

Vision Block Screen [9A78]

Special Tools:

none

Language of origin: English

Intended Use:

To obscure patient's vision of a general procedure / examination.



Caution



Assure chair is clear of any obstacles that may interfere with the operation of the vision block screen.

Note

The vision block screen works with both Midmark® accessory rail and surgical rail.

Installation of vision block screen...

- Loosen lower knob on swivel block.
- Slide swivel block onto exam chair rail.
- Position at desired location and tighten knob.

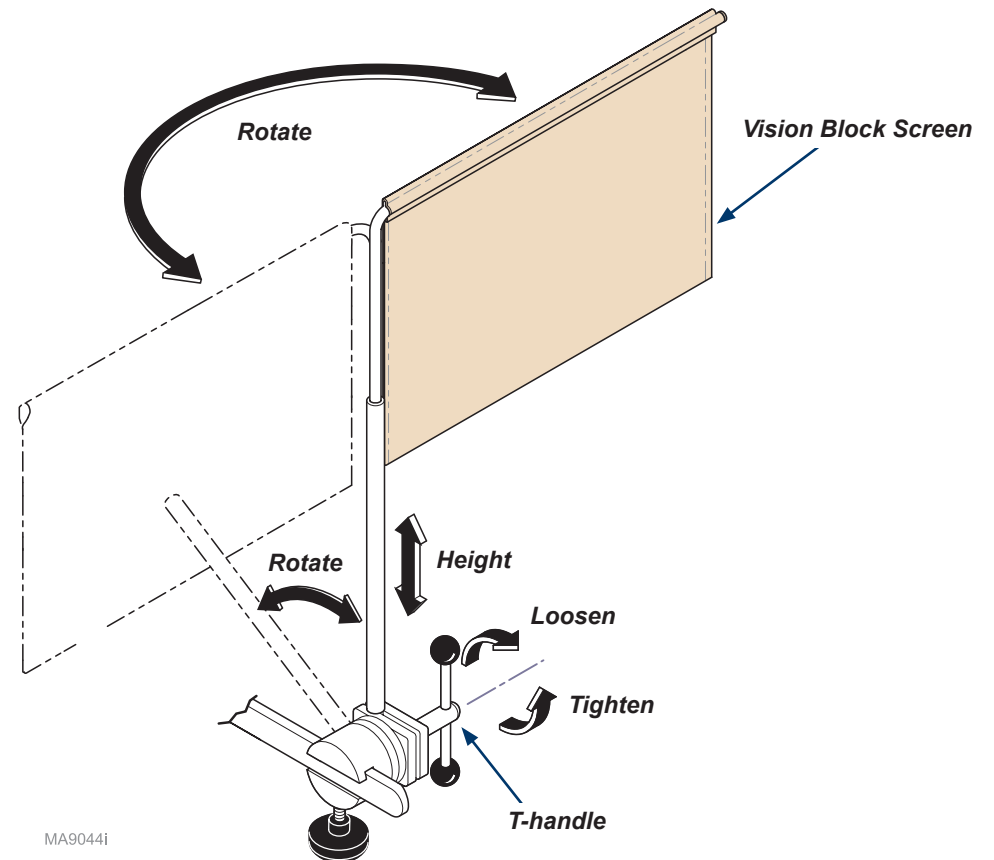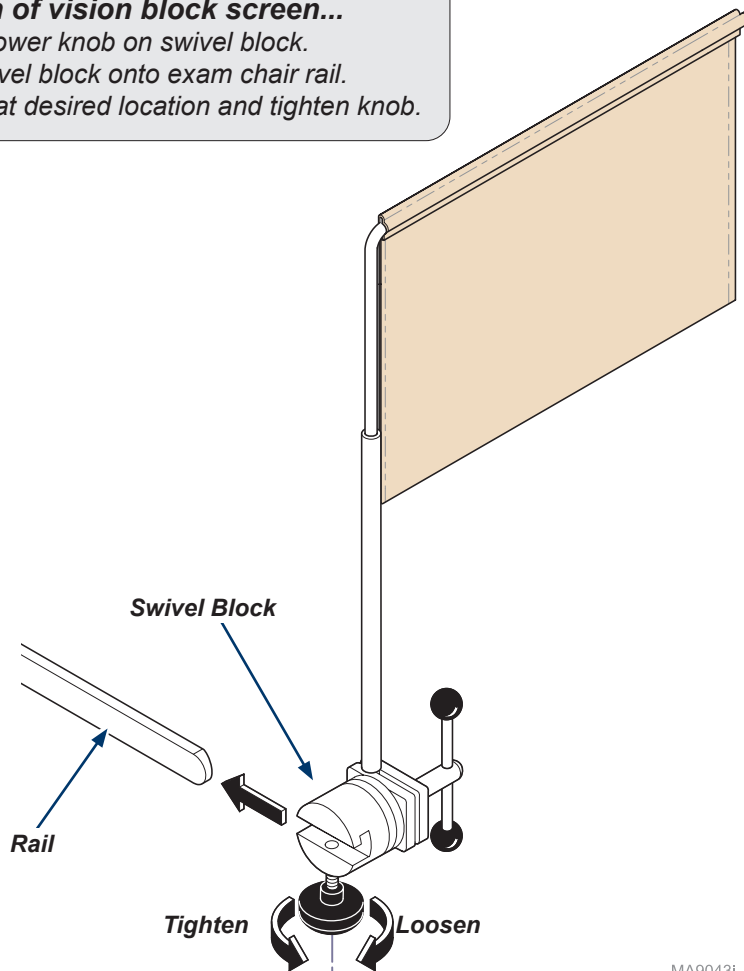

Operation of vision block screen...

- Support vision block screen and loosen T-handle.
- Position screen to desired height and angle.
- Tighten T-handle.