

QC3R-01 QUICKCLEAN ULTRASONIC CUTOUT INSTRUCTIONS

1. ENSURE THAT THE MODEL NUMBER REFERENCED ON THIS TEMPLATE AND THE MODEL NUMBER OF THE MIDMARK ULTRASONIC CLEANER MATCH APPROPRIATELY.
2. LOCATE TEMPLATE TO PROPER POSITION (LEFT-RIGHT) ON THE COUNTERTOP. USE THE CENTERLINE ON THE TEMPLATE TO REFERENCE POSITIONING.
3. SLIDE THE TEMPLATE SO THAT THE EDGE MARKED "BACK EDGE" IS FLUSH AGAINST THE BACKSPLASH (ALLOW THE TEMPLATE TO RIDE UP THE CORNER RADIUS), AND THEN TAPE THE TEMPLATE IN PLACE.
4. USING A DRILL AND 1/2" BIT, DRILL THE (4) CORNER HOLES AT THE LOCATIONS SHOWN.
5. REMOVE THE TEMPLATE AND DRAW A STRAIGHT LINE CONNECTING THE OUTSIDE TANGENT POINTS OF EACH 1/2" DIA. HOLE TO REPRESENT THE CUTOUT.
6. APPLY A LAYER OF PAPER TAPE TO THE BOTTOM SURFACE OF A SABER SAW, MAKING CERTAIN THAT NO METAL IS EXPOSED. THIS WILL PREVENT SCRATCHING OF THE COUNTERTOP.
7. USING THE SABER SAW AND A SMOOTH CUT WOOD BLADE (32 TEETH/INCH), FOLLOW AND CUT THE CUTOUT LINE PERIMETER.

NOTES:
 1. MATERIAL: 60# WHITE OFFSET PAPER.
 2. INK COLOR: BLACK
 3. PART TO BE SUPPLIED FOLDED; APPROX. 8.25" X 10.75"
 4. FULL SIZE PRINT ON SHEET 2.

ITEM	QTY	PART NUMBER	DESCRIPTION
MATERIAL SPEC			
UNSPECIFIED DIMENSIONAL TOLERANCES:		INCH (MILLIMETERS)	
EC0100120	01/01/2023	RG	ADDED OUTLINE OF ULTRASONIC CLEANER, ADDED NOTE
EC0052440	5/8/17	KFS	RELEASED
ECO NO	DATE	CH BY	CHANGE
REV			
COMMERCIAL TOLERANCES SHALL APPLY ON DRG, ROD, WIRE, SHEET, ETC.			
APPROVED BY	DATE	PART NUMBER	DESCRIPTION
	04/06/17	003-10019-00	TEMPLATE, CUTOUT, QC3R-0X



SHEET 1 OF 2


REV AA2

REGATCHELL 7/7/2023 2:52:53 PM

WALL

BACKSPLASH

BACK EDGE

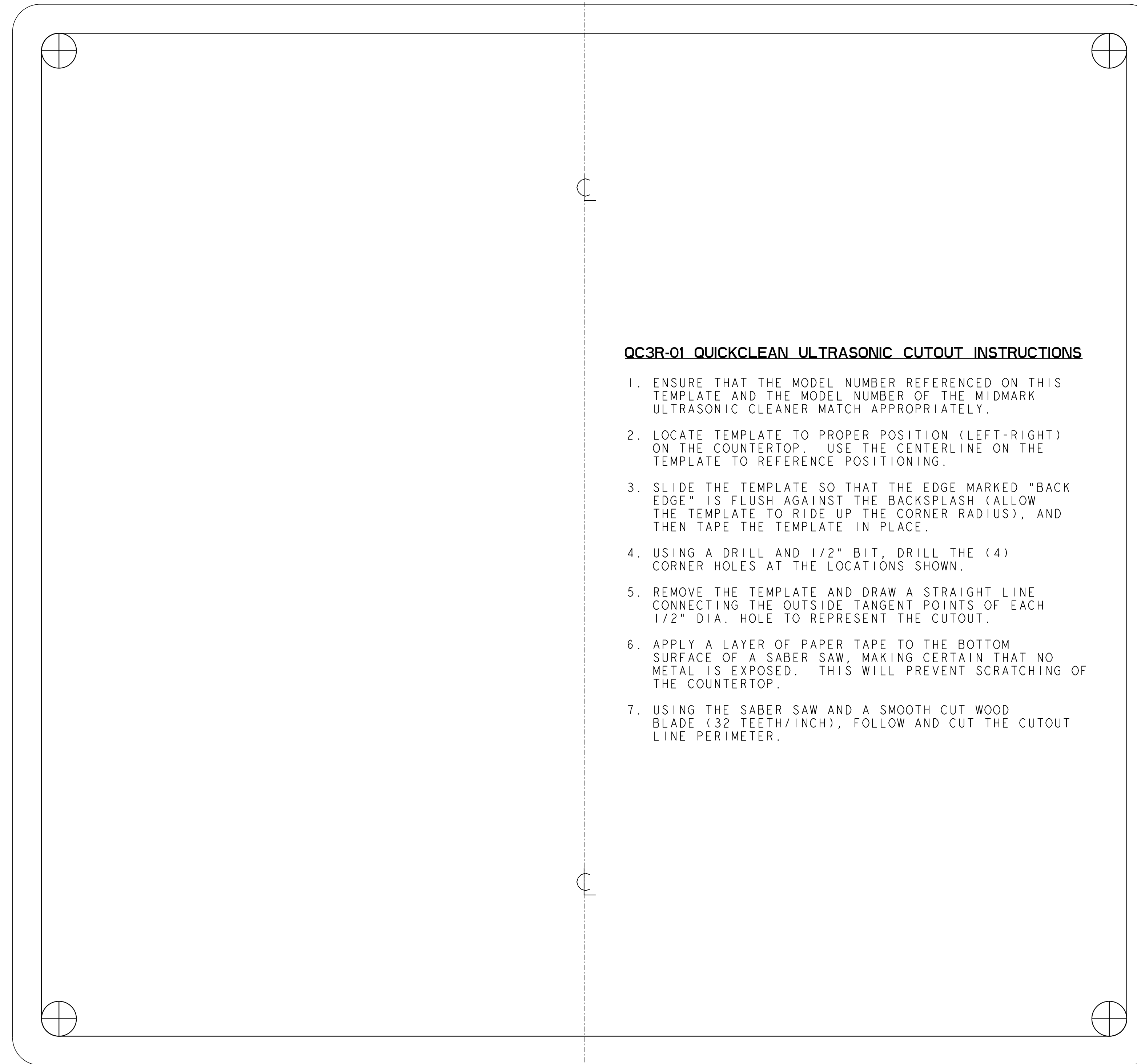

QC3R-01 QUICKCLEAN ULTRASONIC CUTOUT INSTRUCTIONS

1. ENSURE THAT THE MODEL NUMBER REFERENCED ON THIS TEMPLATE AND THE MODEL NUMBER OF THE MIDMARK ULTRASONIC CLEANER MATCH APPROPRIATELY.
2. LOCATE TEMPLATE TO PROPER POSITION (LEFT-RIGHT) ON THE COUNTERTOP. USE THE CENTERLINE ON THE TEMPLATE TO REFERENCE POSITIONING.
3. SLIDE THE TEMPLATE SO THAT THE EDGE MARKED "BACK EDGE" IS FLUSH AGAINST THE BACKSPLASH (ALLOW THE TEMPLATE TO RIDE UP THE CORNER RADIUS), AND THEN TAPE THE TEMPLATE IN PLACE.
4. USING A DRILL AND 1/2" BIT, DRILL THE (4) CORNER HOLES AT THE LOCATIONS SHOWN.
5. REMOVE THE TEMPLATE AND DRAW A STRAIGHT LINE CONNECTING THE OUTSIDE TANGENT POINTS OF EACH 1/2" DIA. HOLE TO REPRESENT THE CUTOUT.
6. APPLY A LAYER OF PAPER TAPE TO THE BOTTOM SURFACE OF A SABER SAW, MAKING CERTAIN THAT NO METAL IS EXPOSED. THIS WILL PREVENT SCRATCHING OF THE COUNTERTOP.
7. USING THE SABER SAW AND A SMOOTH CUT WOOD BLADE (32 TEETH/INCH), FOLLOW AND CUT THE CUTOUT LINE PERIMETER.

OUTLINE OF ULTRASONIC CLEANER (FOR REFERENCE ONLY)

FRONT

003-10019-00 REV AA2