

**QC6R-01 QUICKCLEAN ULTRASONIC CUTOUT INSTRUCTIONS**

1. ENSURE THAT THE MODEL NUMBER REFERENCED ON THIS TEMPLATE AND THE MODEL NUMBER OF THE MIDMARK ULTRASONIC CLEANER MATCH APPROPRIATELY.
2. LOCATE TEMPLATE TO PROPER POSITION (LEFT-RIGHT) ON THE COUNTERTOP. USE THE CENTERLINE ON THE TEMPLATE TO REFERENCE POSITIONING.
3. SLIDE THE TEMPLATE SO THAT THE EDGE MARKED "BACK EDGE" IS FLUSH AGAINST THE BACKSPLASH (ALLOW THE TEMPLATE TO RIDE UP THE CORNER RADIUS), AND THEN TAPE THE TEMPLATE IN PLACE.
4. USING A DRILL AND 1/2" BIT, DRILL THE (4) CORNER HOLES AT THE LOCATIONS SHOWN.
5. REMOVE THE TEMPLATE AND DRAW A STRAIGHT LINE CONNECTING THE OUTSIDE TANGENT POINTS OF EACH 1/2" DIA. HOLE TO REPRESENT THE CUTOUT.
6. APPLY A LAYER OF PAPER TAPE TO THE BOTTOM SURFACE OF A SABER SAW, MAKING CERTAIN THAT NO METAL IS EXPOSED. THIS WILL PREVENT SCRATCHING OF THE COUNTERTOP.
7. USING THE SABER SAW AND A SMOOTH CUT WOOD BLADE (32 TEETH/INCH), FOLLOW AND CUT THE CUTOUT LINE PERIMETER.

25.25 REF

6.065

14.37

22.48

Ø .50  
(4) PLCS

AA2 - OUTLINE OF ULTRASONIC CLEANER (FOR REFERENCE ONLY)

FRONT


WALL

BACKSPLASH

BACK EDGE

- NOTES:
1. MATERIAL: 60# WHITE OFFSET PAPER.
  2. INK COLOR: BLACK
  3. PART TO BE SUPPLIED FOLDED; APPROX. 8.25" X 10.75"
  4. FULL SIZE PRINT ON SHEET 2.

ITEM	QTY	PART NUMBER	DESCRIPTION
MATERIAL SPEC			
INCH [MILLIMETERS]		INCH [MILLIMETERS]	
UNSPECIFIED DIMENSIONAL TOLERANCES:		UNSPECIFIED DIMENSIONAL TOLERANCES:	
ANGULAR +/-	2.0°	SCALE	1:000
FRACTIONAL +/-	0.1	DATE	04/16/17
DECIMAL .XX +/-	0.03	PREPARED BY	KSCHMITZ
DECIMAL .XXX +/-	0.015	DATE	04/16/17
ECO NO	DATE	CH BY	CHANGE
ECO NO	DATE	CH BY	CHANGE
ECO NO	DATE	CH BY	CHANGE
THIS PRINT AND THE INFORMATION CONTAINED THEREIN ARE PROPERTY OF MIDMARK CORPORATION AND ARE LOANED TO THE USER IN CONFIDENCE AND ARE NOT TO BE REPRODUCED, USED, OR DISCLOSED TO OTHERS WITHOUT THE EXPRESS WRITTEN PERMISSION OF MIDMARK CORPORATION.		COMMERCIAL TOLERANCES SHALL APPLY ON DIA, ROD, WIRE, SHEET, ETC.	
APPROVED BY	DATE	PART NUMBER	003-10020-00
REV	AA2		

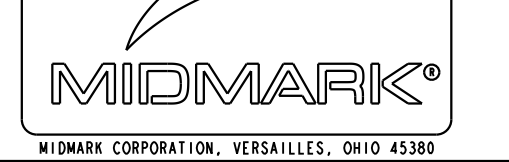

SHEET 1 OF 2

TEMPLATE, CUTOUT, QC6R-0X


REV AA2

7/7/2023 2:51:41 PM

WALL

BACKSPLASH

BACK EDGE

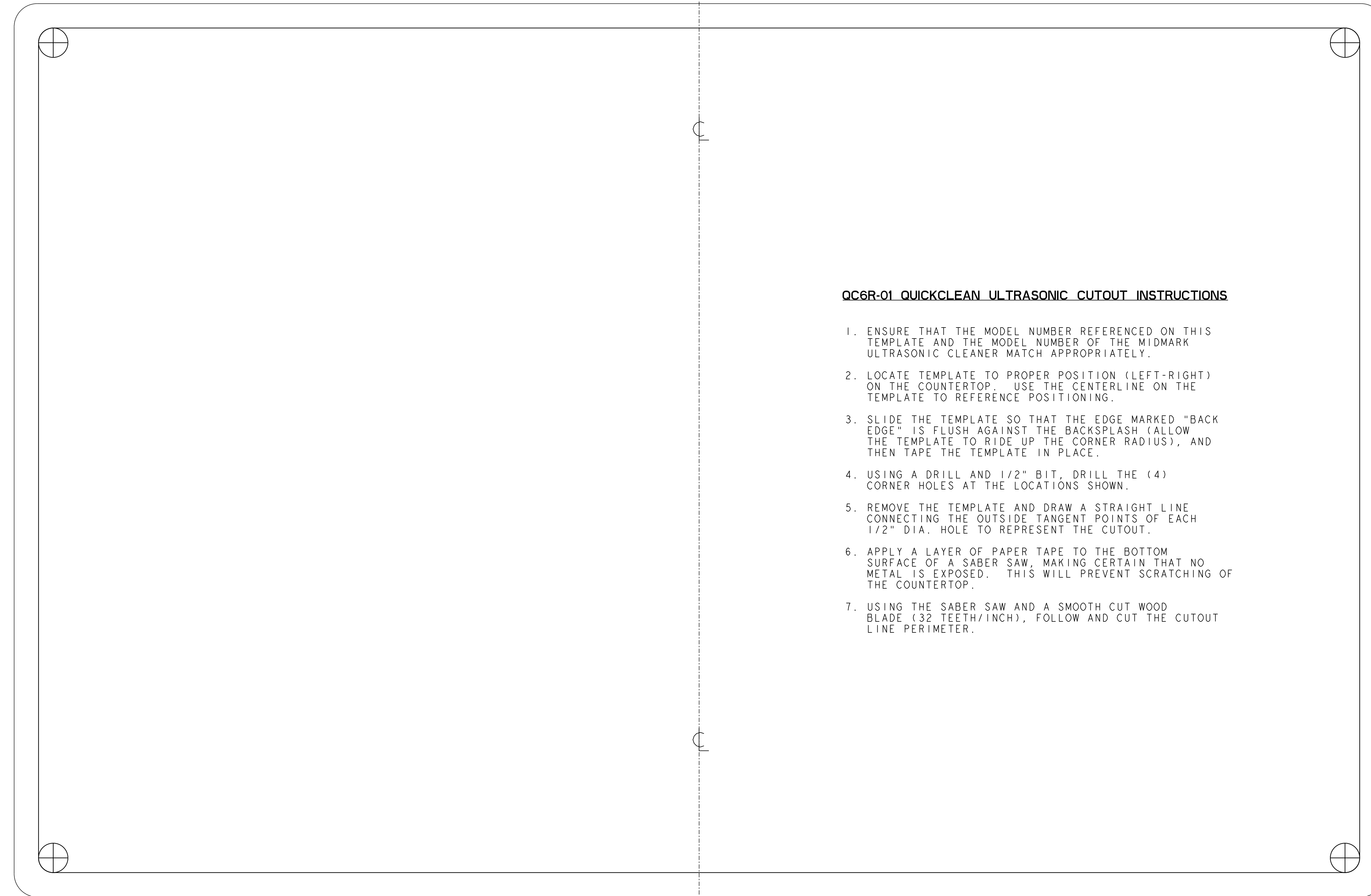

**QC6R-01 QUICKCLEAN ULTRASONIC CUTOUT INSTRUCTIONS**

1. ENSURE THAT THE MODEL NUMBER REFERENCED ON THIS TEMPLATE AND THE MODEL NUMBER OF THE MIDMARK ULTRASONIC CLEANER MATCH APPROPRIATELY.
2. LOCATE TEMPLATE TO PROPER POSITION (LEFT-RIGHT) ON THE COUNTERTOP. USE THE CENTERLINE ON THE TEMPLATE TO REFERENCE POSITIONING.
3. SLIDE THE TEMPLATE SO THAT THE EDGE MARKED "BACK EDGE" IS FLUSH AGAINST THE BACKSPLASH (ALLOW THE TEMPLATE TO RIDE UP THE CORNER RADIUS), AND THEN TAPE THE TEMPLATE IN PLACE.
4. USING A DRILL AND 1/2" BIT, DRILL THE (4) CORNER HOLES AT THE LOCATIONS SHOWN.
5. REMOVE THE TEMPLATE AND DRAW A STRAIGHT LINE CONNECTING THE OUTSIDE TANGENT POINTS OF EACH 1/2" DIA. HOLE TO REPRESENT THE CUTOUT.
6. APPLY A LAYER OF PAPER TAPE TO THE BOTTOM SURFACE OF A SABER SAW, MAKING CERTAIN THAT NO METAL IS EXPOSED. THIS WILL PREVENT SCRATCHING OF THE COUNTERTOP.
7. USING THE SABER SAW AND A SMOOTH CUT WOOD BLADE (32 TEETH/INCH), FOLLOW AND CUT THE CUTOUT LINE PERIMETER.

—OUTLINE OF ULTRASONIC CLEANER (FOR REFERENCE ONLY)

FRONT

003-10020-00 REV. AA2