
Ritter®

Barrier-Free® Exam Chair

[English](#)

[Español](#)

[Français](#)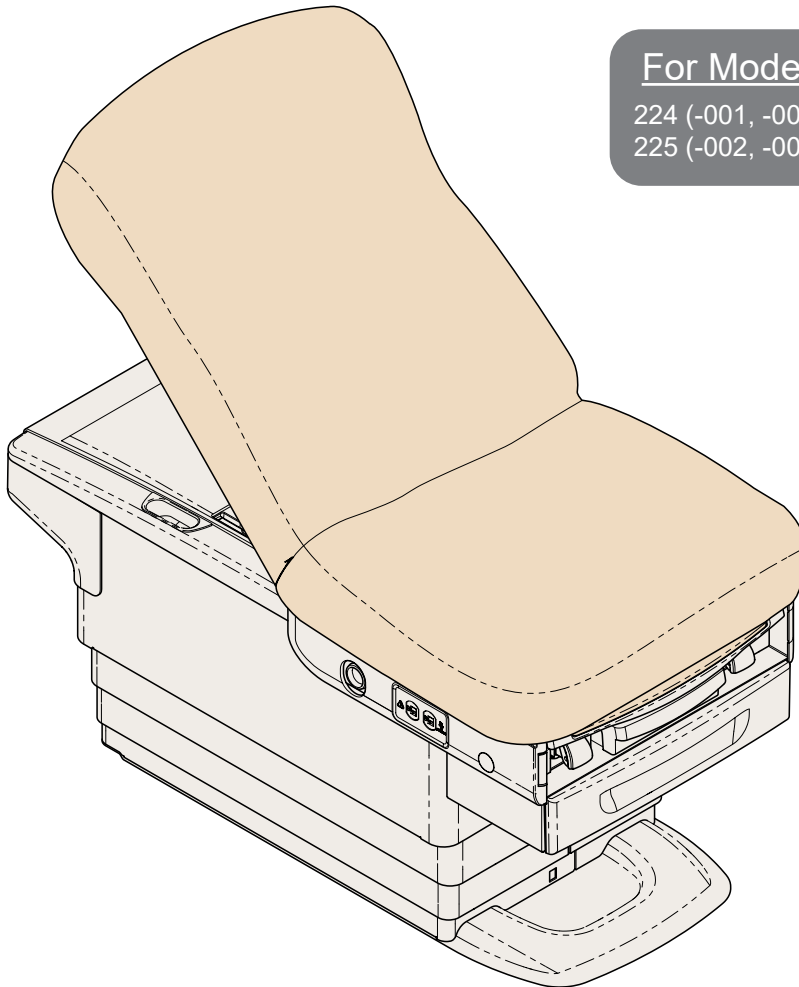

For Models:

224 (-001, -002, -003)

225 (-002, -003)

MA10327i

User Guide

003-10025-99 Rev. AA6 (11/10/21)

TP202 20-42-FO-00014 Rev A2 C2169

Style P

Product Information


Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

**Model / Serial Number Label
Located on the Back Mounting Frame**
(exact location may vary)

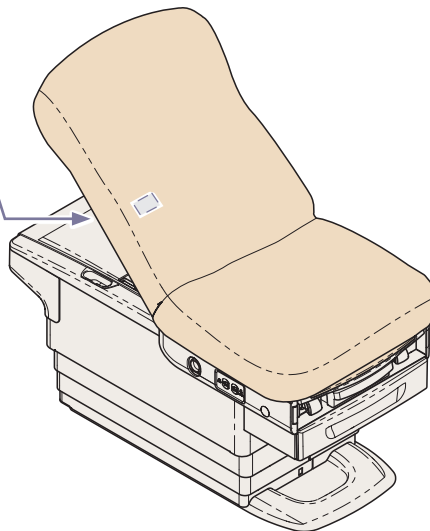MA100311

Table of Contents

Important Information

Safety Symbols.....	4
Product Registration.....	5
Transportation / Storage Conditions.....	5
Operating Conditions.....	5
Disposal of Equipment.....	5
Authorized Representatives.....	6
Intended Use.....	7
Electrical Requirements.....	7
Electromagnetic Interference.....	7
Safety Instructions.....	9

Operation

Active Sensing Technology™.....	11
Base Up / Down Function.....	12
Back Up / Down Function.....	13
Paper Roll & Tear Strap.....	15
Stirrups.....	16
Footrest & Treatment Pan.....	17
Drawer w/ Heater.....	18
Pelvic Tilt.....	19
Duplex Receptacles.....	20
Clean Assist™ Roller System.....	21

Maintenance

Calling for Service.....	22
Cleaning.....	22
Preventative Maintenance.....	22

Specifications

Specifications Chart.....	23
Model Identification / Compliance Chart.....	24
Range of Motion.....	25
Dimensions.....	26

Warranty Information

Limited Warranty.....	27
-----------------------	----

Important Information - Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a situation which could result in equipment damage.

Note

Amplifies a procedure, practice, or condition.

Symbol Glossary

Duty Cycle (motor run time)
30 seconds ON, 5 minutes OFF

Serial Number

Proper shipping orientation

Keep dry


Fragile

Maximum stacking height
(palletted units)

Type B, Applied Part
(Upholstery)

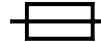Fuse rating specification

Protective earth ground

Dangerous voltage / shock hazard

Patient Weight Limit

Refer to manual

Manufacturer

Catalogue Number

Operating Instructions

Product Registration

To register your product, go to www.midmark.com

Transportation / Storage Conditions

Ambient Temperature Range:.....-30°C to +60°C (-22°F to +140°F)

Relative Humidity.....10% to 90% (non-condensing)

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)

Relative Humidity.....30% to 75% (non-condensing)

Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,

Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800

Intended Use

The examination chair is intended to be used as a chair / table to provide positioning and support of patients during general examination procedures conducted by medical professionals.

Electrical Requirements

Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.

Caution

To completely isolate the chair from electrical mains supply, power cord must be unplugged.

Caution

Prior to use of high frequency surgical devices or endocardial catheters, consult the operating instructions for those devices. Failure to comply may result in electric shock or burns to the patient.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room
- Plug chair into isolated circuit
- Increase separation between chair and interfering device
- Contact Midmark if interference persists

Equipment Alert

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Exam Chair including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

EMC - Manufacturer's Declaration and Guidance

Note

The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals. The exam chair is not intended to be used in a residential environment.

Immunity Test	Immunity Test Level
Electrostatic Discharge	± 8 kV Contact, ±15 kV Air
Radiated RF EM Fields	3V/m 80MHz-2.7 GHz 80% AM at 1kHz
Rated Power Frequency Magnetic Fields	30 A/m
Electrical Fast Transients	±2 kV at 100kHz repetition frequency
Surges	Line to Line: ±1 kV Line to Ground: ±2 kV
Conducted RF	3V: 0.15MHz -80MHz 6V in ISM Bands between 0.15MHz - 80MHz 80% AM at 1kHz
Voltage Dips	0 % UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0°
Voltage Interruptions	0 % UT; 250/300 cycle

Radiated Test	Radiated Test Level
Conducted and Radiated RF Emissions	CISPR 11 Group 1 Class A

Safety Instructions

WARNING

No modification of this equipment is allowed.

WARNING

Connecting equipment to the multiple socket outlet effectively leads to creating a Medical Electrical System and the result can be a reduced level of safety.

WARNING

To avoid a fire hazard or risk of electric shock, this equipment must only be connected to a supply mains with protective earth ground.

WARNING

Ensure that the surrounding area is clear of obstructions prior to each use. Otherwise improper operation and serious injury could occur.

WARNING

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Safety Instructions - continued

WARNING

Be sure that all personnel and equipment are clear of the chair before activating any function. Keep a minimum 6 in. (15.25 cm) clearance surrounding the chair. Failure to do so could result in personal injury.

At the chair's lowest height [18 in. (46 cm)], the clearance between the bottom of the drawer and the floor is approximately 5 in. (12.7 cm). Make sure patient (esp. feet) and all objects are clear of this area before lowering chair.


Equipment Alert

Chair must be positioned in normal use such that the power cords can be easily accessed to unplug.

If the chair malfunctions, immediately release controller, unplug the chair power cord, and assist the patient from the chair. If chair continues to malfunction, call for service.

Failure to comply with these instructions may result in injury to patient and/or damage to equipment.

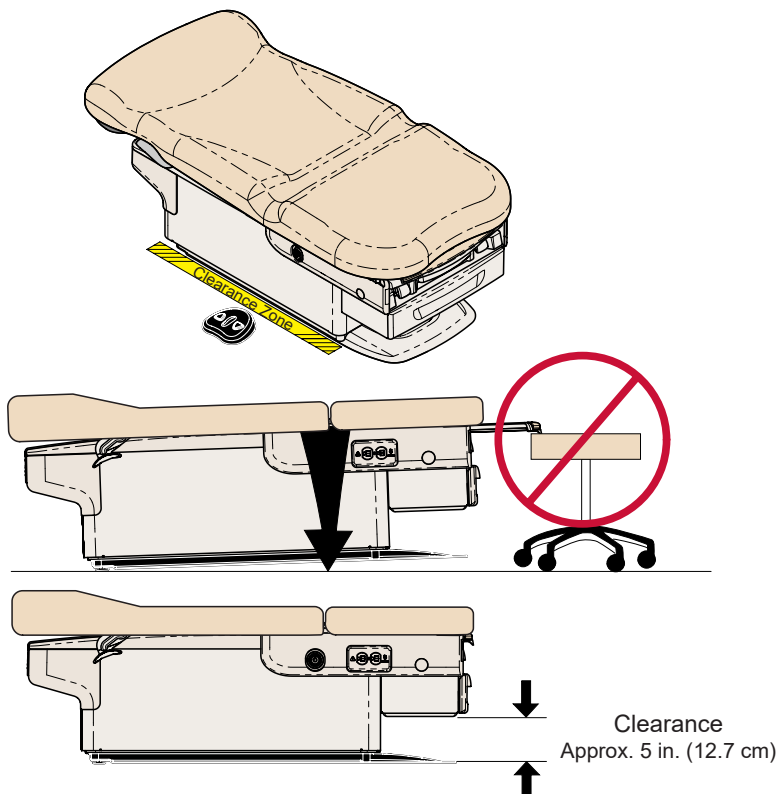Operation

Active Sensing Technology™


WARNING

Be sure that all personnel and equipment are clear of the chair before activating any function. Failure to do so could result in personal injury.

The Active Sensing Technology™ prevents the chair from moving Down if the bottom of the drawer contacts another object. If the Active Sensing Technology™ is activated, it will sound a series of “Beeps,” disable the Base Down function and move chair in the Base Up direction. To resume Base Down operation, remove object from under the bottom of the drawer and press the Base Up / Down buttons on controller.

Note

The Active Sensing Technology™ is not available on models without drawer.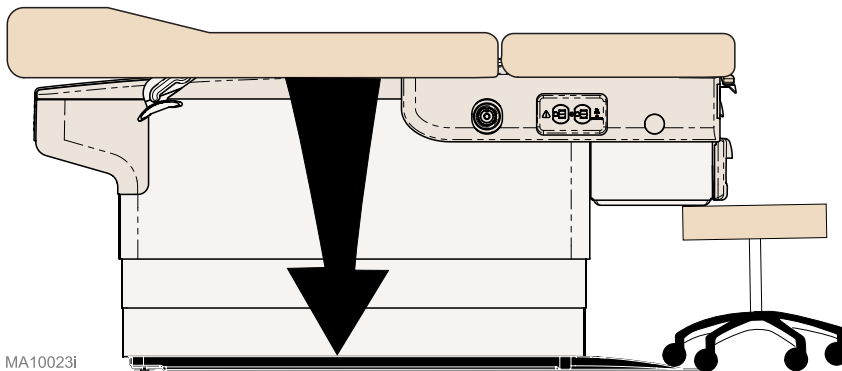

Operation

Base Up / Down Function

Attention!

The Active Sensing Technology™ will stop the Base Down function and retract Up if the bottom of the drawer contacts another object.

(You will hear a series of “Beeps” if the Active Sensing Technology™ is activated.)

To resume Base Down operation:


Remove object from under the bottom of the drawer.

Press Base Down button.


Note

The chair automatically stops at its maximum and minimum height.

To move the chair Up or Down...
Press and hold the appropriate button.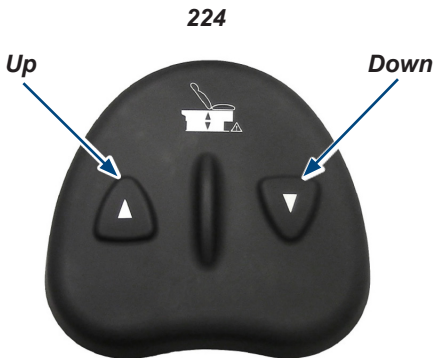

ArtHalf

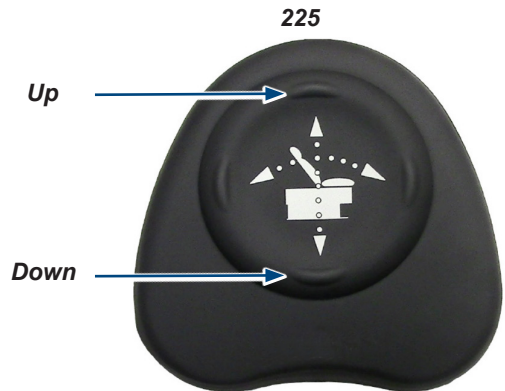
Operation - continued

Back Up / Down Function - 224

To position the back section...

- A) While holding the back section in place, squeeze the Back Release Handle (either side).
- B) Move the back section.
- C) Release handle to lock back section in position.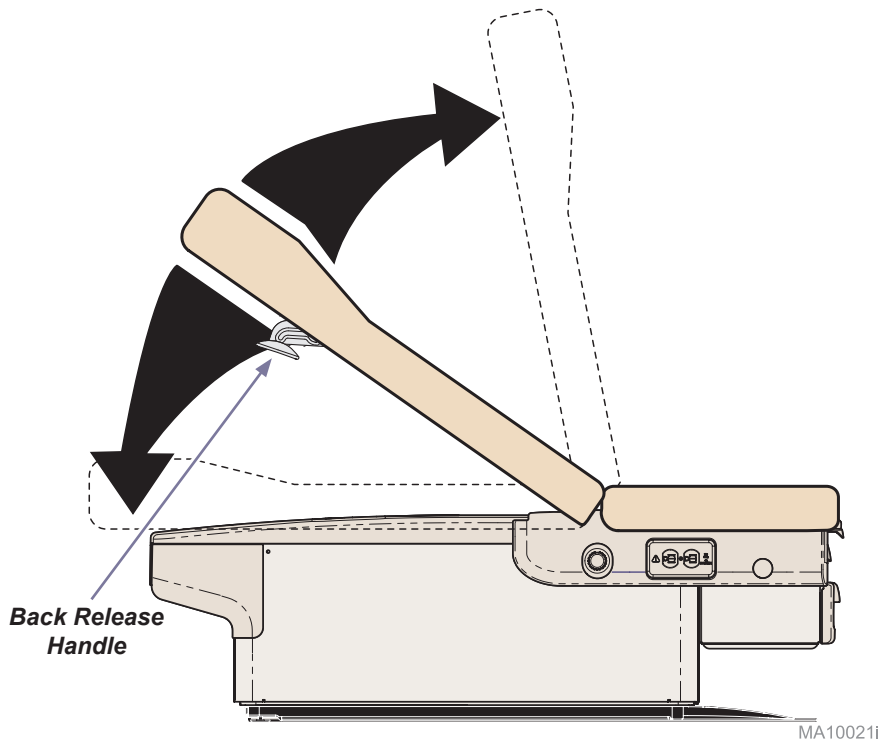

Operation - continued


Back Up / Down Function - 225

Note

The back section automatically stops at the top and bottom of its travel.

To position the back section...

Press and hold the appropriate button.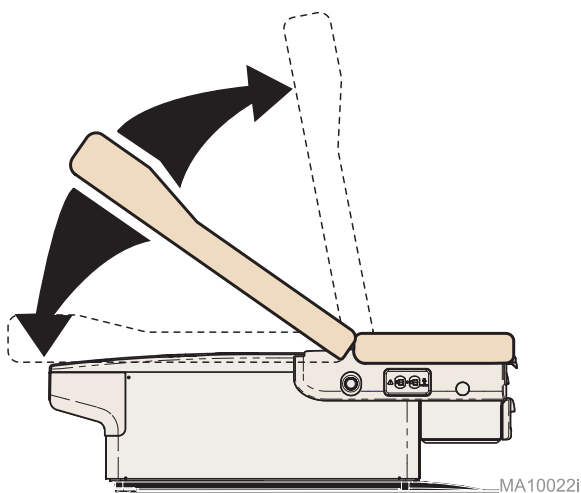


Paper Roll & Tear Strap

Paper Roll Size (maximum):..... 24 in. wide x 3.5/8 in. diameter
(61 cm x 9.2 cm)


Note

One additional paper roll may be stored under the back section.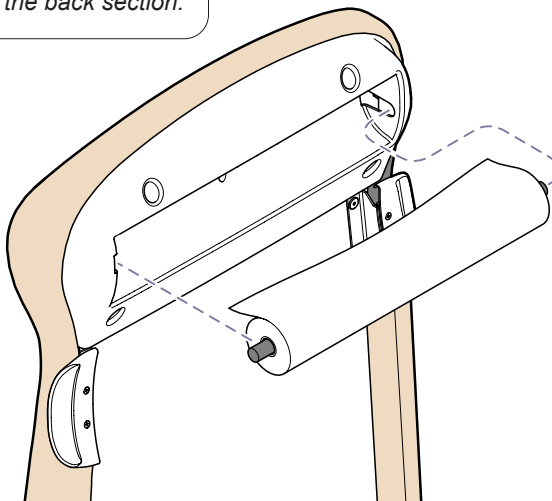

WARNING

To avoid injury, do not put head or extremities under the paper tear strap.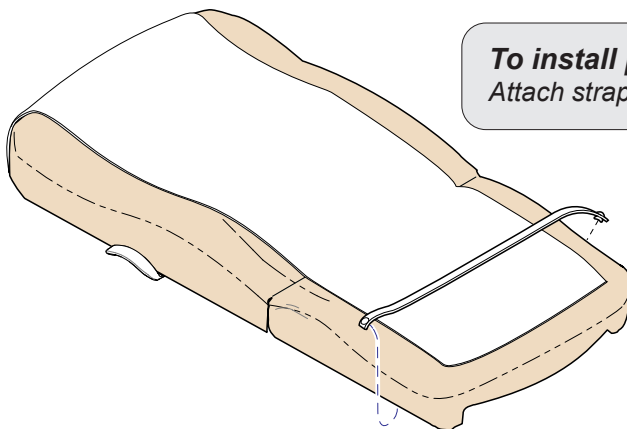

To install paper tear strap...

Attach strap to snaps on each side of chair.

Stirrups


Caution

*Be sure the stirrups are locked in place before using.
The stirrups will not support the patient's entire weight.*

To position the stirrups...

- A) *Pull the stirrup out, then unfold.*
- B) *Lift the stirrup slightly, then move it left or right as desired.*
- C) *Release stirrup to lock in position.*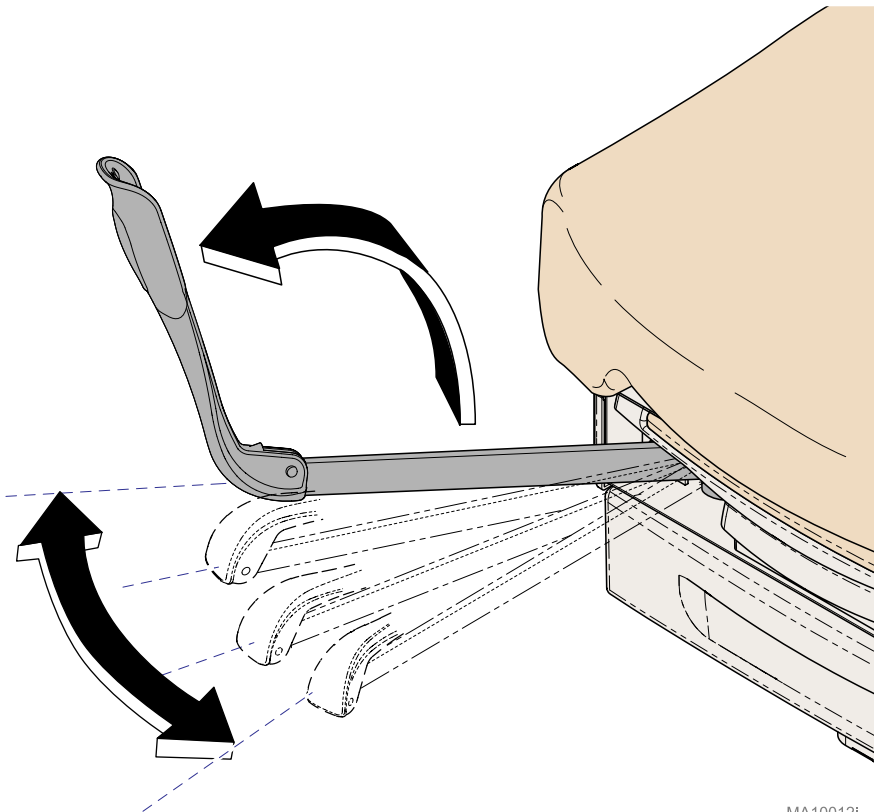

MA10012i

Footrest & Treatment Pan (not available on all models)


WARNING

Do not use footrest to support patient's entire weight.

To extend footrest...

Pull footrest out to desired position.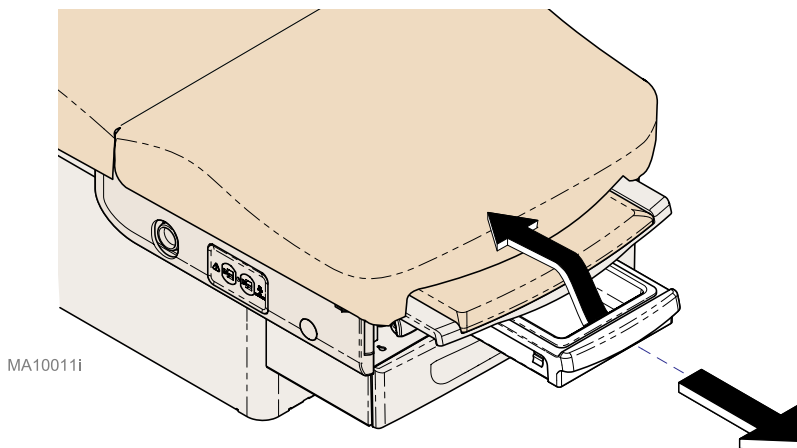

To access the treatment pan...


Pull treatment pan out to desired position.

Drawer w/ Heater (not available on all models)

The drawer heater warms to 92°F (33°C) to 112°F (44°C) measured at the center of the drawer at room temperature and nominal input voltage.

To turn the drawer heater ON / OFF...
Press and release the drawer heater switch.

Note: Switch will illuminate when heater is ON.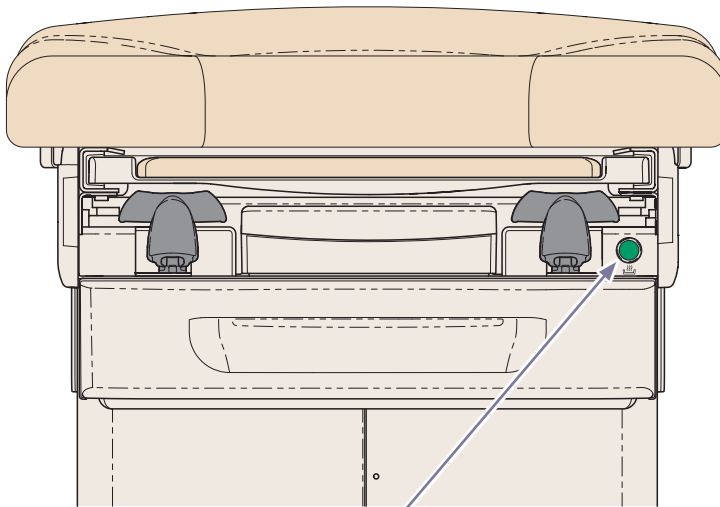

***Drawer Heater
Switch***

MA10015i

Pelvic Tilt (not available on all models)


WARNING

To avoid risk of injury do not put extremities under the seat.

To elevate seat section...


Lift seat until pelvic support drops into place.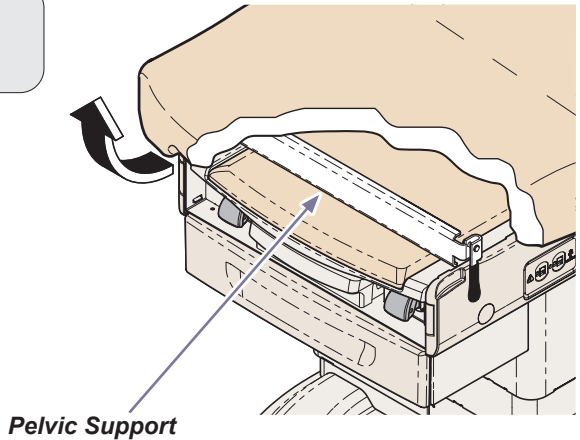

MA10017i

To return seat section to flat position...

- A) *Lift seat section slightly.*
- B) *Pivot pelvic support handle upward.*
- C) *Lower seat section slowly.*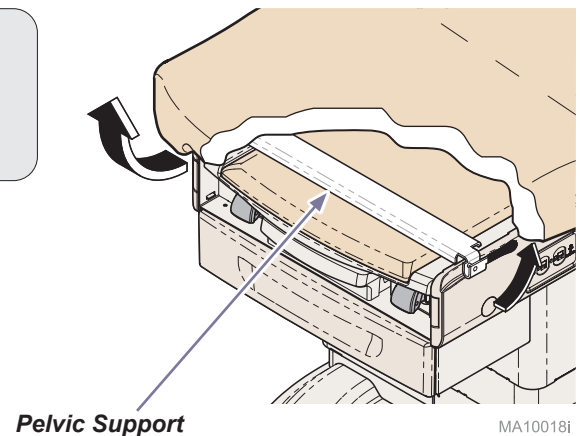


MA10018i

Duplex Receptacles (not available on all models)

The duplex receptacles on each side of chair provide power for accessories used during medical procedures. There are two circuit breakers located under the seat section. If the receptacles' maximum load is exceeded, the circuit breakers interrupt power to the receptacles.

Maximum Load.....115 VAC, 4 amps

To reset the circuit breaker...
Push circuit breaker switch.

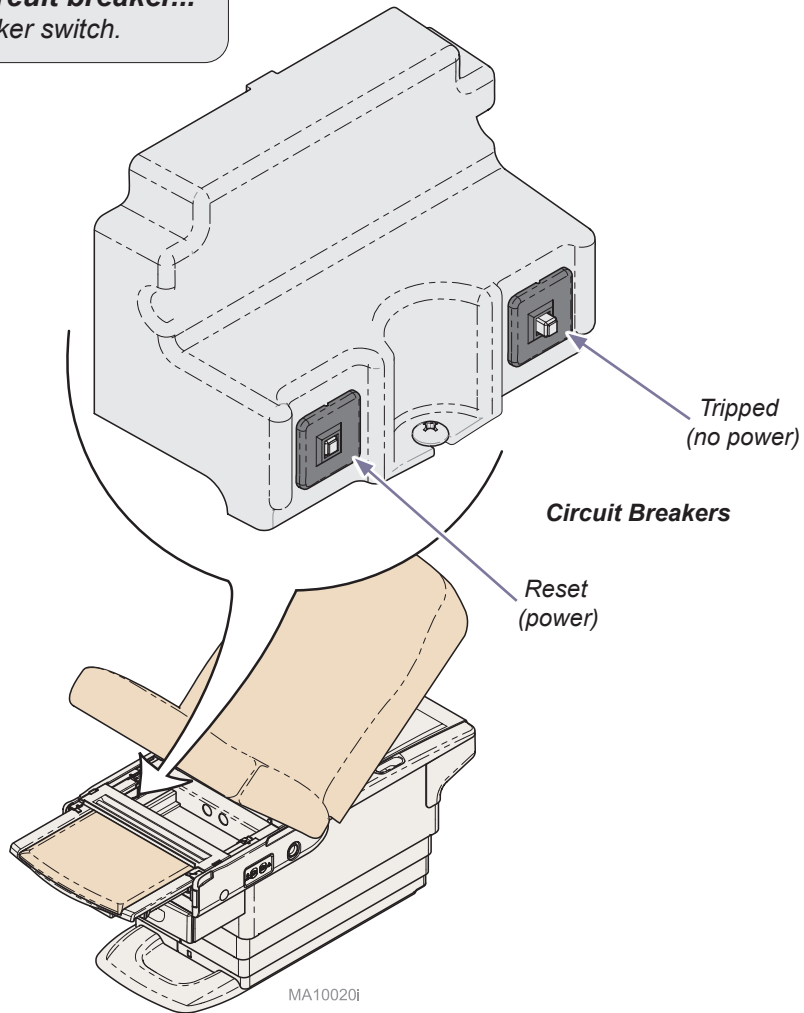Clean Assist™ Roller System (not available on all models)

Control levers located under the back section on each side, activate and deactivate the **Clean Assist™ Roller System**. The rollers are located on each corner of the chair to allow positioning of chair.

Caution

To prevent injury, never activate / deactivate Clean Assist™ Roller System or move chair with patient aboard. Use caution when moving chair.

Equipment Alert

The intended use of the Clean Assist™ Roller System is for chair positioning within the exam room only.

Equipment Alert

Be sure to activate both control levers.


WARNING

To avoid risk of fall, the Clean Assist™ Roller System must be disengaged while a patient ingresses or egresses the chair.

To activate Clean Assist™ Roller System...

- Raise back section 30 to 45 degrees.
- Ensure chair is at least two inches higher than its lowest position.
- Move control levers on each side of chair to the Clean Assist™ Roller System engagement position.
- Move chair to the lowest position to activate rollers.


Clean Assist™ Roller System Engagement Position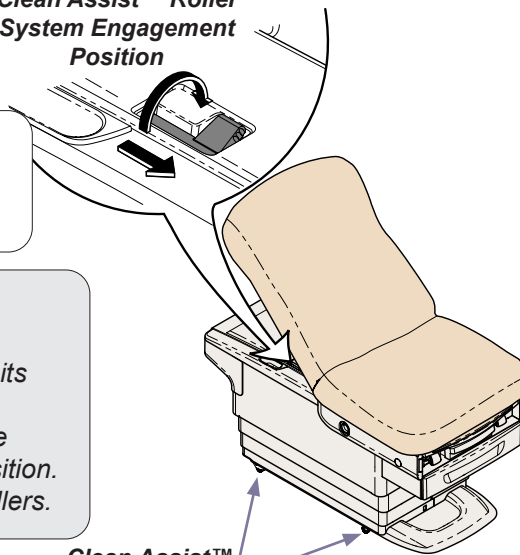

Clean Assist™ Roller System

MA10033i

Clean Assist™ Roller System Disengagement Position

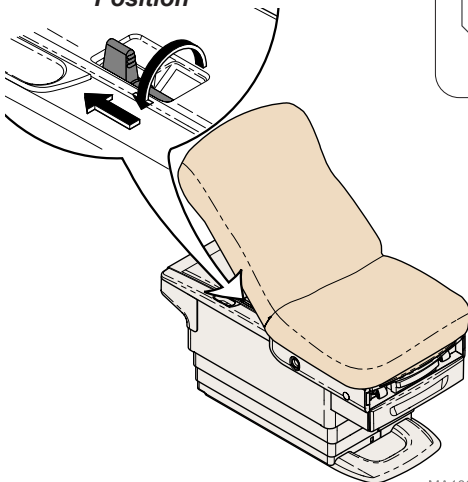MA10029i

Equipment Alert

After Clean Assist™ Roller System has been activated, disconnect power cord and foot control and stow away from chair to prevent damage to cords.

To deactivate Clean Assist™ Roller System...

- Move chair to desired location.
- Reconnect power cord.
- Raise chair at least 2" to deactivate Clean Assist™ Roller System.

Note: An audible click will occur to confirm deactivation.

Note

Control levers will automatically reset to the Clean Assist™ Roller System disengagement position.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact your authorized Midmark dealer. To contact Midmark directly:
1.800.MIDMARK (1.800.643.6275)
8:00 am until 5:00 pm Monday - Friday (ET) [excluding standard US holidays]

Cleaning

WARNING

The upholstery should be inspected before each use and replaced if punctured or otherwise damaged.

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Preventative Maintenance

Periodically inspect the following areas:

- Power cord(s) should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

Periodically lubricate the back hinge to maintain quiet, smooth, operation (use light machine oil).

Have a qualified service technician inspect your equipment every twelve months.

WARNING

Should damage and or loose parts be identified, the examination chair should be removed from use and serviced. Failure to do so may result in injury.

Specifications



Specifications Chart	
Patient Weight (<i>maximum</i>)	500 lbs (227 kg)
Paper Roll (<i>maximum size</i>):	24 in. long x 3.5/8 in. diameter (61 cm x 9.2 cm)
Range of Motion & Dimensions:	<i>[See Range of Motion & Dimensions pages]</i>
Weight of Chair: w/upholstery w/package & skid (<i>no uph.</i>) Uph. w/package (<i>shipped separately</i>)	445 lbs (201.8 kg) 455 lbs (206.4 kg) 40 lbs (18.1 kg)
Power Cord Length:	8 ft. (244 cm) long
Electrical Requirements:	<i>[See Model Identification / Compliance Chart]</i>
Foot / Hand Control Voltage:	3.3 VDC, SELV (Safety Extra Low Voltage)
Duplex Receptacles Maximum Load: (<i>not available on all models</i>)	115 VAC, 4A, 50/60 Hz
Fuse Ratings: <i>IEC inlet fuses</i> F1 (<i>on Main PC board</i>) F2 (<i>on Main PC board</i>)	T8AH, 250V, 5 x 20 mm T6.3AH, 250V, 5 x 20 mm T250mAH, 250V, 5 x 20 mm
Duty Cycle (Motor Run Time):	Intermittent Operation <i>[30 seconds ON / 5 minutes OFF]</i>
Receptacles, Drawer Heaters:	Continuous Operation
Classifications:	Class I, Type B Applied Part, Intermittent Operation <i>[30 seconds ON / 5 minutes OFF]</i>
Certifications:	ES 60601-1:2012, CSA C22.2 No. 60601-1:14; (Safety Standards) IEC 60601-1-2:2014 (EMC Standards)
Protection against ingress of fluids:	IPX0 Chair IPX1 Foot Control

Model Identification / Compliance Chart

Model	Description	Complies To:					Electrical Ratings:		
		ANSI/AAMI ES60601-1	CAN/CSA-C22.2 No. 60601-1:14	IEC 60601-1-2	NFPA 99	CE	VAC +/-10%	Amps	Cycles (Hz)
224-001	Ritter® Basic Chair	•	•	•	•	•	115	9	50 / 60
224-002	Ritter® Basic Chair w/ Receptacles, Pelvic Tilt, & Drawer Heater	•	•	•	•		115	11.5	50 / 60
224-003	Ritter® Basic Chair w/ Receptacles, Recessed Rollers, Pelvic Tilt, & Drawer Heater	•	•	•	•		115	11.5	50 / 60
225-002	Ritter® Power Back Chair w/ Receptacles, Pelvic Tilt, & Drawer Heater	•	•	•	•		115	11.5	50 / 60
225-003	Ritter® Power Back Chair w/ Receptacles, Recessed Rollers, Pelvic Tilt, & Drawer Heater	•	•	•	•		115	11.5	50 / 60



Fire Code Ratings: All upholstery complies with California Bureau of Home Furnishing [Technical Bulletin 117](#) and California Code of Regulations, Sect. 93120-93120.12, Title 17. Optional upholstery is available that complies with California Bureau of Home Furnishing [Technical Bulletin 133](#).

Range of Motion: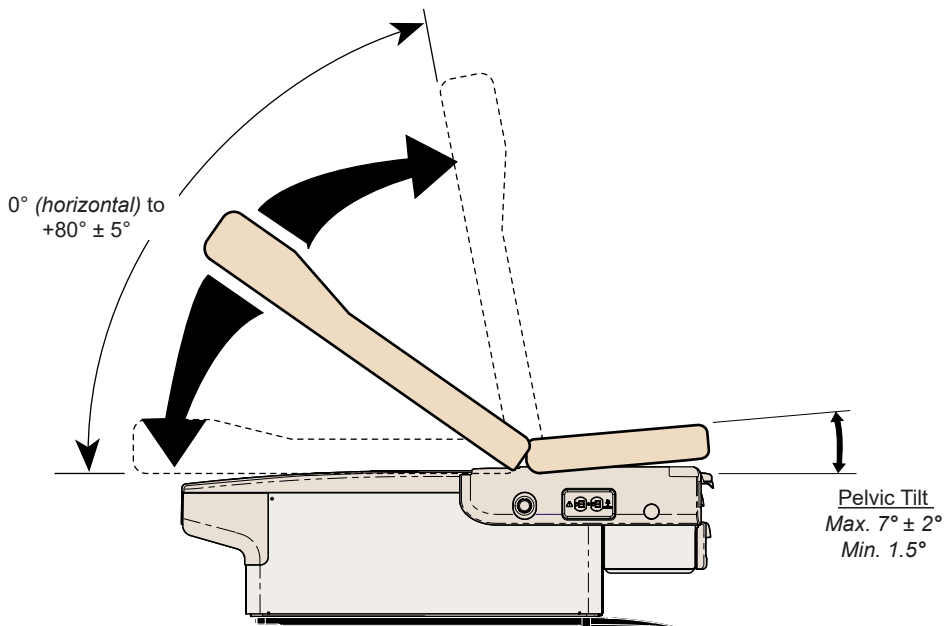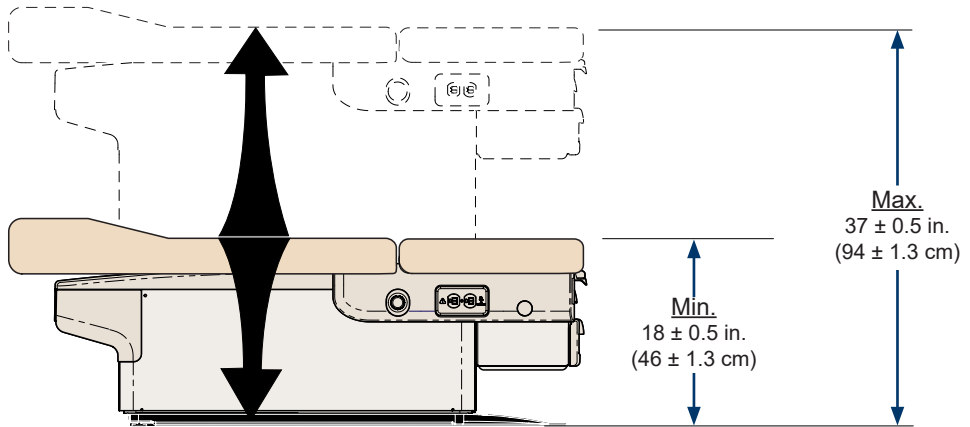

MA10035I

Dimensions:

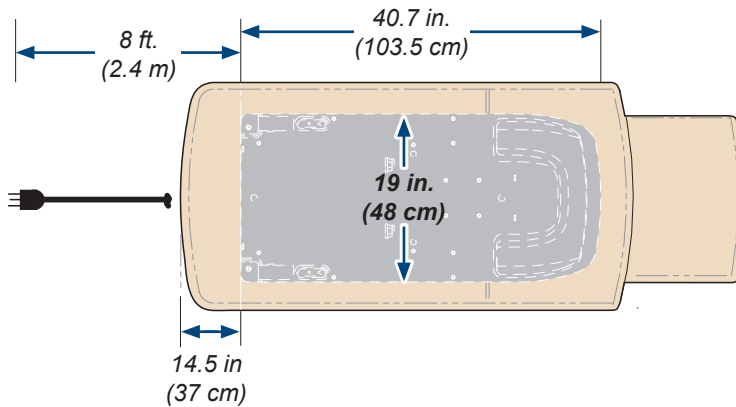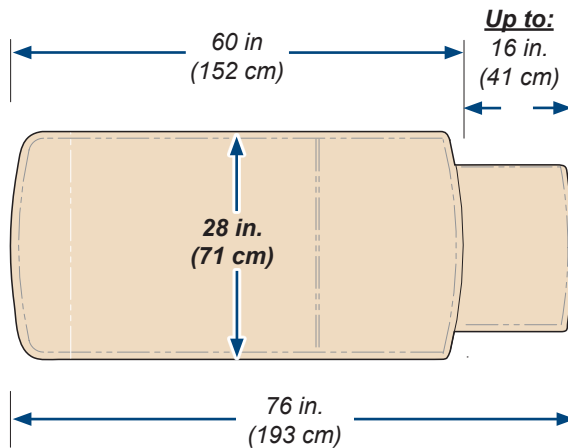MA10036i

Midmark® + Ritter® Medical Equipment (Tables, Chairs, Physician Seating, Lights, Workstations and Sterilizers) Limited Warranty

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components. QuickClean® Ultrasonic Cleaners are warranted for a period of three (3) years.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at midmark.com; by phone at 1.800.MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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© Midmark Corporation 2017