Barrier-Free® Exam Chair

w/ Midmark Digital Scale and
Vital Signs Connections

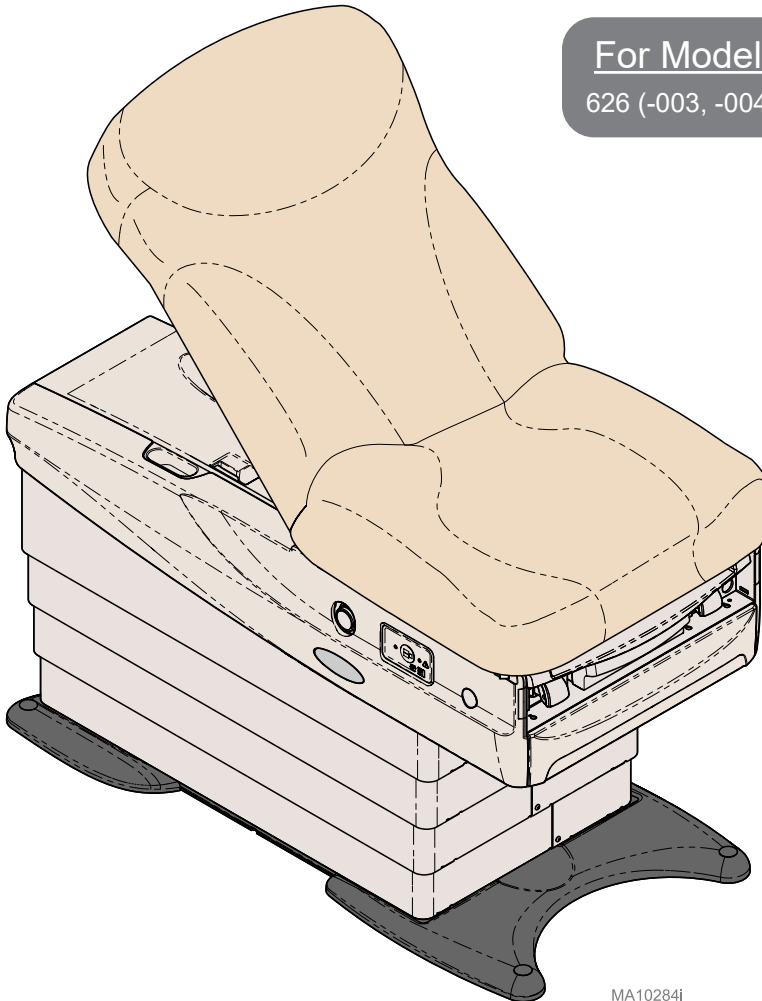
[English](#)

[Español](#)

[Français](#)

For Models:

626 (-003, -004, -005, -006)

MA10284i

User Guide

0 0 3 - 1 0 0 2 7 - 9 9

003-10027-99 Rev. AB4 (01/22/25)

TP202 20-42-FO-00014 Rev A1 C2169

Style P

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

Model / Serial Number Label
Located on the Back Mounting Frame
(exact location may vary)

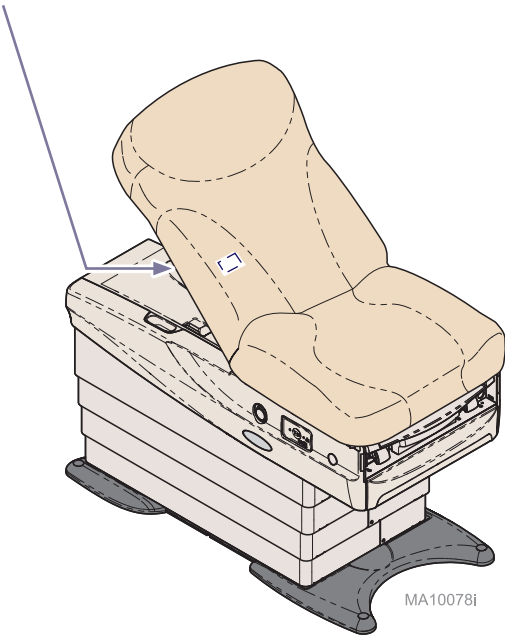
Table of Contents

Important Information

Safety Symbols.....	4
Product Registration	5
Transportation / Storage Conditions.....	5
Operating Conditions.....	5
Disposal of Equipment.....	5
Authorized Representatives	6
Compliance Information.....	7
Intended Use	8
Electrical Requirements.....	8
Electromagnetic Interference.....	8
Digital Scale Accuracy.....	9
Safety Instructions.....	10

Operation

Midmark Digital Scale Functions	13
Active Sensing Technology®	19
Base / Back Functions.....	20
Programmable 'Home' Function	21
Quick Exam® Function.....	22
Control Lockout	23
Association Procedure.....	24
Paper Roll and Tear Strap	25
Stirrups	26
Footrest & Treatment Pan	27
Drawer Heater	28
Upholstery Heater (-004 /-006).....	29
Pelvic Tilt.....	30
Simplex Receptacles.....	31
Data Connections.....	32
Clean Assist Roller System	35

Maintenance

Calling for Service	36
Cleaning	36
Periodic Maintenance	36
Troubleshooting.....	37

Specifications

Specifications Chart.....	38
Model Identification / Compliance Chart.....	39
Range of Motion	40
Dimensions.....	41

Warranty Information

Limited Warranty	42
------------------------	----

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a situation which could result in equipment damage.

Note
Amplifies a procedure, practice or condition.

Symbol Glossary

Stop

Proper shipping orientation

Fragile

Refer to manual

Protective earth ground

Patient Weight Limit

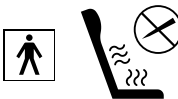
Manufacturer

Catalogue Number

Operating Instructions

The heated upholstery is a Type BF, Applied Part. Do not penetrate upholstery with sharp objects, such as needles, knives, etc.

Type B, Applied Part
(Standard Upholstery)

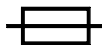
Serial Number

Keep dry

Maximum stacking height
(**palletted** units)

Fuse rating specification

Dangerous voltage / shock hazard

Duty Cycle (motor run time)
30 seconds ON, 5 minutes OFF

Medical Device

Product Registration

Transportation / Storage Conditions

Ambient Temperature Range:.....-30°C to +60°C (-22°F to +140°F)
Relative Humidity.....10% to 90% (non-condensing)

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,

Ladrón de Guevara, 44600,

Guadalajara, Jalisco, México | Tel (33) 3669 9800

Compliance Information

FCC Compliance Statement

This equipment has been tested and found to comply with the limits for a class A digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at their own expense.

Any modifications to this product may violate the rules of the Federal Communications Commission and make operation of the product unlawful.

IC Compliance Statement

Interference

"This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device."

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Antennas

"Under Industry Canada regulations, this radio transmitter may only operate using an antenna of a type and maximum (or lesser) gain approved for the transmitter by Industry Canada. To reduce potential radio interference to other users, the antenna type and its gain should be so chosen that the equivalent isotropically radiated power (e.i.r.p.) is not more than that necessary for successful communication."

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Intended Use

The examination chair is intended to be used as a chair / table to provide positioning and support of patients during general examination procedures conducted by medical professionals.

With the addition of the integrated digital scale, the examination chair is additionally intended to be used for regular monitoring to assess patient weight change, as well as measuring patient weight to assist medical diagnosis. The integrated digital scale is not intended to be used for patients that weigh less than 30 pounds (13.6 kilograms) or more than 650 pounds (295 kilograms).

Electrical Requirements

Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.

Caution

To completely isolate the chair from electrical mains supply, power cord must be unplugged.

Caution

Prior to use of high frequency surgical devices or endocardial catheters, consult the operating instructions for those devices. Failure to comply may result in electric shock or burns to the patient.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room
- Plug chair into isolated circuit
- Increase separation between chair and interfering device
- Contact Midmark if interference persists

Equipment Alert

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Exam Chair including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Digital Scale Accuracy

Equipment Alert

Before using the digital scale, verify the scale is measuring accurately using a reference weight of known value. In addition, refer to the Preventive Maintenance Schedule for the recommended frequency of this check. If the scale is determined to be out of calibration, contact your authorized Midmark service provider before using the scale.

Caution

For models with a wired hand control, placing tension on the hand control cord can have an effect on the scale accuracy.

EMC - Manufacturer's Declaration and Guidance

Note

The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals. The exam chair is not intended to be used in a residential environment.

Immunity Test	Immunity Test Level
Electrostatic Discharge	± 8 kV Contact, ±15 kV Air
Radiated RF EM Fields	3V/m 80MHz-2.7 GHz 80% AM at 1kHz
Rated Power Frequency Magnetic Fields	30 A/m
Electrical Fast Transients	±2 kV at 100kHz repetition frequency
Surges	Line to Line: ±1 kV Line to Ground: ±2 kV
Conducted RF	3V: 0.15MHz -80MHz 6V in ISM Bands between 0.15MHz - 80MHz 80% AM at 1kHz
Voltage Dips	0 % UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0°
Voltage Interruptions	0 % UT; 250/300 cycle

Radiated Test	Radiated Test Level
Conducted and Radiated RF Emissions	CISPR 11 Group 1 Class A

Safety Instructions

WARNING

No modification of this equipment is allowed.

WARNING

Connecting equipment to the multiple socket outlet effectively leads to creating a Medical Electrical System and the result can be a reduced level of safety.

WARNING

To avoid a fire hazard or risk of electric shock, this equipment must only be connected to a supply mains with protective earth ground.

WARNING

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

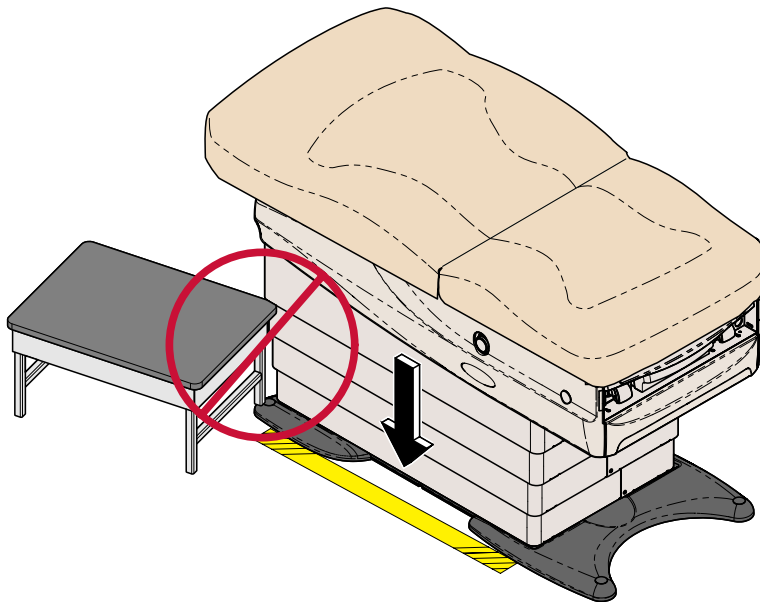
Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Safety Instructions - continued

WARNING

Be sure that all personnel and equipment are clear of the chair before activating any function. Keep a minimum 6 in. (15.25 cm) clearance surrounding the chair. Failure to do so could result in personal injury.

Safety Instructions - continued

WARNING

Be sure that all personnel and equipment are clear of the chair before activating any function. Failure to do so could result in personal injury.

At the chair's lowest height [14.6 in. (37 cm)] w/ compressed foam, the clearance between the bottom of the drawer and the floor is approximately 0.37 in. (0.94x cm). Make sure patient (esp. feet) and all objects are clear of this area before lowering chair.

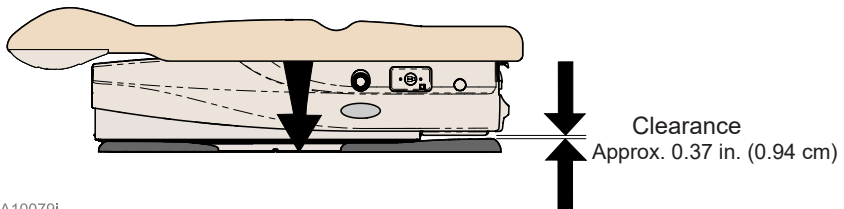
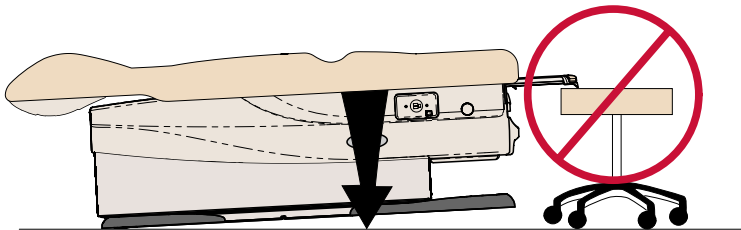
Equipment Alert

Chair must be positioned in normal use such that the power cords can be easily accessed to unplug.

If the chair malfunctions, immediately release controller, unplug the chair power cord, and assist the patient from the chair. If chair continues to malfunction, call for service.

Failure to comply with these instructions may result in injury to patient and/or damage to equipment.

MA10079i

Operation

Midmark Digital Scale Functions

Best practice is to always zero the scale prior to weighing the patient. For details, please refer to the weighing guidelines in this manual.

Caution

Clear any obstruction under or around chair base to ensure the measured weight of patient is accurate.

This includes the cord for the hand or foot control on wired models.

Caution

For models with a wired hand control, place hand control in holster before pressing the Zero button to ensure highest scale accuracy.

Equipment Alert

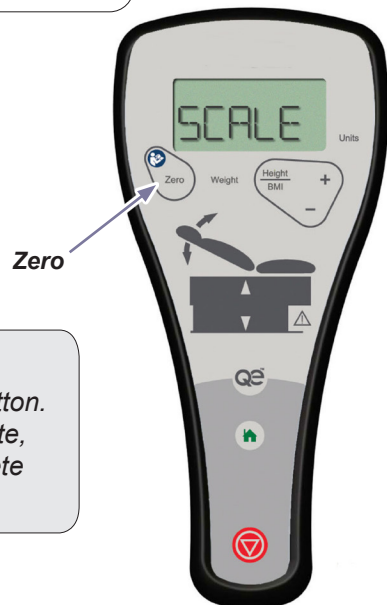
The integrated digital scale is not intended to be used for patients that weigh less than 30 lbs (13.6 kg) or more than 650 lbs (295 kg).

Note

Before seating patient on exam chair, ensure the scale has been zeroed. Failure to do so may affect the measurement of the patient's weight.

To zero the scale...

Before seating patient, press and release the Zero button. The display scrolls "ZEROING" and then when complete, scrolls "SCALE READY"; the zero operation is complete and the patient may be seated.

Midmark Digital Scale Functions - continued

WARNING

The scale is intended for use with patients who weigh between 30 and 650 lbs (13.6 / 295 kg). For weights below 30 lbs (13.6 kg) or above 650 lbs (295 kg), the display will flash the weight reading to indicate the weight is out of range. Do not use an out of range (flashing) weight reading for patient measurements.

Caution

For models with a wired hand control, place hand control in holster before pressing the Weight button to ensure highest scale accuracy.

To measure the patient's weight...


With patient in the seated position, press and release the Weight button.

Note: To prevent the patient's feet from touching the ground, the chair will automatically rise to a height of twenty five inches prior to displaying patient weight.

Note


Patient's weight will be displayed no more than three seconds after the "weight" button has been pressed.

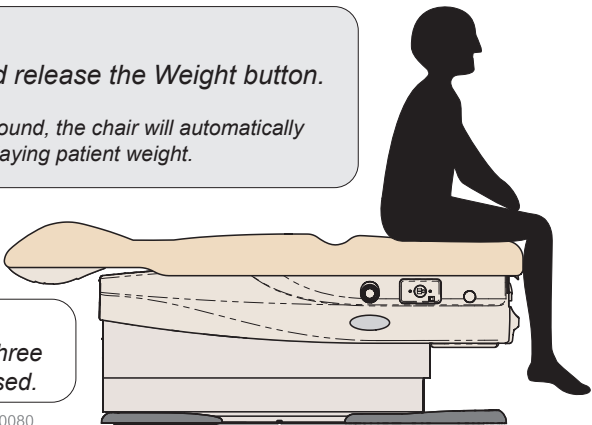MA10080

To calculate the patient's BMI...

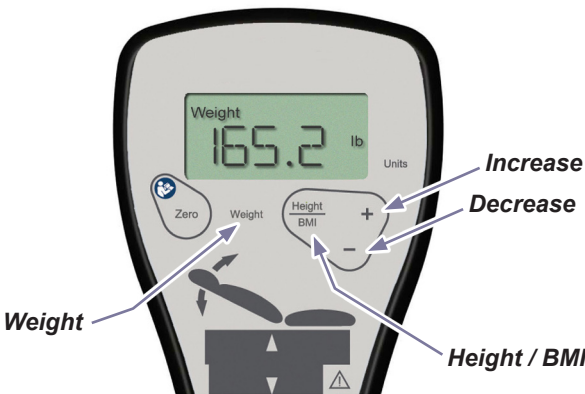- a) After patient weight is displayed, press and release the Height / BMI button.
- b) Press the (+) or (-) buttons to increase or decrease patient height on display.
- c) Press the Height / BMI button again to display patient BMI.


Auto Zero

Best practice is to always zero the scale prior to weighing the patient. For details, please refer to the weighing guidelines in this manual.

In addition, at regular intervals the digital scale will zero itself automatically. This auto zero will only be performed when there is no patient on the exam chair.

Note

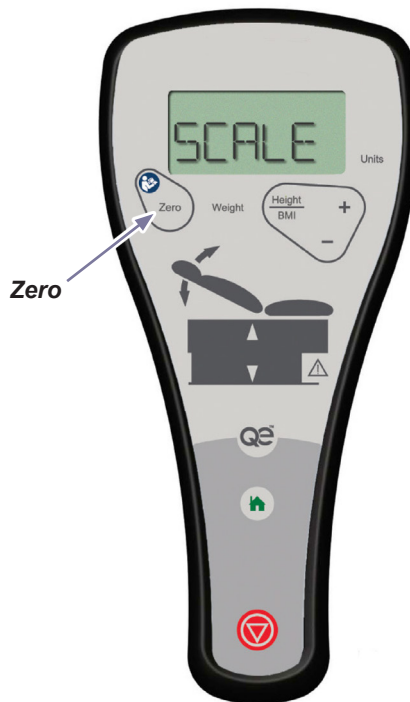Before seating patient on exam chair, ensure the scale has been zeroed. Failure to do so may affect the measurement of the patient's weight.

Units Configuration

The Units key will be used to lock the units (English or Metric), and to unlock the units. The Units key will behave differently when the units are locked than when they are unlocked.

If Units are unlocked:

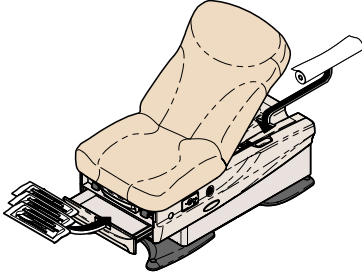
- When the display is blank releasing the Units key will show the current units indicator.
- When the display is showing only the units indicator then releasing the Units key will cause the Units indicator to toggle between displays showing inch/lb and cm/kg.
- When the display is showing only the units indicator then holding the Units key for 3 seconds will cause the LCD to scroll the following message 'UNITS', 'LOCKED', ' ' twice.
- When the display is showing Weight, pressing and releasing the Units key will toggle the units indicator between lb/kg and display the weight in the new units.
- When the display is showing Height, pressing and releasing the Units key will toggle the units indicator between in/cm and display the height in the new units.

If Units are locked:


- When the display is blank releasing the Units key will show the current Units indicator.
- When the display is showing only the units indicator then releasing the Units key will do nothing
- When the display is showing only the units indicator then holding the Units key for 3 seconds will cause the LCD to scroll the following message 'UNITS', 'UNLOCKED', ' ' twice.
- When the display is showing Weight, pressing the Units key will toggle the units indicator between lb/kg and display the weight in the new units. Releasing the Units key will toggle the units indicator back to the previous units.
- When the display is showing Height, pressing the Units key will toggle the units indicator between in/cm and display the height in the new units. Releasing the Units key will toggle the units indicator back to the previous units.

To ensure proper weighing of patients please review the following guidelines.


Do zero the scale after adding or removing supplies.

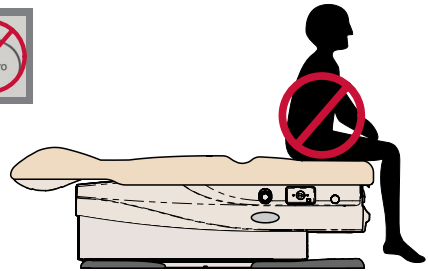
Don't zero the scale while the patient is on the chair.

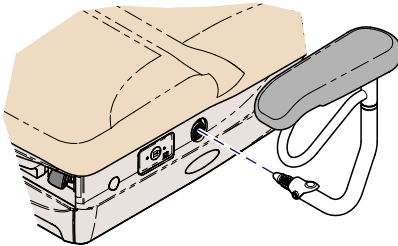
Do zero the scale after adding or removing equipment.

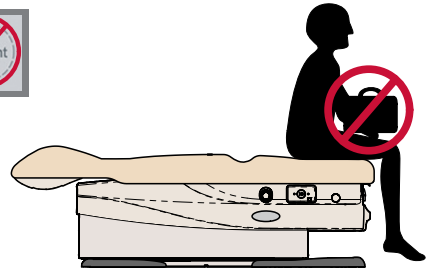
Don't weigh patients holding items such as purses, coats etc.

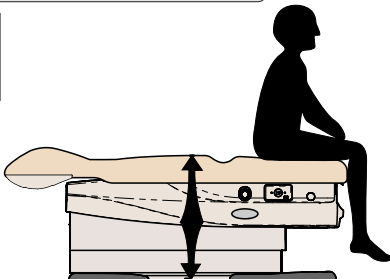Do ensure the patient's feet are off the floor during weighing.

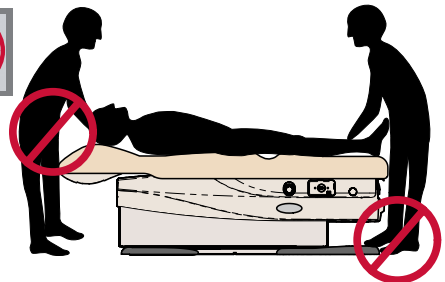
Don't lean, step or otherwise apply load to chair while measuring weight.

Hand / Foot Control Connection

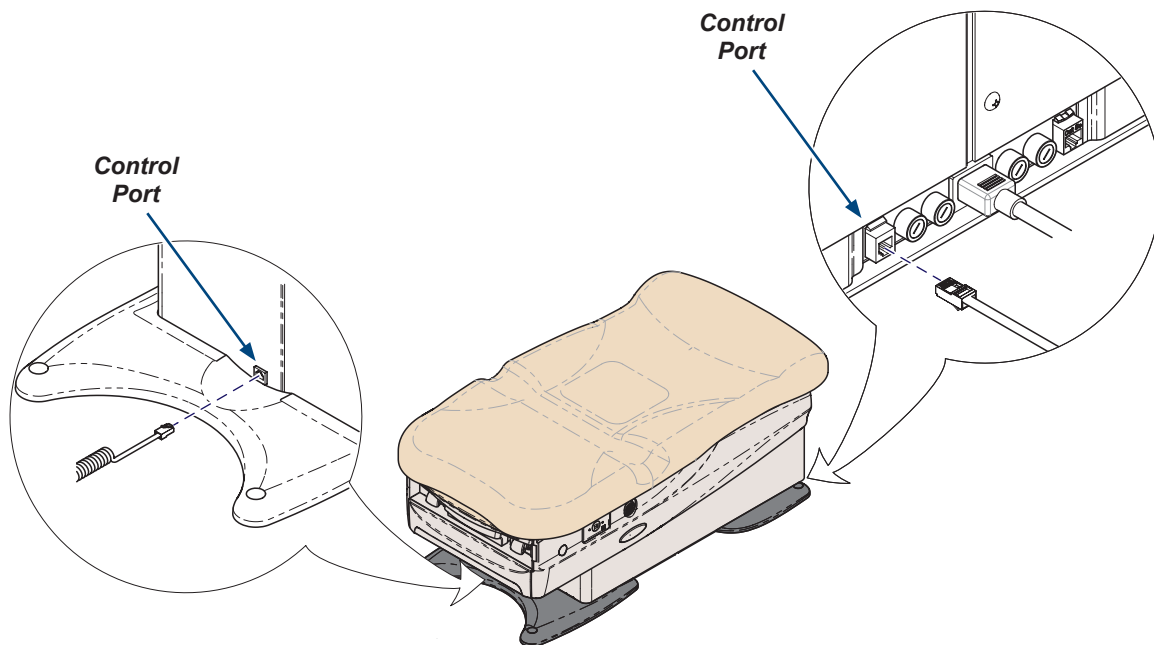Although wireless controls are standard on some models, there are hand / foot control ports for wired controls located on the base at the head and foot end of the chair.

Operation

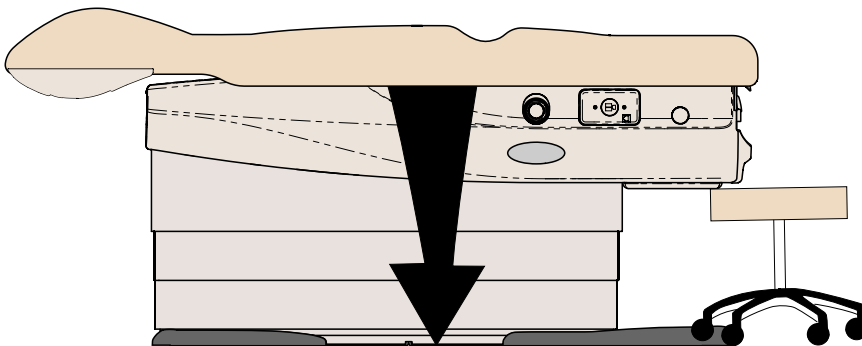
Active Sensing Technology®

WARNING

Be sure that all personnel and equipment are clear of the chair before activating any function. Failure to do so could result in personal injury.

The Active Sensing Technology™ prevents the chair from moving Down if the bottom of the drawer contacts another object. If the Active Sensing Technology™ is activated, it will sound a series of “Beeps,” disable the Base Down function and move chair in the Base Up direction. To resume Base Down operation, remove object from under the bottom of the drawer and press the Base Up / Down buttons on controller.

MA100871

Operation - continued

Base / Back Functions

Attention!

The Active Sensing Technology® will stop the Base Down function and retract Up if the bottom of the drawer contacts another object.

(You will hear a series of “Beeps” if the Active Sensing Technology is activated.)

To resume Base Down operation:

Remove object from under the bottom of the drawer.

Press Base Down button.

To operate the Base / Back functions...

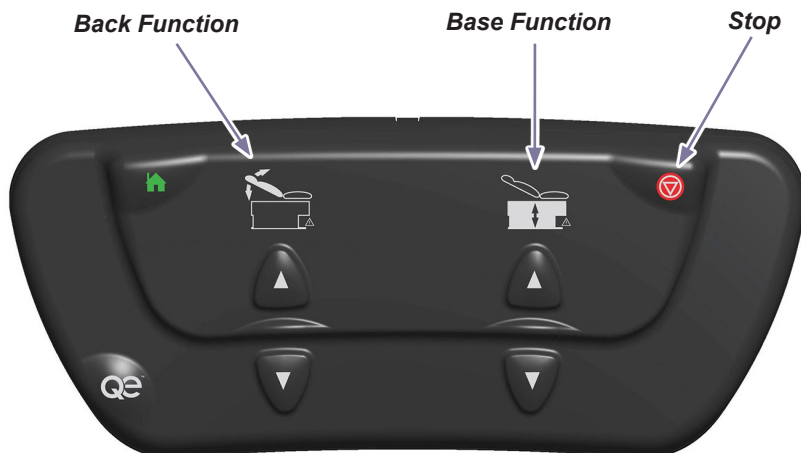
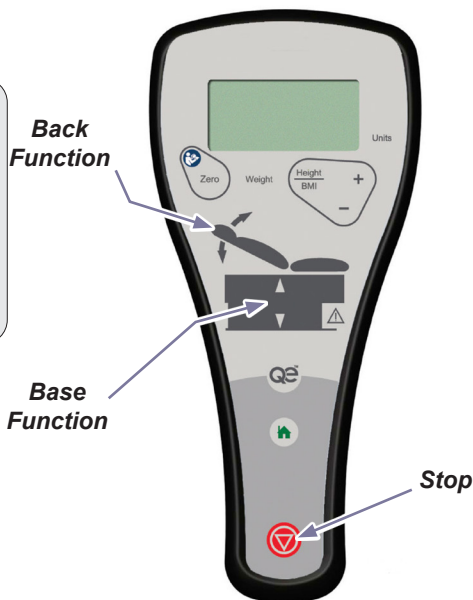
Press and hold the Up or Down arrow button for the desired function.

Note: The chair automatically stops at its maximum and minimum height. At anytime, the Stop button can be pressed to terminate movement.

Equipment Alert

Controllers must be within a Fifteen foot radius of chair to be able to command the chair.

Programmable 'Home' Function

The Home function automatically commands the chair to a programmed position.

The Home function is factory preset to automatically raise or lower the chair height to 17 inches (43 cm) and raise the Back angle to 80 degrees.

To program the Home function...

- Press the Base Up / Down button to move the Base to the desired height.
- Press the Back Up / Down button to move the Back to the desired angle.
- Press the Stop button, then the Home button and hold simultaneously for three to four seconds.

Note: You will hear an audible tone when the new 'Home' position is saved.

To activate the Home function...

Press and release the Home button.

Note: At any time, ANY button can be pressed to terminate movement.

Equipment Alert

Controllers must be within a Fifteen foot radius of chair to be able to command the chair.

Note

An audible tone will sound when the Home button is pressed to indicate auto-movement.

Quick Exam® Function

The Quick Exam function automatically commands the chair to a programmed position. The Quick Exam function is factory preset to automatically raise or lower the chair height to 34 inches (86 cm) and raise the Back angle to 80 degrees.

To program the Quick Exam function...

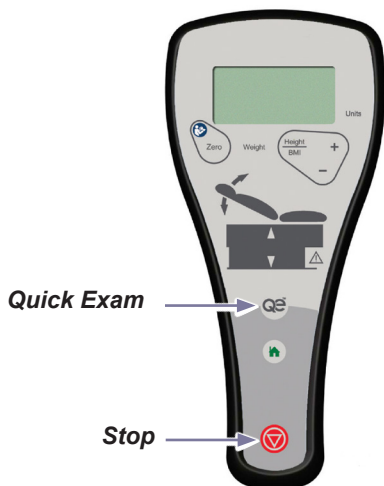
- Press the Base Up / Down button to move the Base to the desired height.
- Press the Back Up / Down button to move the Back to the desired angle.
- Press the Stop button, then the Quick Exam button and hold simultaneously for three to four seconds.

Note: You will hear an audible tone when the new Quick Exam position is saved.

Note

An audible tone will sound when the Quick Exam button is pressed to indicate auto-movement.

To activate the Quick Exam function...

Press and release the Quick Exam button.

Note: At any time, ANY button can be pressed to terminate movement.

Equipment Alert

Controllers must be within a Fifteen foot radius of chair to be able to command the chair.

Control Lockout

The Control Lockout feature provides a means to lock out any movement of the Base and Back functions on the chair. This includes movement initiated by the hand and foot controls. It will also lock out the ability to receive weight from the scale to the hand control.

Note

The factory default setting for the Control Lockout is “disabled” and requires the operator to enable it.

Equipment Alert

Controllers must be within a Fifteen foot radius of chair to be able to command the chair.

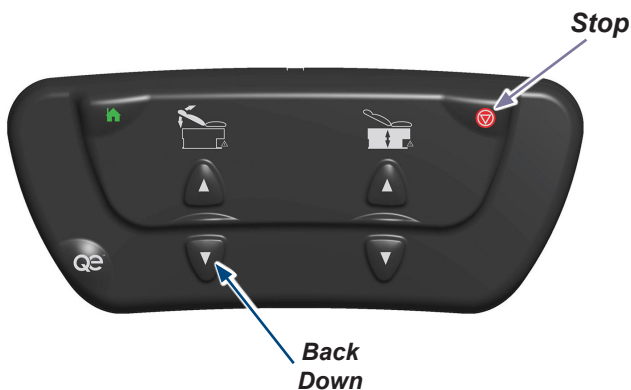
To enable / disable the Control Lockout feature...

Press and hold the Stop and Back Down buttons simultaneously for three to four seconds.

You will hear...

- When locked, a single “Beep”
- When unlocked, “Beeps” until buttons are released.

Note: If motion is attempted while the controls are locked out, you will hear a series of “Beeps” until motion is no longer attempted.

Association Procedure

The wireless controllers will only function with the chair they have been “associated” with. This prevents unwanted movement in locations with more than one chair. Each wireless controller’s association is preset at the factory. This procedure is only required in the event of multiple controllers being used for more than one chair or any time a controller is being used with a chair other than the one it was previously associated with.

Equipment Alert

A maximum of one hand control and one foot control may be associated to the chair at any one time.

Equipment Alert

Controllers must be within a Fifteen foot radius of chair to be able to command the chair.

Association Procedure...

*Disconnect power for at least three seconds.
Reconnect power and wait until the chair “Beeps” three times.
Press and hold the Stop and Base Down buttons until the Control “Beeps” three times.*

Note: Two “Beeps” indicates an unsuccessful association.

Paper Roll and Tear Strap

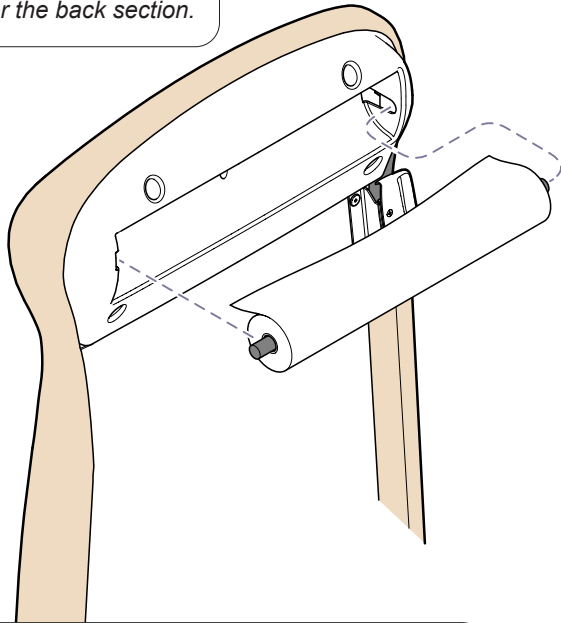
Paper Roll Size (maximum):..... 24 in. wide x 3.5/8 in. diameter
(61 cm x 9.2 cm)

Note

One additional paper roll may be stored under the back section.

To install paper roll...

*Insert rod ends into sockets
under head end of upholstery.*


WARNING

To avoid injury, do not put head or extremities under the paper tear strap.

To install paper tear strap...

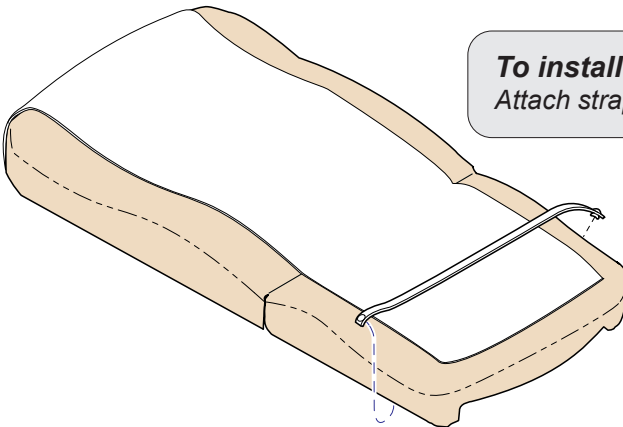Attach strap to snaps on each side of chair.

MA10088I

Stirrups

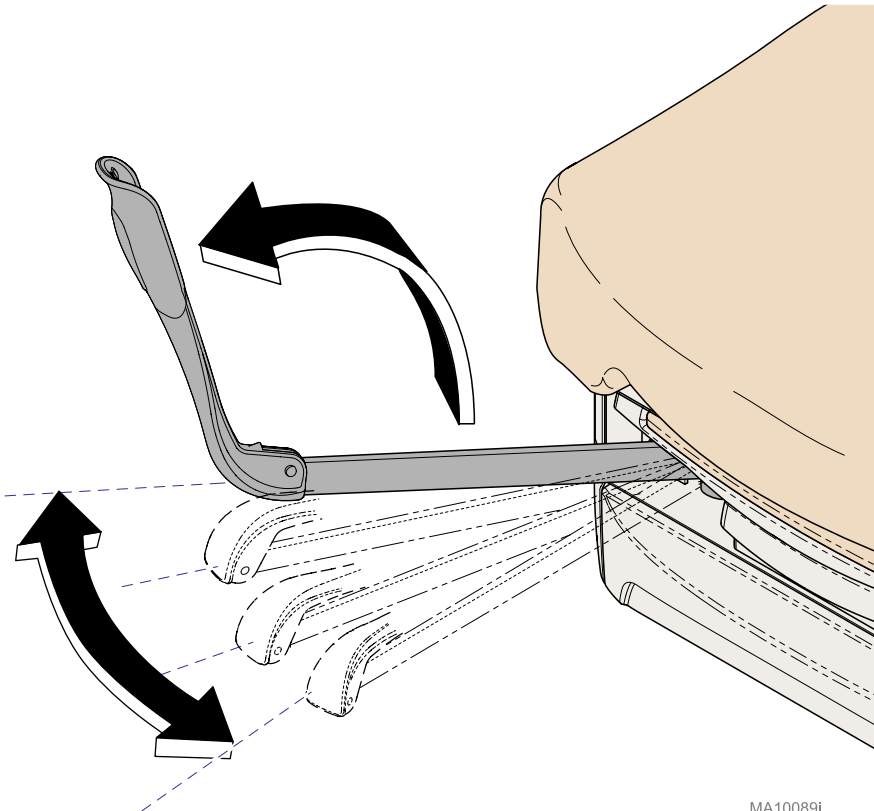
Caution

*Be sure the stirrups are locked in place before using.
The stirrups will not support the patient's entire weight.*

To position the stirrups...

- A) *Pull the stirrup out, then unfold.*
- B) *Lift the stirrup slightly, then move it left or right as desired.*
- C) *Release stirrup to lock in position.*

MA10089i

Footrest and Treatment Pan

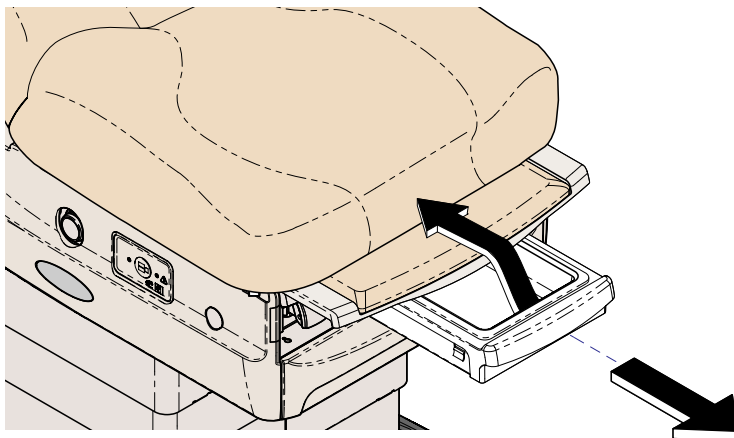
WARNING

Do not use footrest to support patient's entire weight.

To extend footrest...

Pull footrest out to desired position.

MA100901

To access the treatment pan...

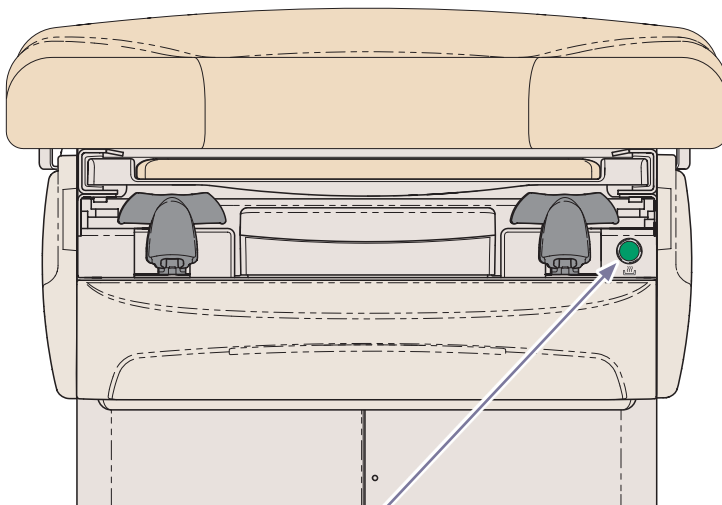
Pull treatment pan out to desired position.

Drawer Heater

The drawer heater warms to 92°F (33°C) to 112°F (44°C) measured at the center of the drawer at room temperature and nominal input voltage.

To turn the drawer heater ON / OFF...
Press and release the drawer heater switch.

Note: Switch will illuminate when heater is ON.

***Drawer Heater
Switch***

MA10091i

Upholstery Heater (-004 /-006)

The nominal surface temperature of the LOW setting is approximately 95° +/- 5° F, and the nominal surface temperature of the HIGH setting is approximately 102° +/- 5° F at an ambient temperature of 70° +/- 2° F. There will be approximately a difference of 7° +/- 2° F between the high and low setting. The upholstery heater will automatically shut off after ten hours of operation. It takes the upholstery approximately 20 to 25 minutes to reach 99° F from an ambient temperature of 73° F.

Caution

Use of the upholstery heater in combination with other heating devices could cause elevated body temperature.

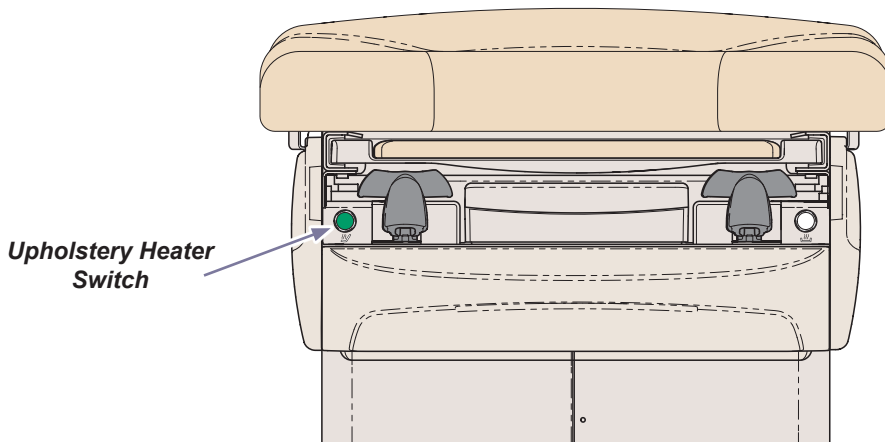
To turn the upholstery heater ON / OFF...

Press and release the upholstery heater switch once for the LOW setting and twice for the High setting. Pressing the heater switch a third time will turn upholstery heater off.

Note: Switch will illuminate "Dim Green" on the LOW setting, and illuminate "Bright Green" on the HIGH setting.

Note

Harnesses protruding out of the chair from underneath the back weldment are to be used for heated upholstery only.

MA10104I

Pelvic Tilt

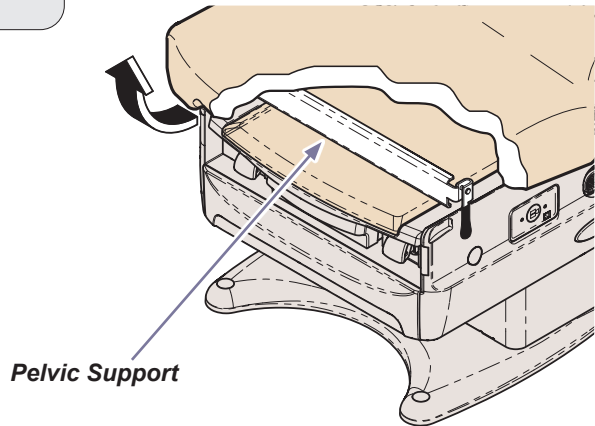
WARNING

To avoid risk of injury do not put extremities under the seat.


To elevate seat section...

Lift seat until pelvic support drops into place.

To return seat section to flat position...

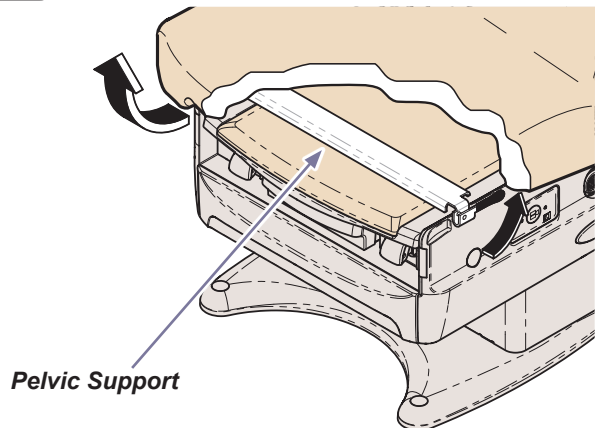
- A) *Lift seat section slightly.*
- B) *Pivot pelvic support handle upward.*
- C) *Lower seat section.*

Simplex Receptacles

The simplex receptacle on each side of chair provides power for accessories used during medical procedures. There are two circuit breakers located under the seat section. If the receptacles' maximum load is exceeded, the circuit breakers interrupt power to the receptacles.

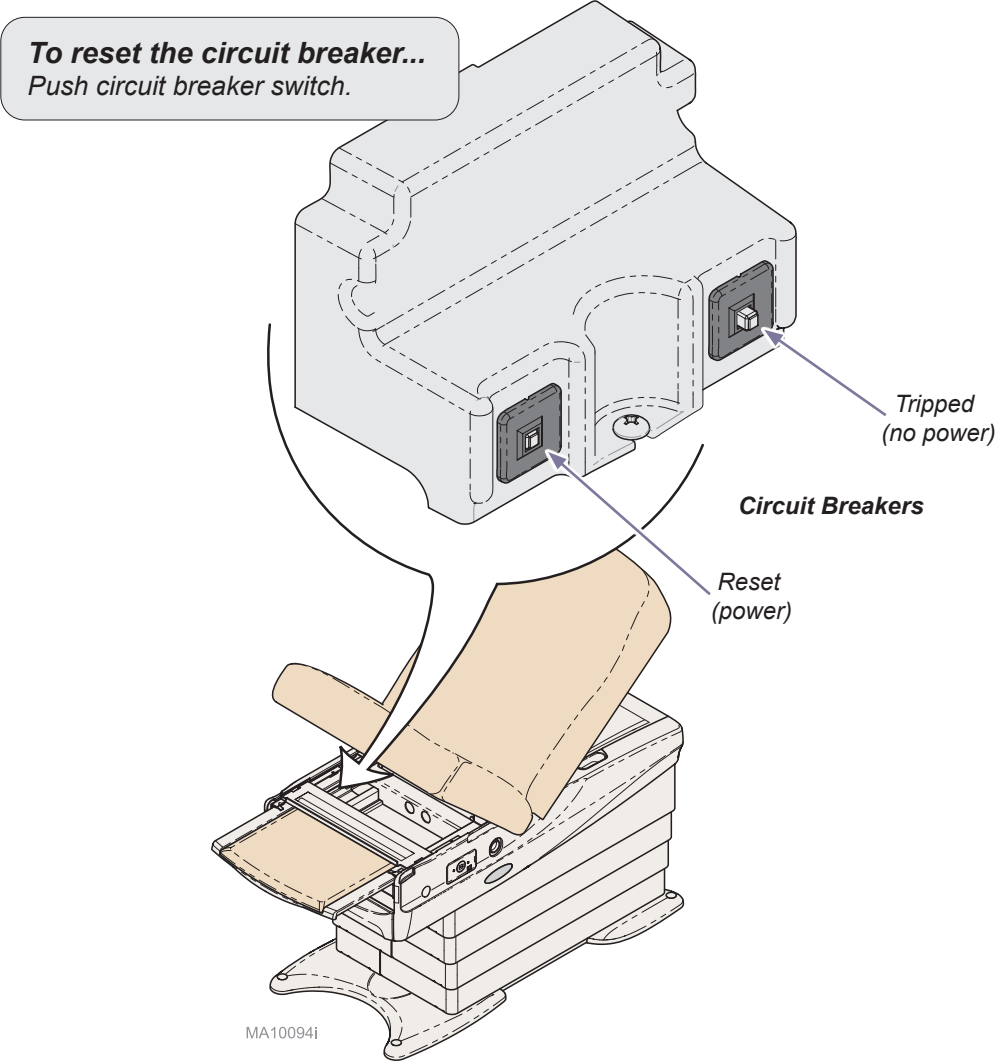Maximum Load.....115 VAC, 4 amps


Data Connections

Data connections are located on each side of chair and on rear base of chair.

Port 1, 2 and 3: Vital Signs connections.

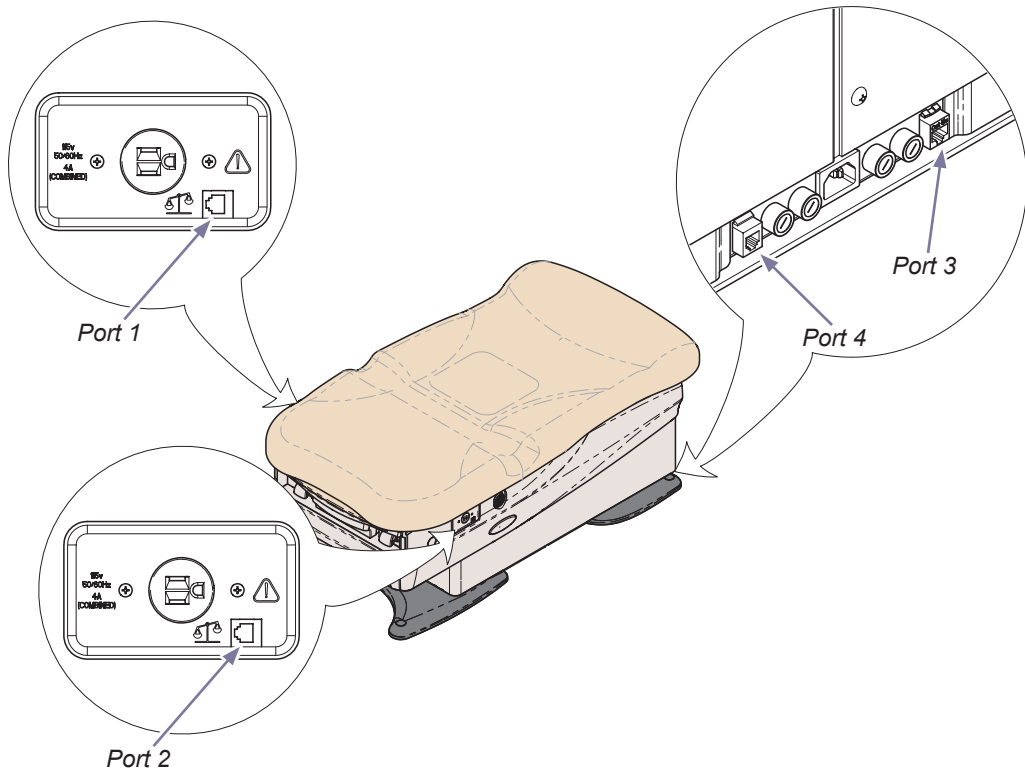Port 4: Optional wired hand or foot control.

Data Connections - continued

Vital Signs Connections

The serial ports on the chair provide connection to a vital signs device. This allows for the patient weight to be captured from the digital scale and transferred to the vital signs device for display and capture.

Supported vital signs devices are:

- Midmark Digital Vital Signs Device
- Welch Allyn® Spot Vital Signs® LXi *
- Midmark IQvitals® Zone™

Setup:

Purchase the appropriate serial cable for your equipment:

- Midmark Digital Vital Signs Device and IQvitals Zone Serial Cable, 6 foot length, coiled (9A478001).
- Midmark Digital Vital Signs Device and IQvitals Zone Serial Cable, 15 foot length, straight (9A478002).
- Midmark Digital Vital Signs Device and IQvitals Zone Serial Cable, 30 foot length, straight (9A478003).
- Midmark Digital Vital Signs Device and IQvitals Zone Serial Cable, 50 foot length, straight (9A478004).
- Welch Allyn Spot Vital Signs LXi Serial Cable, 6 foot length, coiled (9A479001).
- Welch Allyn Spot Vital Signs LXi Serial Cable, 15 foot length, straight (9A479002).

Connect one of the vital signs connections (ports 1, 2, or 3) to one end of the serial cable, and the other end to the corresponding port on your vital signs device.

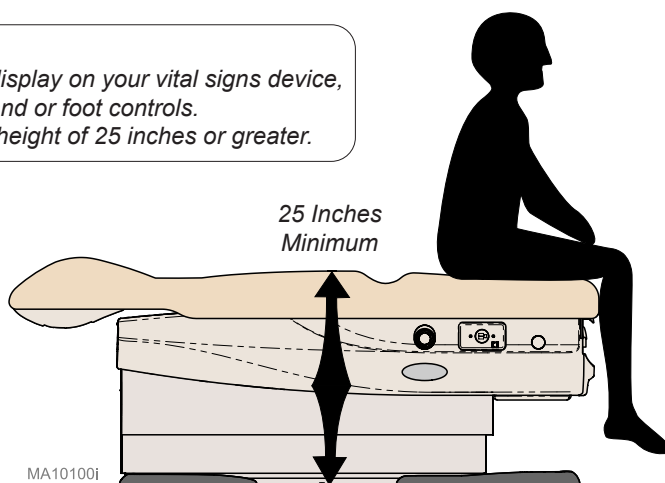
Patient weight will automatically be measured when the patient sits on the chair.

Note

*Before seating patient on exam chair, ensure the scale has been zeroed.
Failure to do so may affect the measurement of the patient's weight.*

Note

*If weight does not display on your vital signs device,
raise chair using hand or foot controls.
Chair must be at a height of 25 inches or greater.*




* Welch Allyn® and Spot Vital Signs® are registered trademarks of Welch Allyn, Inc.

Data Connections - continued

Connecting Chair to Vital Signs Device

The serial cable may be connected to one of three ports located on the chair.

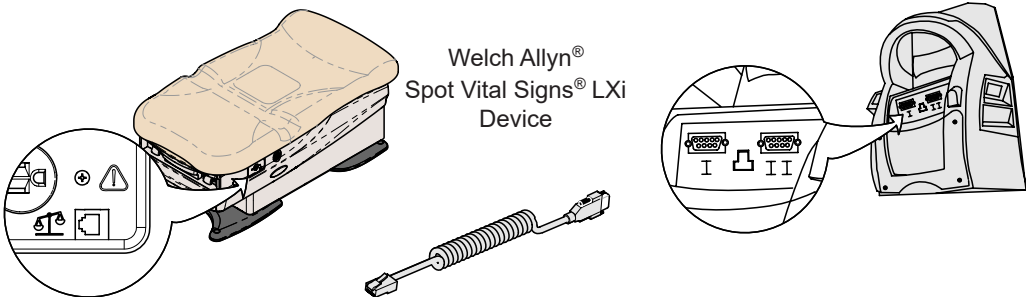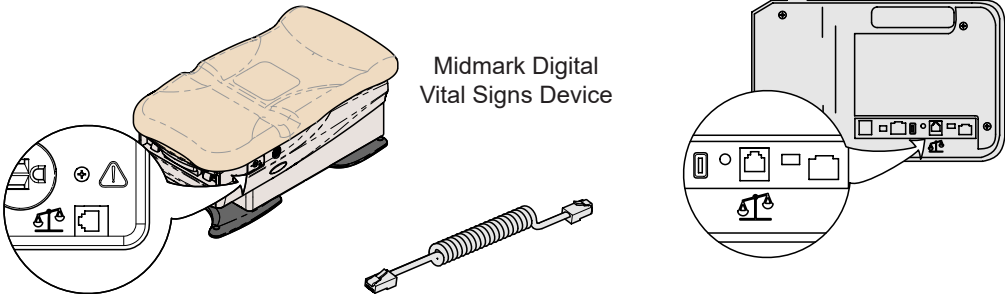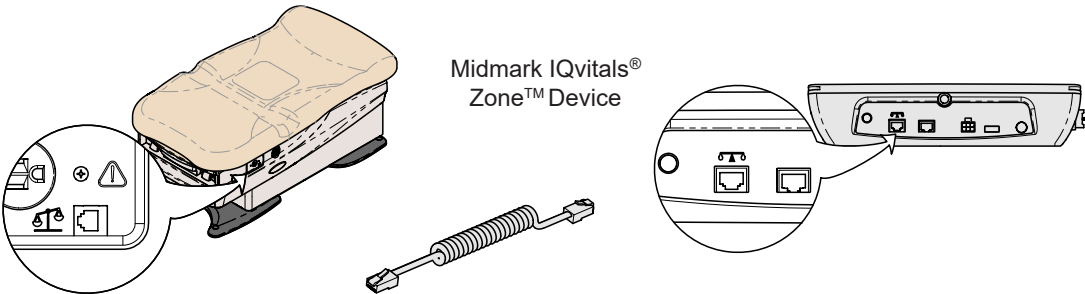Note
Only one vital signs device may be connected at a time.

Clean Assist Roller System

Control levers located under the back section on each side, activate and deactivate the **Clean Assist Roller System**. The rollers are located on each corner of the chair to allow positioning of chair.

Caution

To prevent injury, never activate / deactivate Clean Assist Roller System or move chair with patient aboard. Use caution when moving chair.

Equipment Alert

The intended use of the Clean Assist Roller System is for chair positioning within the exam room only.

Equipment Alert

Be sure to activate both control levers.

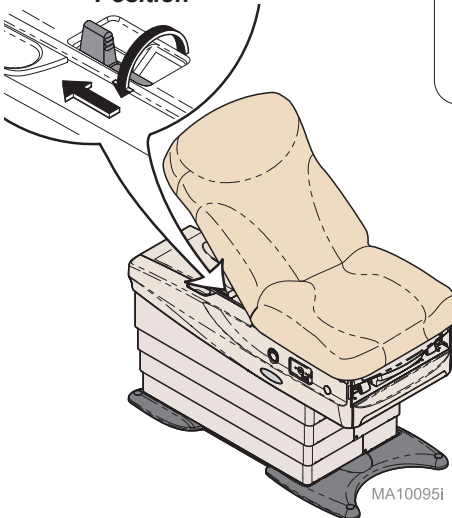
WARNING

To avoid risk of fall, the Clean Assist Roller System must be disengaged while a patient ingresses or egresses the chair.


To activate Clean Assist Roller System...

- Raise back section 30 to 45 degrees.
- Ensure chair is at least two inches higher than its lowest position.
- Move control levers on each side of chair to the Clean Assist Roller System engagement position.
- Move chair to the lowest position to activate rollers.

Clean Assist Roller System Disengagement Position

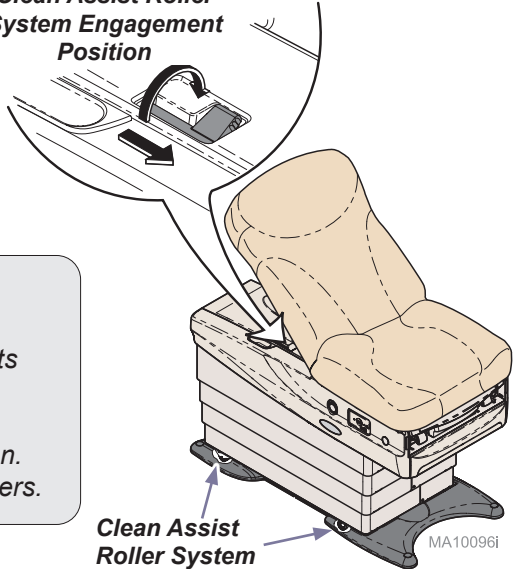Clean Assist Roller System Engagement Position

Equipment Alert

After Clean Assist Roller System has been activated, disconnect power cord and foot control and stow away from chair to prevent damage to cords.

To deactivate Clean Assist Roller System...

- Move chair to desired location.
- Reconnect power cord.
- Raise chair at least 2" to deactivate Clean Assist™ Roller System.

Note: An audible click will occur to confirm deactivation.

Note

Control levers will automatically reset to the Clean Assist Roller System disengagement position. Zero the scale after the chair is moved for cleaning.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM Friday 8:00 AM until 5:00 PM (ET)

Cleaning

WARNING

The upholstery should be inspected before each use and replaced if punctured or otherwise damaged.

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [midmark.com](https://www.midmark.com) in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

- Power cord(s) should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

Periodically lubricate the back hinge to maintain quiet, smooth, operation (use light machine oil).

Have a qualified service technician inspect your equipment every twelve months.

Periodic calibration of the scale is not required. It is recommended to verify the scale is measuring accurately using a reference weight of known value every six months.

WARNING

Should damage and or loose parts be identified, the examination chair should be removed from use and serviced. Failure to do so may result in injury.

Troubleshooting

Symptom	Probable Cause	Correction
No functions will operate.	Facility supply voltage.	Secure power cord connections. Reset facility circuit breaker.
Chair stops and beeps.	Patient weight exceeded 650 lbs.	Inform staff that the maximum patient weight limit is 650 lbs.
No power at chair receptacles. All other functions work.	Chair circuit breaker(s) tripped.	Lift seat section to access circuit breaker(s), press to reset.
When any control button is pressed, that controller sounds a single beep. (All functions operate)	Low battery.	Replace batteries. (<i>size: AA</i>)
When the weight button is pressed, the controller displays "Chair Low".	Momentary power interruption.	Lower chair all the way down.
Chair does not move when pressing Quick Exam® button, or does not go to the correct position.	Quick Exam® positioning is user adjustable and may need to be reprogrammed.	See Quick Exam® instructions in this manual.
	Active Sensing Technology™ activated.	Remove object from under the bottom of the drawer.
Chair does not move when pressing 'Home' button, or does not go to the correct position.	'Home' positioning is user adjustable and may need to be reprogrammed.	See 'Home' instructions in this manual.
	Active Sensing Technology™ activated.	Remove object from under the bottom of the drawer.

Specifications



Specifications Chart	
Patient Weight (maximum):	650 lbs (295 kg)
Scale Accuracy ≤ 12 hours after using Clean Assist Roller system*: 30 - 100 lb (13.6 - 45.4 kg) 100 - 250 lb (45.4 - 113.4 kg) 250 - 650 lb (113.4 - 294.8 kg) <i>*Note: While table stabilizes on floor substrate.</i>	± 2 lb (0.9 kg) ± 3 lb (1.4 kg) ± 4 lb (1.8 kg)
Scale Resolution: Scale Accuracy: 30 - 100 lb (13.6 - 45.4 kg) 100 - 300 lb (45.4 - 136.1 kg) 300 - 500 lb (136.1 - 226.8 kg) 500 - 650 lb (226.8 - 294.8 kg)	0.2 lb (0.1kg) ± 0.2 lb (0.1 kg) ± 0.4 lb (0.2 kg) ± 0.6 lb (0.3 kg) ± 0.8 lb (0.4 kg)
Paper Roll (maximum size):	24 in. long x 3.5/8 in. diameter (61 cm x 9.2 cm)
Range of Motion & Dimensions:	(See <i>Range of Motion & Dimensions</i> page)
Weight of Chair: w/upholstery w/packaging & skid (no uph.) Uph. w/packaging (shipped separately)	475 lbs (215.5 kg) 485 lbs (220 kg) 40 lbs (18.1 kg)
Power Cord Length:	8 ft. (244 cm)
Electrical Requirements:	[See <i>Model Identification / Compliance Chart</i>]
Foot / Hand Control Voltage:	3.3 VDC, SELV (Safety Extra Low Voltage)
Simplex Receptacles (maximum load)	115 VAC, 4A, 50/60 Hz
Fuses: <i>IEC inlet fuses</i> <i>F1 (on Main PC board)</i> <i>F2 (on Main PC board)</i> <i>Models w/ upholstery heaters</i> <i>F1 & F2 (on Uph Heater PC board)</i>	T8AH, 250V, 5 x 20 mm T250mA, 250V, 5 x 20 mm T6.3AH, 250V, 5 x 20 mm T3.15AH, 250V, 5 x 20 mm
Duty Cycle (Motor Run Time):	Intermittent Operation [30 seconds ON / 5 minutes OFF]
Receptacles, Drawer Heater:	Continuous Operation
Classifications:	Class I, Type B Applied Part, except as noted for optional heated upholstery, Intermittent Operation [30 seconds ON / 5 minutes OFF]
Certifications	ES 60601-1:2012, CSA C22.2 No. 60601-1:14; AAMI/ISO 80601-2-35:2009, CSA C22.2 No. 80601-2-35:12, IEC 60601-1-2:2014
	Optional heated upholstery, Type BF Applied Part, complies with IEC 80601-2-35
	IPX0 Chair IPX1 Foot Control IPX2 Heated Upholstery
Transmit / Receive Frequencies	2.402GHz - 2.480GHz

Model Identification / Compliance Chart


Model	Description	Complies To:				Electrical Ratings:		
		ANSI/AAMI ES60601-1	CAN/CSA-C22.2 No. 60601-1:14	IEC 60601-1-2	NFPA 99	VAC +/-10%	Amps	Cycles (Hz)
626-003	Two-function chair (Base / Back) w/ Simplex Receptacles, Recessed Rollers, Pelvic Tilt, Drawer Heater, Wireless Controls, Midmark Digital Scale, & IQhub®	•	•	•	•	115	12	50 / 60
626-004	Two-function chair (Base / Back) w/ Simplex Receptacles, Recessed Rollers, Pelvic Tilt, Drawer Heater, Heated Upholstery, Wireless Controls, Midmark Digital Scale, & IQhub®	•	•	•	•	115	12	50 / 60
626-005	Two-function chair (Base / Back) w/ Simplex Receptacles, Recessed Rollers, Pelvic Tilt, Drawer Heater, Wired Controls, Midmark Digital Scale, & IQhub®	•	•	•	•	115	12	50 / 60
626-006	Two-function chair (Base / Back) w/ Simplex Receptacles, Recessed Rollers, Pelvic Tilt, Drawer Heater, Heated Upholstery, Wired Controls, Midmark Digital Scale, & IQhub®	•	•	•	•	115	12	50 / 60

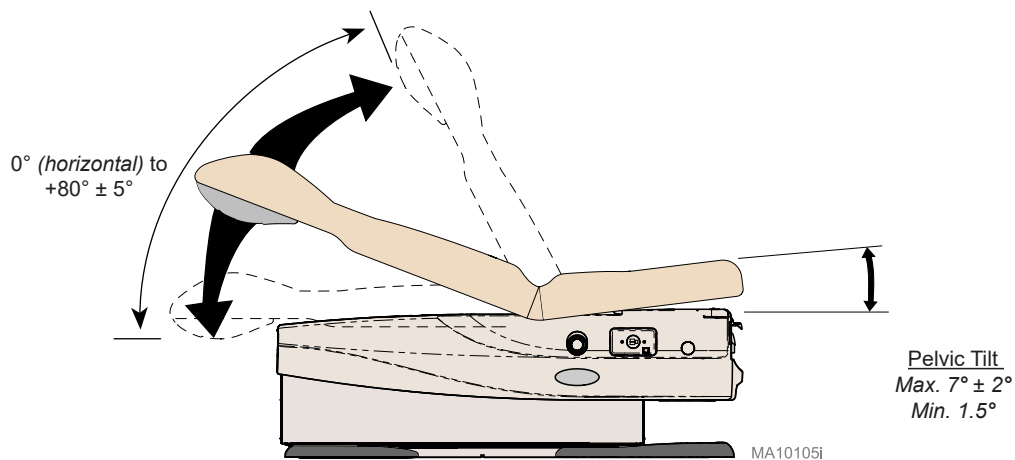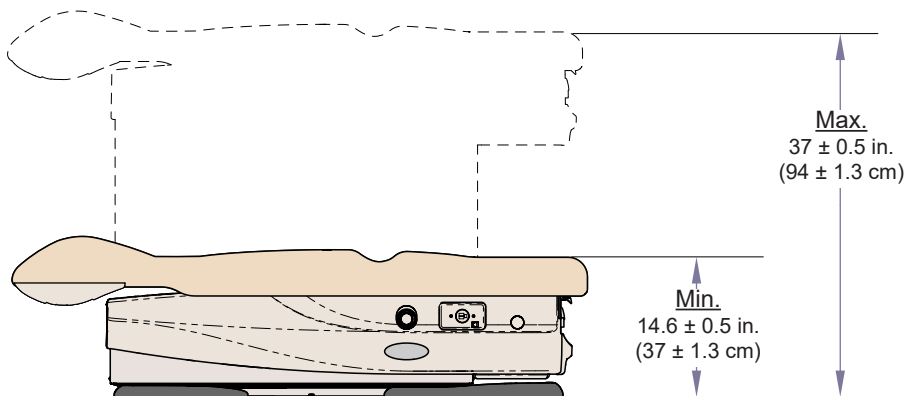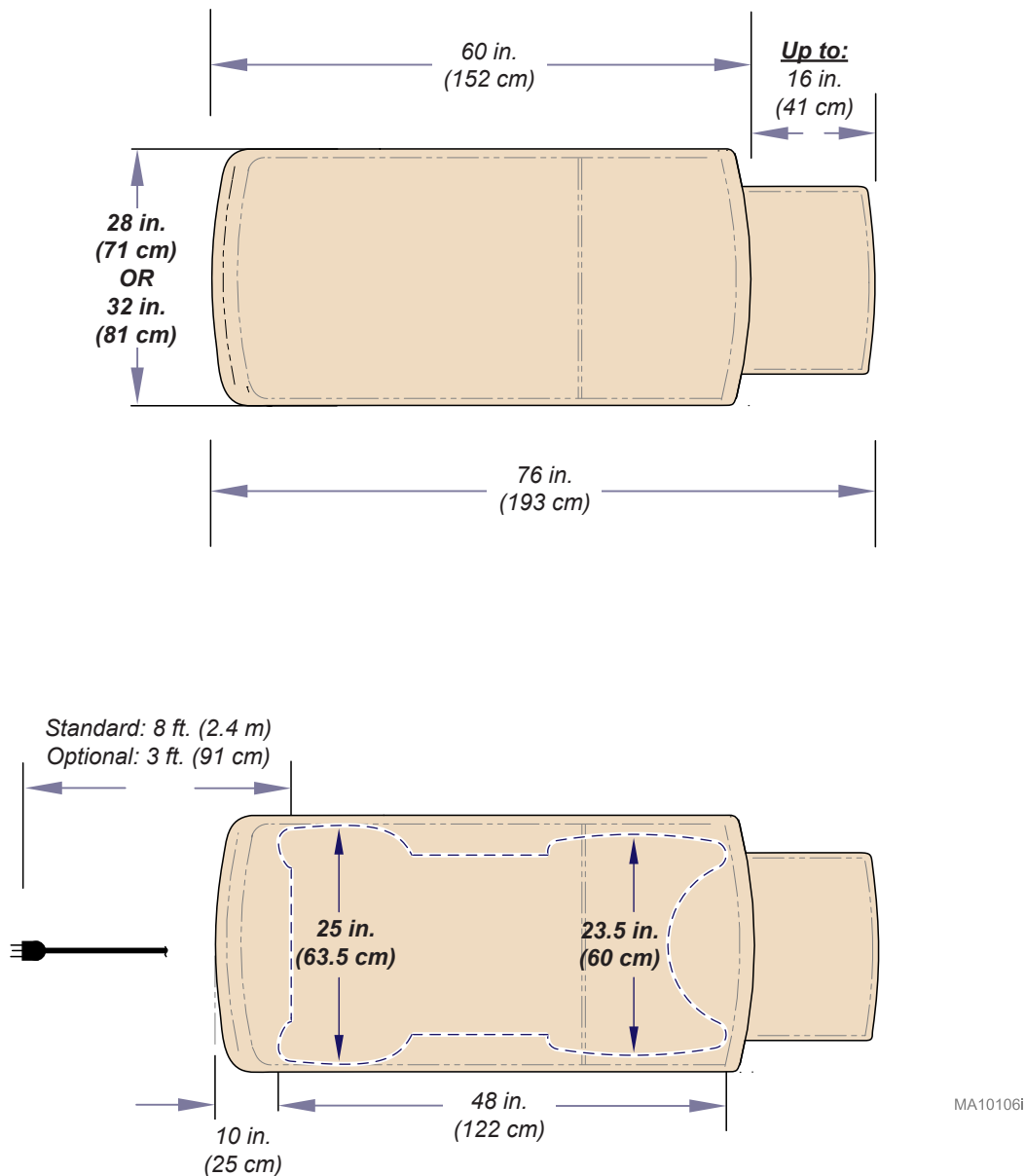Fire Code Ratings: All upholstery complies with California Bureau of Home Furnishing [Technical Bulletin 117](#) and California Code of Regulations, Sect. 93120-93120.12, Title 17. Optional upholstery is available that complies with California Bureau of Home Furnishing [Technical Bulletin 133](#).

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Midmark Corporation