224 / 225 Barrier-Free® Power Examination Chair Setup Instructions

Tools:

Battery Powered Drill / Impact Driver
Standard Wrench Set
Standard Deep Well Socket Set
Box Cutter
Furniture Dolly (4 Wheel)

Caution

Heavy object. Use lifting aids and proper lifting techniques when moving chair to minimize muscle strains or back injury.

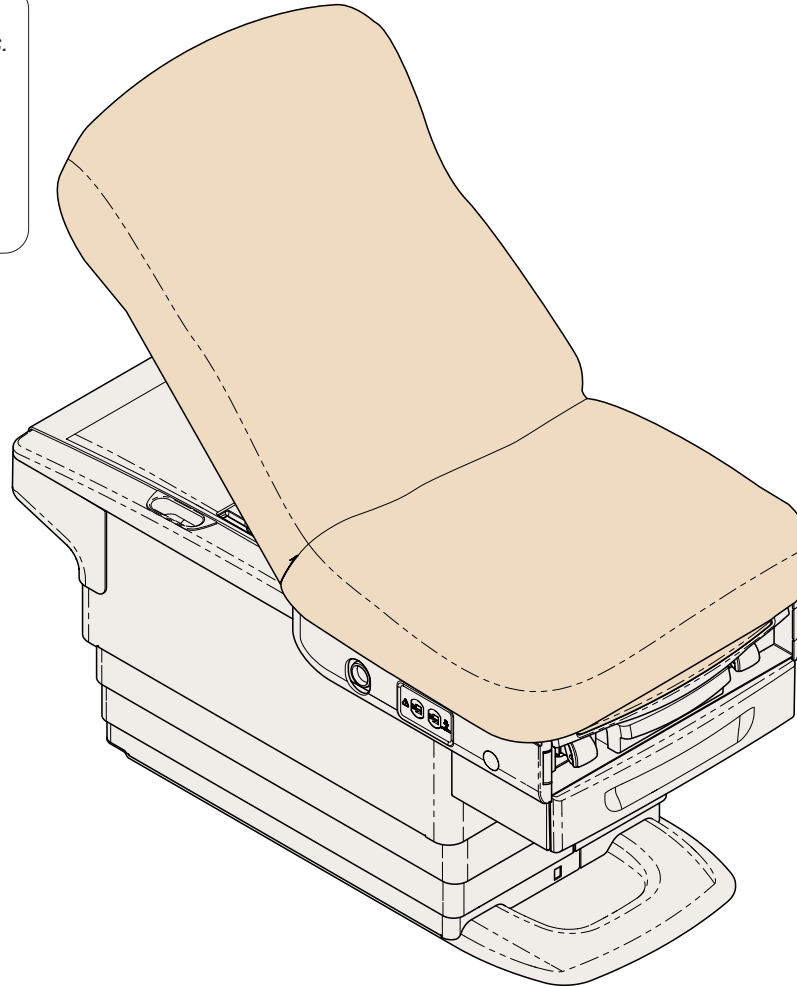
Equipment Alert

When moving the chair, you must adhere to the following instructions.

ALWAYS: a) Transport chair in full-down position with base shroud supports in place.


NEVER: a) Push on base shrouds.
b) Lift chair by stirrups, back section, or upholstered top.


0 0 3 - 1 0 0 3 3 - 9 9

MA10250i

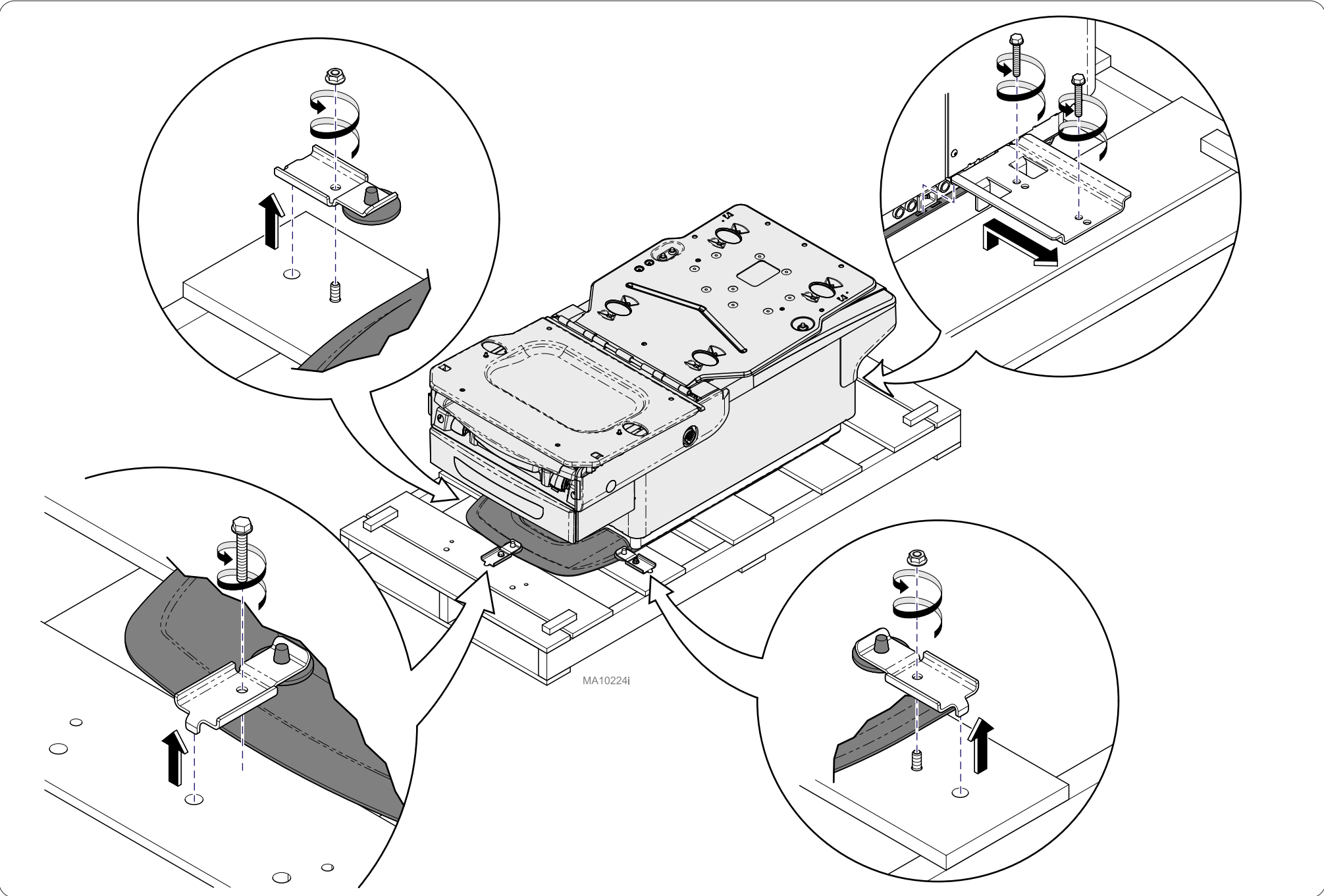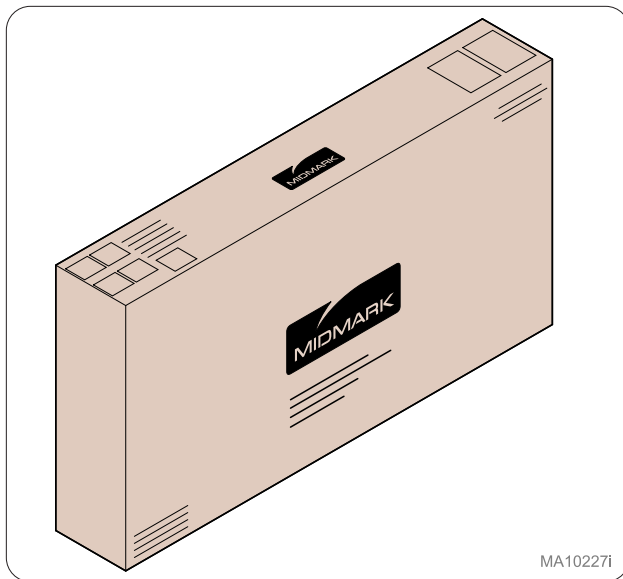
Step 1:
Remove brackets (4) from skid to release chair.


MA10227i

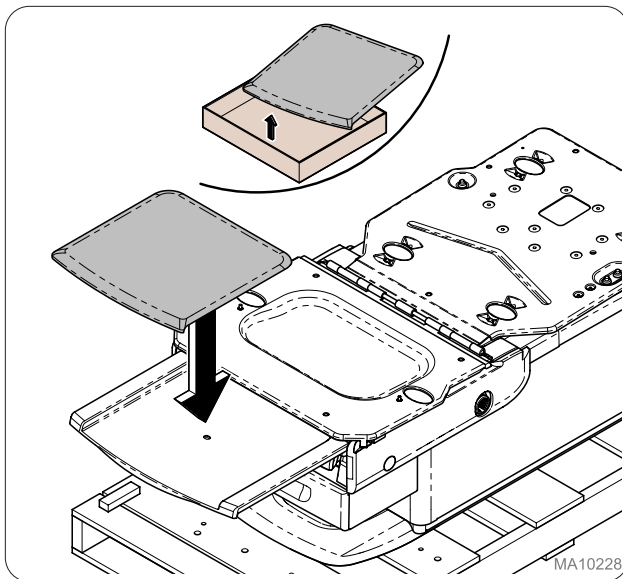Step 2:
Locate box containing upholstery.


MA10228i

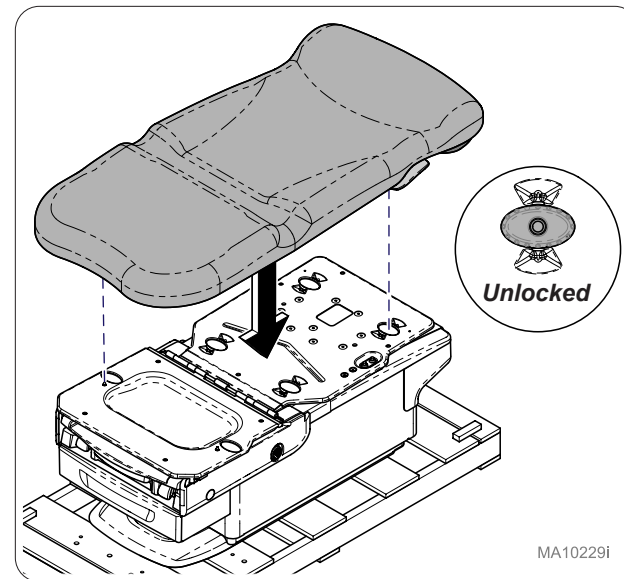Step 3:
Place footrest upholstery, inserting pin on bottom of upholstery into footrest shelf hole.


MA10229i

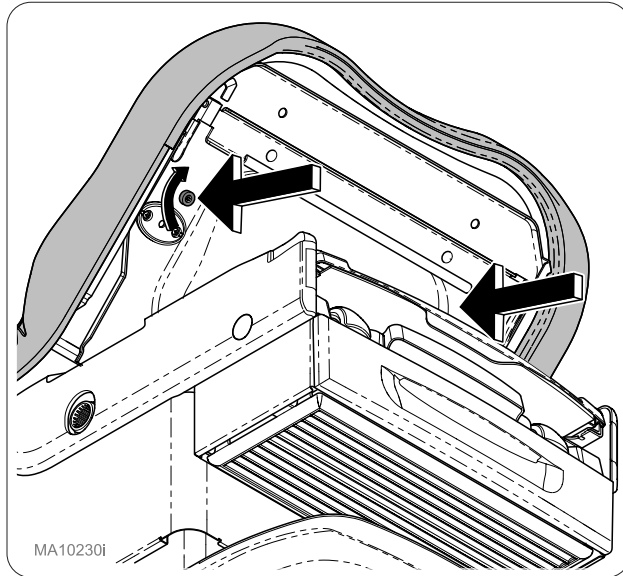Step 4:
With locking knobs (4) in the unlocked position, place upholstery on chair top aligning locking knobs with holes.


MA10230i

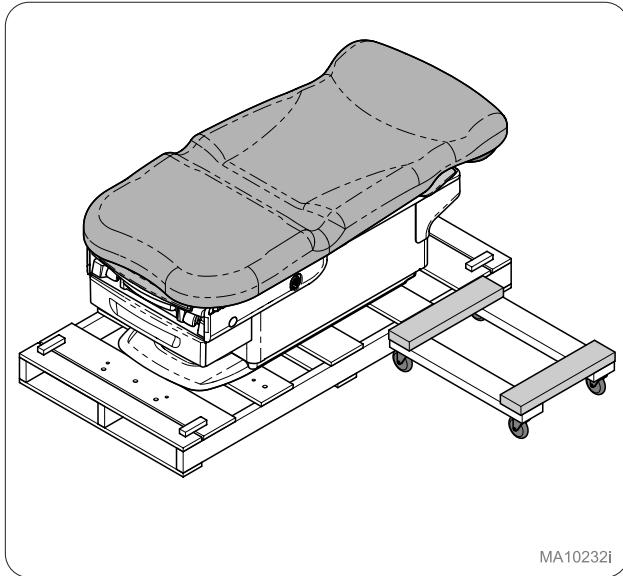Step 5:
Raise seat pan 90 degrees and secure seat section using attached thumb screws (2). Lower seat pan.


MA10232i

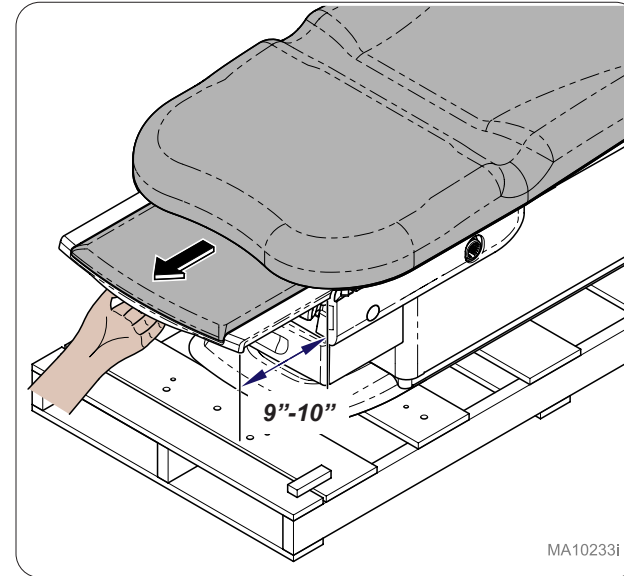Step 6:
Place dolly next to head end of chair.

MA10233i

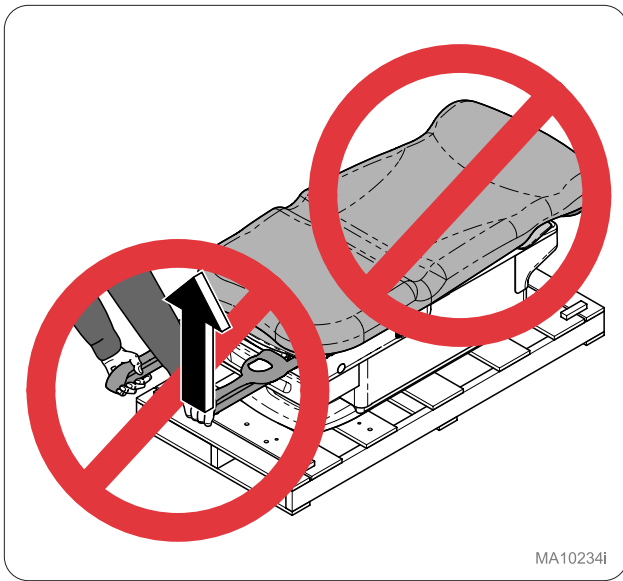Step 7:
Slide footrest 9"-10" out of chair.


MA10234i

Equipment Alert

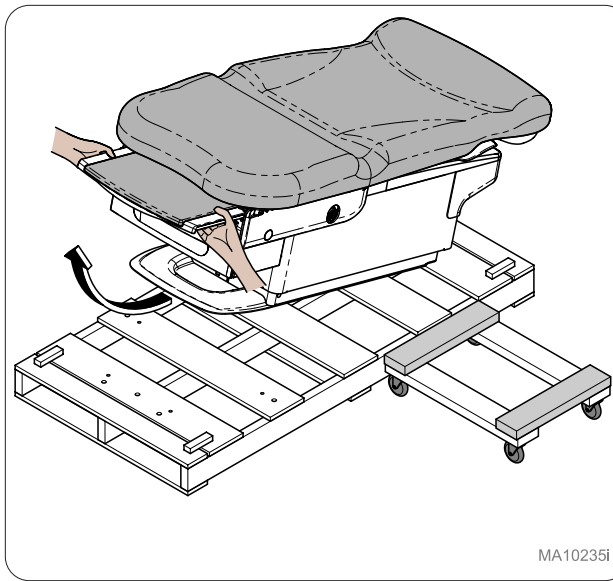
Do **NOT** lift chair by Upholstery or Stirrups.

MA10235i

Step 8:

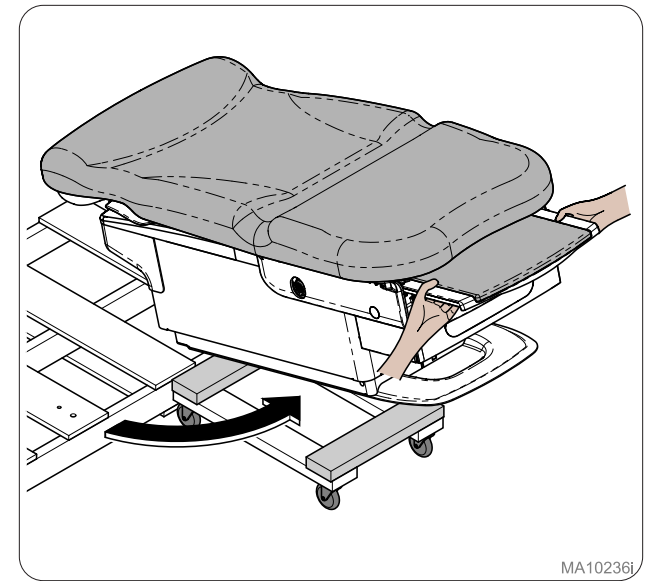
Using proper lifting technique, lift chair off front of skid.

MA10236i

Step 9:

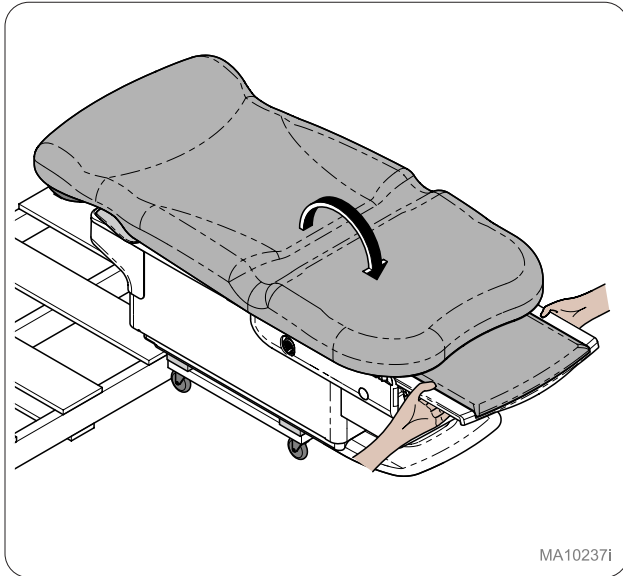
Rotate chair to dolly.

MA10237i

Step 10:

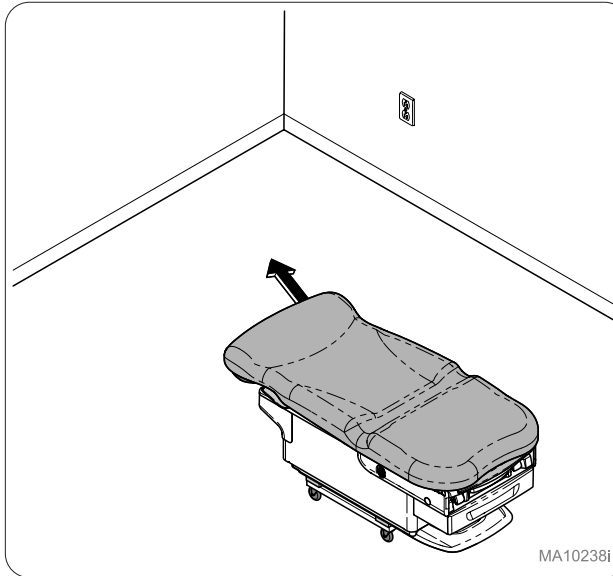
Lower chair onto dolly.

MA10238i

Step 11:

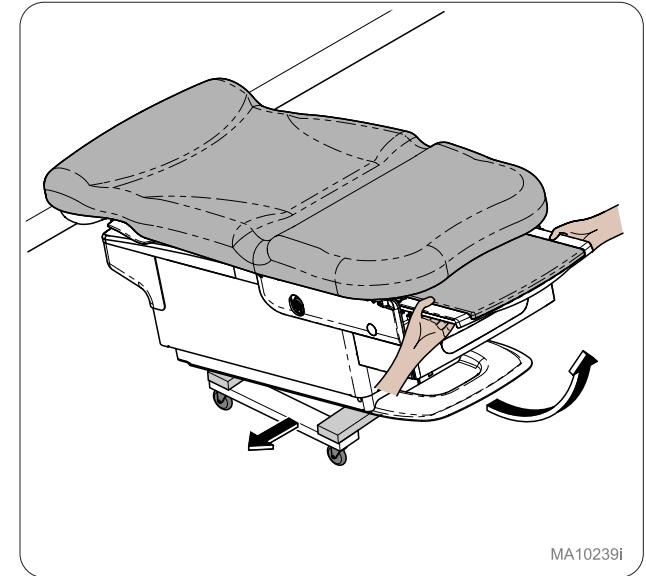Move chair to desired location.


MA10239i

Step 12:

Remove chair from dolly.

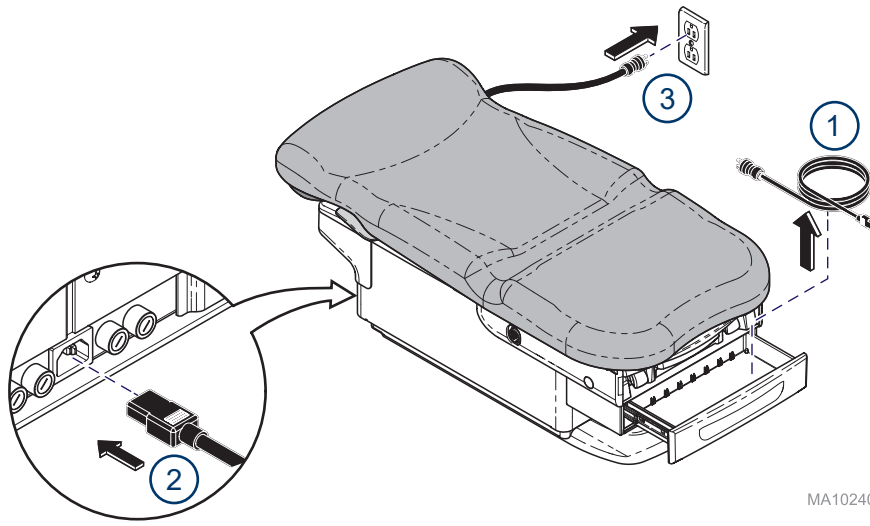
Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local, state, and federal electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.

MA10240i

Step 13:

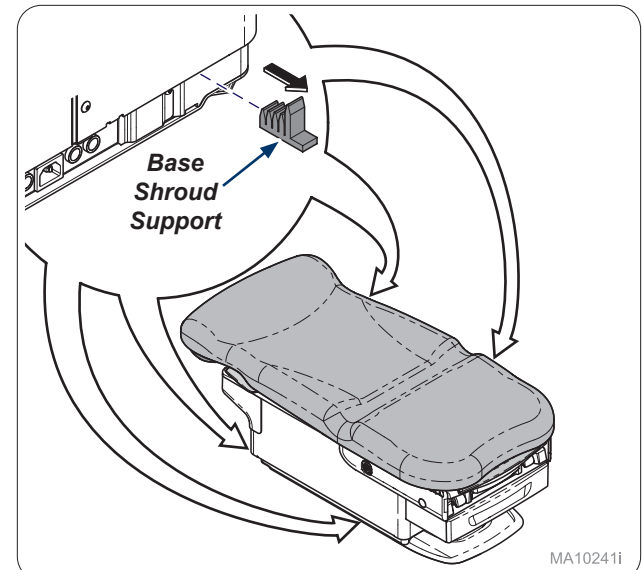
Remove power cord from drawer, connect to rear of chair, and plug into wall or floor outlet.

MA10241i


Step 14:

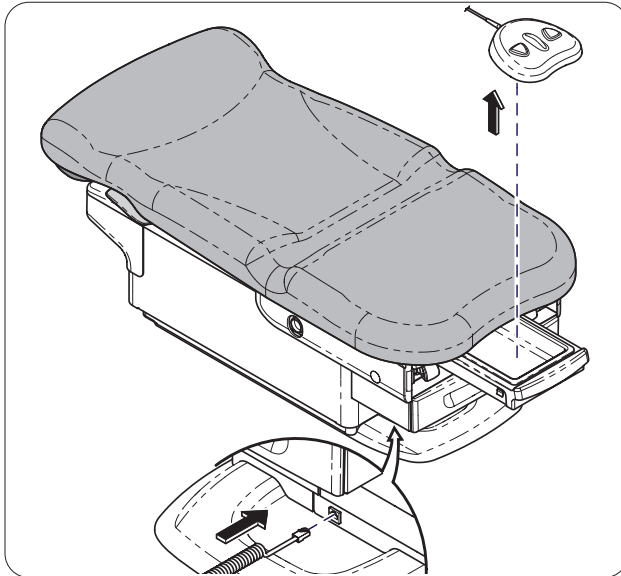Remove base shroud supports (4).

Step 15:

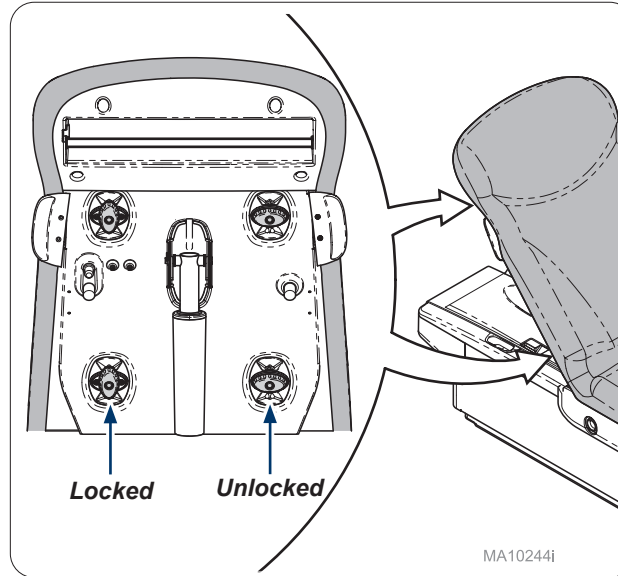
Remove foot control from treatment pan. Connect foot control to front base of chair.

MA10244i

Step 16:

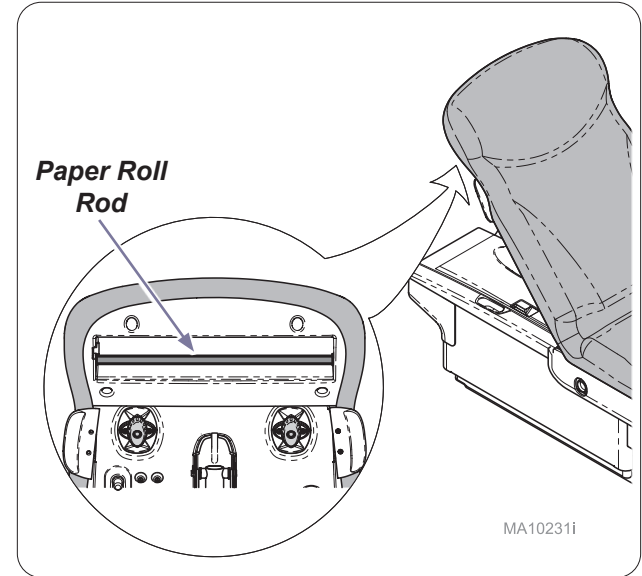Raise back and latch upholstery locking knobs (4).

MA10231i

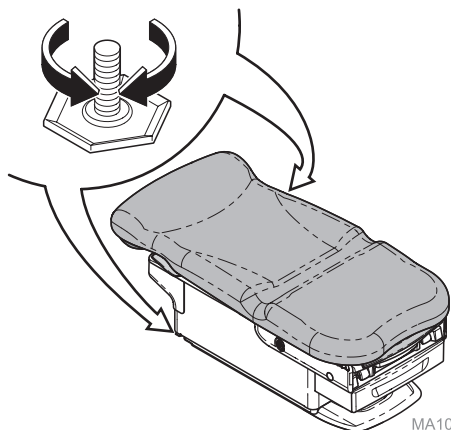
Step 17:

Ensure paper roll rod is in place.

Equipment Alert


The chair must be leveled or stable to ensure proper operation.

MA10245i

Step 18:

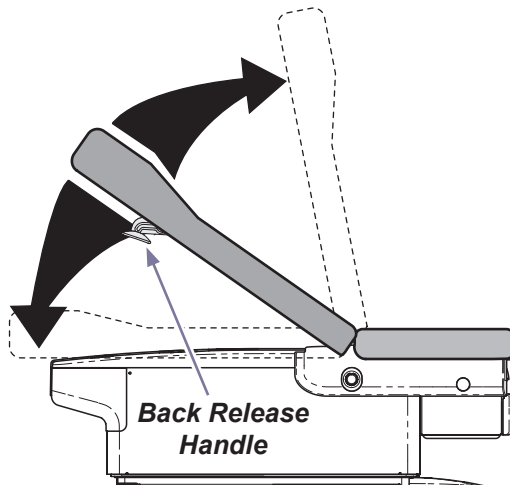Adjust leveling screws (2) at head end of chair to level or balance chair.

MA10246i

Step 19: (224 Only)

Verify proper manual Back operation.

Note

If a malfunction is detected contact Midmark Technical Service by calling: 1-800-MIDMARK.

Base Up

Base Down

Step 20: (224 Only)

Verify proper operation of foot control. Press buttons for Base Up/Down.

Note

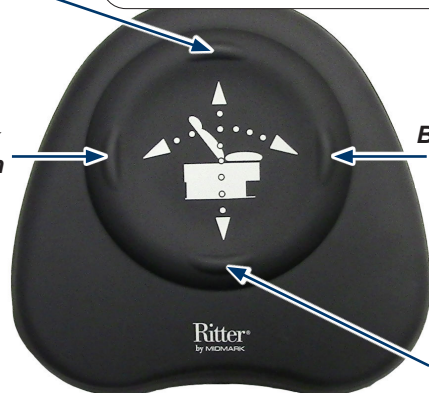
If a malfunction is detected contact Midmark Technical Service by calling: 1-800-MIDMARK.

Base Up

Back Down


Back Up

Base Down


Step 21: (225 Only)

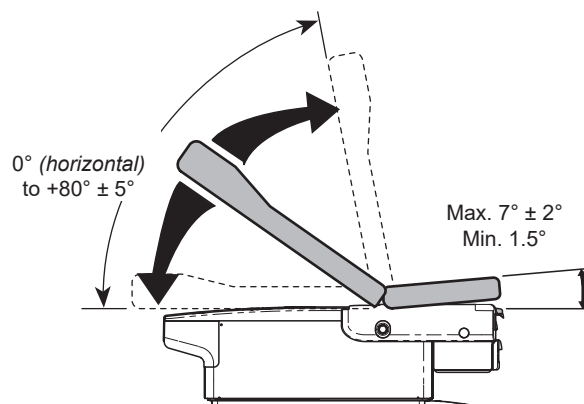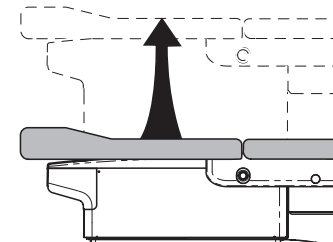Verify proper operation of foot control. Press buttons for Base Up/Down and Back Up/Down.

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