626 Barrier-Free® Power Examination Chair Setup Instructions

Tools:

Battery Powered Drill / Impact Driver
Standard Wrench set
Standard Socket Set
Box Cutter
Furniture Dolly (4 Wheel)

Caution

Heavy object. Use lifting aids and proper lifting techniques when moving chair to minimize muscle strains or back injury.

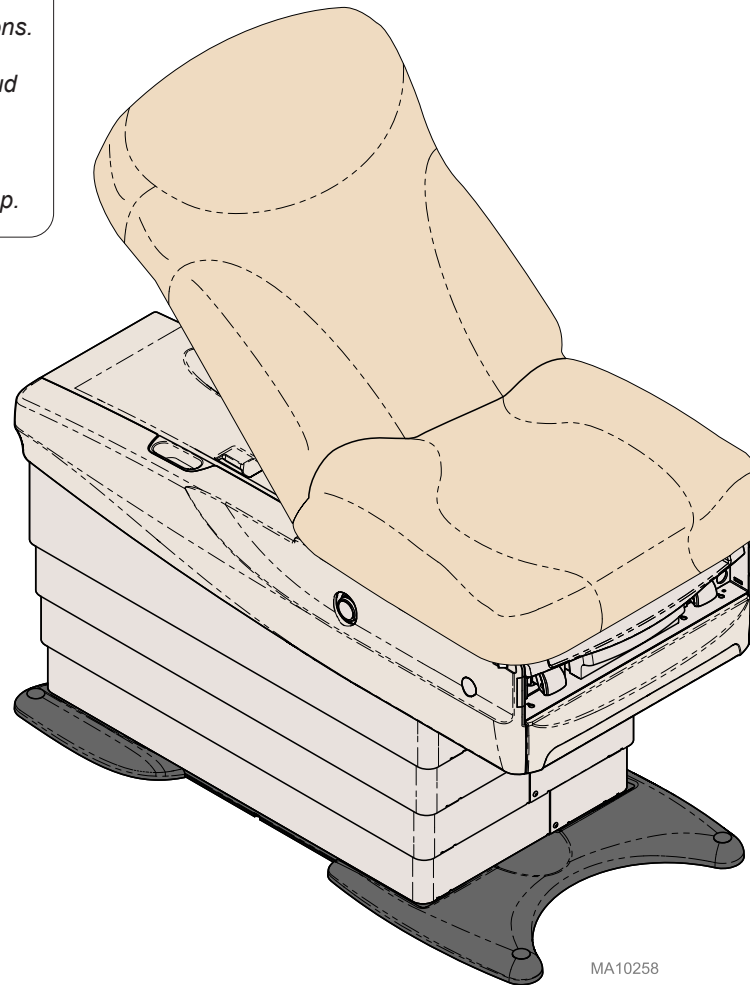
Equipment Alert

When moving the chair, you must adhere to the following instructions.

ALWAYS: a) Transport chair in full-down position with base shroud supports in place.

NEVER: a) Push on base shrouds.
b) Lift chair by stirrups, back section, or upholstered top.

MA10258

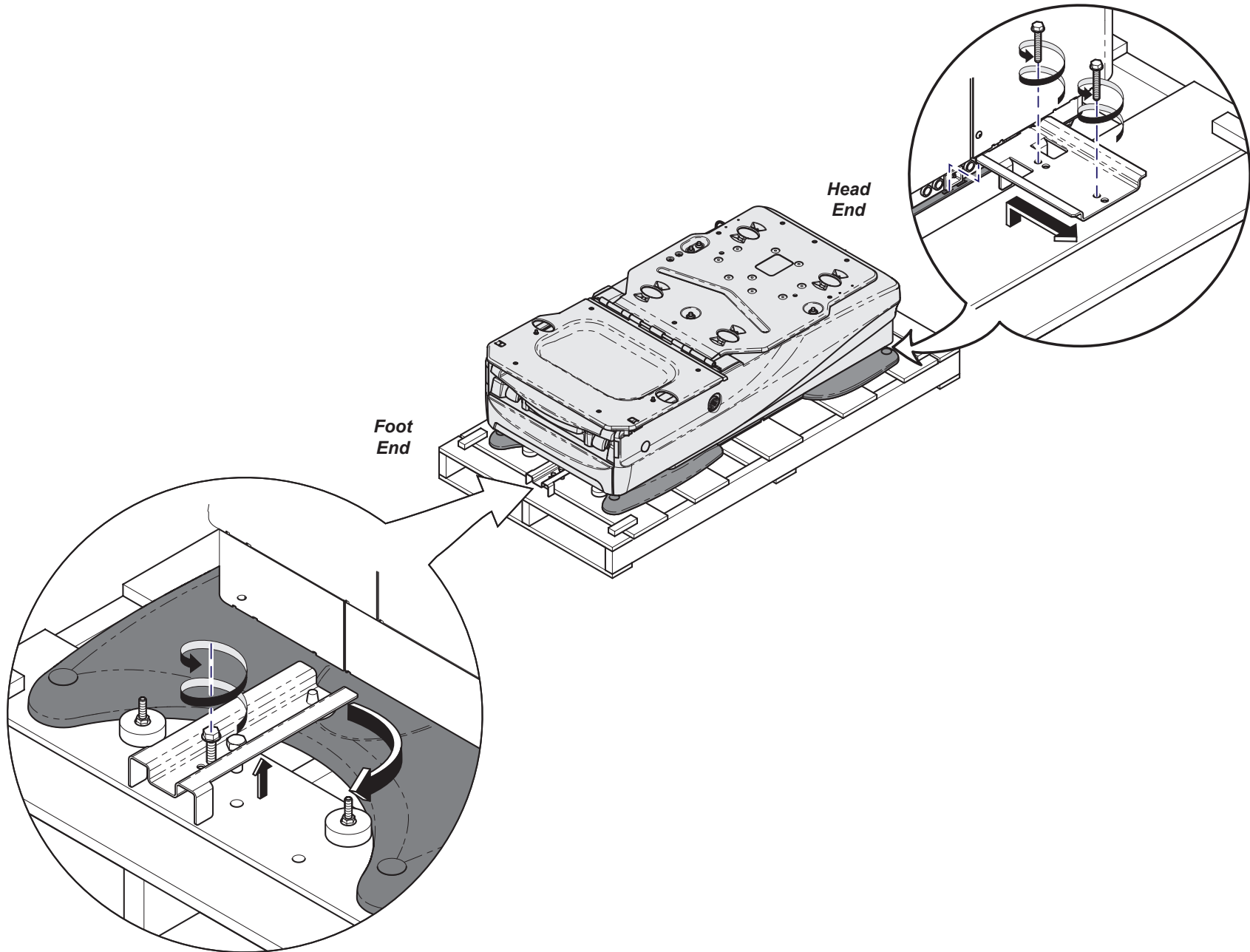
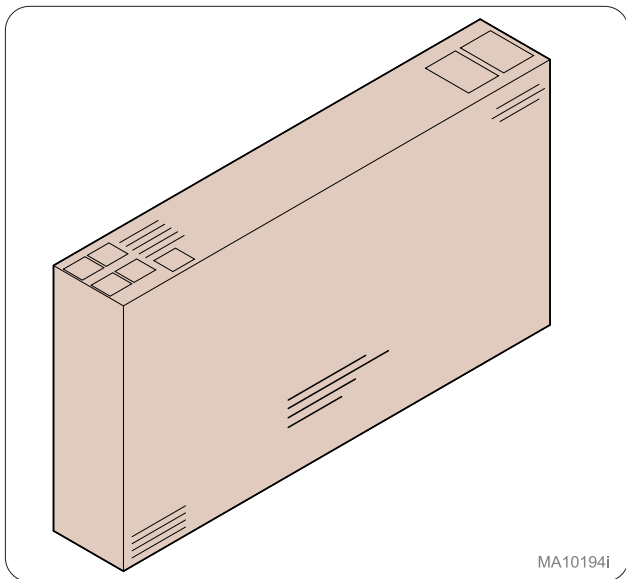
Step 1:

Remove brackets:


Head End - Remove two lag bolts, slide bracket out and up to remove.

Foot End - Loosen lag bolt enough to lift bolt/pin out of hole in skid and rotate bracket out of the way.


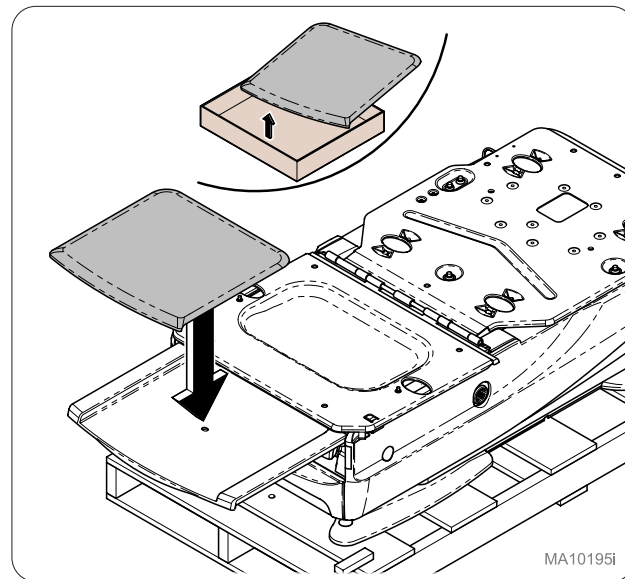
Step 2:
Locate box containing upholstery.

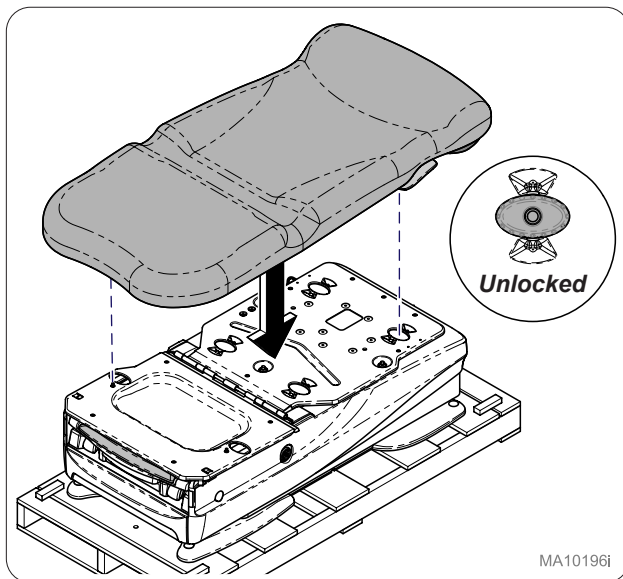
Step 3:
Install footrest upholstery, inserting pin on bottom of upholstery into footrest shelf hole.

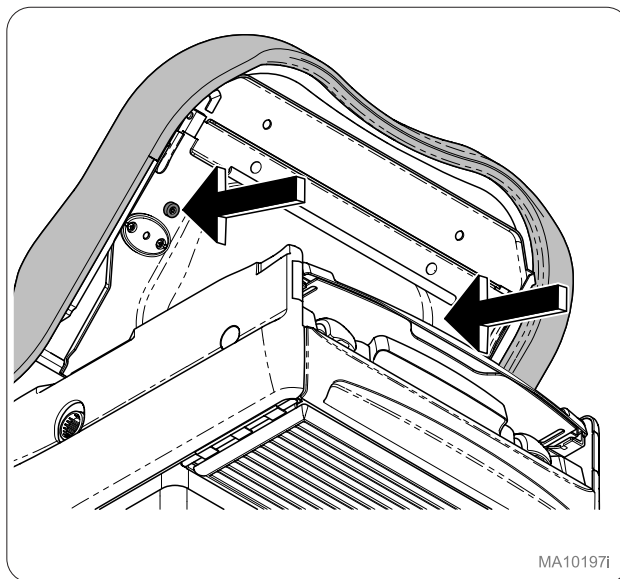
Step 4:
With locking knobs in the unlocked position and chair flat, place upholstery on chair top aligning locking knobs with holes.

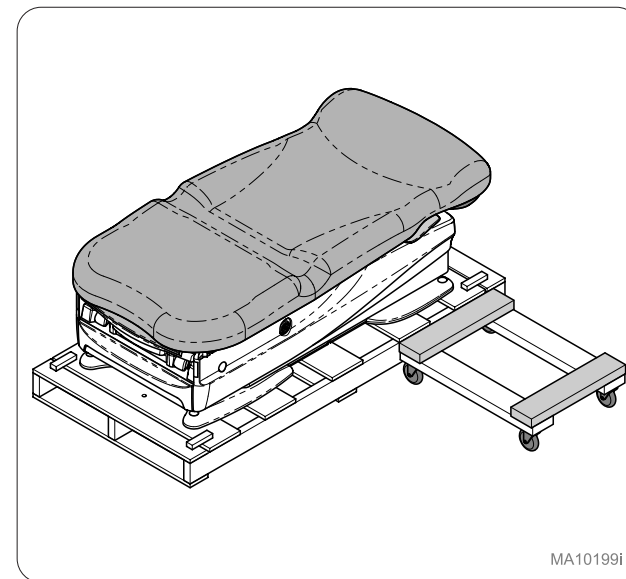
Step 5:
Raise seat pan 90 degrees and secure seat section using attached thumb screws (2). Lower seat pan.

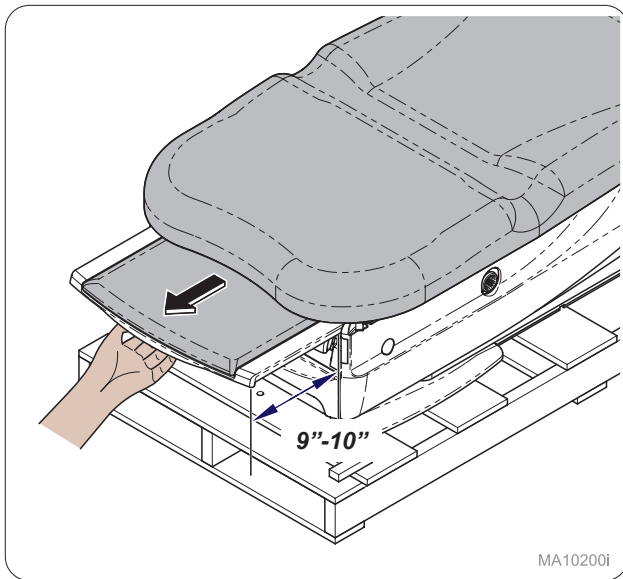
Step 6:
Place dolly next to head end of chair.

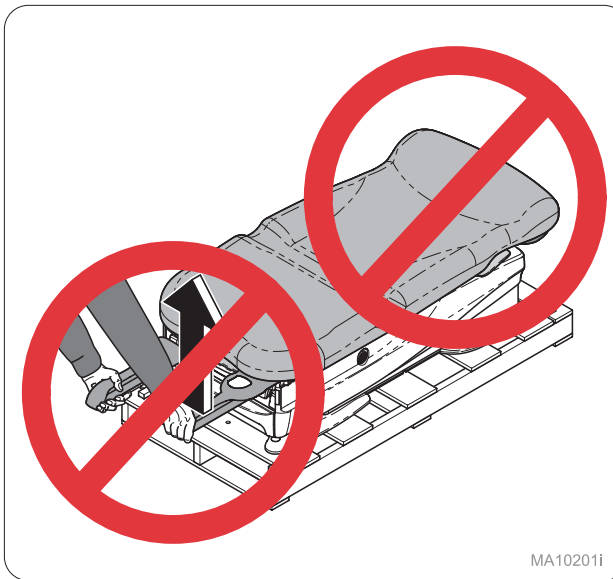
Step 7:
Slide footrest 9"-10" out of chair.

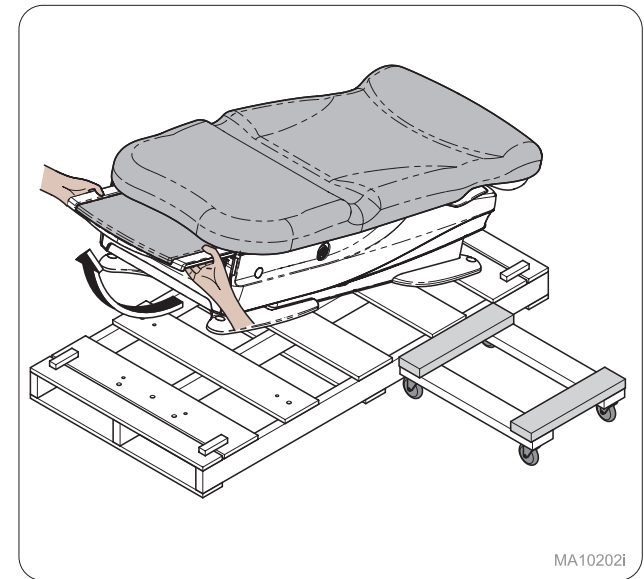
Equipment Alert
Do **NOT** lift chair by Upholstery
or Stirrups.

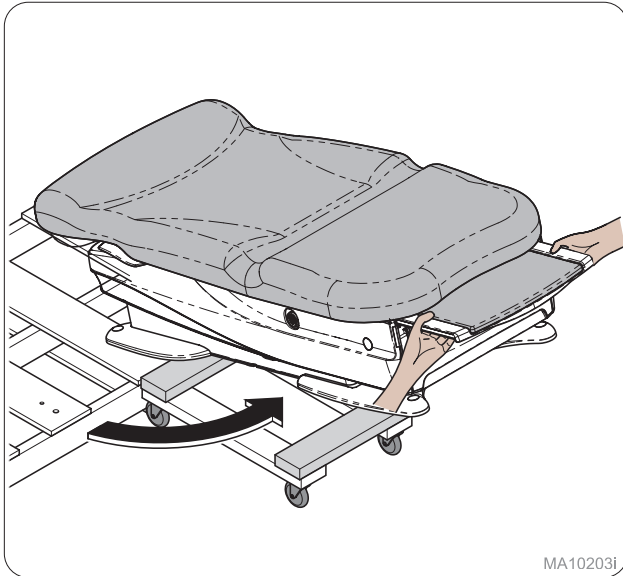
Step 8:
Using proper lifting technique lift chair off front of skid.

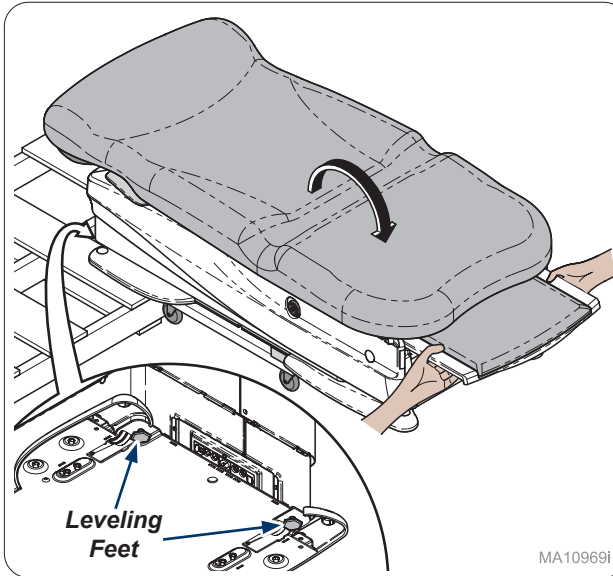
Step 9:
Rotate chair to dolly.

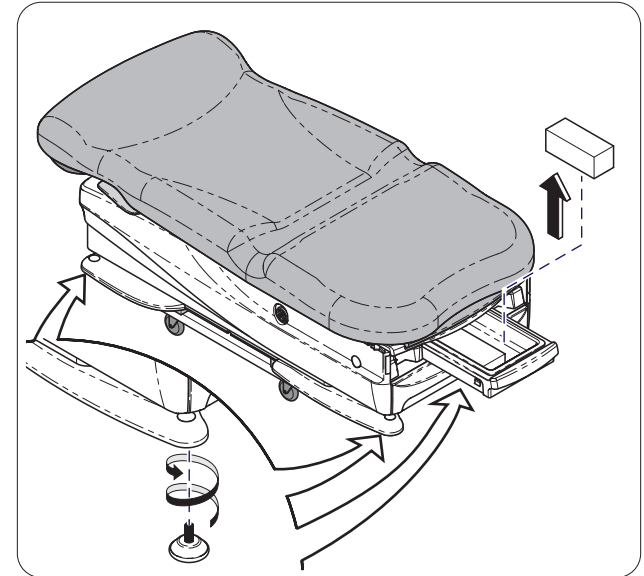Step 10: Non Scale Models (Only)
Lower chair onto dolly. Look under base for leveling feet.

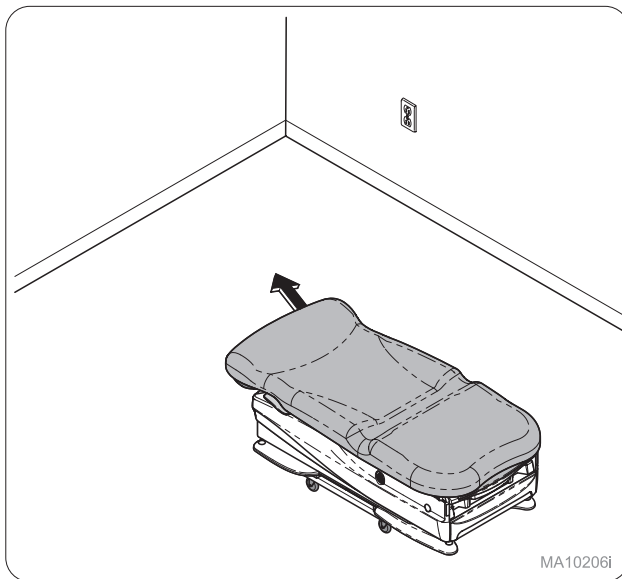
Step 11: Scale Models (ONLY)
Remove box containing feet (4) from the treatment pan
and install into base.

MA10206i

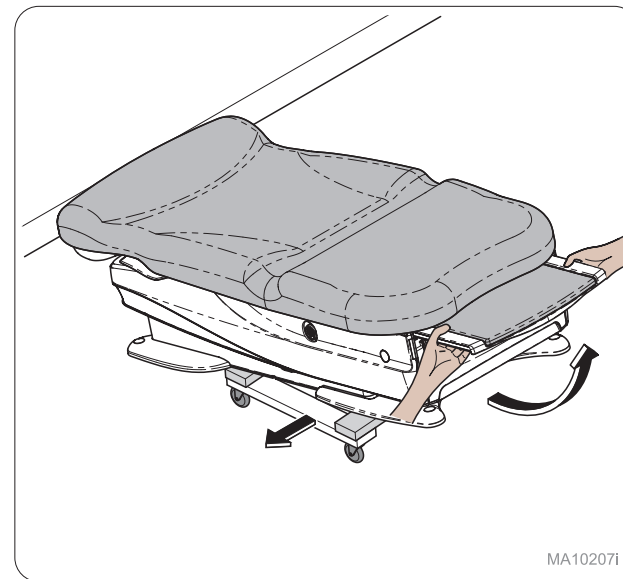
Step 12:
Move chair to desired location.


MA10207i

Step 13:
Remove chair from dolly.

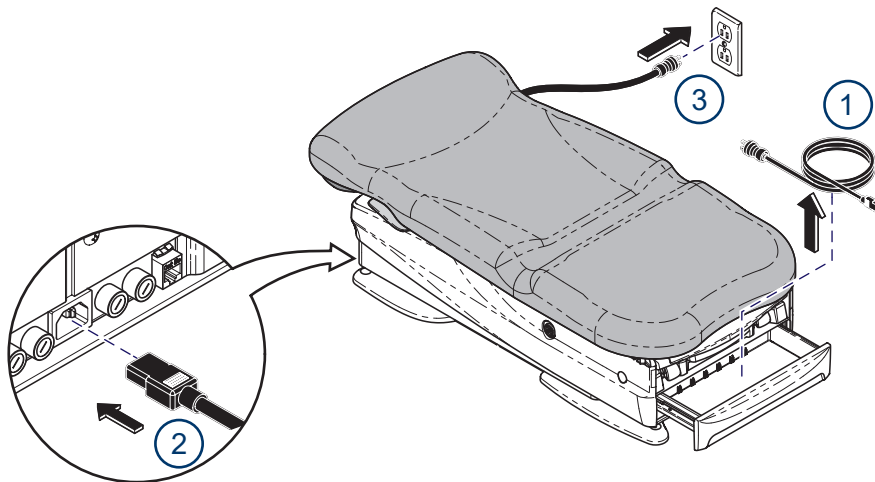Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local, state, and federal electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.

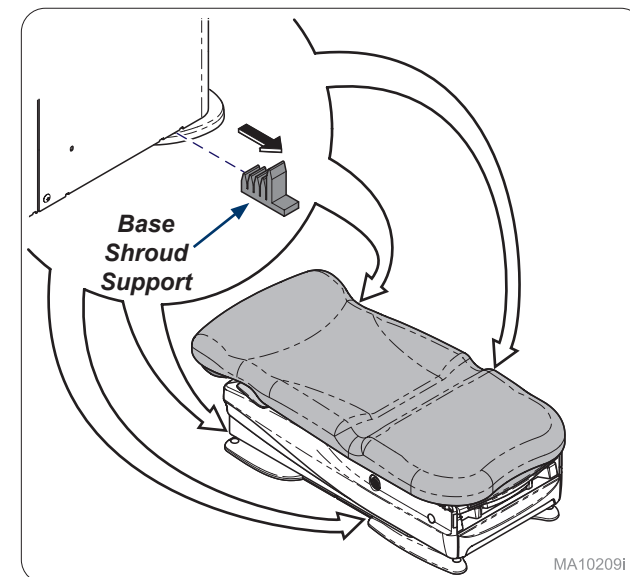
Step 14:
Remove power cord from drawer, connect to rear of chair, and plug into wall or floor outlet.


MA10209i

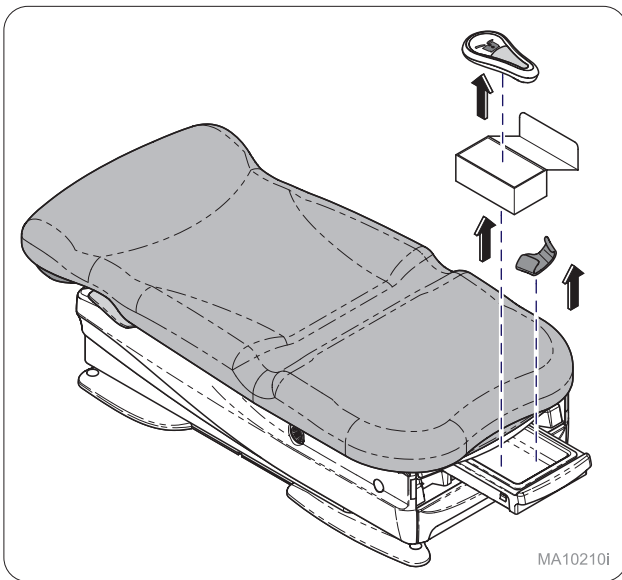
Step 15:
Remove base shroud supports (4).

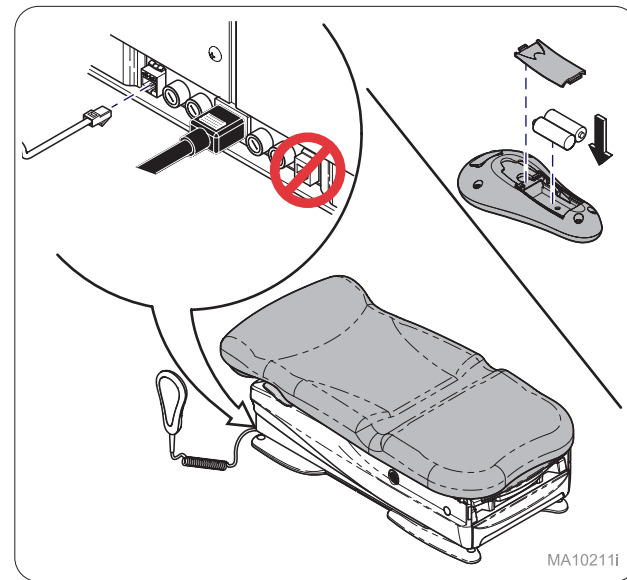
Step 16:
Remove hand control and holster from treatment pan.

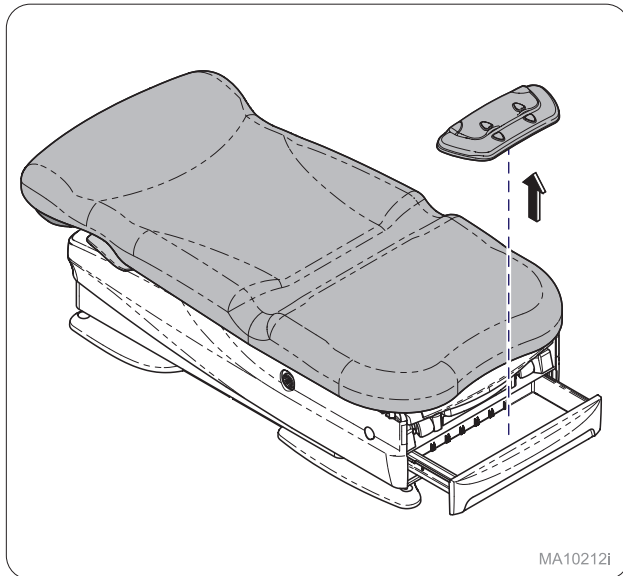
Step 17:
Connect hand control to bottom of chair. If hand control is wireless insert batteries.

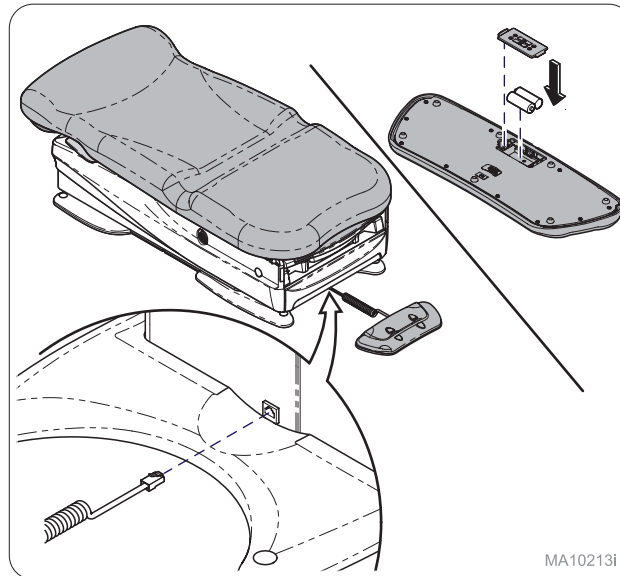Step 18:
Remove foot control from drawer.

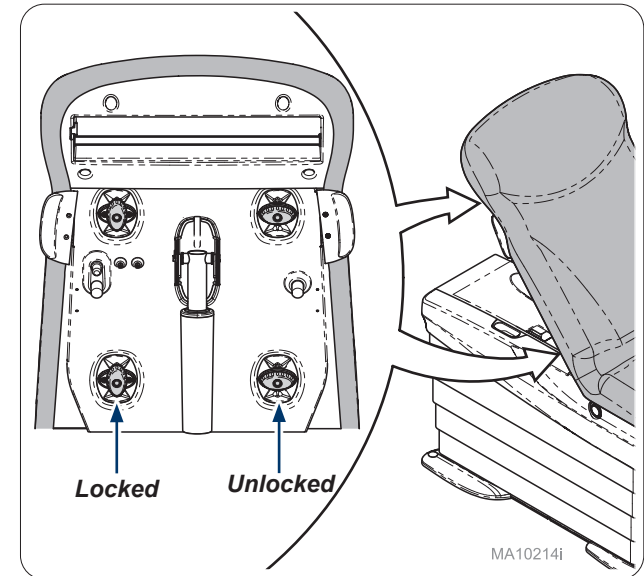Step 19:
Connect foot control to front base of chair. If foot control is wireless insert batteries.


Step 20:
Use foot or hand control to raise chair back and latch upholstery locking knobs (4).

Equipment Alert

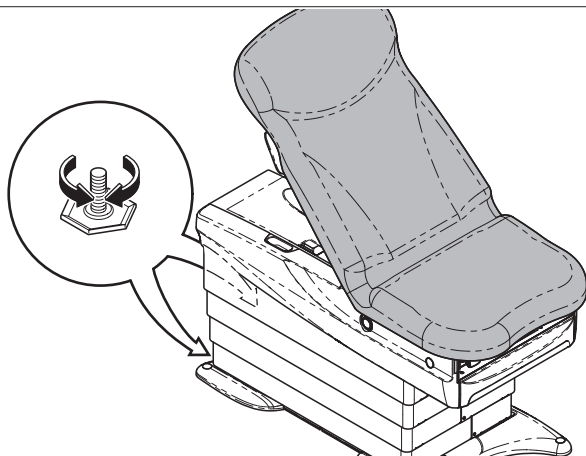The chair must be leveled or stable to ensure proper operation.

MA11482i

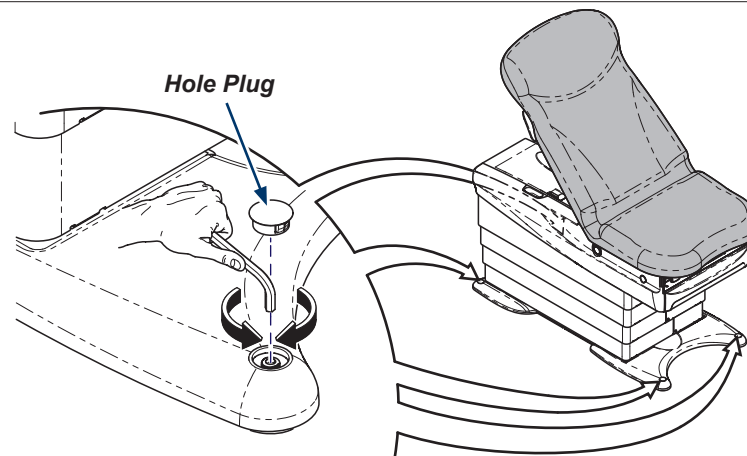
Step 21: Non Scale Models (ONLY)

Adjust leveling feet (2) at head end of chair to level or balance chair.

Equipment Alert

The chair must be leveled or stable to ensure proper operation.


MA10215-1i

Step 22: Scale Models (ONLY)

Remove hole plugs (4) and level or balance chair using Allen wrench. Leveling feet should be a minimum of 1/2 turn from fully tight (ideally 1-2 turns). Wired models only: bottom of table should be 1/2" - 11/16" from floor. Ensure foot control cord cannot fit under table. Reinstall hole plugs (4) when finished.

Note

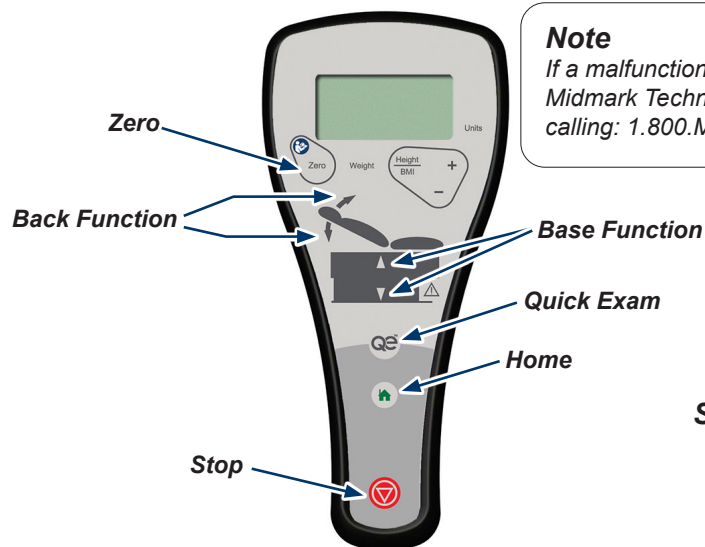If a malfunction is detected contact Midmark Technical Service by calling: 1.800.MIDMARK.


Scale Model shown

Step 23:

Verify proper operation of hand control. Press the buttons for Base Up/Down, Back Up/Down, Quick Exam® and Home. Press Home button, while chair is moving press Stop. (All movement should stop)


Note

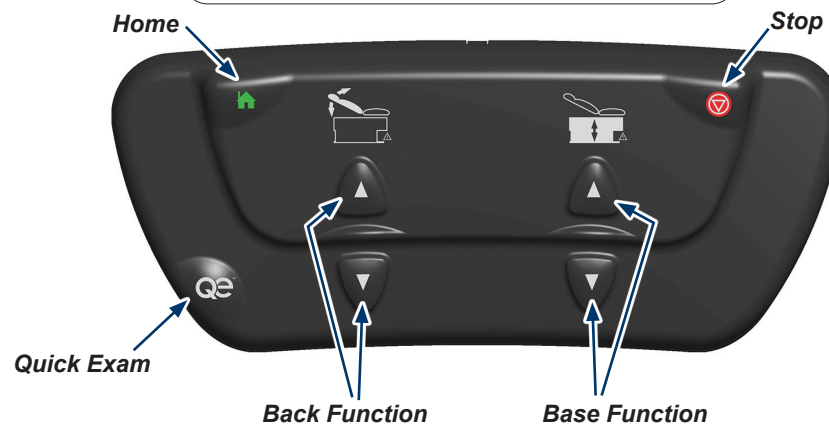If a malfunction is detected contact Midmark Technical Service by calling: 1.800.MIDMARK.


Step 24:

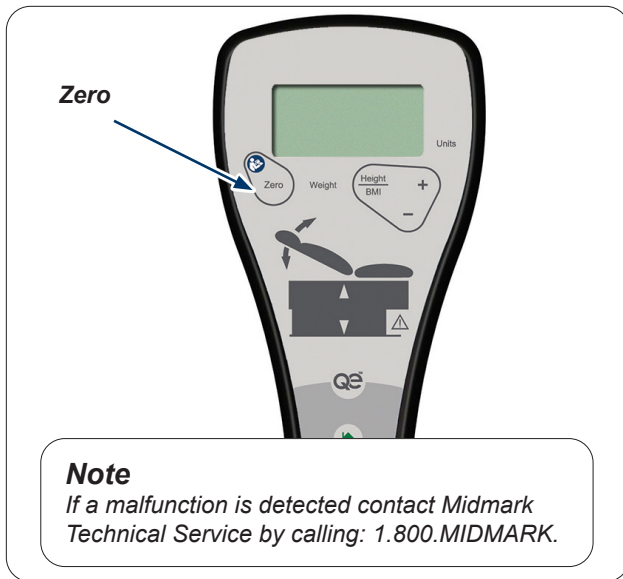
Verify proper operation of foot control. Press the buttons for Base Up/Down, Back Up/Down, Quick Exam and Home. Press Home button, while chair is moving press Stop. (All movement should stop)

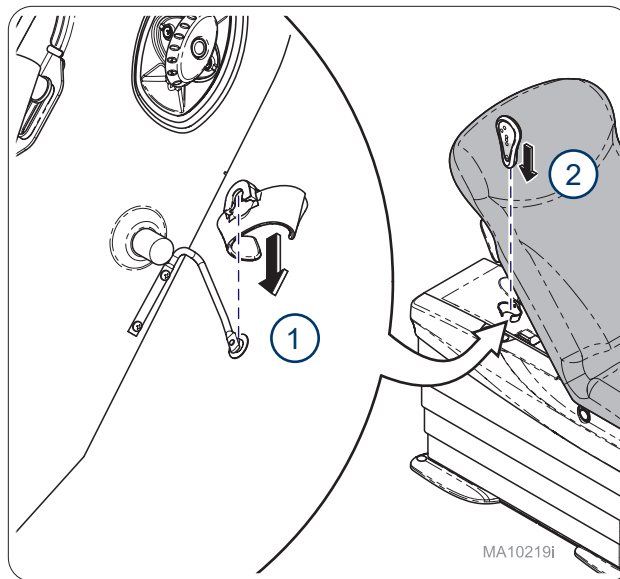
Step 25: Scale Models (ONLY)
Use hand control and press Zero button to calibrate chair. Display should cycle and read "Scale Ready".

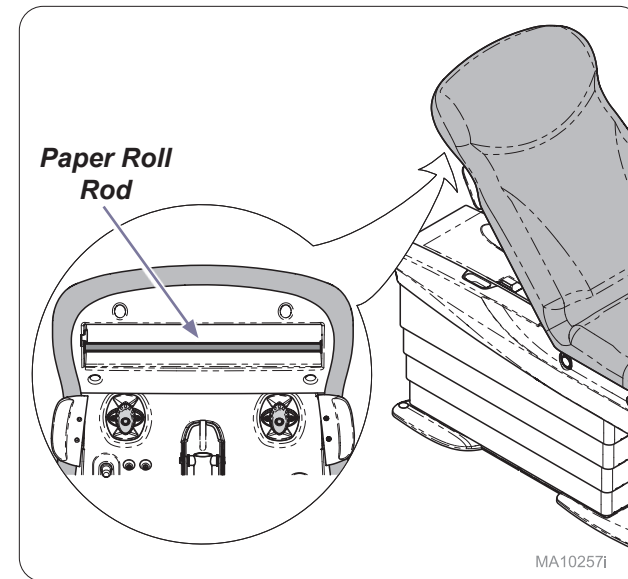
Step 26:
Snap on holster to hanging bar and place hand control into holster.

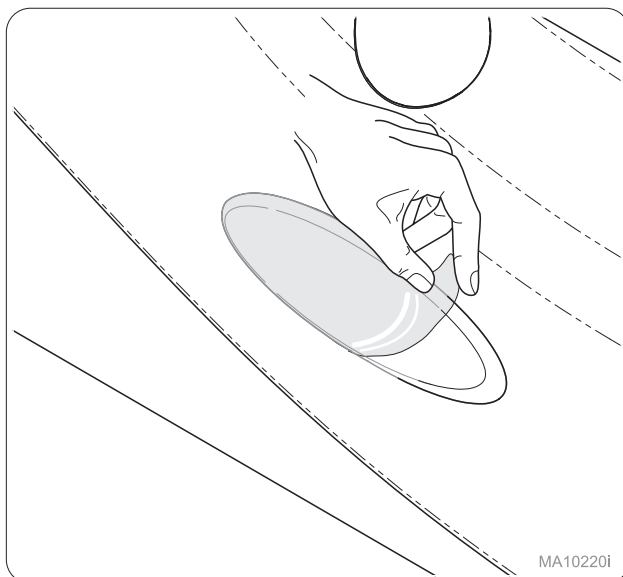Step 27:
Ensure paper roll rod is in place.

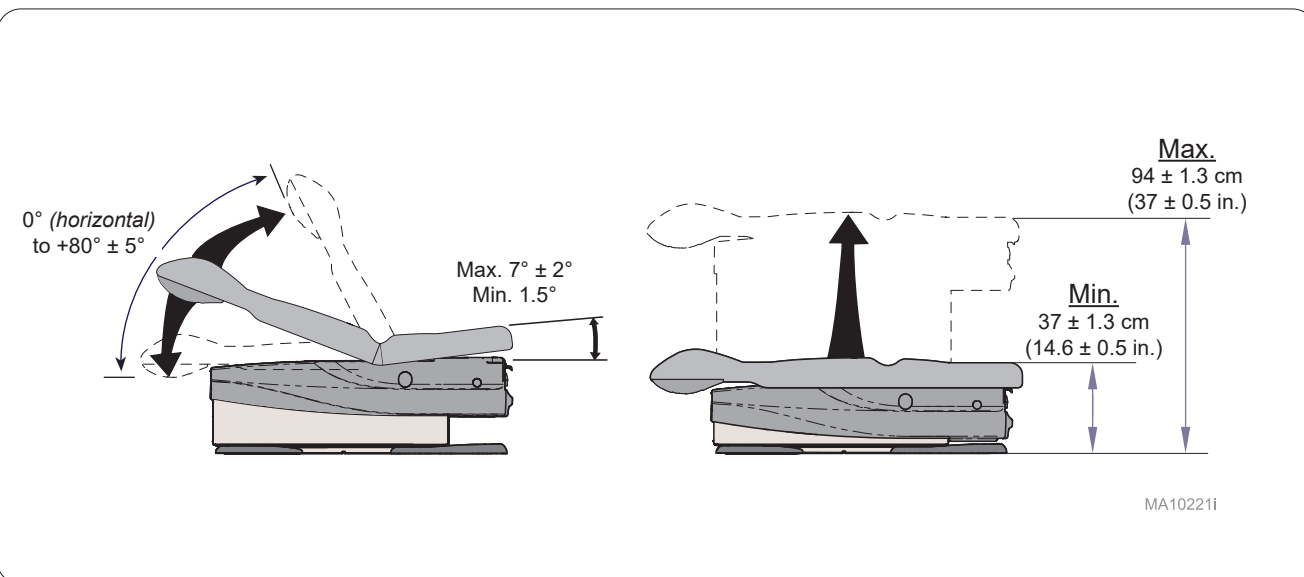Step 28:
Remove clear plastic cover (2) from Midmark name plates.

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