627 Barrier-Free® Power Examination Chair Setup Instructions

Tools:

Battery Powered Drill / Impact Driver
Standard Wrench Set
Standard Deep Well Socket Set
Box Cutter
Furniture Dolly (4 Wheel)

Caution

Heavy object. Use lifting aids and proper lifting techniques when moving chair to minimize muscle strains or back injury.

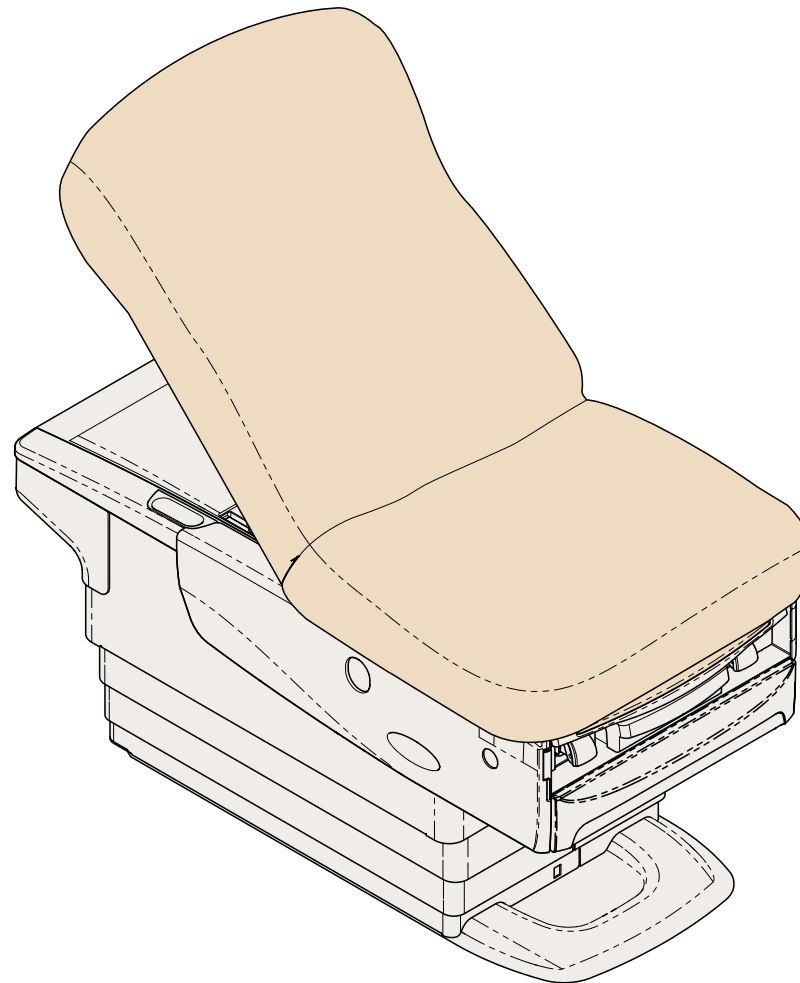
Equipment Alert

When moving the chair, you must adhere to the following instructions.

ALWAYS: a) Transport chair in full-down position with base shroud supports in place.

NEVER: a) Push on base shrouds.
b) Lift chair by stirrups, back section or upholstered top.



MA10259i


Step 1:

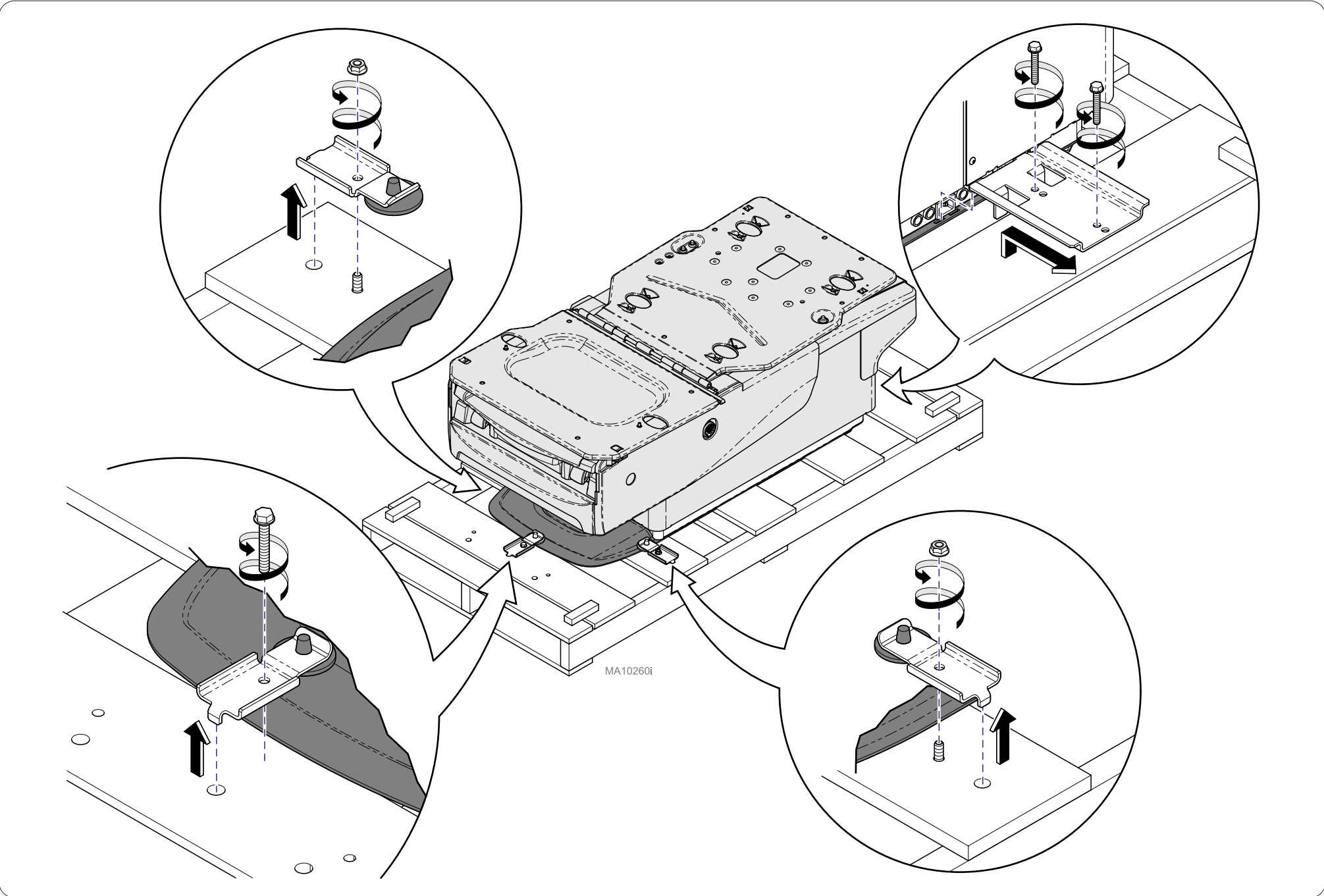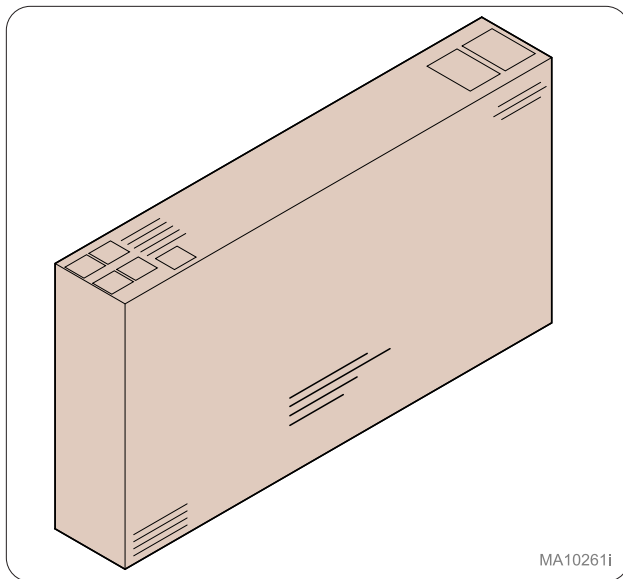Remove brackets (4) from skid to release chair.


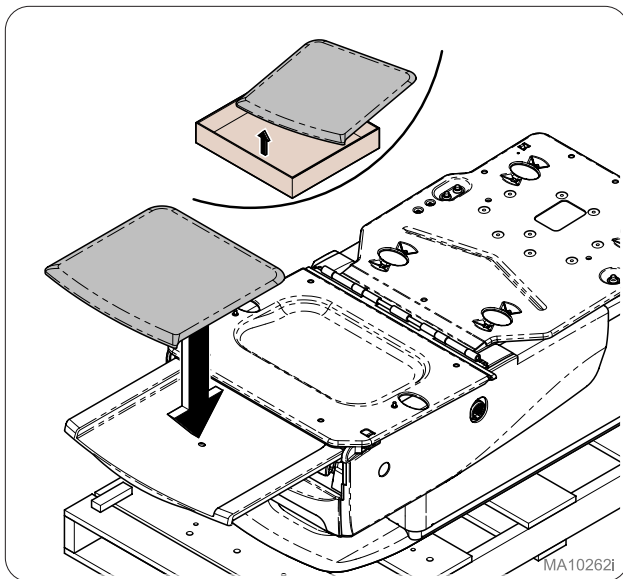
Step 2:
Locate box containing upholstery.

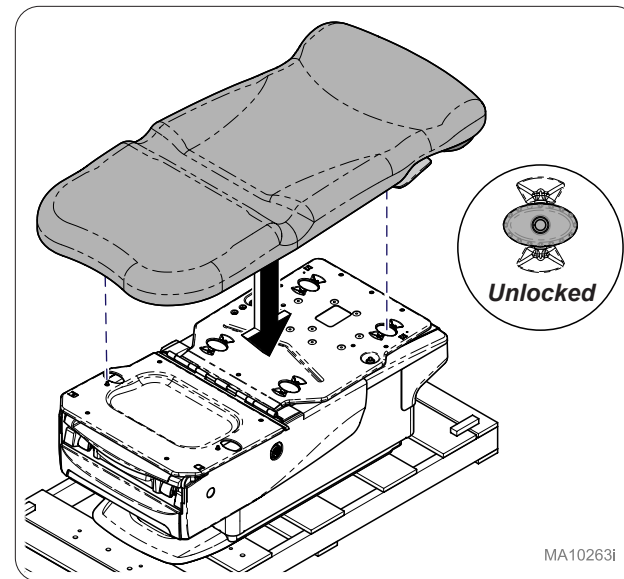
Step 3:
Place footrest upholstery, inserting pin on bottom of upholstery into footrest shelf hole.

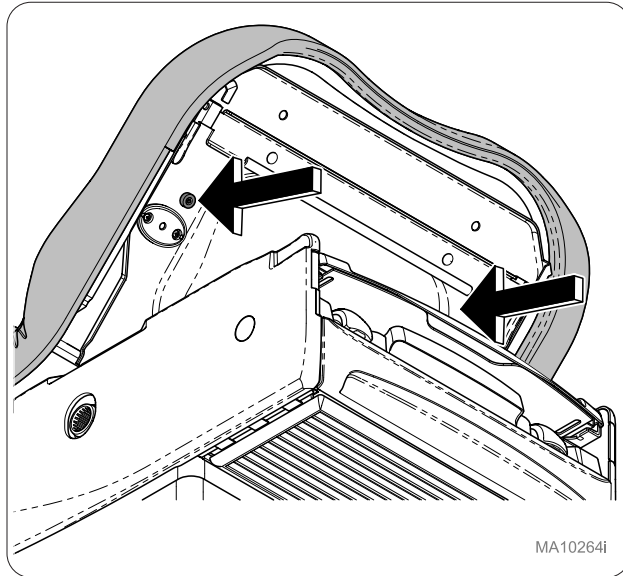
Step 4:
With locking knobs (4) in the unlocked position, place upholstery on chair top aligning locking knobs with holes.

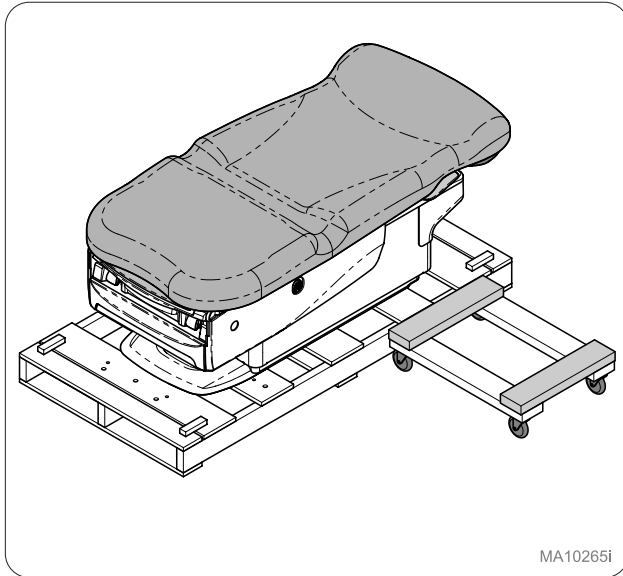
Step 5:
Raise seat pan 90 degrees and secure seat section using attached thumb screws (2). Lower seat pan.

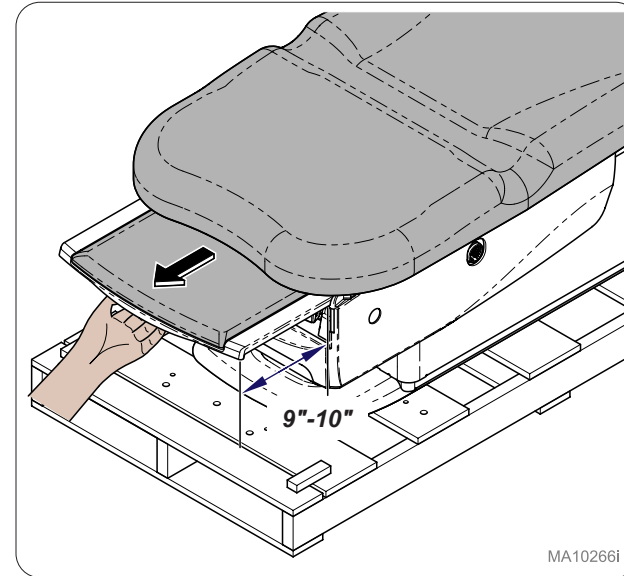Step 6:
Place dolly next to head end of chair.

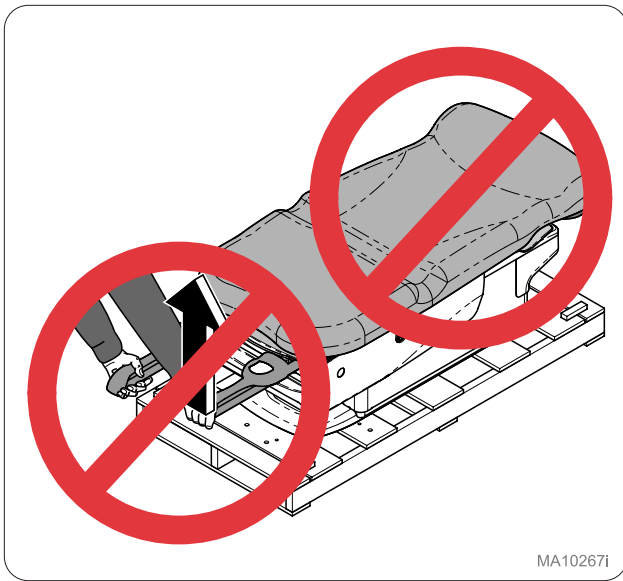Step 7:
Slide footrest 9"-10" out of chair.


MA10267i

Equipment Alert

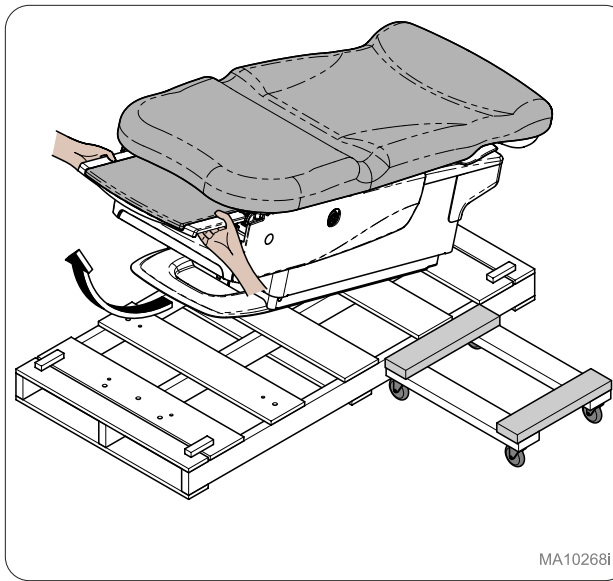
Do **NOT** lift chair by Upholstery or Stirrups.

MA10268i

Step 8:

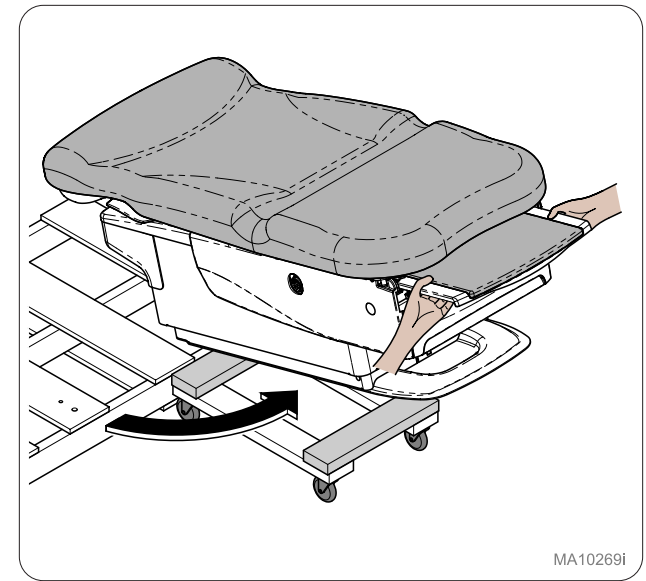
Using proper lifting technique, lift chair off front of skid.

MA10269i

Step 9:

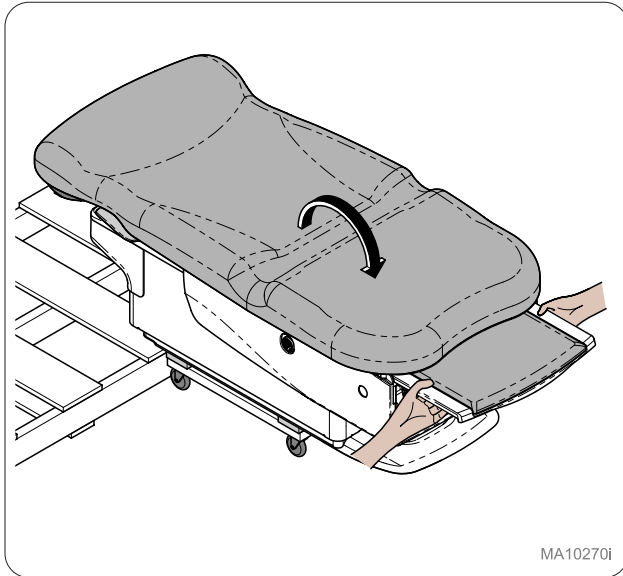
Rotate chair to dolly.

MA10270i

Step 10:

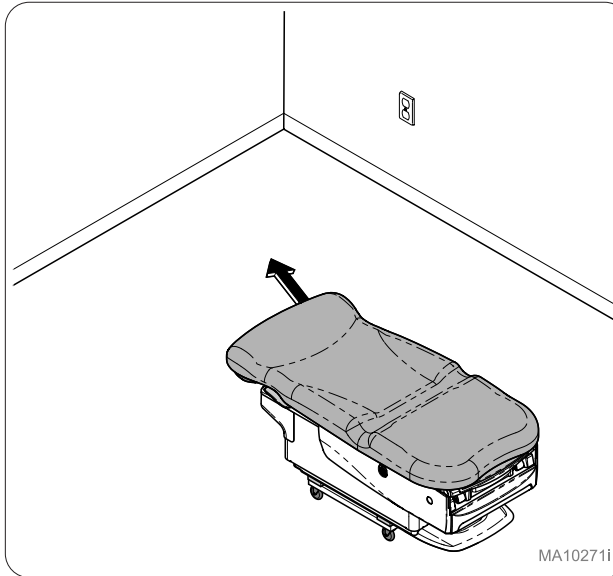
Lower chair onto dolly.

MA10271i

Step 11:

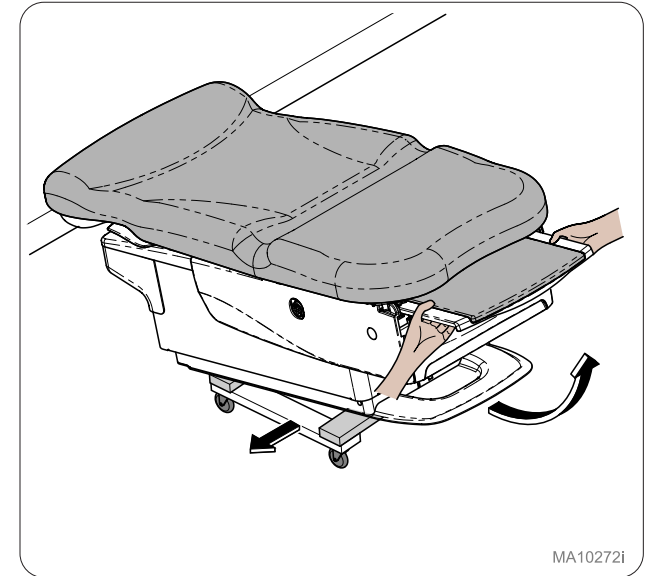Move chair to desired location.


MA10272i

Step 12:

Remove chair from dolly.

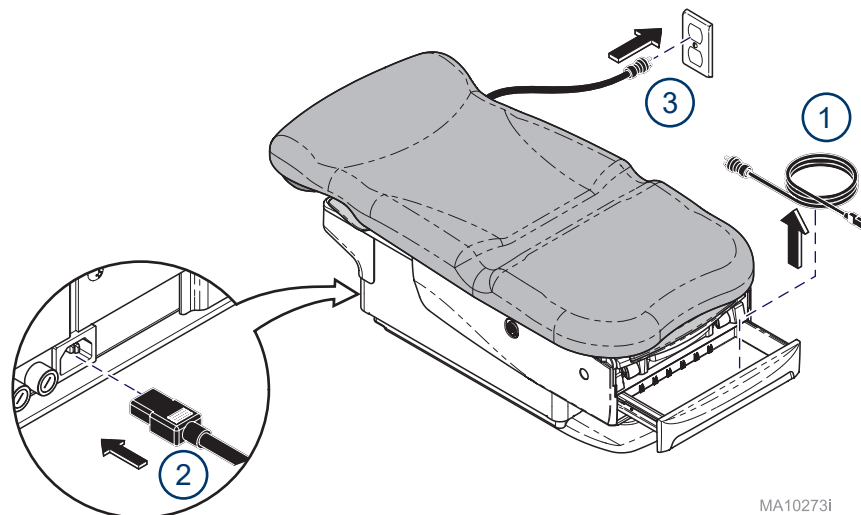
Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all applicable electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements.

MA10273i


Step 13:

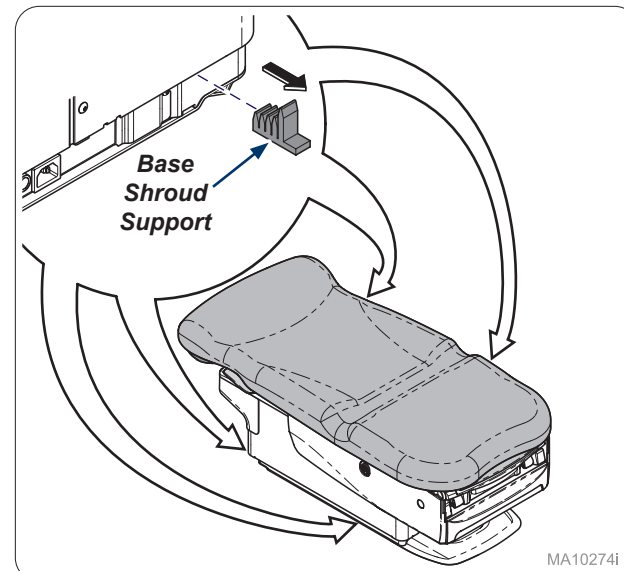Remove power cord from drawer, connect to rear of chair, and plug into wall or floor outlet.


Step 14:

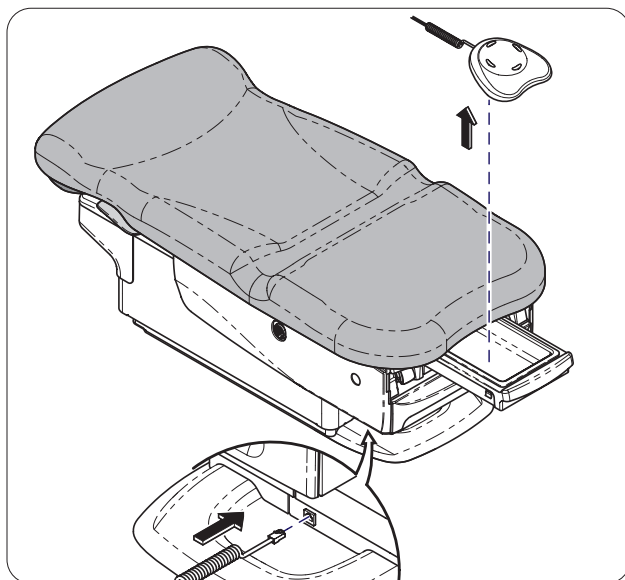Remove base shroud supports (4).


Step 15:

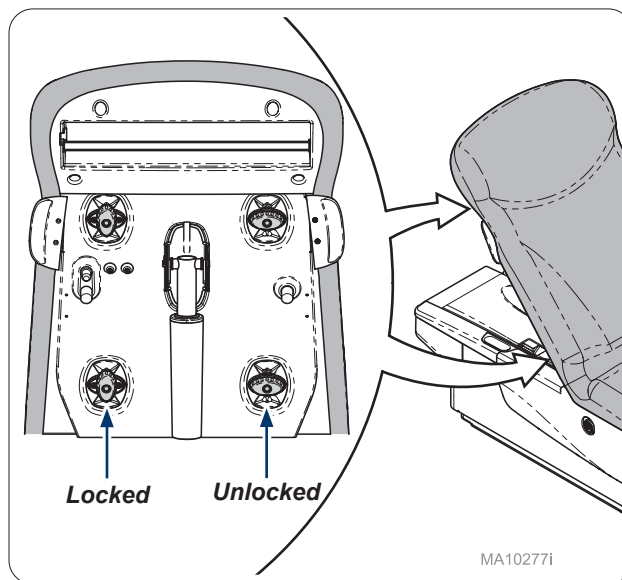Remove foot control from treatment pan. Connect foot control to front base of chair.

Step 16:

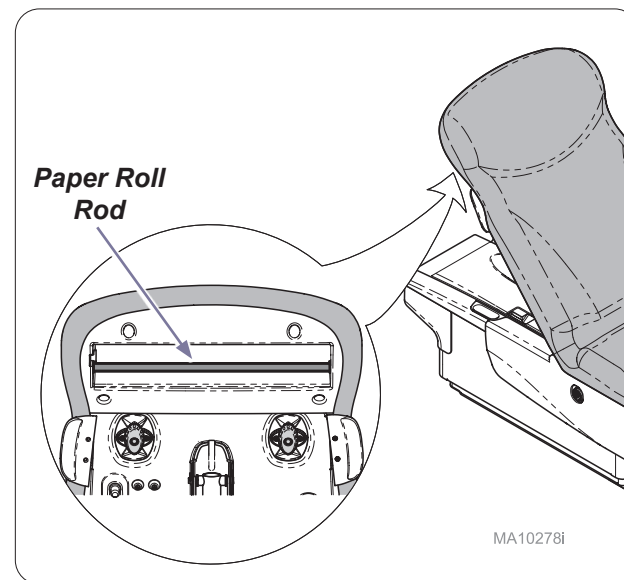Raise back and latch upholstery locking knobs (4).


Step 17:

Ensure paper roll rod is in place.

Equipment Alert

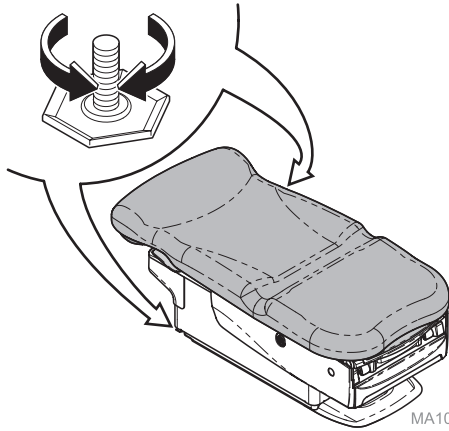The chair must be leveled or stable to ensure proper operation.


Step 18:

Adjust leveling screws (2) at head end of chair to level or balance chair.

Note

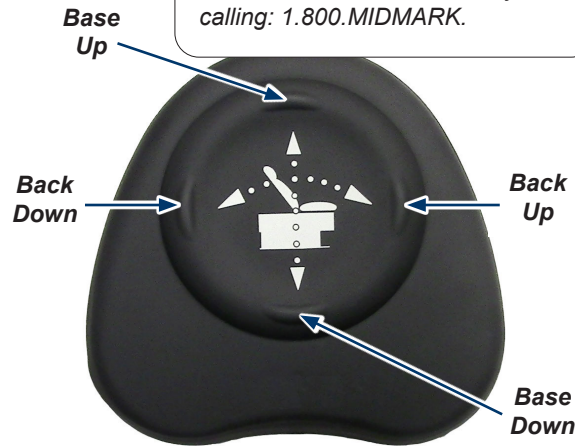If a malfunction is detected contact Midmark Technical Service by calling: 1.800.MIDMARK.




Step 19:

Verify proper operation of foot control. Press buttons for Base Up/Down and Back Up/Down.

Step 20:

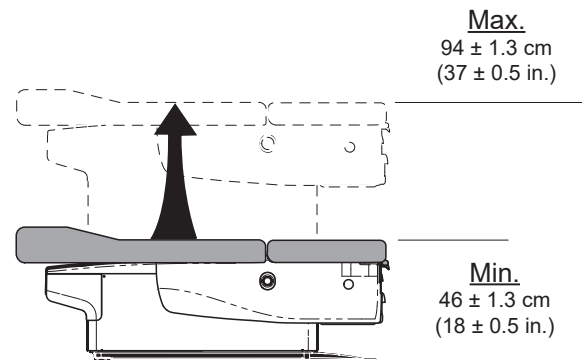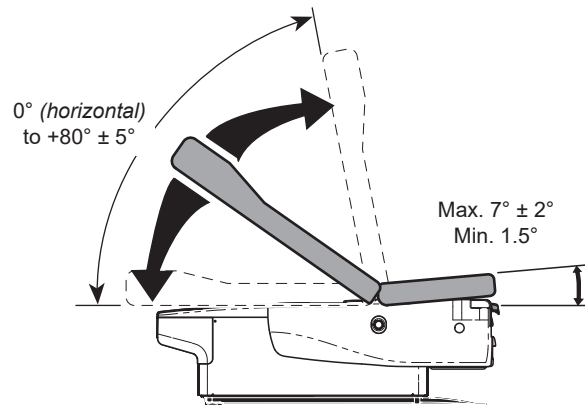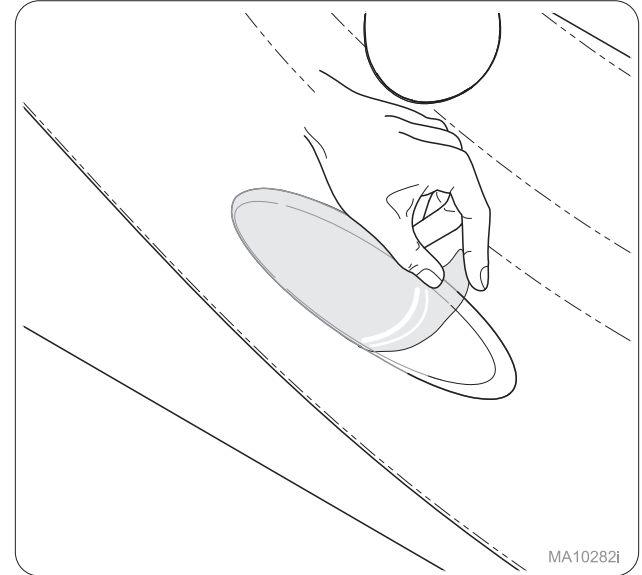Remove clear plastic cover (2) from Midmark name plates.

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