

GENERAL NOTES

- OUTSIDE EDGE OF TEMPLATE INDICATES OVERALL CABINET SILHOUETTE. POSITION TEMPLATE ON WALL IN DESIRED LOCATION PER ROOM LAYOUT DRAWING.

PLUMBING SPECIFICATIONS

NOTE: HAVE A LICENSED PLUMBER AND ELECTRICIAN INSTALL ALL UTILITIES. LOCAL PLUMBING AND ELECTRICAL CODES SHOULD BE FOLLOWED FOR ALL CONNECTIONS. MIDMARK CORP. CANNOT BE HELD RESPONSIBLE FOR INSTALLATIONS WHICH DO NOT CONFORM TO LOCAL CODES.

- STUBS FOR WATER, AIR AND VACUUM CAN BE PLACED ANYWHERE WITHIN THE DOUBLE-LINE PERIMETERS.
- WATER: 3/4" DIAMETER HOLE REQUIRED FOR 1/2" NPT PIPE STUB. STUB TO EXTEND 1" MIN / 2-1/2" MAX FROM FINISHED WALL SURFACE. INSTALL MANUAL SHUT-OFF VALVE (SUPPLIED). FLUSH ALL DEBRIS FROM LINE BEFORE INSTALLING VALVE. WATER PRESSURE NOT TO EXCEED 50 PSI.
- AIR: 3/4" DIAMETER HOLE REQUIRED FOR 1/2" NPT PIPE STUB. STUB TO EXTEND 1" MIN / 2-1/2" MAX FROM FINISHED WALL SURFACE. INSTALL MANUAL SHUT-OFF VALVE (SUPPLIED). FLUSH ALL DEBRIS FROM LINE BEFORE INSTALLING VALVE. AIR PRESSURE TO BE 80 PSI (MIN) TO 100 PSI (MAX).
- VACUUM: PLUMBING TO THE CASEWORK SHOULD BE SPECIFIED BY CENTRAL VACUUM SUPPLIER. DRILL HOLE IN WALL AS REQUIRED. EXTEND PIPE STUB 1" MIN / 2-1/2" MAX FROM FINISHED FLOOR. TERMINATE PLUMBING WITH A 5/8" OD TUBING CONNECTOR.

LEFT SIDE PLUMBING
ROUGH-IN PERIMETER

42.00"

CENTER PLUMBING
ROUGH-IN PERIMETER

RIGHT SIDE PLUMBING
ROUGH-IN PERIMETER

LEFT SIDE PLUMBING
ROUGH-IN PERIMETER

CENTER PLUMBING
ROUGH-IN PERIMETER

RIGHT SIDE PLUMBING
ROUGH-IN PERIMETER

PLACE THIS EDGE ON FLOOR
(CONSIDERING FINISHED FLOOR HEIGHT)



NOTES:
1. MATERIAL: 20# WHITE OFFSET PAPER.
2. INK COLOR: BLACK

ECO NO	DATE	CH BY	CHANGE	REV
EC0063469	9/21/18	PTR	ADDED DIMENSIONS AND UPDATED LOGO.	AA2
EC0053637	5/25/17	BLS	MODIFIED FRACTIONAL SHEET TOLERANCE TO 1/4 FROM 0.1	AA1
EC0052253	2/17/17	BLS	RELEASED	A1

INCH (MILLIMETER)	INCH (MILLIMETER)
UNSPECIFIED GENERAL TOLERANCES	
FRACTIONAL +/- 1/4	
DECIMAL XX +/- 0.03	
DECIMAL XXX +/- 0.015	

ITEM	QTY	PART NUMBER	DESCRIPTION
MATERIAL SPEC			
SCALE 0.500			CK BY DATE
PREPARED BY BSTACHLER			DATE 02/16/17
APPROVED BY			DATE
PART NAME			TEMPLATE, WALL - TS
PART NUMBER			003-10042-00
REV			AA2




SHEET 1 OF 2

003-10042-00

REV AA2

PRATICE 9/28/2018 12:44:59 PM

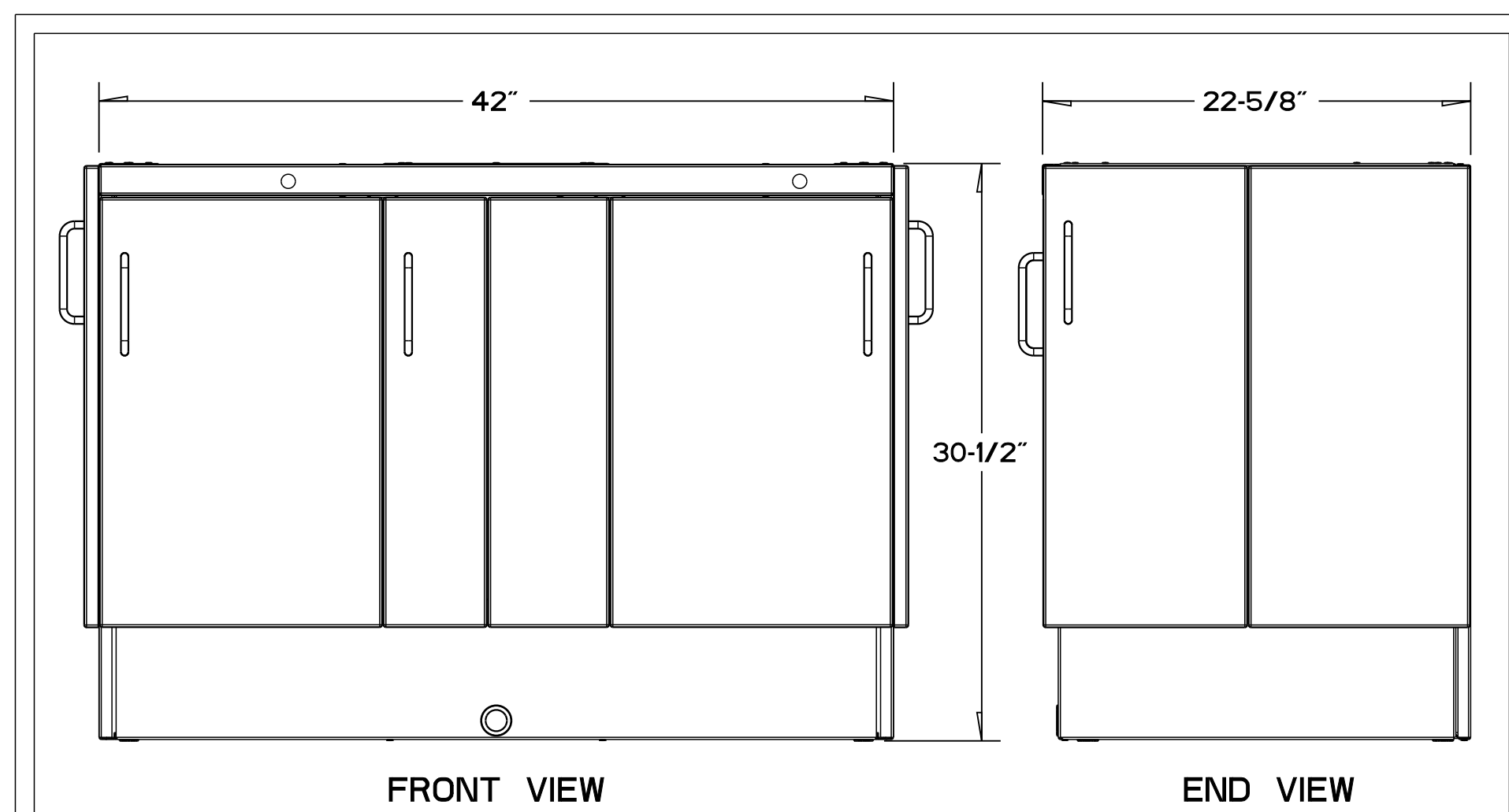

GENERAL NOTES

1. OUTSIDE EDGE OF TEMPLATE INDICATES OVERALL CABINET SILHOUETTE. POSITION TEMPLATE ON WALL IN DESIRED LOCATION PER ROOM LAYOUT DRAWING.

PLUMBING SPECIFICATIONS

- NOTE: HAVE A LICENSED PLUMBER AND ELECTRICIAN INSTALL ALL UTILITIES. LOCAL PLUMBING AND ELECTRICAL CODES SHOULD BE FOLLOWED FOR ALL CONNECTIONS. MIDMARK CORP. CANNOT BE HELD RESPONSIBLE FOR INSTALLATIONS WHICH DO NOT CONFORM TO LOCAL CODES.
1. STUBS FOR WATER, AIR AND VACUUM CAN BE PLACED ANYWHERE WITHIN THE DOUBLE-LINE PERIMETERS.
 2. WATER: 3/4" DIAMETER HOLE REQUIRED FOR 1/2" NPT PIPE STUB. STUB TO EXTEND 1" MIN / 2-1/2" MAX FROM FINISHED WALL SURFACE. INSTALL MANUAL SHUT-OFF VALVE (SUPPLIED). FLUSH ALL DEBRIS FROM LINE BEFORE INSTALLING VALVE. WATER PRESSURE NOT TO EXCEED 50 PSI.
 3. AIR: 3/4" DIAMETER HOLE REQUIRED FOR 1/2" NPT PIPE STUB. STUB TO EXTEND 1" MIN / 2-1/2" MAX FROM FINISHED WALL SURFACE. INSTALL MANUAL SHUT-OFF VALVE (SUPPLIED). FLUSH ALL DEBRIS FROM LINE BEFORE INSTALLING VALVE. AIR PRESSURE TO BE 80 PSI (MIN) TO 100 PSI (MAX).
 4. VACUUM: PLUMBING TO THE CASEWORK SHOULD BE SPECIFIED BY CENTRAL VACUUM SUPPLIER. DRILL HOLE IN WALL AS REQUIRED. EXTEND PIPE STUB 1" MIN / 2-1/2" MAX FROM FINISHED FLOOR. TERMINATE PLUMBING WITH A 5/8" OD TUBING CONNECTOR.

LEFT SIDE PLUMBING
ROUGH-IN PERIMETER

42.00"
←→

CENTER PLUMBING
ROUGH-IN PERIMETER

RIGHT SIDE PLUMBING
ROUGH-IN PERIMETER

30.50"
↑↓

LEFT SIDE PLUMBING
ROUGH-IN PERIMETER

CENTER PLUMBING
ROUGH-IN PERIMETER

RIGHT SIDE PLUMBING
ROUGH-IN PERIMETER

∨ PLACE THIS EDGE ON FLOOR ∨
(CONSIDERING FINISHED FLOOR HEIGHT)