

**Note**  
All wall plates and wiring connections within wall are provided by facility.

**Note**  
Connecting adapters E with cables F and G to table and PC are optional, and only required to activate the IQhub®(s) on side of table and fully control the table from the PC.

**625 Table connections to Midmark Digital Vital Signs Device and PC**


Item	Description	Part Number
A	6-ft Serial Cable (straight)	015-3074-04
B	4-ft Ethernet Patch Cable	015-10124-00
C	USB/Ethernet Adapter 1.1	015-10122-00
D	Digital Vital Signs USB Cable (comes w/ Vitals)	3-009-0016
E	USB/Ethernet Adapter 2.0 w/ power plug	015-10121-00
F	4-ft Ethernet Patch Cable	015-10124-00
G	USB A to B	015-2945-00


**625 Table to PC**

- For 015-10121-00: After the USB and Ethernet cables are connected to each of the two adapters, ensure the adapter with the power source is plugged into the power source before connecting the other adapter into the PC. If done incorrectly unplug adapter from computer and replug it in for proper functionality.

**Midmark Digital Vital Signs Device on PC Setup**

- Open Midmark Digital Vital Signs Device Control Panel
- Press the "CONNECT" button to connect the Vitals machine through USB port
- Go to Device Manager on PC
- Under Ports (COM & LPT) note the COM # of the 625 Table
- Exit Device Manager
- In Midmark Digital Vital Signs Device Control Panel click "Settings"
- Ensure USB is selected in the "Connection Method" box
- In the Scale Settings box click "Settings"
- Under Device ensure "Midmark 625 Scale" is selected in the Scale drop down
- Under Com Port, select the COM # that matches the COM # listed in the device manager
- Go back to the Vitals screen by pressing the "Measurement" Tab
- Click on the image of the Table and your Scale and Table will be connected and ready to use


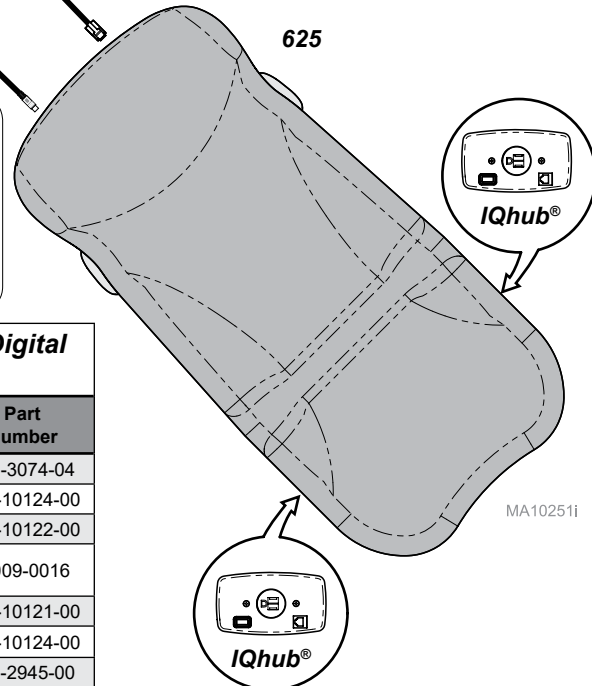


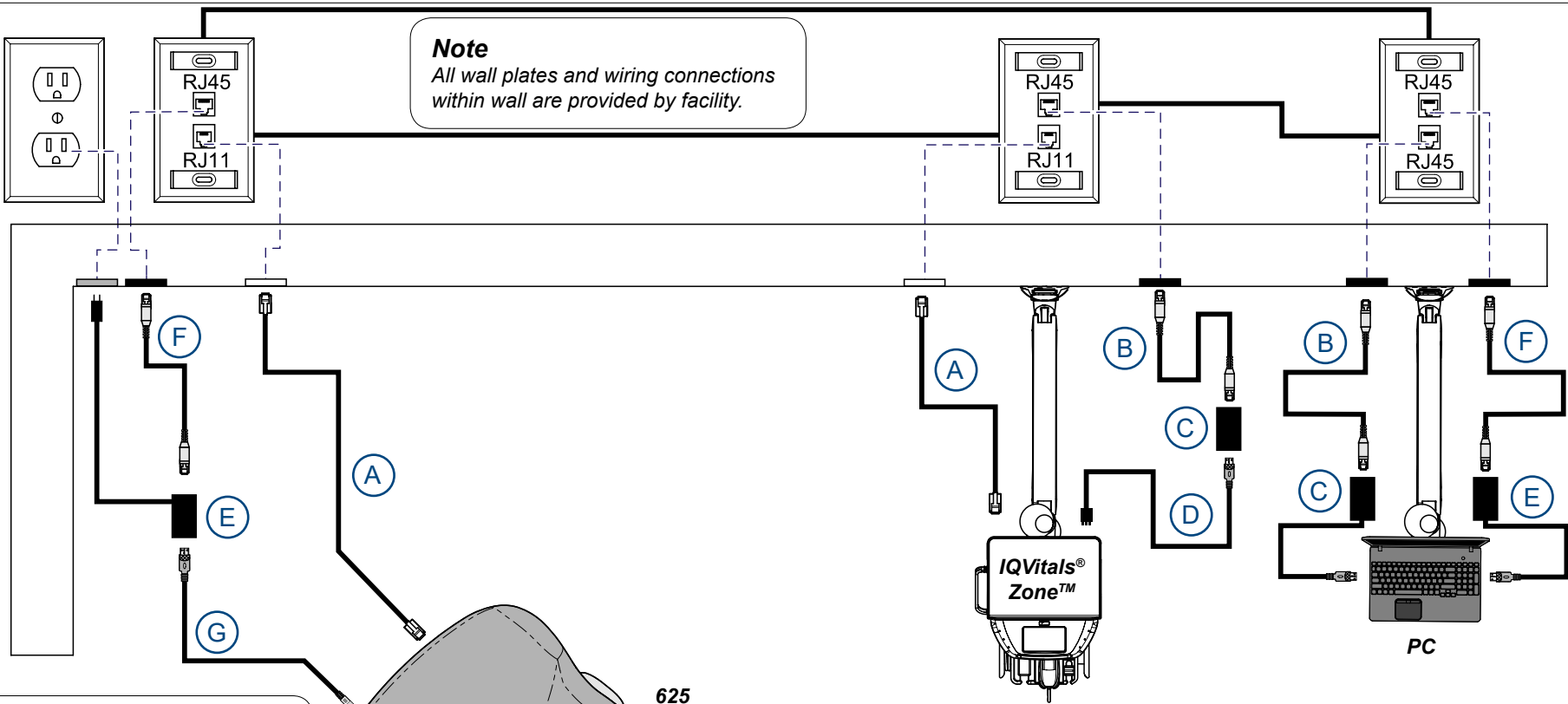

**Note**  
All wall plates and wiring connections within wall are provided by facility.

**Note**  
Connecting adapters E with cables F and G to table and PC are optional, and only required to activate the IQhub® (s) on side of table.

625

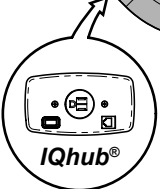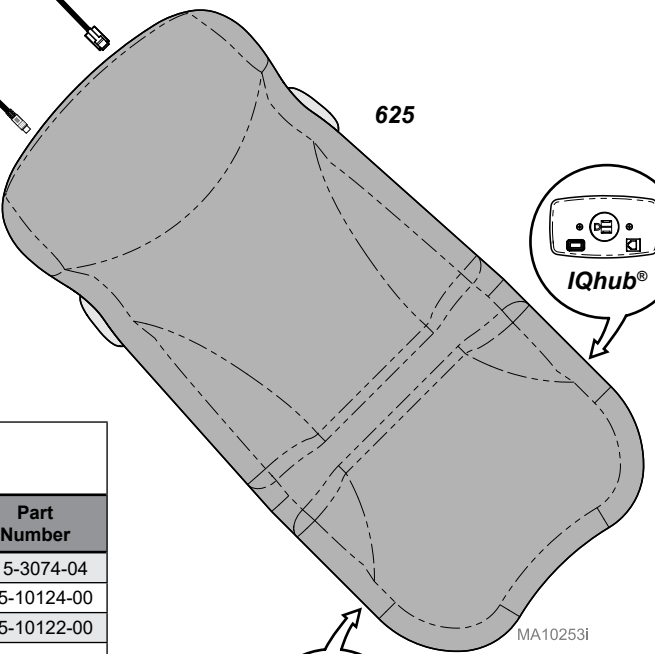

**625 Table connections to IQvitals® Zone™ and PC**

Item	Description	Part Number
A	6-ft Serial Cable (straight)	015-3074-04
B	4-ft Ethernet Patch Cable	015-10124-00
C	USB/Ethernet Adapter 1.1	015-10122-00
D	IQvitals® Zone™ USB Cable (comes w/ Vitals)	2-200-0004
E	USB/Ethernet Adapter 2.0 w/ power plug	015-10121-00
F	4-ft Ethernet Patch Cable	015-10124-00
G	USB A to B	015-2945-00

**Midmark IQvitals® Zone™ on PC Setup**

- Open the Midmark Digital Vitals Control Panel
- A loading sign should appear over the screen if no device is connected
- Once the USB's are plugged into the computer the loading sign disappears and the device should be connected