


**Note**

All wall plates and wiring connections within wall are provided by facility.

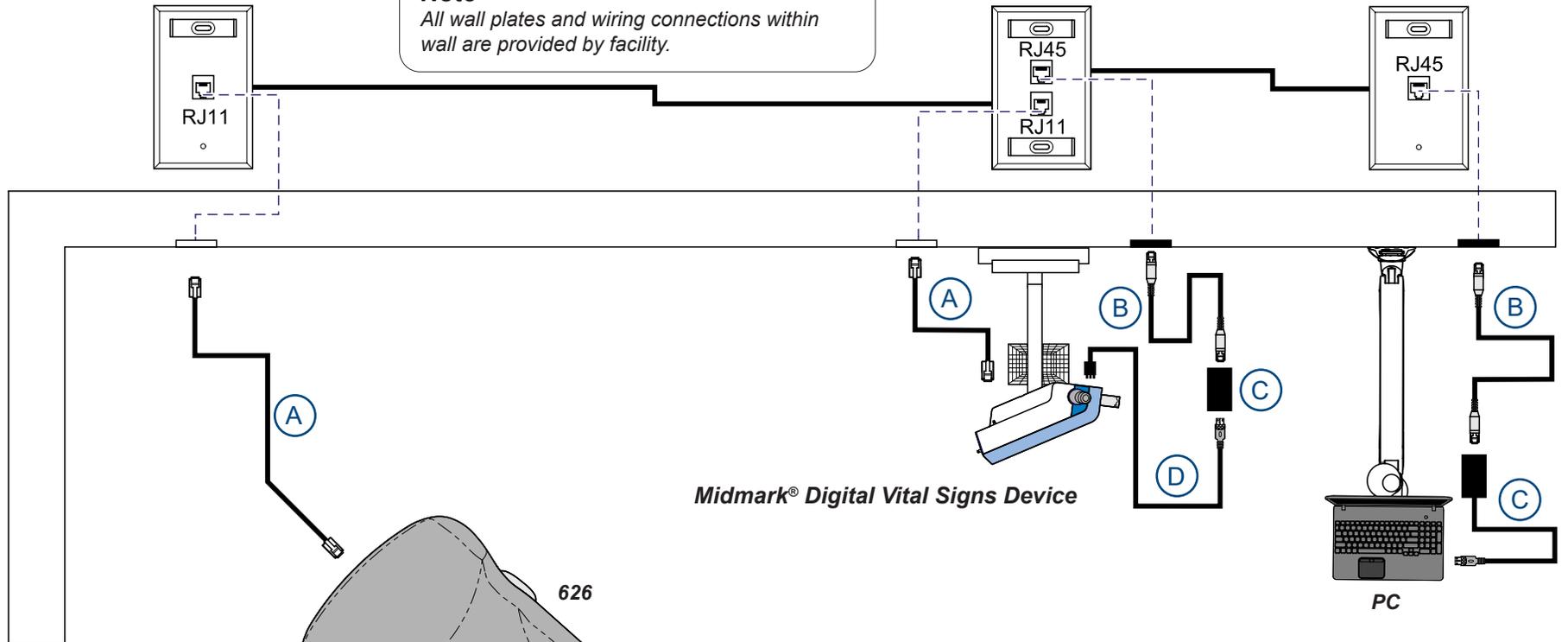


**626 Table connections to Midmark Digital Vital Signs Device and PC**

Item	Description	Part Number
A	6-ft Serial Cable (straight)	
B	4-ft Ethernet Patch Cable	
C	USB/Ethernet Adapter 1.1	
D	Midmark Digital Vital Signs Device USB Cable (comes w/ Device)	3-009-0016

**Midmark Digital Vital Signs Device on PC Setup**

- Open Midmark Digital Vital Signs Device Control Panel
- Press the "CONNECT" button to connect the device through USB port
- Device will then be connected and ready to use

**Note**  
All wall plates and wiring connections within wall are provided by facility.

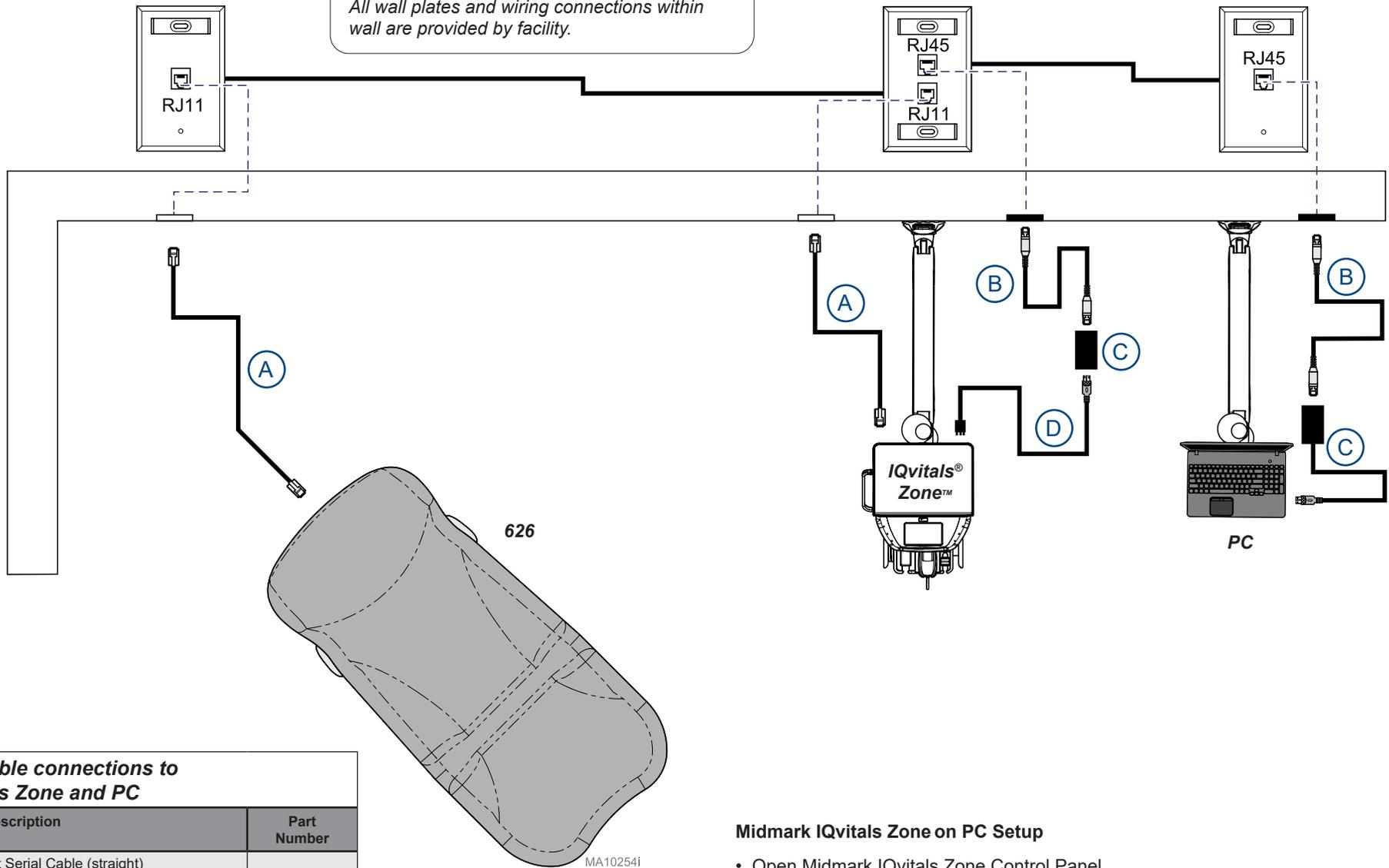

**626 Table connections to IQvitals Zone and PC**

Item	Description	Part Number
A	6-ft Serial Cable (straight)	
B	4-ft Ethernet Patch Cable	
C	USB/Ethernet Adapter 1.1	
D	IQvitals® Zone™ USB Cable (comes w/ Vitals)	2-200-0004

**Midmark IQvitals Zone on PC Setup**

- Open Midmark IQvitals Zone Control Panel
- A loading sign should appear over the screen if no device is connected
- Once the USBs are plugged into the computer the loading sign disappears and the device should be connected