

Wireless Hand / Foot Control Installation

Wireless Hand Control	002-10049-01
Wireless Hand Control W/Scale	002-10049-02
Wireless Foot Control	002-10050-01

Step 1: Associate Hand/Foot Control(s) to Base Station PC board.

- Install batteries in back of Hand/Foot Control(s).
- Disconnect chair power for at least three seconds.
- Reconnect power and wait until the chair "Beeps" three times.
- Press and hold the Stop and Base Down buttons until the Control "Beeps" three times.

Note: Two "Beeps" indicates an unsuccessful association..



Equipment Alert

A maximum of one hand control and one foot control may be associated to the chair at any one time.



Equipment Alert

Controllers must be within a Fifteen foot radius of chair to be able to command the chair.



Note

The wireless controllers will only function with the chair they have been "associated" with. This prevents unwanted movement in locations with more than one chair.

