Socket Accessory Setup Guide

Applies to: Patient Support Rails [9A600001]

Equipment Alert

Ensure rails do not appear uneven.

Rails should be horizontally straight.

Unevenly positioned rails may not be locked.

Conflict when patient support rails and 253-002 exam light mounted on model 626, 627, 224, or 225 tables. Use 253-003 exam light with extended spline mount instead.


Contents:

Setup 2
Exploded View..3 & 4

Special Tools:

none

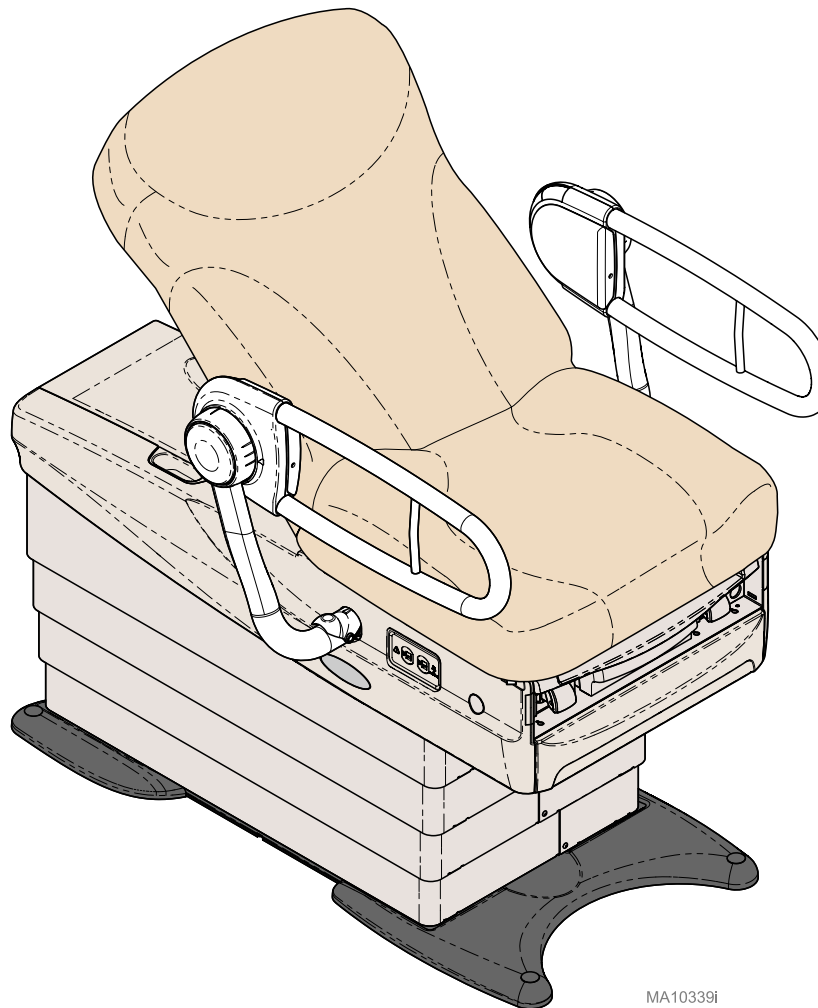
Language of origin: English


MA10339i

**WARNING**

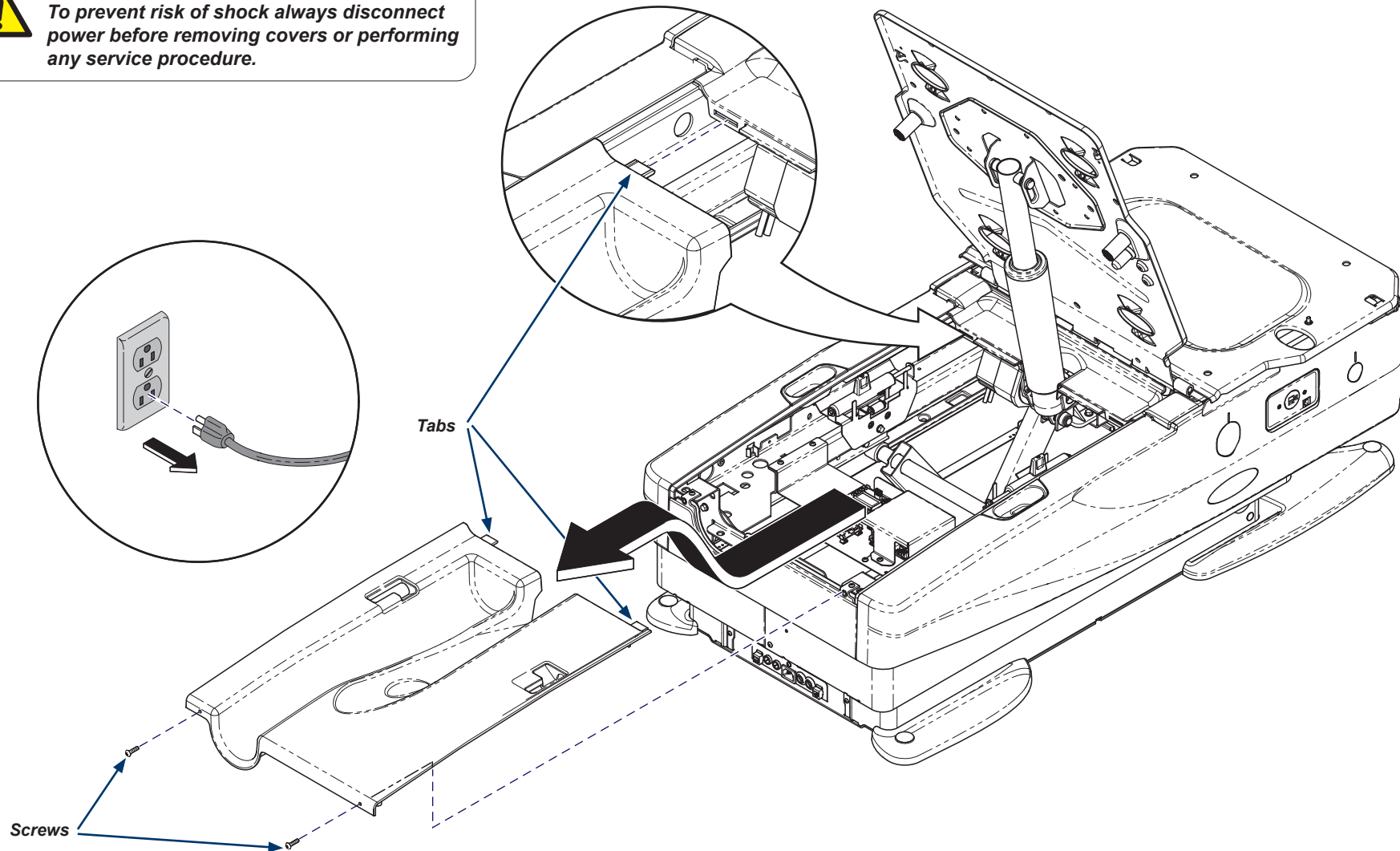To prevent risk of shock always disconnect power before removing covers or performing any service procedure.


**Step 1: Remove Top Cover...**

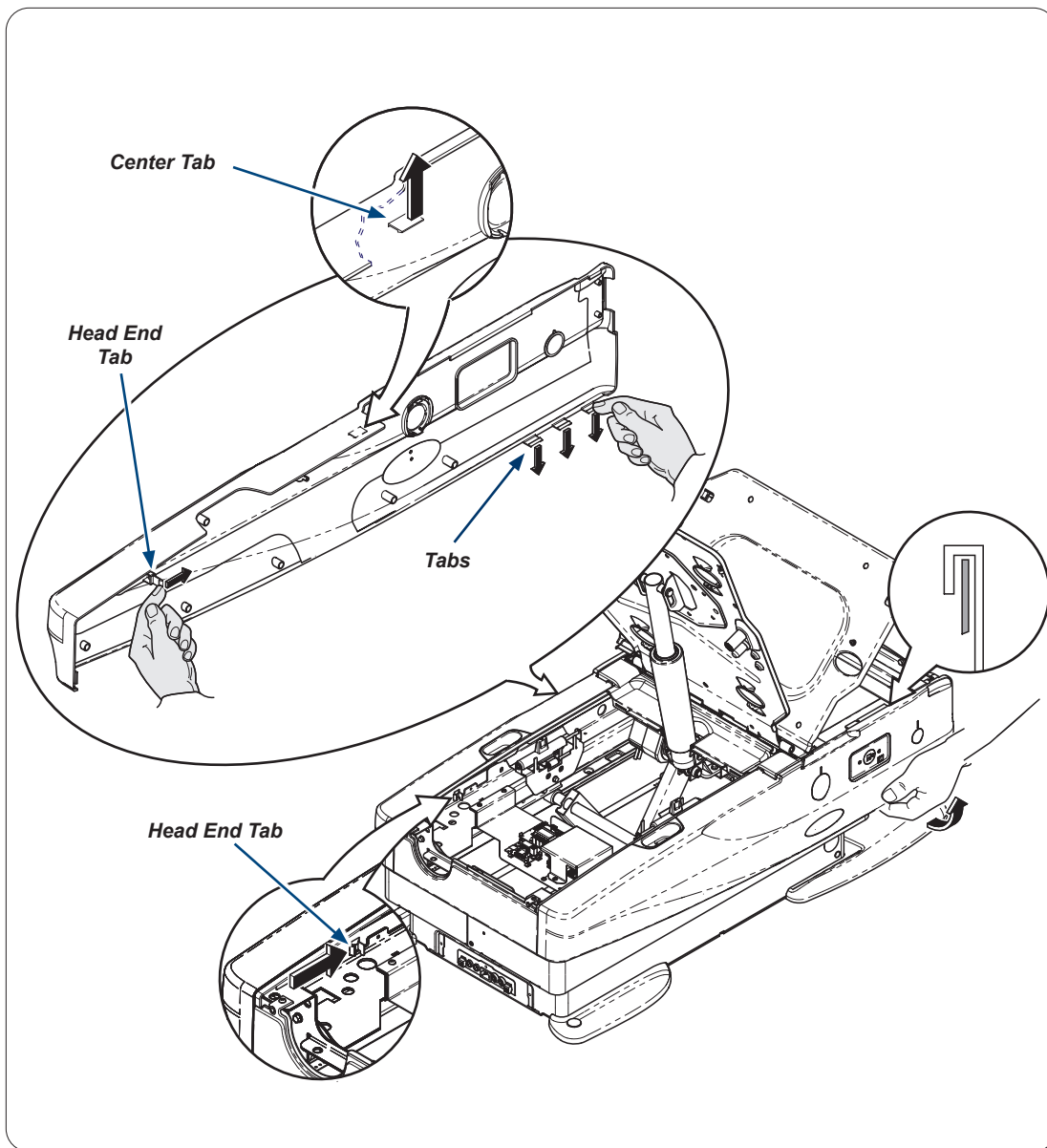- A) Position chair back in the Up position.
- B) Unplug chair.
- C) Remove two screws from back of Top Cover.
- D) Pull cover toward head end to disengage tabs, then lift up to remove cover.


Step 2: Remove Side Cladding...

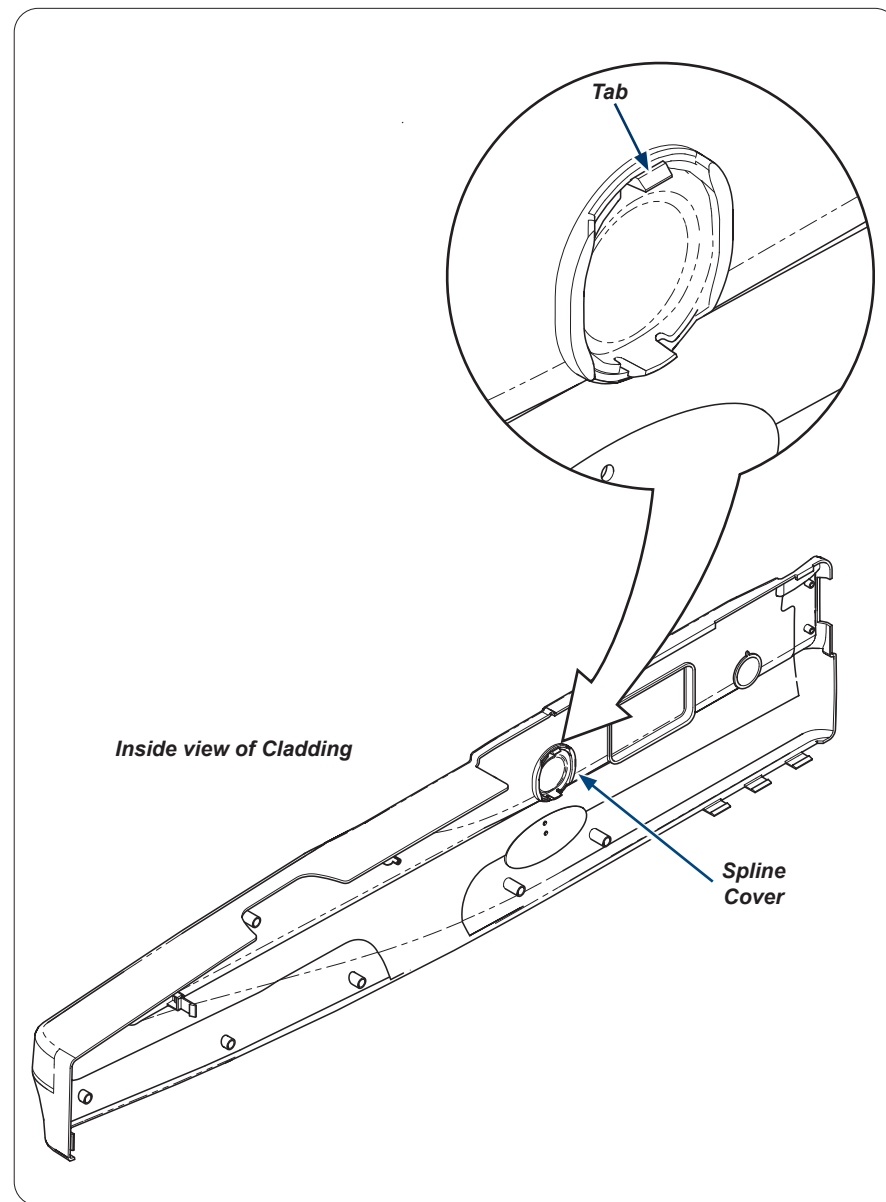- Locate tab inside head end of chair. Push tab toward foot end of chair. Pull cladding away from chair.
- Pull down on three tabs located at the bottom foot end of the cladding and pull cladding away from chair.
- Locate tab inside at center of chair (footrest must be extended) push tab up and pull cladding away from chair.
- Push up on Cladding, tip up and remove.
- Repeat process for opposite side.


Step 3: Remove Spline cover...

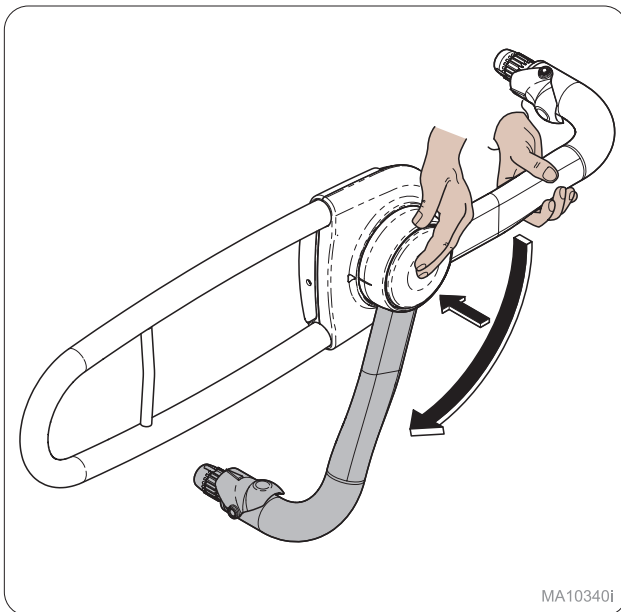- Press down on the top tab of spline cover.
- Push spline cover out of cladding.
- Re-install cladding by hooking top edge onto chair, then pushing in at tab locations until they engage the chair frame.
- Position top cover in place while engaging tabs into chair, secure with two screws.


Step 4: Patient Left Hand

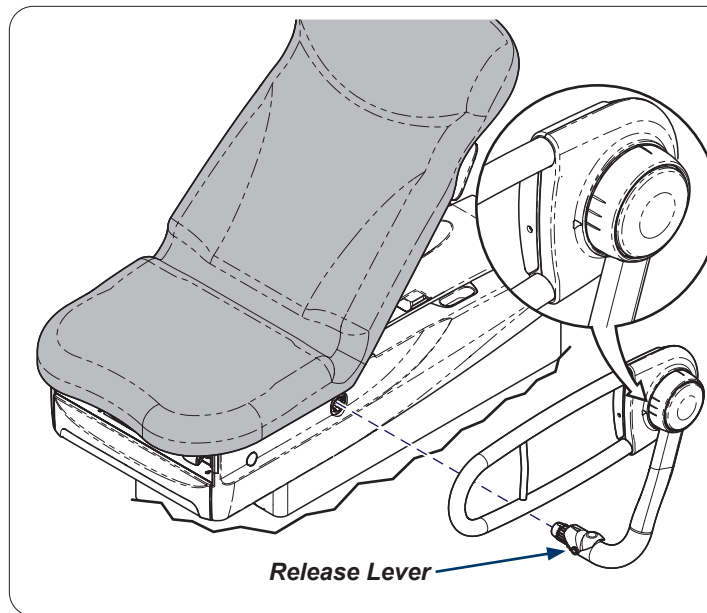Push knob in and rotate as shown.


Step 5: Patient Left Hand

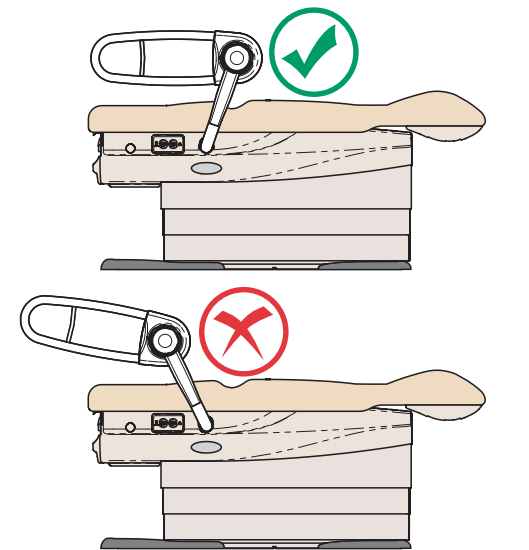Install side rail into socket until release lever locks into place.


MA10341i

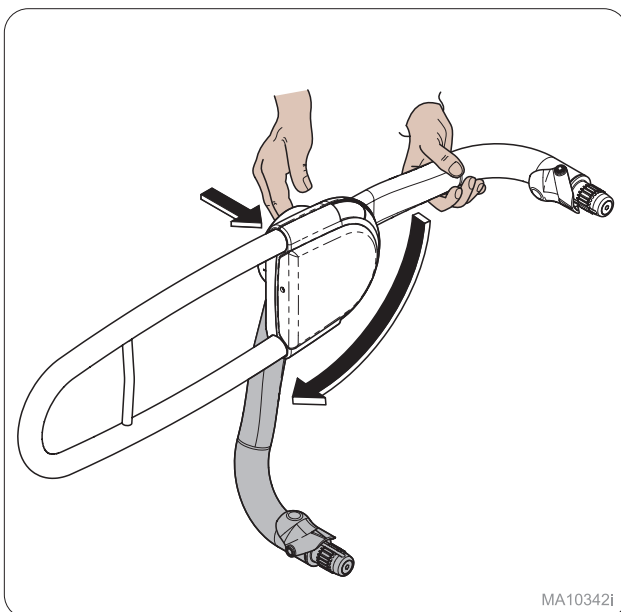
Step 6: Patient Right Hand

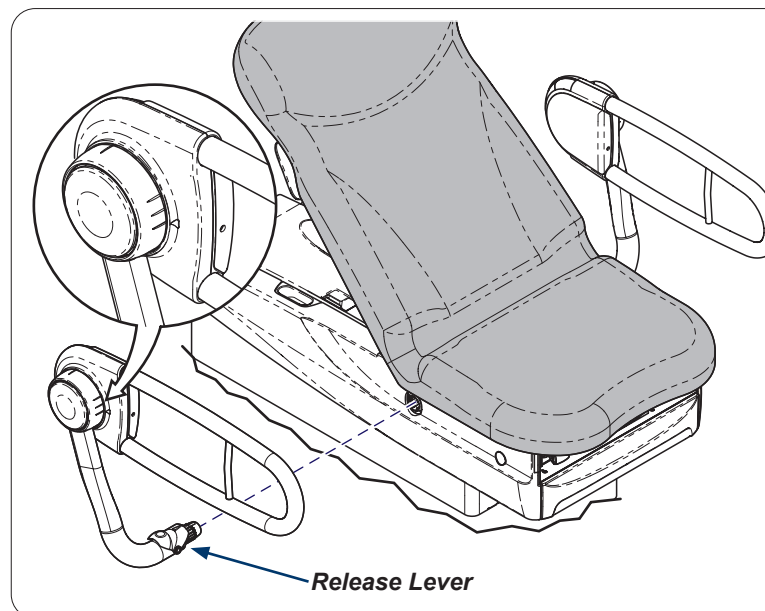Push knob in and rotate as shown.

Step 7: Patient Right Hand

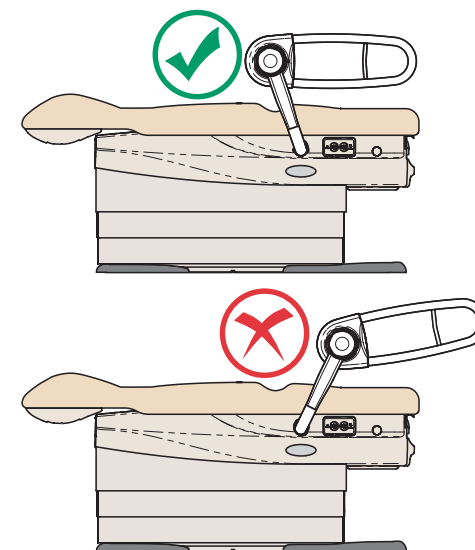Install side rail into socket until release lever locks into place.

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