Manual Examination Table Setup Instructions

Applies to Models:
204 (all)

Language of origin: English

Caution

Heavy object. Use lifting aids and proper lifting techniques when moving table to minimize muscle strains or back injury.

Equipment Alert

When moving the table, you must adhere to the following instructions.

NEVER use stirrups to lift table.

NEVER lift table by upholstered top.

[English](#)

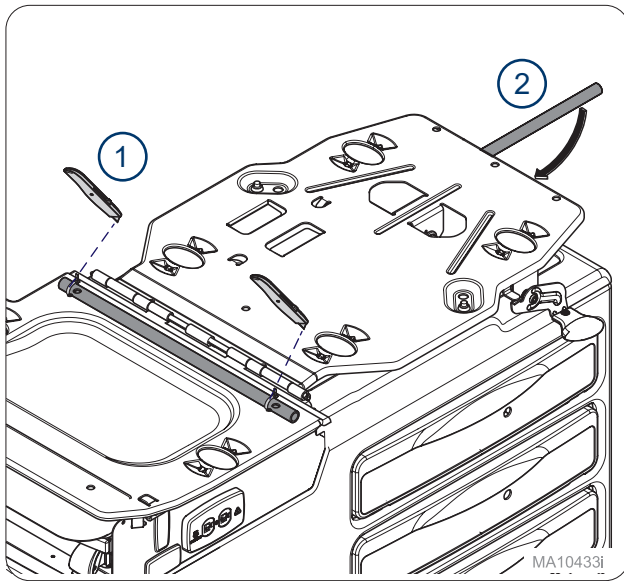
[Deutsch](#)

[Español](#)


[Français](#)


MA706700revB

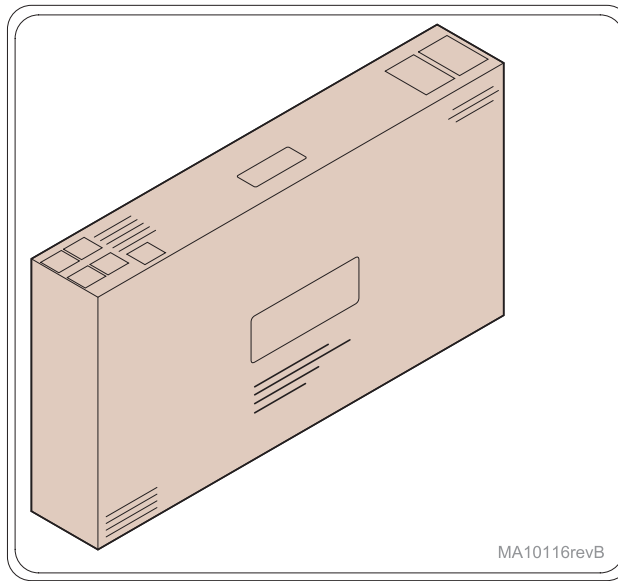
Step 1:
Cut zip ties for paper roll rod and place under table back.

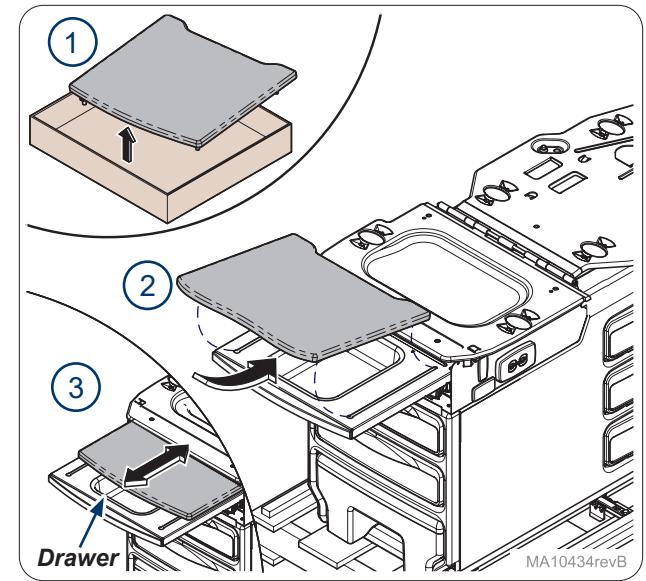
Step 2:
Locate box with upholstery.

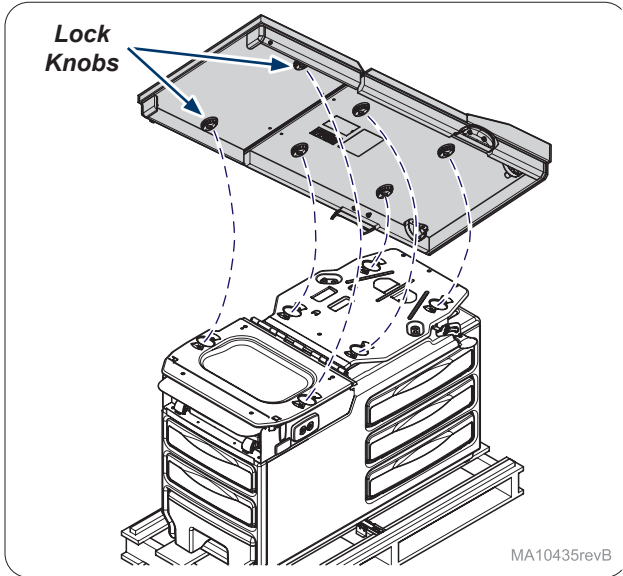
Step 3:
Remove footrest upholstery from box and install on table. Footrest should slide freely from drawer.

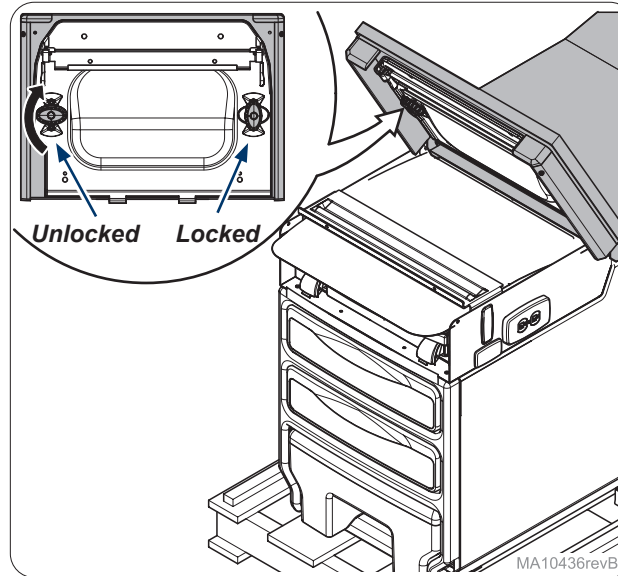
Step 4:
Install upholstery onto table top aligning lock knobs (6) with holes (6).

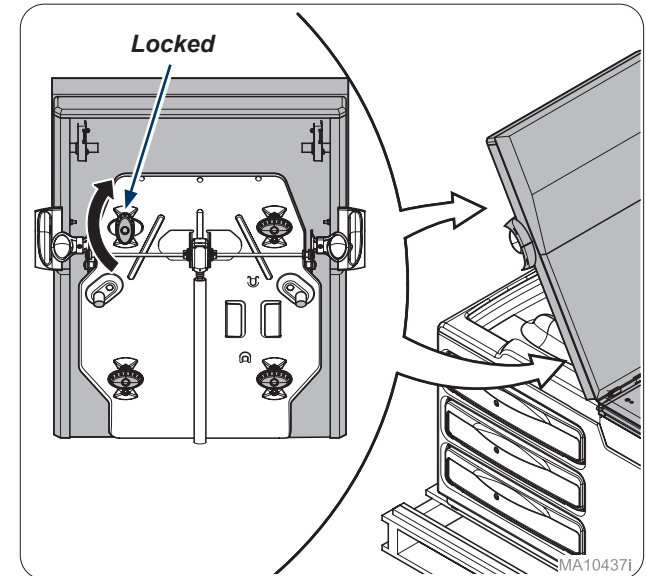
Step 5:
Lift table seat and latch locking knobs (2).

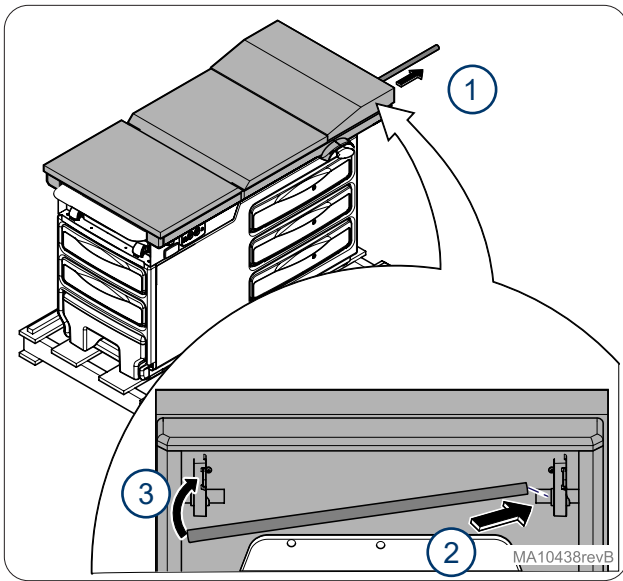
Step 6:
Lift table back using back release handle and latch locking knobs (4).

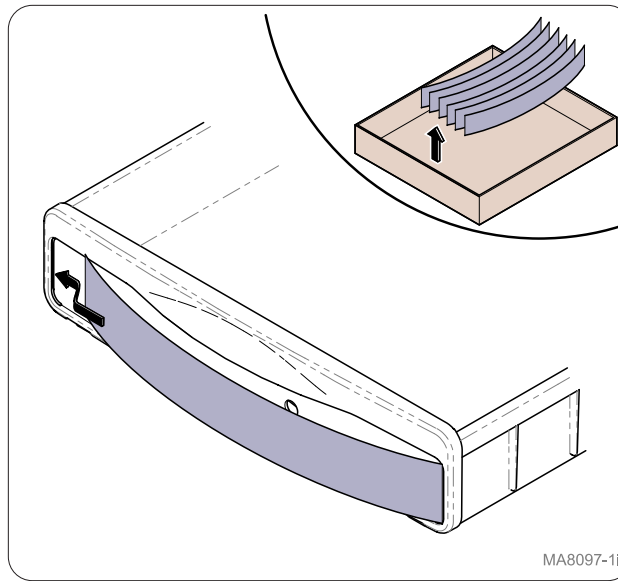
Step 7:
Remove paper towel rod and install into back of upholstery top.

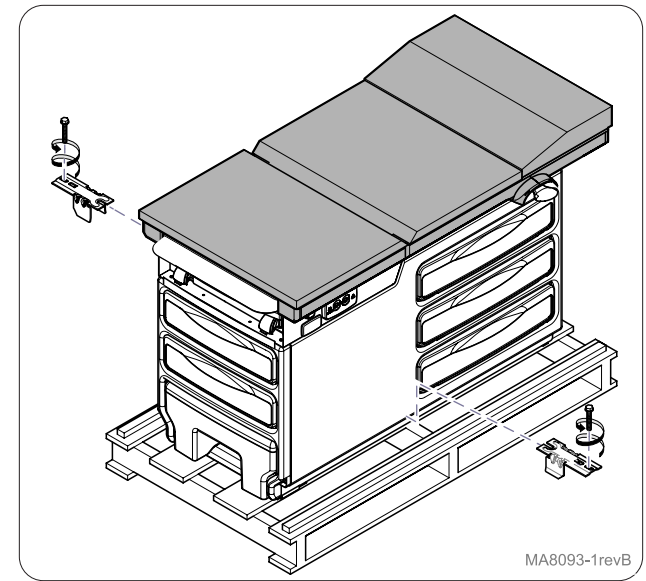
Step 8:
Remove colored drawer inserts from upholstery box. Insert all four corners into drawer fronts.

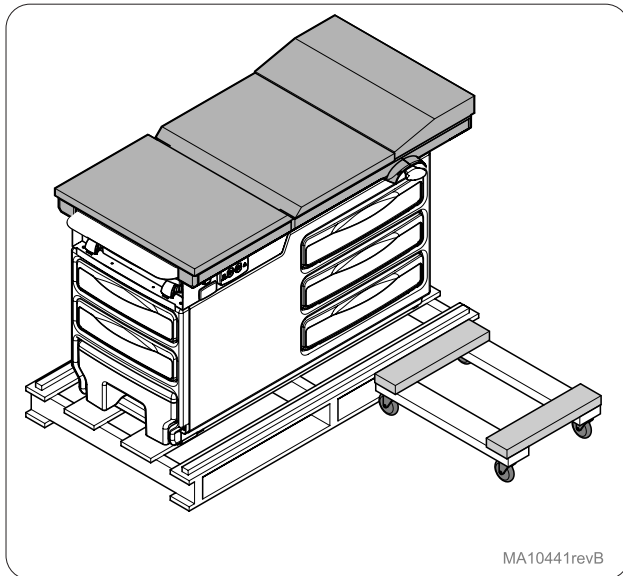
Step 9:
Remove brackets (2) from skid to release table.

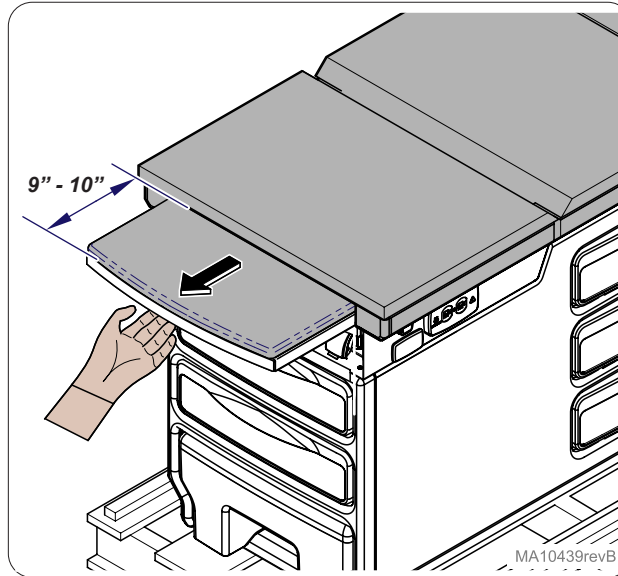
Step 10:
Place dolly next to head end of table.

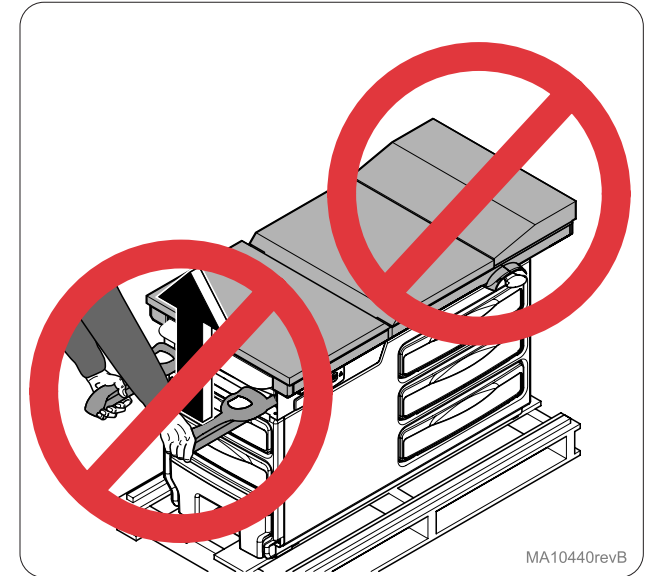
Step 11:
Slide footrest 9"-10" out of table.

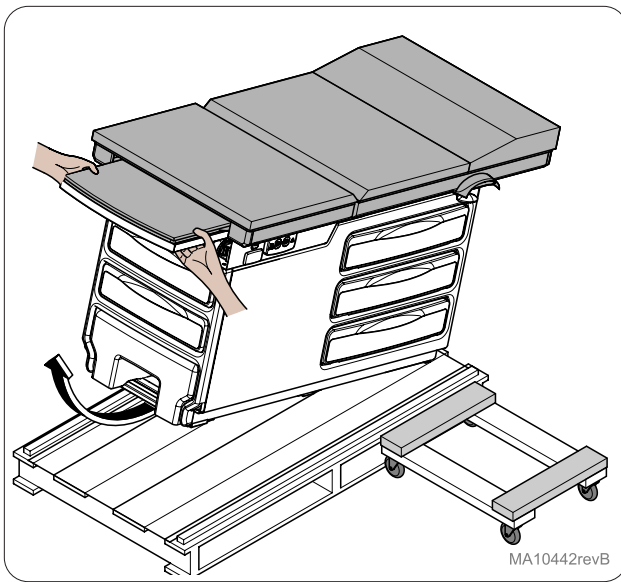
Equipment Alert
Do **NOT** lift table by Upholstery or Stirrups.

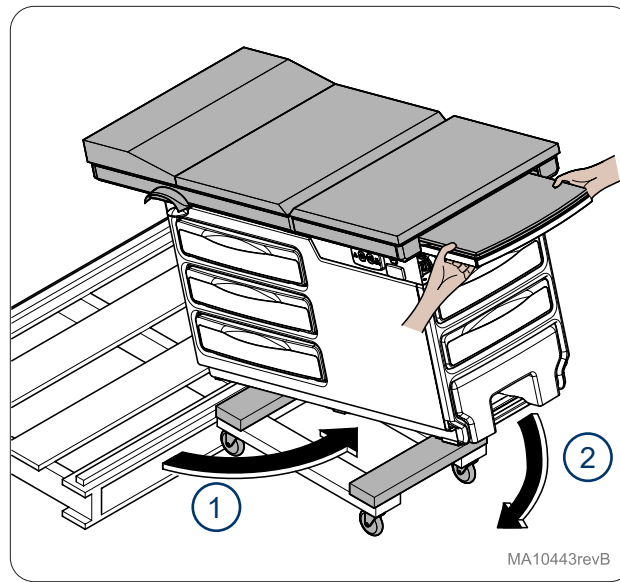
Step 12:
Using proper lifting technique lift table off front of skid.

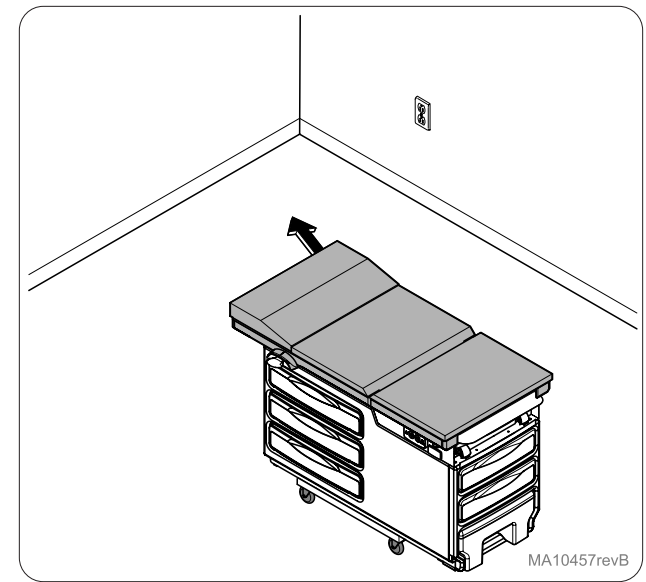Step 13:
Rotate table to dolly, and lower on dolly.

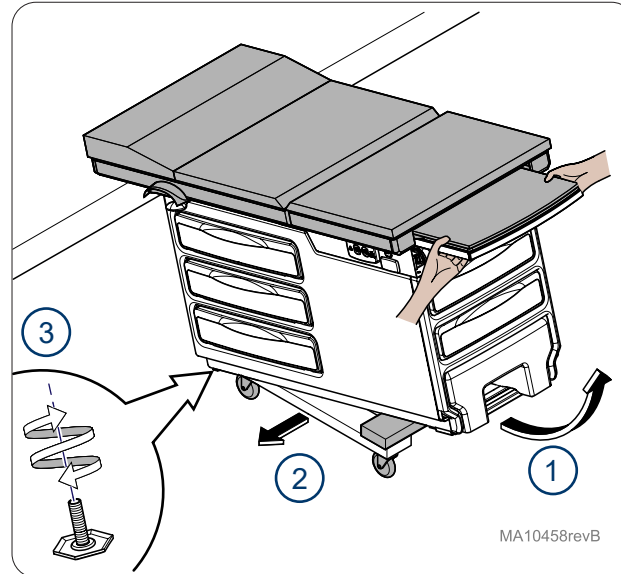
Step 14
Move table to desired location.

Step 15:
Using proper lifting technique lift table off front of dolly. Remove dolly. Adjust leveling feet (2) at front of table to level or balance table if needed.

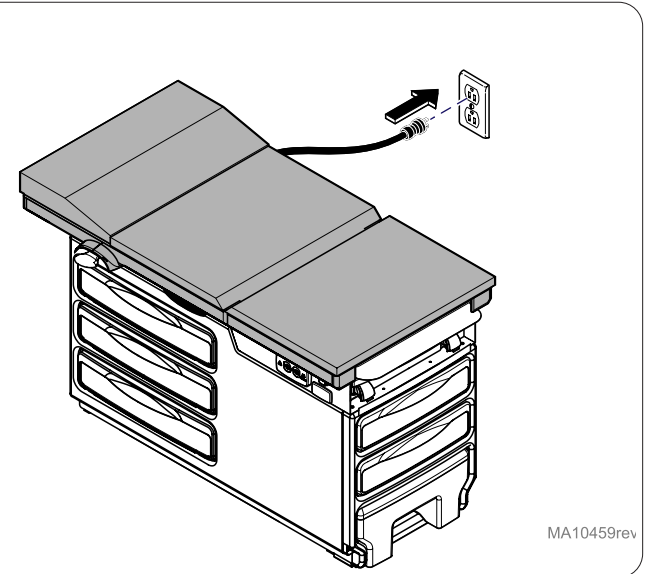Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all local electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements. For 115V rated device models with a current rating at or above 12 Amps, a dedicated branch circuit may be required if the branch circuit protection is provided by a 15 Amp breaker.

Step 16:
Plug table into wall or floor outlet.

*Note: Some models have two power cords. Both cords must be connected for proper operation.

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