224 (-011) Barrier-Free® Power Examination Chair Setup Instructions

Tools:

Battery Powered Drill / Impact Driver
Standard Wrench Set
Standard Deep Well Socket Set
Box Cutter
Furniture Dolly (4 Wheel)

Caution

Heavy object. Use lifting aids and proper lifting techniques when moving chair to minimize muscle strains or back injury.

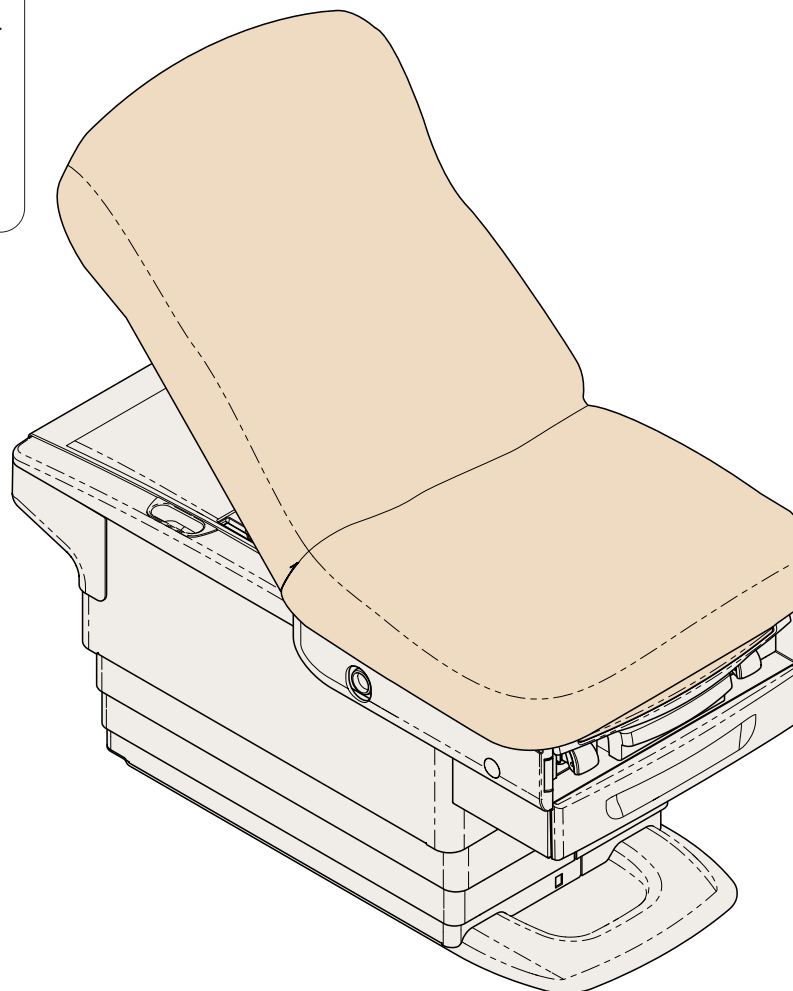
Equipment Alert

When moving the chair, you must adhere to the following instructions.


ALWAYS: a) Transport chair in full-down position with base shroud supports in place.


NEVER: a) Push on base shrouds.
b) Lift chair by stirrups, back section, or upholstered top.

MA10471i

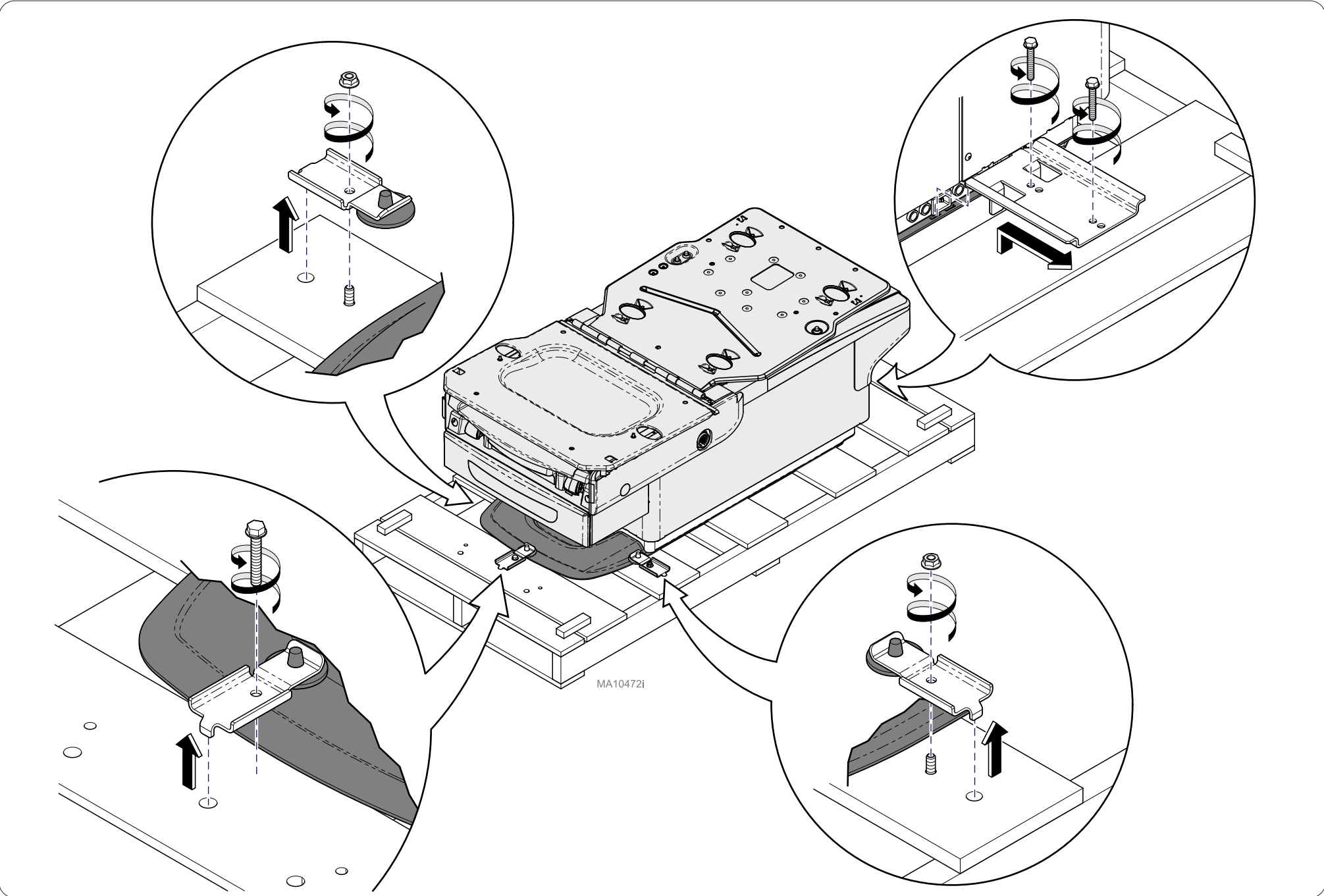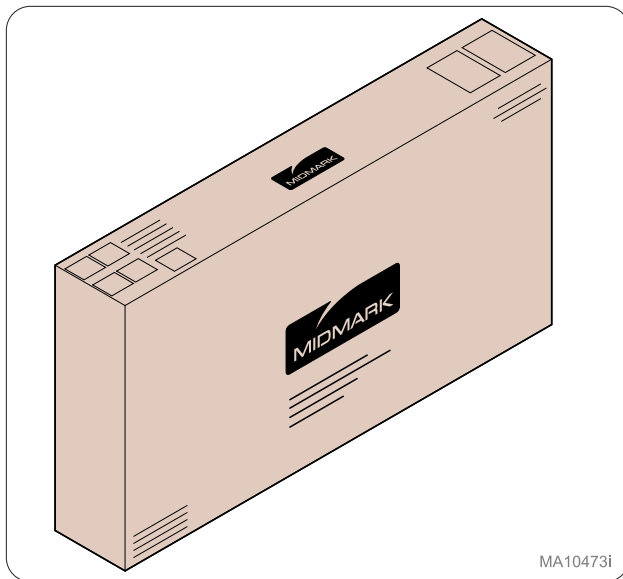
Step 1:
Remove brackets (4) from skid to release chair.


MA10473i

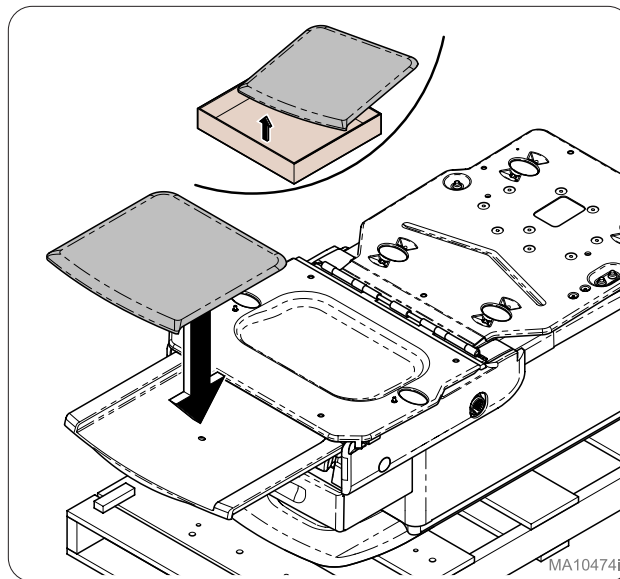Step 2:
Locate box containing upholstery.


MA10474i

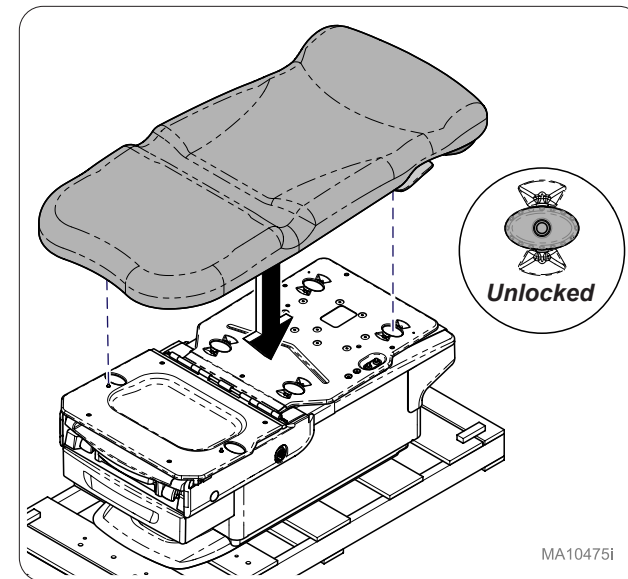Step 3:
Place footrest upholstery, inserting pin on bottom of upholstery into footrest shelf hole.


MA10475i

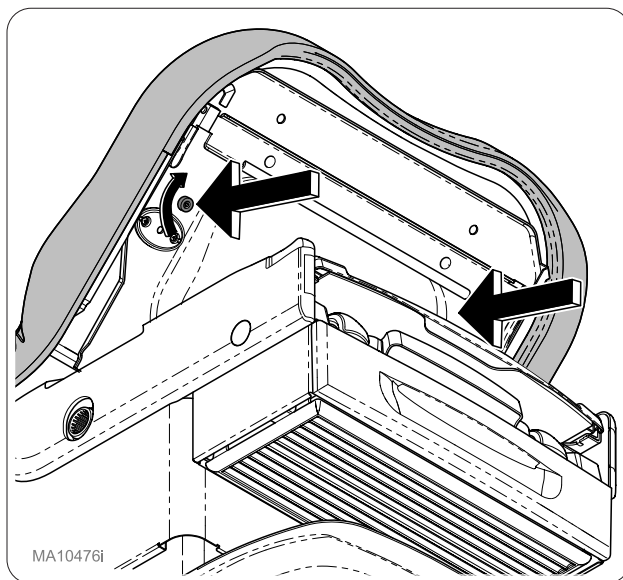Step 4:
With locking knobs (4) in the unlocked position, place upholstery on chair top aligning locking knobs with holes.


MA10476i

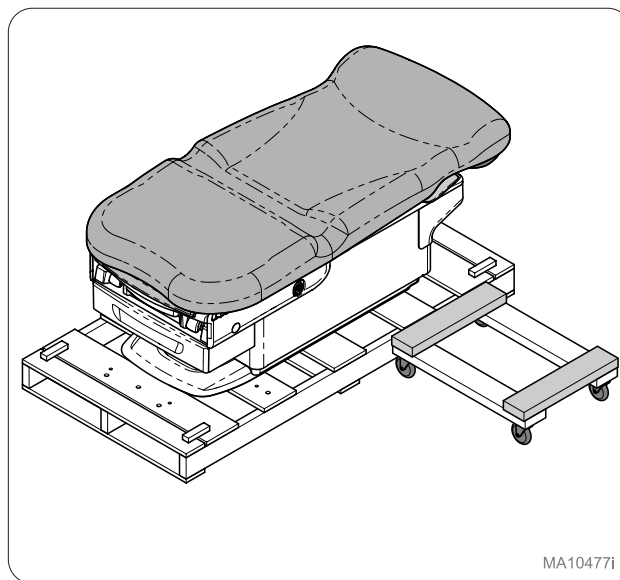Step 5:
Raise seat pan 90 degrees and secure seat section using attached thumb screws (2). Lower seat pan.


MA10477i

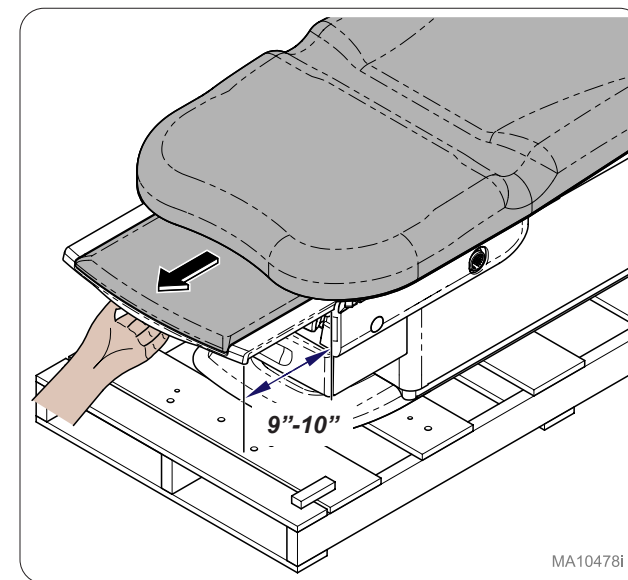Step 6:
Place dolly next to head end of chair.

MA10478i

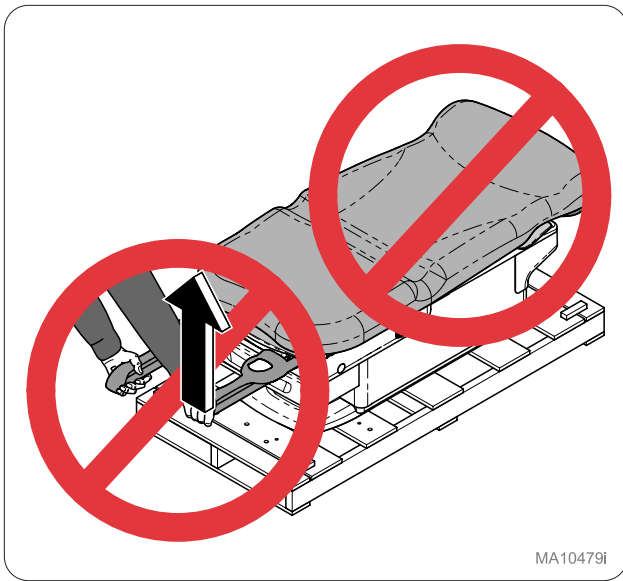Step 7:
Slide footrest 9"-10" out of chair.


MA10479i

Equipment Alert

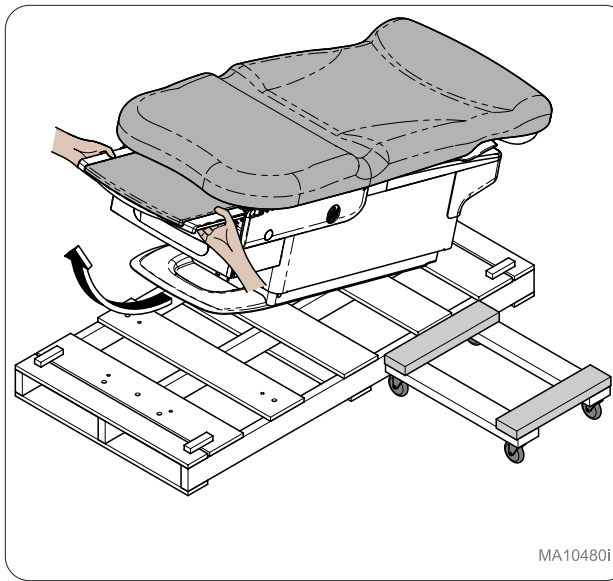
Do **NOT** lift chair by Upholstery or Stirrups.

MA10480i

Step 8:

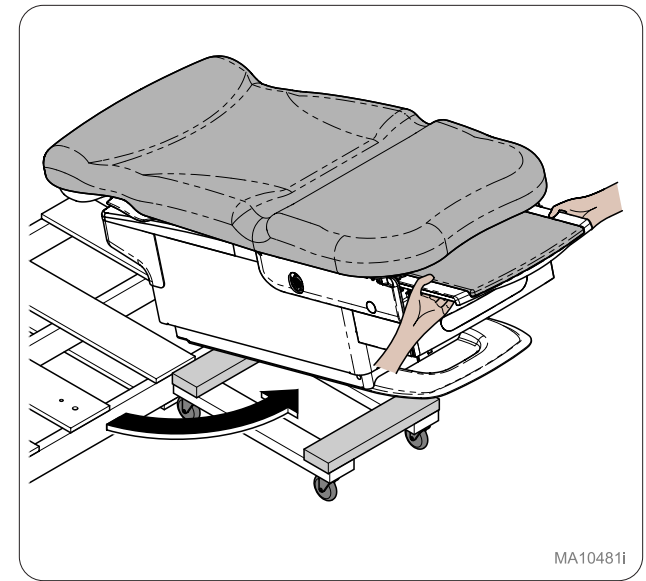
Using proper lifting technique, lift chair off front of skid.

MA10481i

Step 9:

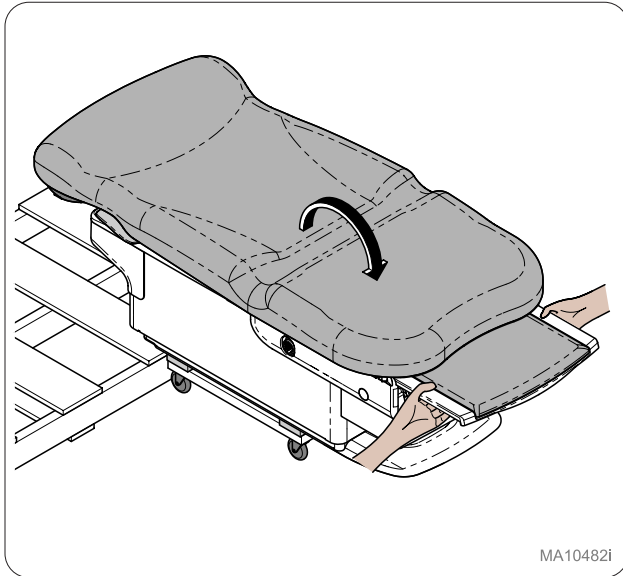
Rotate chair to dolly.

MA10482i

Step 10:

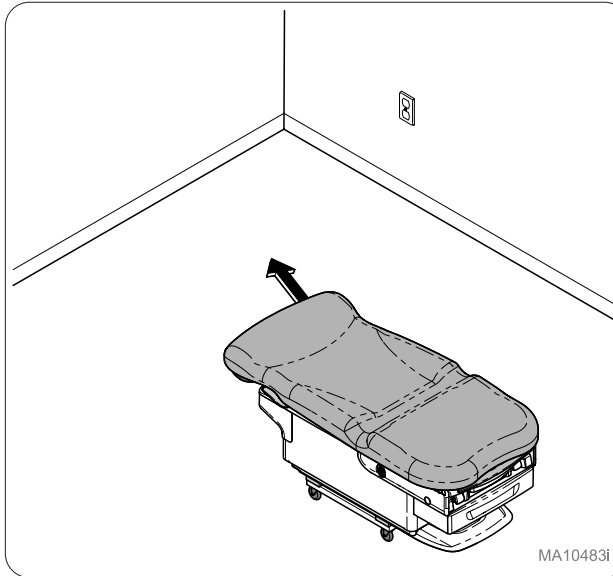
Lower chair onto dolly.

MA10483i

Step 11:

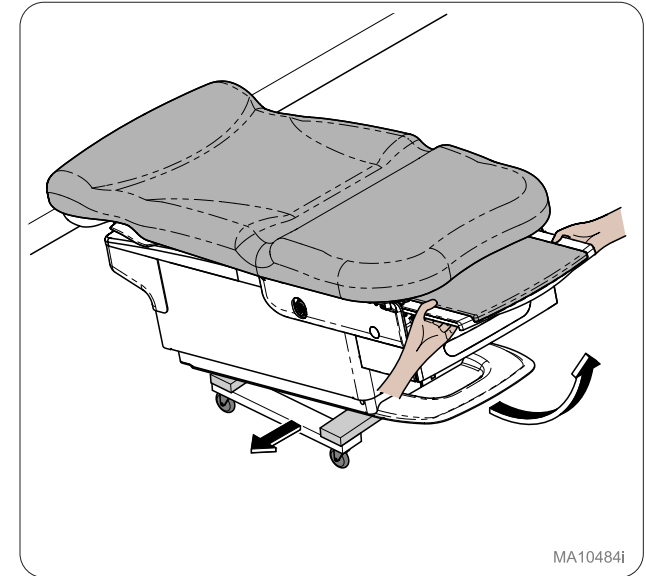Move chair to desired location.


MA10484i

Step 12:

Remove chair from dolly.

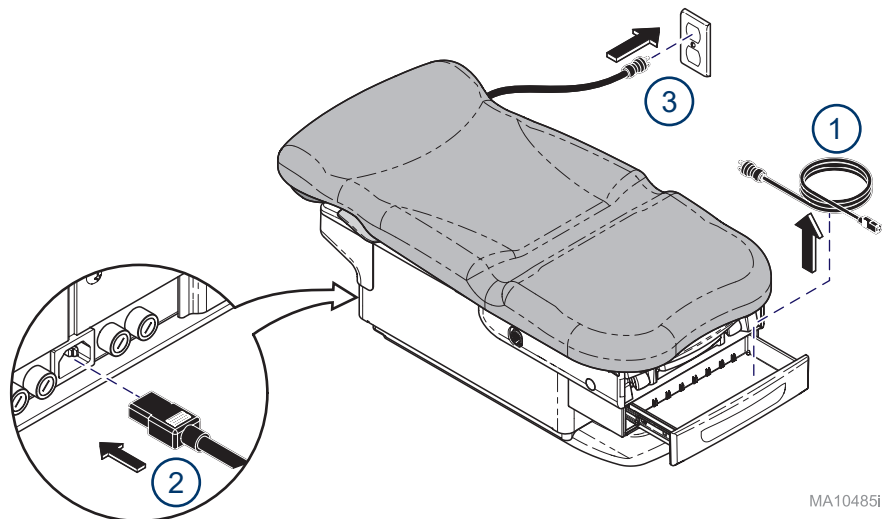
Before connecting the Midmark device power cord into an electrical power source outlet, it is recommended to contact a local licensed electrician or a local Authority Having Jurisdiction over the internal building power distribution to verify that the device when powered up will comply with all applicable electrical codes. Use the electrical power rating marked on the device when determining the appropriate electrical branch circuit and outlet requirements.

MA10485i

Step 13:

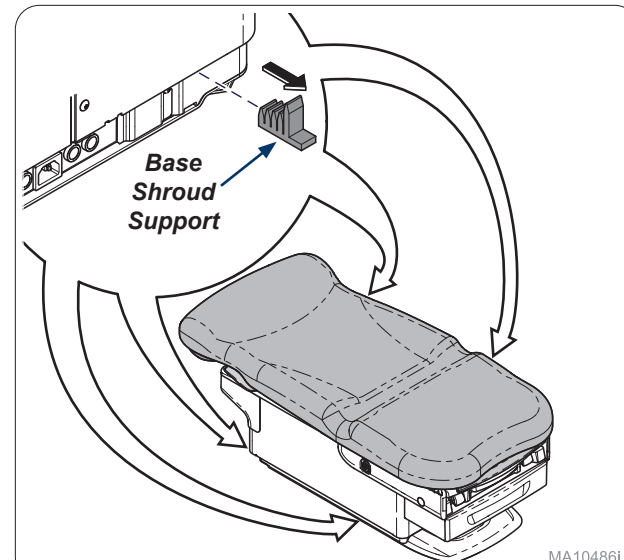
Remove power cord from drawer, connect to rear of chair, and plug into wall or floor outlet.

MA10486i


Step 14:

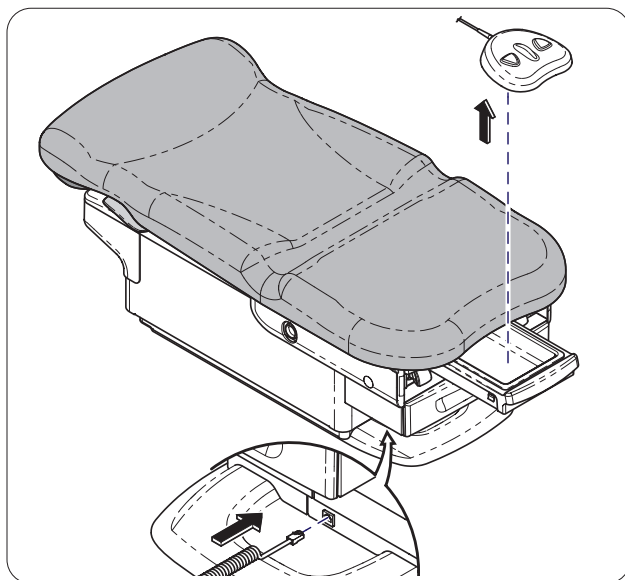Remove base shroud supports (4).

Step 15:

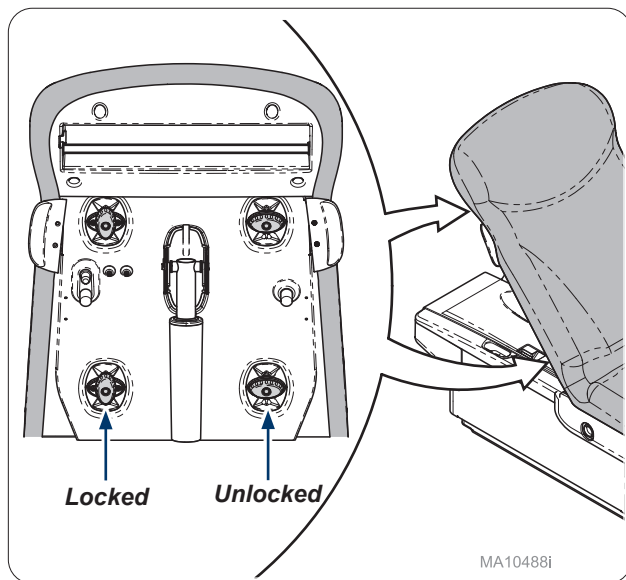
Remove foot control from treatment pan. Connect foot control to front base of chair.

MA10488i

Step 16:

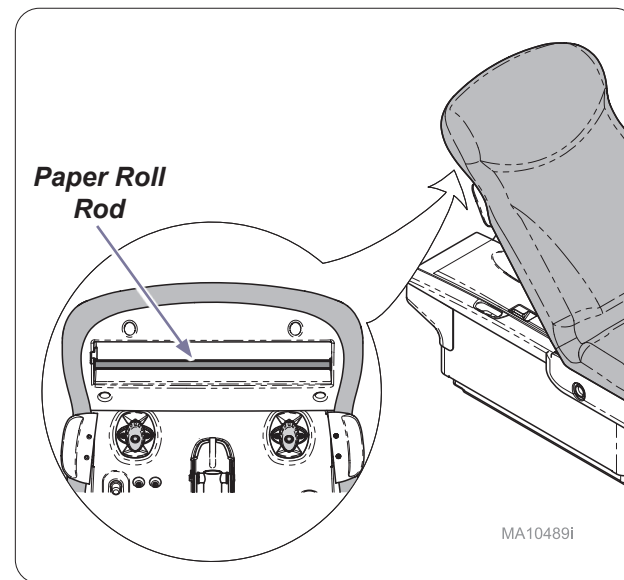Raise back and latch upholstery locking knobs (4).

MA10489i

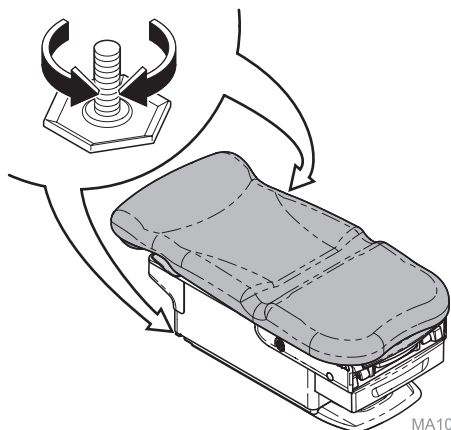
Step 17:

Ensure paper roll rod is in place.

Equipment Alert


The chair must be leveled or stable to ensure proper operation.

MA10490i

Step 18:

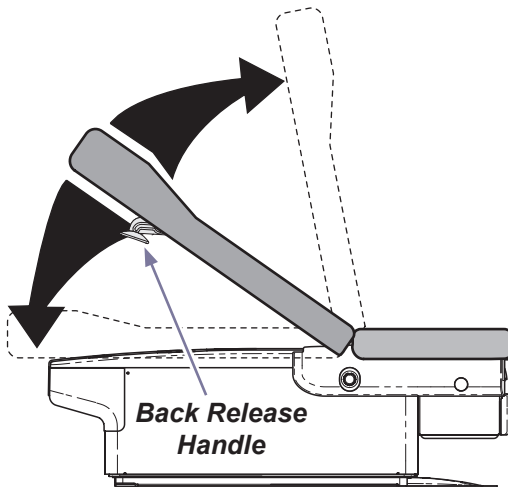Adjust leveling screws (2) at head end of chair to level or balance chair.

MA10491i

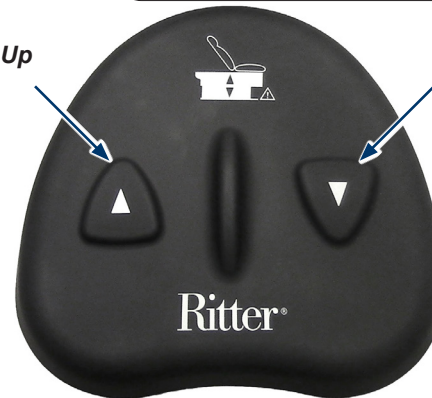
Step 19:

Verify proper manual Back operation.

Note

If a malfunction is detected contact Midmark Technical Service by calling: 1-800-MIDMARK.


Base Up


Base Down

Step 20:

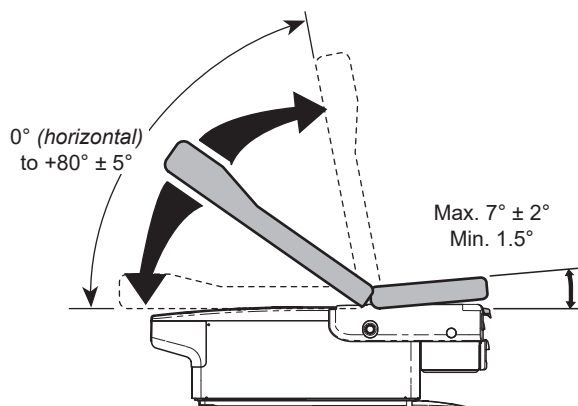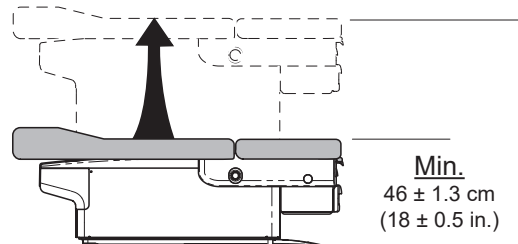Verify proper operation of foot control. Press buttons for Base Up/Down.

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