Podiatry Procedures Chair

[English](#)
[Español](#)
[Français](#)

For Models:

647 Barrier-Free®

User Guide

003-10161-99 Rev. AA6 (9/30/24)
TP202 20-42-FO-00014 Rev A1 C2169
Style P

Product Information

(The information below is required when calling for service.)


Dealer :

Date of Purchase:

Model / Serial Number:

**Midmark Authorized Service
Company:**

**Model / Serial
Number Label**

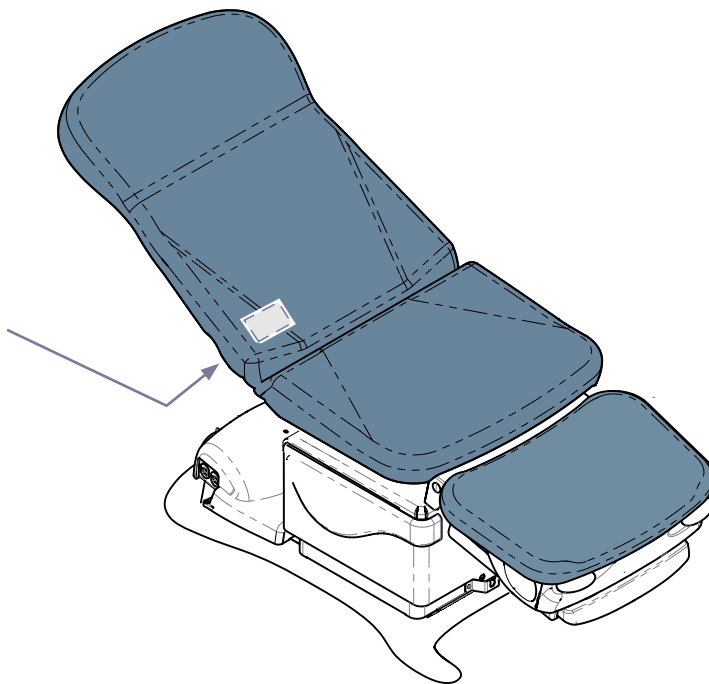MA10670i

Table of Contents

Important Information

<i>Safety Symbols</i>	4
<i>Intended Use</i>	5
<i>Electrical Requirements</i>	5
<i>Electromagnetic Interference</i>	5
<i>Transportation / Storage Conditions</i>	5
<i>Disposal of Equipment</i>	5
<i>Authorized Representatives</i>	6
<i>Power to the Chair</i>	7
<i>Connecting the Foot Control</i>	8
<i>Touch Pad Location</i>	9

Operation

<i>Back / Tilt / Base Operation</i>	10
<i>QuickChair Function</i>	11
<i>Programming Chair Positions (Programmable models only)</i>	12
<i>QuickExam Function (Non-Programmable models only)</i>	13
<i>Foot Section / Debris Tray</i>	14
<i>Chair Arms</i>	15
<i>Rotational Base Operation</i>	16

Cleaning & Maintenance

<i>Calling for Service</i>	17
<i>Cleaning</i>	17
<i>Periodic Maintenance</i>	17

Troubleshooting

<i>Powered Function(s) Not Working Properly</i>	18
<i>* Crash Avoidance System</i>	18

Specifications

<i>Range of Motion / Specifications Table</i>	19
<i>Dimensions</i>	20
<i>Model Identification / Compliance Chart</i>	21

Warranty Information

<i>Warranty Information</i>	22
-----------------------------------	----

Important Information

Safety Symbols

WARNING

Indicates a hazardous situation which could result in serious injury if not avoided.

CAUTION

Indicates a potentially hazardous situation which could result in minor or moderate injury if not avoided. It may also be used to alert against unsafe practices.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note

Amplifies a procedure, practice or condition.

Symbol Glossary

Serial Number


Refer to manual

Protective earth ground

Duty Cycle (motor run time):
30 seconds ON, 5 minutes OFF

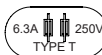Fuse replacement label

Manufacturer

Catalogue Number

Keep dry

Type B, Applied Part

Proper Shipping

Maximum stacking height
(palletted units)

Fragile

Emergency Stop

Intended Use

CAUTION

Be sure that all persons and equipment are clear of the table before activating any function. Failure to do so could result in personal injury.

This product is intended to be used as a chair / table to provide positioning and support for patients during examinations and procedures conducted by qualified medical professionals (*primarily podiatrists*).

Electrical Requirements

CAUTION

When using high frequency surgical devices or endocardial catheters:

- Use non-conductive material to insulate patient from metal portions of the chair.
- Consult operating instructions for the device before using in conjunction with drawer or upholstery heater.

Failure to comply may result in electric shock or burns to patient.

Electromagnetic Interference

This Midmark chair is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this chair:

- Remove interfering device from room
- Plug chair into an isolated circuit
- Increase separation between chair and interfering device
- Contact Midmark if interference persists

Note

Electromagnetic Compliance Statements per EN60601-1-2 are available online:
Go to: www.documark.com (Use **FIND** feature to locate: **077-0501-00**)

Transportation / Storage Conditions

Ambient Temperature Range:..... +41°F to 100°F (+5°C to +38°C)
Relative Humidity..... 10% to 90% (non condensing)
Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, and other consumable goods.

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer.
For more information go to www.p65warnings.ca.gov

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Colombia should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Técnica Electromédica S.A.
Calle 47 # 5-26
Bogota Colombia c.p. 110111
Tel. (+ 57 1) 756 87 87
atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Corporacion Almotec SA
225 metros al sur de oficina de correos
San Jose, Zapote, Costa Rica
Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.


TECNIDENTAL PANAMA S.A.
VILLA MAGNA, VILLA ZAITA
OFFICE B8
PANAMA
Tel/Fax: 343-4049/ 6614-8029
tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios
Carlos Nafarrate SA de CV
Avenida Las Américas 601,
Ladrón de Guevara, 44600,
Guadalajara, Jalisco, México | Tel (33) 3669 9800

Power to the Chair

The power indicator light is illuminated when power is properly connected to the chair. If the chair has no power, all functions will be inoperable.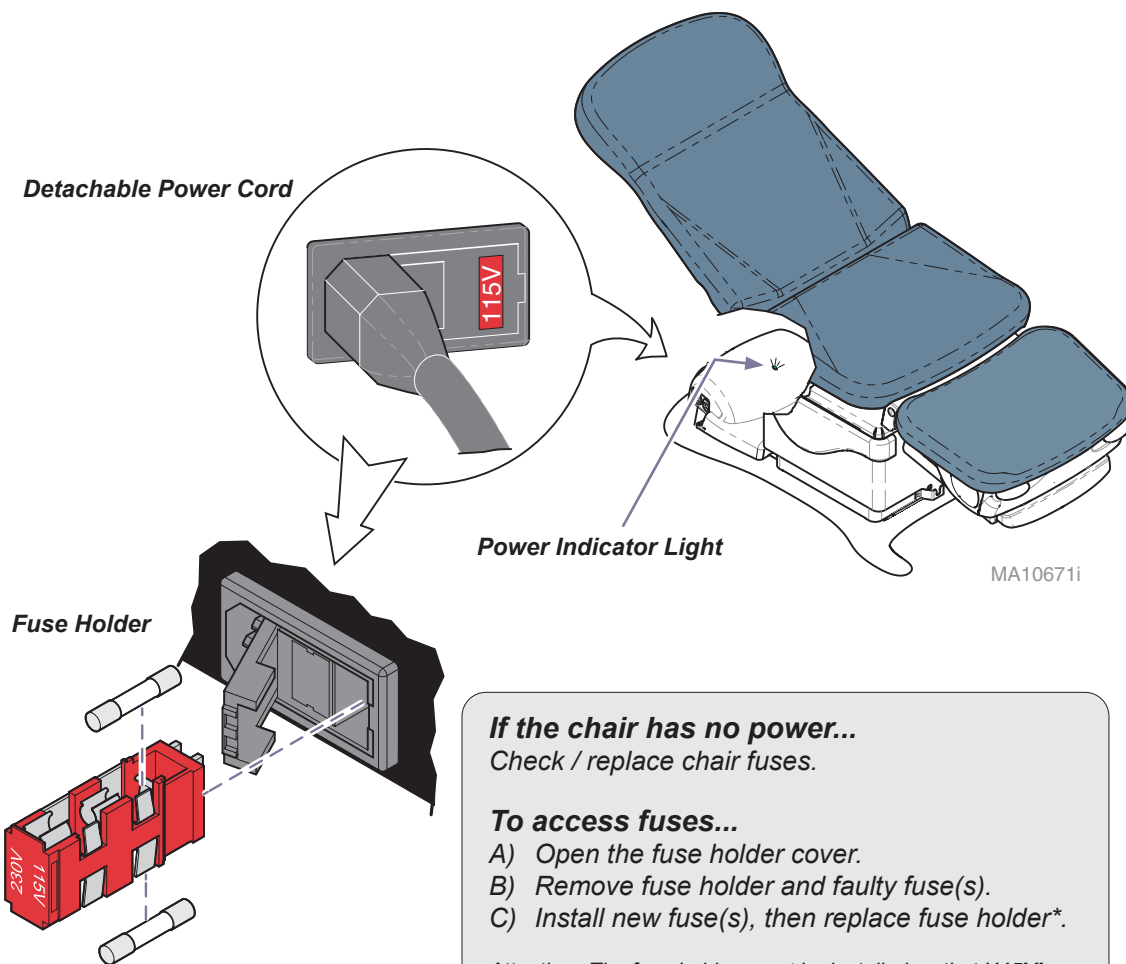

MA10671i


Connecting the Foot Control

The foot control cord inlet is located on the base column at the foot end of the chair.

To connect the foot control cord...


- A) Slide narrow portion of cord connector into bracket.
- B) Connect cord at desired inlet (see Note).
- C) Tighten locking ring.

Note: The cord connector is "keyed" and must be aligned with inlet.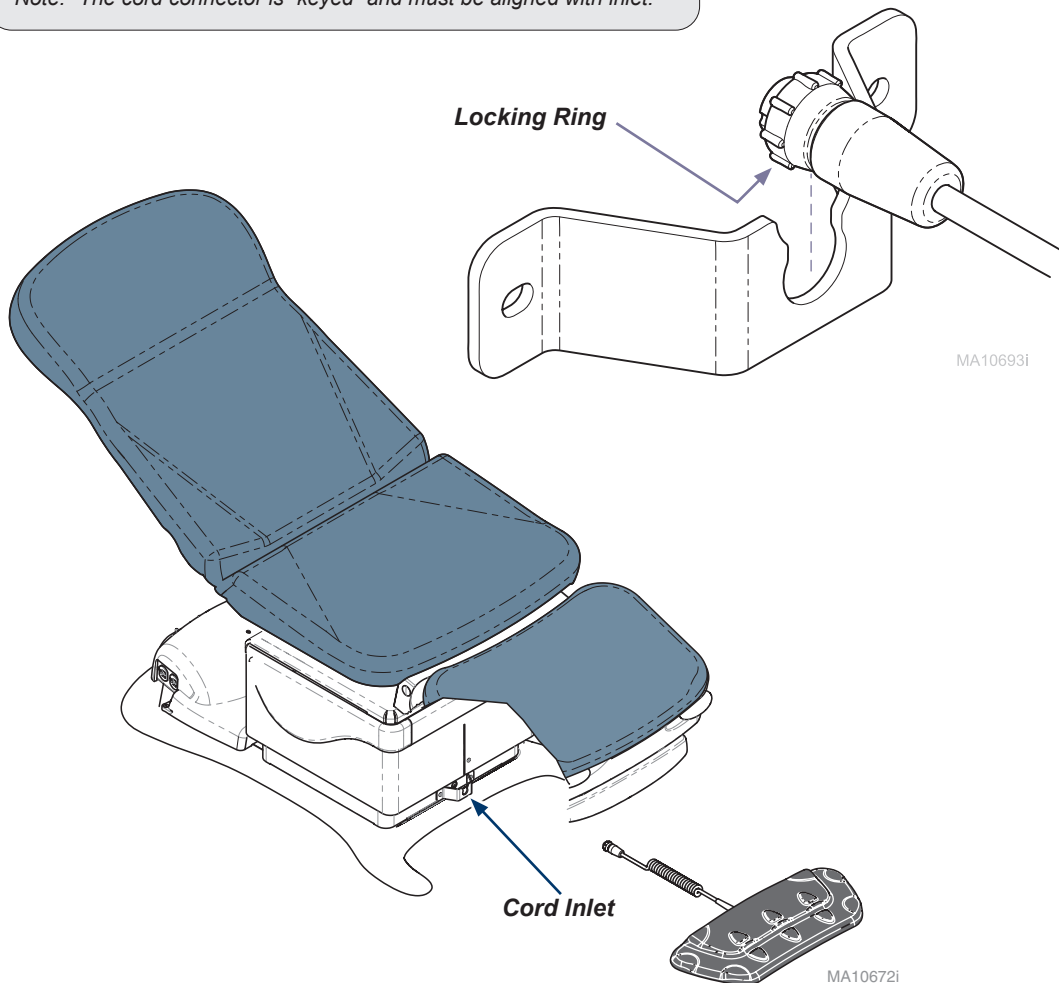

Touch Pad Location

Chair functions can be activated from either of the touch pads located on the foot section.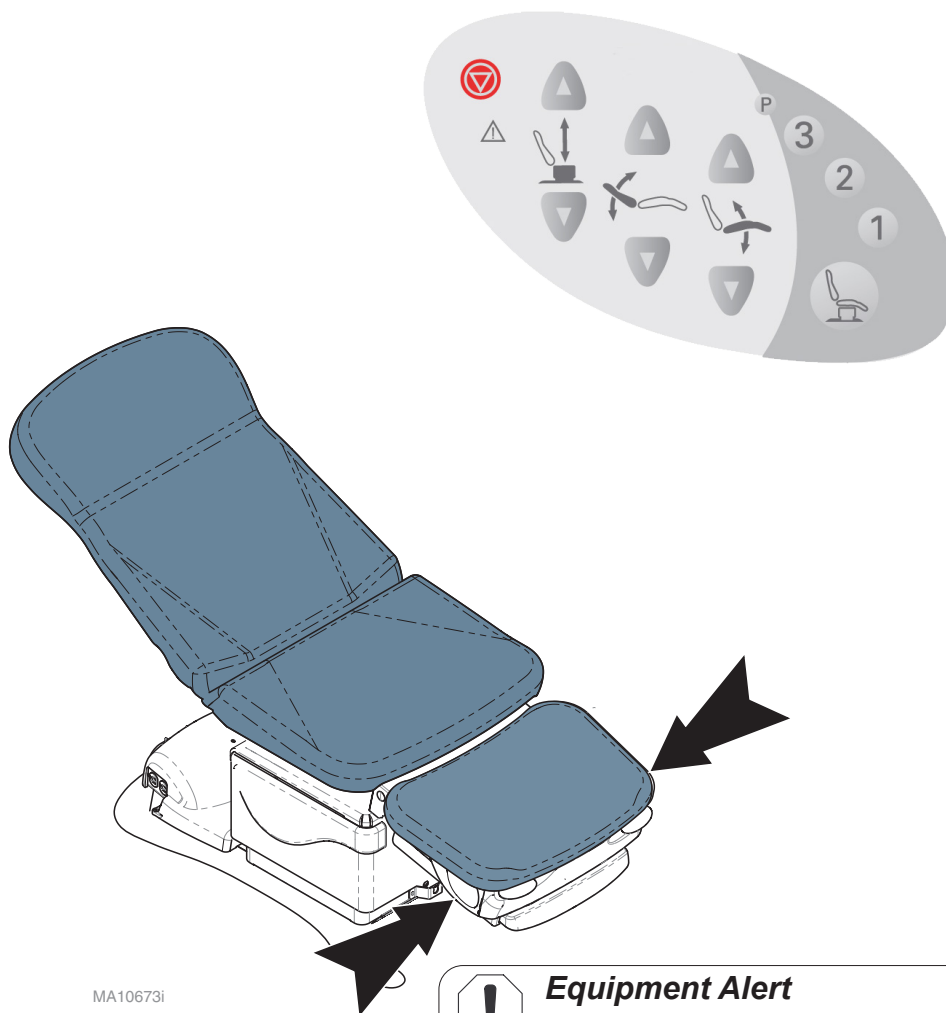

Equipment Alert

The foot section is not a seat. Patients should sit only on the seat section of the chair. Sitting on the foot section may result in damage or inoperable equipment.

Operation


Back / Tilt / Base Operation

To operate any of the chair's powered functions...

Press and hold the Up or Down arrow button for the desired function.


Note: In the event of a malfunction, press the Stop button to terminate movement.

The programmable controls are shown. The applicable symbols used on non-programmable models are the same.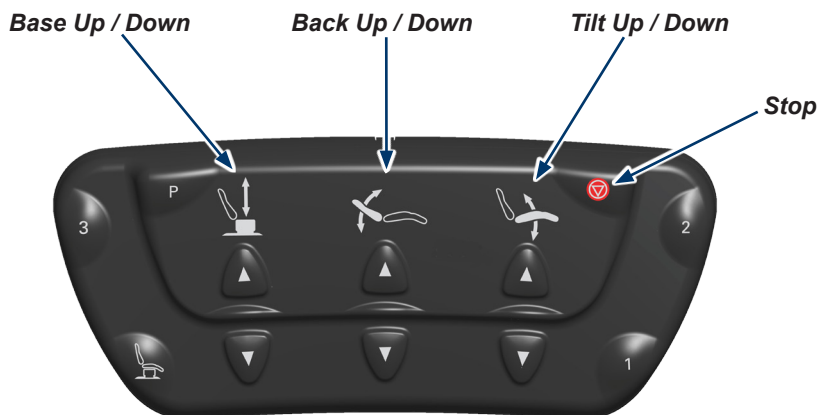

CAUTION

When the Back function is in full down position, **and** the Tilt function is in full up position, there is less than 2 in (5 cm) of clearance between the back section and the floor. Keep all objects and personnel clear to prevent damage / injury.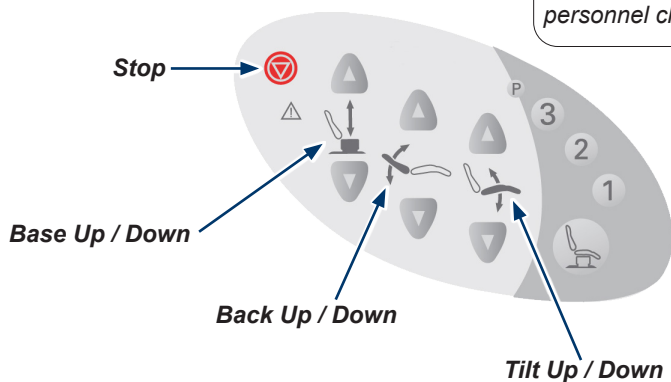

QuickChair Function

The QuickChair function moves the table into the “chair position” with the touch of a single button. The chair functions that are activated vary by model.

Functions activated during QuickChair Operation



Base Down / Back Up / Tilt Down

To activate the QuickChair function...

Press and release the QuickChair button.

Note: In the event of a malfunction, press the Stop button to terminate movement.

The programmable controls are shown. The applicable symbols used on the non-programmable models are the same.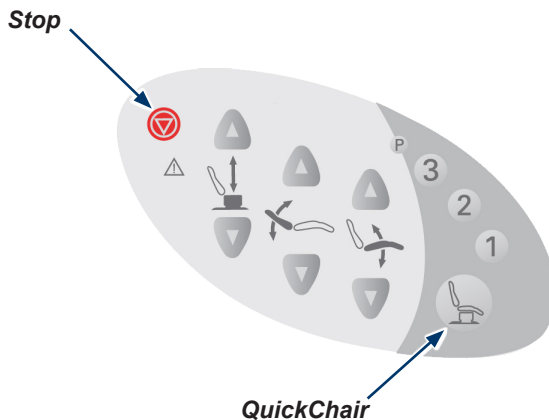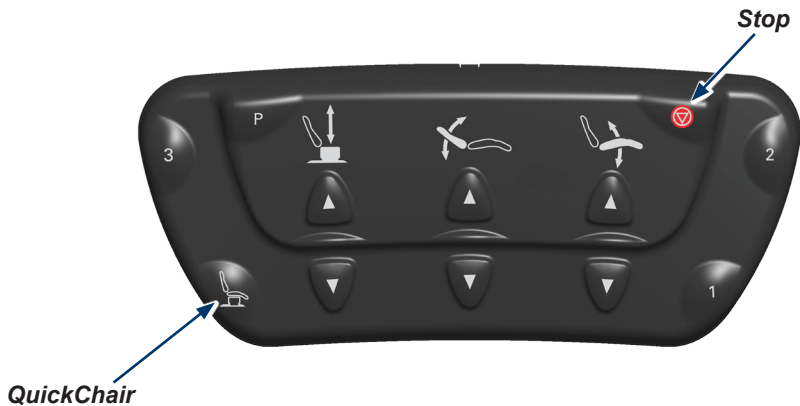

Programming Chair Positions (Programmable models only)

The Programming feature allows you to easily recall frequently used chair positions.

To program a chair position...


- Move the chair to desired position.
- Press the Program button (one “beep”).
- Press the desired Position button (1, 2 or 3).
(three “beeps”)

Tip: You must press the desired Position button within five seconds of pressing the Program button.

To recall a chair position...

Press and release the desired Position button (1, 2 or 3).

Note: In the event of a malfunction, press the Stop button to terminate movement.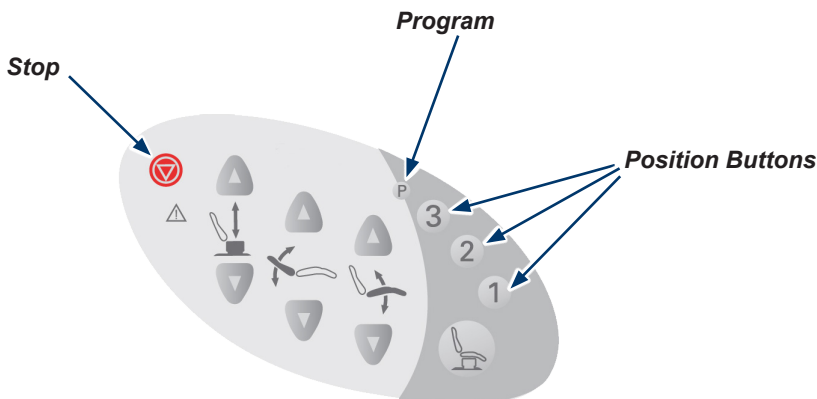

QuickExam Function


(Non-Programmable models only)

The QuickExam function positions the chair for an exam by activating the Base Up and Tilt Up functions simultaneously.

To activate the QuickExam function...


- A) Press and hold the QuickExam button.
- B) Release button when chair reaches desired position.

Note: In the event of a malfunction, press the Stop button to terminate movement.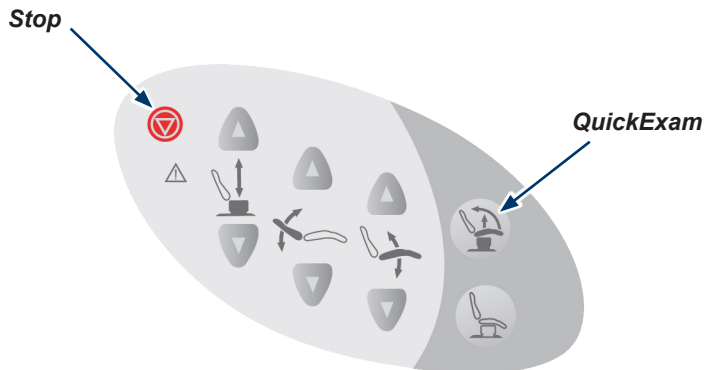

Foot Section / Debris Tray


The foot section can be extended up to 9 in. (23 cm) to accommodate taller patients.
The retractable debris tray is located directly under the foot section.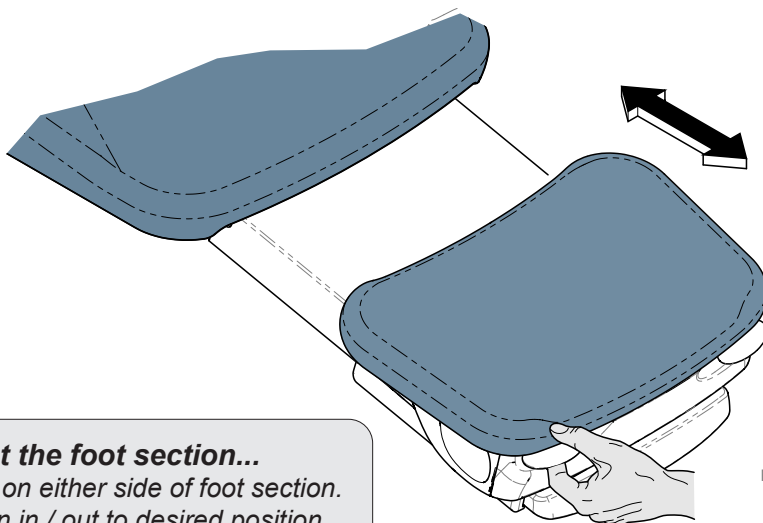

MA10676i

To extend / retract the foot section...

- A) Squeeze handle on either side of foot section.
- B) Move foot section in / out to desired position.
- C) Release handle(s) to lock in position.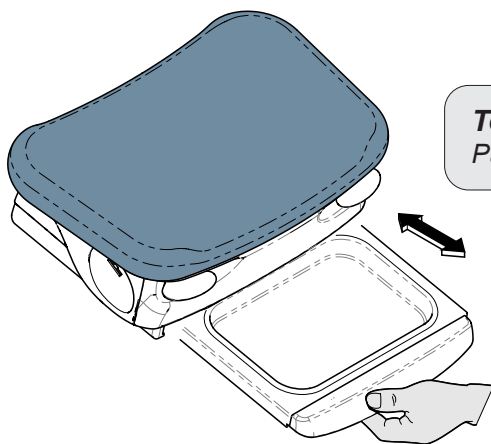

MA10677i

To access the debris tray...


Pull tray outward.

Chair Arms

The chair arms can be rotated 90° to allow the patient to mount / dismount the chair.

To rotate the chair arm(s)...

- A) Press the pivot lever.***
- B) Rotate the chair arm to desired position.***
- C) Release pivot lever to lock in position.***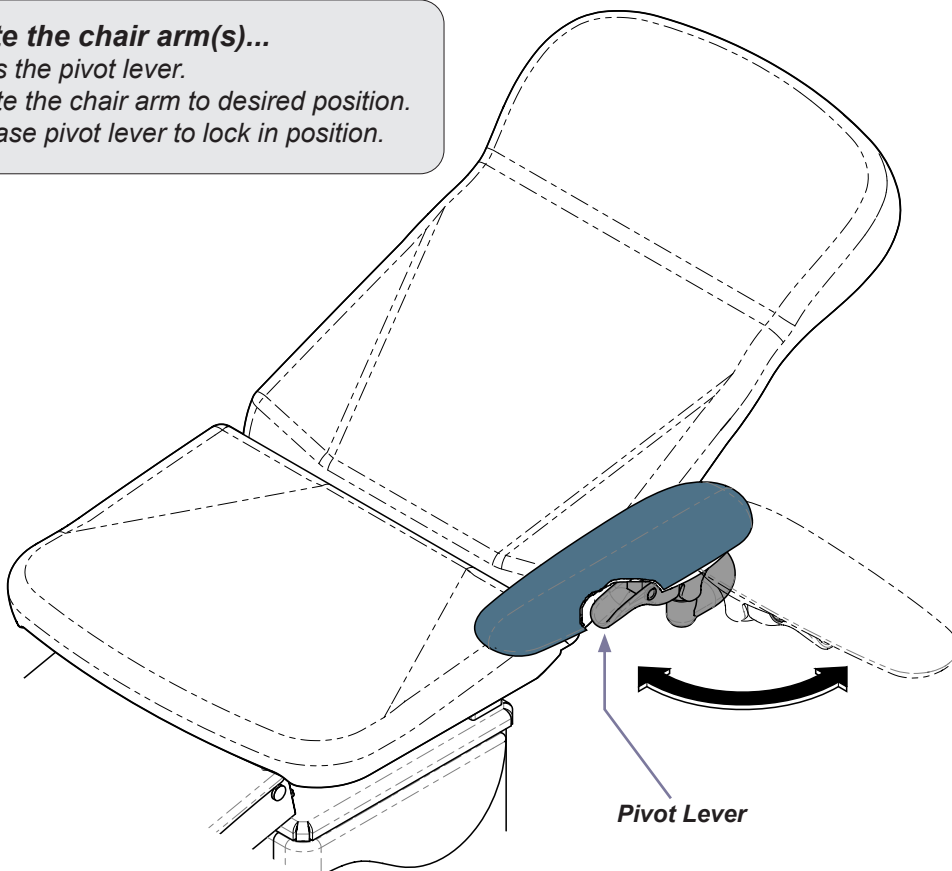

MA10678i

Rotational Base Operation

(Rotational models only)


To rotate the chair...

- A) Press & release the brake pedal.
- B) Rotate the chair to desired position.

To lock in position...

Press & release the brake pedal.

Note: The base will automatically lock in position after two minutes.

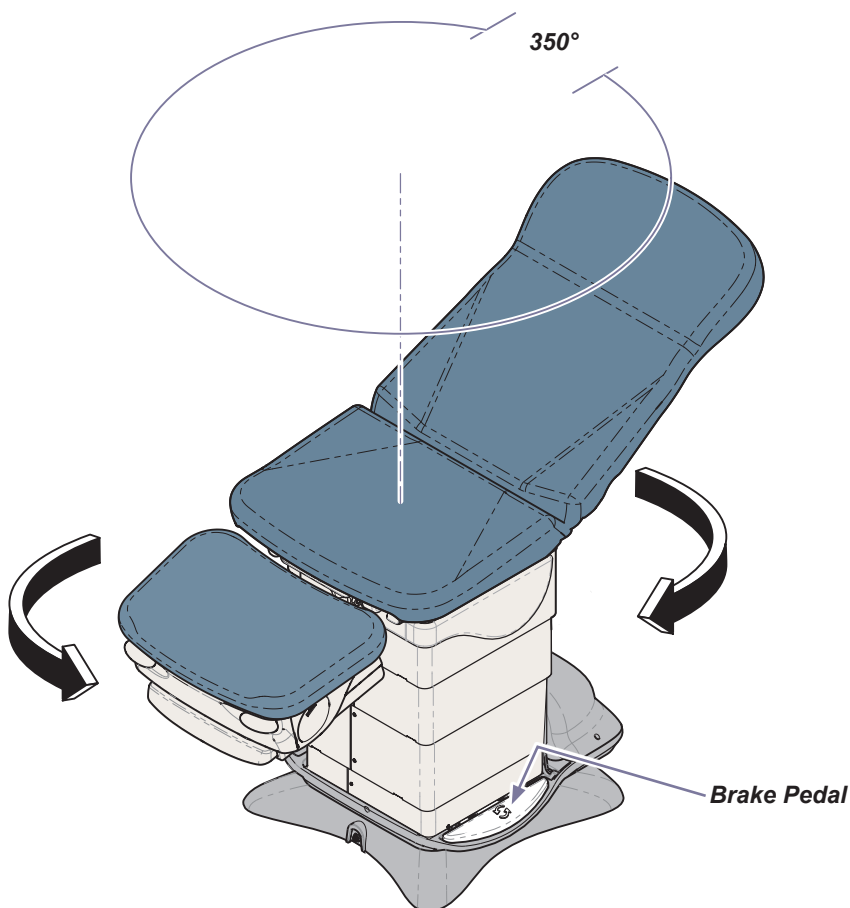MA10679i

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

1.800.MIDMARK (1.800.643.6275)

Monday-Thursday 8:00 AM until 6:00 PM Friday 8:00 AM until 5:00 PM (ET)

Cleaning

Upholstery

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner buildup.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue buildup, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

- Power cord(s) should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

Periodically lubricate the back hinge to maintain quiet, smooth, operation (use light machine oil).

Have a qualified service technician inspect your equipment every twelve months.

Troubleshooting

Powered Function(s) Not Working Properly

Function	Problem	Correction
Back Up	No movement (“beeps”)	Contact your authorized service provider.
Back Down	No movement (“beeps”)	* Activate the Base Up, and Tilt Down functions, then retry Back Down. If problem persists, contact your authorized service provider.
Tilt Up	No movement (“beeps”)	* Activate the Base Up, and Back Up functions, then retry Tilt Up. If problem persists, contact your authorized service provider.
Tilt Down	No movement (“beeps”)	* Remove any obstructions under the foot section. Activate the Tilt Up, and Base Up functions, then retry Tilt Down. If problem persists, contact your authorized service provider.
Base Up	No movement (“beeps”)	Contact your authorized service provider.
Base Down	No movement (“beeps”)	* Remove any obstructions under the foot section. Activate the Tilt Up, and Back Up functions, then retry Base Down. If problem persists, contact your authorized service provider.

* **Crash Avoidance System**

This chair is equipped with Midmark’s Crash Avoidance System. The system is designed to prevent injury and equipment damage by disabling certain functions if the chair is in a potential “crash position”.

The chair is considered to be in a potential “crash position” when the system’s sensors and switches indicate that the foot section is close to colliding with the floor, or another foreign object. On 647 models, the system also monitors the position of the back section to prevent it from colliding with the floor.

The Crash Avoidance System may affect any combination of the functions listed:

Tilt Up / Down, Back Down, Base Down

Specifications

Range of Motion / Specifications Table

Back Function

Full Down: $0^{\circ} \pm 1/2^{\circ}$

Full Up: $80^{\circ} \pm 3^{\circ}$

Base Function

Standard Base

Full Down: 19 in. (48 cm)

Full Up: 40 in. (101 cm)

Rotational Base

Full Down: 22.5 in. (57 cm)


Full Up: 43.5 in. (110 cm)

Tilt Function

Full Down: $0^{\circ} \pm 1/2^{\circ}$

Full Up: $30^{\circ} \pm 2^{\circ}$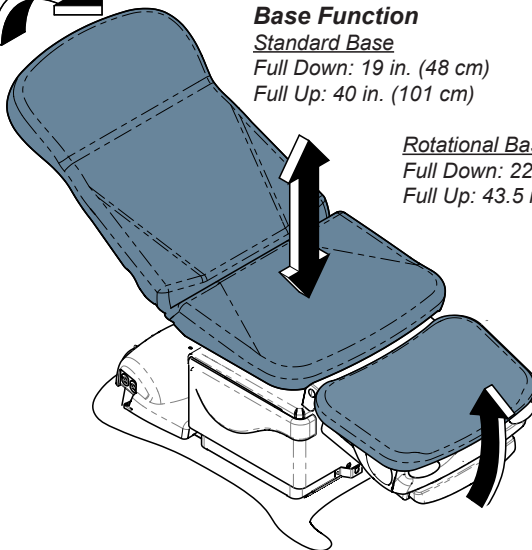

Patient Weight (max):	450 lbs (204 kg)
Weight of Chair:	w/ Standard Base: 421 lbs (191 kg) w/ Rotational Base: 506 lbs (230 kg)
Power Cord Length:	8 ft (244 cm)
Fuses (647 only):	6.3A, 250V, Type "T", 5 x 20 mm
Foot Control Voltage:	10 VAC, SELV (Safety Extra Low Voltage)
Chair Receptacle Maximum Load:	115 VAC, 3 amps, 50 / 60 Hz
Duty Cycle (Motor Run Time):	Intermittent Operation (30 seconds ON - 5 minutes OFF)
Protection against ingress of fluids:	Ordinary Equipment (IPX0) <i>Foot control <u>only</u>:</i> IPX1
Classifications:	Class 1, Type B, Applied Part
Electrical Requirements:	See model identification chart
Regulatory Compliance:	See model identification chart


WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.


Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Dimensions


Upholstered Top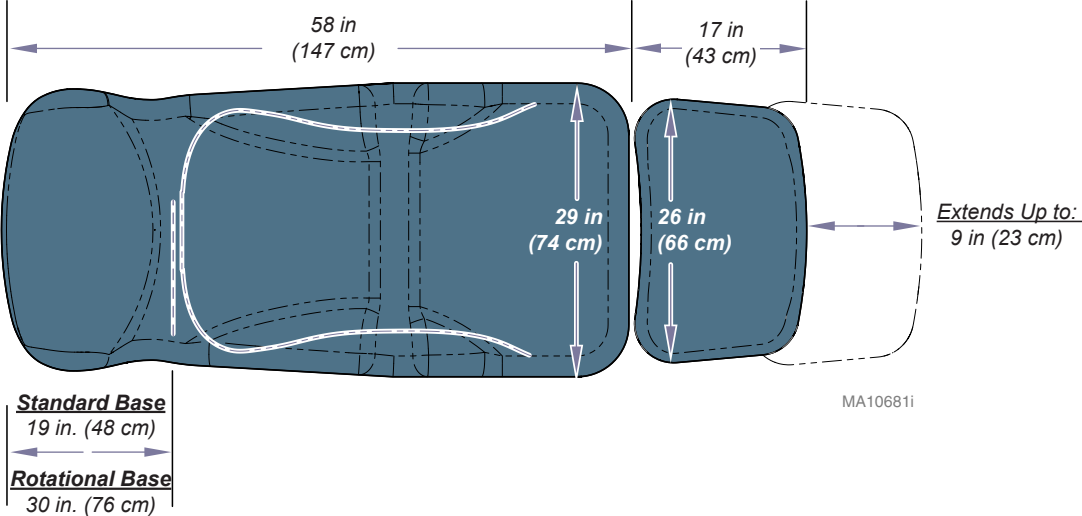

Standard Base Casting

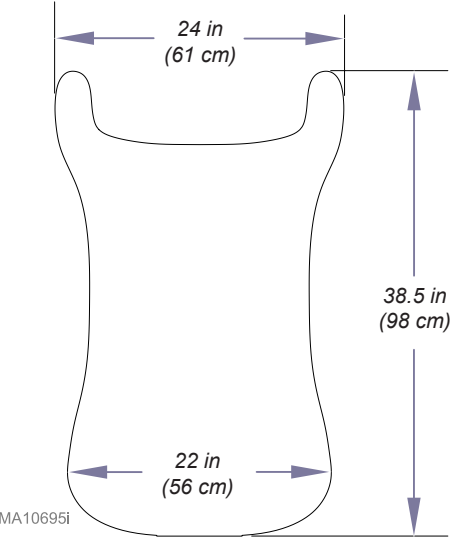Rotational Base Casting

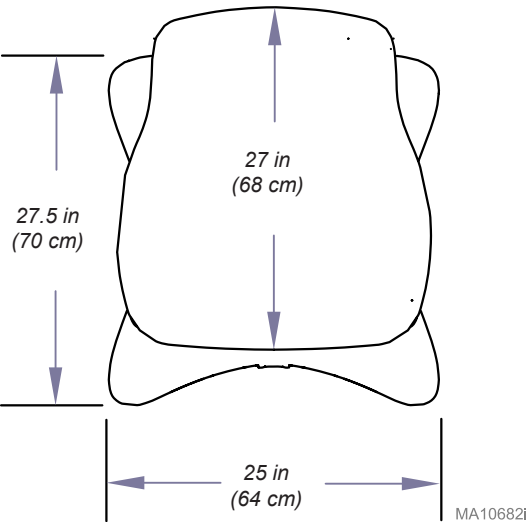Model Identification / Compliance Chart

Model	Description	Complies To:			Electrical Ratings:		
		UL 60601-1	CAN / CSA 22.2, #601.1-M90	EN 60601-1-2 (EMC)	VAC +/- 10%	Amps	Cycles (Hz)
647-001	Three-Function Chair (Base, Back & Tilt) Non-Programmable w/Receptacles	•		•	115	10.5	60
647-002	Three-Function Chair (Base, Back & Tilt) Programmable w/Receptacles	•		•	115	10.5	60
647-003	Three-Function Chair (Base, Back & Tilt) Non-Programmable w/Rotational Base	•	•	•	115	8.5	60
647-004	Three-Function Chair (Base, Back & Tilt) Programmable w/Rotational Base	•	•	•	115	8.5	60
647-010	Three-Function Chair (Base, Back & Tilt) Non-Programmable w/Receptacles	•	•	•	115	10.5	60

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Midmark Corporation