



# Barrier-Free® Exam Chair Heated Upholstery Jumper Kit Instructions


**Applies to Models:**  
Midmark 626 (-002 + -004)

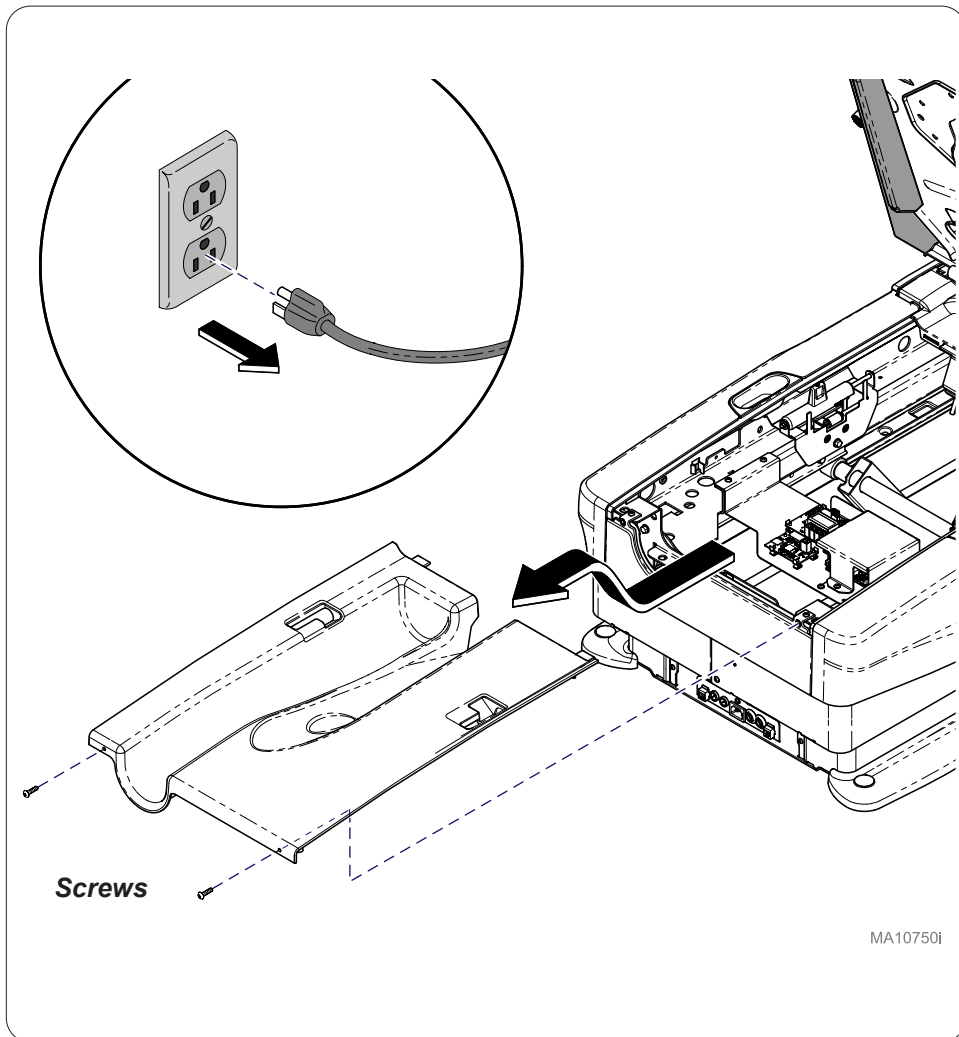


**Step 1:**  
Unplug chair. Remove screws (2) and top cover.

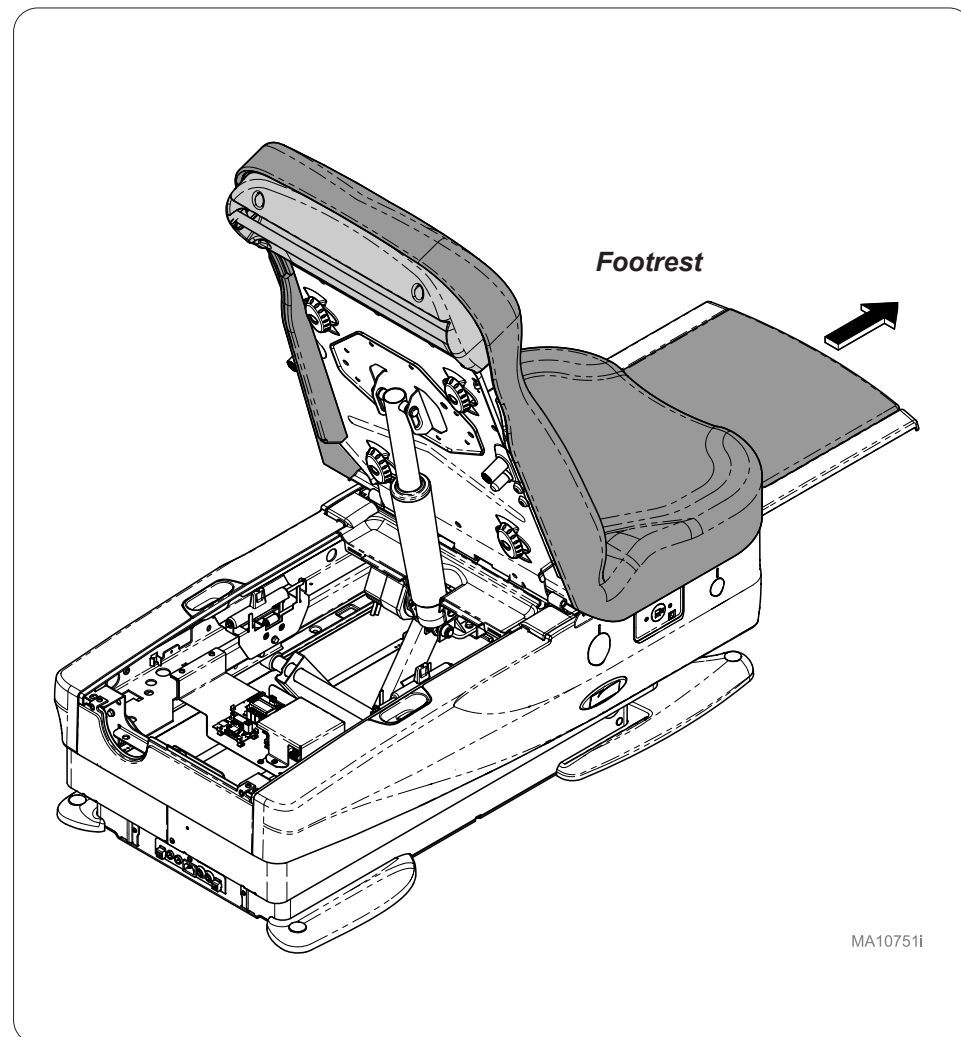


**Step 2:**  
Slide footrest out to allow access to remove cladding on patients right.

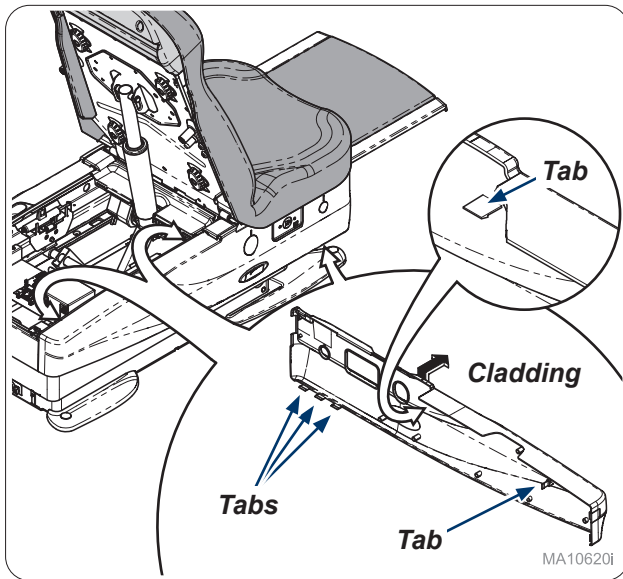


**Step 3:**  
Release tabs on cladding and remove by lifting up and out.

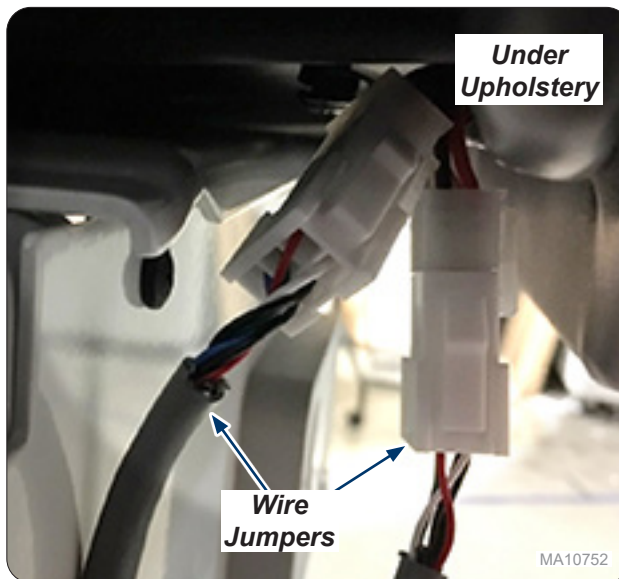


**Step 4:**  
Connect wire jumpers cables to upholstery wires under upholstery.

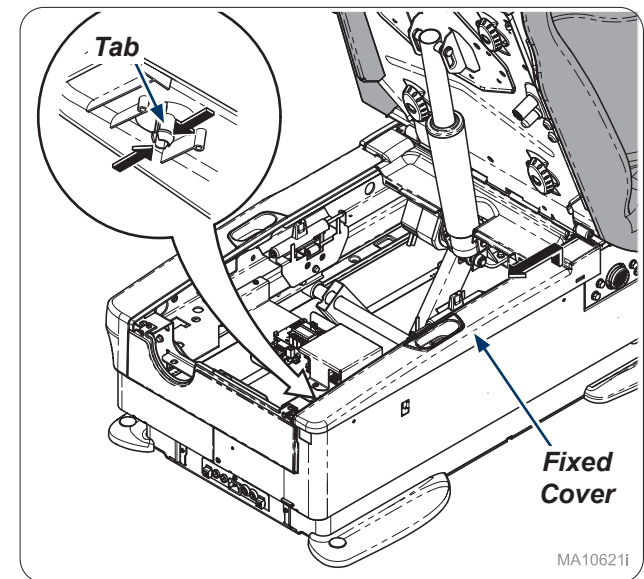


**Step 5:**  
Release patient right tab on fixed cover and move up and towards rear of chair.

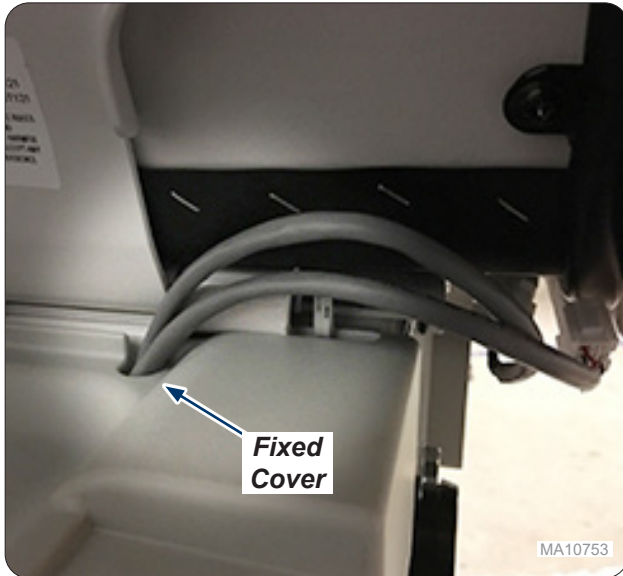

**Step 6:**  
Install wire harnesses through fixed cover.

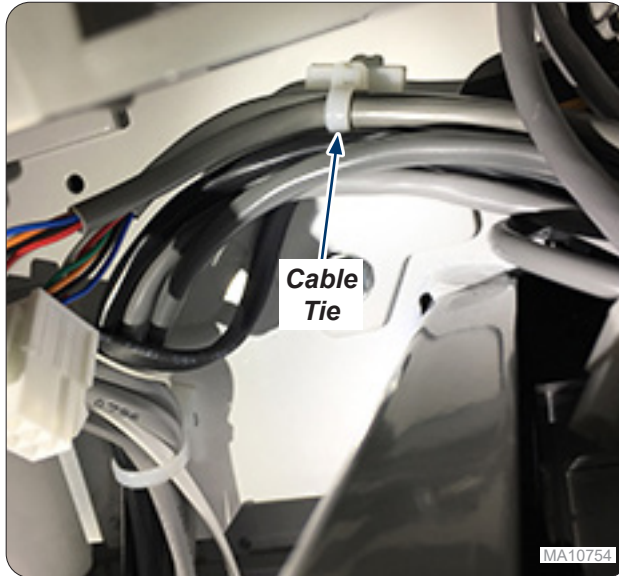

**Step 7:**  
Remove cable tie to make connections for Wire Jumper wires to PC board wires.

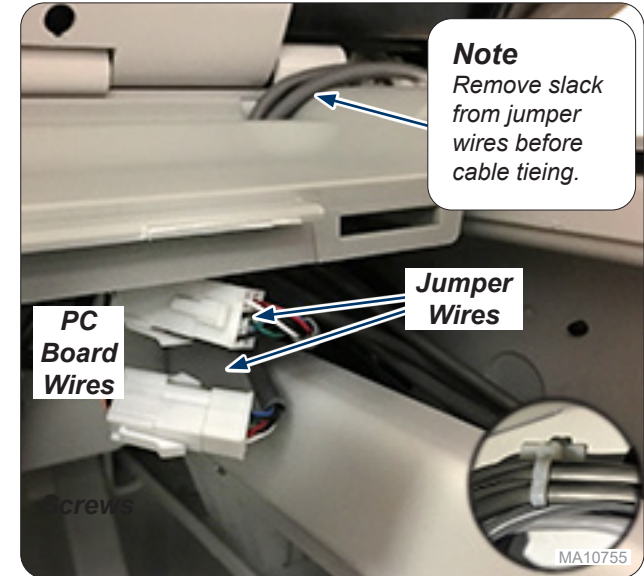

**Step 8:**  
Plug jumper wires to PC board wires and reattach all four wires with provided cable tie.