Intended Use:

To mount 253 exam light to Ritter / Midmark chairs.

Restrictions:

Cannot be used on the same side of the chair as the following:

On models 204/604

9A314, 9A627 Chair Arm

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On models 625 and 222/223

9A561 Patient Support Rails (w/ 253-012). Use 253-013.

Model: 604

Cladding Removal / Installation

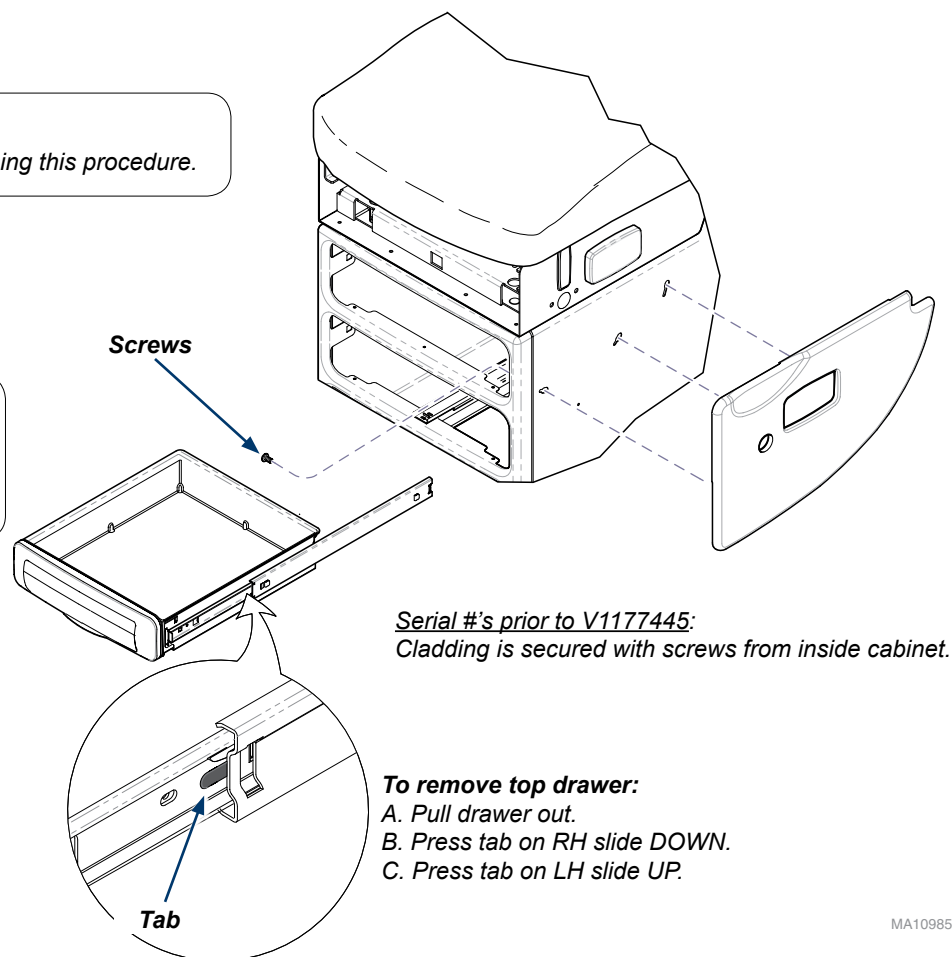
Note

Remove all accessories (ex. chair arms) before performing this procedure.

Step 1: Remove side cladding.

Note

The cladding on 604 models with serial numbers V1177445 thru present is magnetic. It is **not** necessary to remove drawers / screws.

MA10985I

Models: 224 / 225

Cladding Removal / Installation

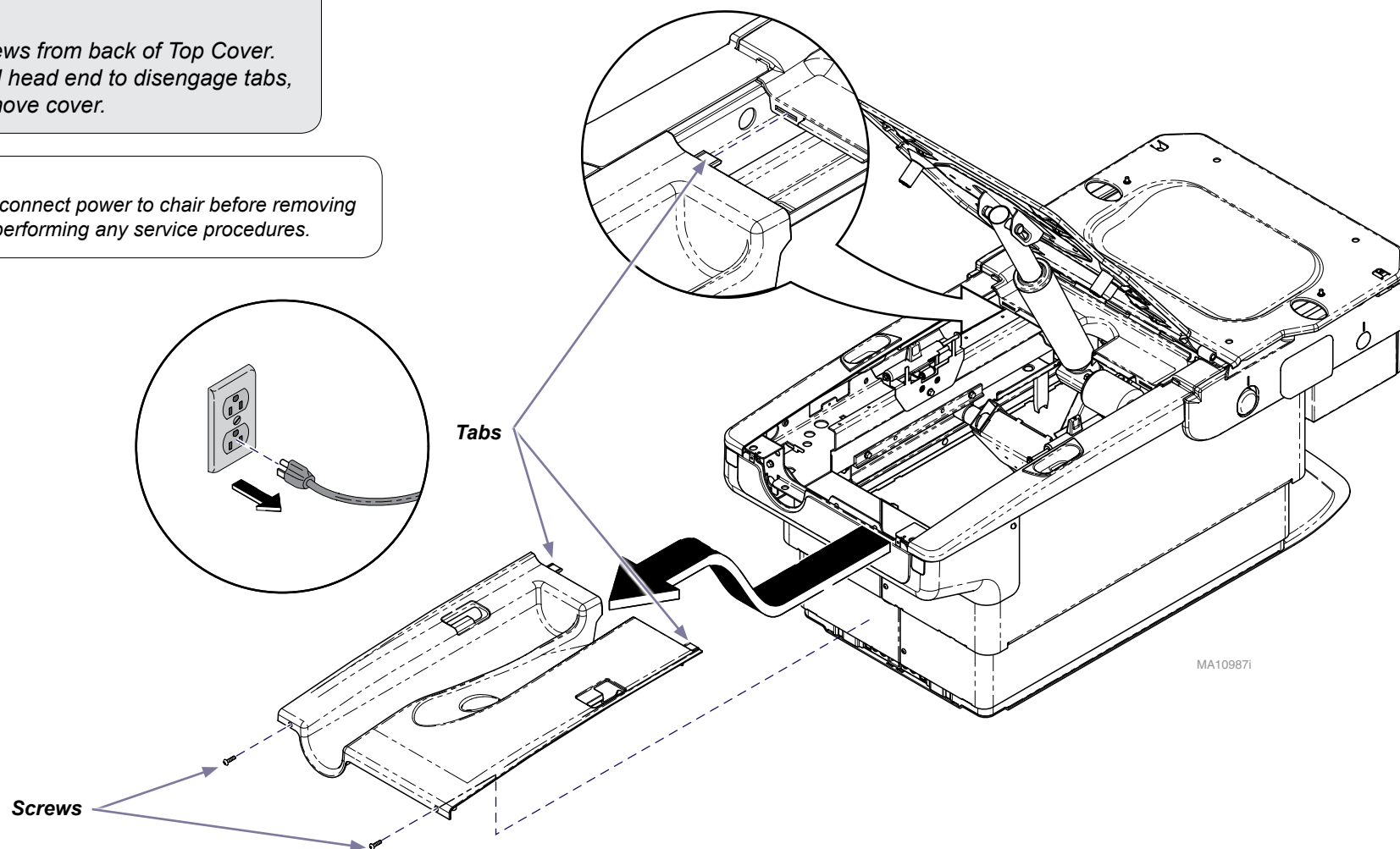
Step 1: Remove top cover

- A) Raise chair back.
- B) Unplug chair.
- C) Remove two screws from back of Top Cover.
- D) Pull cover toward head end to disengage tabs, then lift up to remove cover.

Caution

Always disconnect power to chair before removing covers or performing any service procedures.


Cladding Removal / Installation

Note

Remove all accessories (ex. chair arms) before performing this procedure.

Step 2: Remove side cladding.

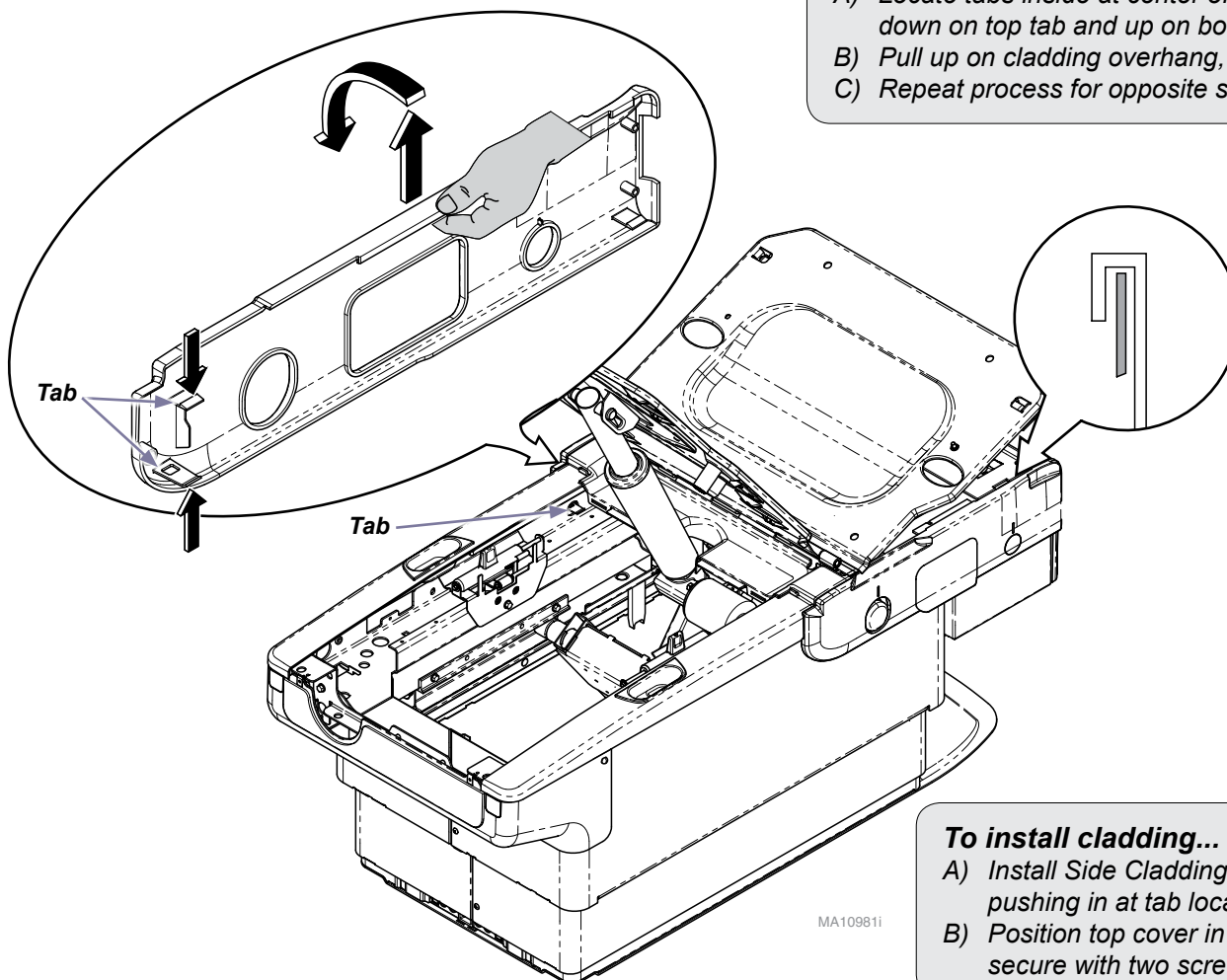- A) Locate tabs inside at center of chair (foot rest must be extended) push down on top tab and up on bottom tab, pull cladding away from chair.
- B) Pull up on cladding overhang, and remove.
- C) Repeat process for opposite side.

To install cladding...

- A) Install Side Cladding by hooking top edge onto chair, then pushing in at tab locations until they engage the chair frame.
- B) Position top cover in place while engaging tabs into chair, secure with two screws.

Model: 626

Cladding Removal / Installation

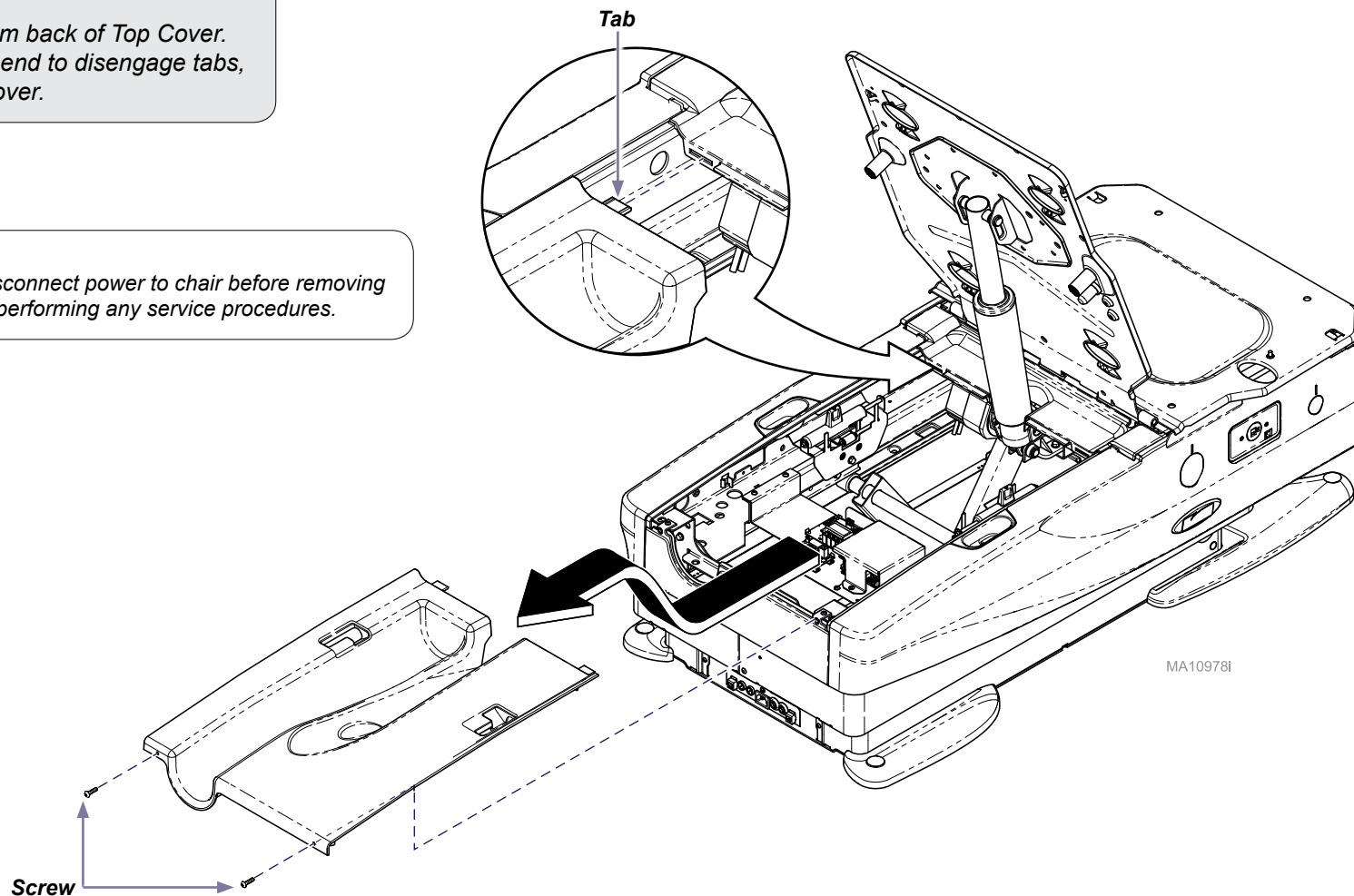
Step 1: Remove top cover

- A) Raise chair back.
- B) Unplug chair.
- C) Remove two screws from back of Top Cover.
- D) Pull cover toward head end to disengage tabs, then lift up to remove cover.

Caution

Always disconnect power to chair before removing covers or performing any service procedures.

Remove all accessories (ex. chair arms) before performing this procedure.

- A) Locate tab inside head end of chair. Push tab toward foot end of chair. Pull cladding away from chair.
- B) Pull down on three tabs located at the bottom foot end of the cladding and pull cladding away from chair.
- C) Locate tab inside at center of chair (footrest must be extended) push tab up and pull cladding away from chair.
- D) Push up on Cladding, tip up and remove.
- E) Repeat process for opposite side.

- Install Side Cladding by hooking top edge onto chair, then pushing in at tab locations until they engage the chair frame.
- Position top cover in place while engaging tabs into chair, secure with two screws.

Model: 627

Cladding Removal / Installation

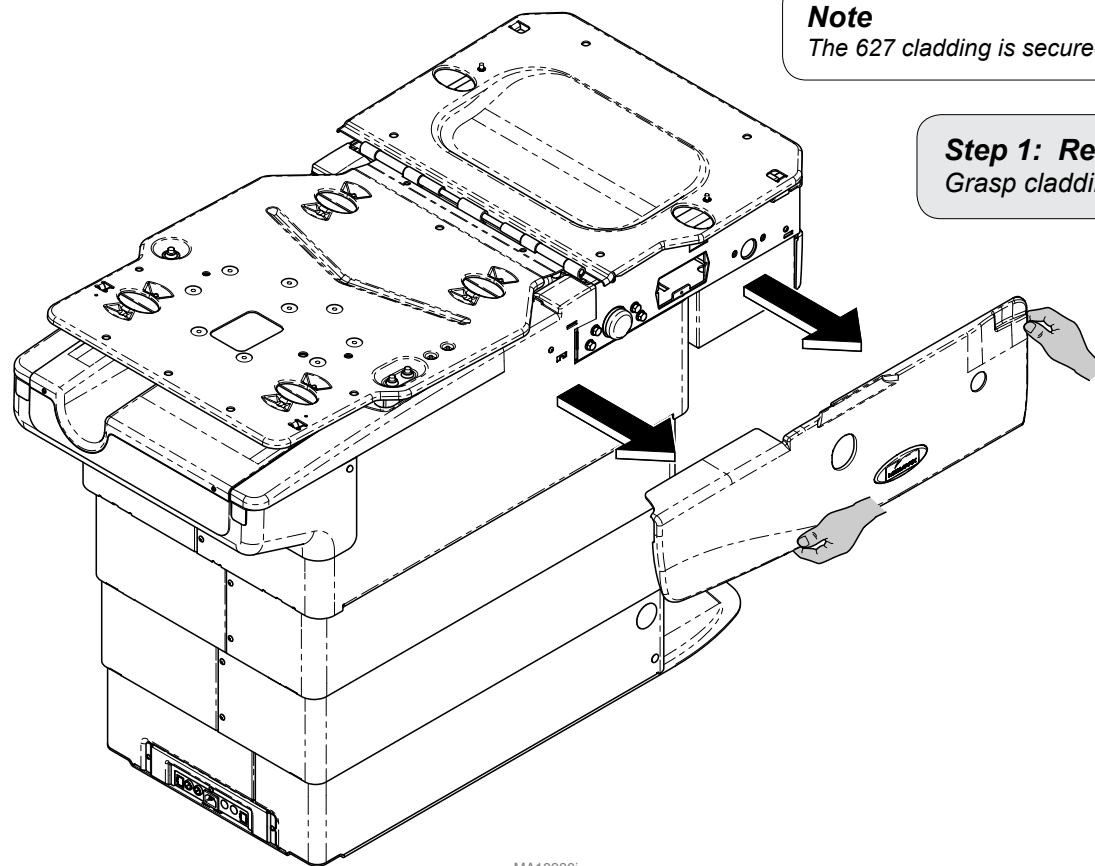
Note

Remove all accessories (ex. chair arms) before performing this procedure.

Note

The 627 cladding is secured with magnets only.

Step 1: Remove side cladding.
Grasp cladding firmly and pull outward.

MA10980i

Models: 622 / 623

Cladding Removal / Installation

Note

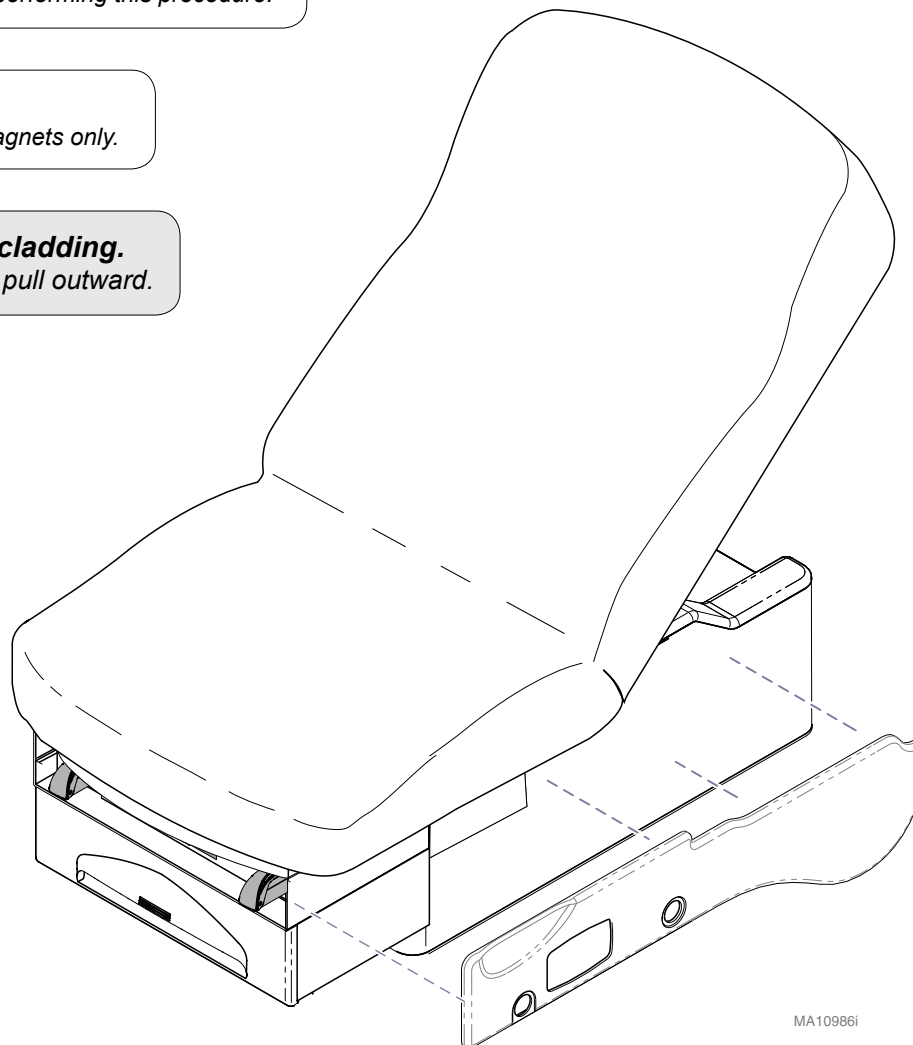
Remove all accessories (ex. chair arms) before performing this procedure.

Note

The 622 / 623 cladding is secured with magnets only.

Step 1: Remove side cladding.

Grasp cladding firmly and pull outward.


MA10986i

Model: 625

Cladding Removal / Installation


Note

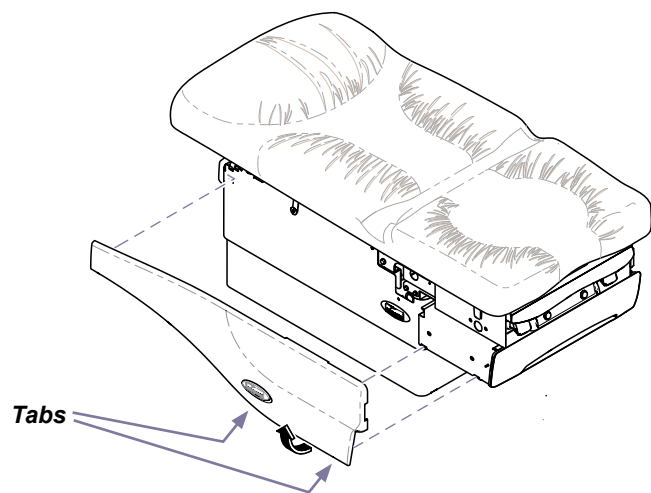Remove all accessories (ex. chair arms) before performing this procedure.


To remove cladding...

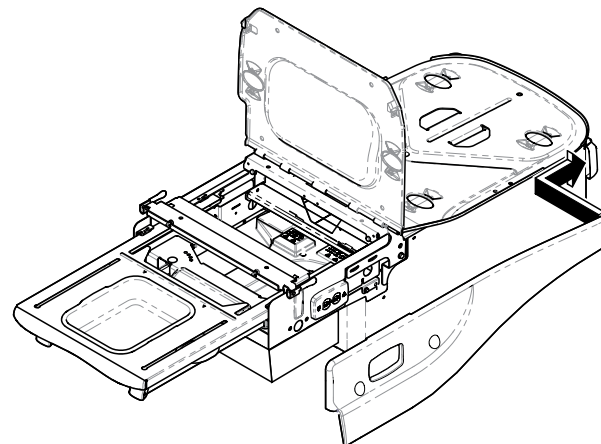- A) Press two tabs.
- B) Pull bottom of cladding away from chair.
- C) Lift cladding up to remove.

To install cladding...

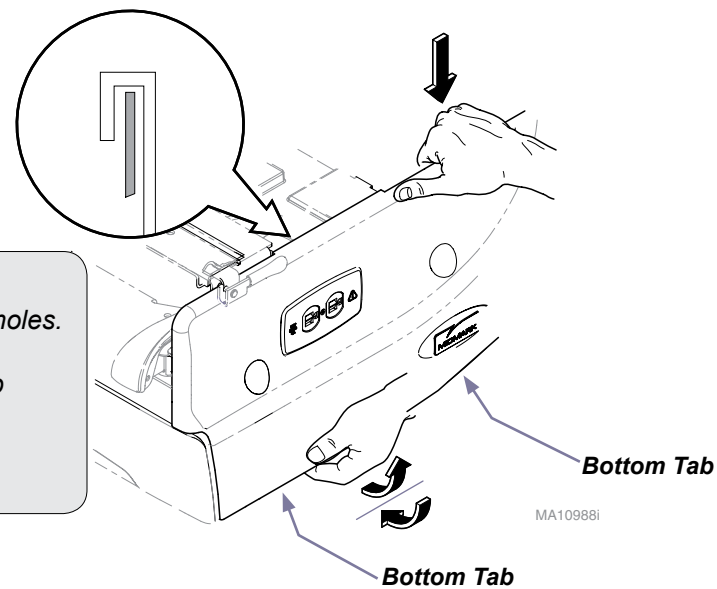- A) Insert tabs at head-end of cladding into side cladding holes.
- B) Slide cladding over upperwrap.
- C) Pull bottom of cladding out, then push down on the top to engage the long tab onto top edge of upperwrap.
- D) Push bottom of cladding in, until two bottom tabs snap into place.

Models: 204 / 222 / 223 / 224 / 225 / 604 / 622 / 623 / 625 / 626 / 627

Spline Socket Installation

Caution

Always disconnect power to chair before removing covers or performing any service procedures.

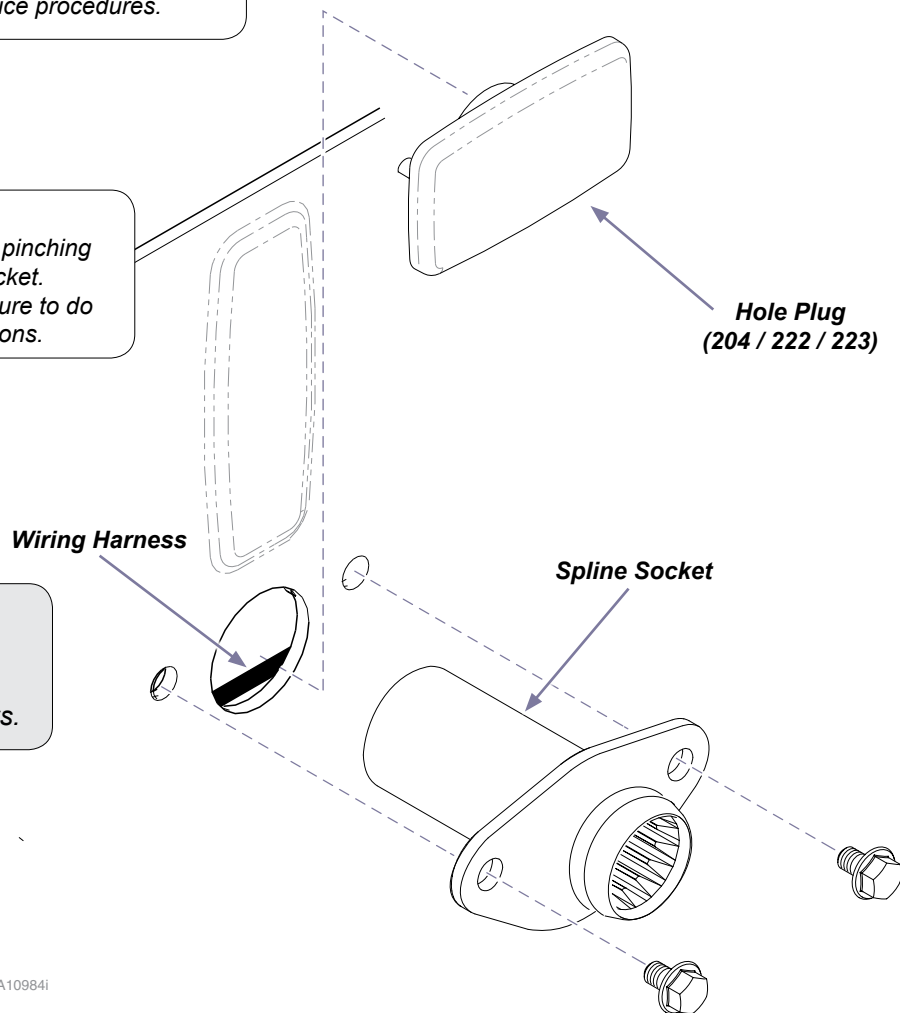
Equipment Alert

Care must be taken to avoid pushing or pinching wiring harness when inserting spline socket. Failure to do so could result in damage to heater switch connections.

Step 1: Install spline weldment.

- A) Remove hole plug.
- B) Insert spline socket into hole.
- C) Secure spline socket with two hex screws.


MA10984i

Models: 204 / 222 / 223 / 224 / 225 / 604 / 622 / 623 / 625 / 626 / 627

Spline Mount / Light Installation

Step 1: Install light & cord restraint.

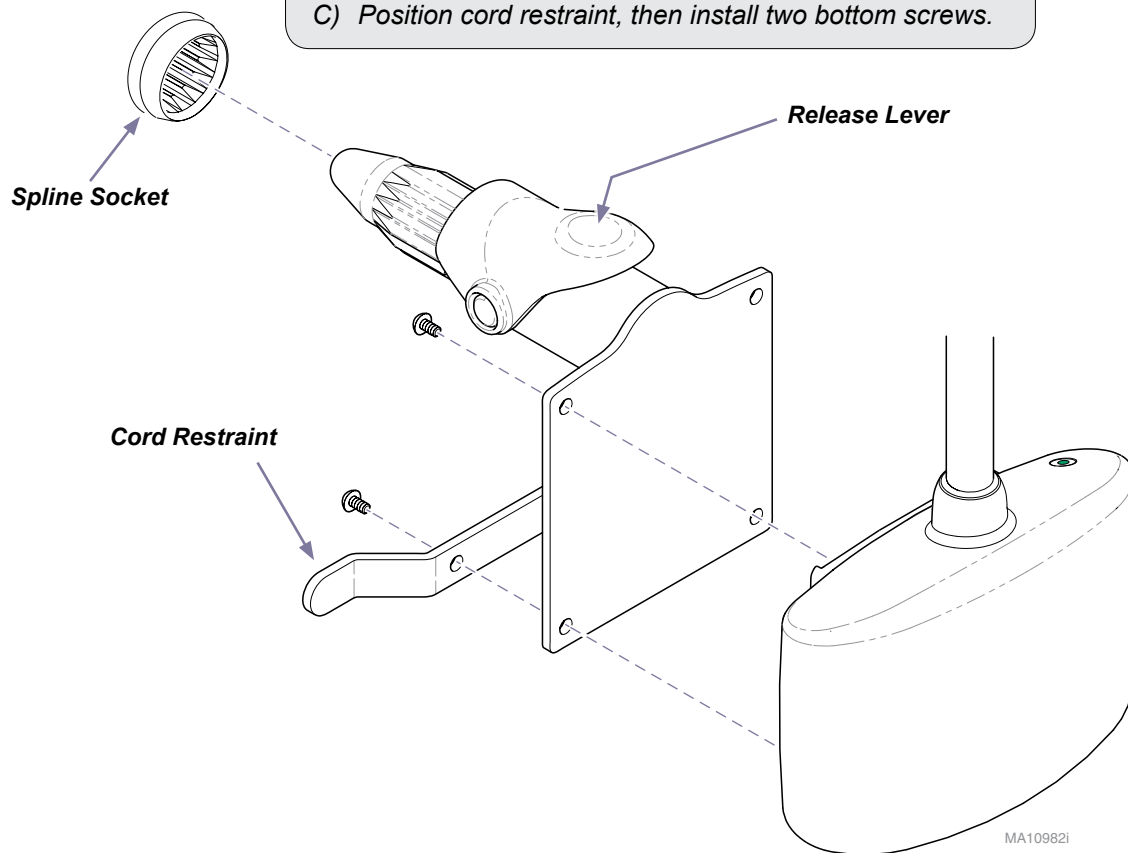- A) Align light with holes in mounting plate.
- B) Secure light by installing two screws thru the top holes.
- C) Position cord restraint, then install two bottom screws.


Step 2: Install spline mount / light assembly into spline socket.

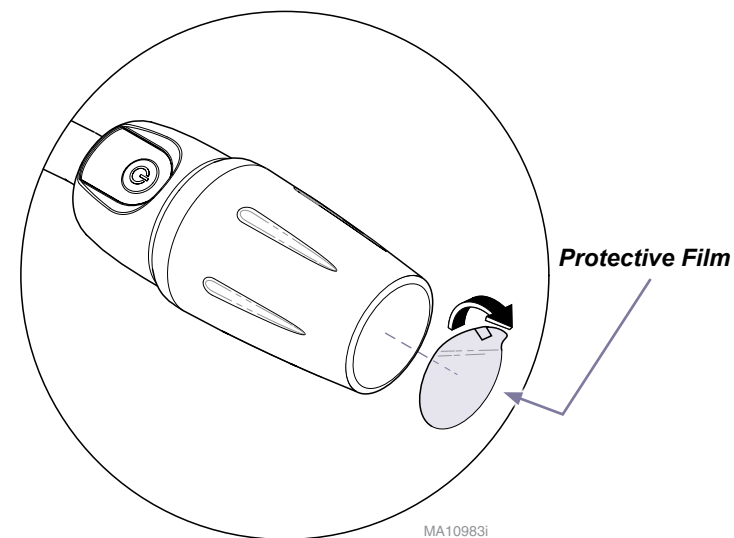- A) Reinstall cladding to chair.
- B) Push bracket assembly into spline until release lever locks in place.

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