Install Assist Tool

(9A680001)

[English](#)

[Español](#)


[Français](#)

For Models:

(222, 223, 622, 623 and 625)
(230, 630, 641 and 647)
(224, 225, 626 and 627)
(244)

Note

For additional languages please visit technicallibrary.midmark.com

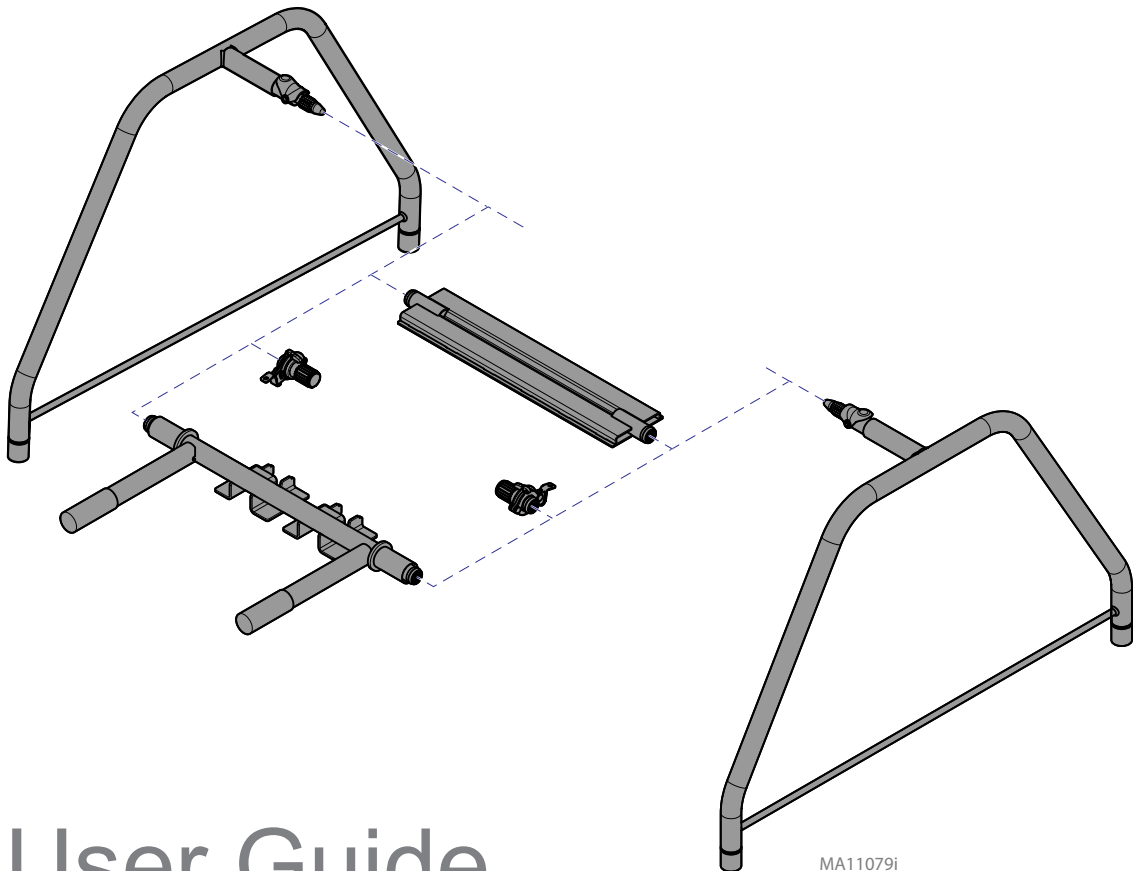User Guide

MA11079i

Style P

Table of Contents

Important Information

<u>Safety Instructions</u>	3
<u>Safety Symbols</u>	3
<u>Shipping Symbols</u>	3
<u>Transportation / Storage Conditions</u>	3
<u>Intended Use</u>	4
<u>Install Assist Tool Warnings and Cautions</u>	4

Operation

<u>Models 224, 225, 626 and 627</u>	5
<u>Models 222, 223, 622, 623 and 625</u>	6
<u>Models 230, 630 (-001 thru -009), 641 and 647</u>	7
<u>Model 244, 630 (-010 thru -013) (-020 thru -023)</u>	8

Maintenance

<u>Parts List</u>	9
-------------------------	---

Warranty Information

<u>Limited Warranty</u>	10
-------------------------------	----

Important Information

These symbols may appear on your equipment and/or in the manuals.

Safety Instructions

The primary concern of Midmark® is that this equipment is operated and maintained with the safety of the user in mind. To assure safe and reliable operation:

- Read and understand this manual before attempting to operate the Install Assist Tool.
- Assure that the appropriate personnel are informed on the contents of this manual.

Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage.

Shipping Symbols

Proper Shipping Orientation

Caution Shipping Damage

Keep Dry

Manufacturer

Transportation / Storage Conditions

Ambient Temperature Range..... -22°F to 140°F (-30°C to +60°C)
Relative Humidity..... 10% to 90% (non-condensing)
Atmospheric Pressure 500 hPa to 1060 hPa

Important Information

Intended Use

The Install Assist Tool is designed to provide a method to raise medical chairs from a shipping pallet or furniture dolly for easier installation.

Install Assist Tool Warnings and Cautions

WARNING

- *Designed to only be used with supported Midmark chairs.*
- *Risk of injury if arms are not fully engaged in the adapter plate prior to use.*
- *Use only on clean level surface.*
- *Do not lean/sit on chair during use.*
- *Do not use if the install assist tool or chair is damaged.*
- *Risk of severe injury or death if the load is suspended over people.*
- *Do no service chair while lifted.*
- *Do not place objects on the chair during use.*
- *Do not use while patient is on chair.*
- *Do not remove, cover, damage, or otherwise render labels unreadable.*
- *Maximum load: 550 pounds.*



Caution

Ensure dolly is centered under the chair.

Operation

Models 224, 225, 626 and 627

Step 1: Slide spline adapters onto Install Assist Tool legs.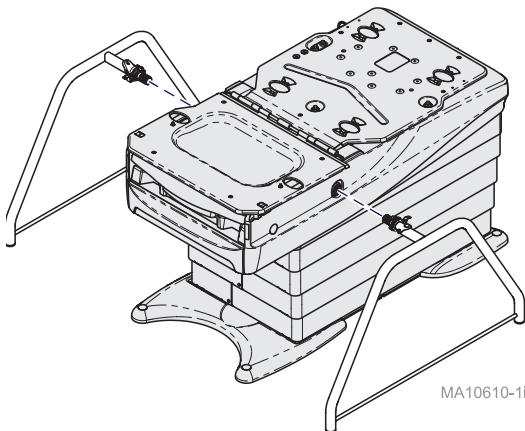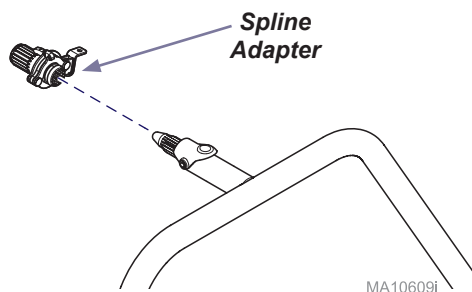


Step 2:

- A) Press Chair Up on hand control to raise chair height.
- B) Insert Install Assist Tool legs into spline receptacles.
- C) Press Chair Down on hand control to lift the chair base to desired height.

To remove Install Assist Tool legs...

- A) Press Chair Up on hand control to lower chair base to floor or dolly.
- B) Lift up on spline adapter release lever.
- C) Slide Install Assist Tool legs from spline receptacles.

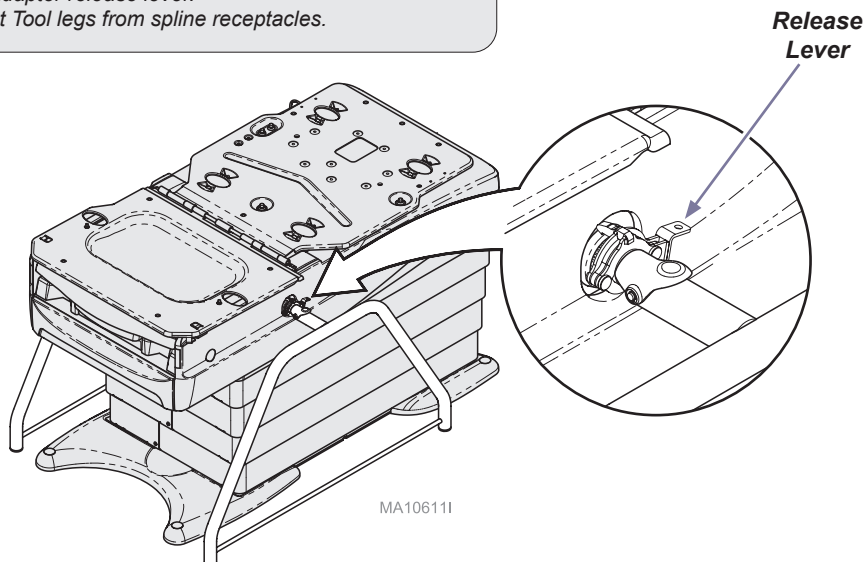
Operation

Models 222, 223, 622, 623 and 625


Equipment Alert

Only attach Install Assist Tool to Cross Member designed for these models. To prevent damage DO NOT attach directly to the chair splines.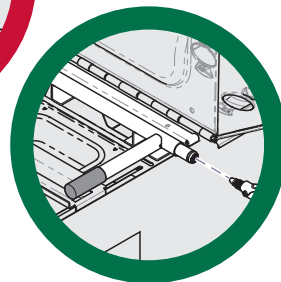


Step 1:

- Lift seat section.
- Hook Exam Chair Cross Member onto hinge lip.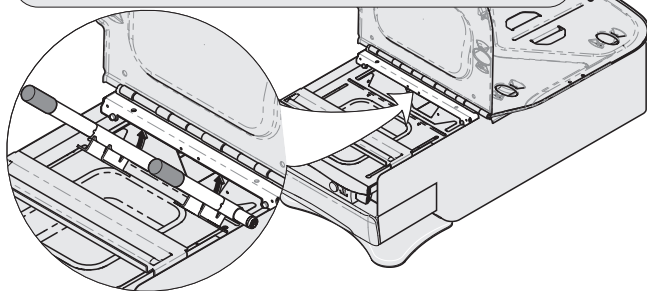

Step 2: Lay Exam Chair Cross Member flat on metal crossbar.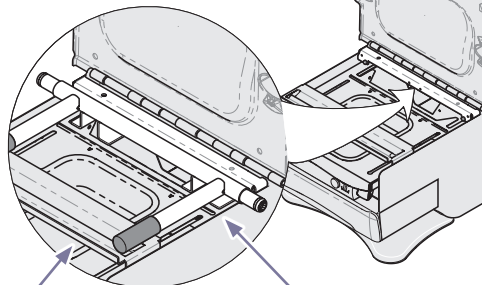


Metal Crossbar


Exam Chair Cross Member

Step 3:

- Press Base Up on hand control to raise chair height.
- Insert Install Assist Tool into spline receptacles.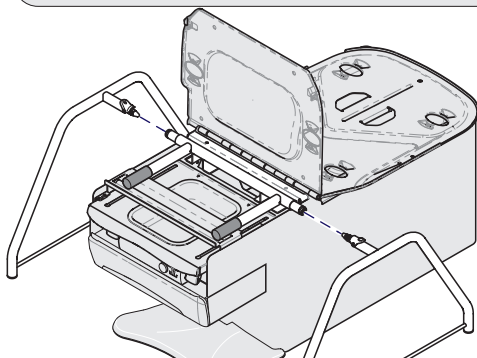

Step 4: Press Base Down on hand control to lift the chair base to desired height.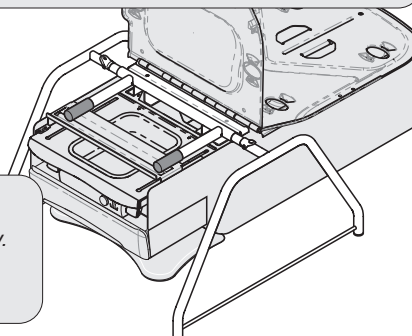

To remove Install Assist Tool legs...

- Press Chair Up on hand control to lower chair base to floor or dolly.
- Lift up on spline adapter release lever.
- Slide Install Assist Tool legs from spline receptacles.




Operation

**Models 230, 630 (-001 thru -009),
641 and 647**

Equipment Alert


Only attach Install Assist Tool to Cross Member designed for these models. To prevent damage **DO NOT** attach directly to the chair splines.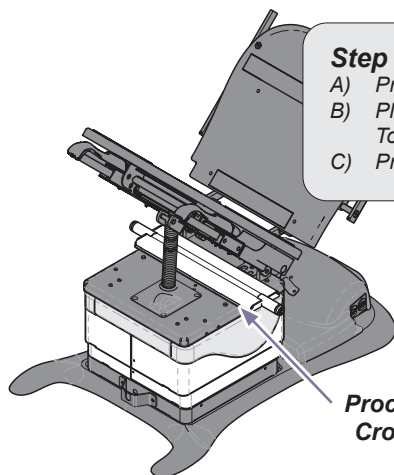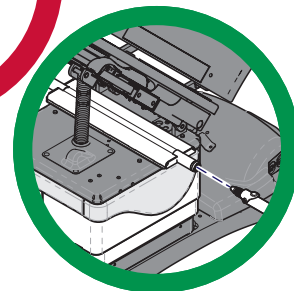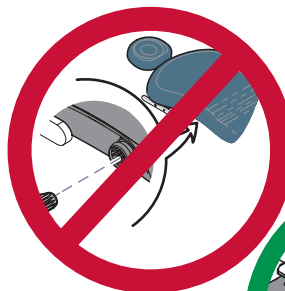

Step 1:


- A) Press Tilt Up on hand control.
- B) Place Procedure Chair Install Assist Tool behind the tilt actuator shaft.
- C) Press Tilt Down on hand control.

**Procedure Chair
Cross Member**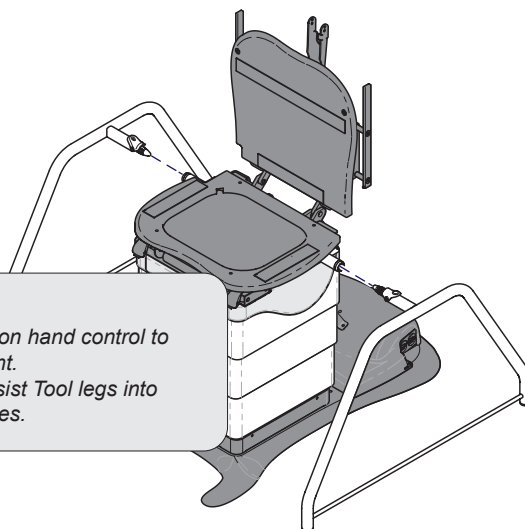

Step 2:

- A) Press Base Up on hand control to raise chair height.
- B) Insert Install Assist Tool legs into spline receptacles.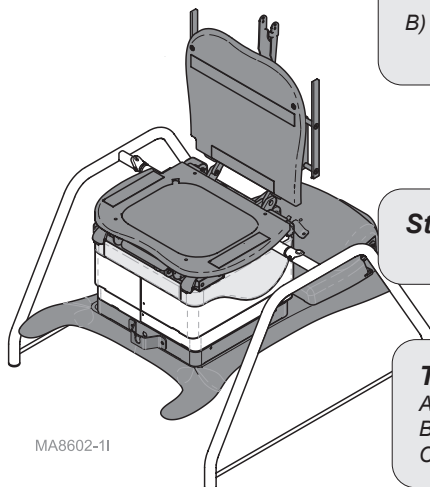

- ### Step 3:
- Press Base Down on hand control to lift the chair base to desired height.

To remove Install Assist Tool legs...

- A) Press Chair Up on hand control to lower chair base to floor or dolly.
- B) Lift up on spline adapter release lever.
- C) Slide Install Assist Tool legs from spline receptacles.

MA8602-11

Operation

Model 244, 630 (-010 thru -013) (-020 thru -023)


Step 1:

- A) Press Chair Up on hand control to raise chair height.
- B) Insert Install Assist Tool legs into spline receptacles.



630

244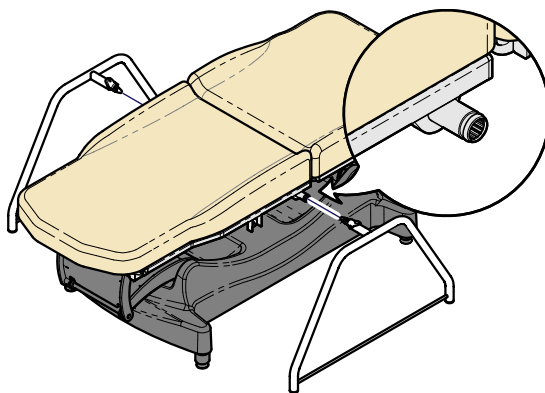

- ### Step 2:
- Press Chair Down on hand control to lift the chair base to desired height.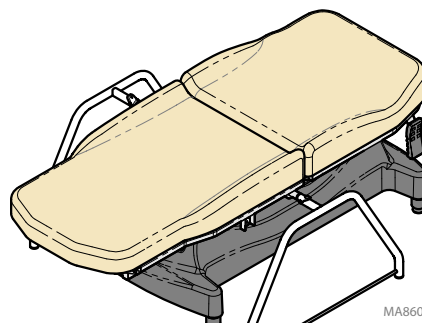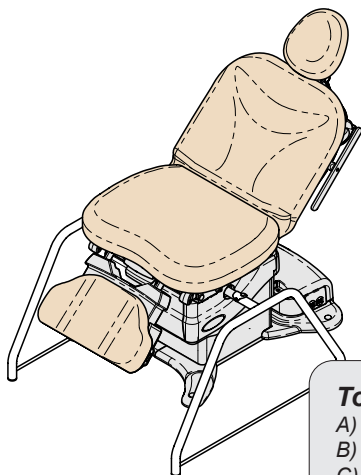


To remove Install Assist Tool legs...

- A) Press Chair Up on hand control to lower chair base to floor or dolly.
- B) Lift up on spline adapter release lever.
- C) Slide Install Assist Tool legs from spline receptacles.

Maintenance

Parts List

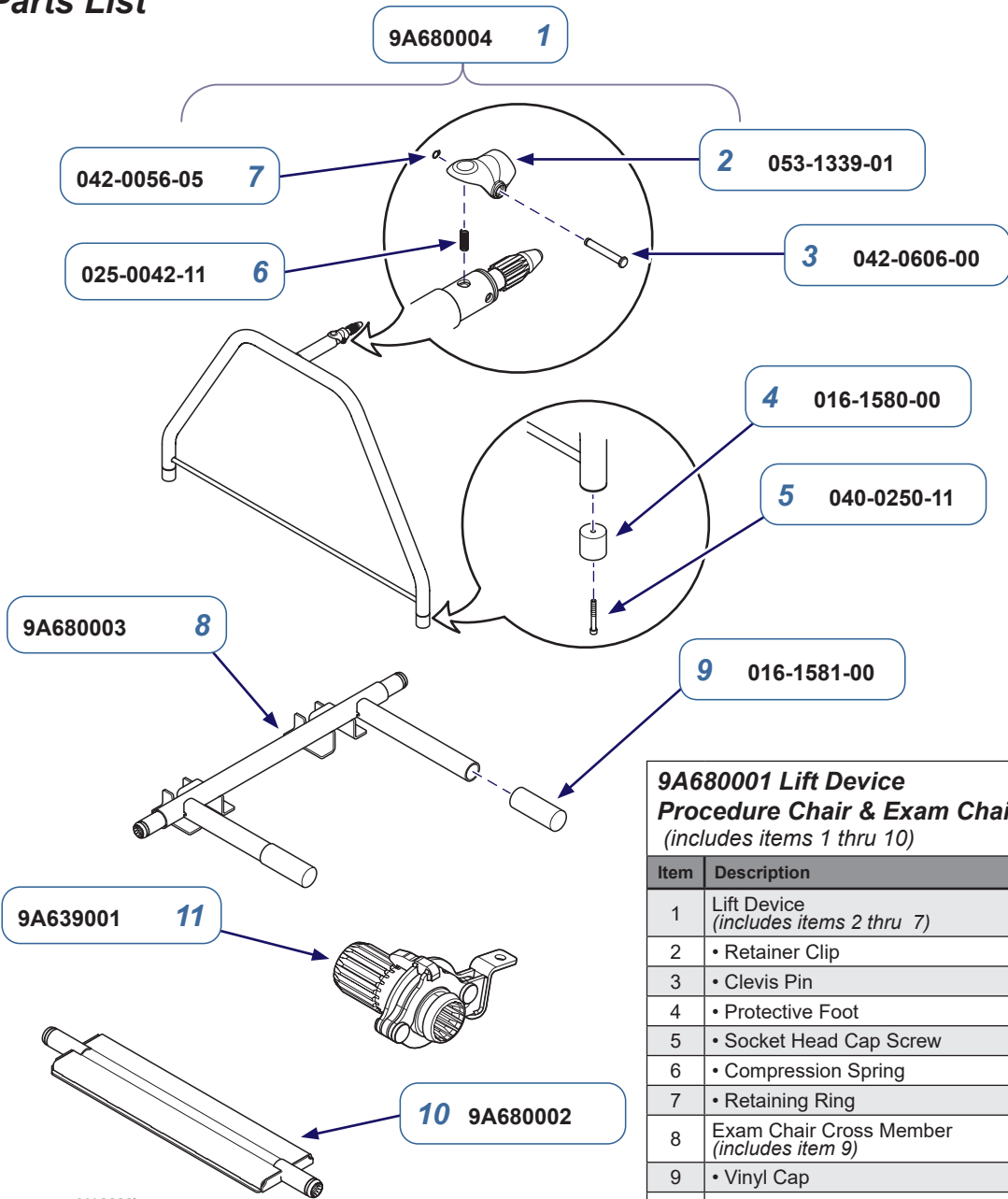9A680001 Lift Device Procedure Chair & Exam Chair (includes items 1 thru 10)		
Item	Description	Qty.
1	Lift Device (includes items 2 thru 7)	2
2	• Retainer Clip	1
3	• Clevis Pin	1
4	• Protective Foot	2
5	• Socket Head Cap Screw	2
6	• Compression Spring	1
7	• Retaining Ring	1
8	Exam Chair Cross Member (includes item 9)	1
9	• Vinyl Cap	2
10	Procedure Chair Cross Member	1
11	Spline Adapter	2

MA8666i

Warranty Information

Warranty Registration

Notes:

Notes:

Note

For additional languages please visit technicallibrary.midmark.com

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Midmark Corporation