

# Light Power Supply Replacement [002-10416-00] (Wall / Cabinet Mount)

**Applies to Models:**  
Wall / Cabinet Mount

**Special Tools:**  
none





## Warning

Always disconnect power before removing covers.  
Failure to do so may result in personal injury.


## Step 1: To access Power Supply.

- A) Disconnect power.
- B) Remove screws and cover(s).

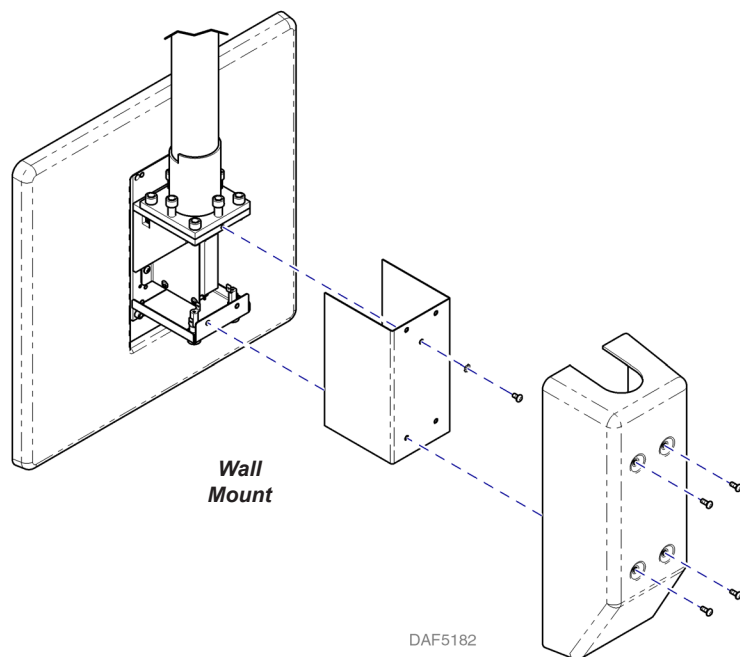
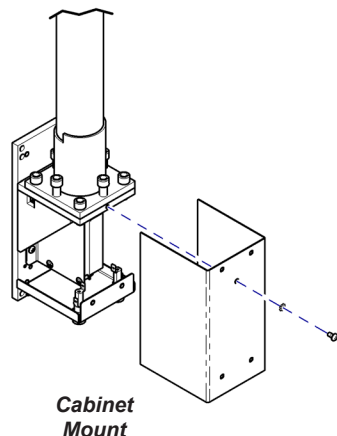


## Step 2: To remove Power Supply.

- A) Disconnect wiring harnesses and discard.
- B) Remove mounting screws.
- C) Remove power supply assembly from light mount assembly.

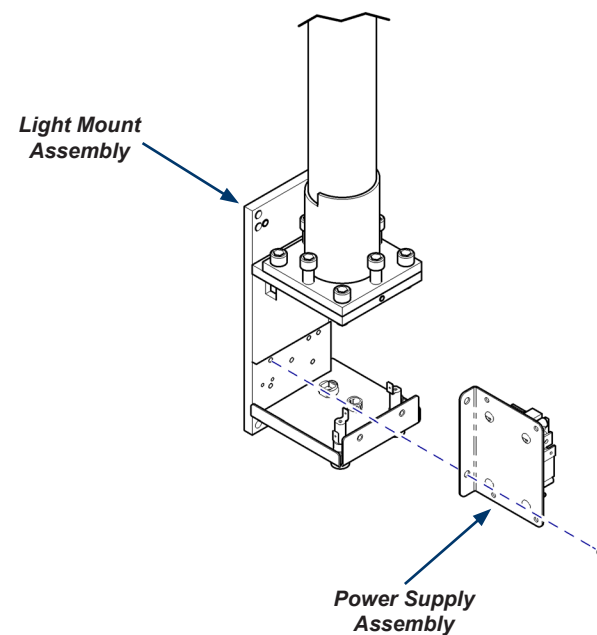

## Harness Connections

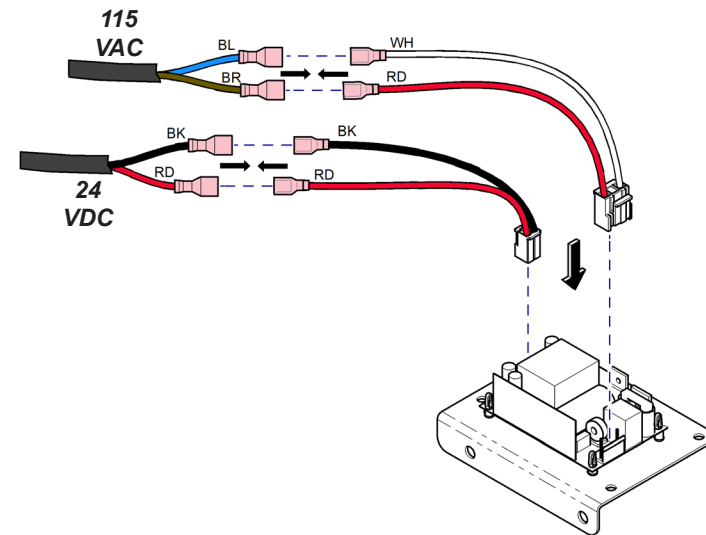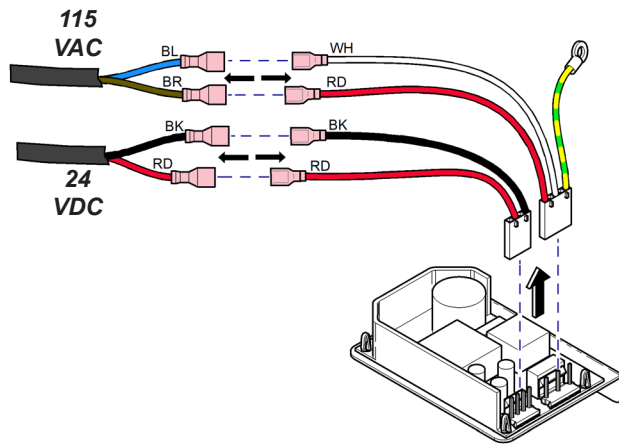

DAF5183

### Step 3: To install...

- A) Position power supply assembly and connect harnesses.
- B) Secure power supply assembly with screws.
- C) Attach cover(s) and secure with screws.
- E) Reconnect power.

Note: Refer to previous page for illustration.