

# Light Power Supply Replacement [002-10417-00] (Track Light Monitor / Ceiling Mount)

## Applies to Models:

Track Mounted Light  
Track Mounted Light w/ Monitor  
Ceiling Mounted Light

## Special Tools:


none



### **Warning**

Always disconnect power before removing covers.  
Failure to do so may result in personal injury.


**Track Light**

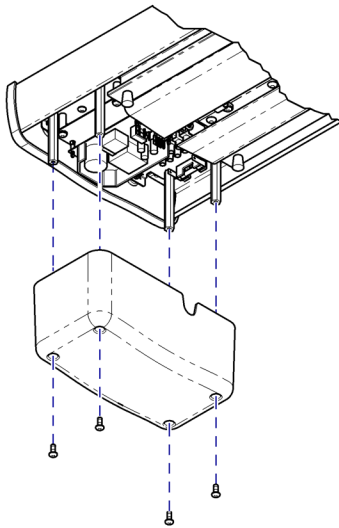


### **Step 1: To access Power Supply.**

- A) Disconnect power.
- B) Remove cover(s) as needed.


**Ceiling Mounted Light**

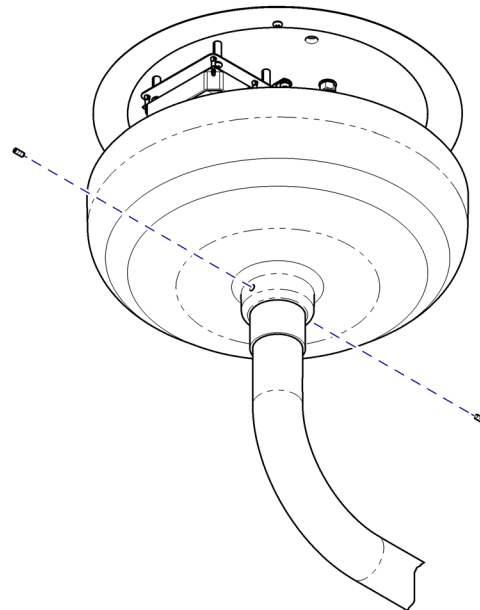

**Track Light / Monitor**

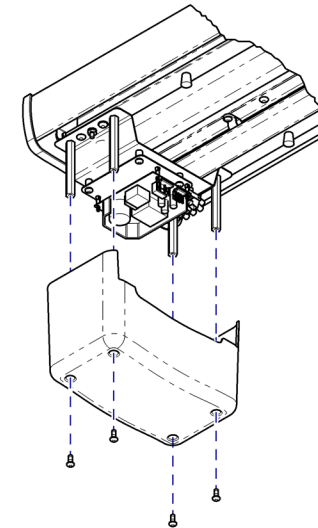


DAF5180

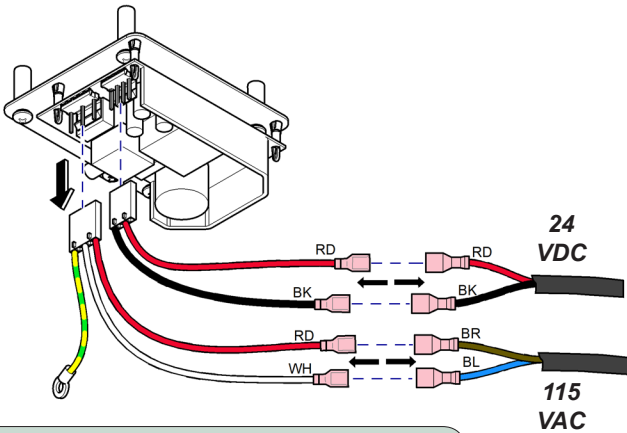

### Step 2:

- Disconnect wiring harnesses and discard.
- Use needle nose pliers to release power supply from mounting plate.

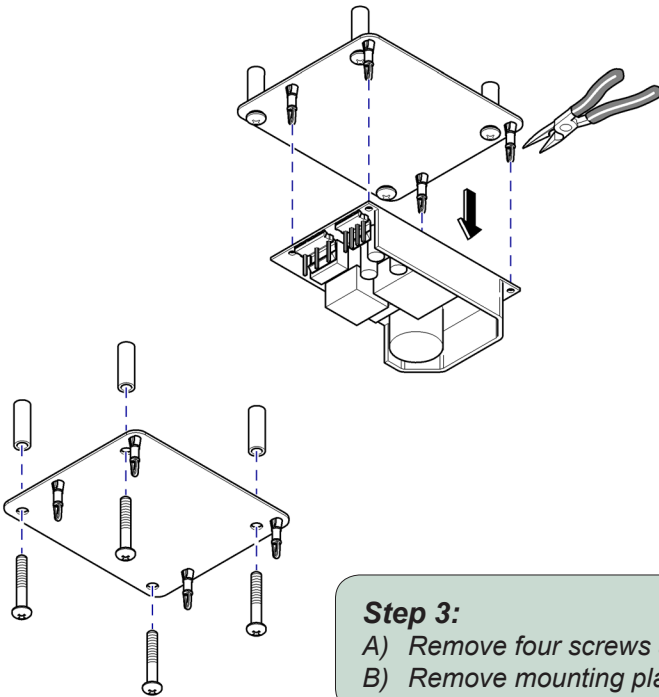


### Step 3:

- Remove four screws and spacers.
- Remove mounting plate and discard.

DAF51181


### Step 4:

- Install four screws and spacers.
- Secure power supply assembly.

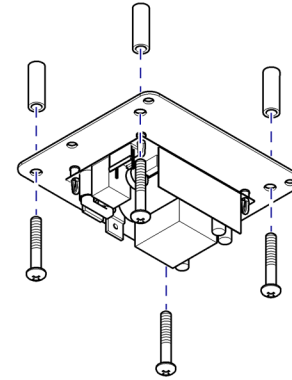

### Step 5:

- Connect supplied harnesses.

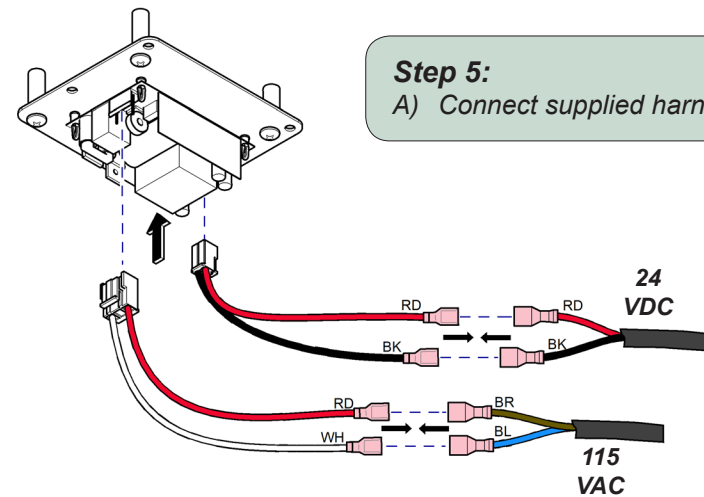

### Step 6:

- Install cover(s).
- Connect power.