

Light Power Supply Replacement [002-10418-00] (Universal Fixed Arm)

Applies to Models:
Universal Fixed Arm

Special Tools:
none



Warning

Always disconnect power before removing covers.
Failure to do so may result in personal injury.


Step 1: To access Power Supply.

- Disconnect power.
- Remove four screws, top cover, and trim ring.
- Remove two cap screws to separate fixed arm from mounting post.
- Remove four screws, spacers and standoffs.
- Disconnect wiring harnesses and discard.
- Remove power supply assembly.





Equipment Alert

Apply 042-0024-00 adhesive to threads of cap screws before reassembling.

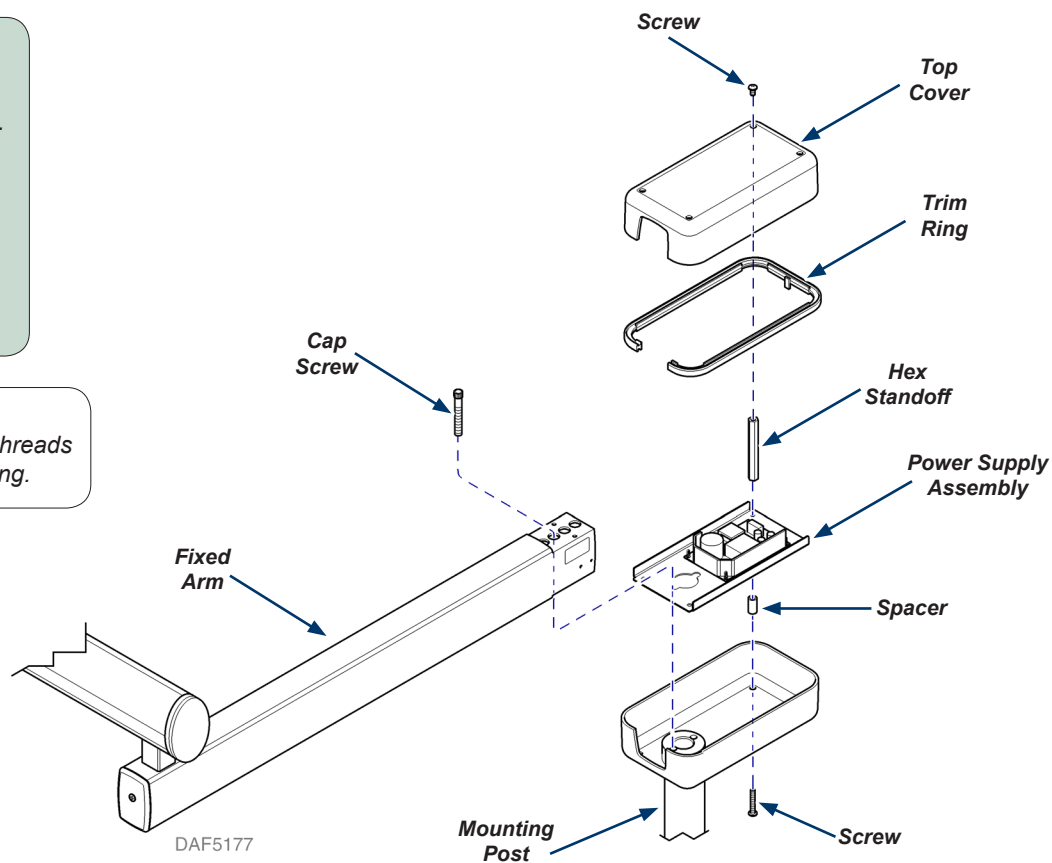

Harness Connections

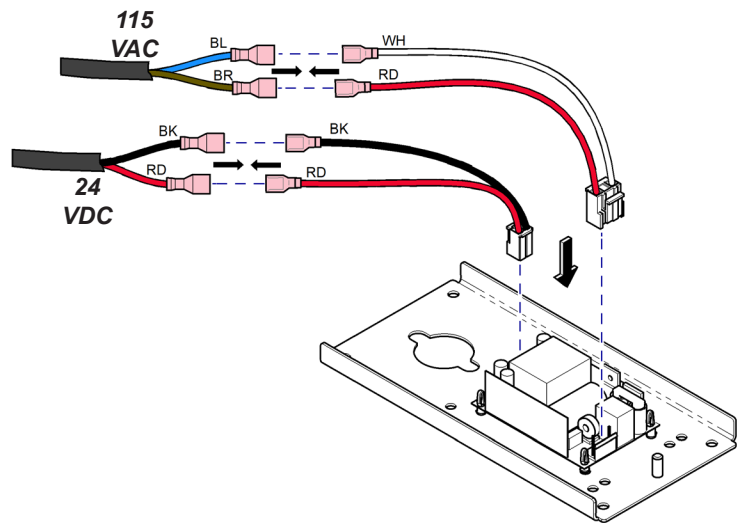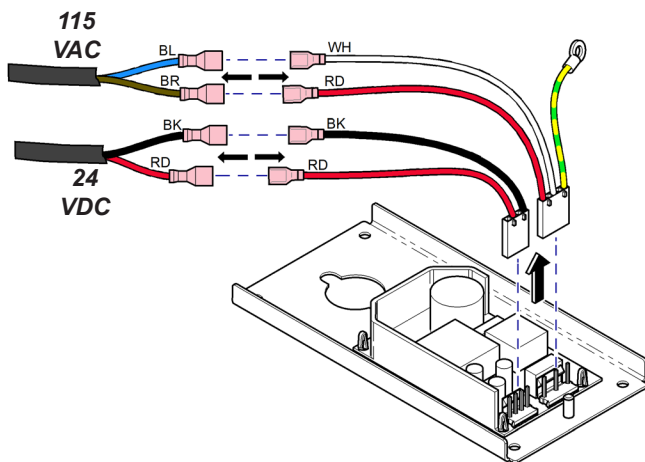

DAF5178



Equipment Alert

Apply 042-0024-00 adhesive to threads of cap screws before reassembling.

Step 2: To install...

- A) Position power supply assembly and connect harnesses.
- B) Secure power supply assembly with four screws, spacers, and standoffs.
- C) Attach fixed arm to mounting post with two cap screws.
- D) Position top cover and trim ring. Secure with four screws.
- E) Reconnect power.

Note: Refer to previous page for illustration.