



# Power Vac Maintenance Kit

002-10413-00

002-10413-01

## Applies to Models:

-00 = P3, P5 & G3, G5

-01 = P7 & G7

## Components:

Conditioning Fluid... 1

Belt ..... 1



### WARNING


To prevent risk of shock always disconnect power before removing covers or performing any service procedure.

### Note

Most of pump pulley keyway is hidden behind washer. Top edge of keyway cut out should be visible.

## 1. Conditioning Fluid

- Unfasten front cover by turning bottom fasteners 1/4 turn counter-clockwise. Lift covers straight up to remove from side panels.
- Disconnect intake hose union.
- Rotate pump pulley until keyway (small notch in pulley) is in 3 or 9 o'clock position.
- Pour conditioning fluid (provided) into intake hose assembly.
- Rotate pump pulley clockwise 5 rotations.
- Replace covers and re-connect intake hose assembly.
- Check exhaust drain and drain tube for any obstructions.

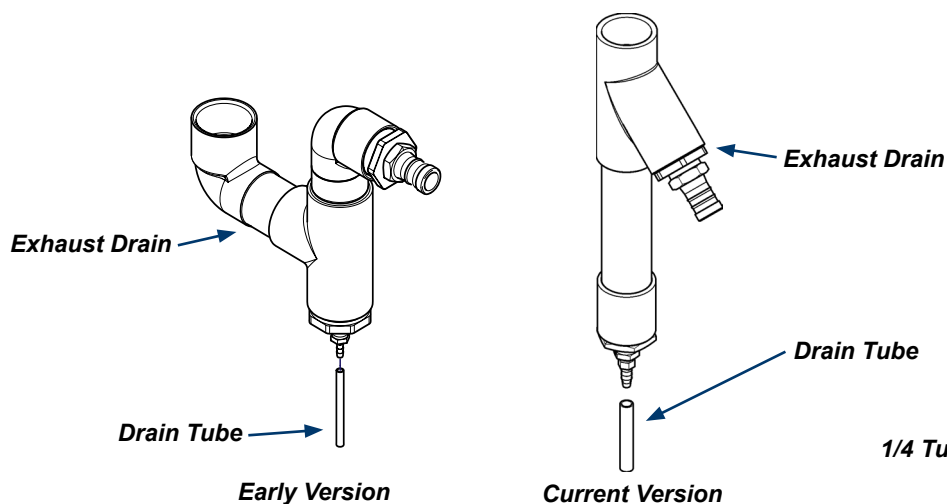

Pump Pulley Keyway  
(position 3 or 9 o'clock)

Intake Hose Union

To Vac

