Pivot Brake Assembly Kit [002-10429-00]

Special Tool:

Torque Wrench
 Allen Wrenches - (Metric & Inch)
 Tack Cloth
 Deburr Tool (File)
 Drill
 Multi-use Lubricant
 3/16" Allen Drive - min. 1-1/2" LG

Note

Torque wrenches should be capable of 70 in-lbs (7.9 Nm) to 200 in-lbs (22.6 Nm), and 33 ft-lbs (44.7 Nm).

Step 1: Inspect components.

Step 2: Position chair.

- Move the chair back to FULL DOWN position.
- Raise / lower the chair to a comfortable working height, no higher than 3/4 the way up.

WARNING

Always disconnect the power source before removing any covers. Failure to do so may result in personal injury.

Step 3: Remove seat upholstery.

- Disconnect power source.
- Rotate seat locking knob.
- Lift seat upholstery, then pull to release tabs from frame.

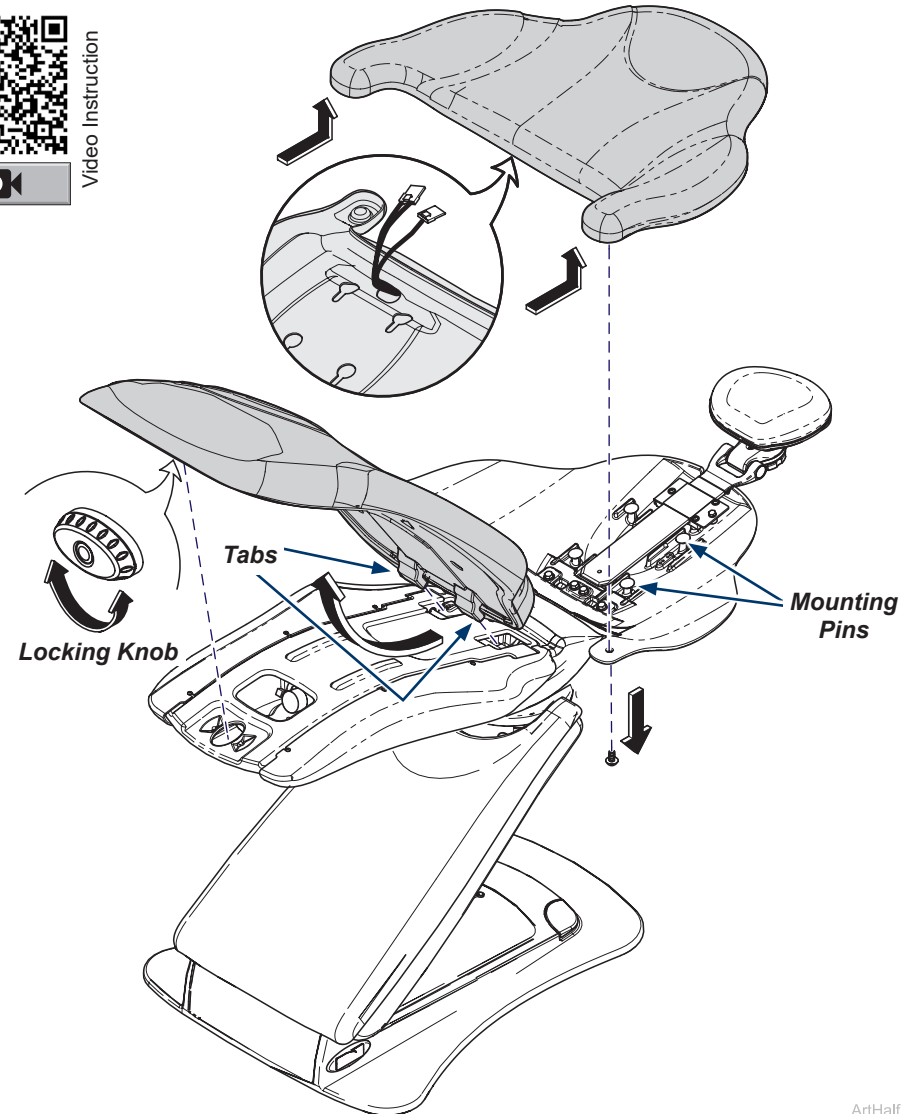
Step 4: Remove back upholstery.

- Remove two screws in arms.
- Lift up on upholstered back to free from mounting pins.
- If present disconnect cables from of back upholstery before removing.

Note: Only heated upholstery will need to be unplugged before removal.

Video Instruction


Note: Retain all shrouds, frame and mounting hardware for re-use. Some mounting hardware may not be shown depending on model.

Note: Shoulder Screws need torqued down 33 ft-lb when reinstalling seat frame.

Step 5: Remove shrouds and frame.

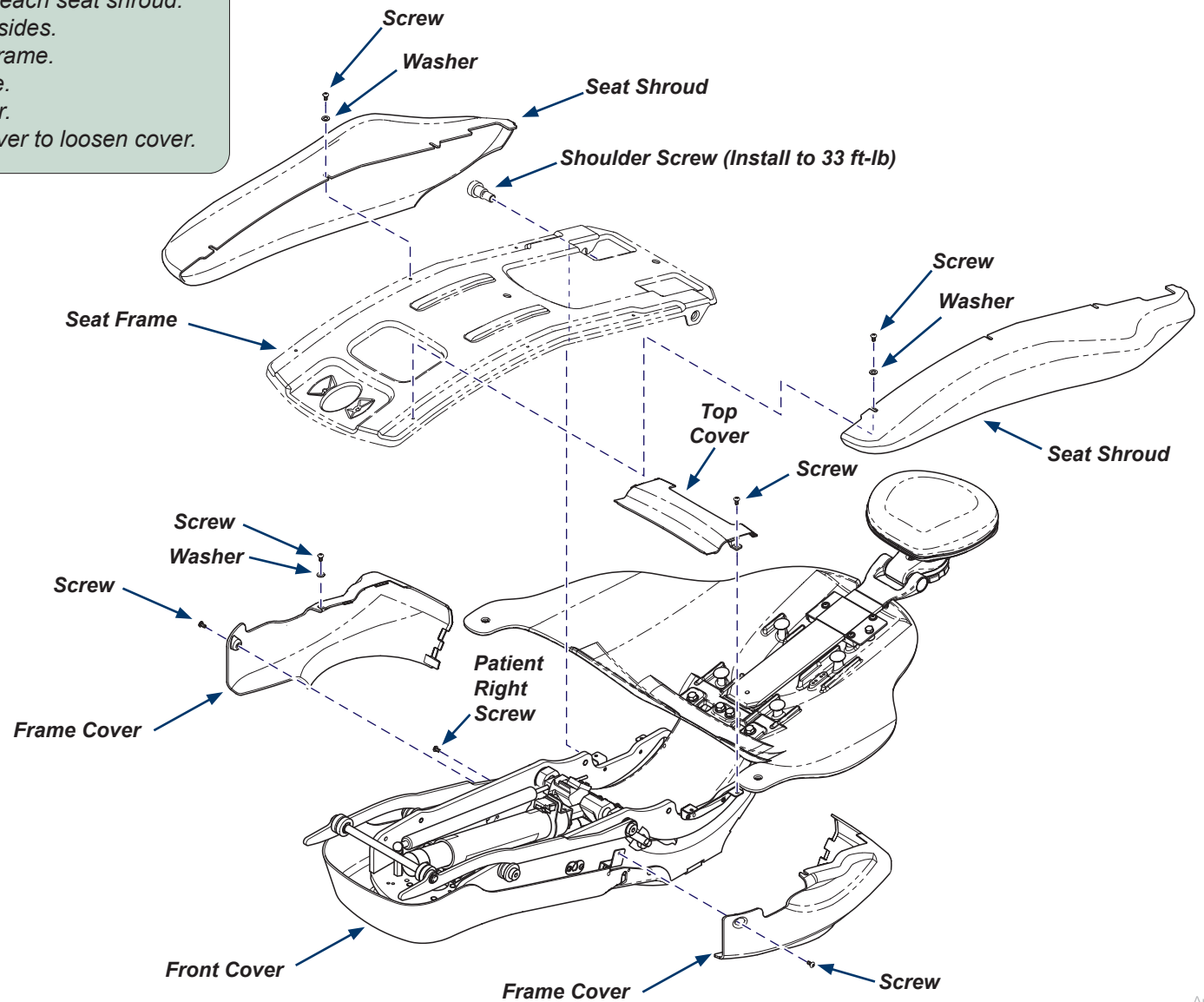
- A) Remove three screws and washers from each seat shroud.
- B) Remove seat shrouds from right and left sides.
- C) Remove two shoulder screws then seat frame.
- D) Remove both frame covers and hardware.
- E) Remove two screws from frame top cover.
- F) Remove patient right screw from front cover to loosen cover.

ArtFull


Step 6: Console mounted on Patient Right - (ONLY).

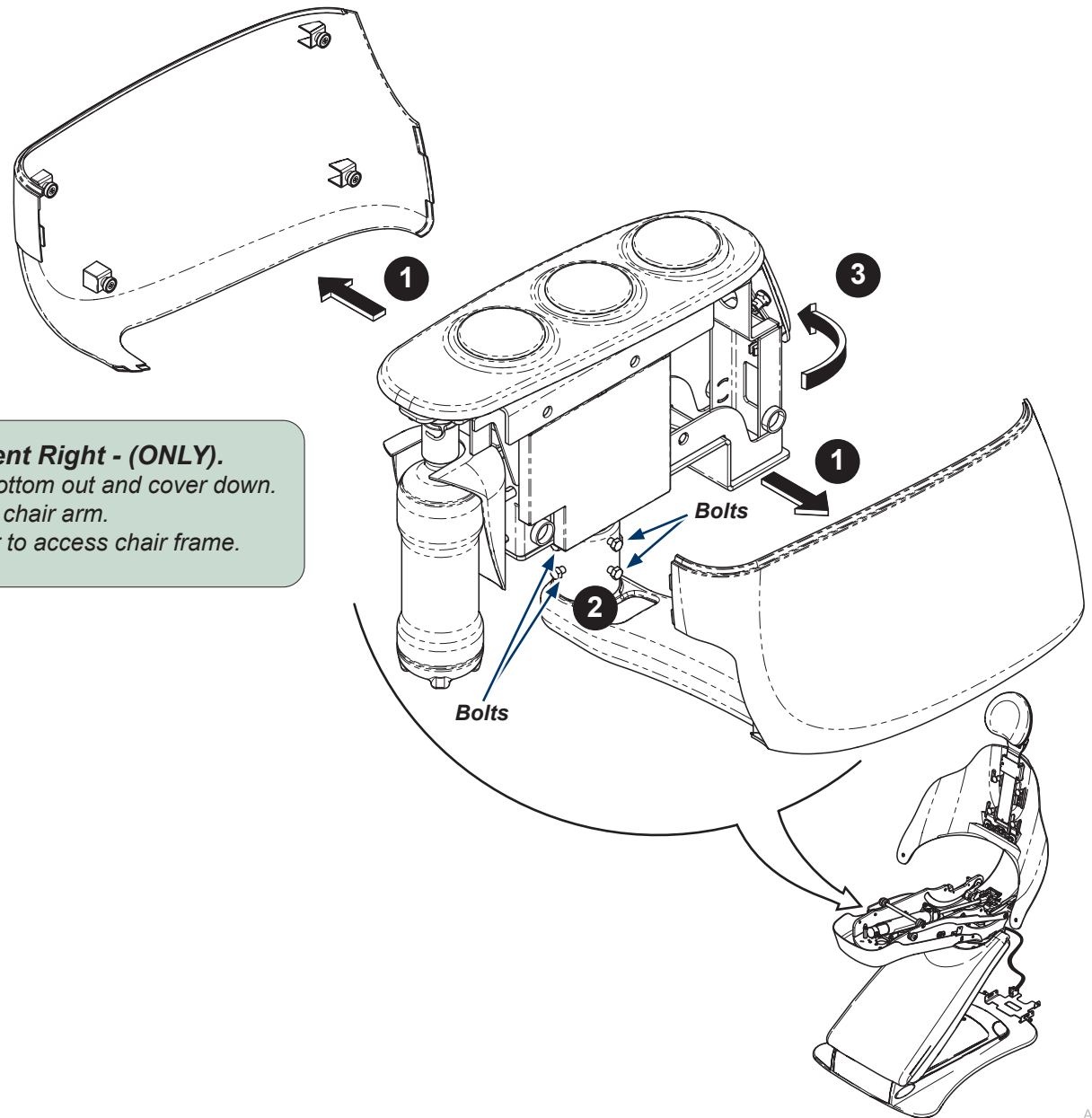- A) Remove console covers by pulling bottom out and cover down.
- B) Loosen eight bolts under console on chair arm.
- C) Rotate console out of way from chair to access chair frame.

Step 7: Removing Z-bracket.

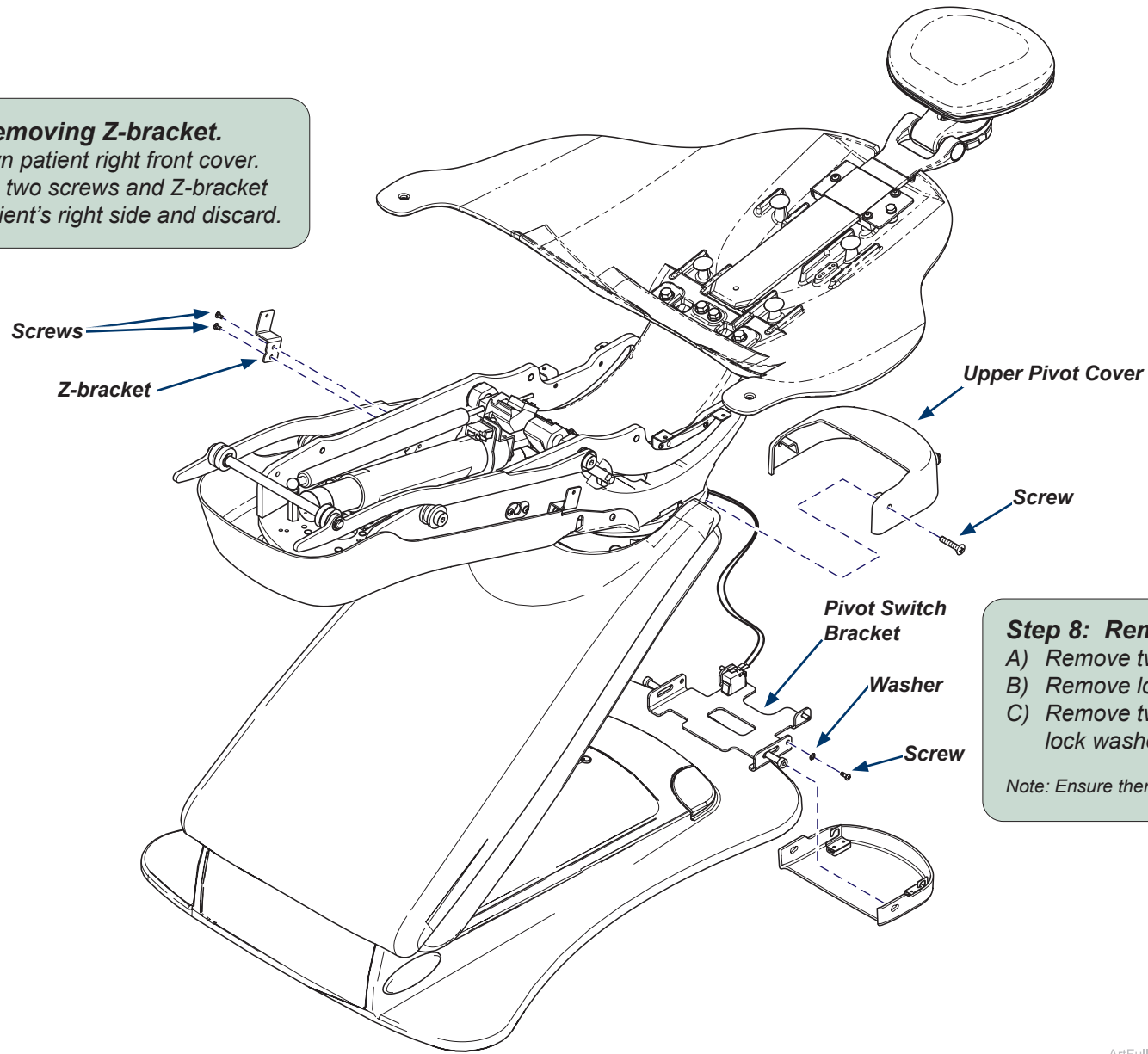- A) Pull down patient right front cover.
- B) Remove two screws and Z-bracket from patient's right side and discard.



Step 8: Removing Pivot Covers and Bracket.

- A) Remove two screws and upper pivot cover.
- B) Remove lower pivot cover.
- C) Remove two screws, two shoulder screws and lock washers to remove pivot switch bracket.

Note: Ensure there is no tension on switch wires.

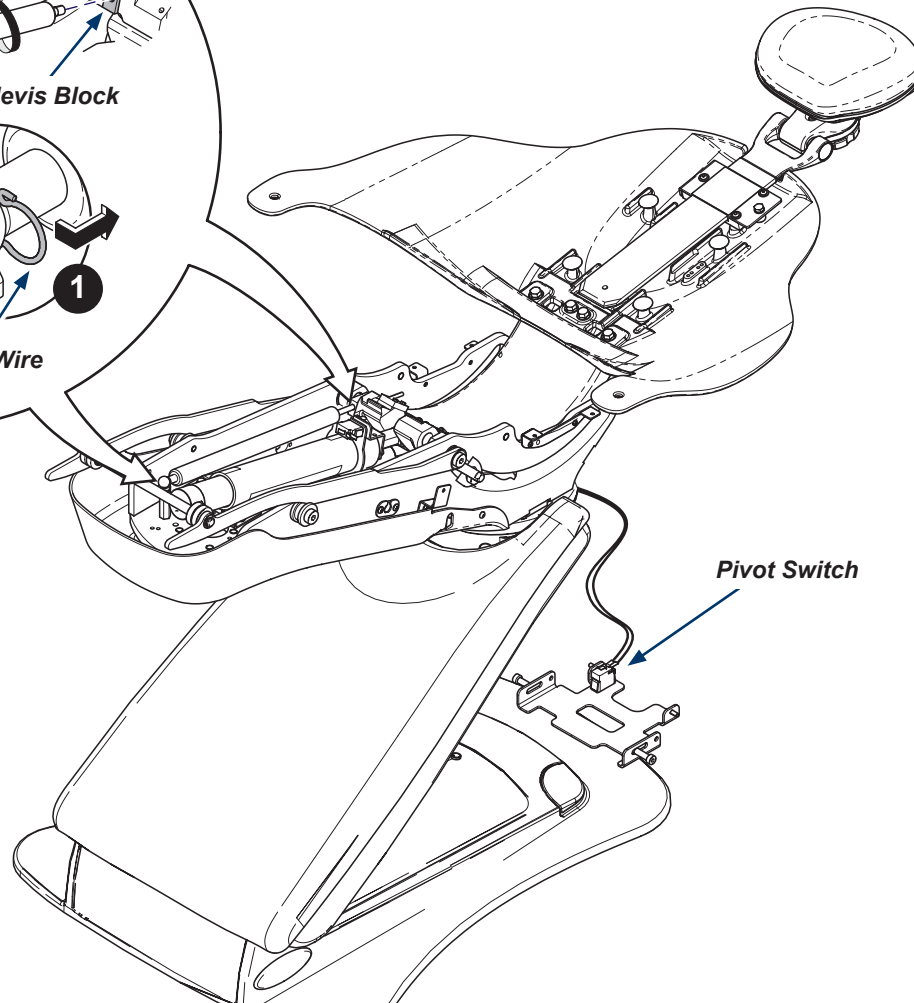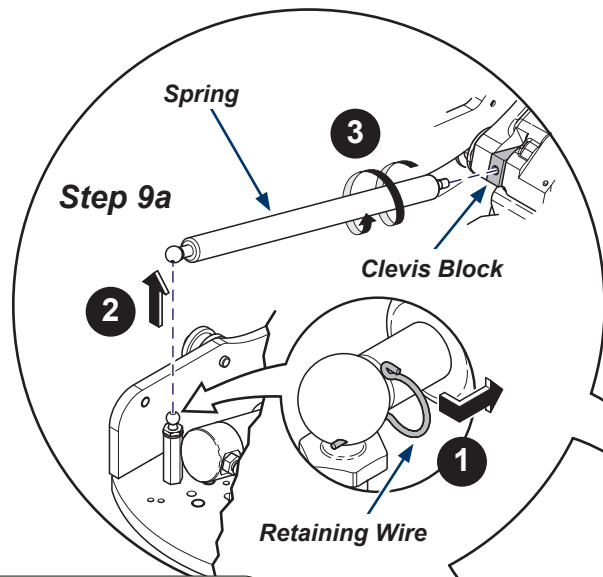ArtFull


Caution
 Move back to be down completely. Retraction spring under tension.

Step 9a: Remove retraction spring.

A) Remove retaining wire.
 B) Pull spring off stud.
 C) Unscrew spring from clevis block.

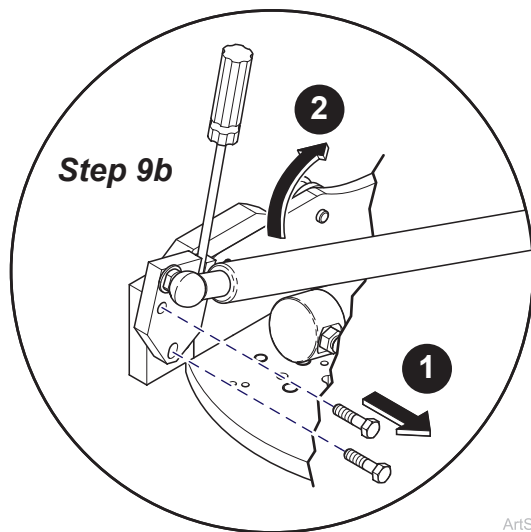Note: If chair back does not move check pivot switch connections.

Caution
 Move back to be down completely. Retraction spring under tension.

Step 9b: Remove retraction spring.

A) Remove bolts (2) and hold screwdriver as shown.
 B) Rotate retraction spring and rest on seat back.

Note: If chair back does not move check pivot switch connections.

ArtFull

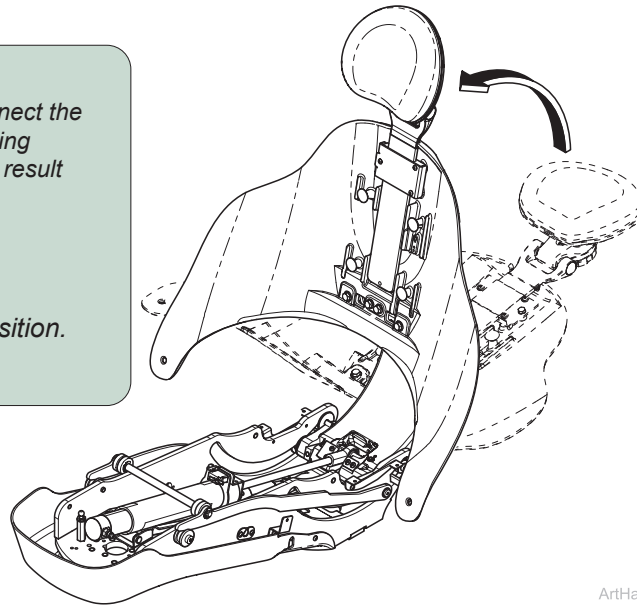
WARNING

After positioning chair disconnect the power source before performing service. Failure to do so may result in personal injury.

Step 10: Position chair.

- A) Connect chair to power source.
- B) Raise chair back to FULL UP position.
- C) Disconnect from power source.

ArtHalf

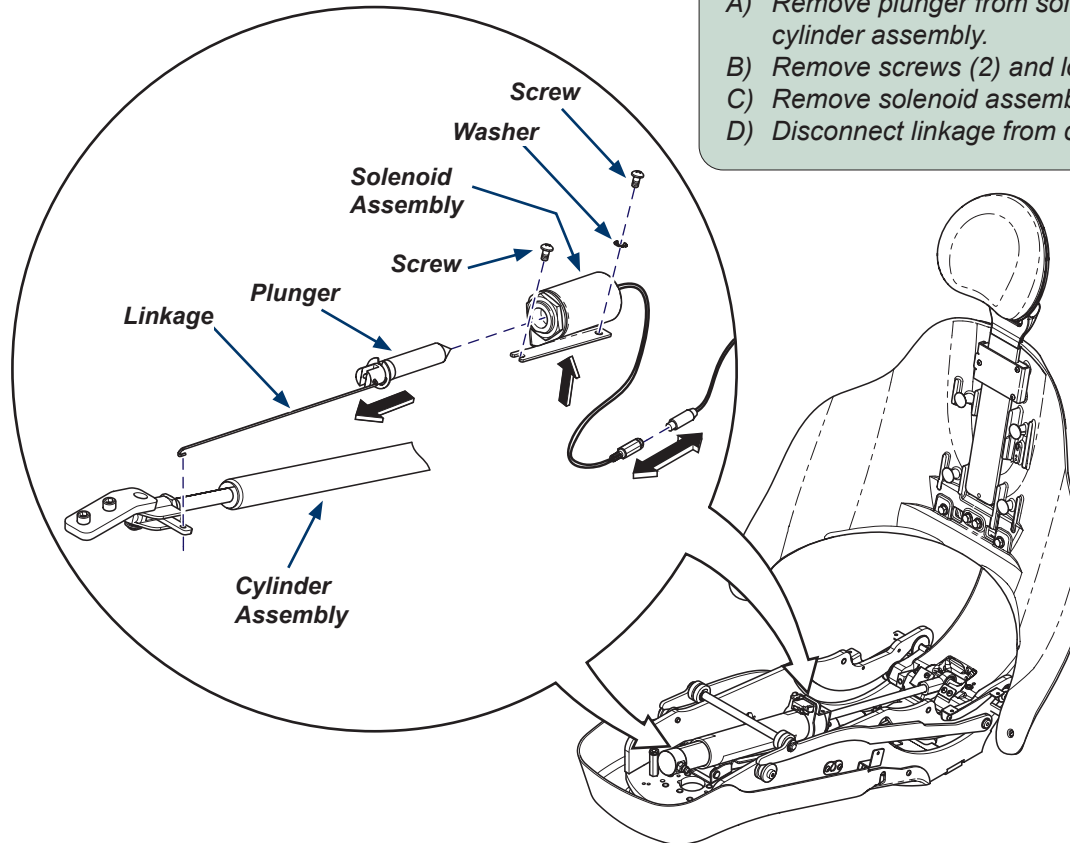
Caution

Do not damage while unplugging solenoid wire from solenoid assembly. Ensure wires are secure.

Step 11: Remove linkage and solenoid assembly.


- A) Remove plunger from solenoid assembly and unhook linkage from cylinder assembly.
- B) Remove screws (2) and lock washer from solenoid assembly.
- C) Remove solenoid assembly and unplug solenoid from chair.
- D) Disconnect linkage from cylinder assembly and remove.

ArtFull

Step 12: Remove cylinder assembly.

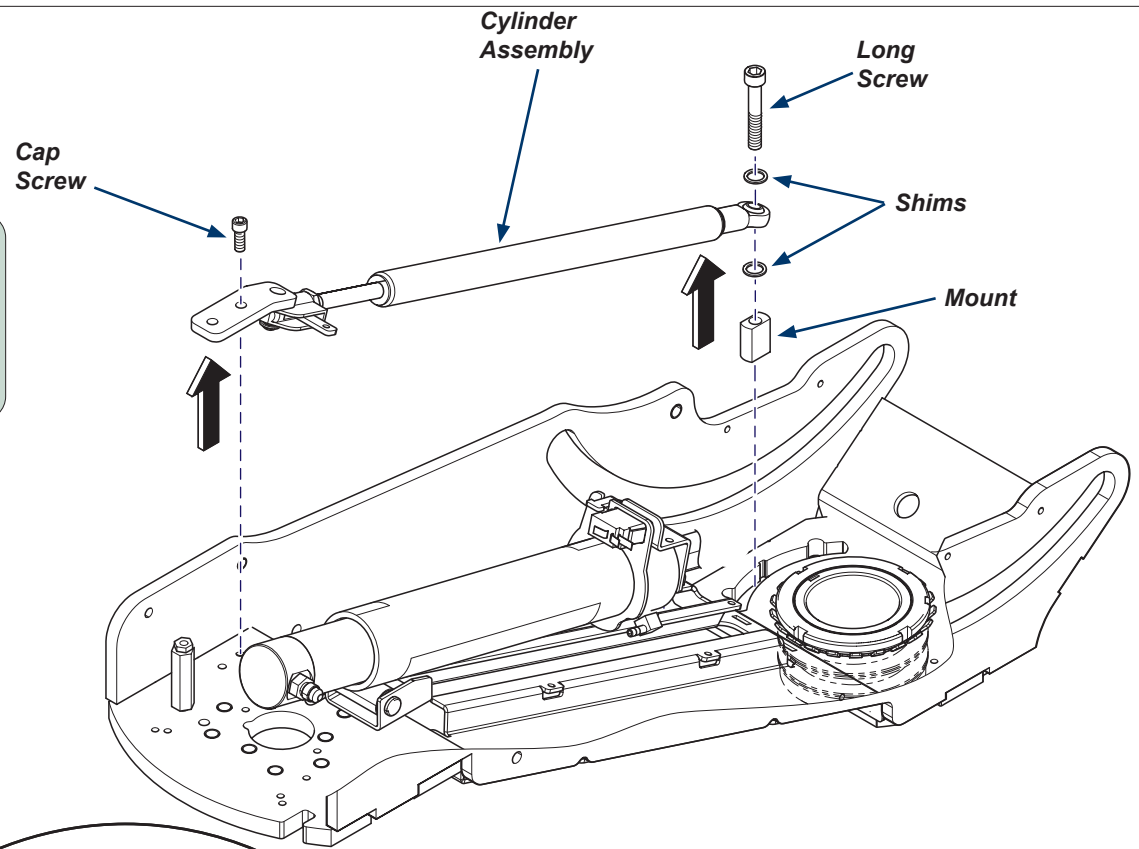
- A) Remove two cap screws from cylinder assembly.
- B) Remove long screw and shims from mount.
- C) Remove mount.
- D) Remove cylinder assembly.


Step 13: Protecting wiring and tubing.

- A) Wires and tubing may need pulled to patient's left side.
- B) Place drill guard in position to protect wiring and tubing.
- C) Secure to wiring and tubing with wire tie.
- D) Place tack cloth under pivot hub to catch shavings.

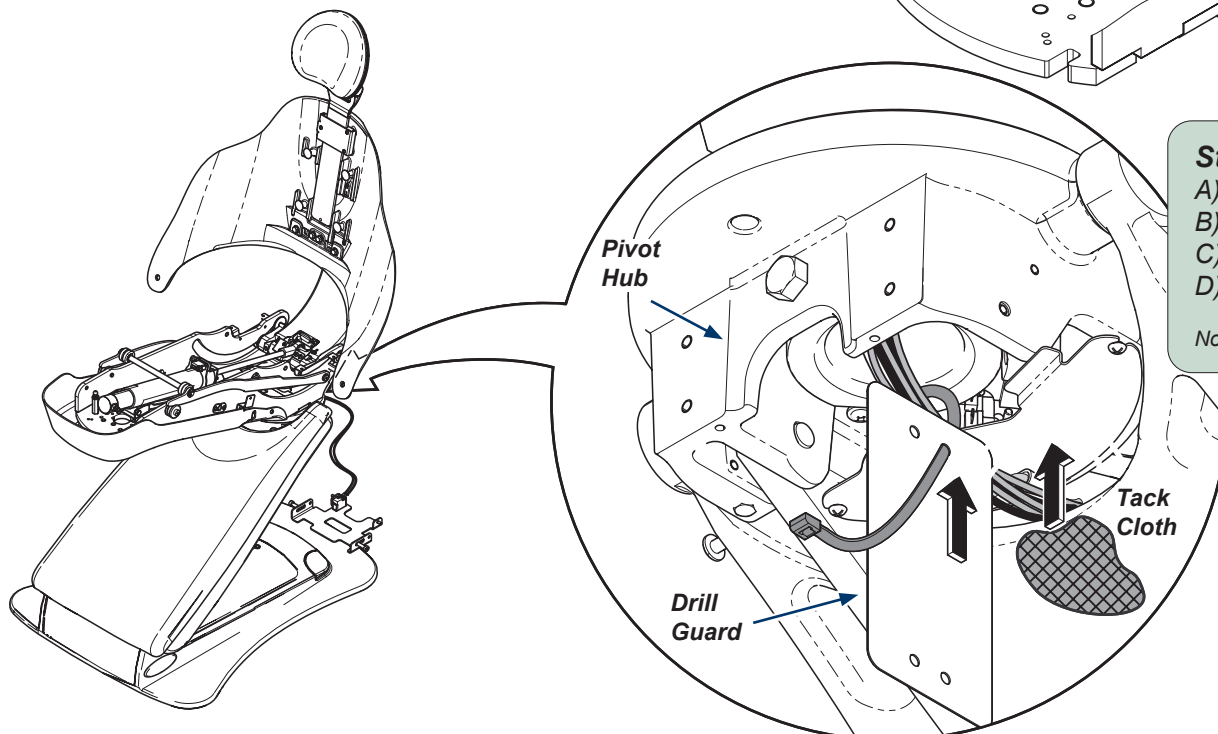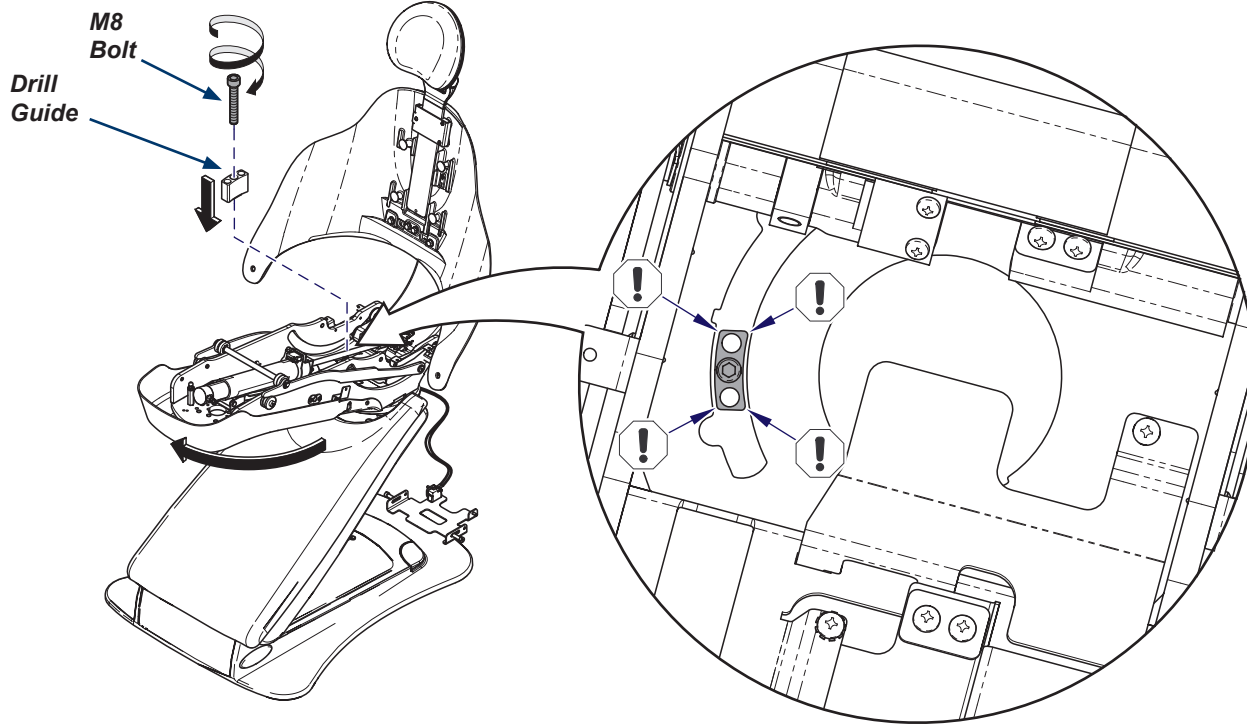Note: Drilling will occur on patient's right side.


Equipment Alert

Position drill guide with spacing on both sides of slot. Failure to do so will create rotation drag. Before drilling any holes, ensure the guide does not touch the chair slot when pivoting the chair.

Step 14: Installing drill guide.

- A) Install drill guide, rectangular step face down, with new M8 bolt.
- B) Position the drill guide with spacing on both sides of slot.

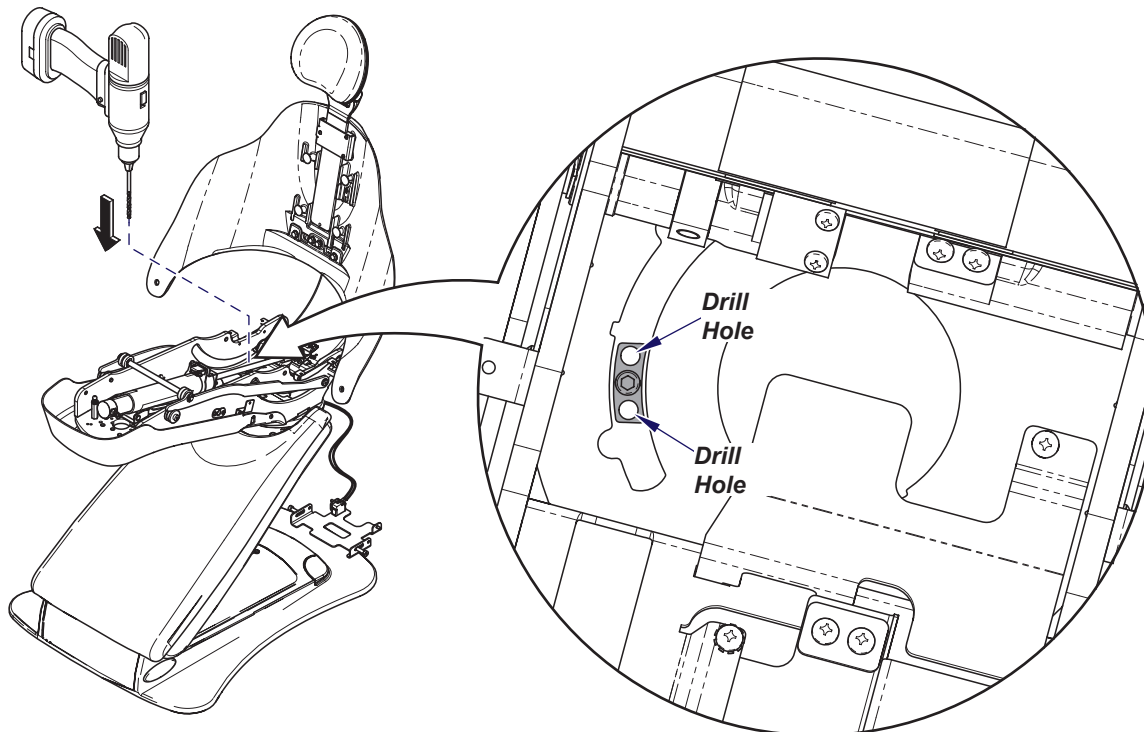ArtFull


Step 15: Drilling holes.

- A) Rotate chair clockwise / patient's right for best access.
- B) Spray drill lubricant into the guide holes before and during drill operation to reduce metal shavings.
- C) Drill two holes thru the dental chair weldment with drill bit provided.

Note: Frequently collect shavings with magnet and discard.

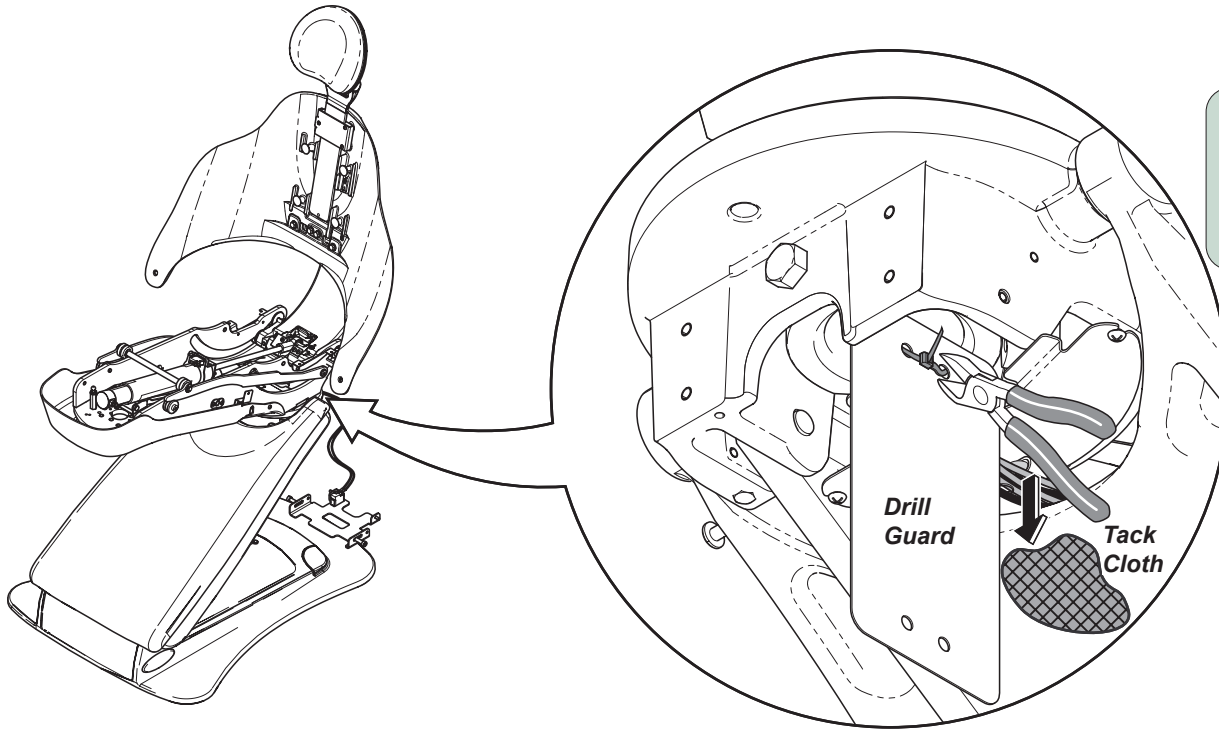ArtFull

Step 16: Removing drill guard.

- A) Remove tack cloth.
- B) Remove drill guard from under chair. Move drill guard to seat weldment shown below.

ArtFull

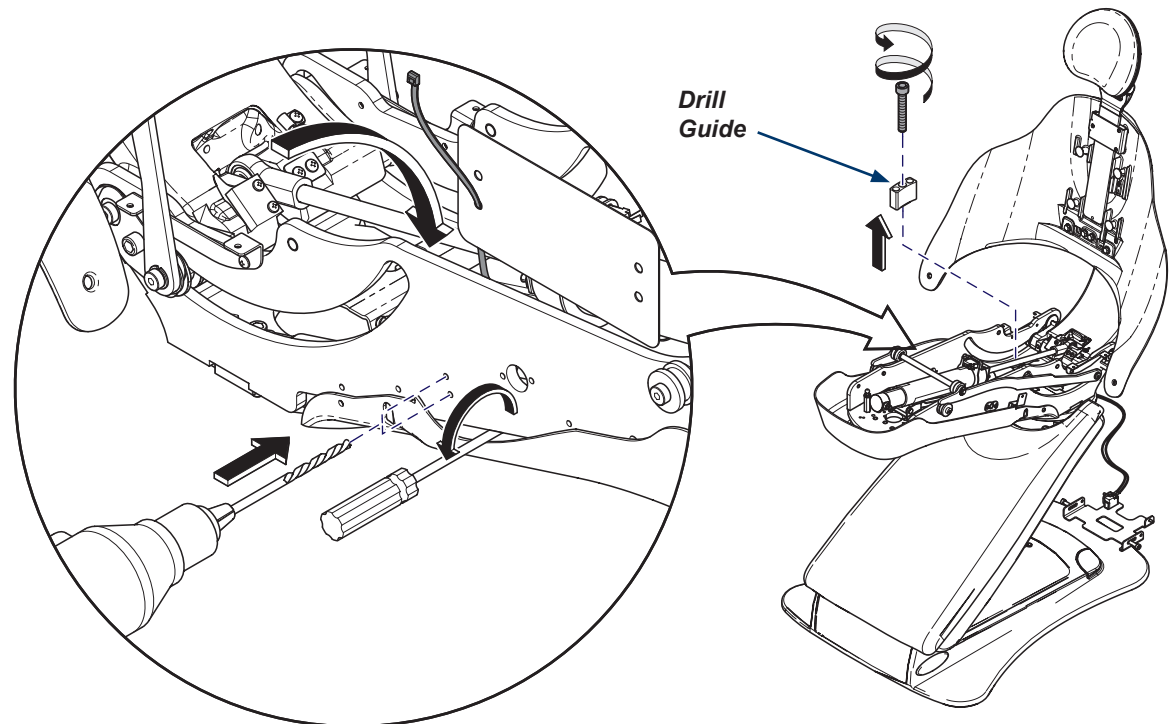
Equipment Alert

Make sure PC Board is covered before drilling holes to prevent shaving from damaging the board.

Step 17: Installing drill guard.

- A) Install drill guard to protect tubing in seat weldment and secure with wire tie.
- B) Rotate chair to the patient left.
- C) Increase the two hole size for Z-bracket holder on side of chair with drill bit provided.
- D) Remove drill guide from top of chair.


Note: May need a screwdriver to hold the bottom cover out of the way of drill.

ArtFull

Step 18: Clean holes.

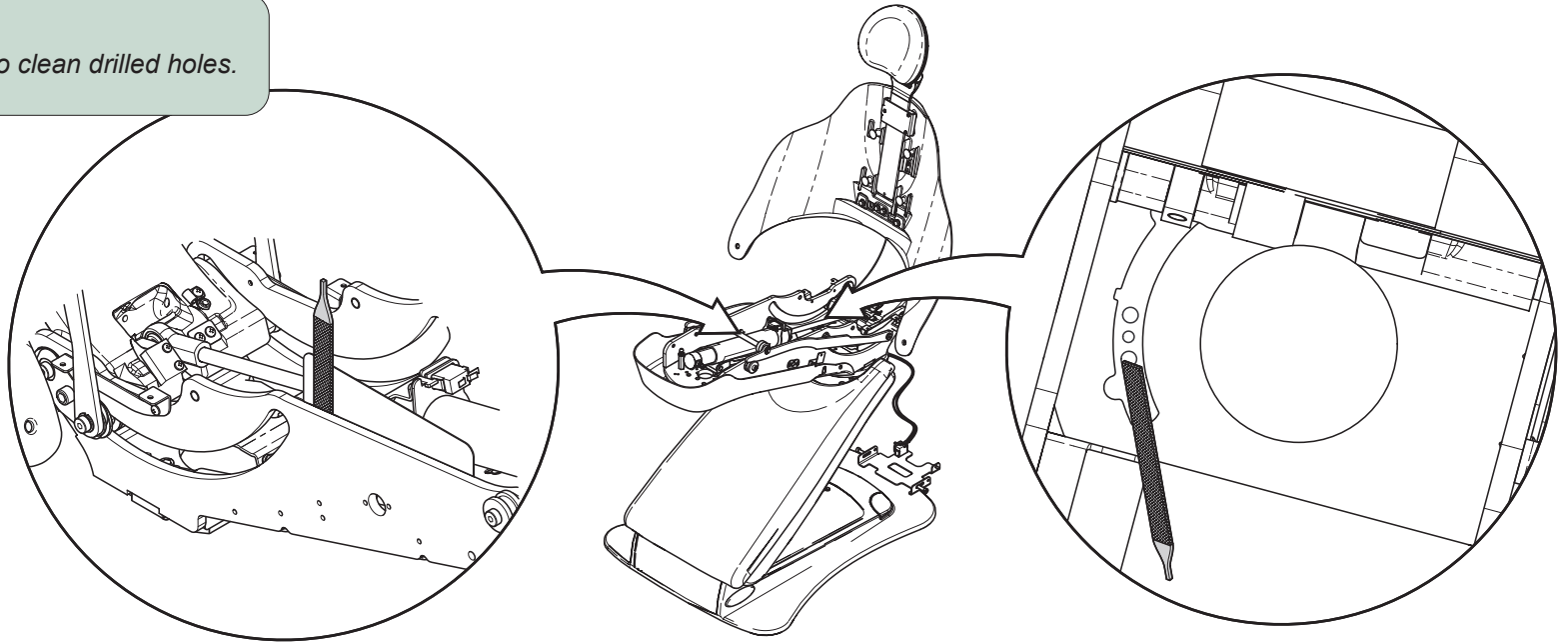A) Use a file or deburring tool to clean drilled holes.


Step 19: Removing guard and clean.

A) Remove drill guard from seat weldment.


B) Clean all metal shavings from chair with tack cloth and hand held magnet before any reassembly.

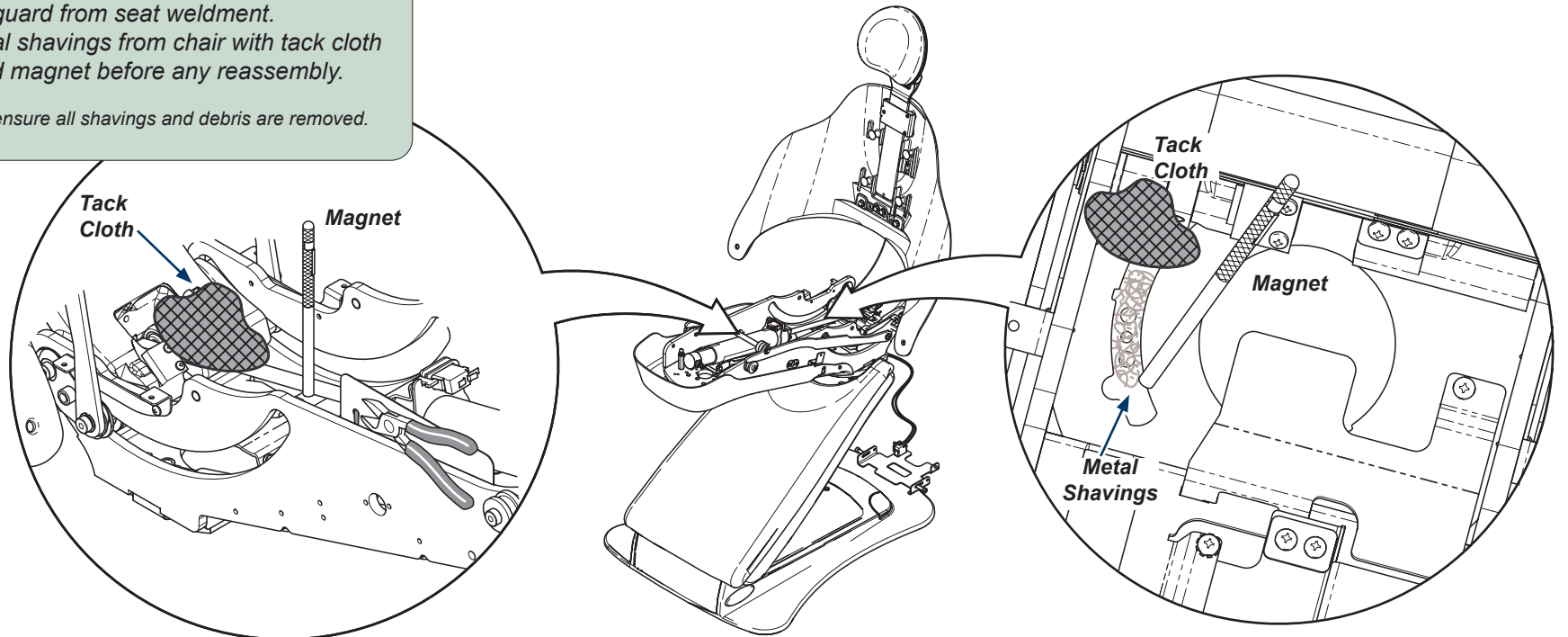Note: Rotate chair to ensure all shavings and debris are removed.

Step 20: Install pivot mount.

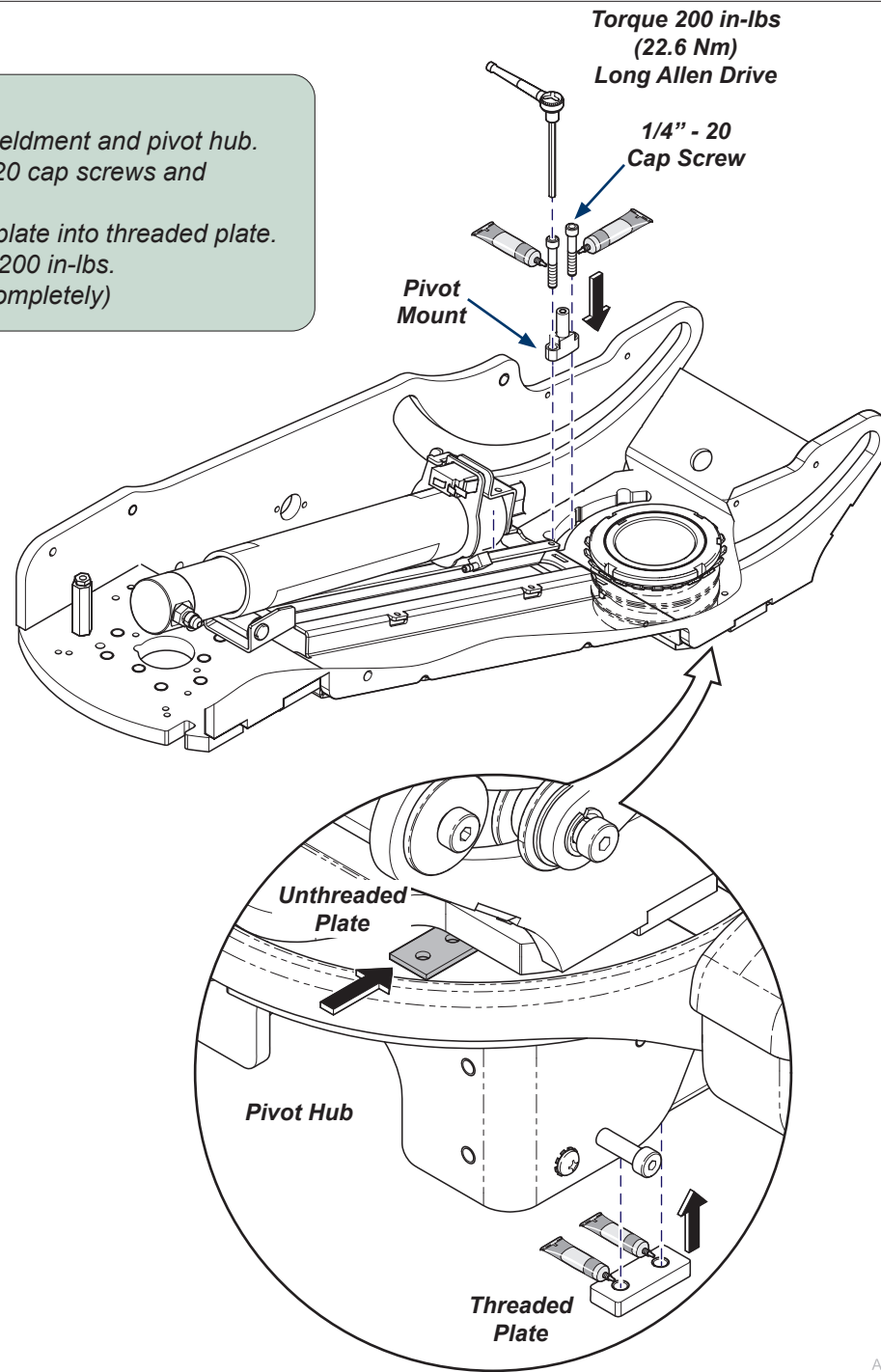
- A) Place unthreaded plate between seat weldment and pivot hub.
- B) Apply thread lock adhesive to two 1/4"-20 cap screws and threaded plate.
- C) Install pivot mount through unthreaded plate into threaded plate.
- D) Once hand tightened, torque screws to 200 in-lbs. (Requires longer allen driver to install completely)

ArtFull

Step 21: Install pivot brake.

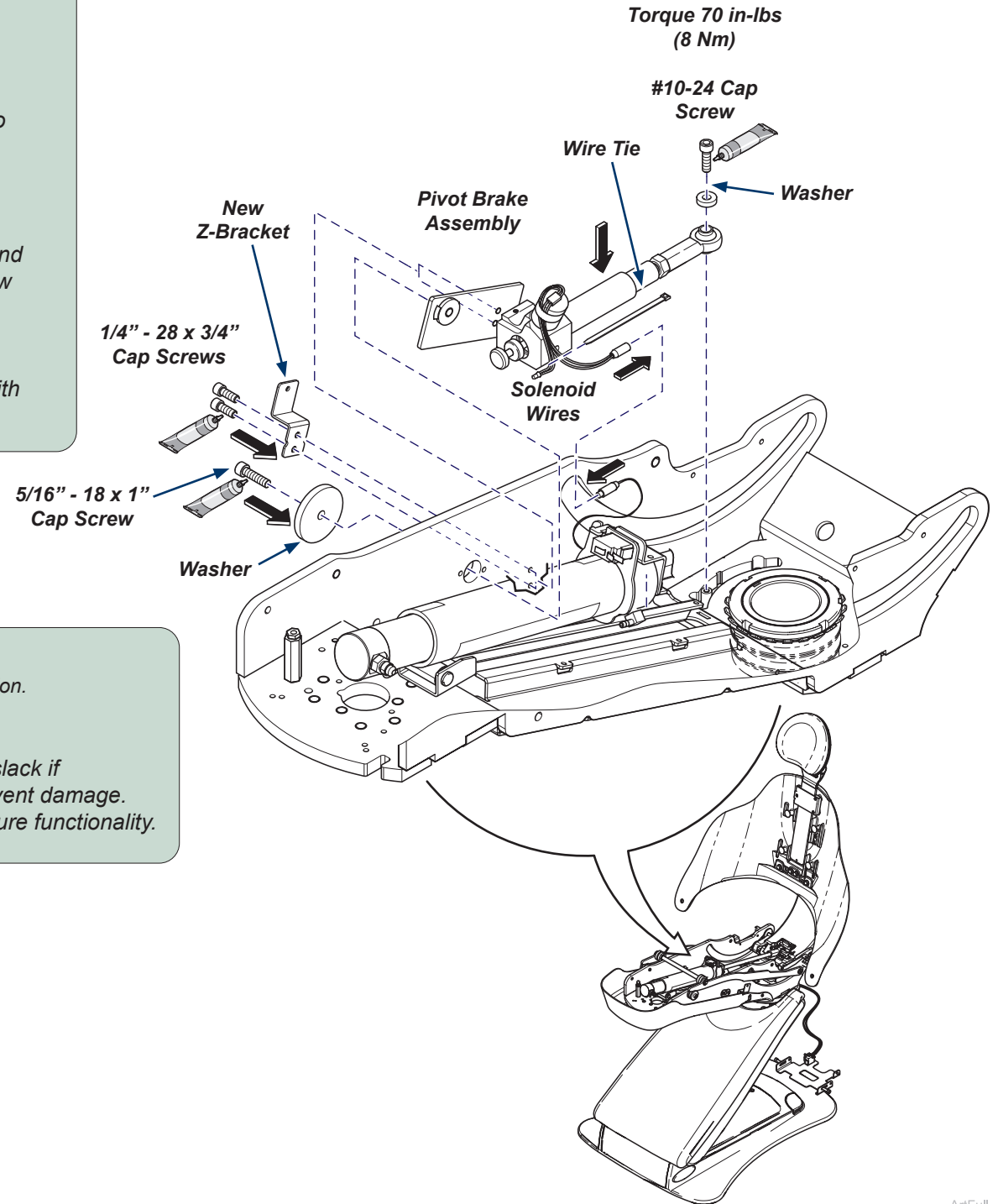- A) Slide ball joint of brake assembly over pivot mount, while also positioning tapered stud of mounting plate within large hole in seat weldment.
- B) Secure ball joint to pivot mount. Apply thread lock adhesive to #10-24 cap screw and thread through washer. Hand-tighten.
- C) Bolt tapered stud to side of seat weldment. Apply thread lock adhesive to 5/16-18 x 1" cap screw and thread through one 5/16" plain washer. Hand tighten.
- D) Apply thread lock adhesive to two 1/4-28 x 3/4" cap screws and secure other side of mounting plate to seat weldment and new Z-bracket. Hand tighten.
- E) Once the plate is properly positioned, torque #10-24 cap screw to 70 in-lbs.
- F) Tighten the 5/16 cap screw and the two 1/4 - 28 cap screws with wrench.

Equipment Alert

Pinching solenoid wires can cause chair to malfunction.

Step 22: Plugging in solenoid wires.

- A) Plug solenoid into existing connector, pull additional slack if necessary. Secure solenoid wires with wire tie to prevent damage.
- B) Plug the chair back in and operate pivot brake to ensure functionality.

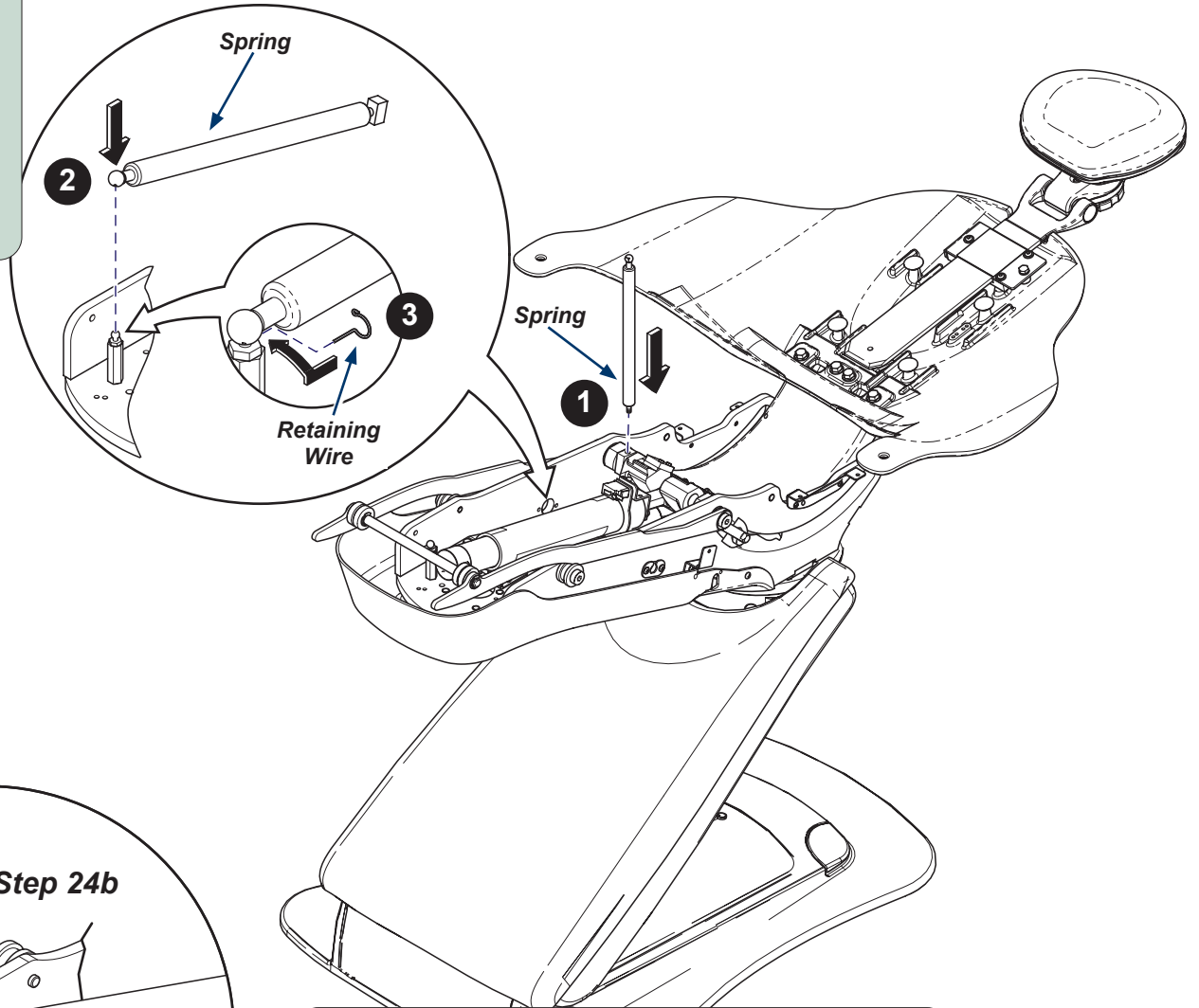
Caution

Retraction spring under pressure.



Step 23: Reinstall retraction spring.

- A) If replacing retraction spring with kit skip to next step.
- B) Screw spring to clevis block.
- C) Lower chair back all the way down using some pressure to assist.
- D) Unplug the chair.
- E) Place spring onto stud.
- F) Install retaining wire.

Step 24a: Replacing retraction spring.

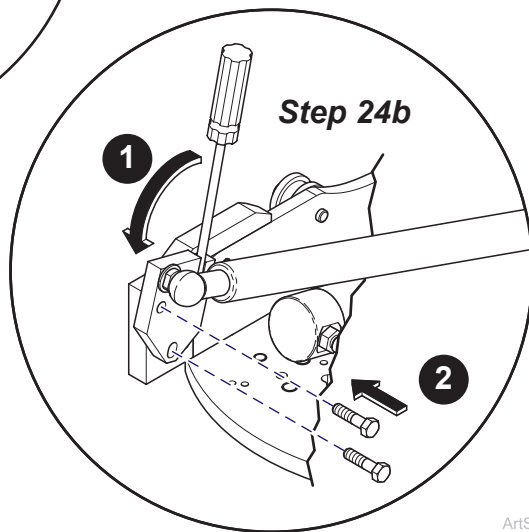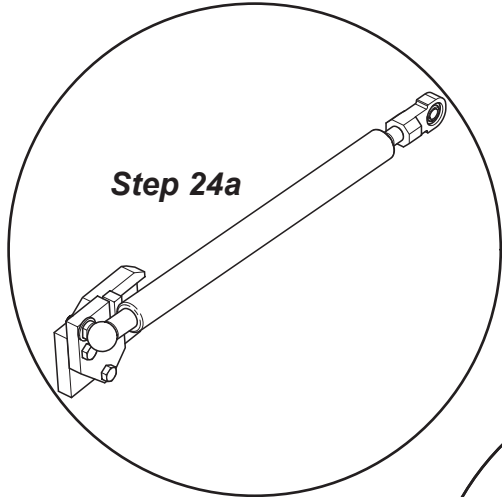- A) To replace spring go to kit number 002-10439-00 instruction manual 003-10274-00.

Caution

Move back to be down completely. Retraction spring under tension.

Step 24b: Reinstall retraction spring.

- A) Rotate retraction spring back into position.
- B) Install bolts (2) and use screwdriver to help rotate.


Note: If chair back does not move check pivot switch connections.


ArtSU3

ArtFull

Step 25: Console mounted on Patient Right - (ONLY).

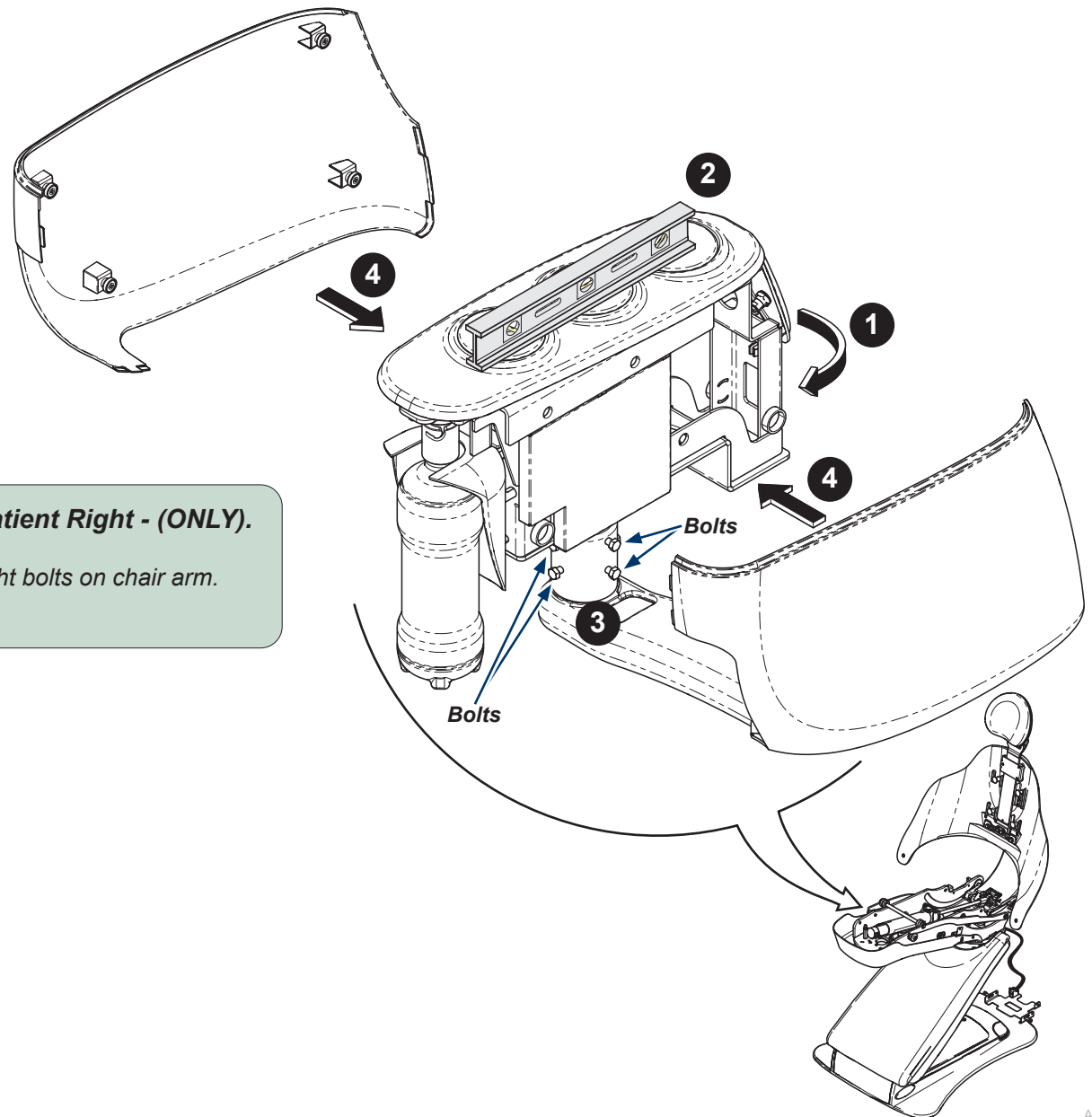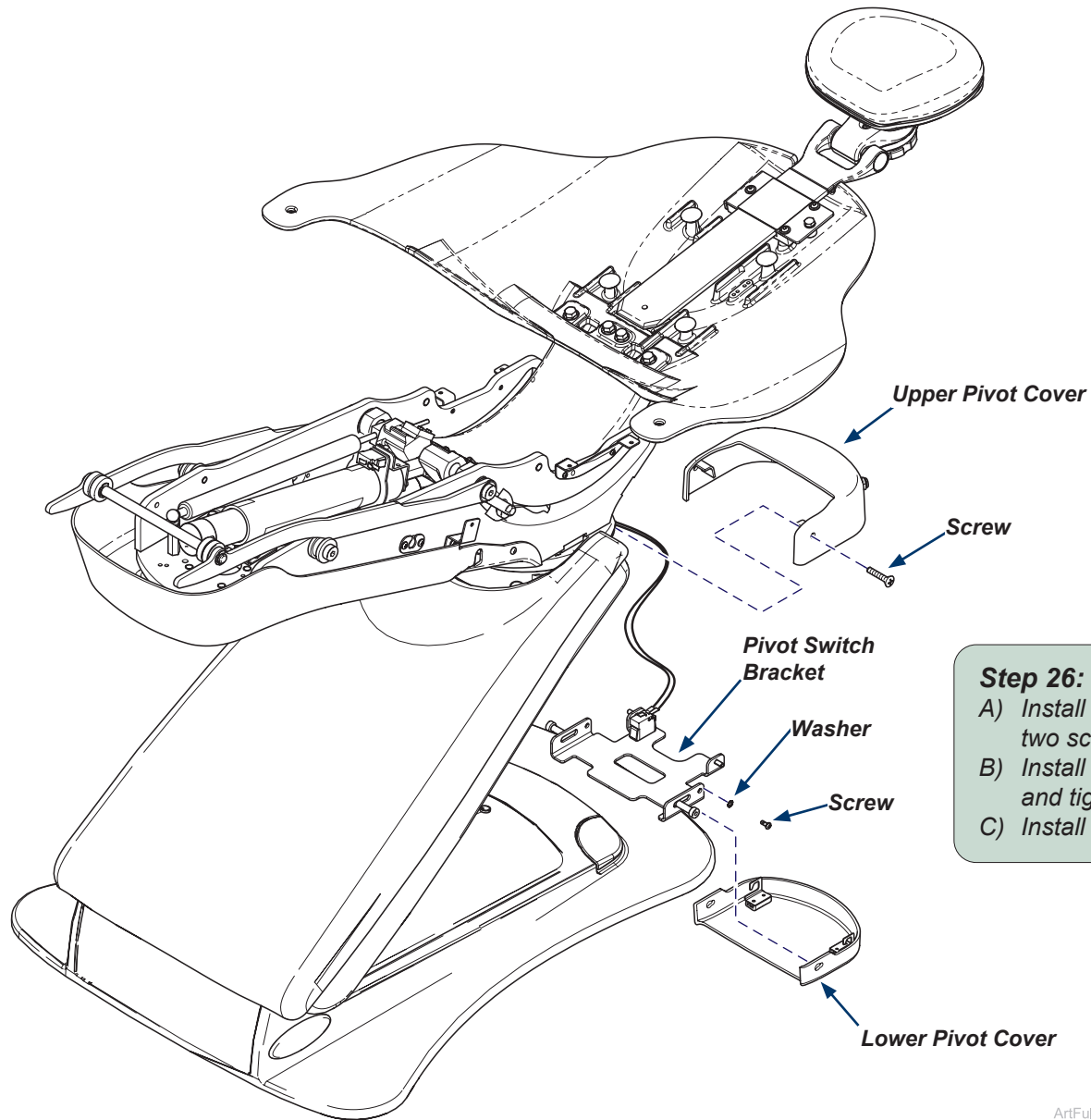- A) Rotate console back into position.
- B) Level console before tightening eight bolts on chair arm.
- C) Install console covers.


Step 26: Install Pivot Covers and Bracket.
 A) Install pivot switch bracket with two lock washers, two screws, and two shoulder screws.
 B) Install lower pivot cover onto two shoulder screws and tighten.
 C) Install upper pivot cover with two screws.

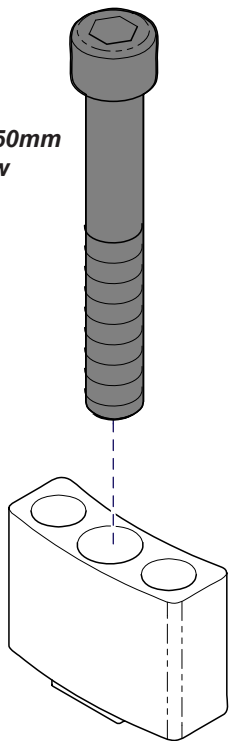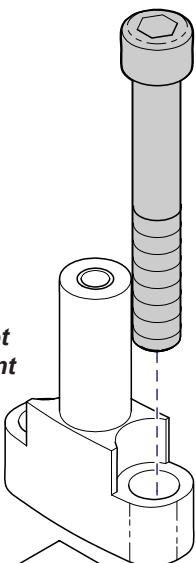
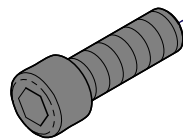
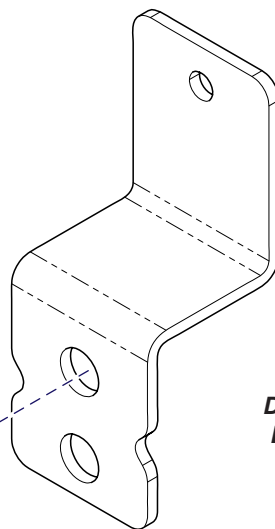
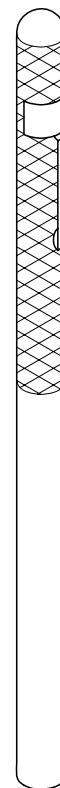
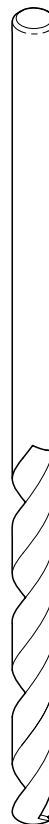
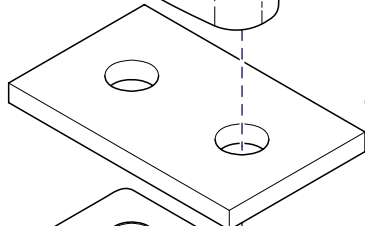
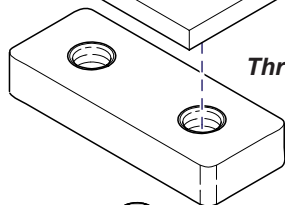
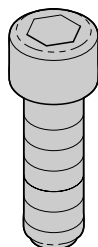
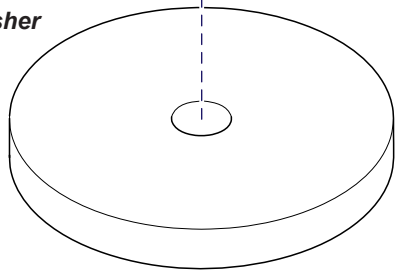
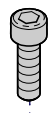
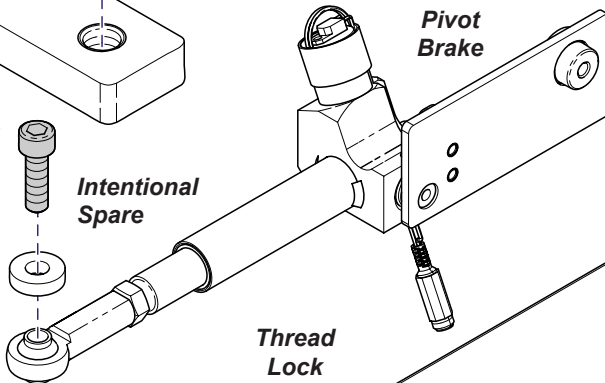
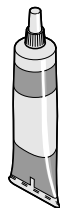
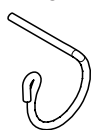
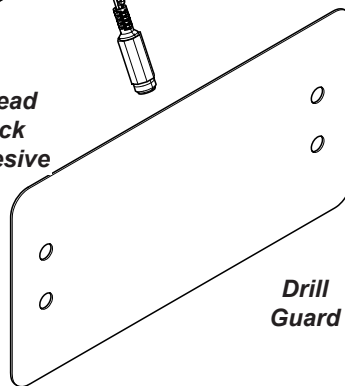
Step 27: Finalize installation.
 A) Install any remaining covers/components (Steps 5 through 2).
 B) Connect chair to power source.
 C) Ensure power is restored for hydraulic functions.
 D) Verify operation.

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