

Midmark® 1000 Solenoid Kit (002-10452-00 & 002-1148-00)



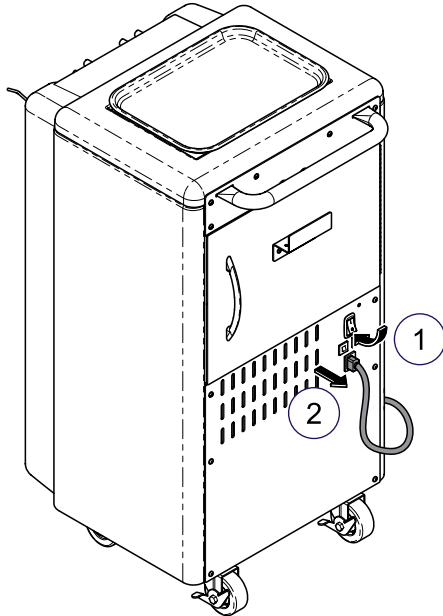
Step 1:

Turn off and unplug unit.



Caution

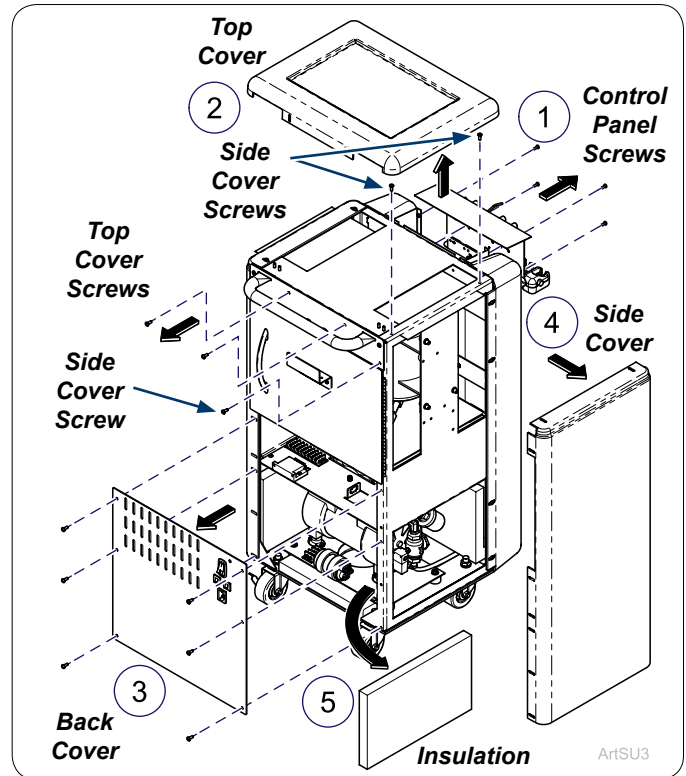
Always disconnect power from unit before opening or removing covers to perform any service to unit.



ArtSU3

Step 2:

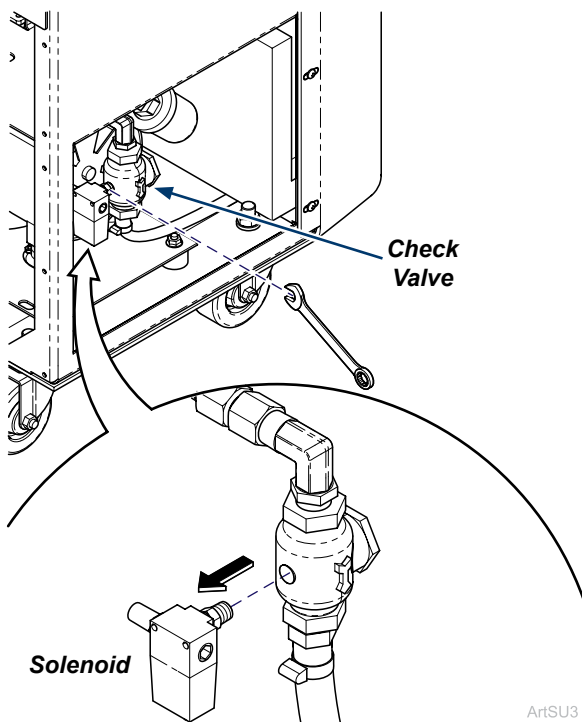
Remove screws (4) from control panel, screws (2) and top cover, screws (6) and back cover, screws (3) and side cover & insulation.



ArtSU3

Step 3:

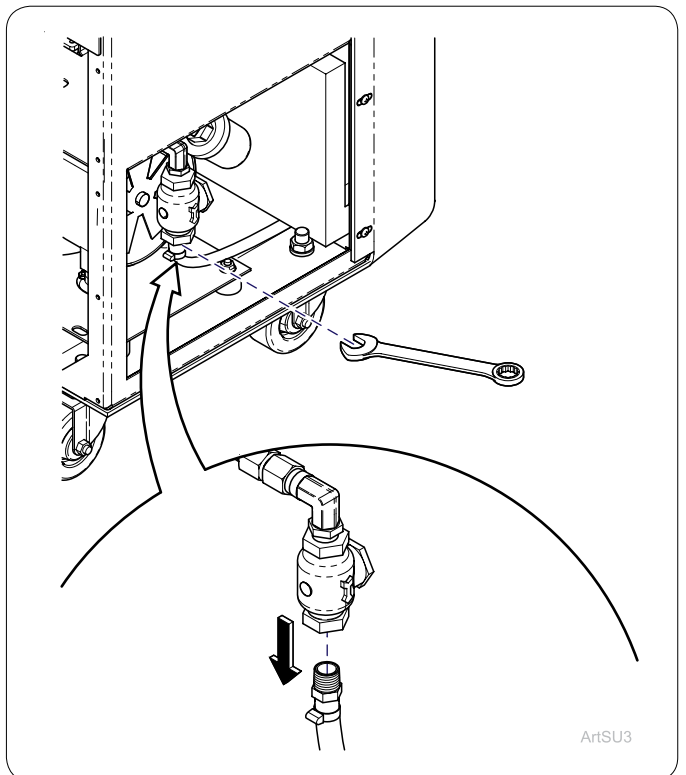
Unplug and remove solenoid and pipe nipple from check valve and set aside.



ArtSU3

Step 4:

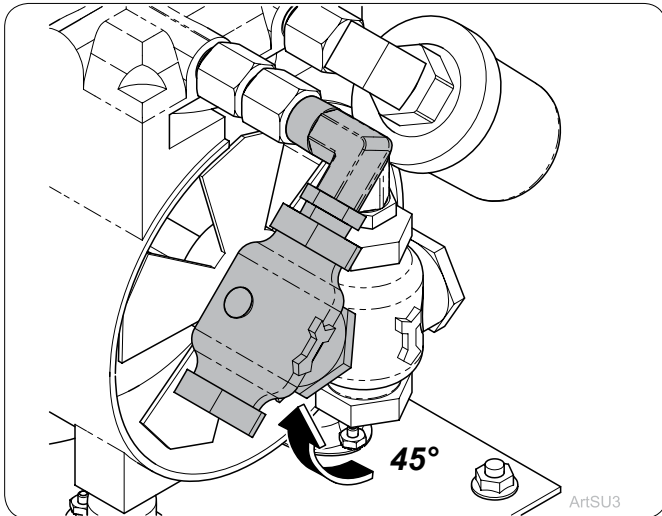
Remove barb fitting and tubing from check valve.



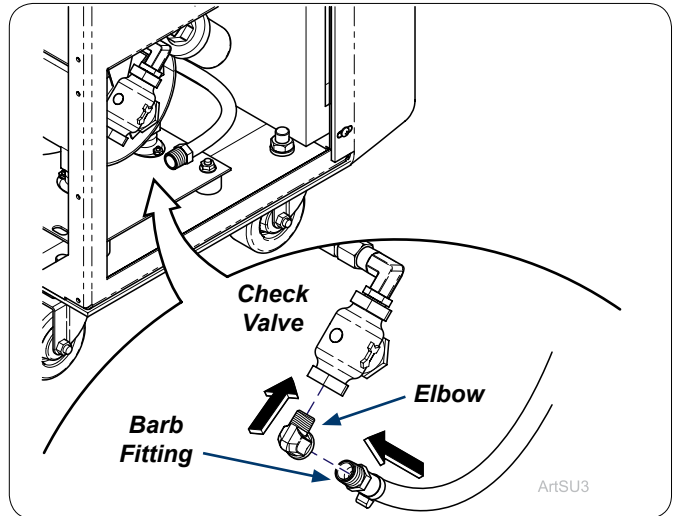
ArtSU3

Step 5:

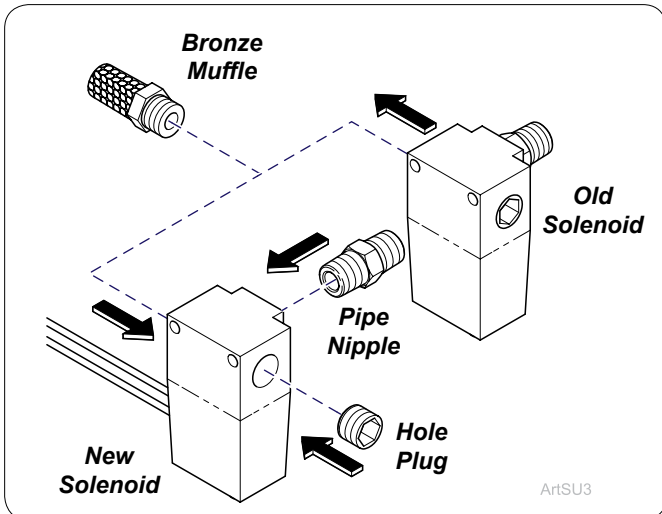
Adjust elbow and check valve by rotating roughly 45 degrees as shown.

**Step 6:**

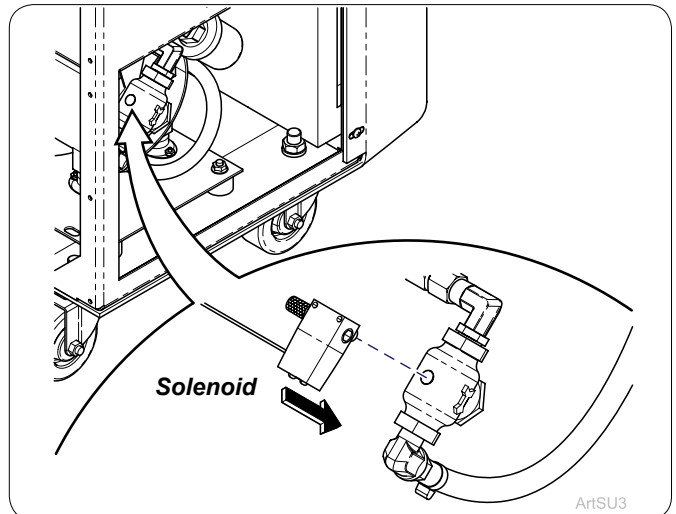
Install elbow from kit into check valve and install barb fitting with tubing onto elbow.

**Step 7:**

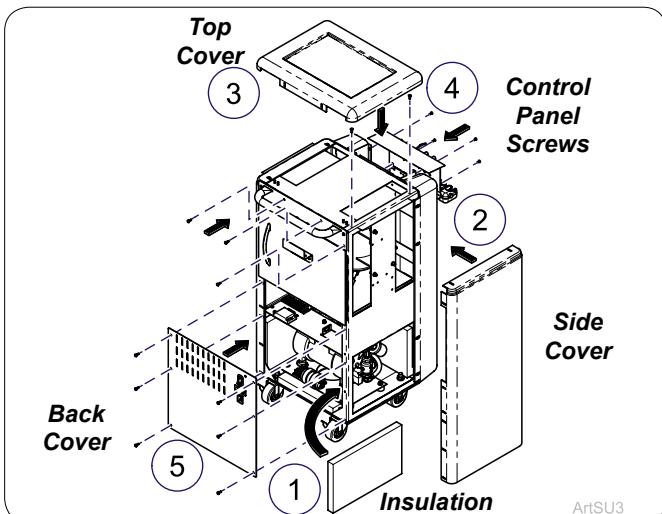
Assemble pipe nipple and hole plug onto solenoid. Remove bronze muffler from old solenoid and install onto new solenoid.

**Step 8:**

Install solenoid with pipe nipple to check valve.

**Step 9:**

Install insulation, side cover with screws (3) top cover with screws (2), control panel with screws (4), & back cover with screws (6).

**Step 10:**

Plug in unit and turn on.

