# Knee Crutches / Cladding

9A410 / 9A411 / 9A412 / 9A413

[English](#)

[Deutsch](#)

[Français](#)

[Español](#)

[Dansk](#)

[Suomi](#)

[Italiano](#)

[Svenska](#)

[Norsk](#)

[Nederlands](#)

[Polski](#)

[Türkçe](#)

## **Applies to Models:**

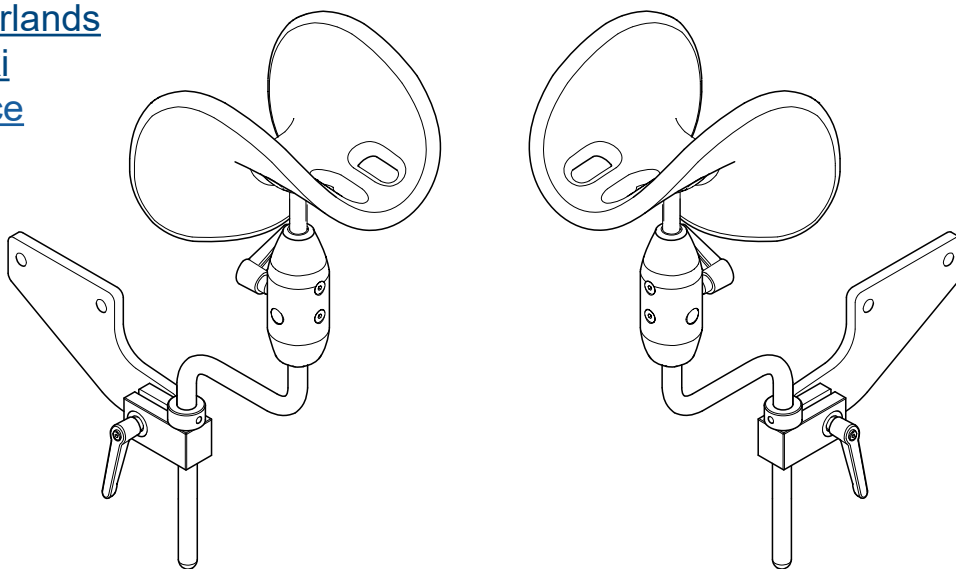
204 / 222 / 223

604 / 622 / 623 / 625

230 / 630

## **Special Tools:**

*none*

# Set Up / User Guide

Style U

003-10286-99 Rev. AA5 (10/11/23)

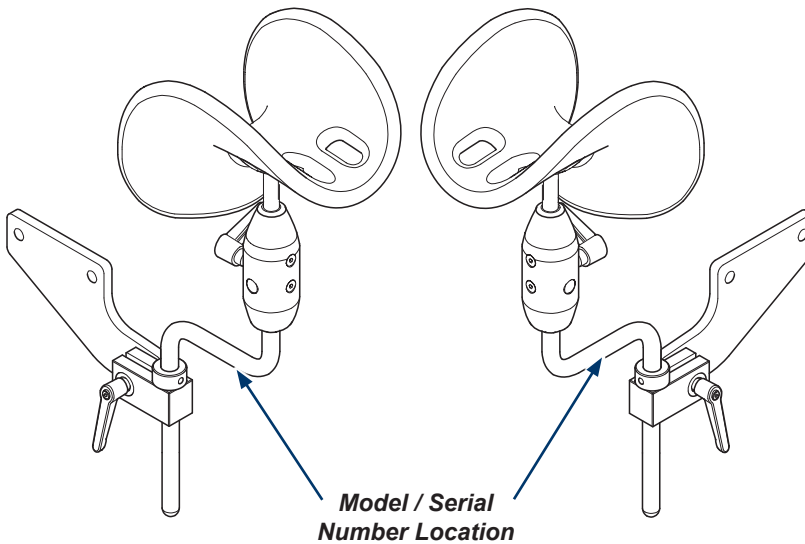
# Product Information

**Dealer:**

**Date of Purchase:**

**Model / Serial Number:**

**Midmark Authorized Service Company:**

# Table of Contents

## **Important Information**

<u>Product Information</u> .....	2
<u>Product Registration</u> .....	4
<u>Operating Conditions</u> .....	4
<u>Disposal of Equipment</u> .....	4
<u>Authorized Representatives</u> .....	4
<u>Important Information</u> .....	6
<u>Intended Use</u> .....	6
<u>Safety Instructions</u> .....	6

## **Set Up**

<u>Cladding Removal (models with cladding)</u> .....	7
<u>Knee Crutch Brackets (204 / 222 / 223 / 604 / 622 / 623 / 625)</u> .....	8
<u>Knee Crutch Brackets (230 / 630)</u> .....	9
<u>Knee Crutches (all models)</u> .....	10

## **Operation**

<u>Knee Crutches (all models)</u> .....	11
---	----

## **Maintenance**

<u>Calling for Service</u> .....	12
<u>Cleaning</u> .....	12
<u>Periodic Maintenance</u> .....	12

## **Warranty Information**

<u>Limited Warranty</u> .....	13
-------------------------------	----

# Product Registration

## Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)  
Relative Humidity.....30% to 75% (non-condensing)  
Altitude.....3000m or less

## Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

## Authorized Representatives

### WARNING

*In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.*

Customers in the EU should direct all questions, incidents, and complaints to Midmark’s Authorized Representative listed below.

CEpartner4U  
Esdoornlaan 13  
3951 DB Maarn, The Netherlands  
Phone: +31 343 442 524  
Fax: +31 343 442 162

Customers in the UK should direct all questions, incidents, and complaints to Midmark’s UK Responsible Person listed below.

UKCApartner4U Ltd.,  
7 CAMPION WAY, BINGHAM, NOTTINGHAM NG13 8TR, UK.  
www.UKCApartner4U.co.uk  
Phone : +44 7905 384429

Customers in Australia should direct all questions, incidents, and complaints to Midmark’s Sponsor listed below.

ICONA Pty Ltd.  
1A/2A Westall Road  
Clayton, Victoria 3168  
Phone: 1300 442 662

## ***Authorized Representatives - continued***

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India  
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)  
Mumbai, 400022  
Tel: +91 22 4915 3000  
Fax: +91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd  
Abdullah M. Al Khalifi Street  
Al Nakheel Dist. II  
P.O. Box 116105  
Jeddah (East)  
21391  
Saudi Arabia  
Tel: + 966 2 286 4707  
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.  
Al Barsha,  
Dubai, UAE  
Phone: +971 4 501 4000  
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Associated Medical Supplies Company LTD. 沒有中文版使用說明  
Room 1201-1202, 12/F, Fo Tan Industrial Center  
26-28 Au Pui Wan Street, Fo Tan, Shatin  
New Territories, Hong Kong  
Phone: +852 2604 9389  
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading  
Midmac Roundabout  
Opposite Al Mana Petrol Station  
Doha, Qatar  
Tel: +974.44337000  
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.  
HaTnufa St 7  
Petah Tikva, Israel  
Tel: +972 3.925.4040  
Fax: +972 3.924.9977

# Important Information - Safety Symbols

## **WARNING**

*Indicates a potentially hazardous situation which could result in serious injury.*

## **Caution**

*Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices*

## **Equipment Alert**

*Indicates a potentially hazardous situation which could result in equipment damage.*

## **Note**

*Amplifies a procedure, practice or condition.*

## Symbol Glossary

Manufacturer

Serial Number

Medical Device

Catalogue Number

## Intended Use

To support patient's knees / legs during examinations & procedures.

## Safety Instructions

## **WARNING**

*Always disconnect power to chair before removing covers or performing any service procedures.*

## **WARNING**

*No modification of this equipment is allowed.*

## **WARNING**

*Do not sit on Knee Crutches.*

## **WARNING**

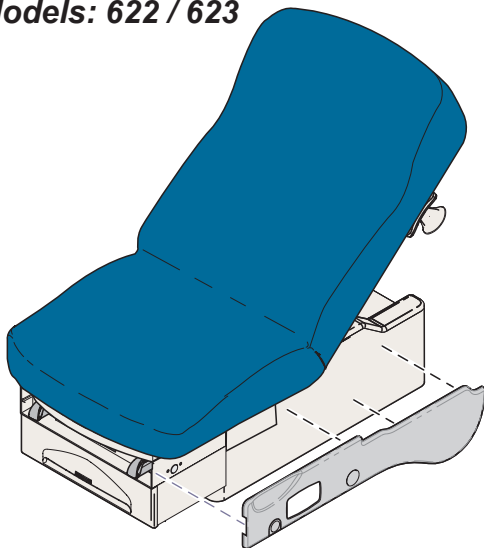
*Do not use Knee Crutches as a gripping surface during ingress egress.*

# Set Up - Cladding Removal (models with cladding)

## Note


Remove all accessories (ex. chair arms) before performing this procedure.

## Models: 622 / 623

622 / 623 cladding is magnetic.  
Pull outward to remove.

## Models: 625
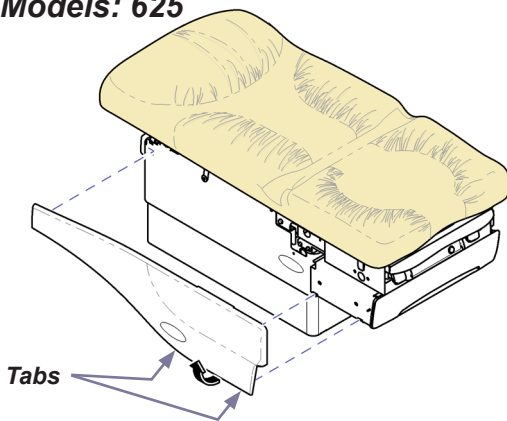
### To remove 625 cladding:

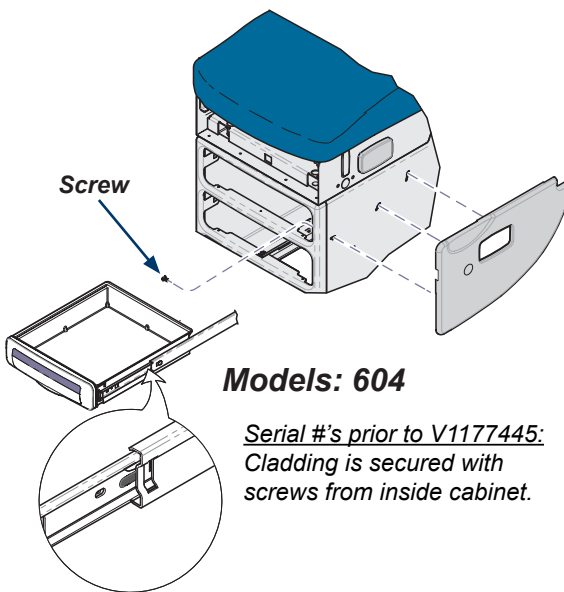
- Press two tabs.
- Pull bottom of cladding away from chair.
- Lift cladding up to remove.

## Requirements:

604 / 622 / 623 models require a special cladding kit to accommodate the knee crutch brackets. Refer to Midmark pricing catalog to determine proper kit.

## Note

The cladding on 604 models with serial numbers V1177445 thru present is magnetic. It is **not** necessary to remove drawers / screws.

## Models: 604

Serial #'s prior to V1177445:  
Cladding is secured with screws from inside cabinet.

### To remove 604 cladding...

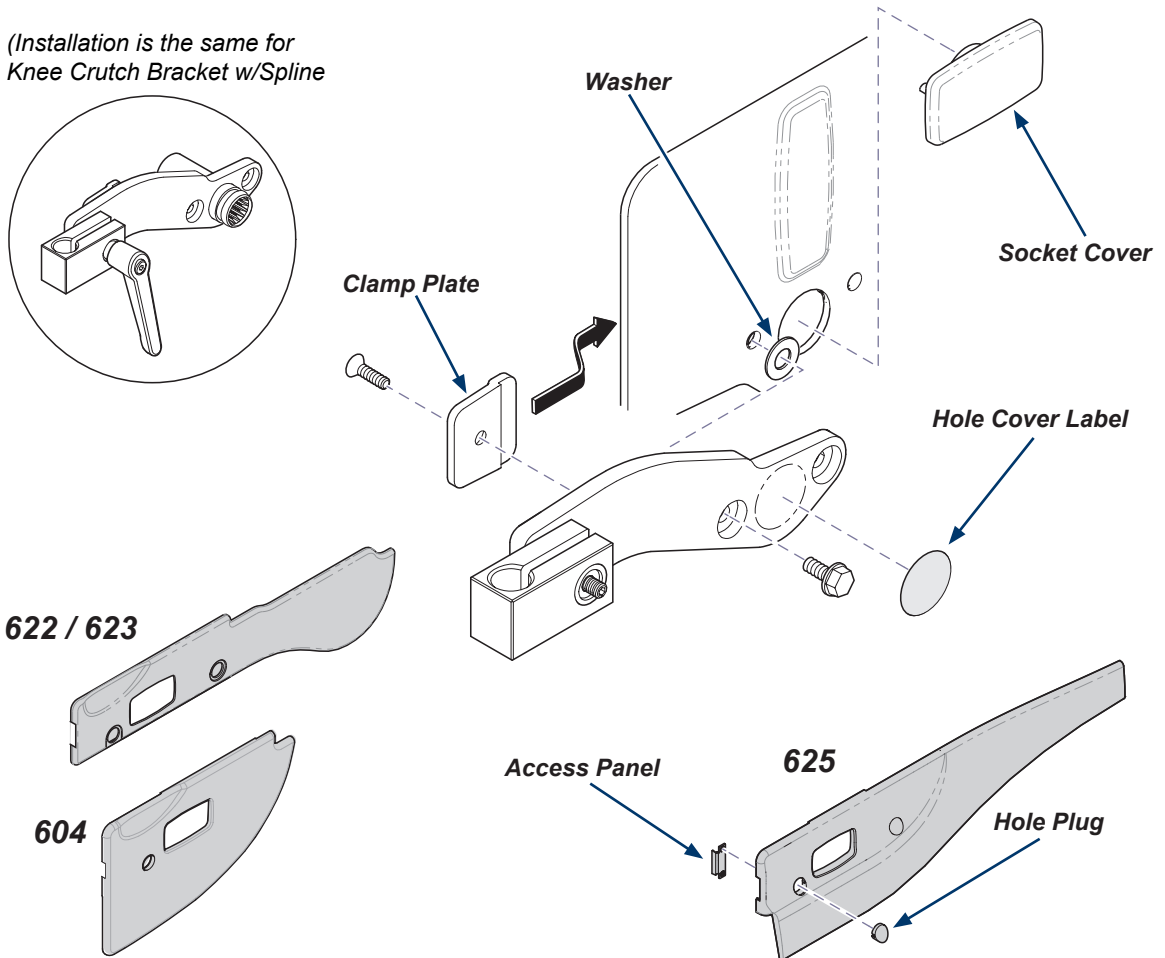
- Pull drawer out.
- Press RH tab Up and LH tab Down and remove drawer.
- Remove screws and cladding.

# Set Up - model's 204 / 222 / 223 / 604 / 622 / 623 / 625

## Note

Remove all accessories (ex. chair arms) before performing this procedure.

(Installation is the same for  
Knee Crutch Bracket w/Spline)


- A) Remove socket cover. (If present)
- B) Position clamp bracket & washers. Secure with two hex screws. Apply hole cover label if desired. Repeat for other bracket.
- C) Position clamp plates with offset against upperwrap as shown, then secure clamp plates to brackets with screws.
- D) (604 / 622 / 623) Install notched side cladding. Note: If installing brackets with splines, you must remove hole plug from the side cladding.
- E) (625) Remove access panels. Install side cladding. Note: If installing brackets with splines, you must remove hole plugs from the side cladding.

# Set Up - model's 230 / 630
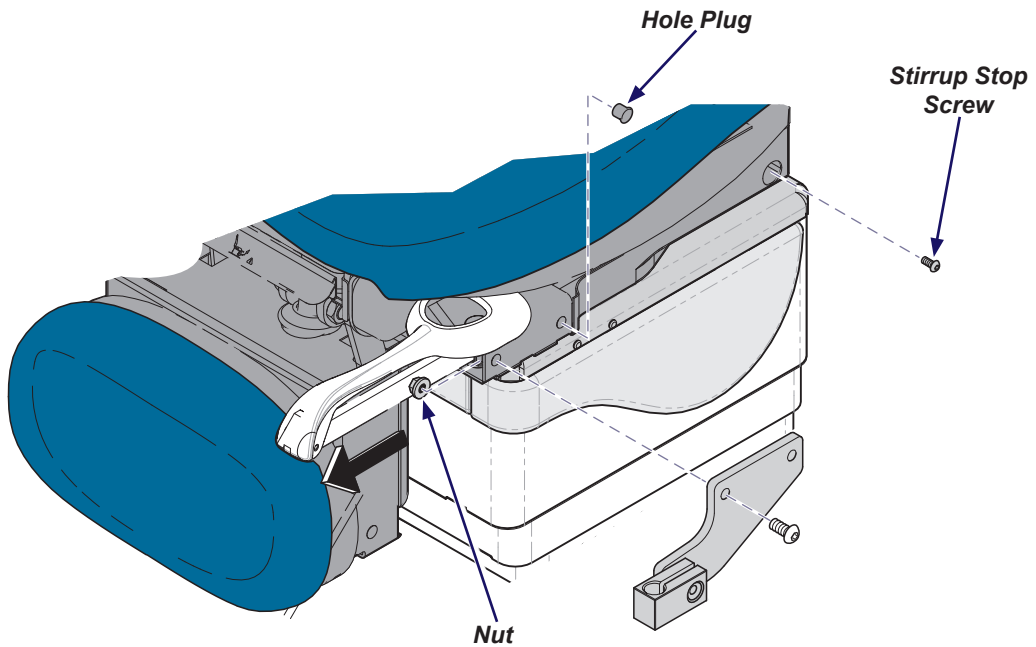
## Note

Remove all accessories (ex. chair arms) before performing this procedure.


- A) Remove two hole plugs.
- B) Remove stirrup stop screw and stirrup assembly.
- C) Position clamp bracket. Secure with two screws / nuts. Repeat for other bracket.
- D) Install stirrup assembly, and stirrup stop screw.

## Set Up - Knee Crutches (all models)
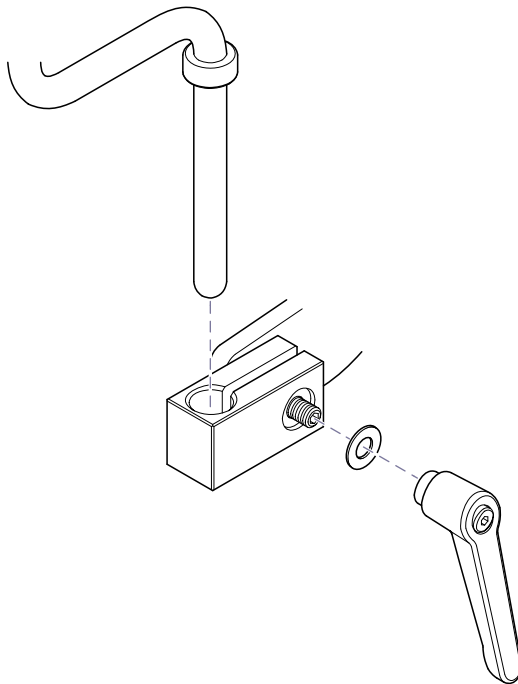
### To install Knee Crutch...



- A) Install knob / washer as shown.  
(Do not tighten knob)
- B) Slide knee crutch into block.
- C) Tighten knob to secure knee crutch.

# Operation - Knee Crutches (all models)

## To Position Knee Crutch...

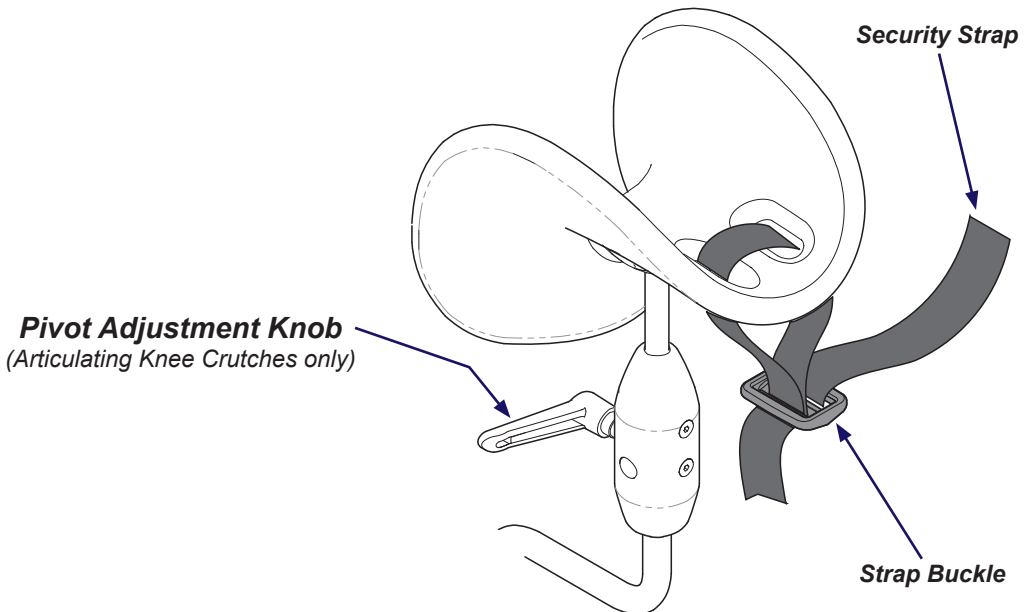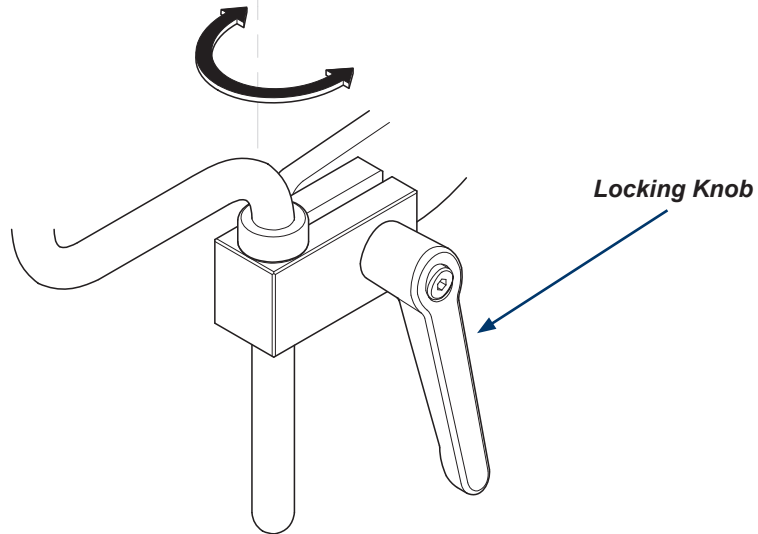- A) Loosen locking knob to position knee crutch.
- B) Tighten locking knob to secure knee crutch.

# Maintenance

## Calling for Service

### Note

*Model / serial number information is required when calling for service.*

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

[midmark.com](http://midmark.com)

## Cleaning

### Equipment Alert

*The Knee Crutches are resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Knee Crutches.*

### Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

## Periodic Maintenance

Periodically inspect the following areas:

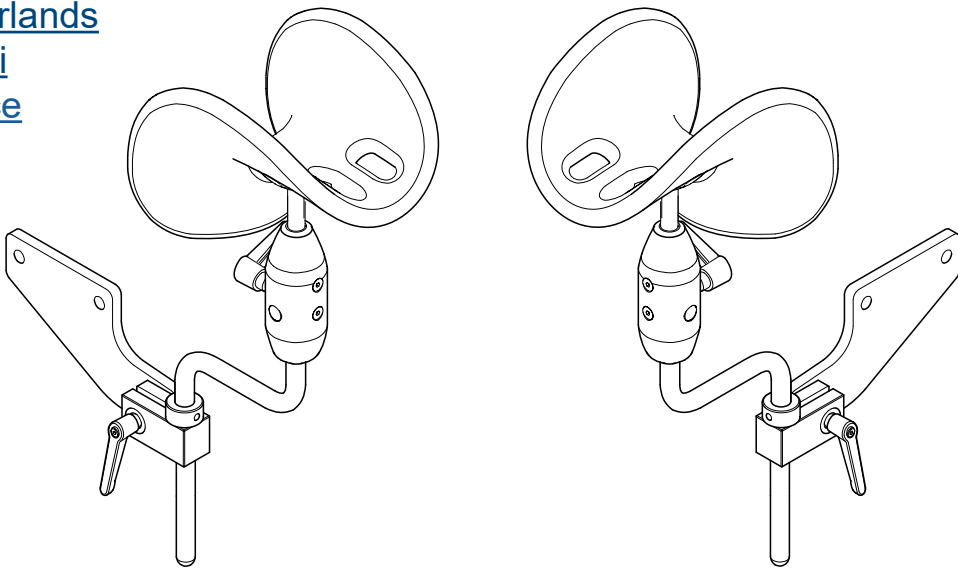
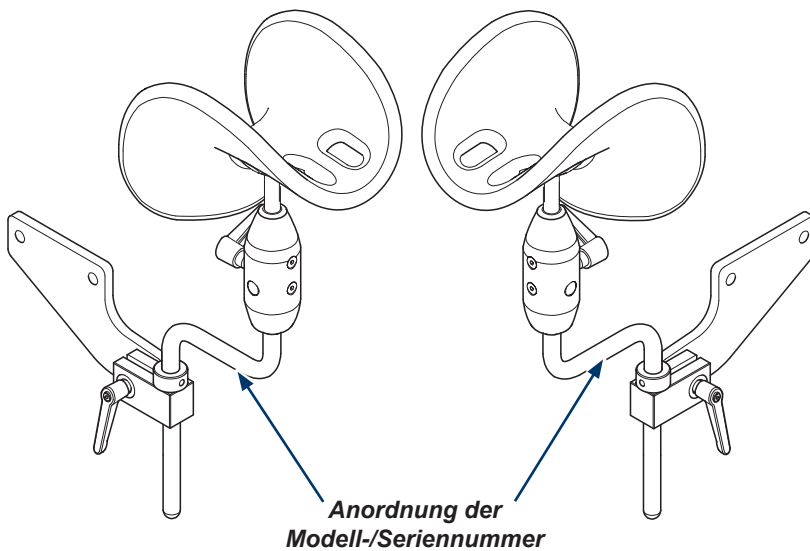
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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**Midmark Corporation**