Knee Crutches

9A641 / 9A643 / 9A645 / 9A678 / 9A679

[English](#)

[Deutsch](#)

[Français](#)

[Español](#)

[Dansk](#)

[Suomi](#)

[Italiano](#)

[Svenska](#)

[Norsk](#)

[Nederlands](#)

[Polski](#)

[Türkçe](#)

Applies to Models:

224 (-001 /-002 /-003 /-011)

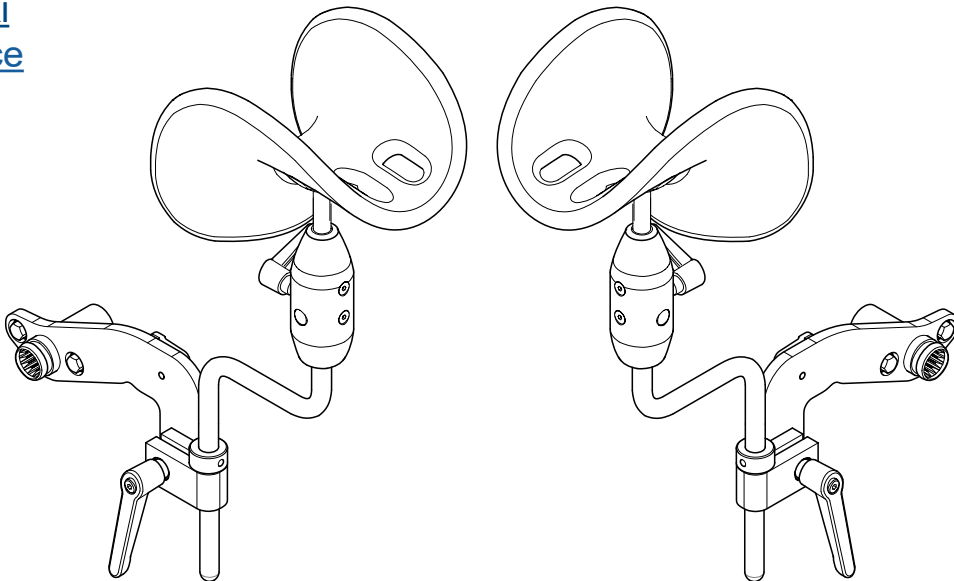
225 (-002 /-003)

626 (-001 /-002 /-003 /-004 /-005 /-006)

627 (-011)

Special Tools:

none

Set Up / User Guide

Style U

003-10289-99 Rev. AA6 (8/31/23)

TP202 20-42-FO-00014 Rev A1 C2169

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

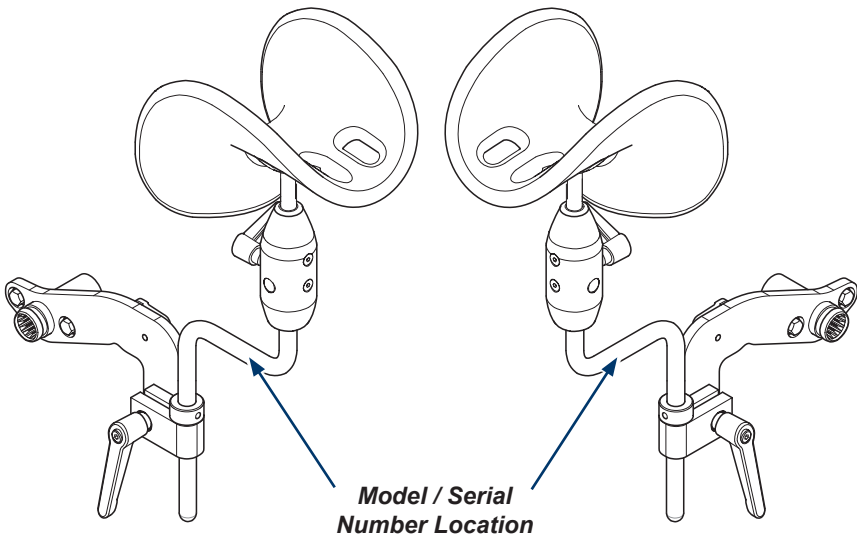
Table of Contents

Important Information

- [Product Information](#) 2
- [Product Registration](#) 4
- [Operating Conditions](#) 4
- [Disposal of Equipment](#) 4
- [Authorized Representatives](#) 4
- [Important Information](#) 6
- [Intended Use](#) 6
- [Safety Instructions](#) 6

Set Up

- [Cladding Removal \(224 / 225\)](#) 7
- [Cladding Removal \(626\)](#) 9
- [Cladding Removal \(627\)](#) 11
- [Knee Crutches \(all models\)](#) 12

Operation

- [Knee Crutches \(all models\)](#) 14

Maintenance

- [Calling for Service](#) 15
- [Cleaning](#) 15
- [Periodic Maintenance](#) 15

Warranty Information

- [Limited Warranty](#) 16

Product Registration

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in the EU should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

CEpartner4U
Esdoornlaan 13
3951 DB Maarn, The Netherlands
Phone: +31 343 442 524
Fax: +31 343 442 162

Customers in the UK should direct all questions, incidents, and complaints to Midmark's UK Responsible Person listed below.

UKCApartner4U Ltd.,
7 CAMPION WAY, BINGHAM, NOTTINGHAM NG13 8TR, UK.
www.UKCApartner4U.co.uk
Phone : +44 7905 384429

Customers in Australia should direct all questions, incidents, and complaints to Midmark's Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Authorized Representatives - continued

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel: +91 22 4915 3000
Fax: +91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Associated Medical Supplies Company LTD. 沒有中文版使用說明
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage.

Note

Amplifies a procedure, practice, or condition.

Symbol Glossary

Manufacturer

Serial Number

Medical Device

Catalogue Number

Intended Use

To support patient's knees / legs during examinations.

Safety Instructions

WARNING

No modification of this equipment is allowed.

WARNING

Do not sit on Knee Crutches.

WARNING

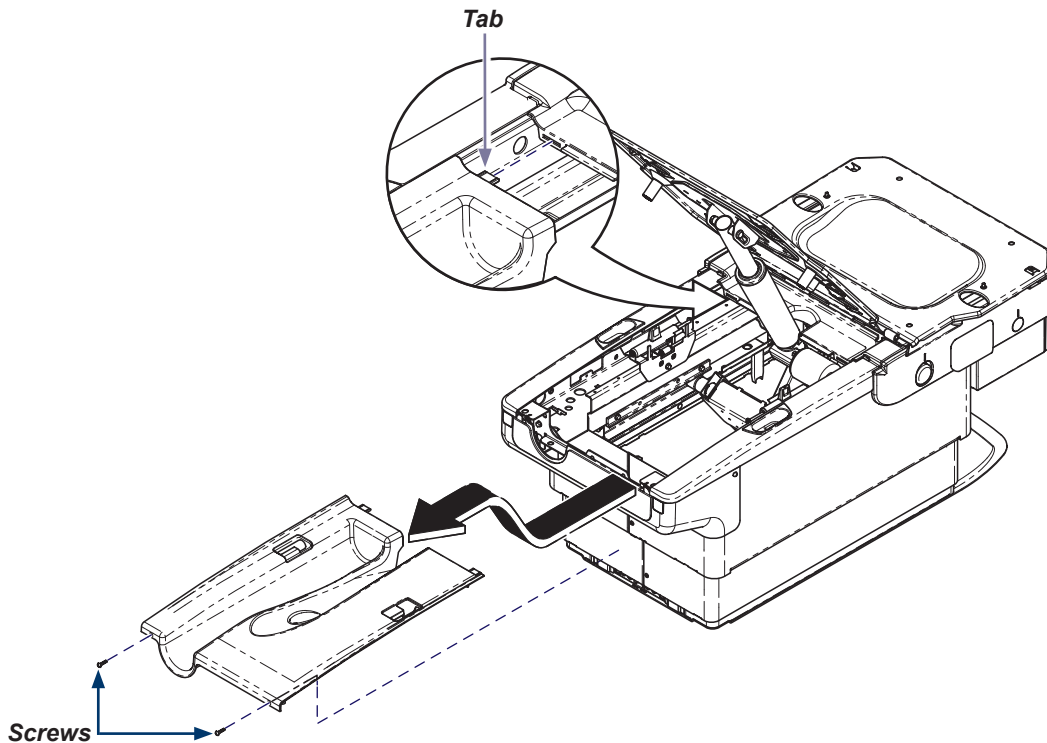
Do not use Knee Crutches as a gripping surface during ingress egress.

Set Up - Cladding Removal (224 / 225)

WARNING

Always disconnect power to chair before removing covers or performing any service procedures.

Step 1: Remove Top Cover:

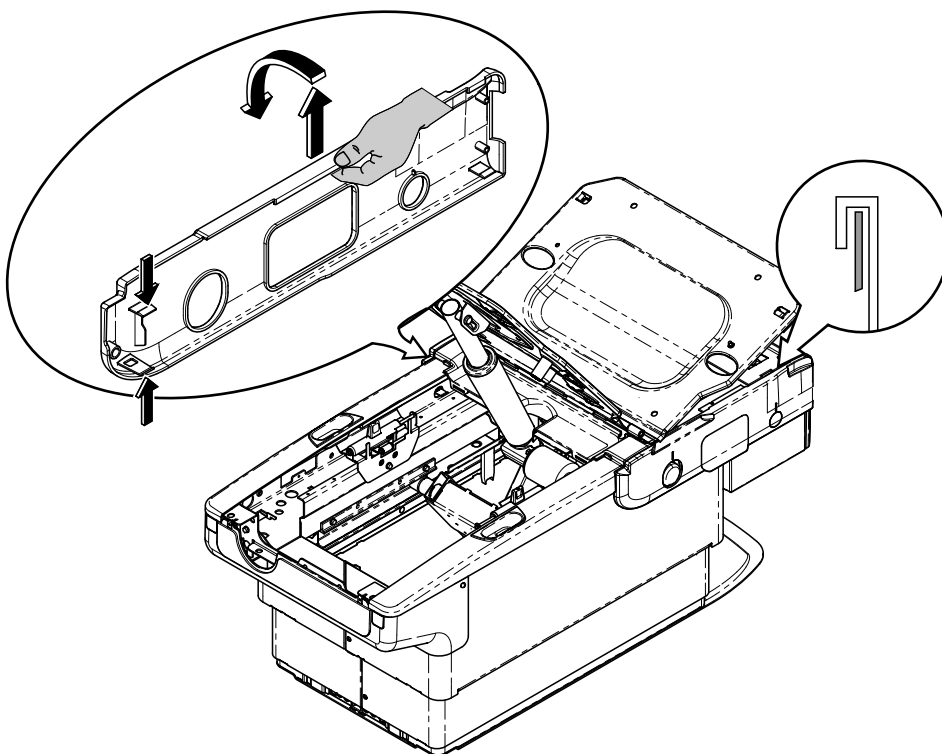
A) Remove two screws from back of top cover.

B) Pull cover toward head end to disengage two tabs, then lift up to remove cover.

Cladding - continued (224 / 225)

Note

Remove all accessories (ex. chair arms) before performing this procedure.

Step 2: Remove Side Cladding:

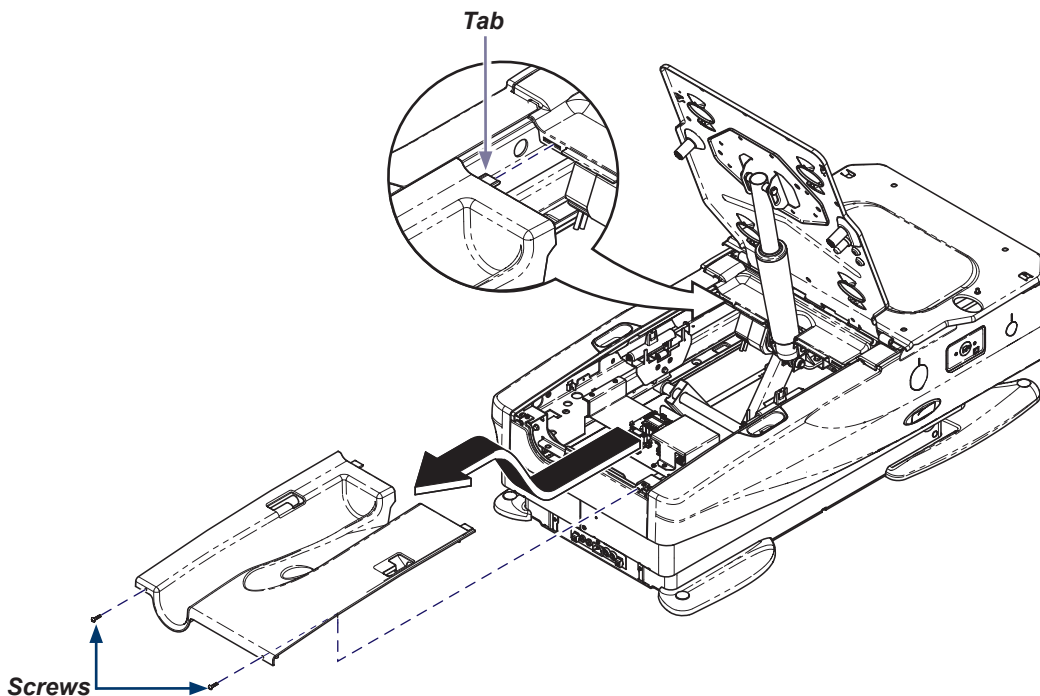
- A) Locate tabs inside at center of chair (foot rest must be extended) push down on top tab and up on bottom tab, pull cladding away from chair.
- B) Pull up on cladding overhang, and remove.
- C) Repeat process for opposite side.

Set Up - Cladding Removal (626)

WARNING

Always disconnect power to chair before removing covers or performing any service procedures.

Step 1: Remove Top Cover:

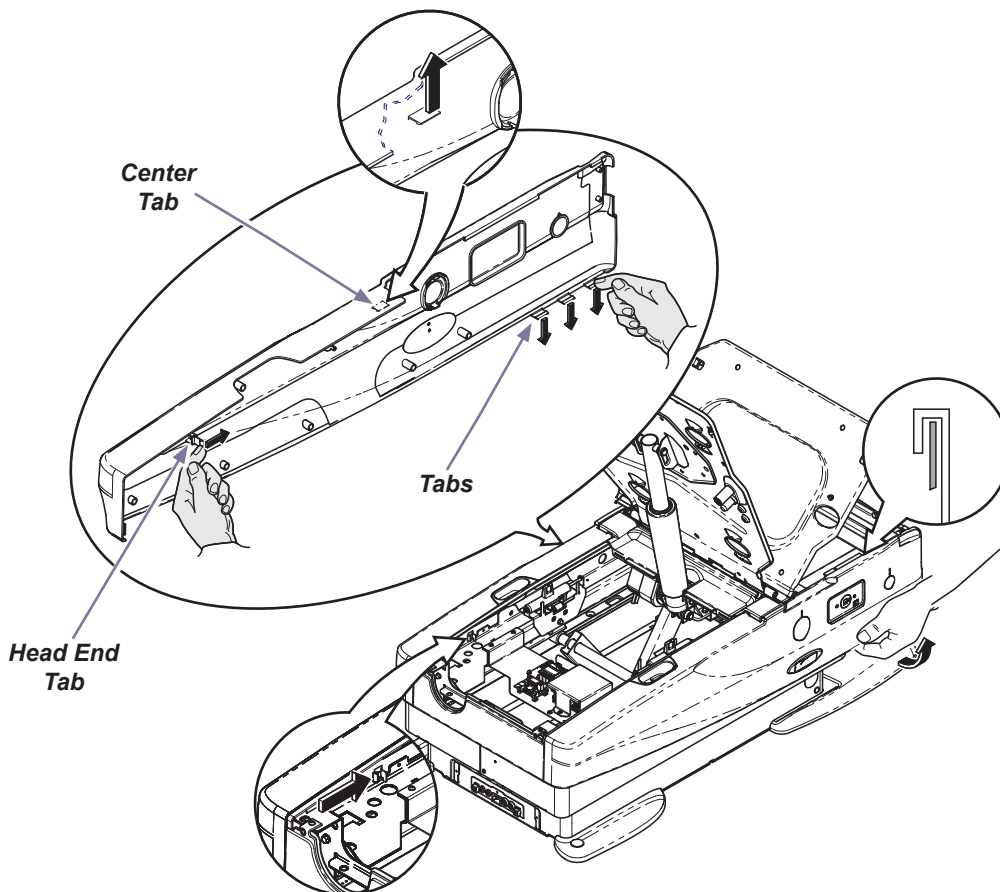
A) Remove two screws from back of top cover.

B) Pull cover toward head end to disengage two tabs, then lift up to remove cover.

Cladding - continued (626)

Note

Remove all accessories (ex. chair arms) before performing this procedure.

Step 2: Remove Side Cladding:

- Locate tab inside head end of chair, push tab toward foot end of chair, pull cladding away from chair.
- Pull down on three tabs located at the bottom foot end of the cladding and pull cladding away from chair.
- Locate tab inside at center of chair (foot rest must be extended) push tab up and pull cladding away from chair.
- Push up on Cladding, tip up and remove.
- Repeat process for opposite side.

Set Up - Cladding Removal (627)

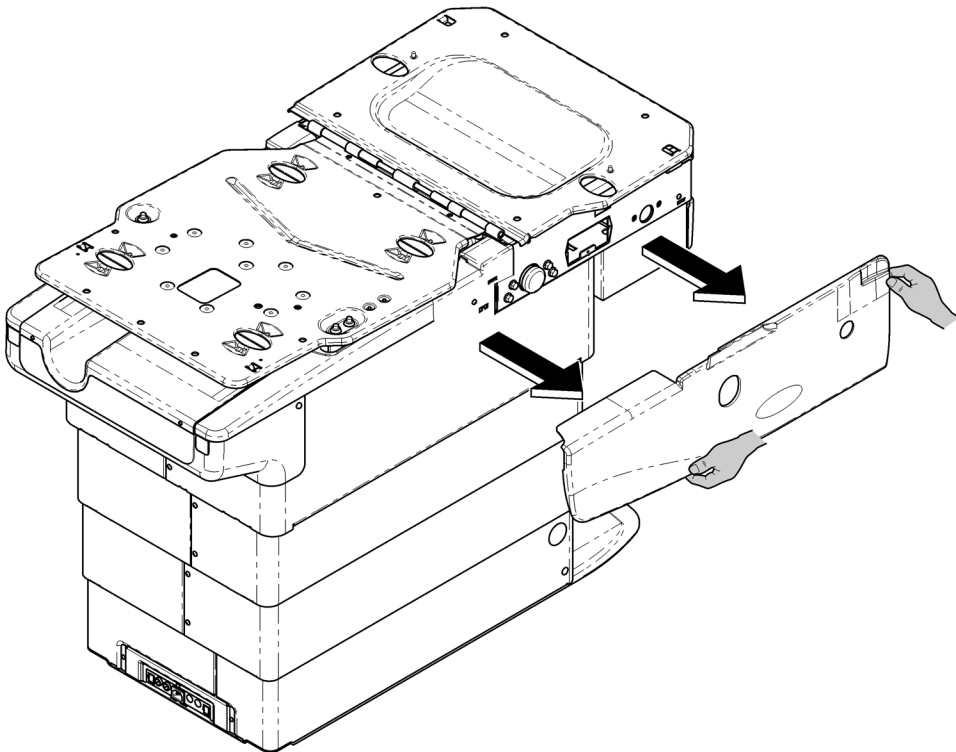
WARNING

Always disconnect power to chair before removing covers or performing any service procedures.

Note

The 627 cladding is secured with magnets only.


To Remove Side Cladding:

Grasp cladding firmly and pull outward.

Set Up - Knee Crutches (all models)

Step 1:

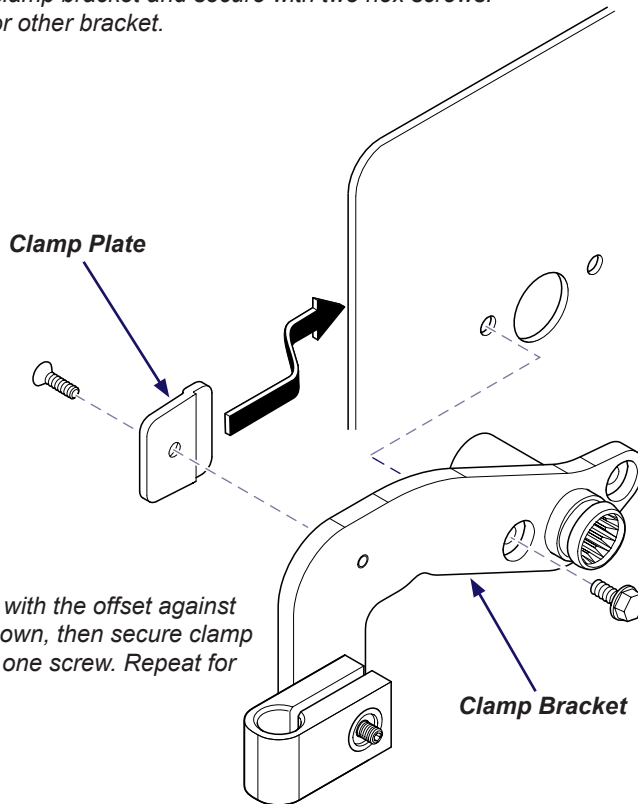Position clamp bracket and secure with two hex screws.
Repeat for other bracket.


Step 2:

Position clamp plate with the offset against the upperwrap as shown, then secure clamp plate to bracket with one screw. Repeat for other bracket.

Step 3:

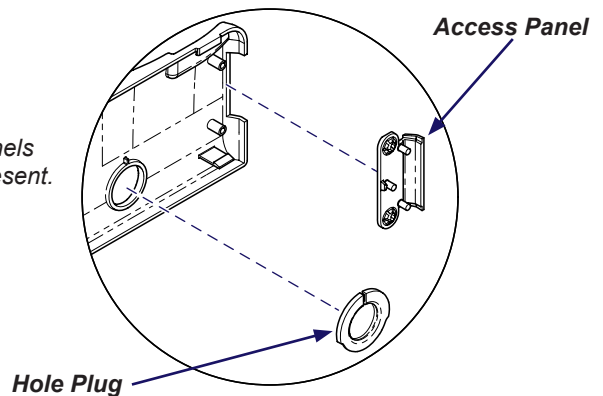
Remove access panels and hole plugs if present.
Install side cladding.

Set Up - Knee Crutches - continued (all models)

To install Knee Crutch...


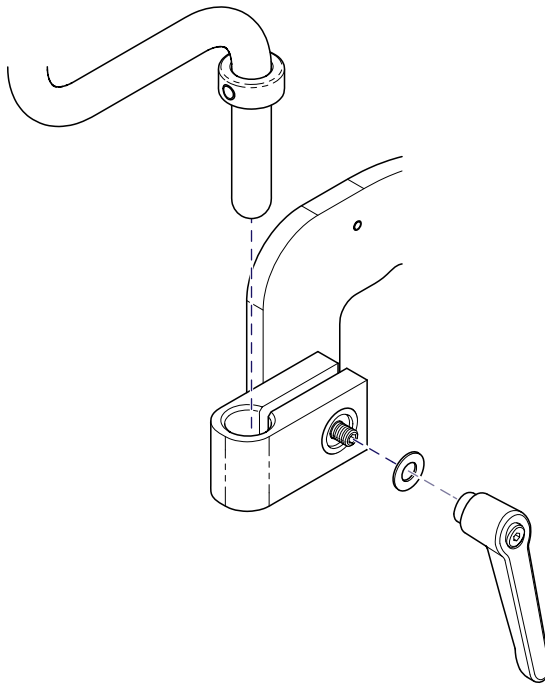
- A) Install knob / washer as shown.
(Do not tighten knob)*
- B) Slide knee crutch into block.*
- C) Tighten knob to secure knee crutch.*

Operation - Knee Crutches (all models)

To Position Knee Crutch...

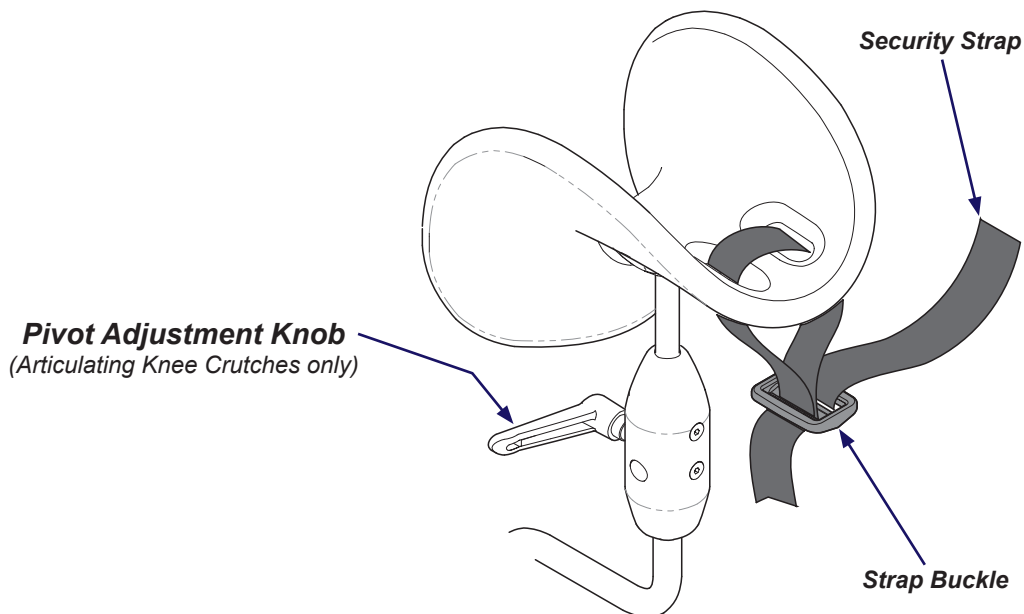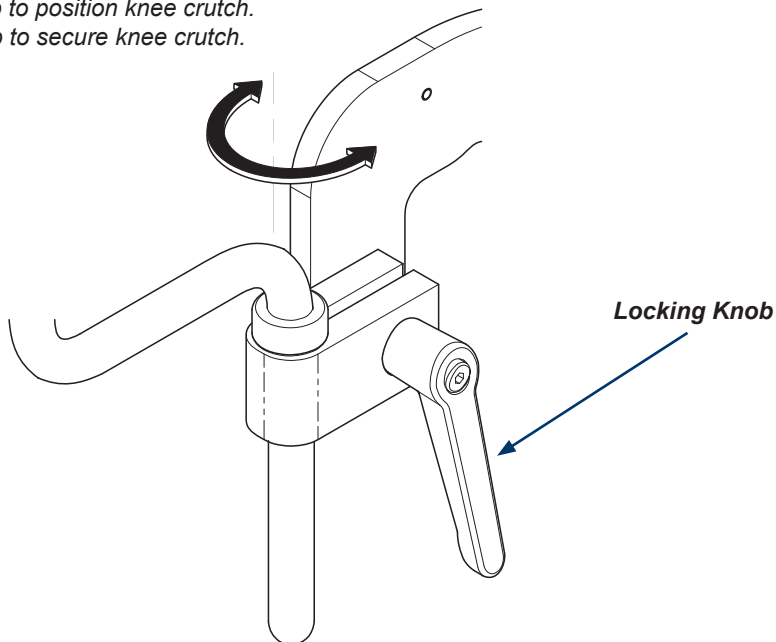- A) Loosen locking knob to position knee crutch.
- B) Tighten locking knob to secure knee crutch.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM
Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The Knee Crutches are resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Knee Crutches.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

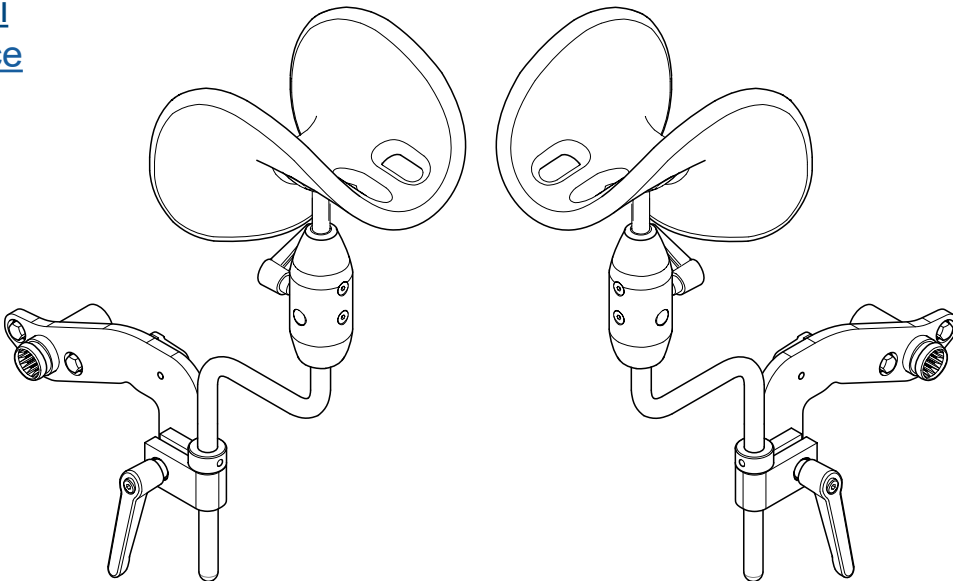
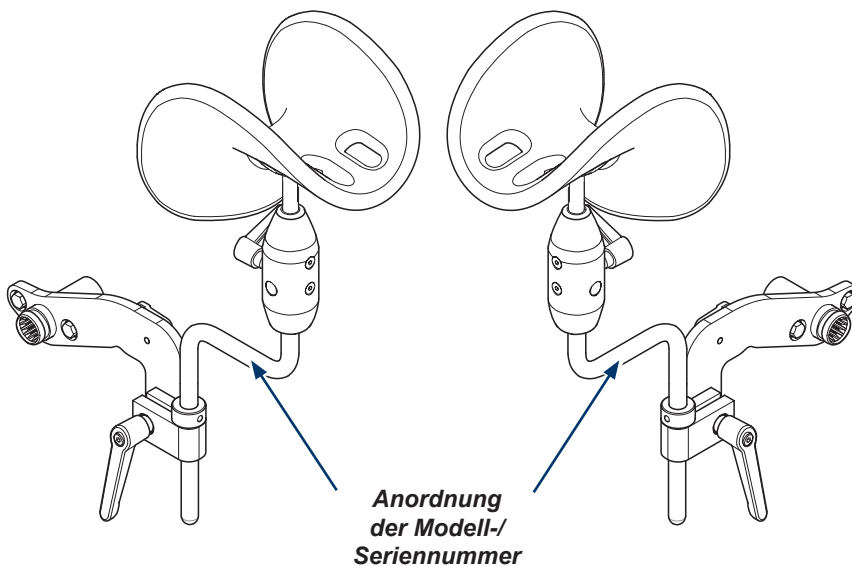
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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© Midmark Corporation 2019
Midmark Corporation