# Adjustable Arm System

9A314001 / 9A627001

[English](#)

[Deutsch](#)

[Français](#)

[Español](#)

[Dansk](#)

[Suomi](#)

[Italiano](#)

[Svenska](#)

[Norsk](#)

[Nederlands](#)

[Polski](#)

[Türkçe](#)

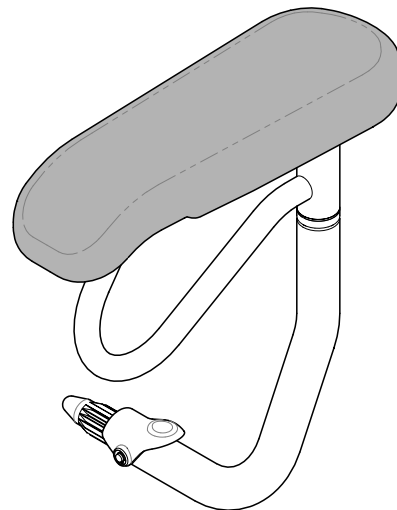
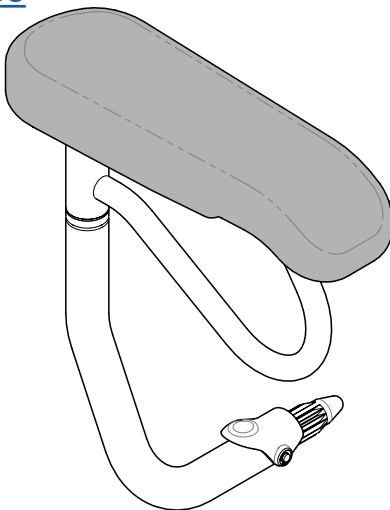
## Applies to Models:

204 / 222 / 223 / 244

604 / 622 / 623 / 625

## Special Tools:

none

## Set Up / User Guide

Style U

003-10290-99 Rev. AA6 (9/6/23)

TP202 20-42-FO-00014 Rev A1 C2169

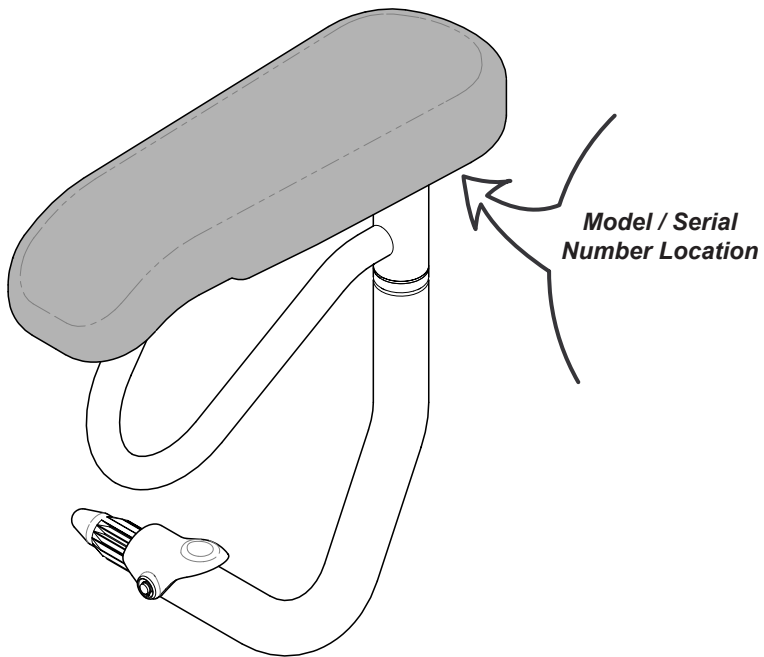
# Product Information

**Dealer:**

**Date of Purchase:**

**Model / Serial Number:**

**Midmark Authorized Service Company:**

# Table of Contents

## **Important Information**

<u>Product Information</u> .....	2
<u>Product Registration</u> .....	4
<u>Transportation / Storage Conditions</u> .....	4
<u>Operating Conditions</u> .....	4
<u>Disposal of Equipment</u> .....	4
<u>Authorized Representatives</u> .....	4
<u>Important Information</u> .....	6
<u>Intended Use</u> .....	7
<u>Requirements (Bracket sets):</u> .....	7
<u>Safety Instructions</u> .....	7

## **Set Up / Operation**

<u>Set Up / Operation (222, 223, 622, 623, 625)</u> .....	8
<u>Set Up / Operation (204 and 604)</u> .....	9
<u>Set Up / Operation (244)</u> .....	10

## **Maintenance**

<u>Calling for Service</u> .....	11
<u>Cleaning</u> .....	11
<u>Periodic Maintenance</u> .....	11

## **Warranty Information**

<u>Limited Warranty</u> .....	12
-------------------------------	----

# Product Registration

## Transportation / Storage Conditions

Ambient Temperature Range:.....-30°C to +60°C (-22°F to +140°F)

Relative Humidity.....10% to 90% (non-condensing)

## Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)

Relative Humidity.....30% to 75% (non-condensing)

Altitude.....3000m or less

## Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

## Authorized Representatives

### WARNING

*In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.*

Customers in the EU should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

CEpartner4U  
Esdoornlaan 13  
3951 DB Maarn, The Netherlands  
Phone: +31 343 442 524  
Fax: +31 343 442 162

Customers in the UK should direct all questions, incidents and complaints to the Midmark UK Responsible Person listed below.

UKCApartner4U Ltd.,  
7 CAMPION WAY, BINGHAM, NOTTINGHAM NG13 8TR, UK.  
www.UKCApartner4U.co.uk  
Phone : +44 7905 384429

Customers in Australia should direct all questions, incidents and complaints to the Midmark Sponsor listed below.

ICONA Pty Ltd.  
1A/2A Westall Road  
Clayton, Victoria 3168  
Phone: 1300 442 662

## ***Authorized Representatives - continued***

Customers in India should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Midmark India  
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)  
Mumbai, 400022  
Tel:+91 22 4915 3000  
Fax:+91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Attieh Medico Ltd  
Abdullah M. Al Khalifi Street  
Al Nakheel Dist. II  
P.O. Box 116105  
Jeddah (East)  
21391  
Saudi Arabia  
Tel: + 966 2 286 4707  
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

GULF DRUG LLC.  
Al Barsha,  
Dubai, UAE  
Phone: +971 4 501 4000  
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Associated Medical Supplies Company LTD. 沒有中文版使用說明  
Room 1201-1202, 12/F, Fo Tan Industrial Center  
26-28 Au Pui Wan Street, Fo Tan, Shatin  
New Territories, Hong Kong  
Phone: +852 2604 9389  
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Tadmur Trading  
Midmac Roundabout  
Opposite Al Mana Petrol Station  
Doha, Qatar  
Tel: +974.44337000  
Fax: +974.44337100

Customers in Israel should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Medtechnica Ltd.  
HaTnufa St 7  
Petah Tikva, Israel  
Tel: +972 3.925.4040  
Fax: +972 3.924.9977

# Important Information - Safety Symbols

## **WARNING**

*Indicates a potentially hazardous situation which could result in serious injury.*

## **Caution**

*Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices*

## **Equipment Alert**

*Indicates a potentially hazardous situation which could result in equipment damage.*

## **Note**

*Amplifies a procedure, practice or condition.*

## Symbol Glossary

Pressure Limit

Proper shipping orientation

Fragile

Manufacturer

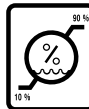
Medical Device

Temperature Limit

Keep dry

Humidity

Serial Number

Catalogue Number

## **Intended Use**

Support the patient's arms during examination.

## **Requirements (Bracket sets):**

204 / 604.....9A632003  
222 / 223 / 622 / 623.....9A314005  
244.....none  
625.....9A314005 / 9A314007

## **Safety Instructions**

### **WARNING**

*No modification of this equipment is allowed.*

### **WARNING**

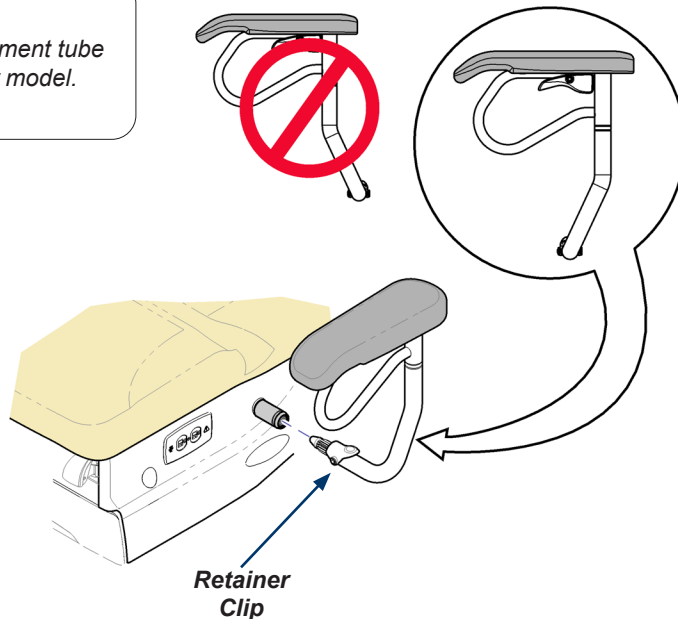
*Do not sit on Chair Arm or Chair Arm pad.*

# Set Up / Operation (222, 223, 622, 623, 625)


- A) Position Chair Arm as indicated. (see note)
- B) Insert arm into socket until retainer clip locks arm into place.

## Note

Be sure the angle of the arm weldment tube is properly positioned for the chair model.  
See illustration for details.

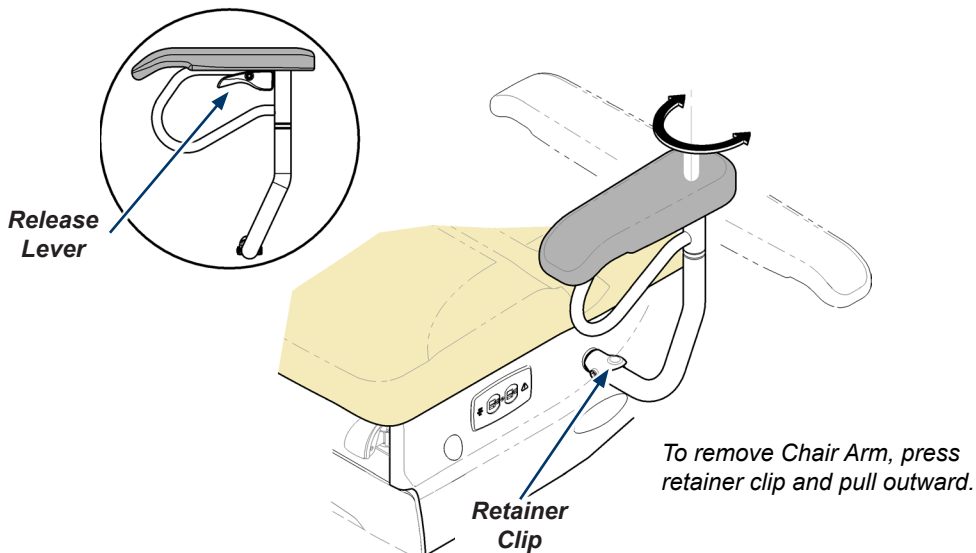- A) To position Chair Arm, squeeze release lever.
- B) Rotate Chair Arm to desired position.


# Set Up / Operation (204 and 604)


- A) Position Chair Arm as indicated. (see note)
- B) Insert arm into socket until retainer clip locks arm into place.

## Note

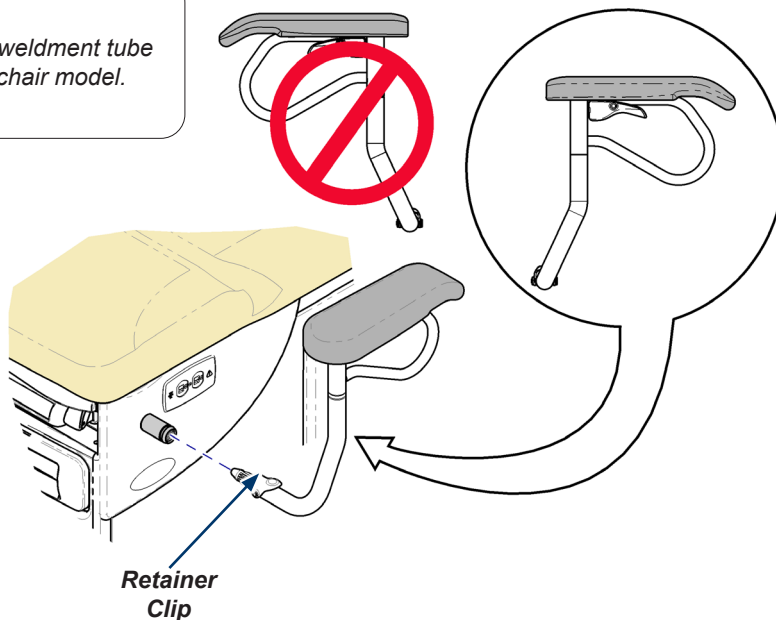Be sure the angle of the arm weldment tube is properly positioned for the chair model. See illustration for details.

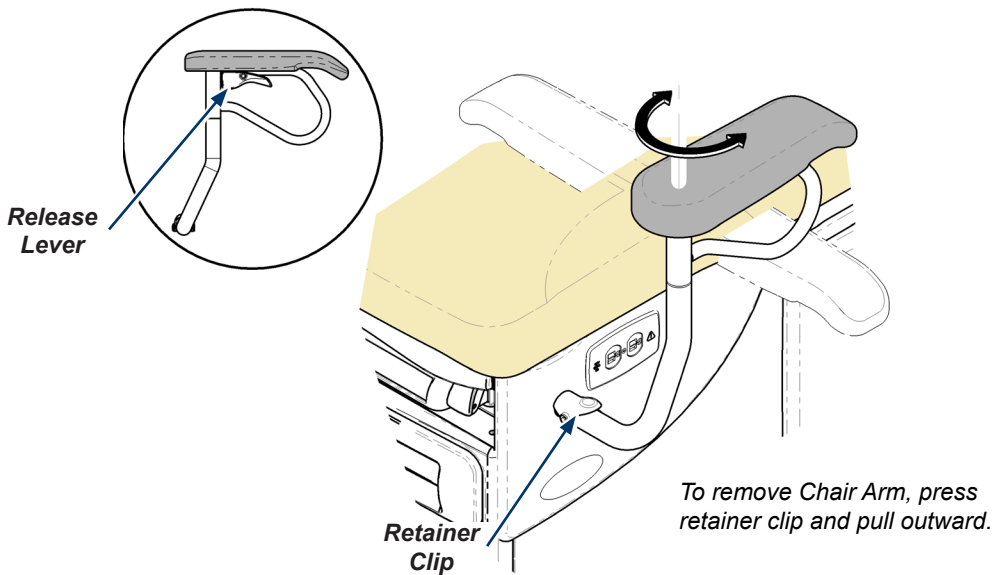- A) To position Chair Arm, squeeze release lever.
- B) Rotate Chair Arm to desired position.


# Set Up / Operation (244)


- A) Position Chair Arm as indicated. (see note)
- B) Insert arm into socket until retainer clip locks arm into place.

## Note

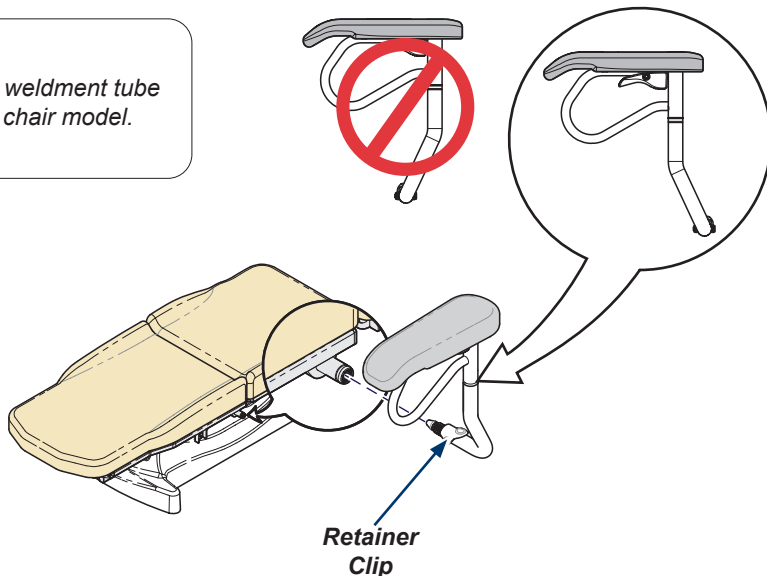Be sure the angle of the arm weldment tube is properly positioned for the chair model. See illustration for details.

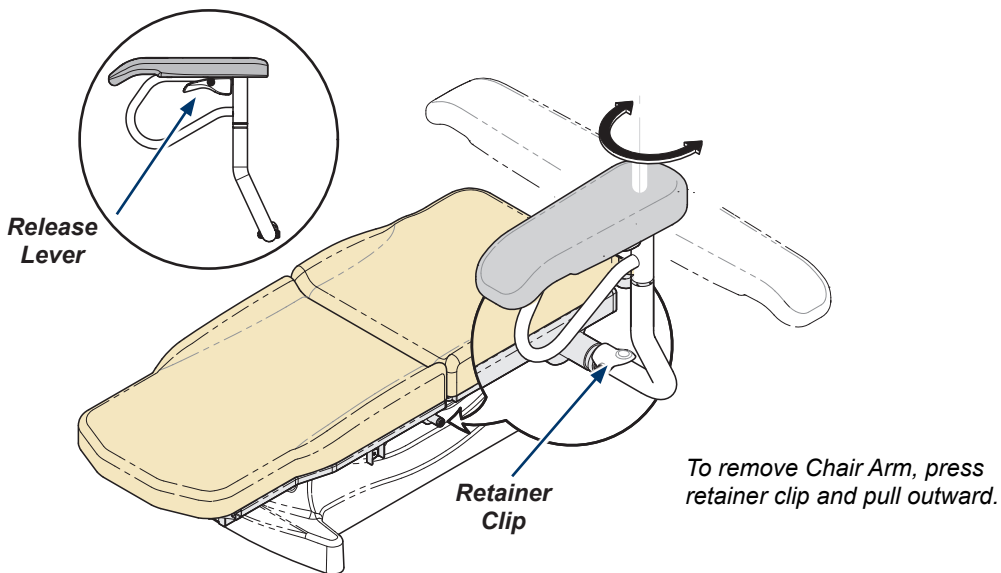- A) To position Chair Arm, squeeze release lever.
- B) Rotate Chair Arm to desired position.

# Maintenance

## Calling for Service

### Note

*Model / serial number information is required when calling for service.*

If service is required, contact Midmark directly

US 844.856.1230 / Canada 937.526.8585  
Monday-Thursday 8:00 AM until 6:00 PM  
Friday 8:00 AM until 5:00 PM (ET)

[midmark.com](http://midmark.com)

## Cleaning

### Equipment Alert

*The Chair Arm is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Chair Arm.*

### Chair Arm Pad

Wash your Chair Arm weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your Chair Arm with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the Chair Arm surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [midmark.com](http://midmark.com) in the Technical library under the User Information tab for your product.

### Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

## Periodic Maintenance

Periodically inspect the following areas:

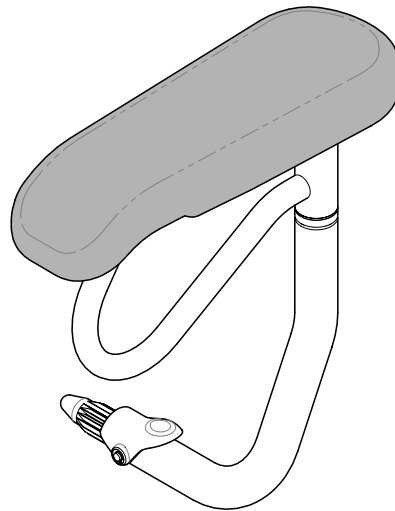
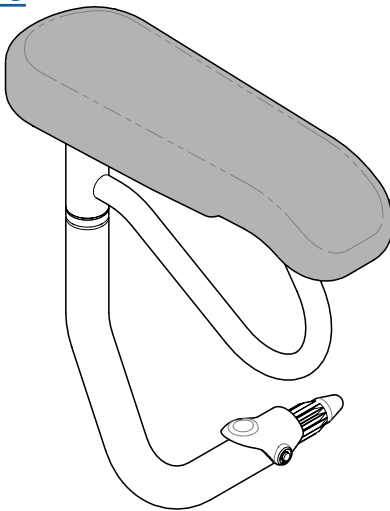
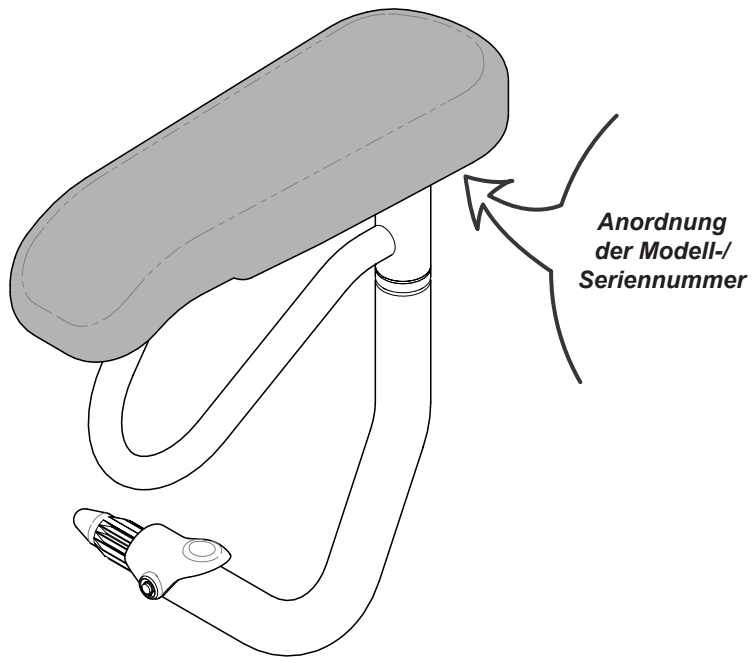
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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© Midmark Corporation 2019
**Midmark Corporation**