Articulating Armboard 9A81001

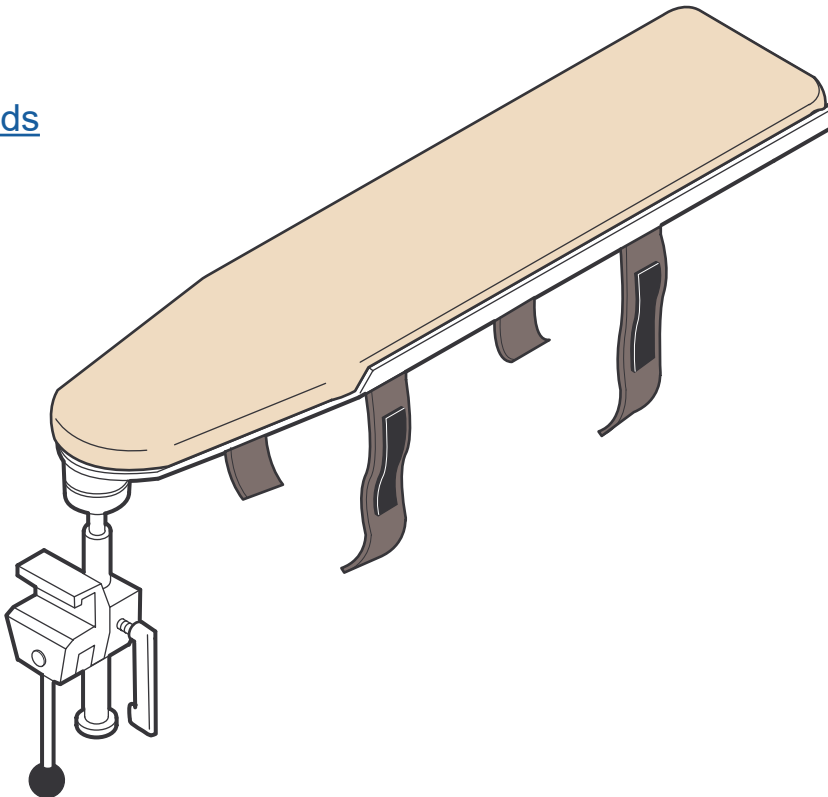
[English](#)
[Deutsch](#)
[Français](#)
[Español](#)
[Dansk](#)
[Suomi](#)
[Italiano](#)
[Svenska](#)
[Norsk](#)
[Nederlands](#)
[Polski](#)
[Türkçe](#)

Applies to Models:

230 / 630 / 641

Special Tools:

none

Set Up / User Guide

Style U

003-10308-99 Rev. AA6 (1/8/24)

TP202 20-42-FO-00014 Rev A1 C2169

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

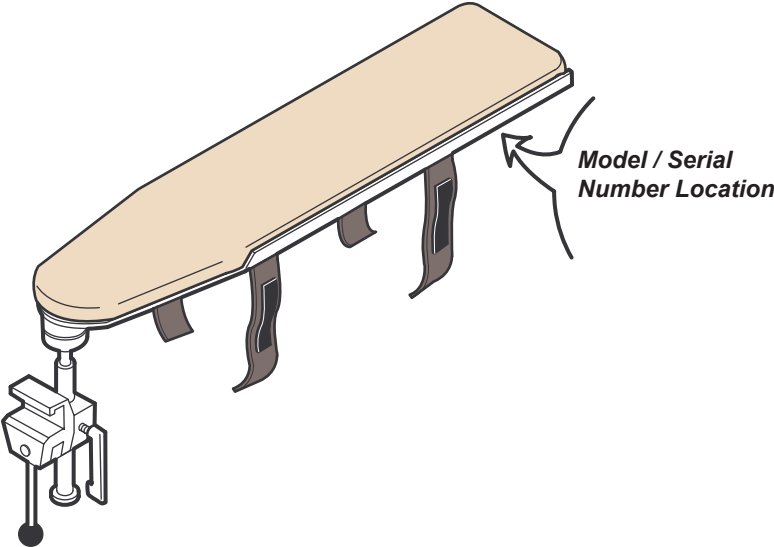
Table of Contents

Important Information

- [Product Information](#) 2
- [Product Registration](#) 4
- [Transportation / Storage Conditions](#) 4
- [Operating Conditions](#) 4
- [Disposal of Equipment](#) 4
- [Authorized Representatives](#) 4
- [Important Information](#) 6
- [Intended Use](#) 7
- [Requirements](#) 7
- [Safety Instructions](#) 7

Set Up

- [Set Up Instructions](#) 8

Operation

- [Articulation](#) 9
- [Height Adjustment](#) 9

Adjustment

- [Locking System](#) 10

Maintenance

- [Calling for Service](#) 11
- [Cleaning](#) 11
- [Periodic Maintenance](#) 11

Warranty Information

- [Limited Warranty](#) 12

Product Registration

Transportation / Storage Conditions

Ambient Temperature Range:.....+5°C to +38°C (+41°F to +100°F)
Relative Humidity.....10% to 90% (non-condensing)
Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

WARNING

*In the event of any serious incident in relation to the device,
please contact Midmark and the appropriate competent authority.*

Customers in the EU should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

CEpartner4U
Esdoornlaan 13
3951 DB Maarn, The Netherlands
Phone: +31 343 442 524
Fax: +31 343 442 162

Customers in the UK should direct all questions, incidents, and complaints to Midmark's UK Responsible Person listed below.

UKCApartner4U Ltd.,
7 CAMPION WAY, BINGHAM, NOTTINGHAM NG13 8TR, UK.
www.UKCApartner4U.co.uk
Phone : +44 7905 384429

Customers in Australia should direct all questions, incidents, and complaints to Midmark's Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Authorized Representatives - continued

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel: +91 22 4915 3000
Fax: +91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Associated Medical Supplies Company LTD. 沒有中文版使用說明
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice or condition.

Symbol Glossary

Proper shipping orientation

Fragile

Manufacturer

Keep dry

Serial Number

Catalogue Number

Medical Device

Intended Use

To support the patient's arm during procedures and administering of IVs.

Requirements

Depending on chair model, a back mounted accessory rail may need to be installed.

Safety Instructions

WARNING

No modification of this equipment is allowed.

WARNING

Do not sit on Armboard or Armboard pad.

WARNING

Do not press any button during a procedure.

WARNING

Do not use Armboard as a gripping surface during ingress egress.

Set Up

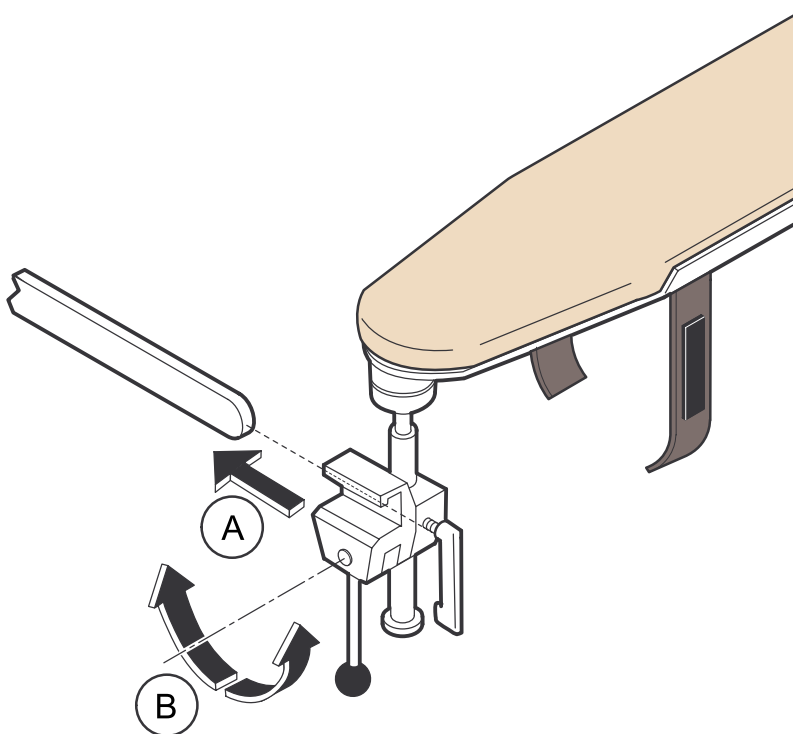
WARNING

Do not use with any other accessories on same side of chair. When used with rail in down position assure patient is restrained to prevent accidentally falling from unit during procedures. Assure all positions are solidly locked to prevent sudden movement during procedure.

Caution

Always support Armboard assembly when changing positions on it.


A) Slide swivel block onto rail to desired position.

B) Rotate lock handle to one side to lock swivel block onto rail.


Operation - Articulation

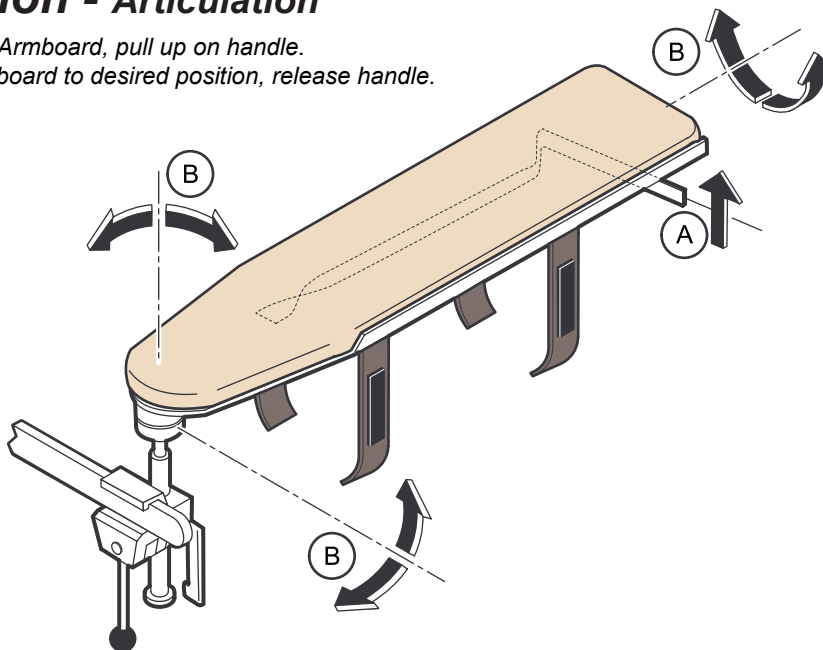- A) Supporting Armboard, pull up on handle.
- B) Rotate Armboard to desired position, release handle.

Operation - Height Adjustment

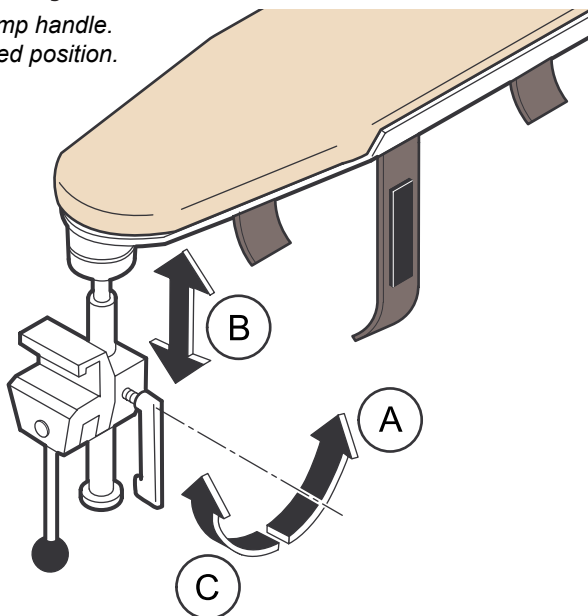- A) Supporting Armboard, loosen clamp handle.
- B) Raise or lower Armboard to desired position.
- C) Tighten clamp handle.


Adjustment - Locking System

Equipment Alert

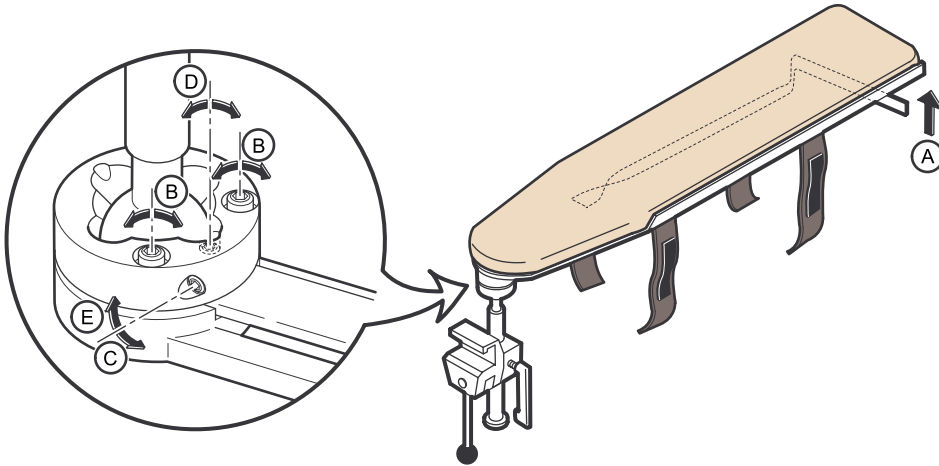It may be necessary to adjust locking system periodically. Use a 3mm Allen wrench.

To check Armboard adjustment:

Check handle for 1/8" (3.2 mm) to 3/16" (4.8 mm) free movement (distance handle moves without releasing).

Note

If handle free movement is not within above dimensions continue with the following steps.

To tighten or loosen Armboard movement:

- A) Turn both screws 1/8 to 1/4 turn clockwise to tighten or counterclockwise to loosen movement.
- B) Loosen locking screw.
- C) Turn adjustment screw clockwise or counterclockwise until free movement of handle is within tolerance.
- D) Tighten locking screw.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM
Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The Armboard is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Armboard.

Armboard Pad

Wash your Armboard weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your Armboard with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the Armboard surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

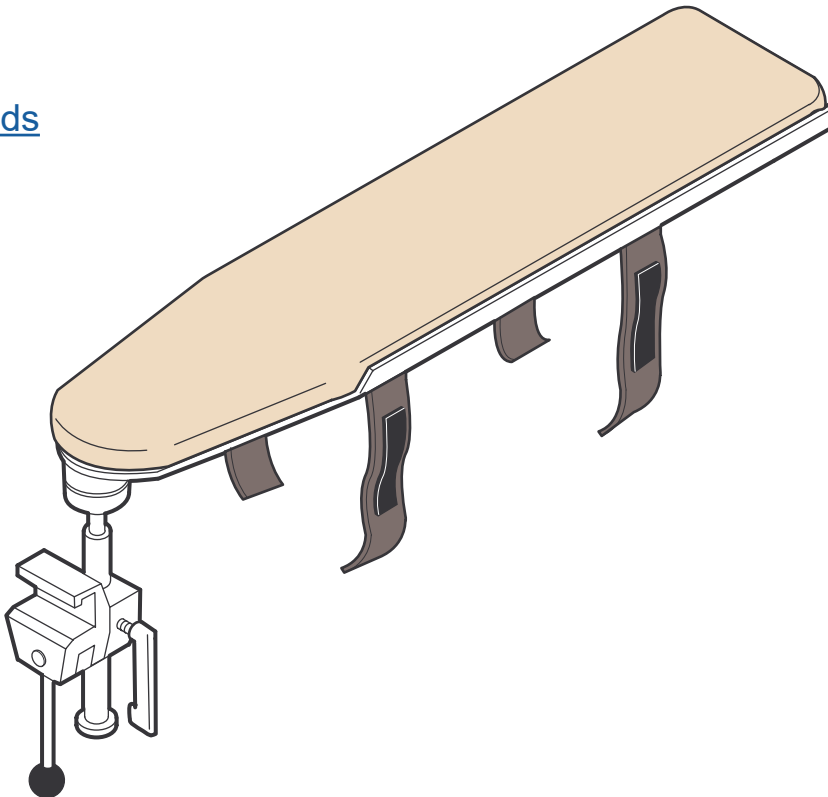
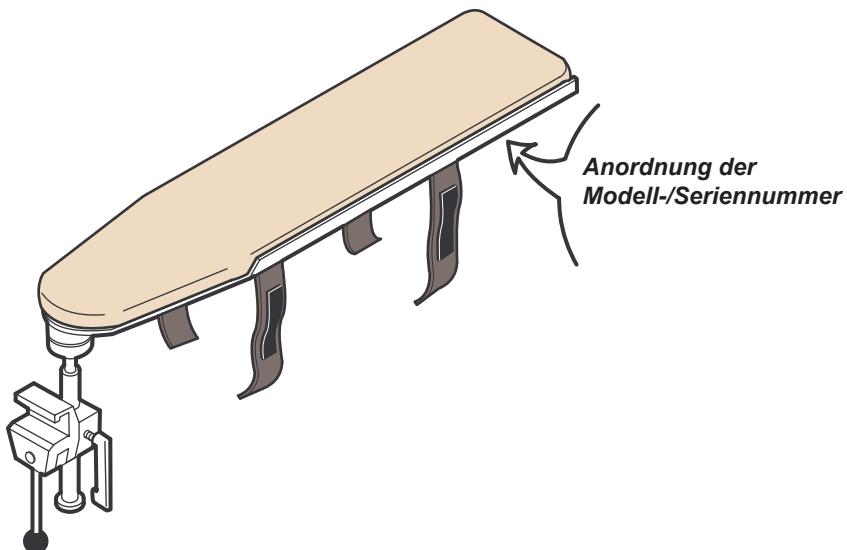
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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Midmark Corporation