[English](#)
[Deutsch](#)
[Français](#)
[Español](#)
[Dansk](#)
[Suomi](#)
[Italiano](#)
[Svenska](#)
[Norsk](#)
[Nederlands](#)
[Polski](#)
[Türkçe](#)

IV Pole

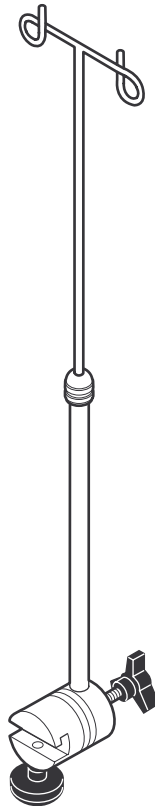
9A77001

Applies to Models:

230 / 630 / 641 / 646 / 647

Special Tools:

none

Set Up / User Guide

Style U

003-10313-99 Rev. AA5 (9/13/23)

TP202 20-42-FO-00014 Rev A2 C2169

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

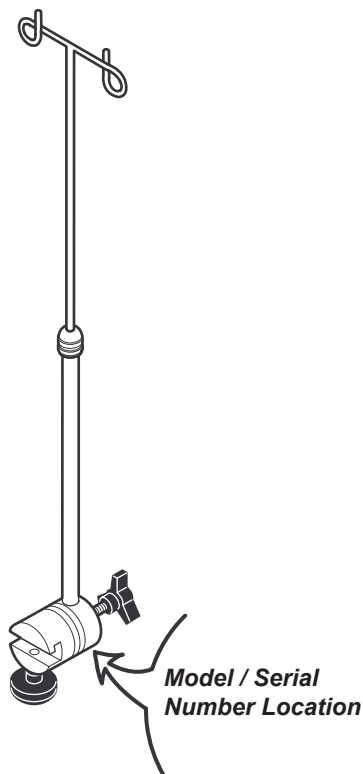
Table of Contents

Important Information

- [Product Information](#) 2
- [Product Registration](#) 4
- [Transportation / Storage Conditions](#) 4
- [Operating Conditions](#) 4
- [Disposal of Equipment](#) 4
- [Authorized Representatives](#) 4
- [Important Information](#) 6
- [Intended Use](#) 7
- [Requirements](#) 7
- [Safety Instructions](#) 7

Set Up

- [Set Up Instructions](#) 8

Operation

- [Rotational Positioning](#) 9
- [Height Adjustment](#) 10

Maintenance

- [Calling for Service](#) 11
- [Cleaning](#) 11
- [Periodic Maintenance](#) 11

Warranty Information

- [Limited Warranty](#) 12

Product Registration

Transportation / Storage Conditions

Ambient Temperature Range:.....+5°C to +38°C (+41°F to +100°F)
Relative Humidity.....10% to 90% (non-condensing)
Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

Warning
*In the event of any serious incident in relation to the device,
please contact Midmark and the appropriate competent authority.*

Customers in the EU should direct all questions, incidents, and complaints to Midmark’s Authorized Representative listed below.

CEpartner4U
Esdoornlaan 13
3951 DB Maarn, The Netherlands
Phone: +31 343 442 524
Fax: +31 343 442 162

Customers in Australia should direct all questions, incidents, and complaints to Midmark’s Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Authorized Representatives - continued

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel: +91 22 4915 3000
Fax: +91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Associated Medical Supplies Company LTD. 沒有中文版使用說明
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice, or condition.

Symbol Glossary

Fragile

Keep dry

Manufacturer

Serial Number

Medical Device

Catalogue Number

Intended Use

To provide secure support for fluid for an IV infusion. Only use with a base mounted rail.

Requirements

Depending on chair model, an accessory/base rail may need to be installed.

Safety Instructions

Warning

No modification of this equipment is allowed.

Warning

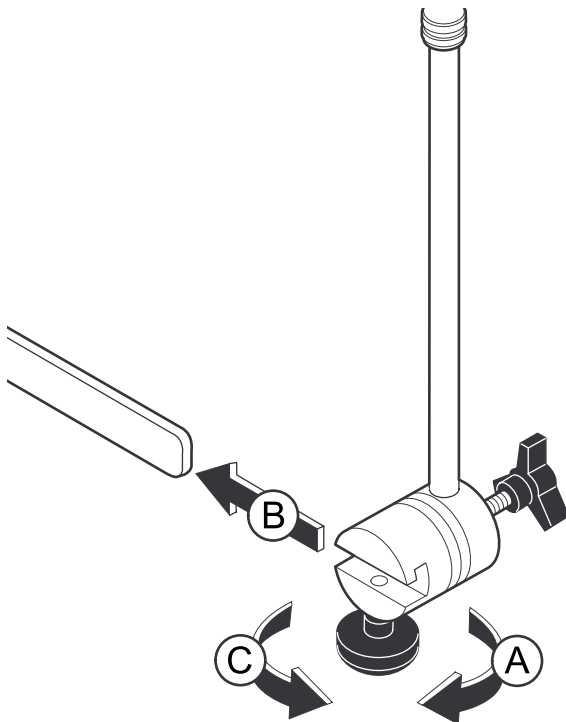
Do not use IV Assembly as a gripping surface during ingress egress.

Set Up

Caution

Only use 9A77001 with a base mounted rail. Assure chair is clear of any obstacles that may interfere with operation of IV Pole. IV Pole may interfere with 8 ft. (2.44 m) ceiling. Always support IV Pole assembly when changing positions on it.


Caution

Ensure the IV pole is fully tighten after positioning.

- A) Loosen lower knob on swivel block.
- B) Slide swivel block onto exam chair base mounted rail.
- C) Position at desired location and tighten knob.

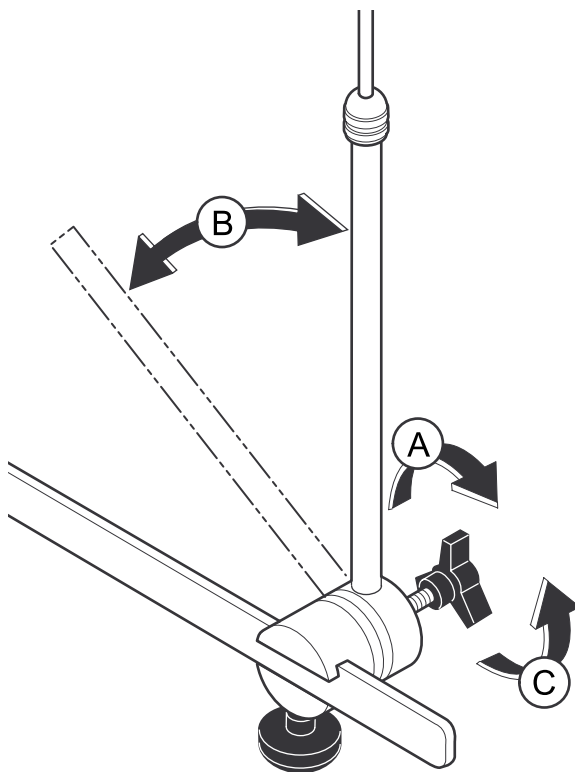Operation


Rotational Positioning

A) Loosen knob.

B) Rotate IV pole to desired position.

C) Tighten knob.

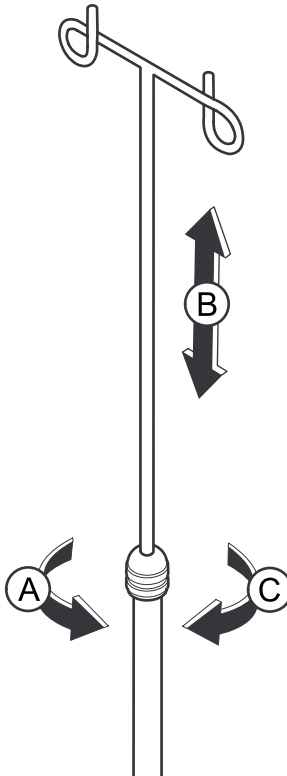Operation - continued

Height Adjustment

- A) Support top IV rod, loosen ferrule.*
- B) Raise or lower top IV rod to desired position.*
- C) Tighten ferrule.*

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The IV Assembly is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the IV Assembly.

Painted Metal / Plastic Surfaces

Clean the plated metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

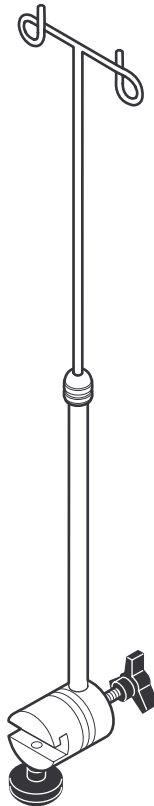
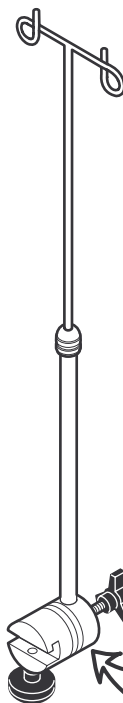
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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Midmark Corporation