Instrument Tray (Double Arm) 9A267

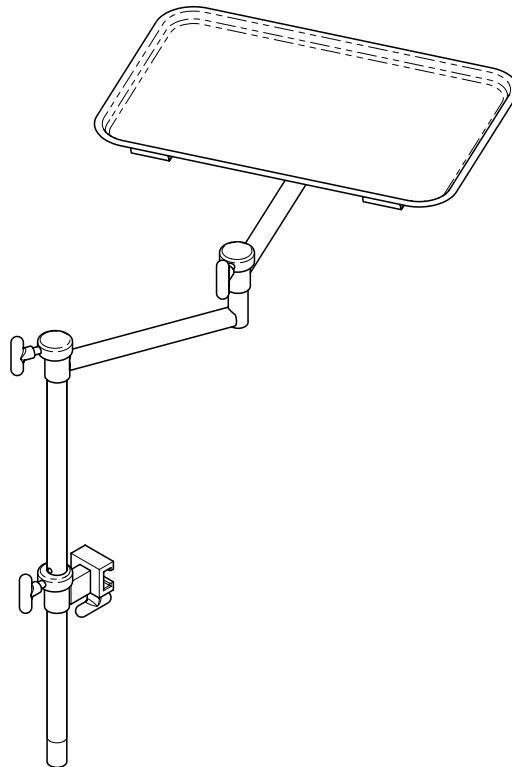
[English](#)
[Deutsch](#)
[Français](#)
[Español](#)
[Dansk](#)
[Suomi](#)
[Italiano](#)
[Svenska](#)
[Norsk](#)
[Nederlands](#)
[Polski](#)
[Türkçe](#)

Applies to Models:

230 / 630 / 641

Special Tools:

none

Set Up / User Guide

Style U

003-10314-99 Rev. AA4 (9/26/23)
TP202 20-42-FO-00014 Rev A1 C2169

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

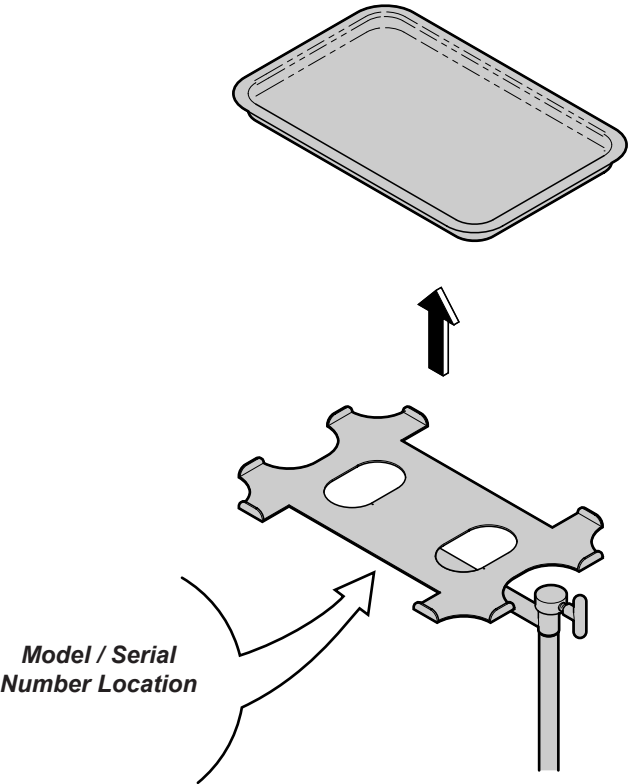
Table of Contents

Important Information

Product Information 2

Product Registration 4

Operating Conditions 4

Transportation / Storage Conditions 4

Disposal of Equipment 4

Authorized Representatives 4

Important Information 6

Intended Use 7

Requirements 7

Safety Instructions 7

Set Up

Set Up Instructions 8

Operation

Operating Instructions 10

Maintenance

Calling for Service 11

Cleaning 11

Periodic Maintenance 11

Warranty Information

Limited Warranty 12

Product Registration

SCAN ME

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)

Relative Humidity.....30% to 75% (non-condensing)

Altitude.....3000m or less

Transportation / Storage Conditions

Ambient Temperature Range:.....-5°C to +38°C (+23°F to 100°F)

Relative Humidity.....10% to 90% (non-condensing)

Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

EC

REP

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in the EU should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

CEpartner4U
Esdoornlaan 13
3951 DB Maarn, The Netherlands
Phone: +31 343 442 524
Fax: +31 343 442 162

Customers in the UK should direct all questions, incidents, and complaints to Midmark's UK Responsible Person listed below.

UKCApartner4U Ltd.,
7 CAMPION WAY, BINGHAM, NOTTINGHAM NG13 8TR, UK.
www.UKCApartner4U.co.uk
Phone : +44 7905 384429

Customers in Australia should direct all questions, incidents, and complaints to Midmark's Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

003-10314-99

English - 4

© Midmark Corporation 2020

TP202 20-42-FO-00014 Rev A1 C2169

Authorized Representatives - continued

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel: +91 22 4915 3000
Fax: +91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Associated Medical Supplies Company LTD. 沒有中文版使用說明
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice, or condition.

Symbol Glossary

Proper shipping orientation

Fragile

Manufacturer

Keep dry

Serial Number

Catalogue Number

Medical Device

Intended Use

To hold instruments for an examination or general procedure. Only use with a base mounted rail.

Requirements

Depending on chair model, either a base rail or base mounted accessory rail must be installed.

Safety Instructions

Warning

No modification of this equipment is allowed.

Warning

Do not use Instrument Tray as a gripping surface during ingress egress.

Set Up

Warning

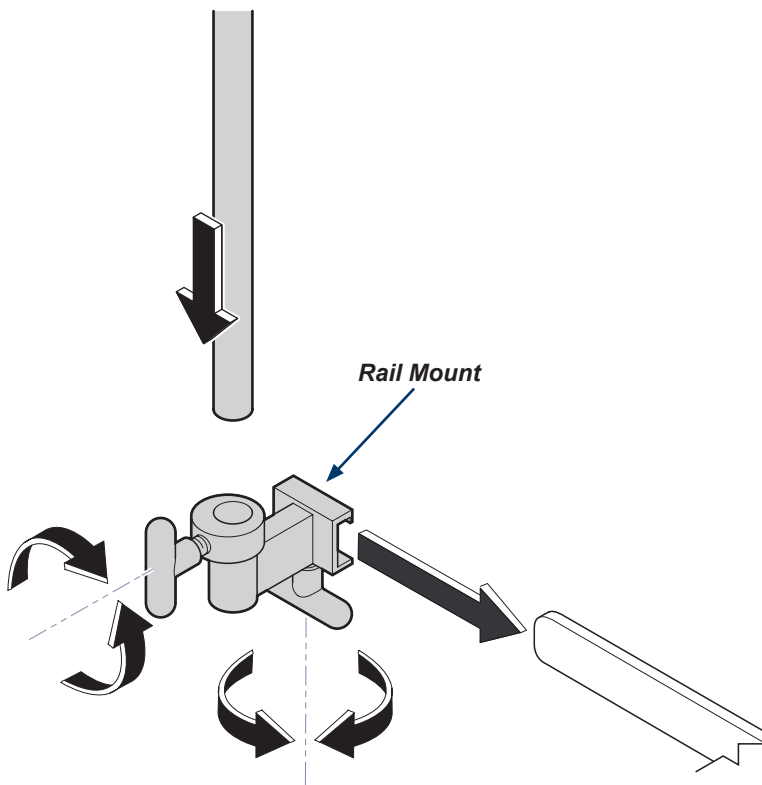
T-handle that secures rail mount must be positioned below base mounted rail so lip on rail mount hooks over the top of rail.

Equipment Alert

Place chair in desired position before mounting 9A267 Instrument Tray.

To prevent damage to the chair / tray, do not operate the chair after the Instrument Tray has been installed.

Step 1:

- A) Loosen lower T-handle on rail mount.
- B) Slide rail mount onto base mounted rail.
- C) Tighten lower T-handle on rail mount.
- D) Loosen upper T-handle on rail mount.
- E) Insert mounting rod.
- F) Tighten upper T-handle.

Set Up - continued

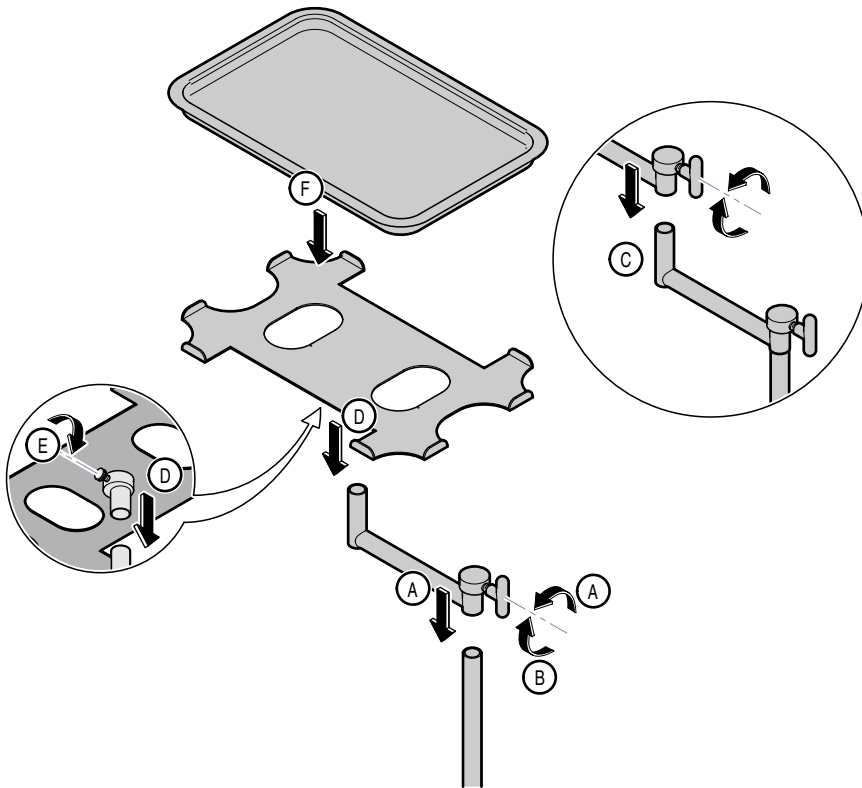
Warning

Assure Arm Extensions are completely engaged when mounting before tightening T-handles. Failure to comply could result in disengagement during procedure and personal injury.

Note

To reposition Tray while supporting assembly, loosen appropriate handle(s), reposition, then retighten handles.

Step 2:

- A) Loosen T-handle on Arm Extension and insert onto Mounting Rod.
- B) Tighten T-handle.
- C) Mount second Arm Extension to mounting tube of first Arm Extension.
- D) Install Tray Mount Weldment onto mounting tube of Arm Extension.
- E) Tighten thumb screw.
- F) Place tray in position.

Operation

Equipment Alert

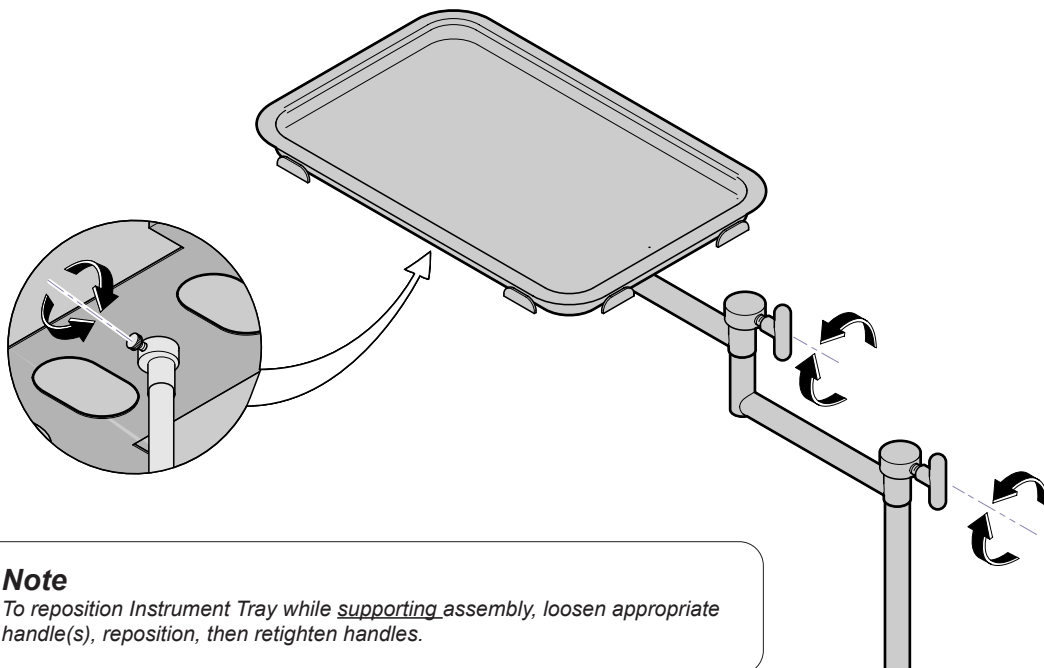
Place chair in desired position before mounting 9A267 Instrument Tray. To prevent damage to the chair / tray, do not operate the chair after the Instrument Tray has been installed.

Equipment Alert

Do not use more than one instrument tray simultaneously on the same side of the chair.

Max. weight (centered on tray) 15 lbs (6.8 kg)

Note

To reposition Instrument Tray while supporting assembly, loosen appropriate handle(s), reposition, then retighten handles.

- A) Loosen T-handle(s), rotate Arm Extension(s) to desired position.
- B) Tighten T-handle(s).
- C) Loosen thumb screw, rotate tray to desired position.
- D) Tighten thumb screw.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The accessory is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the accessory.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces using a clean soft cloth, and mild cleaner.

Equipment Alert

Do not use bleaching agents or any abrasive materials / substances on stainless steel surfaces (i.e. bleach, steel wool, wire brush, scouring powder, etc.).

Failure to comply may result in damage to the accessory.

Stainless Steel

Frequent cleaning will prolong the life and help maintain the natural luster of stainless steel:

- When cleaning, always rub in the direction of the “grain” of the metal.
- Avoid use of steel wool or steel brushes when cleaning stainless steel. These steel particles may become embedded into the stainless steel and cause rust.
- Avoid the use of chlorine bleach. Chemicals in bleach will cause pitting, corrosion, and discoloration of the metal.
- Despite the name, stainless steel actually can stain and mark. However, stains and marks do not affect the integrity of the metal.

Periodic Maintenance

Periodically inspect the following areas:

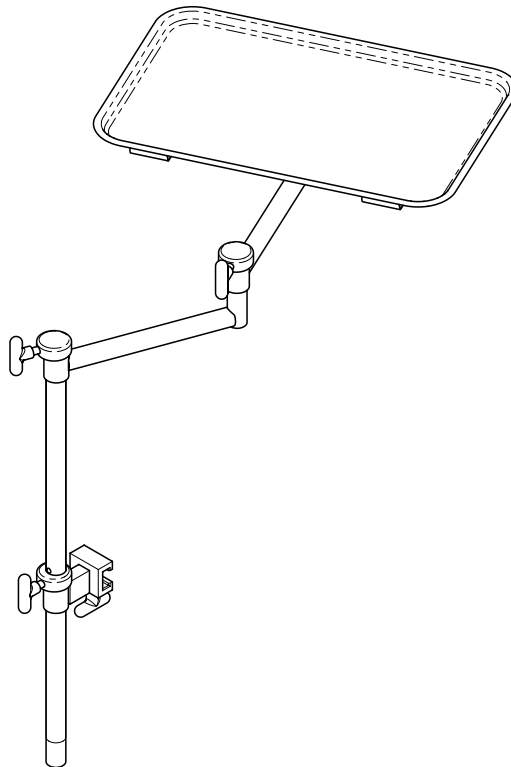
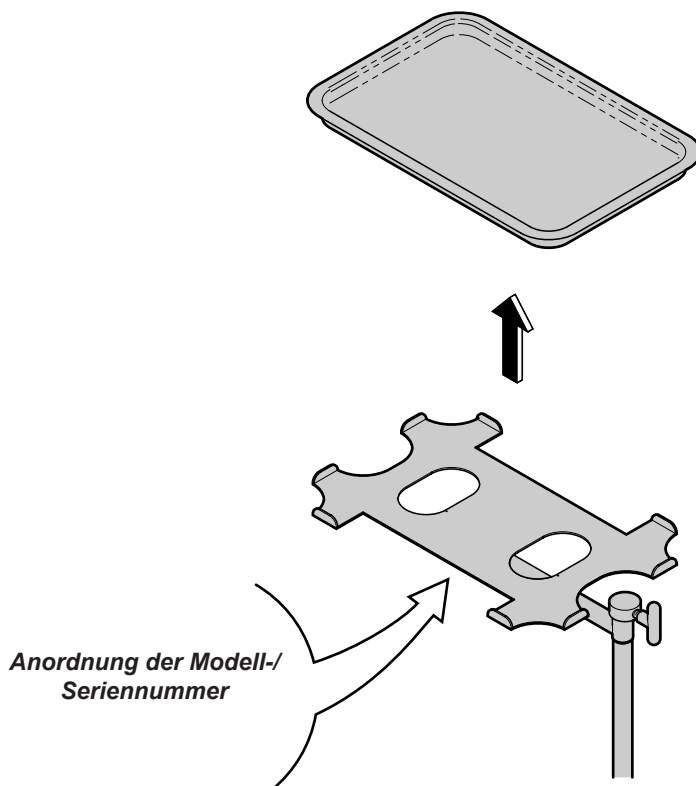
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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