
# Urology Drain System (Field Installed) 9A329004

[English](#)  
[Español](#)  
[Français](#)

**Applies to Models:**  
*230 (-001, -002 and -004)*

**Special Tools:**  
*none*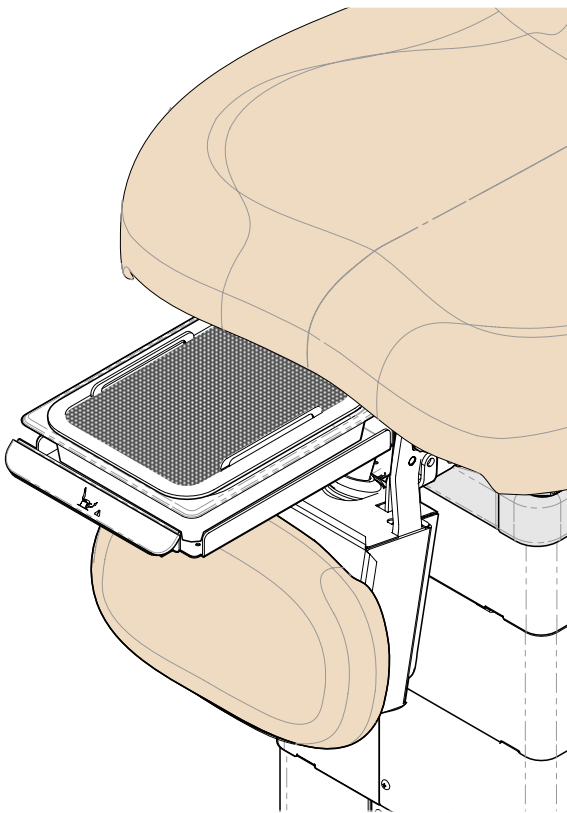

## Set Up / Operation

Style P

003-10330-99 Rev. AA3 (10/11/23)


# Product Information

**Dealer:**

**Date of Purchase:**

**Model Number:**

**Midmark Authorized Service Company:**

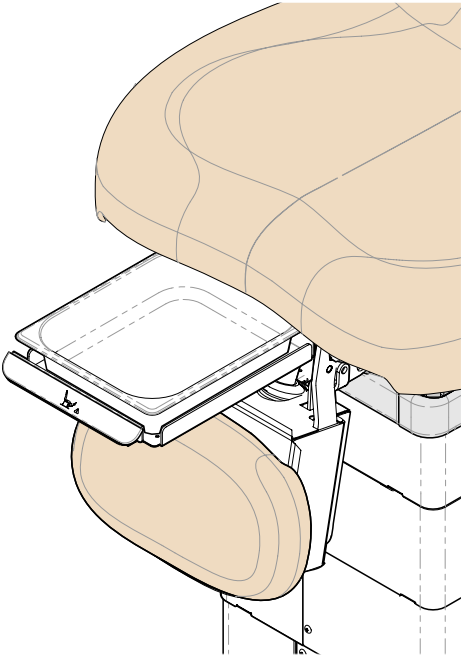# Table of Contents

## Important Information

- [Product Information](#) ..... 2
- [Product Registration](#) ..... 4
- [Operating Conditions](#) ..... 4
- [Disposal of Equipment](#) ..... 4
- [Authorized Representatives](#) ..... 4
- [Important Information](#) ..... 6
- [Intended Use](#) ..... 7
- [Restrictions](#) ..... 7
- [Safety Instructions](#) ..... 7

## Set Up

- [Set Up Instructions](#) ..... 8
- [Operation](#) ..... 11

## Maintenance

- [Calling for Service](#) ..... 12
- [Cleaning](#) ..... 12
- [Periodic Maintenance](#) ..... 12

## Warranty Information

- [Limited Warranty](#) ..... 13

## Product Registration

## Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)  
Relative Humidity.....30% to 75% (non-condensing)  
Altitude.....3000m or less

## Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

## Authorized Representatives

### Warning

*In the event of any serious incident in relation to the device,  
please contact Midmark and the appropriate competent authority.*

Customers in Australia should direct all questions, incidents and complaints to Midmark's Sponsor listed below.

ICONA Pty Ltd.  
1A/2A Westall Road  
Clayton, Victoria 3168  
Phone: 1300 442 662

Customers in India should direct all questions, incidents and complaints to Midmark's Authorized Representative listed below.

Midmark India  
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)  
Mumbai, 400022  
Tel:+91 22 4915 3000  
Fax:+91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd  
Abdullah M. Al Khalifi Street  
Al Nakheel Dist. II  
P.O. Box 116105  
Jeddah (East)  
21391  
Saudi Arabia  
Tel: + 966 2 286 4707  
Fax: + 966 2 286 4744

## ***Authorized Representatives - continued***

Customers in the United Arab Emirates should direct all questions, incidents and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.  
Al Barsha,  
Dubai, UAE  
Phone: +971 4 501 4000  
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents and complaints to Midmark's Authorized Representative listed below.

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Associated Medical Supplies Company LTD.  
Room 1201-1202, 12/F, Fo Tan Industrial Center  
26-28 Au Pui Wan Street, Fo Tan, Shatin  
New Territories, Hong Kong  
Phone: +852 2604 9389  
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading  
Midmac Roundabout  
Opposite Al Mana Petrol Station  
Doha, Qatar  
Tel: +974.44337000  
Fax: +974.44337100

Customers in Israel should direct all questions, incidents and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.  
HaTnufa St 7  
Petah Tikva, Israel  
Tel: +972 3.925.4040  
Fax: +972 3.924.9977

# Important Information - Safety Symbols

**WARNING**  
*Indicates a potentially hazardous situation which could result in serious injury.*

**Caution**  
*Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices*

**Equipment Alert**  
*Indicates a potentially hazardous situation which could result in equipment damage.*

**Note**  
*Amplifies a procedure, practice or condition.*

## Symbol Glossary

Manufacturer

Serial Number

Catalogue Number

## ***Intended Use***

To provide drainage for urology procedures.

## ***Restrictions***

Not for use with rotational base units.

## ***Safety Instructions***

### ***Warning***


*No modification of this equipment is allowed.*


### ***Warning***

*Do not sit on the drain pan / drain pan holder assembly.*


# Set Up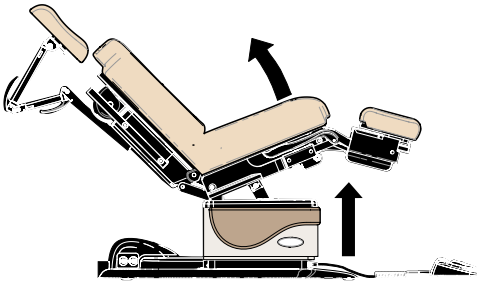


**Step 1:**  
*Run BASE and TILT all the way UP.*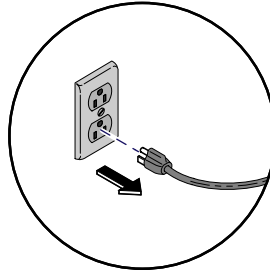

**Step 2:**  
*Unplug power cord.*

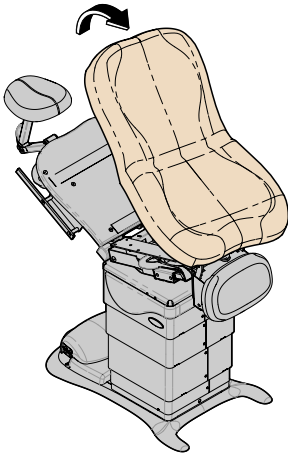**Step 3:**  
*Remove seat / back upholstery.*

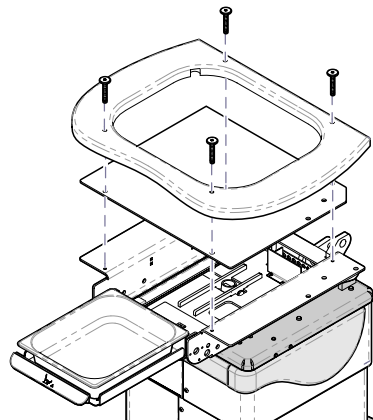
**Step 4:**  
*Remove mounting board and support plate.*

# Set Up - continued

## Step 5:


Remove four nuts and pan holder assembly.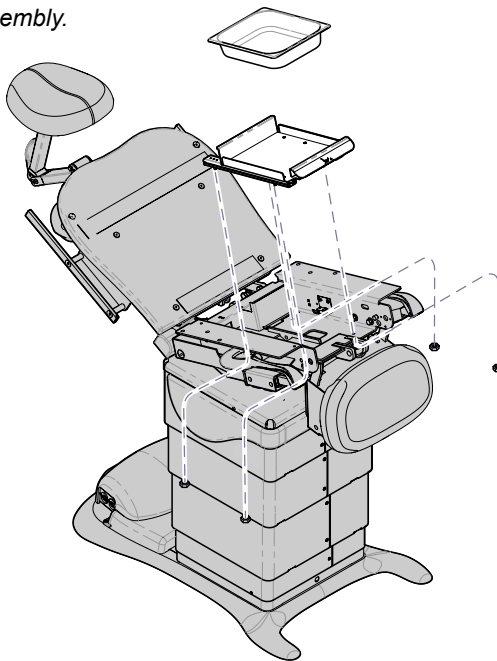

## Step 6:

Remove two screws from wire harness cover.

Note: Keep screws to secure cage bracket.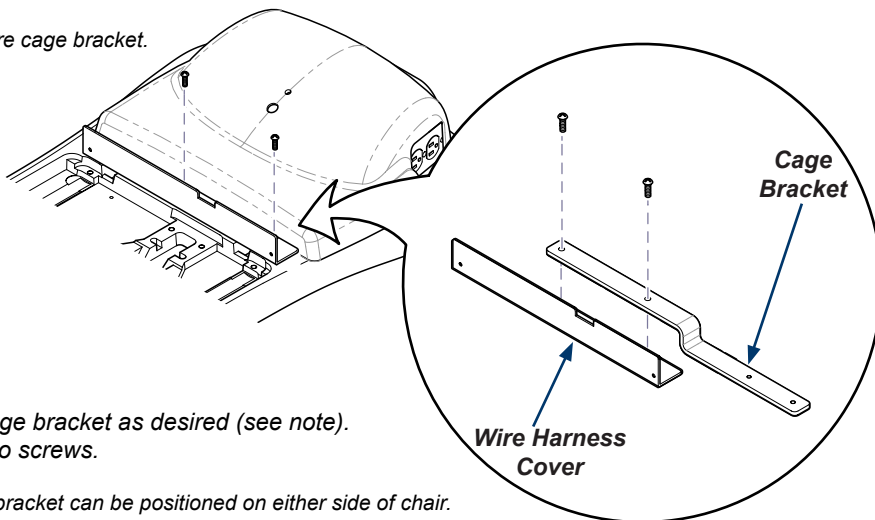

## Step 7:

Position the cage bracket as desired (see note).  
Secure with two screws.


Note: The cage bracket can be positioned on either side of chair.

## Set Up - continued

### Step 9:


*Install washers on new pan holder.*

*Note: Install three washers on both front studs and one washer on right rear stud.*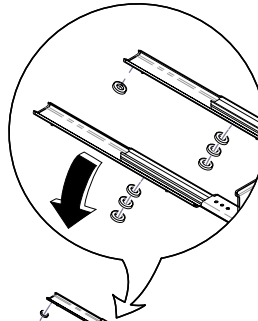

### Step 8:

*Install hose guide.*

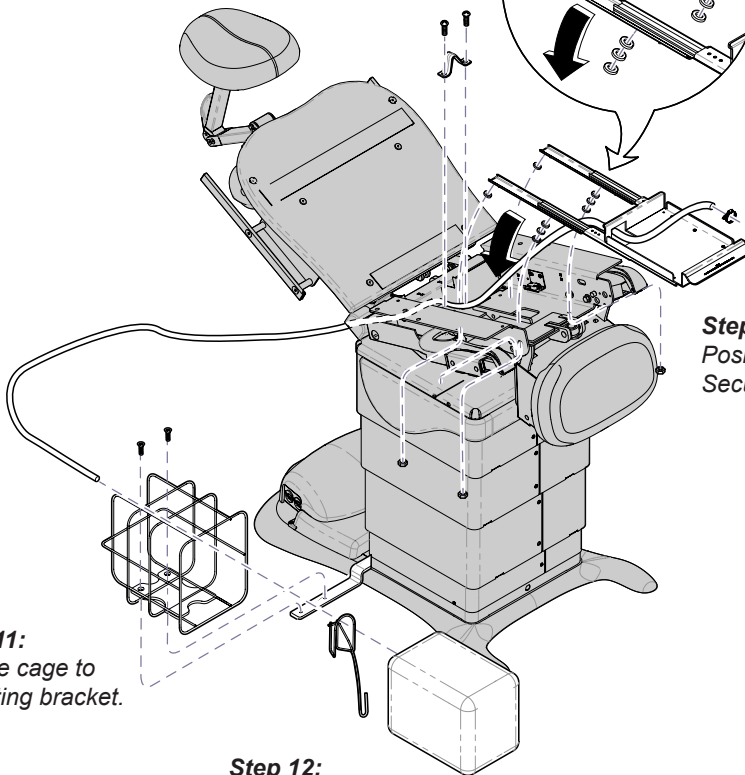### Step 13:

*Route hose thru chair.  
Attach hose to drain  
pan with hose clamp.*

### Step 10:

*Position pan holder.  
Secure with four nuts.*

### Step 11:

*Secure cage to  
mounting bracket.*

### Step 12:

*Attach hose to hose clip.  
Insert hose clip into bottle.  
Install bottle into cage.*

# Operation

## **Attention!**

**The Foot Up function is disabled when the treatment pan is pulled out.**

*(You will hear several “beeps” if Foot Up is activated with the pan pulled out.)*

*To resume Foot Up operation:*

*Push the pan all the way in.*

*Press Foot Up button.*

## **To access the drain pan:**

*Lower the foot section.*

*Pull the drain pan slide out.*

# Maintenance

## Calling for Service

### Note

*Model / serial number information is required when calling for service.*

If service is required, contact Midmark® directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM Friday 8:00 AM until 5:00 PM (ET)

[midmark.com](http://midmark.com)

## Cleaning

### Equipment Alert

*The accessory is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the accessory.*

### Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces using a clean soft cloth, and mild cleaner.

### Equipment Alert

*Do not use bleaching agents or any abrasive materials / substances on stainless steel surfaces (i.e. bleach, steel wool, wire brush, scouring powder, etc.).*

*Failure to comply may result in damage to the accessory.*

### Stainless Steel

Frequent cleaning will prolong the life and help maintain the natural luster of stainless steel:

- When cleaning, always rub in the direction of the “grain” of the metal.
- Avoid use of steel wool or steel brushes when cleaning stainless steel. These steel particles may become embedded into the stainless steel and cause rust.
- Avoid the use of chlorine bleach. Chemicals in bleach will cause pitting, corrosion, and discoloration of the metal.
- Despite the name, stainless steel actually can stain and mark. However, stains and marks do not affect the integrity of the metal.

## Periodic Maintenance

Periodically inspect the following areas:

- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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## **Midmark Corporation**