
Urology Drain System (Plant Installed) 9A329003

[English](#)
[Español](#)
[Français](#)

Applies to Models:
230 (-001, -002 and -004)

Special Tools:
none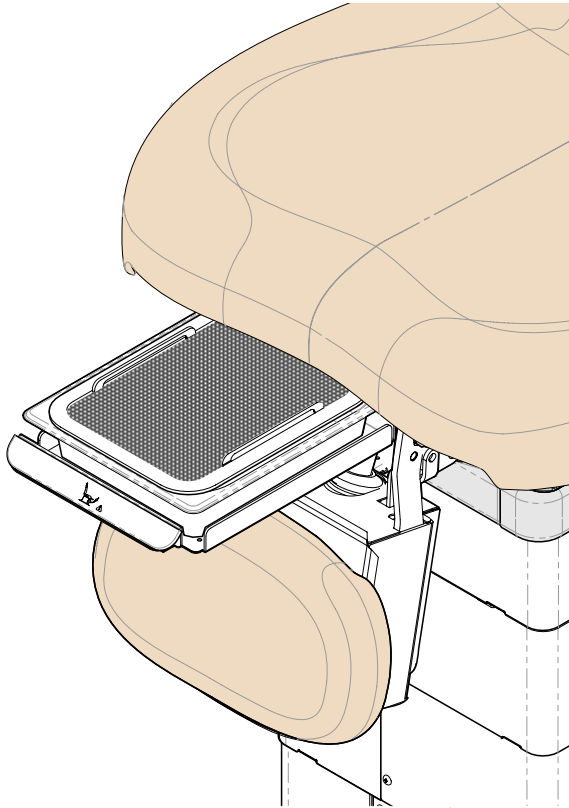

Set Up / User Guide

003-10331-99 Rev. AA3 (10/11/23)

Product Information

Dealer:

Date of Purchase:

Model Number:

Midmark Authorized Service Company:

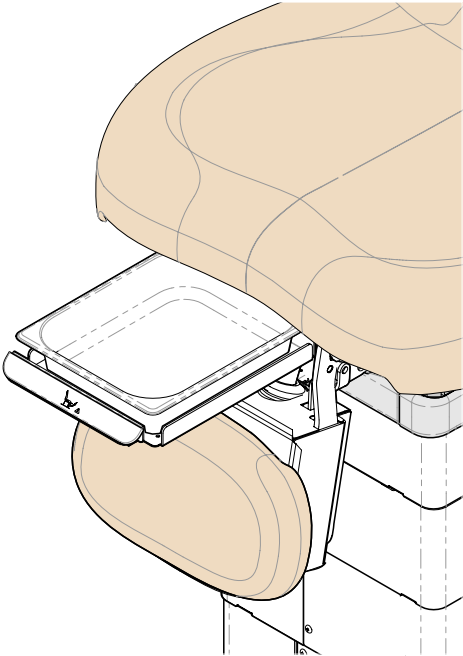
Table of Contents

Important Information

- [Product Information](#) 2
- [Product Registration](#) 4
- [Operating Conditions](#) 4
- [Disposal of Equipment](#) 4
- [Authorized Representatives](#) 4
- [Important Information](#) 6
- [Intended Use](#) 6
- [Restrictions](#) 6
- [Safety Instructions](#) 6

Set Up

- [Set Up Instructions](#) 7

Operation

- [Operating Instructions](#) 9

Maintenance

- [Calling for Service](#) 10
- [Cleaning](#) 10
- [Periodic Maintenance](#) 10

Warranty Information

- [Limited Warranty](#) 11

Product Registration

SCAN ME

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)

Relative Humidity.....30% to 75% (non-condensing)

Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

Warning

*In the event of any serious incident in relation to the device,
please contact Midmark and the appropriate competent authority.*

Customers in Australia should direct all questions, incidents and complaints to the Midmark Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Customers in India should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel:+91 22 4915 3000
Fax:+91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Authorized Representatives - continued

Customers in the United Arab Emirates should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

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Associated Medical Supplies Company LTD.
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice, or condition.

Symbol Glossary

Manufacturer

Serial Number

Catalogue Number

Intended Use

To provide drainage for urology procedures.

Restrictions

Not for use with rotational base units.

Safety Instructions


Warning
No modification of this equipment is allowed.


Warning
Do not sit on the drain pan / drain pan holder assembly.

Set Up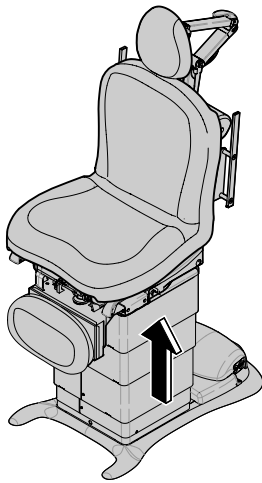


Step 1:
Run BASE function all the way UP.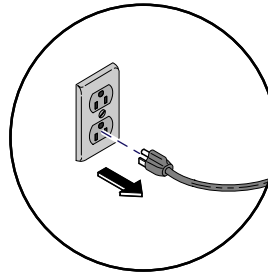

Step 2:
Unplug Power Cord.

Step 3:
Remove two screws from wire harness cover.


Note: Keep screws to secure cage bracket.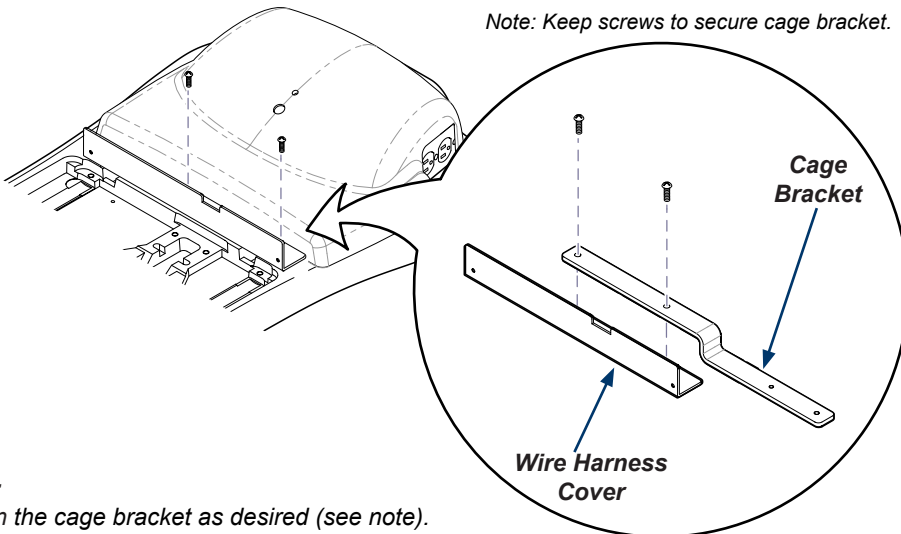

Step 4:
*Position the cage bracket as desired (see note).
Secure with two screws.*

Note: The cage bracket can be positioned on either side of chair.

Set Up - continued

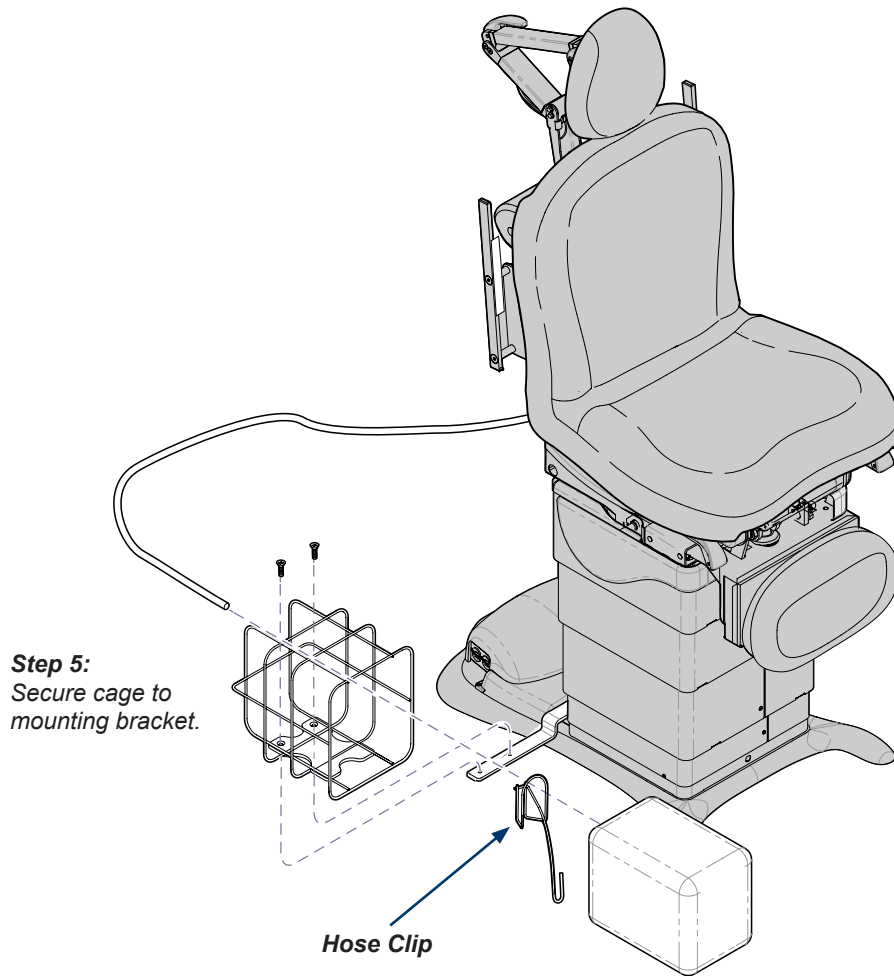Step 5:
Secure cage to
mounting bracket.

Hose Clip

Step 6:
Attach hose to hose clip.
Insert hose clip into bottle.
Install bottle into cage.

Operation

Attention!

The Foot Up function is disabled when the treatment pan is pulled out.

(You will hear several “beeps” if Foot Up is activated with the pan pulled out.)

To resume Foot Up operation:

Push the pan all the way in.

Press Foot Up button.

To access the drain pan:

Lower the foot section.

Pull the drain pan slide out.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark® directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The accessory is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the accessory.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces using a clean soft cloth and mild cleaner.

Equipment Alert

Do not use bleaching agents or any abrasive materials / substances on stainless steel surfaces (i.e. bleach, steel wool, wire brush, scouring powder, etc.). Failure to comply may result in damage to the accessory.

Stainless Steel

Frequent cleaning will prolong the life and help maintain the natural luster of stainless steel:

- When cleaning, always rub in the direction of the “grain” of the metal.
- Avoid use of steel wool or steel brushes when cleaning stainless steel. These steel particles may become embedded into the stainless steel and cause rust.
- Avoid the use of chlorine bleach. Chemicals in bleach will cause pitting, corrosion and discoloration of the metal.
- Despite the name, stainless steel actually can stain and mark. However, stains and marks do not affect the integrity of the metal.

Periodic Maintenance

Periodically inspect the following areas:

- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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© Midmark Corporation 2020
Midmark Corporation