Gynecology Debris Tray 9A555 / 9A556

[English](#)
[Deutsch](#)
[Français](#)
[Español](#)
[Dansk](#)
[Suomi](#)
[Italiano](#)
[Svenska](#)
[Norsk](#)
[Nederlands](#)
[Polski](#)
[Türkçe](#)

Applies to Models:

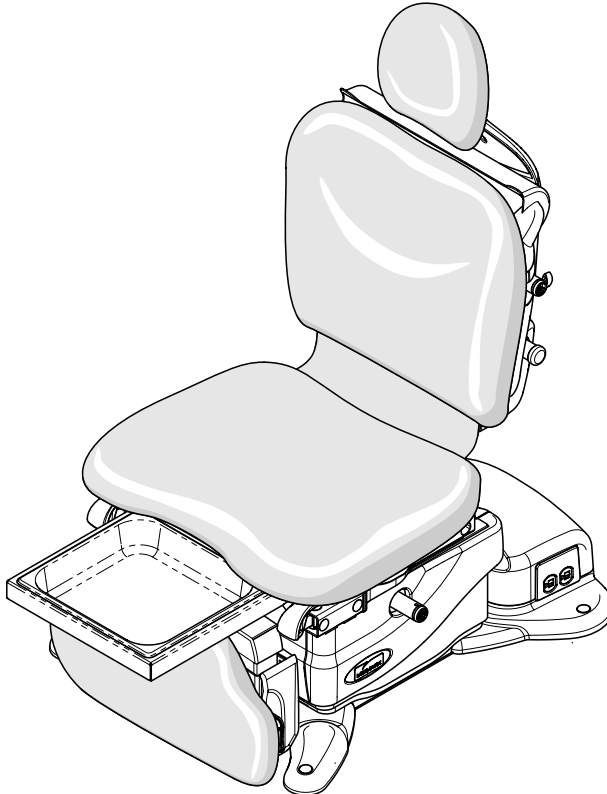
630 (-010 thru -013)

630 (-020 thru -023)

631-001

Special Tools:

none

Set Up / User Guide

Style U

003-10332-99 Rev. AA5 (2/28/25)

TP202 20-42-FO-00014 Rev A1 C2169

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

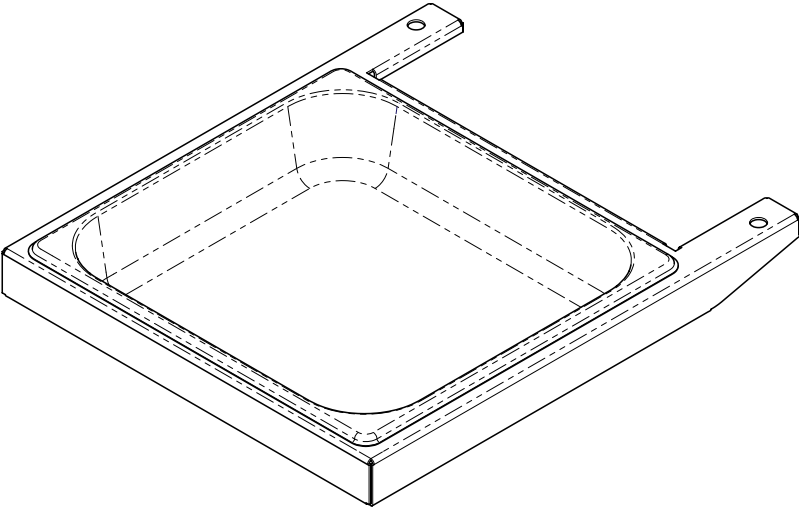
Table of Contents

Important Information

Product Information 2

Product Registration 4

Transportation / Storage Conditions 4

Operating Conditions 4

Disposal of Equipment 4

Authorized Representatives 4

Important Information 6

Intended Use 7

Safety Instructions 7

Set Up

Set Up Instructions 8

Operation

Operating Instructions 9

Maintenance

Calling for Service 10

Cleaning 10

Periodic Maintenance 10

Warranty Information

Limited Warranty 11

Product Registration

SCAN ME

Transportation / Storage Conditions

Ambient Temperature Range:.....-5°C to +38°C (+23°F to +100°F)
Relative Humidity.....10% to 90% (non-condensing)
Pressure Limit.....7.2 Psi to 15.3 Psi (50 kPa to 106 kPa)

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

EC REP

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in the EU should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

CEpartner4U
Esdoornlaan 13
3951 DB Maarn, The Netherlands

Phone: +31 343 442 524
Fax: +31 343 442 162

Customers in the UK should direct all questions, incidents, and complaints to Midmark's UK Responsible Person listed below.

UKCApartner4U Ltd.,
7 CAMPION WAY, BINGHAM, NOTTINGHAM NG13 8TR, UK.
www.UKCApartner4U.co.uk

Phone : +44 7905 384429

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Sponsor listed below.

Carlos Nafarrate SA de CV
Avenida Las Américas 601,
Ladrón de Guevara, 44600,
Guadalajara, Jalisco, México
<https://www.nafarrate.com/> [nafarrate.com]

Phone: (33) 3669 9800

Authorized Representatives - continued

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India

Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)

Mumbai, 400022

Tel:+91 22 4915 3000

Fax:+91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd

Abdullah M. Al Khalifi Street

Al Nakheel Dist. II

P.O. Box 116105

Jeddah (East)

21391

Saudi Arabia

Tel: + 966 2 286 4707

Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.

Al Barsha,

Dubai, UAE

Phone: +971 4 501 4000

Fax: +971 4 501 4100

Customers in Kuwait should direct all questions, incidents, and complaints to the Midmark Authorized Representative:

Central Circle Co.

P.O. Box 1015 Salmieh

22011 Kuwait

Al-Sharq

Khalid Bin Al-Waleed Street

Al-Kazmi Building 10

Third Floor

Email : cencico@centralcircleco.com

Phone : +965 22411758, +965 22411748, +965 22444228

Fax :+965 22445457

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Emergo Costa Rica S.R.L.

Edificio Ribleh 2 Piso, Diagonal a la Biblioteca Municipal

Heredia, Belén

La Ribera, Costa Rica

E-mail: LAvigilance@ul.com

Phone: +(506) 2201-14 53

Customers in Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tecnidental Panama

VILLA MAGNA, VILLA ZAITA

OFFICE B8 PANAMA

Panama

E-mail: tecnidentalpanama@gmail.com

<https://tecnidentalpa.com/> [tecnidentalpa.com]

Phone: +507 6416-2235, +507 6492-0884

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading

Midmac Roundabout

Opposite Al Mana Petrol Station

Doha, Qatar

Tel: +974-44337000

Fax: +974-44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.

HaTnufa St 7

Petah Tikva, Israel

Tel: +972 3-925-4040

Fax: +972 3-924-9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice or condition.

Symbol Glossary

Pressure Limit

Proper shipping orientation

Fragile

Manufacturer

Medical Device

Temperature Limit

Keep dry

Humidity

Serial Number

Catalogue Number

Intended Use

For use with gynecological exams and procedures.

Safety Instructions

Warning

No modification of this equipment is allowed.

Warning

Do not sit on Debris Tray.

Warning

Do not press any button during a procedure.

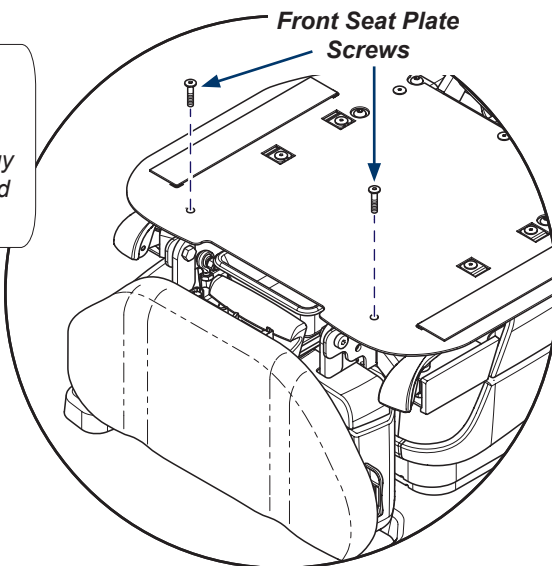
Set Up

Equipment Alert


On the 631-001 the front RH and LH seat plate screws must be replaced with the longer screws provided with the Gynecology Debris Tray. The longer screws are required to "lock" the accessory in place.

- A) Lift seat upholstery.
- B) Remove two (shorter) front seat plate screws.
- C) Install two (longer) front seat plate screws.
- D) Place the seat upholstery on the chair insuring that the front is up against the seat plate and centered.
- E) Press on seat upholstery firmly to secure.

Caution

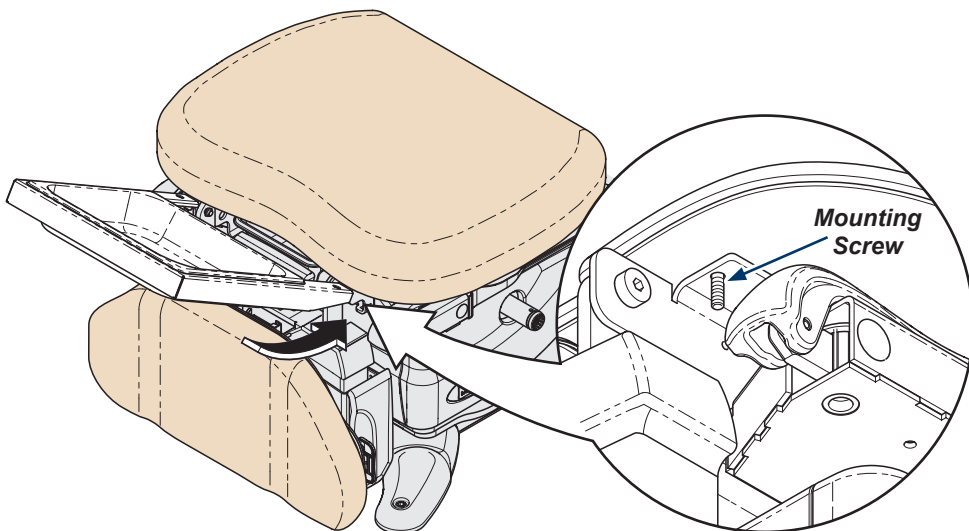Do not operate Foot section with debris tray attached.


- A) Position debris tray holder as shown.
- B) Align holes in debris tray holder with two mounting screws.
- C) Assure debris tray holder is locked in place.

Operation

**Caution**

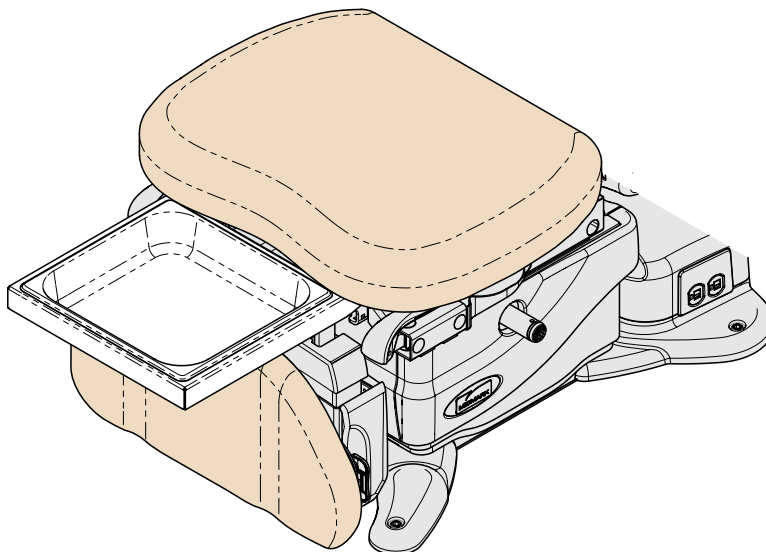Do not operate Foot section
with debris tray attached.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The accessory is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the accessory.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces using a clean soft cloth, and mild cleaner.

Equipment Alert

Do not use bleaching agents or any abrasive materials / substances on stainless steel surfaces (i.e. bleach, steel wool, wire brush, scouring powder, etc.). Failure to comply may result in damage to the accessory.

Stainless Steel

Frequent cleaning will prolong the life and help maintain the natural luster of stainless steel:

- When cleaning, always rub in the direction of the “grain” of the metal.
- Avoid use of steel wool or steel brushes when cleaning stainless steel. These steel particles may become embedded into the stainless steel and cause rust.
- Avoid the use of chlorine bleach. Chemicals in bleach will cause pitting, corrosion, and discoloration of the metal.
- Despite the name, stainless steel actually can stain and mark. However, stains and marks do not affect the integrity of the metal.

Periodic Maintenance

Periodically inspect the following areas:

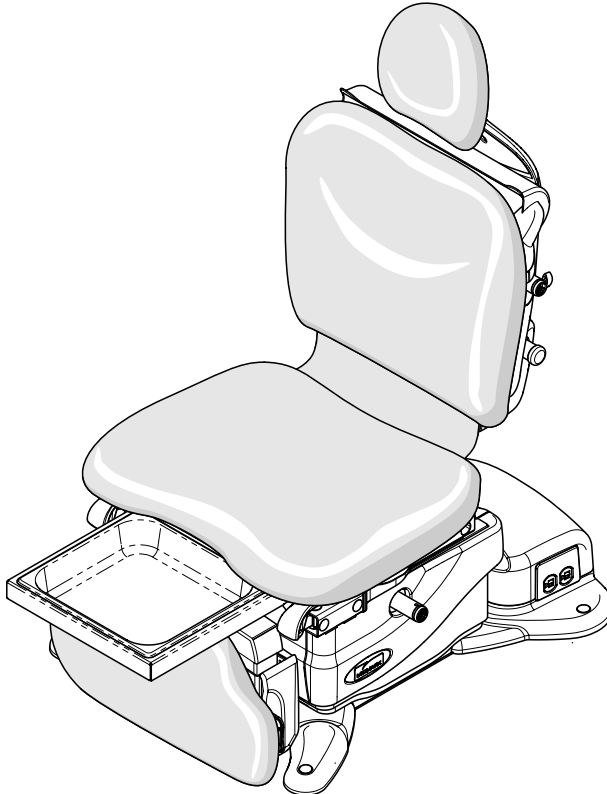
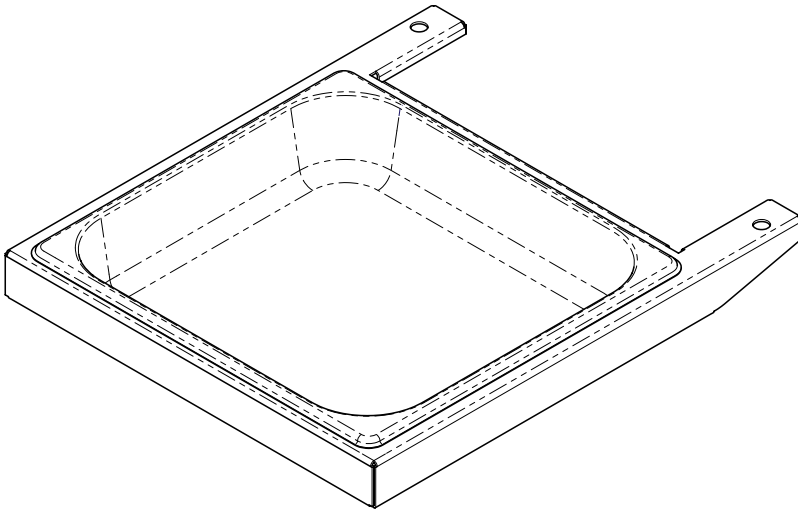
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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© Midmark Corporation 2020
Midmark Corporation