
# ! **Equipment Alert**

***Use these installation instructions when replacing an existing Data Logger only.***

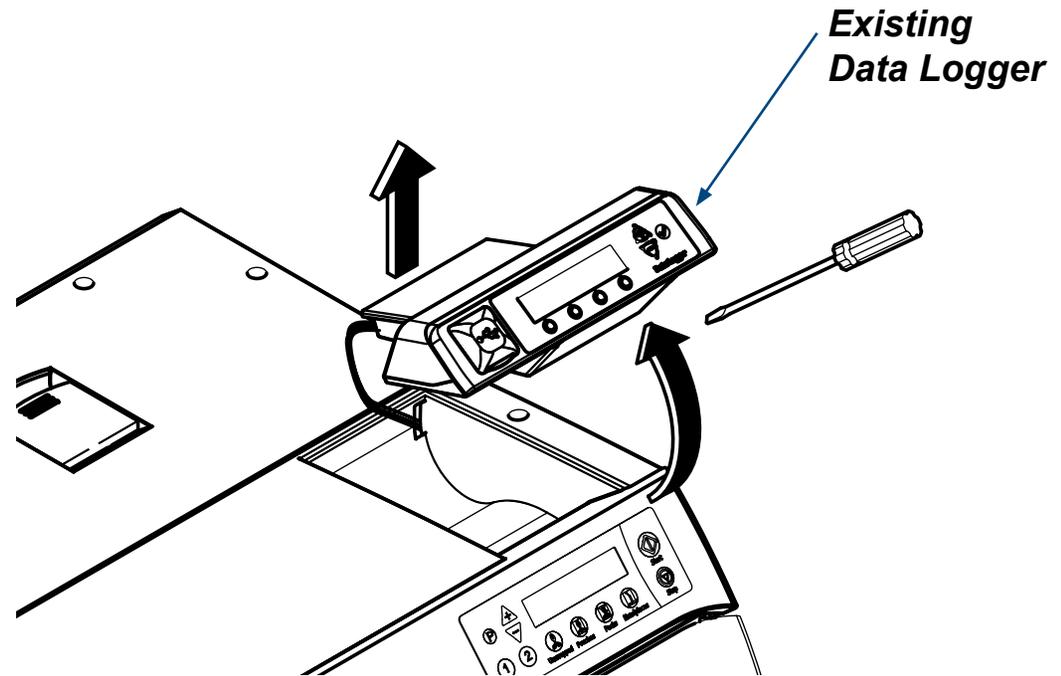
**Caution**

*Failure to follow these instructions when replacing an existing data logger could result in data loss.*

### Applies to Models:

M9	(-020 /-021 /-022)
	(-033 /-034)
	(-040 /-041 /-042 /-043)
M9-D	(-020 /-022 /-042)
M11	(-020 /-021 /-022)
	(-033 /-034)
	(-040 /-041 /-042 /-043)
M11-D	(-020 /-022)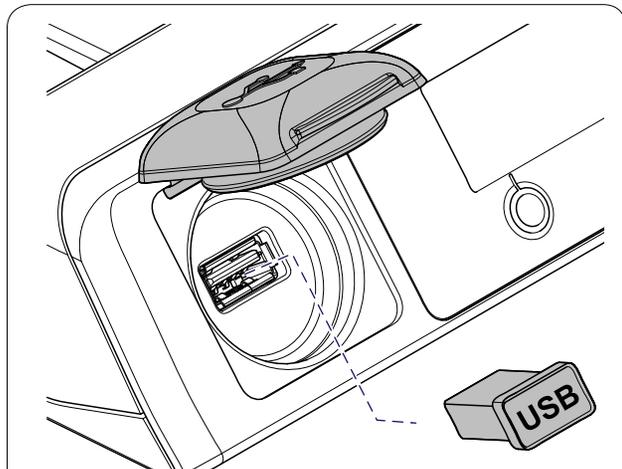

### Caution

The file folder labeled as the Sterilizer Serial Number will need to be renamed to prevent data loss.

### WARNING


To prevent risk of shock always disconnect power before removing covers or performing any service procedure.

### Step 1: Backup Data Onto PC and Rename Folder on PC.

Remove the USB device from the Data Logger. Plug USB into a PC and navigate to the File Explorer on the PC. Open up the USB folder on the PC. Navigate to the correct folder labeled as the sterilizer serial number. Transfer data to the PC. Remove the USB from the PC.

**Rename the folder on the PC currently labeled as the Sterilizer Serial Number to prevent data loss. Midmark suggest renaming the folder by adding the current date after the serial number (IE: VXXXXXXXXX 4-12-2020).**


Consult with an appropriate IT professional for support with file transfer if required.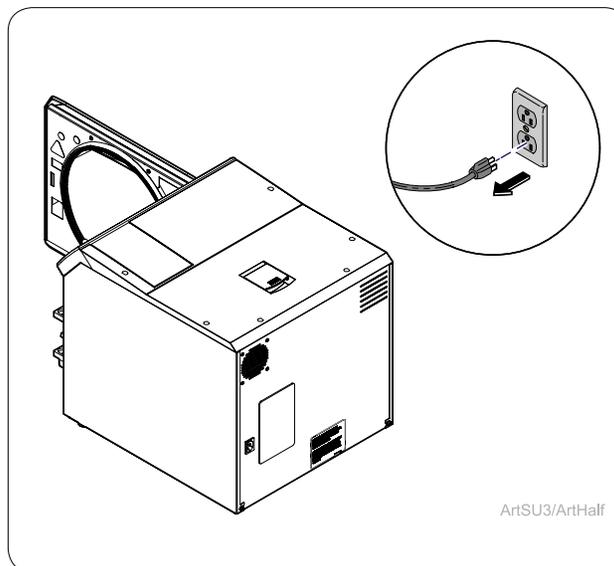

ArtSU3/ArtHalf

### Step 2: Unplug Sterilizer.


Unplug power cord from wall outlet or power inlet on back of the sterilizer.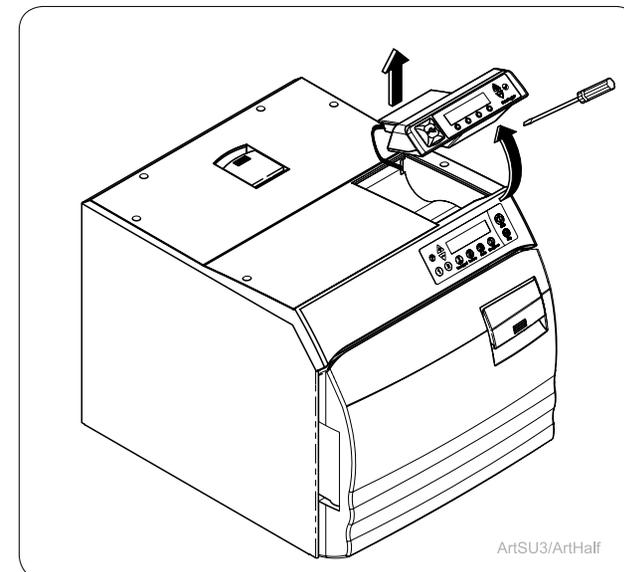


ArtSU3/ArtHalf

### Step 3: Remove Existing Data Logger.


Remove Data Logger from top right of the sterilizer. Remove any double-sided tape left on the sterilizer.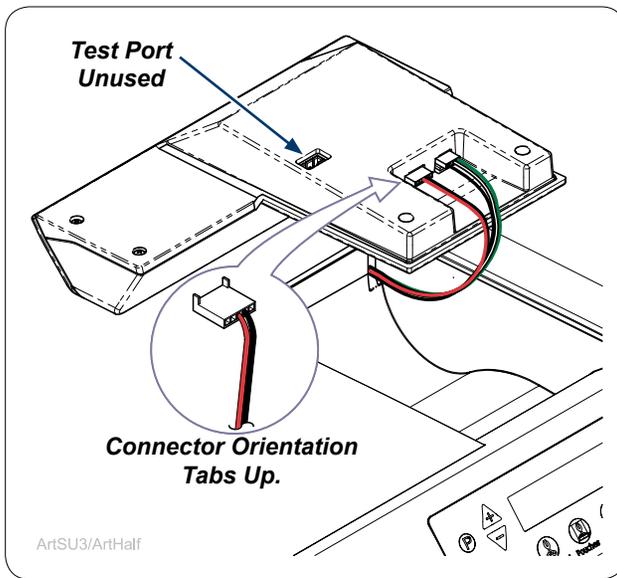


**Step 4: Connect wires to Data Logger.**  
Remove wires from existing Data Logger and connect them to the replacement Data Logger.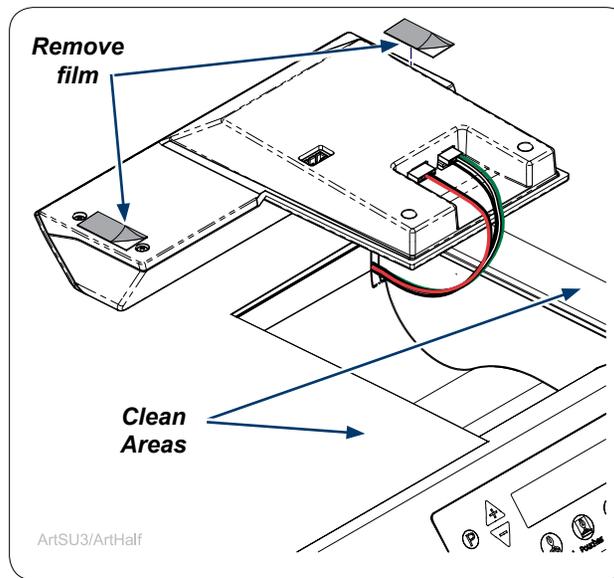


**Step 5: Remove film from tape.**  
Remove film from double sided tape.  
**NOTE:** Thoroughly clean top cover of sterilizer where Data Logger adheres.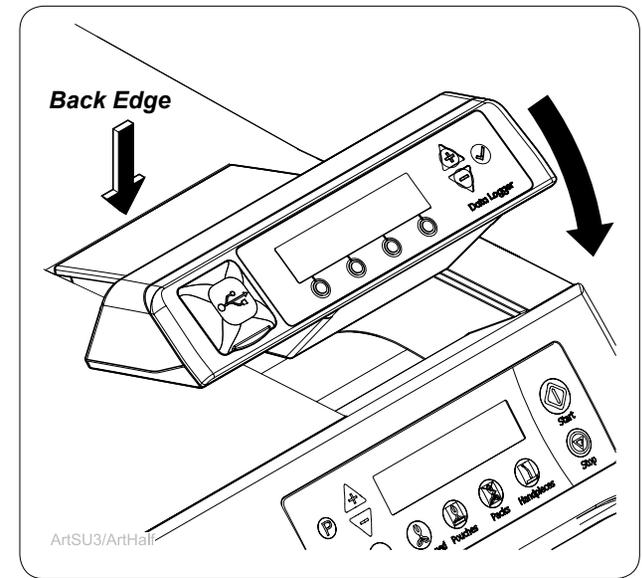

**Step 6: Install Data Logger.**  
Line up back edge of Data Logger into back edge of sterilizer cavity. Make sure the wires do not interfere with placing the Data Logger and rotate front edge of Data Logger down into place. Press down firmly on the Data Logger to secure.

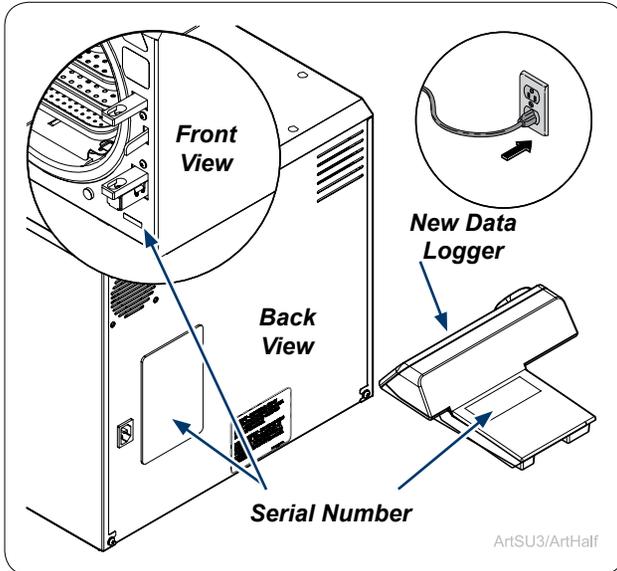
**Step 7: Plug in Sterilizer & Record SN's.**  
Plug power cord into wall outlet and power inlet on sterilizer back. Record serial numbers of sterilizer and Data Logger.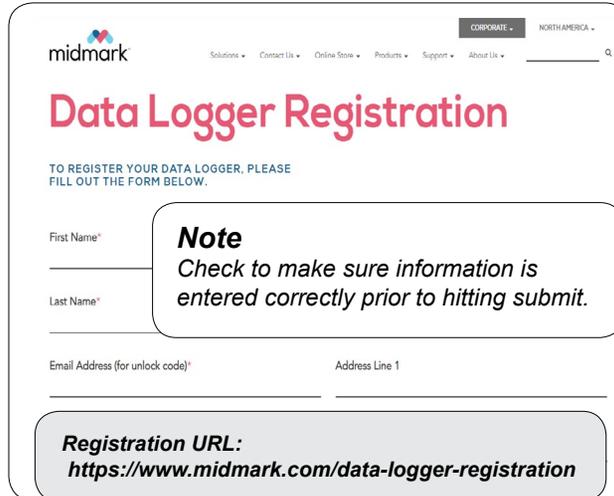

**Step 8: Obtain Unlock Code.**  
Access the Data Logger Registration web page at the on-line address above. Complete the Data Logger registration form on the web page to obtain an Unlock Code for the Data Logger that will be sent to you by e-mail. Once you have the Unlock Code, you can choose to continue the Data Logger set-up using either a manual or on-line method.

**Caution**  
Prior to discarding the original Data Logger. Make sure all electronic documents are transferred to a PC and the file folder on the PC is renamed to prevent data loss.

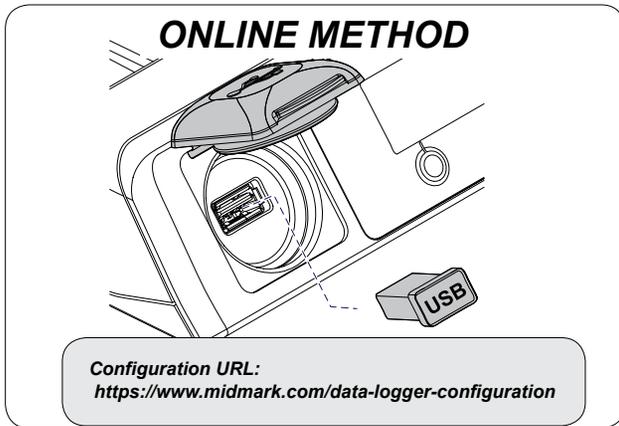
**Online Method Step 1: Setup.**

Remove the USB device from the Data Logger. Transfer the USB device to an internet enabled computer. Navigate to <https://www.midmark.com/data-logger-configuration> using the computer. Complete the form on the page. Once the form is completed, a file named "config.ini" will be downloaded onto the computer. Locate the file in the computer's "Downloads" folder. Copy this file to the Data Logger's USB device installed on the computer. Make sure the "config.ini" file has been successfully copied to the USB device. Do not rename the file. Consult with an appropriate IT professional for support with downloading the config.ini file if required.


**Online Method Step 4: Press Enter (☑).**

Press ENTER (☑) to proceed with normal operation. If an error occurs press ENTER (☑) and restart the online method.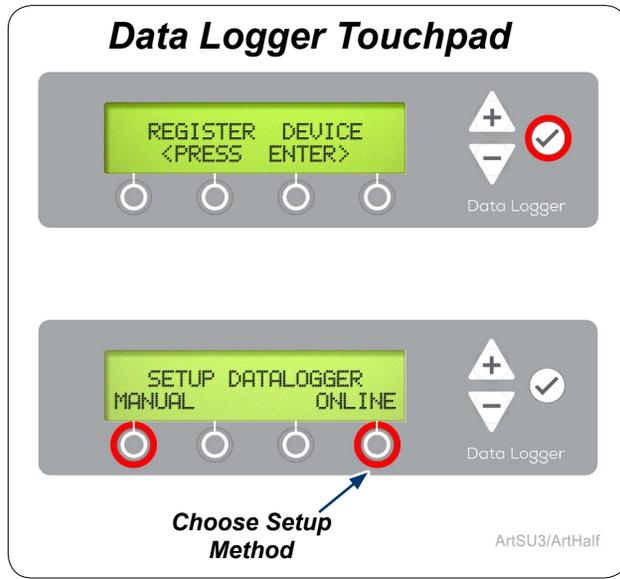

**Online Method Step 2: Data Logger Setup.**

Once you have obtained the Unlock Code. Use the Data Logger Touchpad to press ENTER (☑). Then select the Online method of setting up the Data Logger.

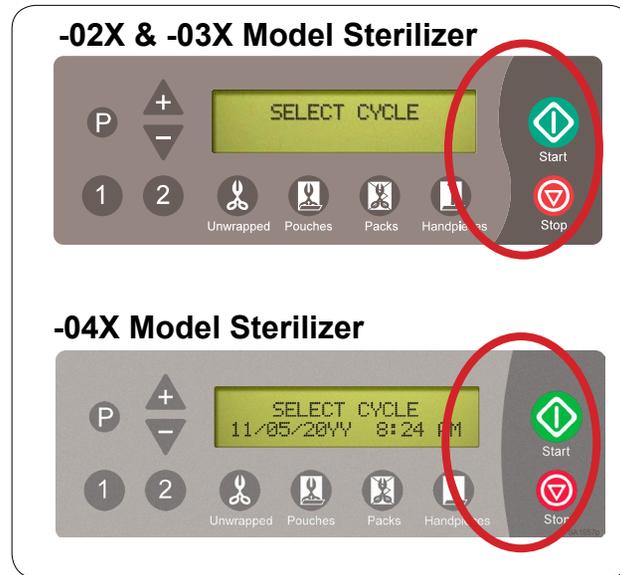
**Online Method Step 5: Identify Sterilizer Model.**

Use images to identify model of sterilizer where Data Logger is installed before completing following step. If you have an -04X model sterilizer setup is finished.

**Online Method Step 3: Install Files.**

Reinstall the USB device containing the downloaded "config.ini" file into Data Logger and close cover. Wait for the "config.ini" file to automatically install on the Data Logger.

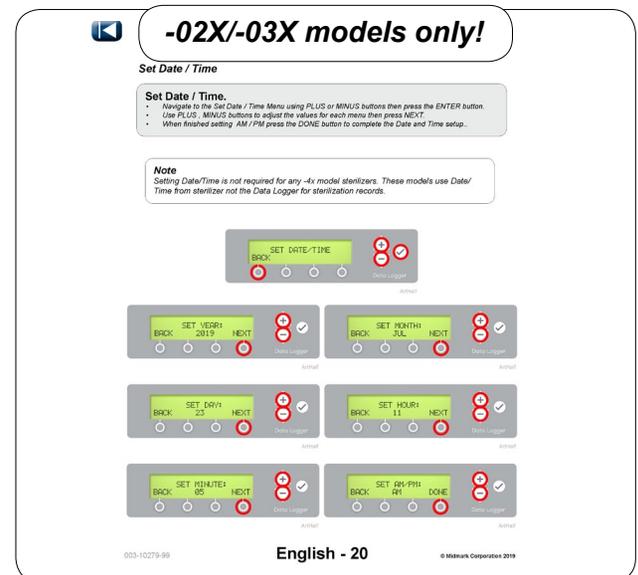**Online Method Step 6: Set Date/Time.**

Sterilizer models -02X and -03X require navigating to the SET DATE/TIME menus and entering Date & Time parameters. Refer to the last page for details.

### Navigating Setup Menus.

- **PLUS, MINUS** changes characters.
  - Press and hold to scroll through available characters when entering data.
- **NEXT** advances to next field.
- **DONE** advances to next menu when entering data is complete.
- **BACK** deletes the previous character/returns to the previous menu.

### Blank Spaces

The blank space character is located between A and 9 when scrolling through characters on the screen.

### Pressing ENTER (✓)

DO NOT press ENTER (✓) in any of the following steps until you have finished entering all information for that step. Refer to the User Guide for details on updating incomplete entries.

### Data Logger Touchpad

Choose Setup Method

ArtSU3/ArtHalf

### Manual Method Step 1 Data Logger Setup.

Once you have obtained the Unlock Code. Use the Data Logger Touchpad to press ENTER (✓). Then select the Manual method of setting up the Data Logger.

### MANUAL METHOD

### Manual Method Step 2: Enter Unlock Code.

Using the Data Logger Touchpad, enter the Unlock Code obtained in Step 8.

ArtSU3/ArtHalf

### Manual Method Step 3: Code Acceptance.

Press ENTER (✓) when the Unlock Code entry is accepted. If an error occurs, press BACK and return to the main menu.

ArtSU3/ArtHalf

### Equipment Alert

Ensure sterilizer serial number being assigned to Data Logger is correct so sterilization data records will be maintained accurately.

### Manual Method Step 4: Enter Serial Number.

Enter sterilizer serial number by using PLUS, MINUS, NEXT/DONE and BACK buttons.

Data Logger


ArtSU3/ArtHalf

### Note

Sterilizer Name will be part of electronic record on receipt. Setting up sterilizer name is optional.


### Manual Method Step 5: Set Sterilizer Name.

Press YES to name the sterilizer. Enter sterilizer name by using PLUS, MINUS, NEXT/DONE and BACK buttons. Or press NO to proceed to SETUP USER menu.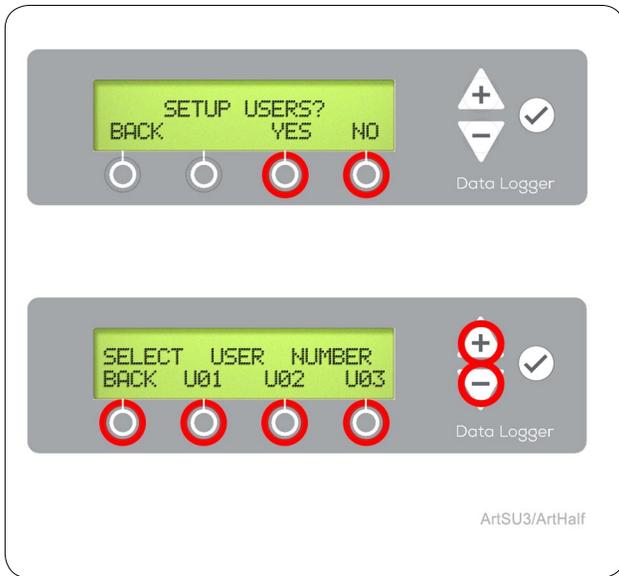


**Manual Method Step 6: Setup Users.**

From 1 to 12 users (designated U01-U12) can be set up in the Data Logger. Use PLUS, MINUS to scroll through the user identifiers and select them for set up.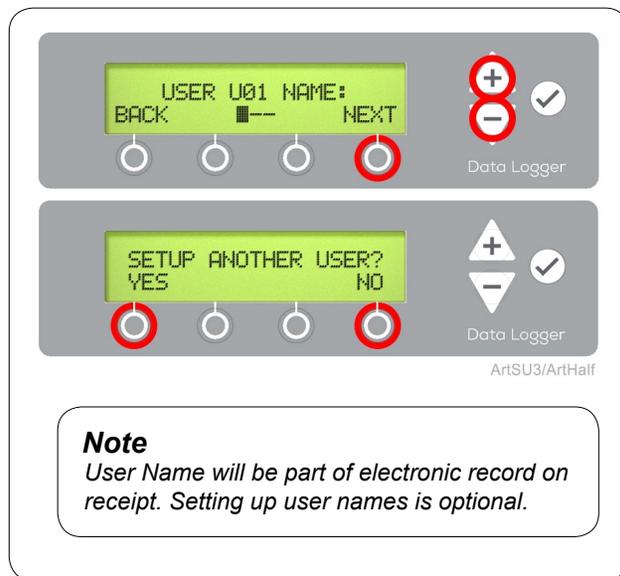


**Manual Method Step 7: Setup User Name.**

Enter a unique 1 to 3 character identifier for each user of the sterilizer. When all users have been entered and identified, select "No" when prompted to set up another user.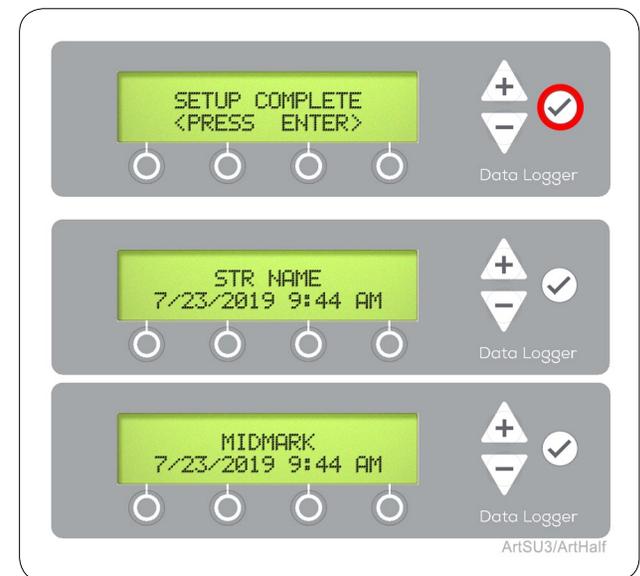


**Manual Method Step 8: Setup Complete.**

Press ENTER (✓) to complete setup. Main menu will appear with either sterilizer name if you entered one, or MIDMARK will appear as default. -04X models will not show Date/Time on Data Logger.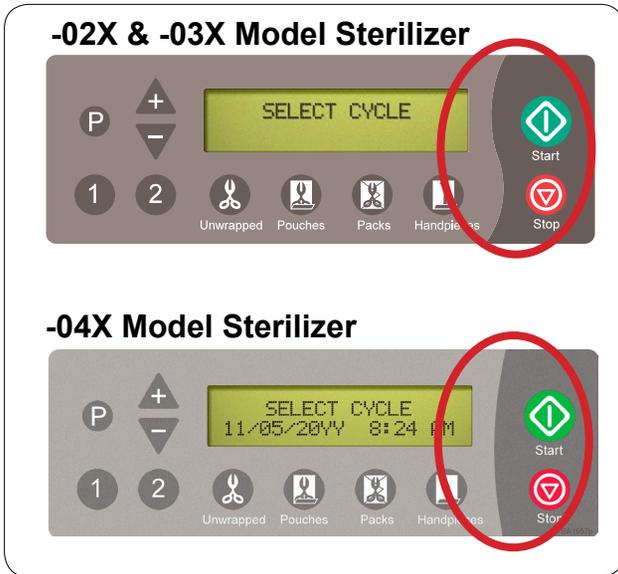


**Manual Method Step 9: Identify Sterilizer Model.**

Use images to identify model of sterilizer where Data Logger is installed before completing following step. If you have an -04X model sterilizer setup is finished.

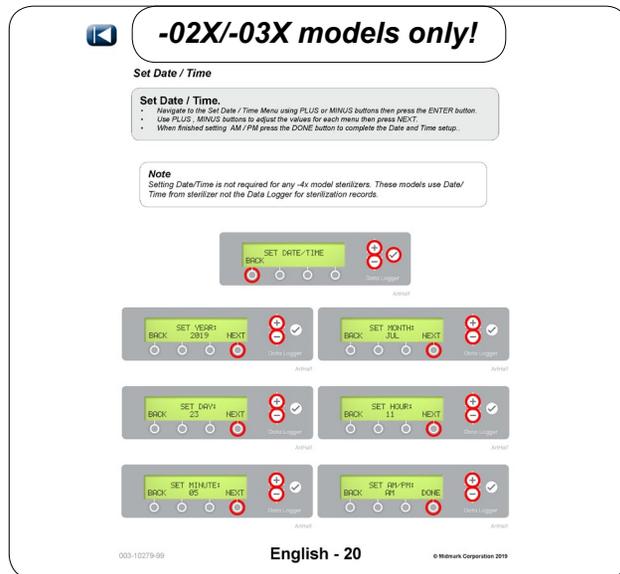**Manual Method Step 10: Set Date/Time.**

Sterilizer Models -02X and -03X require navigating to the Additional Menus and setting up the Date / Time parameters. Refer to the last page for details.

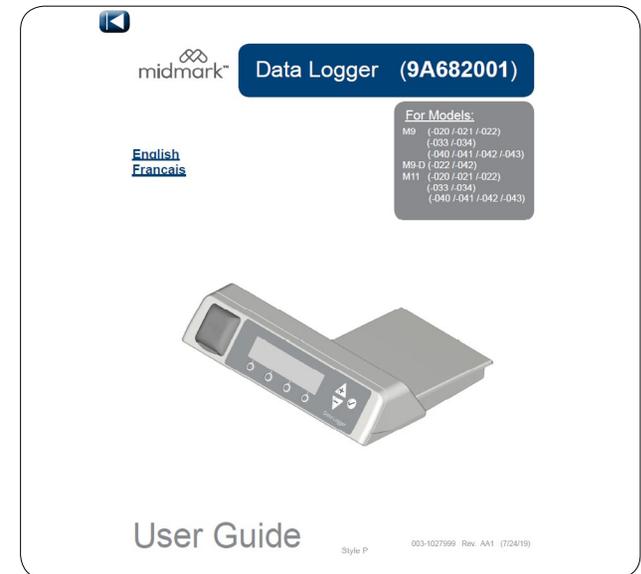**Refer to User Guide.**

Refer to user guide for additional menus and operation.

### Set Date / Time.

- Navigate to the Set Date / Time menu using the MINUS button then press ENTER (✓).
- Use PLUS, MINUS buttons to adjust the values for each menu then press NEXT.
- When finished setting AM / PM press the DONE button to complete the Date and Time setup.

### Note

Setting Date/Time is not required for any -04X model sterilizers. These models use Date/Time from sterilizer not the Data Logger for sterilization records.

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
Set Date / Time

**Set Date / Time.**  
• Navigate to the Set Date / Time Menu using PLUS or MINUS buttons then press the ENTER button.  
• Use PLUS / MINUS buttons to adjust the values for each menu then press NEXT.  
• When finished setting AM / PM press the DONE button to complete the Date and Time setup.

**Note**  
Setting Date/Time is not required for any -4x model sterilizers. These models use Date/Time from sterilizer not the Data Logger for sterilization records.

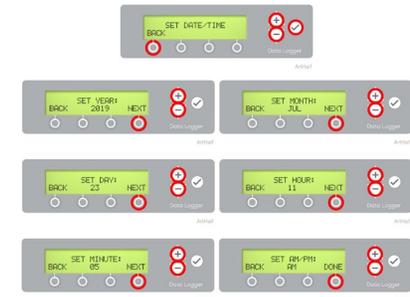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© Midmark Corporation 2019