


# Evacuation Filter Installation


**Applies to Models:**

77000037  
77000193  
77000338  
77000038

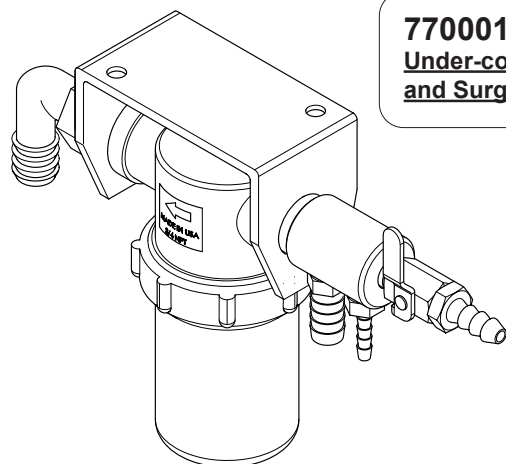

**77000193**

**Under-counter Mount, Hose Fitting Kit  
and Surgical Cut-off Valve**

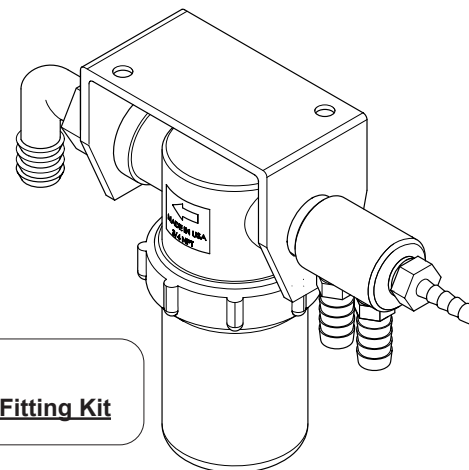


**77000037**

**Under-counter Mount and Hose Fitting Kit**


**77000038**

**Wall Mount 1 Large Hose Hanger, Hose  
Fitting Kit and Surgical Cut-off Valve**

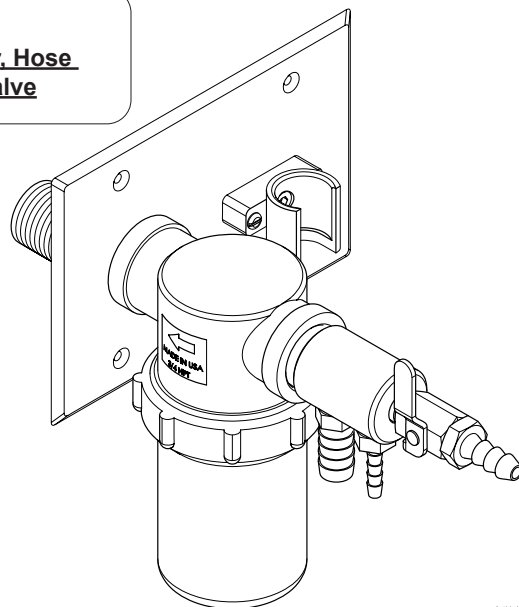


**77000338**

**Wall Mount 1 Large Hose Hanger  
and Hose Fitting Kit**

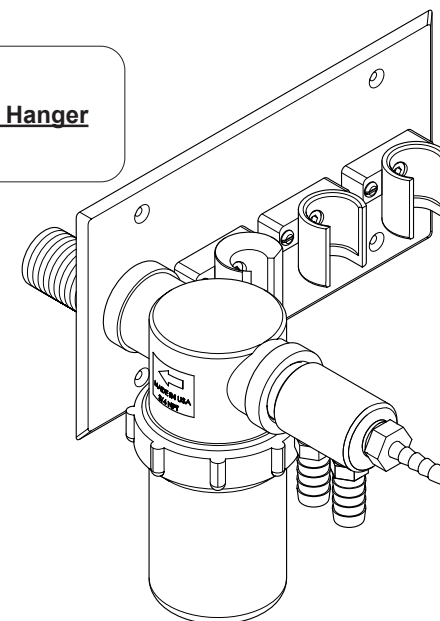


## Wall Mount Installation

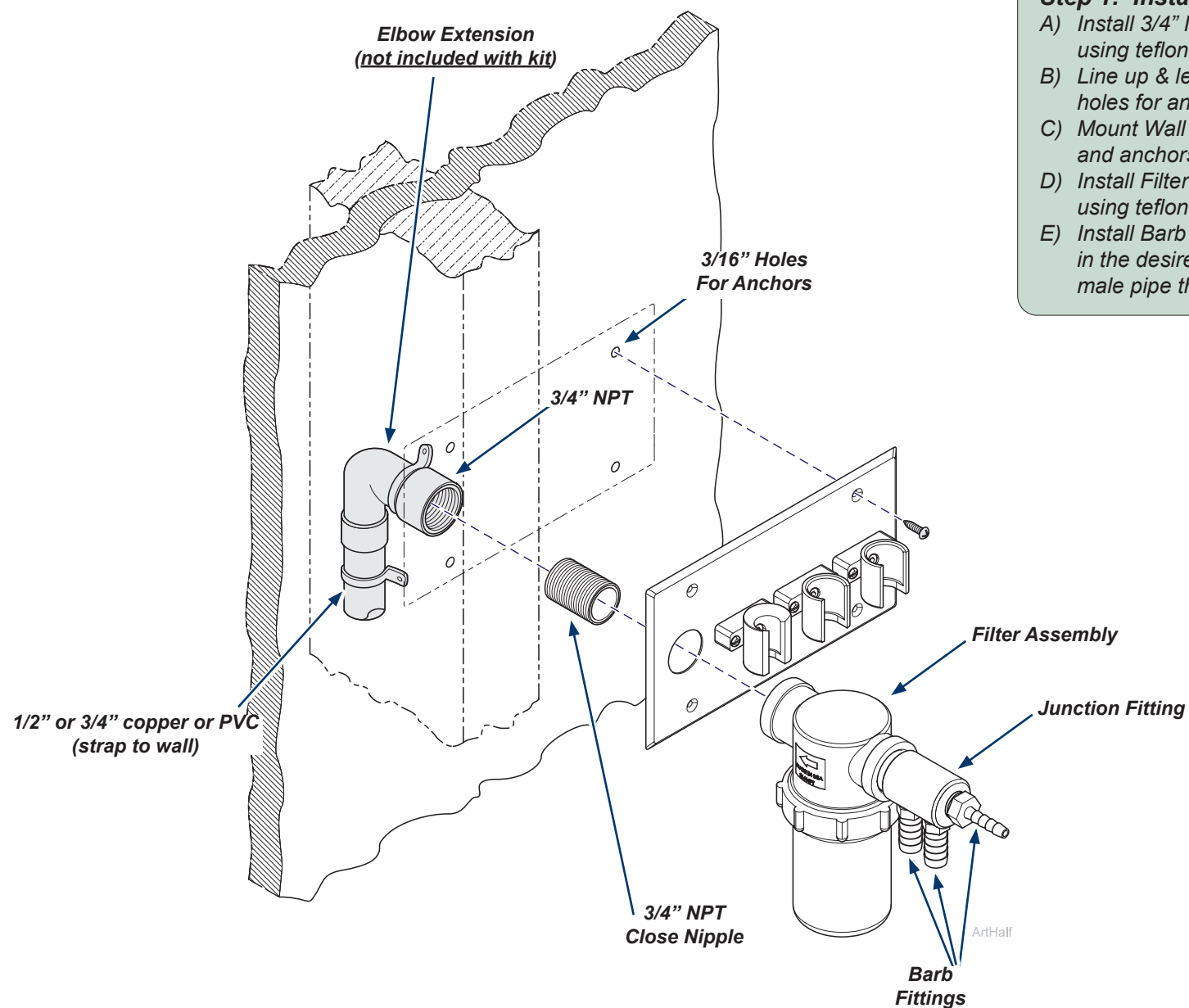

### Step 1: Install Wall Mount Assembly

- Install 3/4" NPT Close Nipple into Elbow Extension using teflon tape on the male pipe threads.
- Line up & level wall plate, mark and drill 3/16" holes for anchors.
- Mount Wall Plate to wall surface with screws and anchors provided.
- Install Filter Assembly onto 3/4" NPT Close Nipple using teflon tape on the male pipe threads.
- Install Barb Fittings or Plugs into Junction Fitting in the desired configuration using teflon tape on the male pipe threads.

## Under-Counter Mount Installation

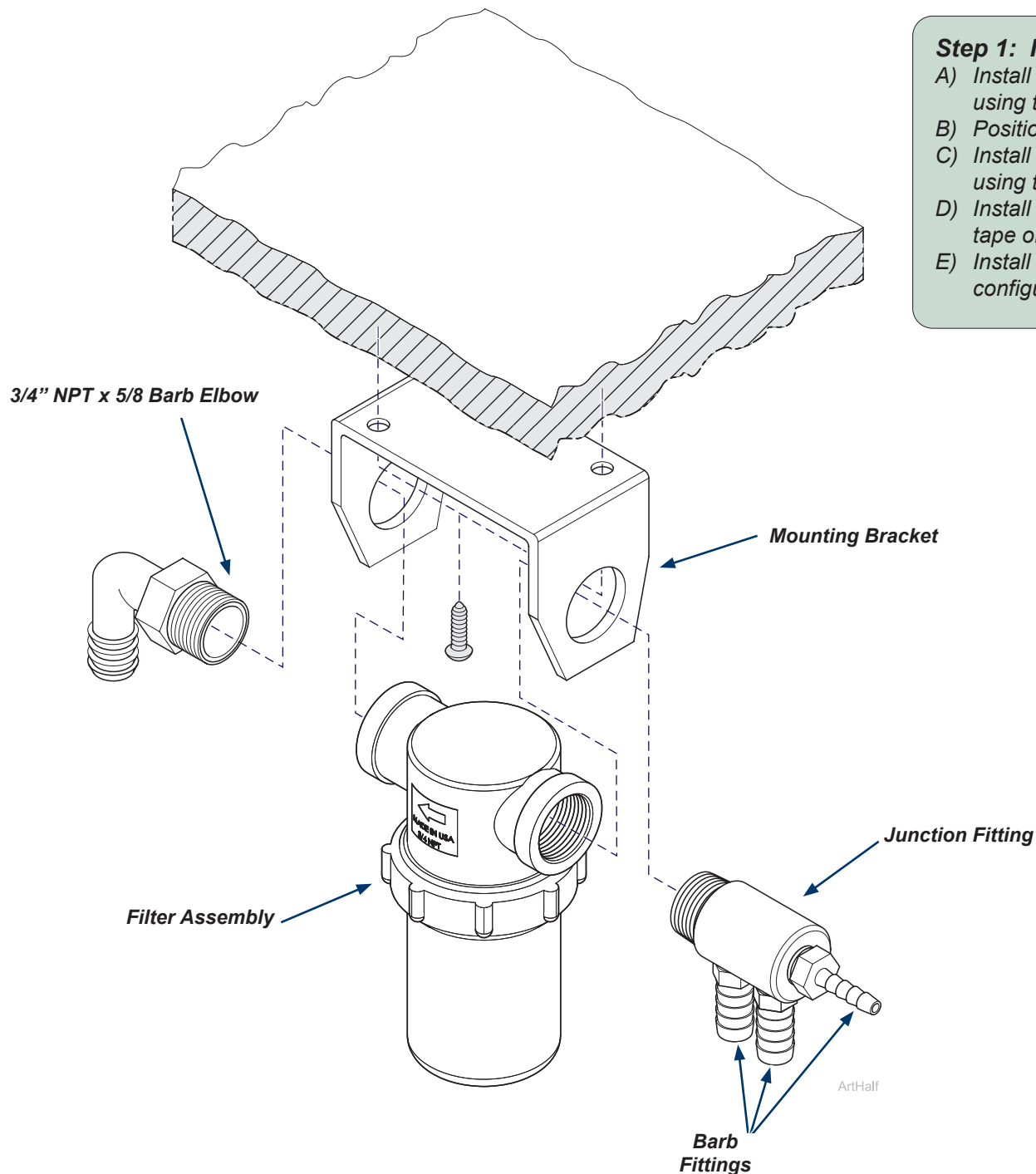

### **Step 1: Install Under-Counter Assembly**

- A) Install Mounting Bracket to the underside of counter surface using the appropriate fasteners for the counter material.
- B) Position Filter Assembly into Mounting Bracket.
- C) Install 3/4" NPT x 5/8" Barb Elbow into back of Filter Assembly using teflon tape on the male pipe threads.
- D) Install Junction Fitting into front of Filter Assembly using teflon tape on the male pipe threads.
- E) Install Barb Fittings or Plugs into Junction Fitting in the desired configuration using teflon tape on the male pipe threads.