

Components:

Diffuser Wrench.....	1
Diffuser.....	1
O-ring.....	1

Step 1: Remove bur or test pin from handpiece chuck.

Step 2: Insert large pin of Diffuser Wrench into handpiece chuck. Rotate Diffuser Wrench while applying a very slight amount of pressure toward handpiece until two small pins drop into place in diffuser.



Equipment Alert

The Diffuser has reverse threads. Rotate Clockwise To Loosen.

Step 3: Rotate Diffuser Wrench clockwise to loosen/remove the Diffuser and O-ring. Then discard used Diffuser and O-ring.

Step 4: Confirm O-ring is Installed onto back of new Diffuser and place on handpiece.



Equipment Alert

The Diffuser has reverse threads. Rotate Counter-Clockwise To Tighten.

Step 5: Rotate diffuser wrench counter-clockwise to tighten the diffuser until snug.

Note: Do not over tighten.

