


ArtQuarter

Note

This tool is used in a portion of the maintenance process for appropriate handpieces. For complete maintenance procedures, refer to the user guide. For best results, clean and lubricate the handpiece swivel bore during each complete handpiece cleaning. Complete handpiece cleaning is recommended after each patient to mitigate cross contamination between patients. This procedure may also be performed between handpiece cleanings if the handpiece appears to be under performing.

Step 1: Screw Swivel nozzle onto spout of handpiece conditioner can.

Step 2: Remove swivel from handpiece and Insert the nozzle into handpiece bore completely.

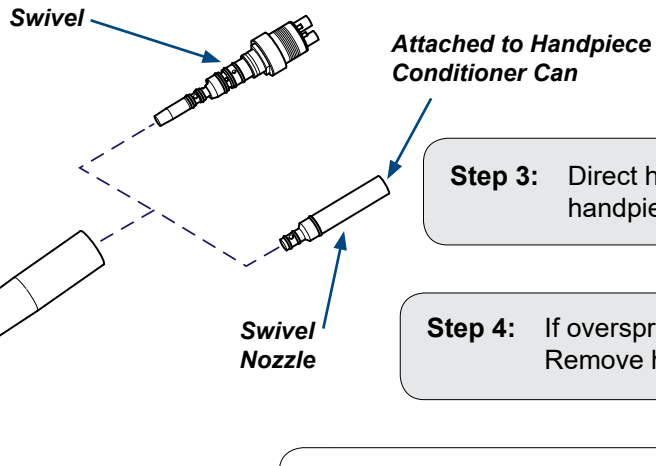

Step 3: Direct handpiece and nozzle away from your body and towards an absorbent towel. Depress handpiece conditioner can top to spray until conditioner flows from handpiece head (1 to 2 seconds).

Step 4: If overspray on the towel appears discolored repeat spray for an additional 2 seconds. Remove handpiece from nozzle and shake over towel to remove excess conditioner.

Note

Refer to user guide for complete cleaning procedures.