


# Midmark Multiparameter Monitor Software Update

**Applies to Models:**  
8019-001 thru -003  
Multiparameter Monitor

**Note**  
Latest available software is available through Midmark Technical Service Department at 1-800-MIDMARK.

**Attention**  
Prior to running this procedure be aware that this will require the monitor to restart, upon restart, ALL current patient alarm data will be purged.

**Step 1: Software Update / Power off**  
A) Contact Midmark Technical Service to download latest software.  
B) Ensure monitor is powered off before software update is run.  
C) If power is on, press power button on side of monitor to power down.

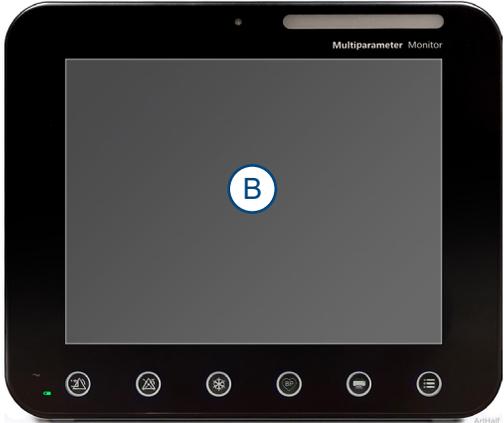


**Step 2: USB**  
A) Insert USB with 8019 software into rear of monitor.



TP201 Rev. A


### Step 3: Software Update

- A) Press power button on side of monitor to turn on power and start software update. Yellow light will flash during software upgrade.
- B) Calibration screen may appear once the software update is complete. This process is not necessary for software update. Press the on/off button on side of monitor to bypass calibration.
- C) Monitor may power off automatically after software update. If prompted to restart the monitor, press and hold power button to power off.

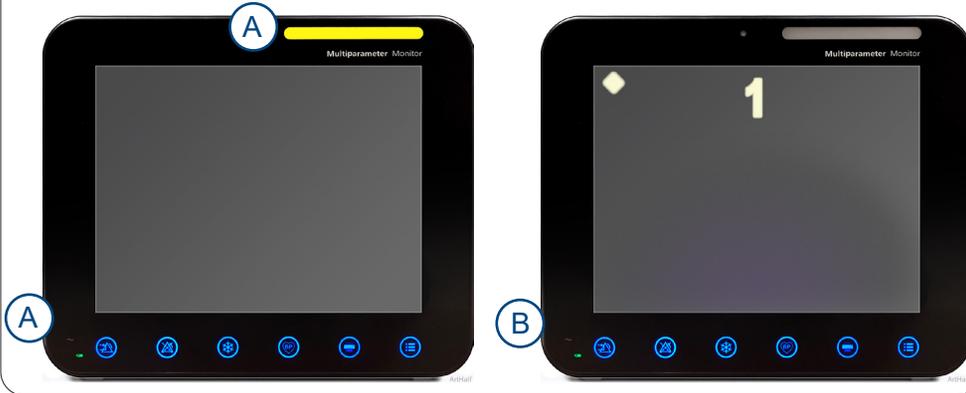

### Step 4: Removing USB

- A) With monitor powered off, remove USB from back of monitor.
- B) Press power button on side of monitor to turn on power.

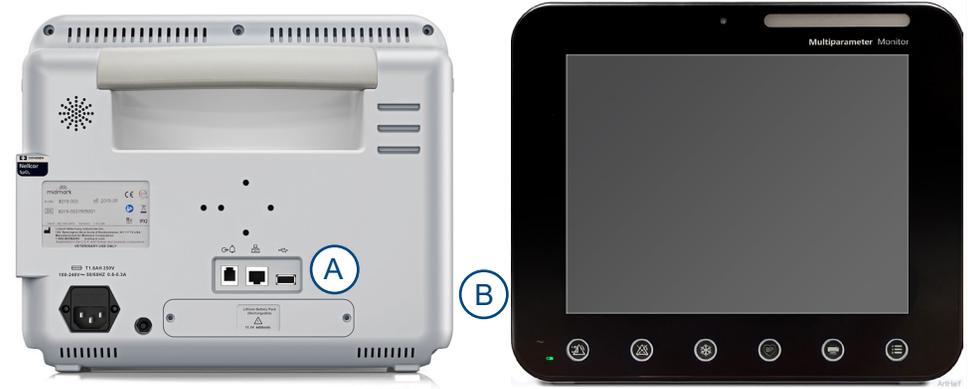

### Step 5: Checking Software Update

- A) Press "Settings" on Touch Screen on bottom display.
- B) Press "Monitor Info" on Touch Screen under Settings menu.



### Step 6: Verifying Software Update

- A) Check that software version matches the software version provided by Midmark Technical Service to confirm successful installation.
- B) Exit Monitor Info menu.

