



TLM Light Harness Guide Kit (002-10557-00) Setup Instructions


Special Tools:
Right Angle Screwdriver
(included in kit)

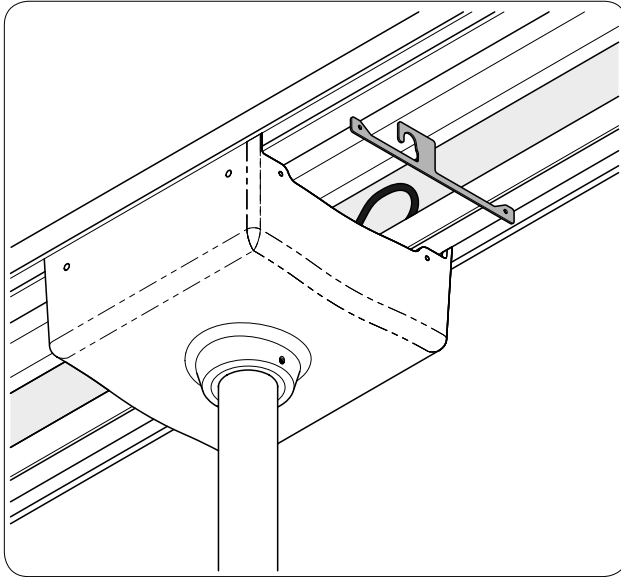


Step 1:
Orient the Guide as shown.
(With hook pointing to the Gray monitor harness)

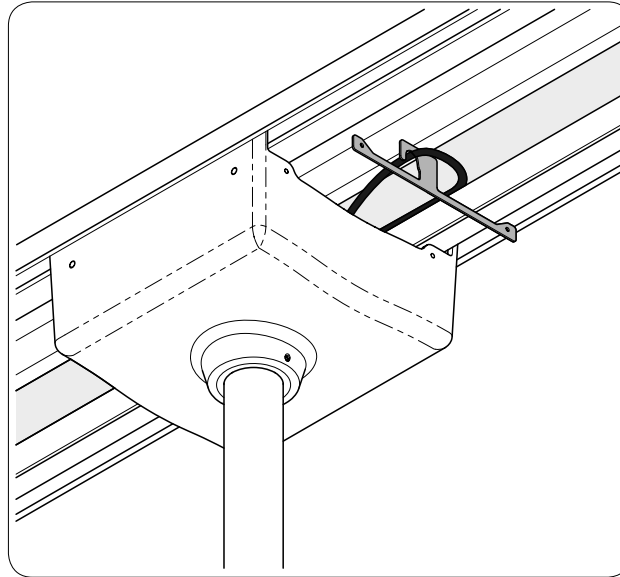


Step 2:
Insert the Black light harness into the hook of Guide.

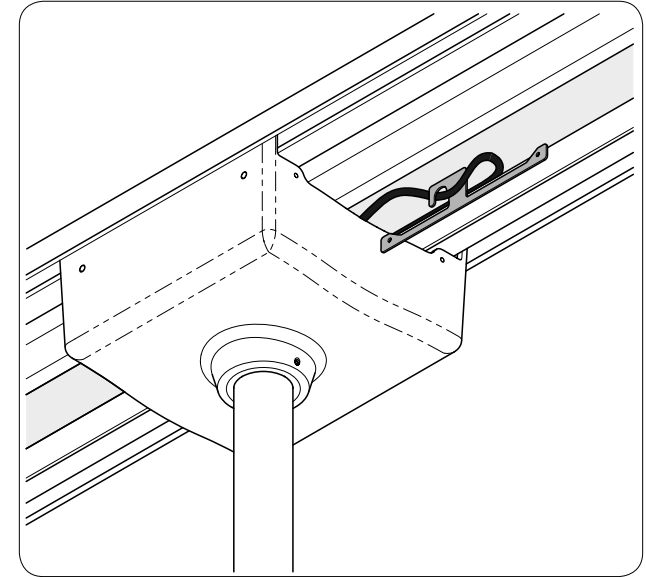


Step 3:
Rotate the Guide 90 degrees and insert between guards.

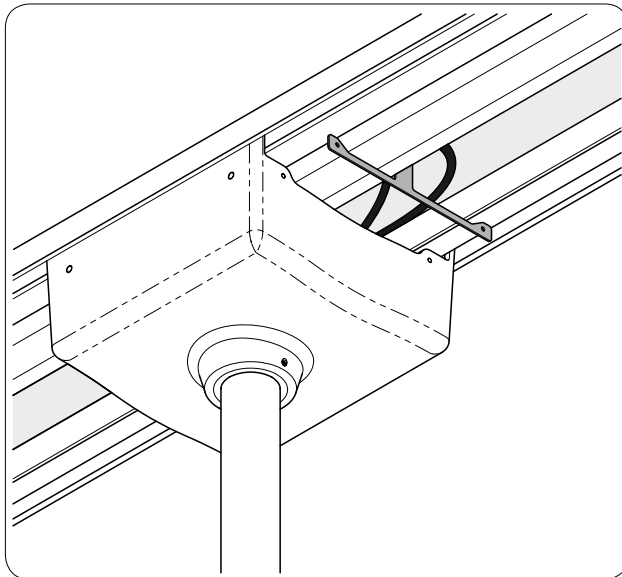

Step 4:
Rotate the Guide back to original orientation.

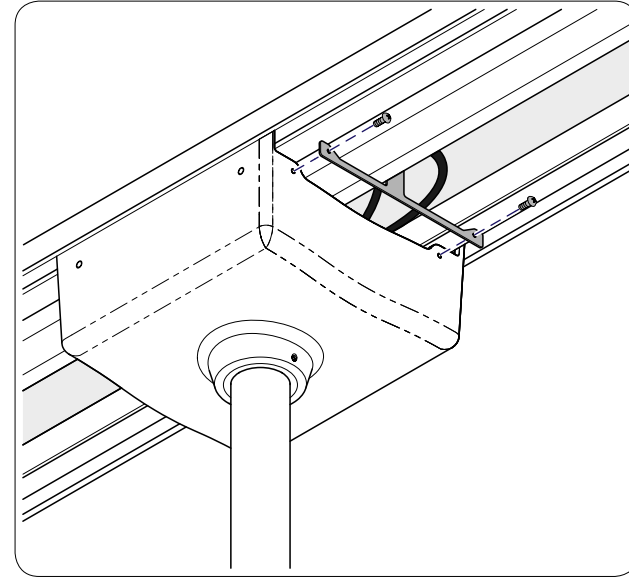


Step 5:
Secure the Guide to light trolley with two screws.

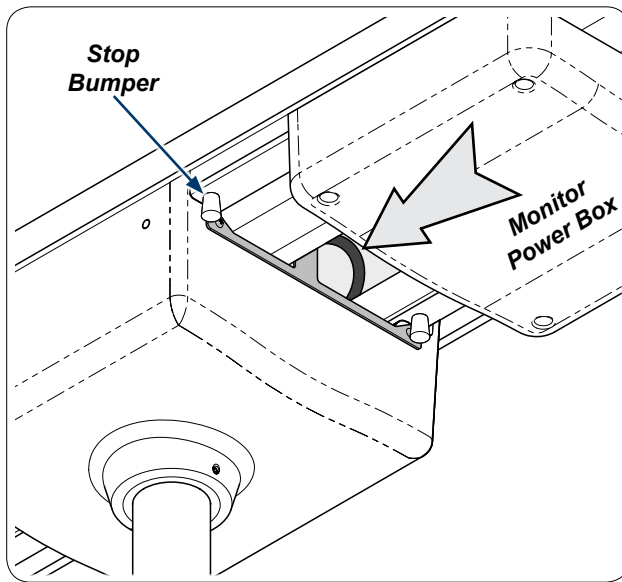

Step 6a:

Move trolley to stop bumpers. Light harness should extend past trolley but not contact Monitor Power box.

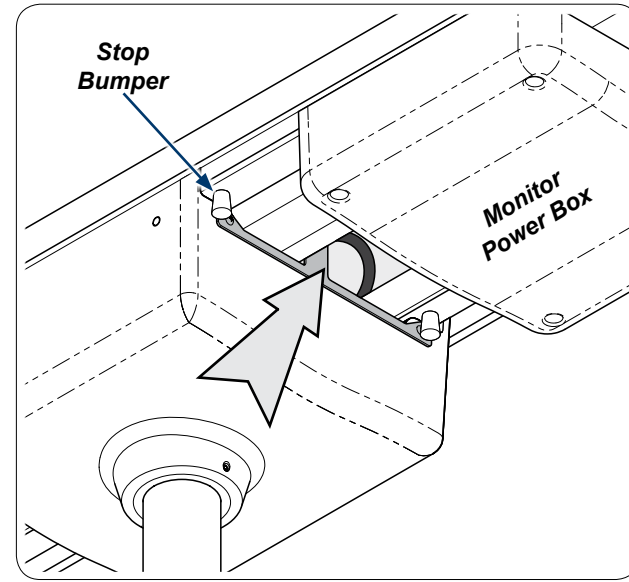

Step 6b:

With trolley against stop bumpers, light harness should not be tight or pulling against the Guide laterally.




Equipment Alert

Failure to check and adjust the harness set length may result in damage to the harness and or track light monitor assembly.

Note

If harness requires adjustment the cable clamps holding the light harness will need loosened.

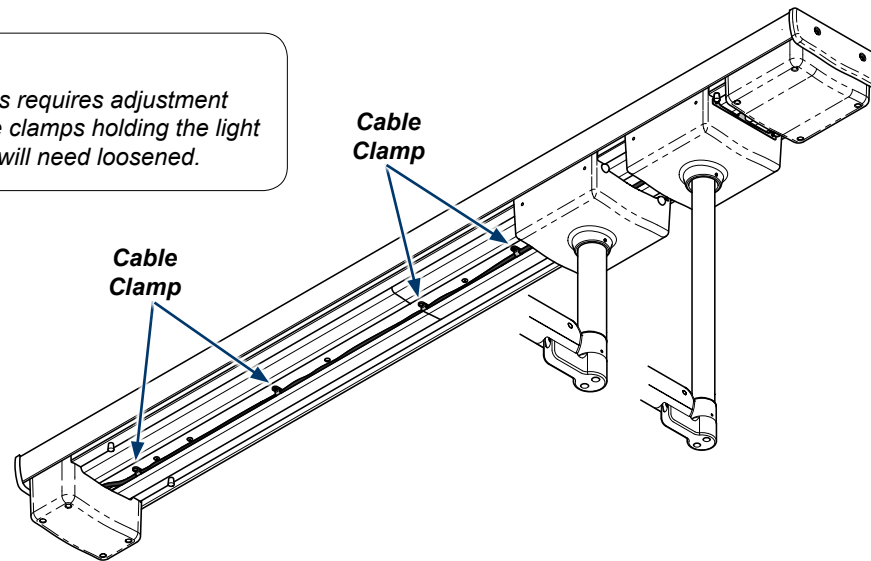

Step 7:

Move both trolleys to the Monitor Power box end of the track. Use a Right Angle screwdriver (included) to loosen cable clamps and adjust the harness set length, then retighten cable clamps.