

Handpiece Illumination System Replacement Kits [002-1251-11 & 002-1251-12] For Asepsis 21® Units mounted on Ultra Chairs

Contents:

Installation	1
Wiring Diagram	2
Tubing Diagram	3



Equipment Alert


Yellow (YL) connects to "I" Common on all handpieces. Orange (OR) connects to "B" and Green (GN) connects to "A" for power from transformer.
Numbered callouts designate hanpiece (HP) numbers (e.g., 1 = first HP, 2 = second HP, etc.).

Step 1: Remove the covers (Console or LR) and attach the Power Pack with the hook and loop strip(s).

Step 2: Connect the tubing and wiring from the umbilical to the Power Pack as illustrated.


Note: Remove Power Pack from hook and loop strip(s) for ease of installing the wiring and tubing.

Step 3: Connect the transformer leads to Orange and Green leads from the umbilical, then plug transformer into electrical mains.

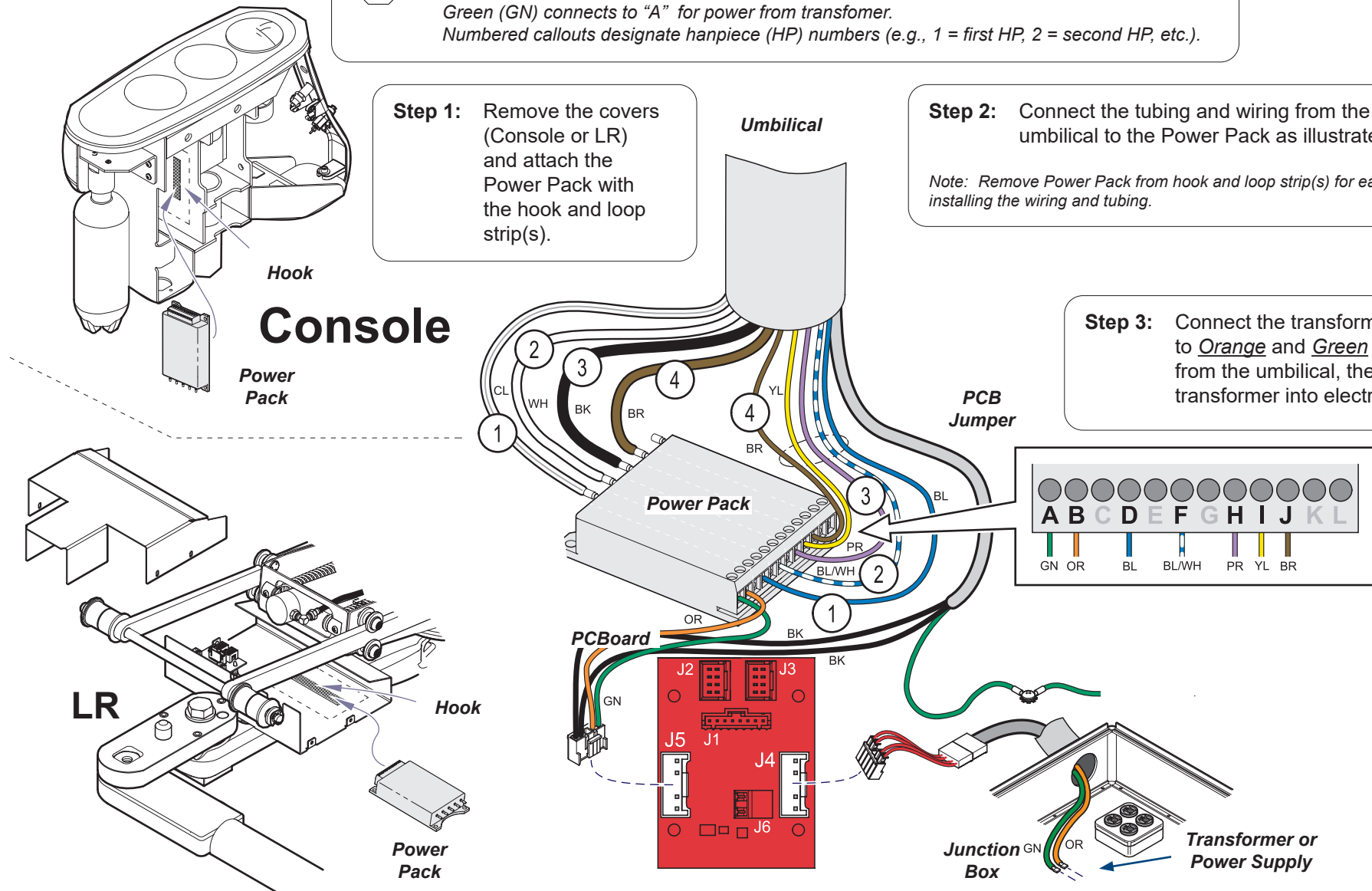


DA3696

Handpiece Wiring Diagram

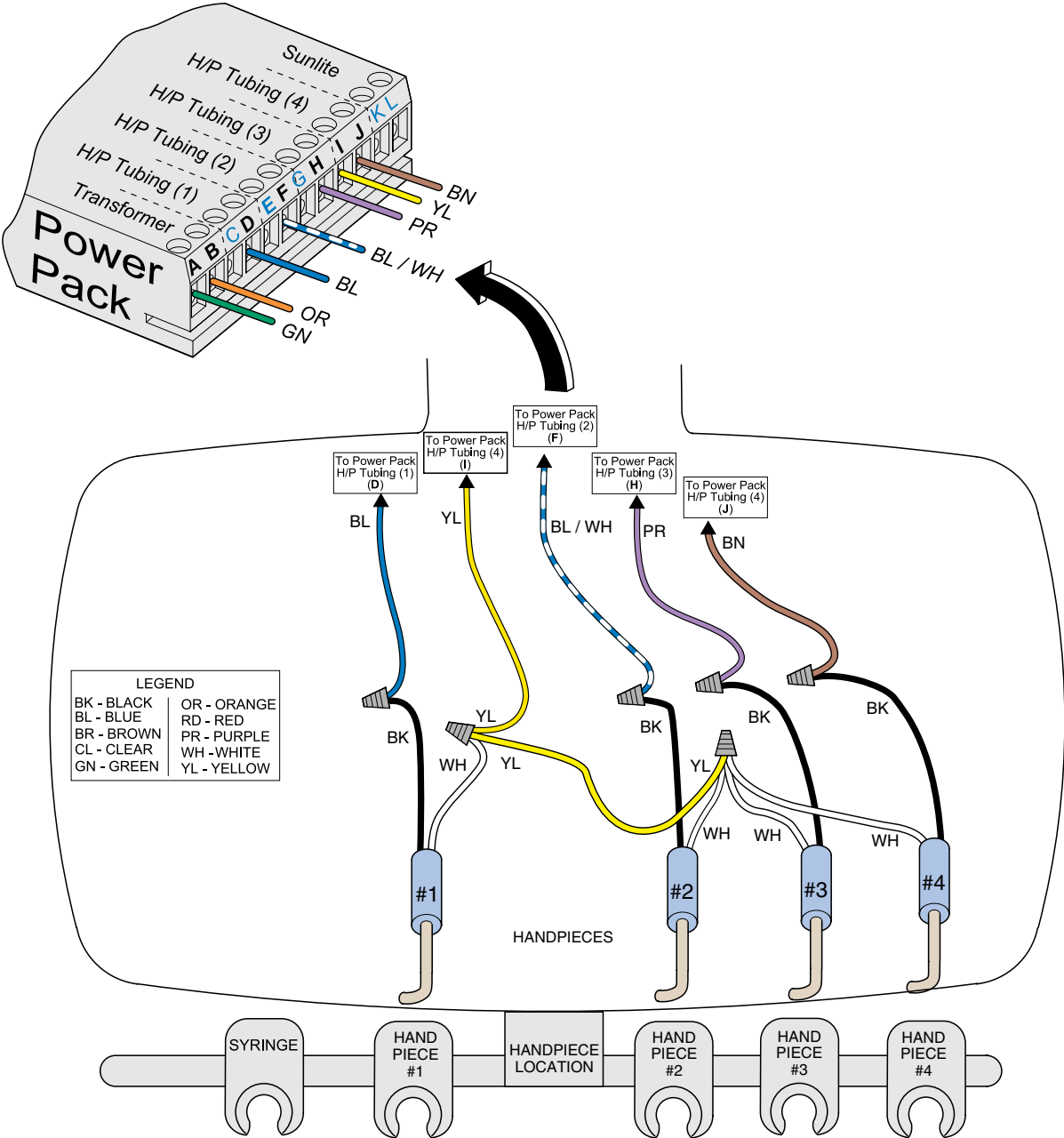

DA2350i

Tubing Flow Diagram

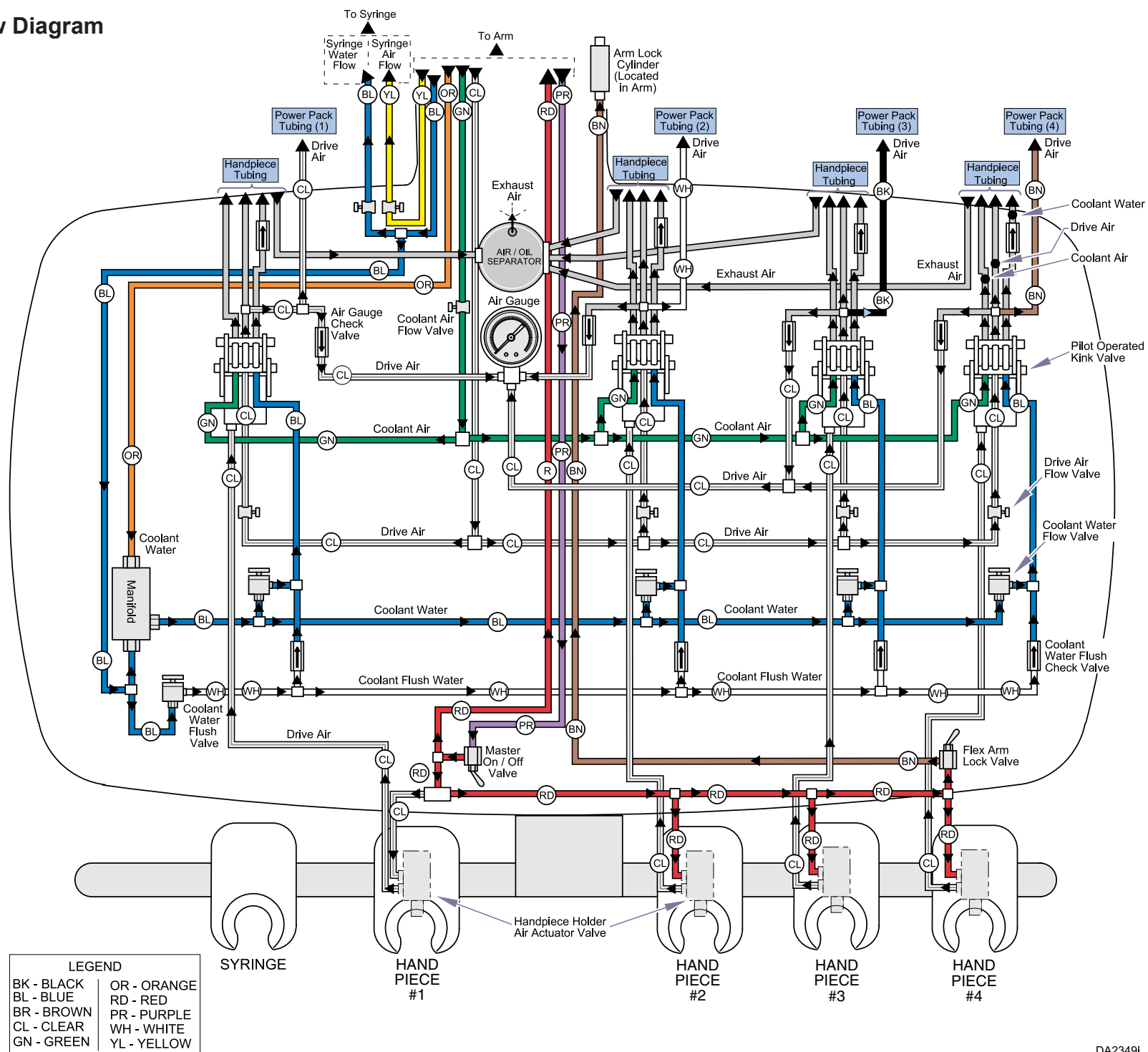

DA2349i