204

Manual Examination Table

Ritter®

[English](#)

[Español](#)

[Français](#)

User's Guide

Style P

003-10362-99 Rev AA5 (10/4/23)

TP202 20-42-FO-00014 Rev A1 C2169

Product Information

(The information below is required when calling for service)

Dealer [name / phone]:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company [name / phone]:

**Model / Serial
Number Location**

ArtFull

Table of Contents

Important Information

Safety Symbols	4
Warnings	5
Intended Use	6
Electrical Requirements	6
Electromagnetic Interference	6
Disposal of Equipment	6
Transportation / Storage Conditions	6
Authorized Representatives	7

Operation

Back Section Positioning	8
Stirrups	9
Footrest and Treatment Pan	10
Footstep	10
Paper Roll and Tear Strap	11
Drawer Heater (optional)	12
Pelvic Tilt (optional)	13
Receptacle (optional)	14

Maintenance

Cleaning	15
Periodic Maintenance	15
Calling For Service	15

Specifications

Weights and Dimensions	16
Upholstery Dimensions	17
Voltage, Current, Frequency Ratings and Compliance Chart	18

Warranty Information

Scope of Warranty	19
-------------------------	----

Important Information

Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note

Amplifies a procedure, practice or condition.

Proper shipping orientation

Fragile

Type B, Applied Part
(Upholstery)

Protective earth ground

Patient Weight Limit

Manufacturer


Serial Number

Keep dry

Maximum stacking height
(palletted units)

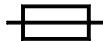Fuse rating specification

Dangerous voltage / shock hazard

Refer to manual

Catalogue Number

Medical Device

Warnings

WARNING

No modification of this equipment is allowed.

WARNING

To avoid injury, do not put head or extremities under the paper tear strap.

WARNING

Connecting equipment to the multiple socket outlet effectively leads to creating a Medical Electrical System and the result can be a reduced level of safety.

WARNING

Table must be positioned in normal use such that the power cords can be easily accessed to unplug.

WARNING

To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Intended Use

This product is intended to be used as a table to provide positioning and support of patients during examination procedures conducted by qualified medical professionals in an office environment.

Electrical Requirements

Caution
To completely isolate the table from electrical mains supply, both power cords must be unplugged.

- When using high frequency surgical devices or endocardial catheters:*
- *Use non-conductive material to insulate patient from metal portions of the table.*
 - *Consult operating instructions for the device before using in conjunction with drawer or upholstery heater.*

Failure to comply may result in electric shock or burns to patient.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room.
- Plug table into an isolated circuit.
- Increase separation between table and interfering device.
- Contact Midmark if interference persists.

Disposal of Equipment

At the end of product life, the table, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Transportation / Storage Conditions

Ambient Temperature Range:.....-5°C to +38°C (+23°F to 100°F)
Relative Humidity.....10% to 90% (non-condensing)
Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.

Calle 47 # 5-26

Bogota Colombia c.p. 110111

Tel. (+ 57 1) 756 87 87

atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA

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San Jose, Zapote, Costa Rica

Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.

VILLA MAGNA, VILLA ZAITA

OFFICE B8

PANAMA

Tel/Fax: 343-4049/ 6614-8029

tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios

Carlos Nafarrate SA de CV

Avenida Las Américas 601,

Ladrón de Guevara, 44600,


Guadalajara, Jalisco, México | Tel (33) 3669 9800

Operation

Back Section Positioning

To position the back section...

- A) While holding the back section in place, squeeze the Back Release Handle (either side).
- B) Move the back section to desired position.
- C) Release handle to lock back section in position.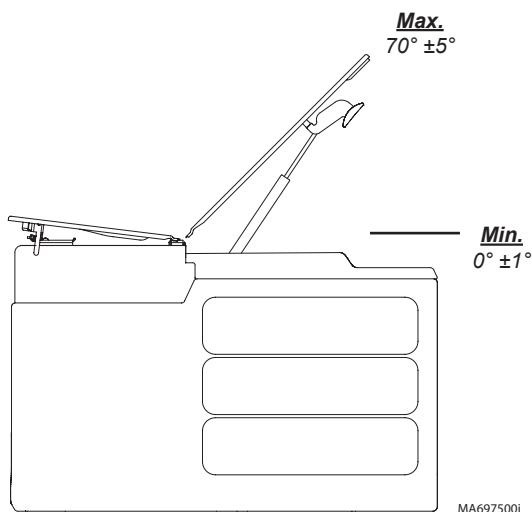

Stirrups


Caution

*Be sure the stirrups are locked in place before using.
The stirrups will not support the patient's entire weight.*

To position the stirrups...

- A) Pull the stirrup out, then unfold.
- B) Lift the stirrup slightly, then move it left or right as desired.
- C) Release stirrup to lock in position.

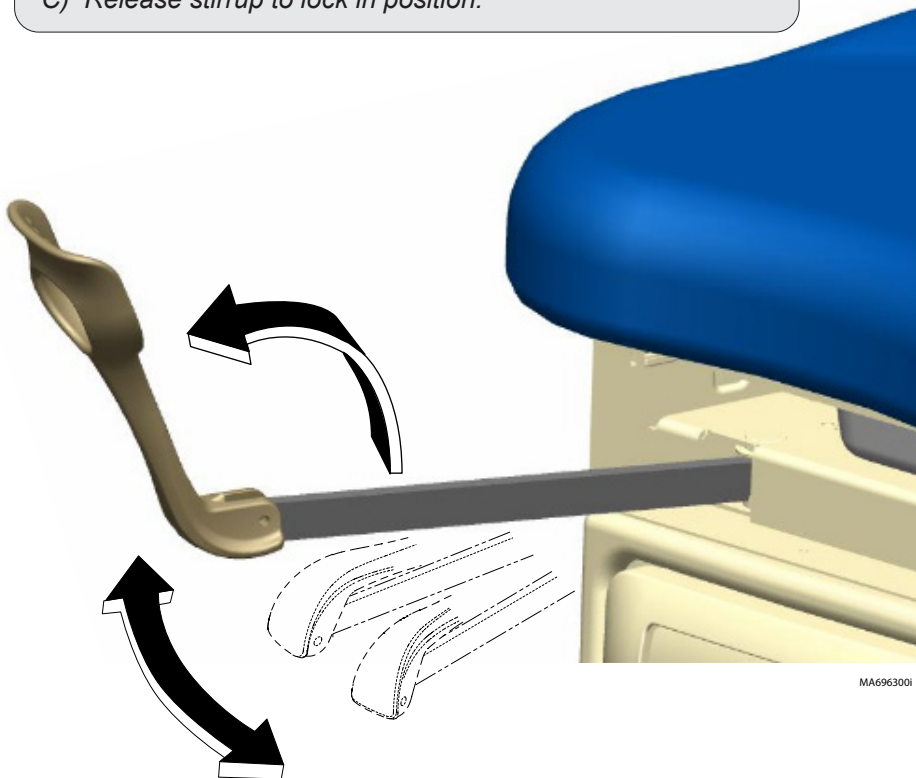Footrest and Treatment Pan

Caution

Do not use footrest to support patient's entire weight.

To access the treatment pan...

Lift footrest pad slightly, then slide pad back into table.

To extend footrest...

Pull footrest out to desired position.

Footstep


Caution

Ensure patients place their feet fully on the flat surface of the footstep when getting aboard and exiting the table.

To access the footstep...

Lift footstep slightly, then pull outward.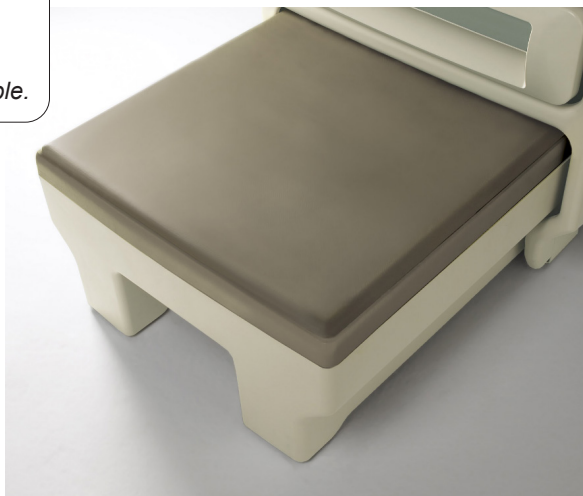

Paper Roll and Tear Strap

Note	
Paper Roll Size (max.):	
Seamless Upholstery.....	21 in. wide x 3.5 in. diameter (53 cm x 9 cm)
Soft Touch Upholstery	21 in. wide x 4.75 in. diameter (53 cm x 12 cm)

MA9895p

MA9896p


Drawer Heater (optional)

Note

The drawer heater switch illuminates when heater is ON (I).

The drawer heater warms the contents of the drawer to approximately 98°F (37°C).

To turn the drawer heater ON/OFF...
Press the heater switch to desired position.

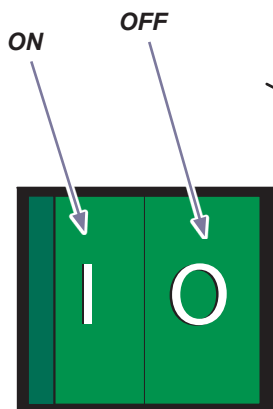MA696500i

Pelvic Tilt (optional)

WARNING



To avoid risk of injury do not put extremities under the seat.

To elevate seat section...

Lift seat until pelvic support drops into place.

To return seat section to flat position...

- A) *Lift seat section slightly.*
- B) *Pivot trigger handle upward.*
- C) *Lower seat section.*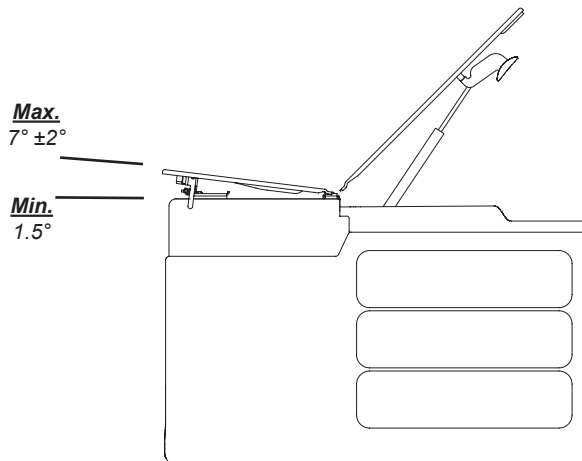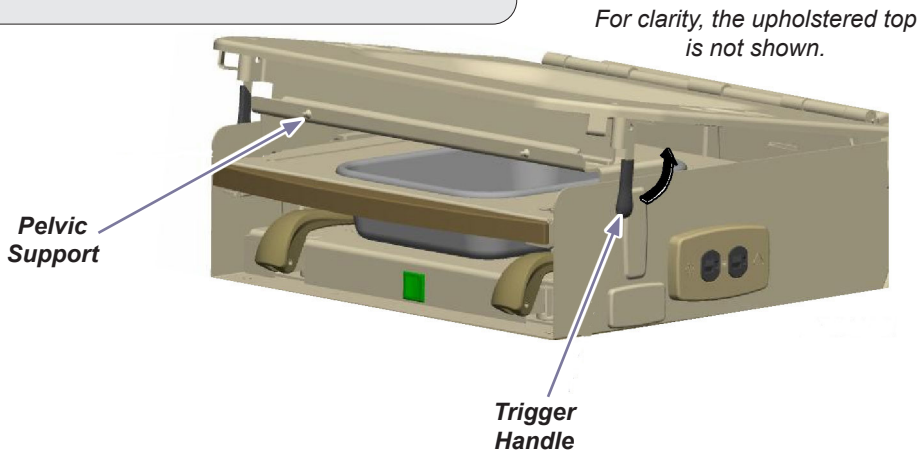

MA6975001



Receptacle (optional)

The duplex receptacle on the side of the table, provides power for accessories used during medical procedures. There are two circuit breakers located under the seat section. If the receptacle's maximum load is exceeded, the circuit breakers interrupt power to the receptacle.

Maximum Load 5 amps

To reset circuit breaker(s)...

Push the circuit breaker toggle switch.

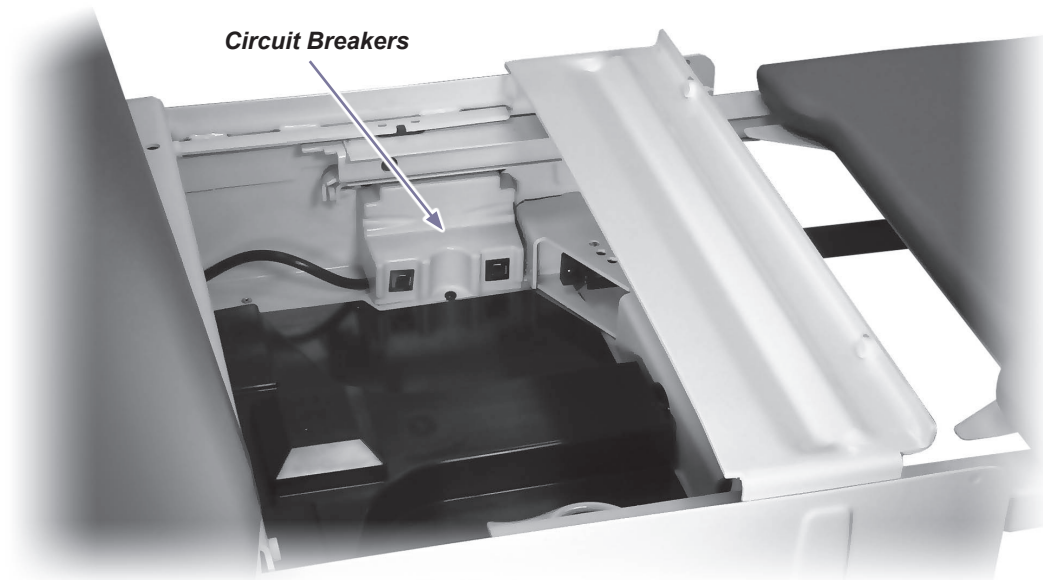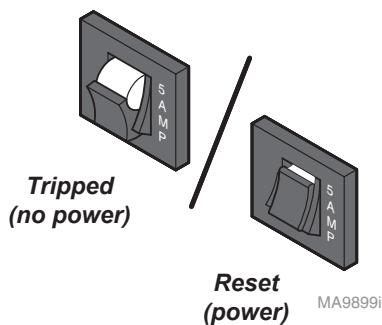MA9894p

Maintenance

Cleaning

Upholstery

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [midmark.com](https://www.midmark.com) in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

Calling For Service

If service is required, contact your authorized Midmark dealer.

Note

Model / Serial Number information is required when calling for service.


If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585
Monday-Thursday 8:00 AM until 6:00 PM
Friday 8:00 AM until 5:00 PM (ET)
[midmark.com](https://www.midmark.com)

Specifications

Weights & Dimensions




Maximum Patient Weight:	500 lbs (226 kg)
Paper Roll <i>(maximum size)</i> :	<i>[See Paper Roll & Tear Strap page]</i>
Range of Motion & Dimensions:	<i>(See illustration below)</i>
Weight of Table:	220 lbs (99.8 kg)
Power Cord:	6 ft. (1.8 m) long

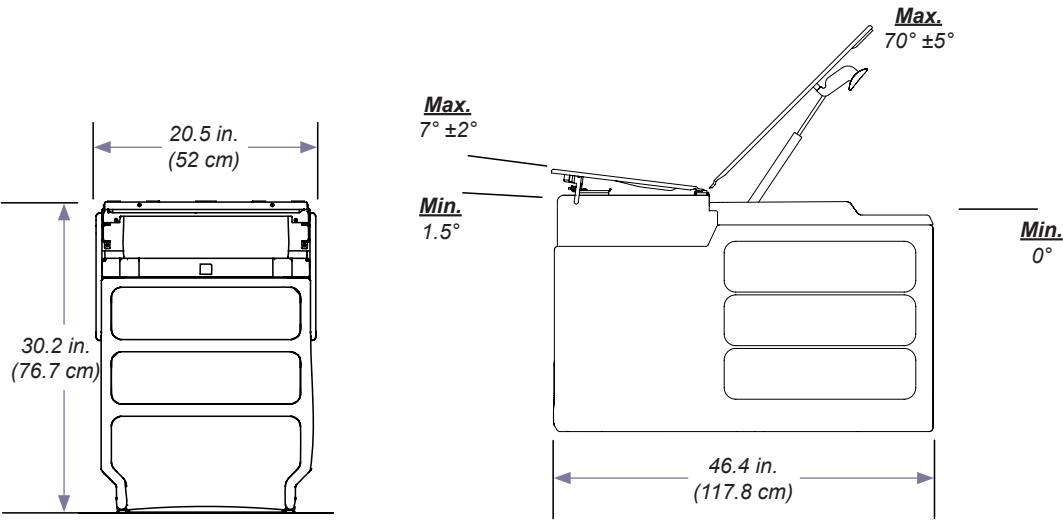MA8376i

Specifications (continued)

Upholstery Dimensions

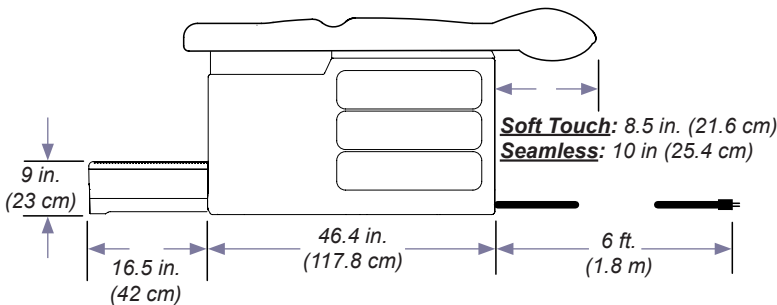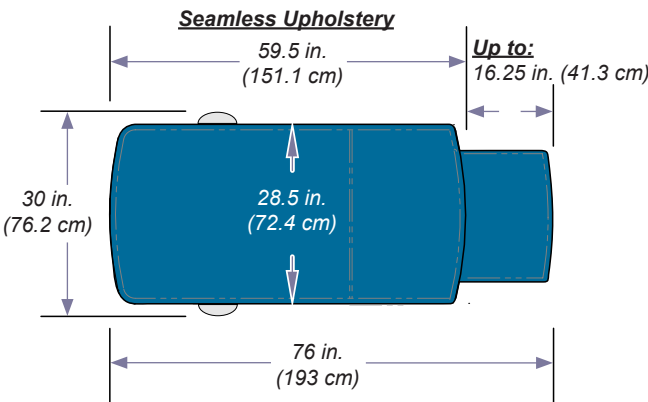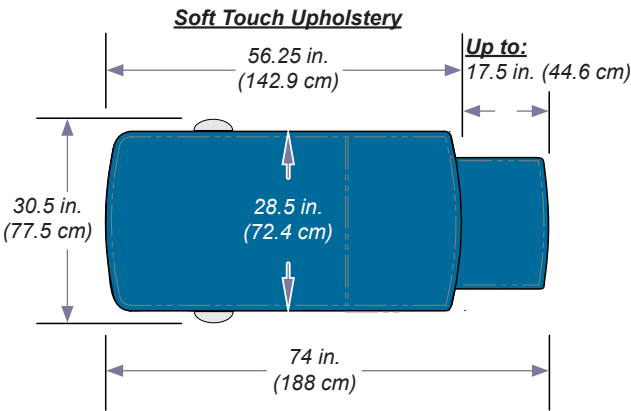MA83771

Specifications *(continued)*

Voltage, Current, Frequency Ratings & Compliance Chart

Model	Description					Electrical Ratings:		
		ES 60601-1	CAN / CSA C22.2, 60601-1	CE	VAC +/- 10%	Amps	Cycles (Hz)	
204-001 204-011	Ritter® Exam Table (Pass-thru Drawers) w/ Receptacle	•	•		115	5	50/60	
204-002 204-012	Ritter® Exam Table (Pass-thru Drawers) w/ Receptacle, Pelvic Tilt, & Drawer Heater	•	•		115	5.5	50/60	
204-003	Ritter® Exam Table (Bulk Storage) w/ Receptacle	•	•		115	5	50/60	
204-004	Ritter® Exam Table (Bulk Storage) w/ Receptacle, Pelvic Tilt, & Drawer Heater	•	•		115	5.5	50/60	

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Midmark Corporation