Security Straps 9A559

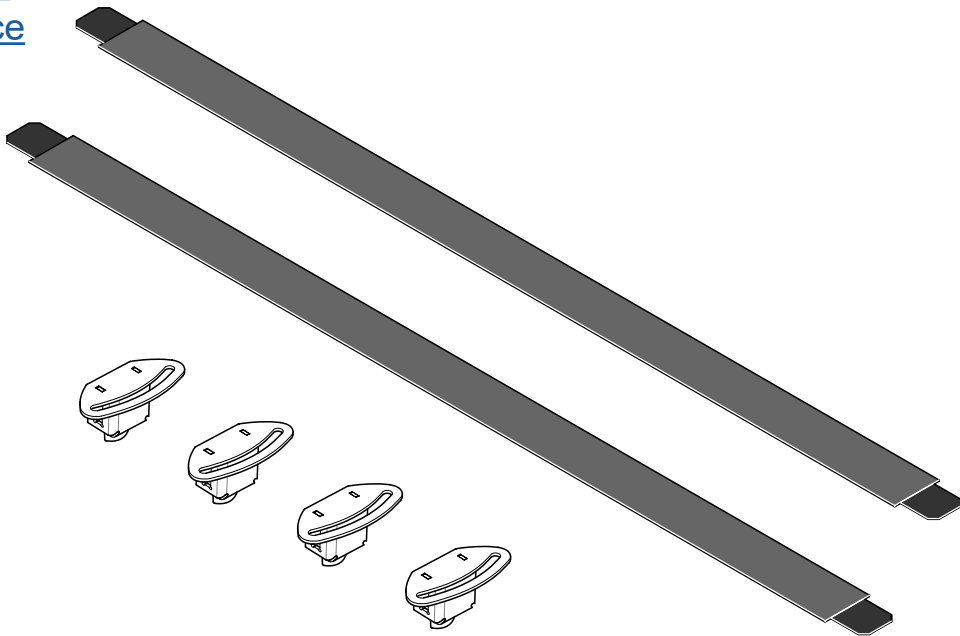
[English](#)
[Deutsch](#)
[Français](#)
[Español](#)
[Dansk](#)
[Suomi](#)
[Italiano](#)
[Svenska](#)
[Norsk](#)
[Nederlands](#)
[Polski](#)
[Türkçe](#)

Applies to Models:

630 (-010 thru -013)
630 (-020 thru -023)
631-001

Special Tools:

none

Set Up / User Guide

Style U

003-10368-99 Rev. AA6 (2/27/25)
TP202 20-42-FO-00014 Rev A2 C2169

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

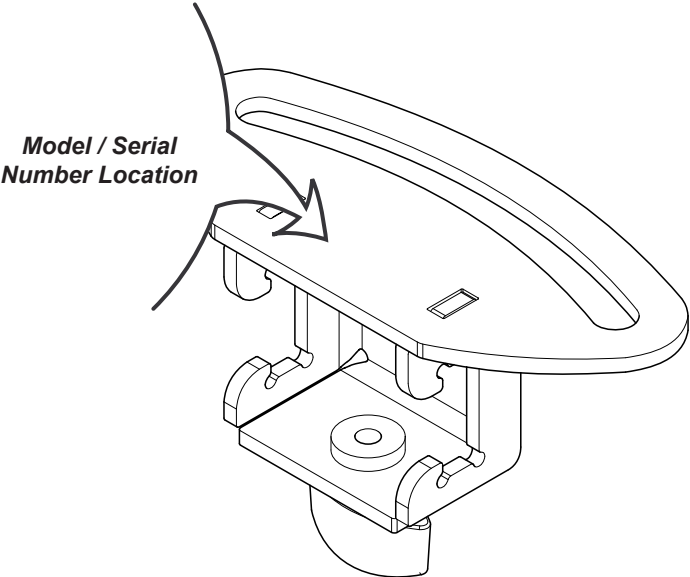
Table of Contents

Important Information

Product Information 2

Product Registration 4

Operating Conditions 4

Disposal of Equipment 4

Authorized Representatives 4

Important Information 6

Intended Use 7

Requirements 7

Safety Instructions 7

Set Up

Set Up Instructions 8

Operation

Operating Instructions 9

Maintenance

Calling for Service 10

Cleaning 10

Periodic Maintenance 10

Warranty Information

Limited Warranty 11

Product Registration

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

WARNING
In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in the EU should direct all questions, incidents, and complaints to Midmark’s Authorized Representative listed below.

CEpartner4U
Esdoornlaan 13
3951 DB Maarn, The Netherlands
Phone: +31 343 442 524
Fax: +31 343 442 162

Customers in the UK should direct all questions, incidents, and complaints to Midmark’s UK Responsible Person listed below.

UKCApartner4U Ltd.,
7 CAMPION WAY, BINGHAM, NOTTINGHAM NG13 8TR, UK.
www.UKCApartner4U.co.uk
Phone : +44 7905 384429

Customers in Mexico should direct all questions, incidents, and complaints to Midmark’s Sponsor listed below.

Carlos Nafarrate SA de CV
Avenida Las Américas 601,
Ladrón de Guevara, 44600,
Guadalajara, Jalisco, México
<https://www.nafarrate.com/> [nafarrate.com]
Phone: (33) 3669 9800

Authorized Representatives - continued

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India

Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)

Mumbai, 400022

Tel:+91 22 4915 3000

Fax:+91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd

Abdullah M. Al Khalifi Street

Al Nakheel Dist. II

P.O. Box 116105

Jeddah (East)

21391

Saudi Arabia

Tel: + 966 2 286 4707

Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.

Al Barsha,

Dubai, UAE

Phone: +971 4 501 4000

Fax: +971 4 501 4100

Customers in Kuwait should direct all questions, incidents, and complaints to the Midmark Authorized Representative:

Central Circle Co.

P.O. Box 1015 Salmieh

22011 Kuwait

Al-Sharq

Khalid Bin Al-Waleed Street

Al-Kazmi Building 10

Third Floor

Email : cencico@centralcircleco.com

Phone : +965 22411758, +965 22411748, +965 22444228

Fax :+965 22445457

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Emergo Costa Rica S.R.L.

Edificio Ribleh 2 Piso, Diagonal a la Biblioteca Municipal

Heredia, Belén

La Ribera, Costa Rica

E-mail: LAvigilance@ul.com

Phone: +(506) 2201-14 53

Customers in Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tecnidental Panama

VILLA MAGNA, VILLA ZAITA

OFFICE B8 PANAMA

Panama

E-mail: tecnidentalpanama@gmail.com

<https://tecnidentalpa.com/> [tecnidentalpa.com]

Phone: +507 6416-2235, +507 6492-0884

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading

Midmac Roundabout

Opposite Al Mana Petrol Station

Doha, Qatar

Tel: +974-44337000

Fax: +974-44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.

HaTnufa St 7

Petah Tikva, Israel

Tel: +972 3-925-4040

Fax: +972 3-924-9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

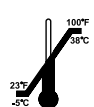
Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice, or condition.

Symbol Glossary

Pressure Limit

Temperature Limit

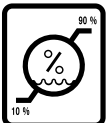
Proper shipping orientation

Keep dry

Fragile

Humidity Limit

Manufacturer

Serial Number

Medical Device

Catalogue Number

Intended Use

To help improve safety by securing the patient to the chair before, during and after procedures, especially when anesthesia is used.

Requirements

Requires Accessory Rail 9A557 or 9A558 and Seat Rail Accessory 9A521 or 9A522.

Safety Instructions

Warning

No modification of this equipment is allowed.

Warning

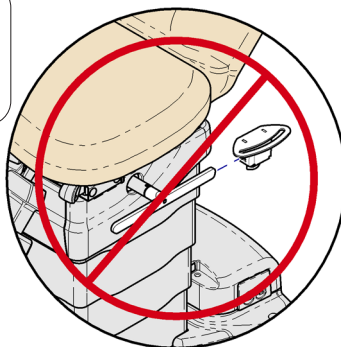
Do not press any button during a procedure.

Set Up


WARNING

*Do not use Security Strap with accessory rail mounted to chair base.
Doing so could result in serious injury.*


To install back section tether bracket...

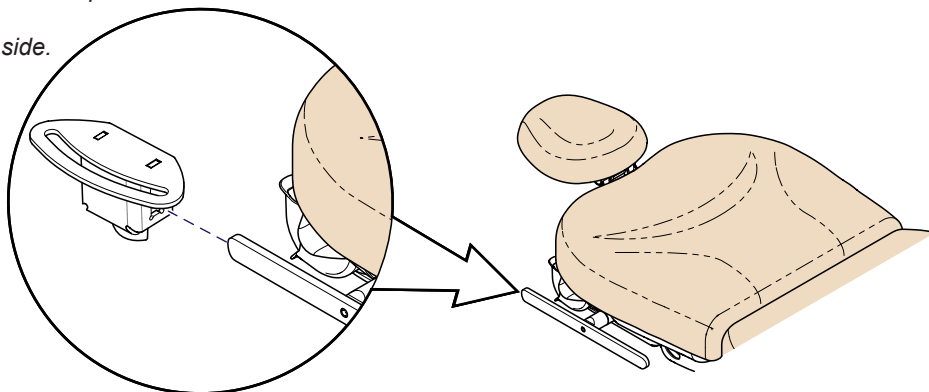- A) Slide tether bracket onto rail as shown.
- B) Adjust bracket to desired position.
- C) Tighten knob.
- D) Repeat for other side.

To install seat section tether bracket...

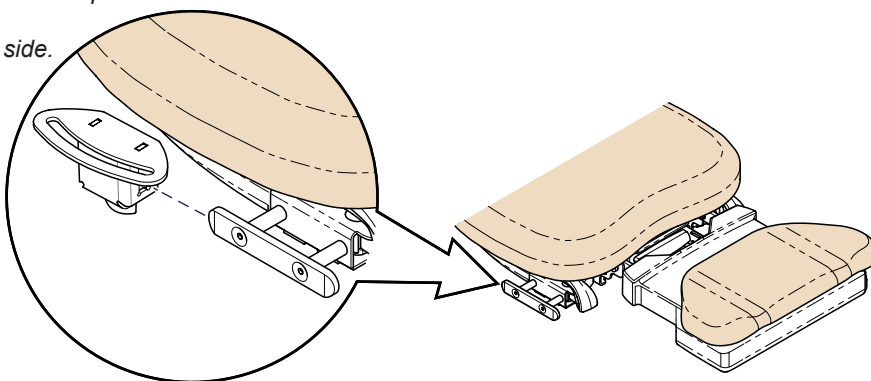- A) Slide tether bracket onto rail as shown.
- B) Adjust bracket to desired position.
- C) Tighten knob.
- D) Repeat for other side.

Operation

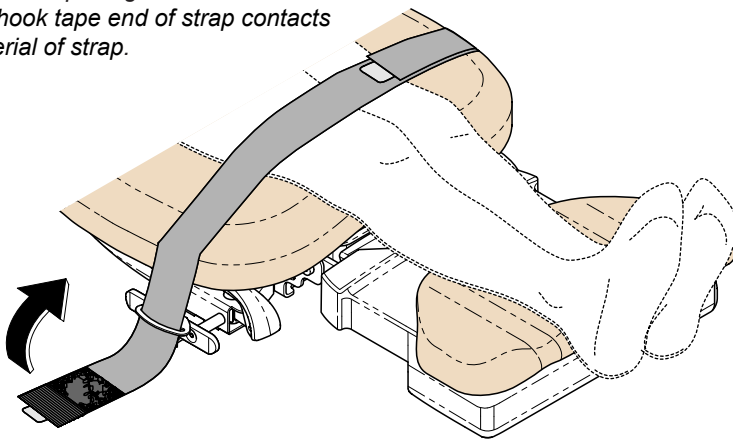
Warning

Do not press any button during a procedure.


Lower security strap:

- A) Place hook tape end of strap through tether bracket opening.
- B) Loop so hook tape end of strap contacts loop material of strap.

Upper security strap:

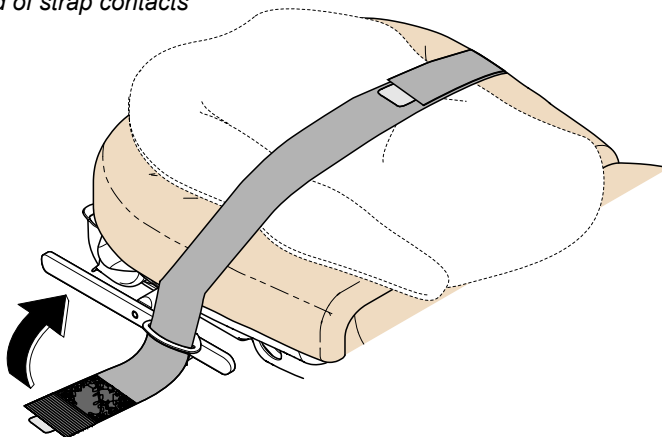- A) Place hook tape end of strap through tether bracket opening.
- B) Loop so hook tape end of strap contacts loop material of strap.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact your authorized Midmark dealer.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The Security Belts are resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Security Belts.

Security Belts

Wash weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

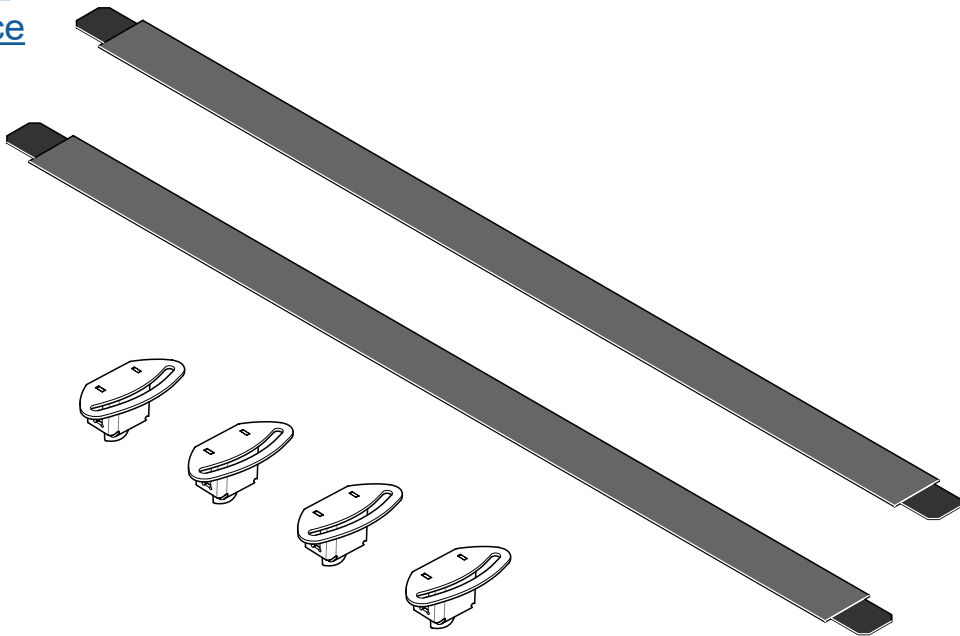
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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© Midmark Corporation 2020
Midmark Corporation