
Security Straps

9A406

[English](#)
[Español](#)
[Français](#)

Applies to Models:
641

Special Tools:
short screwdriver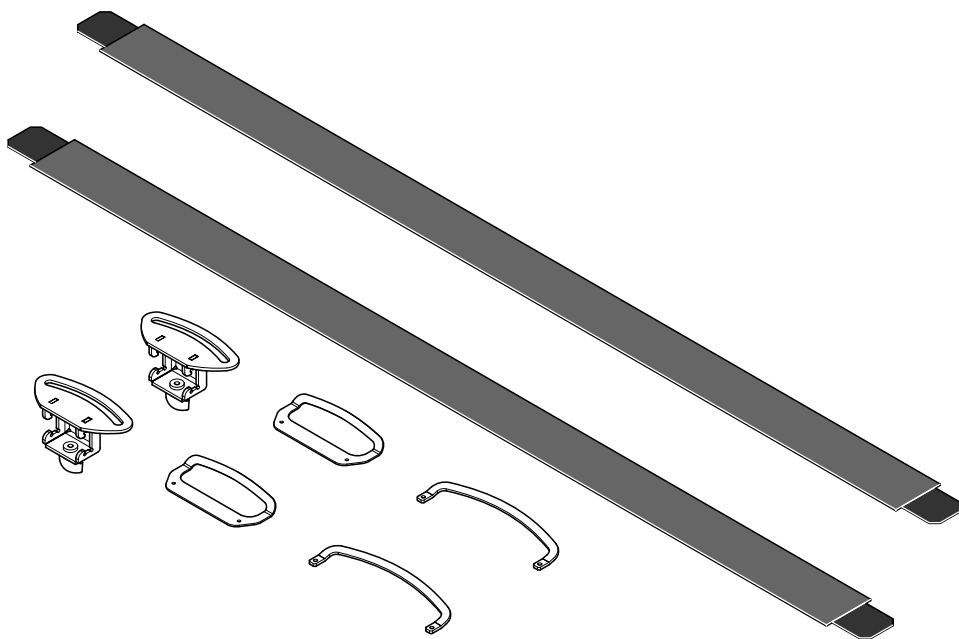

Set Up / User Guide

Style P

003-10395-99 Rev. AA4 (10/5/23)

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

**Model / Serial
Number Location**

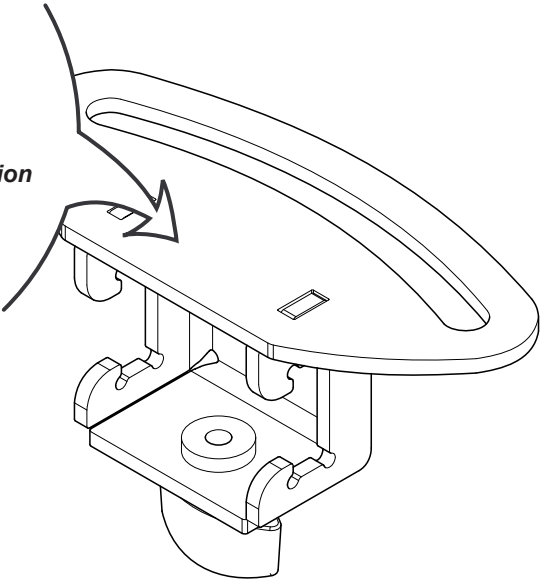
Table of Contents

Important Information

[Product Information](#) 2

[Product Registration](#) 4

[Operating Conditions](#) 4

[Disposal of Equipment](#) 4

[Authorized Representatives](#) 4

[Important Information](#) 6

[Intended Use](#) 7

[Safety Instructions](#) 7

Set Up

[Set Up Instructions](#) 8

Operation

[Operating Instructions](#) 9

Maintenance

[Calling for Service](#) 10

[Cleaning](#) 10

[Periodic Maintenance](#) 10

Warranty Information

[Limited Warranty](#) 11

Product Registration

SCAN ME

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)

Relative Humidity.....30% to 75% (non-condensing)

Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

Warning

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Australia should direct all questions, incidents, and complaints to Midmark's Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel: +91 22 4915 3000
Fax: +91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Authorized Representatives - continued

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

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Associated Medical Supplies Company LTD.
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice, or condition.

Symbol Glossary

Pressure Limit

Proper shipping orientation

Fragile

Manufacturer

Temperature Limit

Keep dry

Humidity Limit

Serial Number

Catalogue Number

Intended Use

To help improve safety by securing the patient to the chair before, during and after procedures, especially when anesthesia is used.

Safety Instructions

Warning

No modification of this equipment is allowed.

Warning

Do not press any button during a procedure.

Set Up

Components:

Tether Brackets:
Back Section..... 2
Seat Section 2
Leg section 2


Security Straps 3
Screws
(#10-24 x 1/2")..... 8

Special Tools:

short screwdriver

To install back section tether bracket...

- A) Slide bracket onto rail as shown.
- B) Adjust bracket to desired position.
- C) Tighten knob.
- D) Repeat for other side.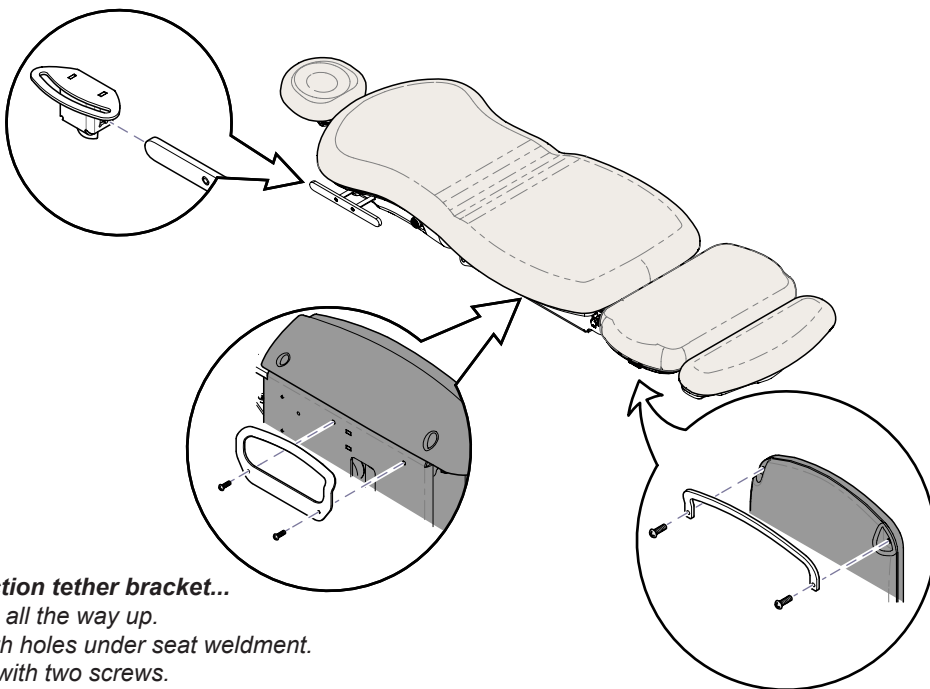

To install seat section tether bracket...

- A) Run Tilt function all the way up.
- B) Align bracket with holes under seat weldment.
- C) Secure bracket with two screws.
- D) Repeat for other side.

To install leg section tether bracket...


- A) Remove two screws from underside of leg section.
- B) Align bracket, then secure with two screws (included w/kit).
- C) Repeat for other side.

Operation

Warning

Do not press any button during a procedure.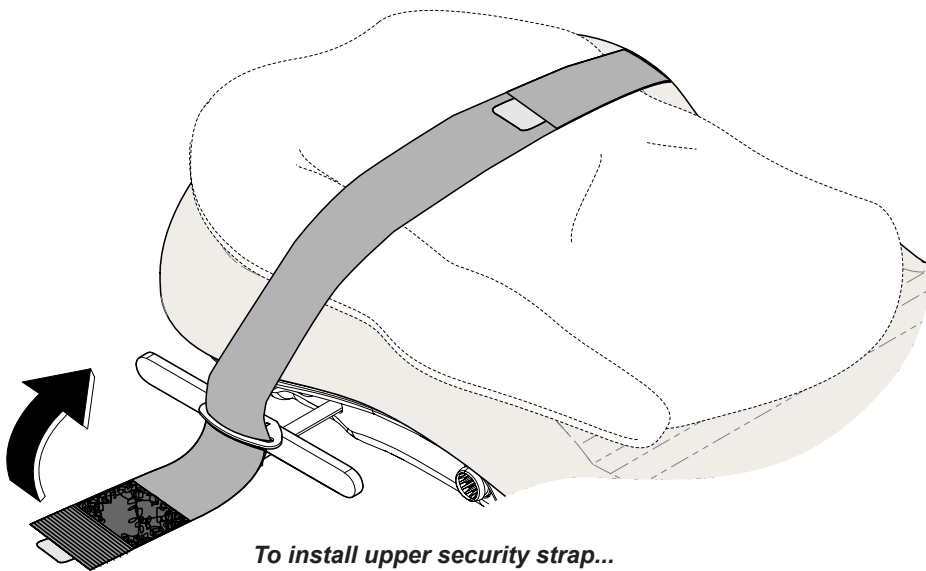

To install upper security strap...

- A) Place hook tape end of strap through tether bracket opening.*
- B) Loop so hook tape end of strap contacts loop material of strap.*
- C) Repeat for Seat and Leg sections.*

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The Security Belts are resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Security Belts.

Security Belts

Wash weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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Midmark Corporation