Swing Arm Instrument Tray / PC Work Shelf 9A427 / 9A428


[English](#)
[Español](#)
[Français](#)

Applies to Models:

230 / 641 / 646 / 647

Special Tools:

none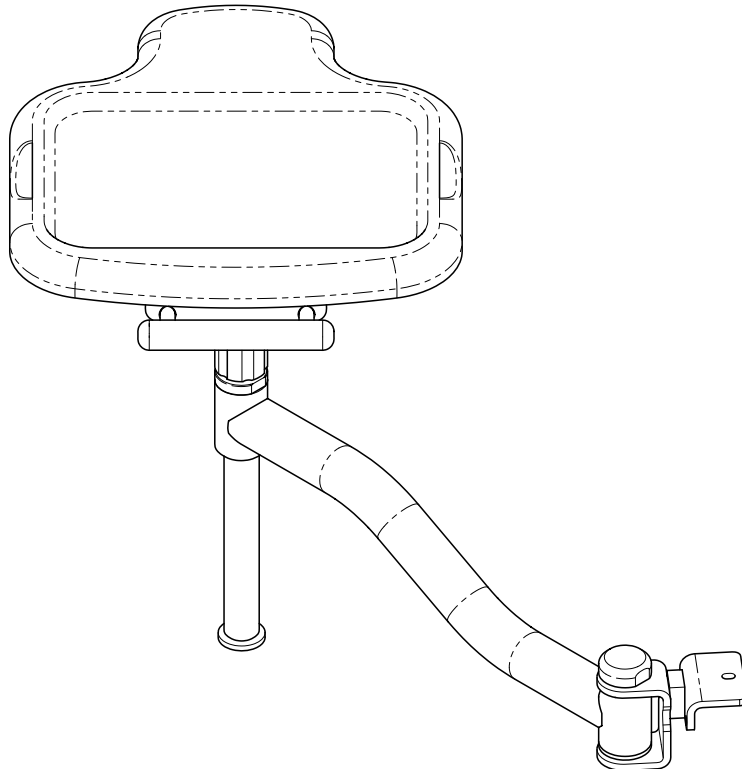

Set Up / User Guide

003-10404-99 Rev. AA4 (10/5/23)

Style P



Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

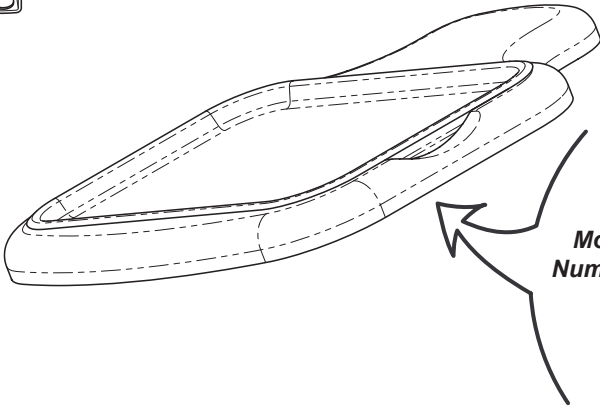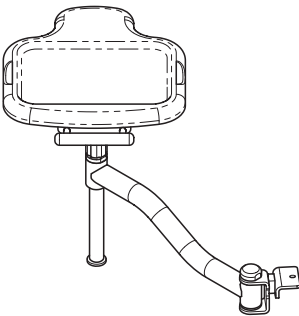**Model / Serial
Number Location**

Table of Contents

Important Information

Product Information 2

Product Registration 4

Transportation / Storage Conditions 4

Operating Conditions 4

Disposal of Equipment 4

Authorized Representatives 4

Important Information 6

Intended Use 6

Restrictions 6

Safety Instructions 6

Set Up

Set Up Instructions 7

Operation

Adjust tray / shelf height 8

Adjust sliding tray position (9A427002) 9

Maintenance

Calling for Service 10

Cleaning 10

Periodic Maintenance 10

Warranty Information

Limited Warranty 11

Product Registration

Transportation / Storage Conditions

Ambient Temperature Range:.....5°C to +38°C (-23°F to +100°F)
Relative Humidity.....10% to 90% (non-condensing)
Atmospheric Pressure500hPa to 1060hPa (0.49atm to 1.05atm)

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

WARNING
*In the event of any serious incident in relation to the device,
please contact Midmark and the appropriate competent authority.*

Customers in Australia should direct all questions, incidents, and complaints to Midmark’s Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Customers in India should direct all questions, incidents, and complaints to Midmark’s Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel:+91 22 4915 3000
Fax:+91 22 4915 3100

Authorized Representatives - continued

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

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Associated Medical Supplies Company LTD.
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.



Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice or condition.

Symbol Glossary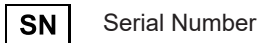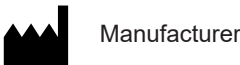

Intended Use

To provide support for small instruments or a laptop PC within easy reach of the healthcare provider.

Restrictions

Cannot be used on the same side as 9A205 Base Rail.

Safety Instructions


WARNING
No modification of this equipment is allowed.

WARNING
Do not use Instrument Tray as a gripping surface during ingress egress.


Set Up

- A) Run Tilt function UP to access top of base column.
- B) Remove three hole plugs from desired side.
- C) Align swing arm with mounting holes.
- D) Secure swing arm with three hex screws.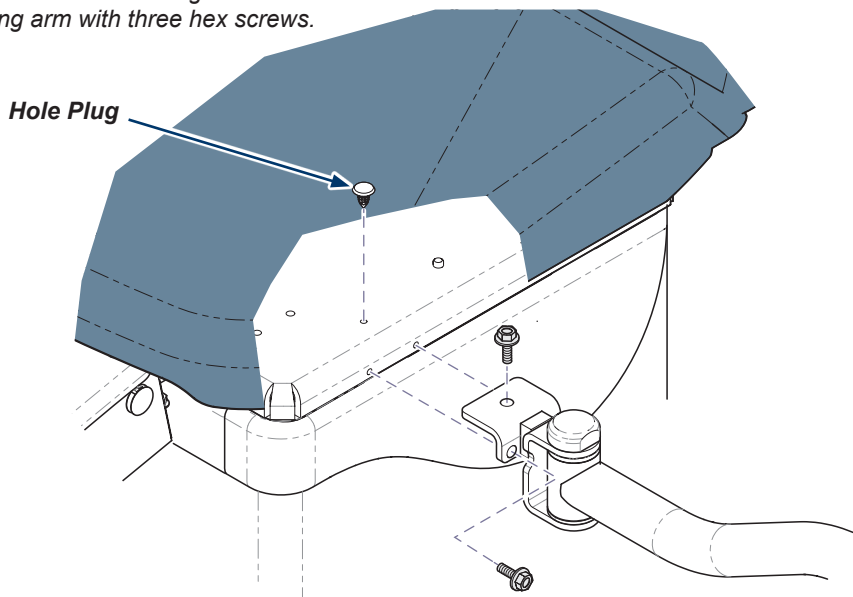


Note

The Fixed Instrument Tray is shown. The installation of the Sliding Instrument Tray, and PC Work Shelf is similar.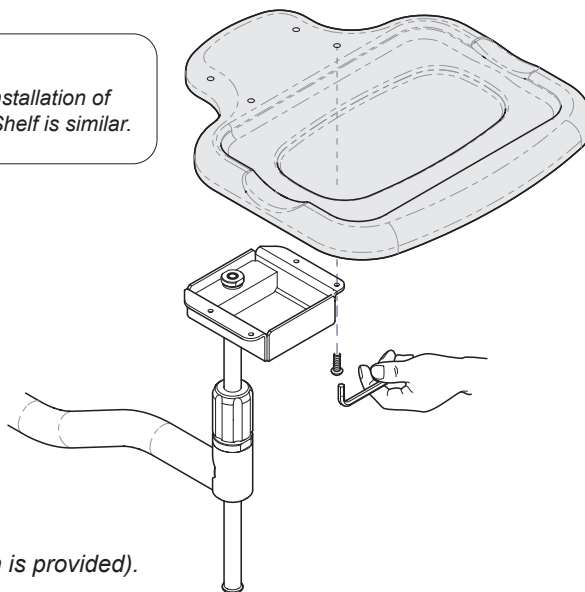


- E) Align tray / shelf with mounting holes.
- F) Secure with four screws (allen wrench is provided).

Operation - To adjust tray / shelf height...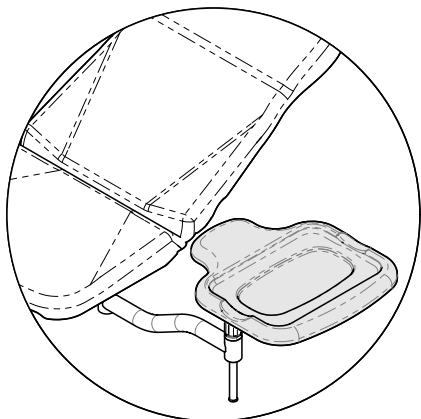

Caution

Assure the tray is clear of the chair while the chair is being positioned to avoid potential trapping hazards. Failure to do so may result in equipment damage or personal injury.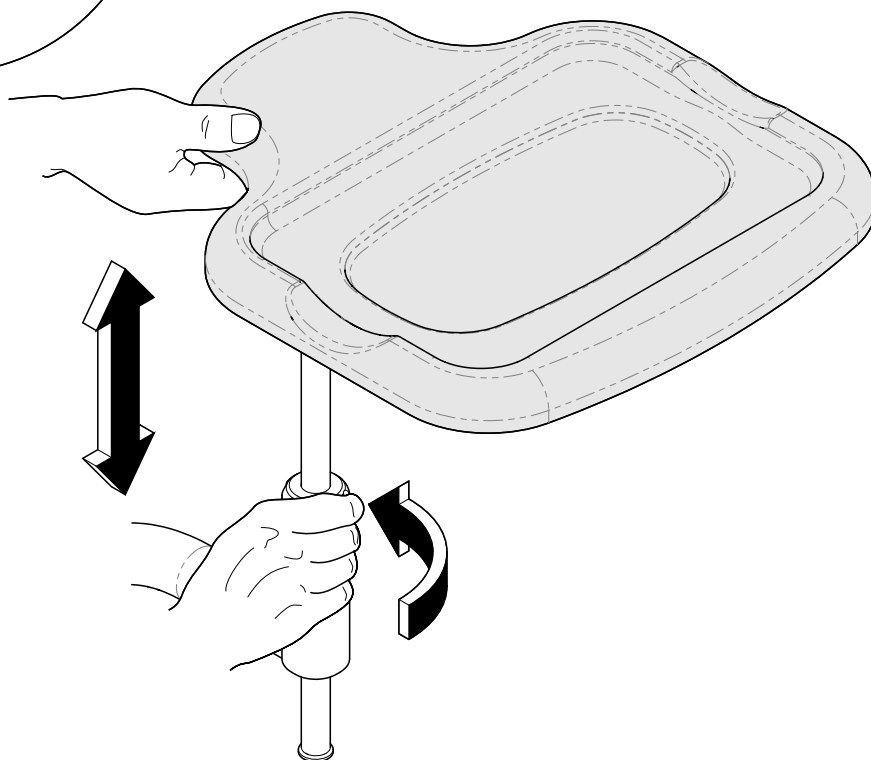

Caution

Be sure to support the tray / shelf as you loosen the adjustment knob. Failure to do so, may result in personal injury.



- A) Loosen adjustment knob.
- B) Raise tray / shelf to desired height.
- C) Tighten adjustment knob.

Operation - To adjust sliding tray position...(9A427002)

Caution

Assure the tray is clear of the chair while the chair is being positioned to avoid potential trapping hazards. Failure to do so may result in equipment damage or personal injury.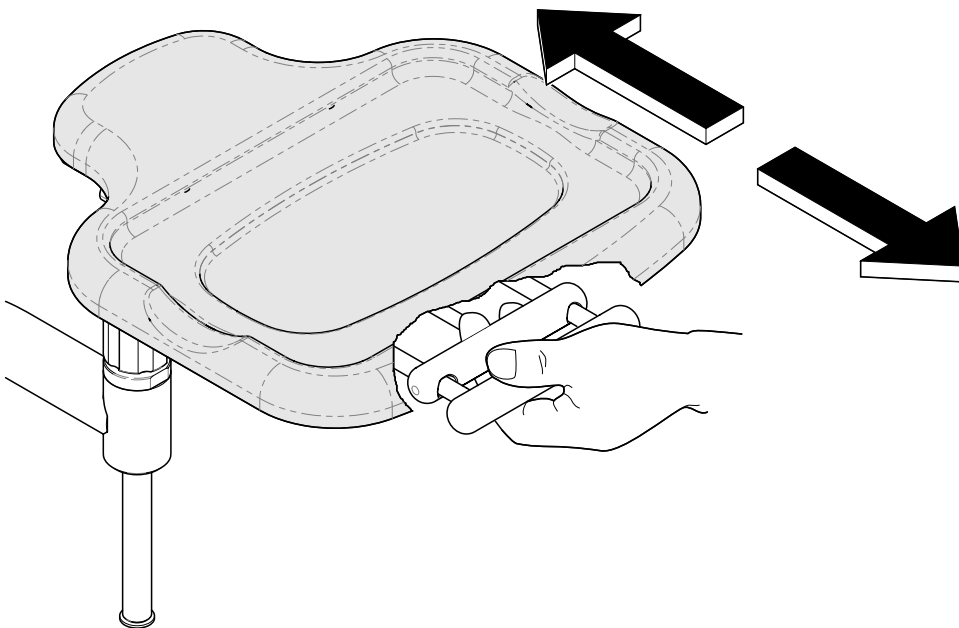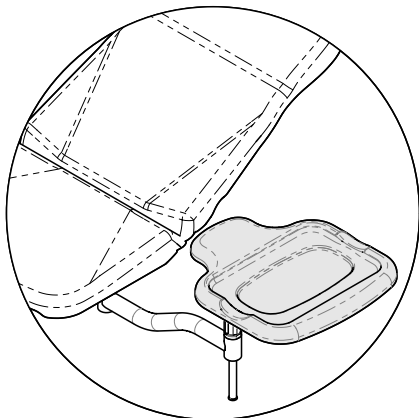

- A. Squeeze tray handles.
- B. Slide tray to desired position.
- C. Release handles to lock in place.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The accessory is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the accessory.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces using a clean soft cloth, and mild cleaner.

Equipment Alert

Do not use bleaching agents or any abrasive materials / substances on stainless steel surfaces (i.e. bleach, steel wool, wire brush, scouring powder, etc.). Failure to comply may result in damage to the accessory.

Stainless Steel

Frequent cleaning will prolong the life and help maintain the natural luster of stainless steel:

- When cleaning, always rub in the direction of the “grain” of the metal.
- Avoid use of steel wool or steel brushes when cleaning stainless steel. These steel particles may become embedded into the stainless steel and cause rust.
- Avoid the use of chlorine bleach. Chemicals in bleach will cause pitting, corrosion, and discoloration of the metal.
- Despite the name, stainless steel actually can stain and mark. However, stains and marks do not affect the integrity of the metal.

Periodic Maintenance

Periodically inspect the following areas:

- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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Midmark Corporation