## Adjustable Arm System

9A328001 / 9A328002 / 9A350003

[English](#)  
[Español](#)  
[Français](#)

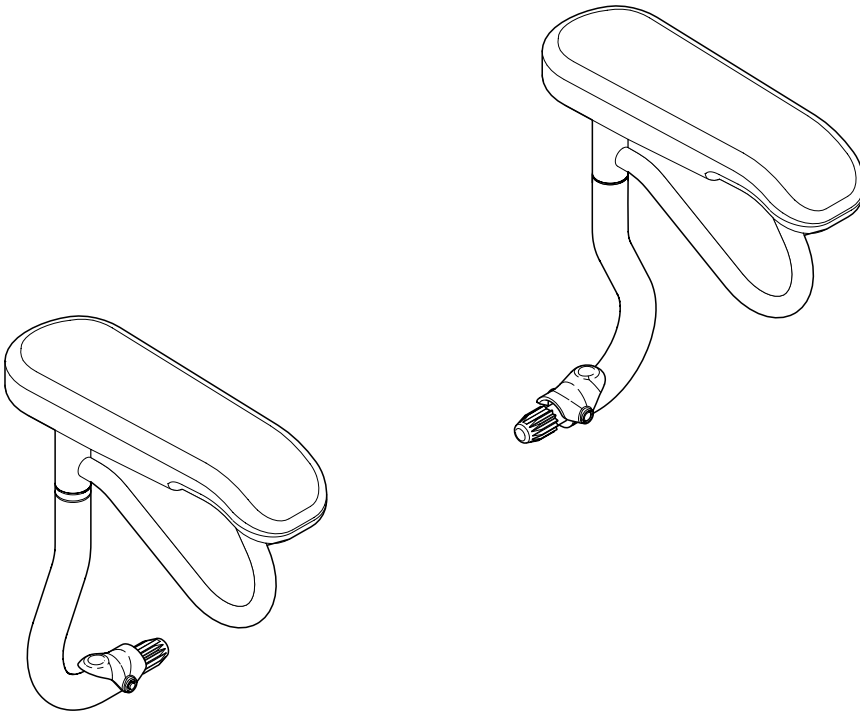
### Applies to Models:

230

630 (-001 thru -009)

### Special Tools:

none

# Set Up / User Guide

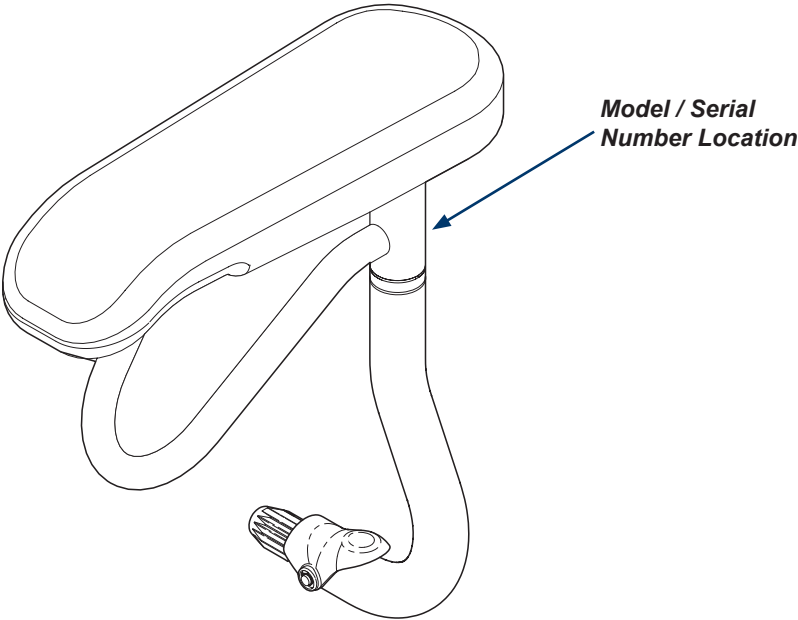
# Product Information

**Dealer:**

**Date of Purchase:**

**Model / Serial Number:**

**Midmark Authorized Service Company:**

# Table of Contents

## Important Information

- Product Information ..... 2
- Product Registration ..... 4
- Operating Conditions ..... 4
- Disposal of Equipment..... 4
- Authorized Representatives ..... 4
- Important Information ..... 6
- Intended Use ..... 7
- Restrictions..... 7
- Safety Instructions ..... 7

## Set Up

- Set Up Instructions ..... 8

## Operation

- Operating Instructions ..... 9

## Maintenance

- Calling for Service ..... 10
- Cleaning ..... 10
- Periodic Maintenance ..... 10

## Warranty Information

- Limited Warranty ..... 11

## Product Registration

SCAN ME

## Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)

Relative Humidity.....30% to 75% (non-condensing)

Altitude.....3000m or less

## Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

## Authorized Representatives

### **WARNING**

*In the event of any serious incident in relation to the device,  
please contact Midmark and the appropriate competent authority.*

Customers in Australia should direct all questions, incidents, and complaints to the Midmark Sponsor listed below.

ICONA Pty Ltd.  
1A/2A Westall Road  
Clayton, Victoria 3168  
Phone: 1300 442 662

Customers in India should direct all questions, incidents, and complaints to the Midmark Authorized Representative listed below.

Midmark India  
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)  
Mumbai, 400022  
Tel:+91 22 4915 3000  
Fax:+91 22 4915 3100

## ***Authorized Representatives - continued***

Customers in Saudi Arabia should direct all questions, incidents, and complaints to the Midmark Authorized Representative listed below.

Attieh Medico Ltd  
Abdullah M. Al Khalifi Street  
Al Nakheel Dist. II  
P.O. Box 116105  
Jeddah (East)  
21391  
Saudi Arabia  
Tel: + 966 2 286 4707  
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to the Midmark Authorized Representative listed below.

GULF DRUG LLC.  
Al Barsha,  
Dubai, UAE  
Phone: +971 4 501 4000  
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to the Midmark Authorized Representative listed below.

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Associated Medical Supplies Company LTD.  
Room 1201-1202, 12/F, Fo Tan Industrial Center  
26-28 Au Pui Wan Street, Fo Tan, Shatin  
New Territories, Hong Kong  
Phone: +852 2604 9389  
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to the Midmark Authorized Representative listed below.

Tadmur Trading  
Midmac Roundabout  
Opposite Al Mana Petrol Station  
Doha, Qatar  
Tel: +974.44337000  
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to the Midmark Authorized Representative listed below.

Medtechnica Ltd.  
HaTnufa St 7  
Petah Tikva, Israel  
Tel: +972 3.925.4040  
Fax: +972 3.924.9977

# Important Information - Safety Symbols

**WARNING**  
*Indicates a potentially hazardous situation which could result in serious injury.*

**Caution**  
*Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices*

**Equipment Alert**  
*Indicates a potentially hazardous situation which could result in equipment damage.*

**Note**  
*Amplifies a procedure, practice or condition.*

## Symbol Glossary

These symbols may appear on packaging, equipment, and/or in the manuals.

Pressure Limit

Humidity Limit

Temperature Limit

Temperature Limit

Proper shipping orientation

Keep dry

Fragile

Serial Number

Manufacturer

Catalogue Number

## ***Intended Use***

To provide patient support when mounting / dismounting chair.

## ***Restrictions***

Cannot use arm on same side as a base rail mounted 253 light.

## ***Safety Instructions***

### ***WARNING***

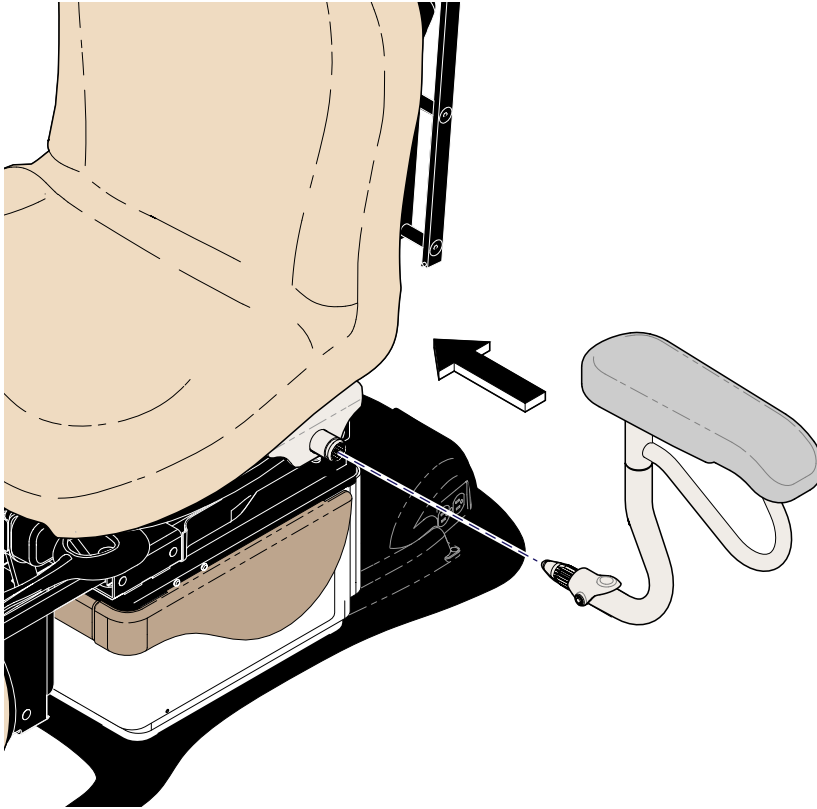
*No modification of this equipment is allowed.*

### ***WARNING***

*Do not sit on Chair Arm or Chair Arm pad.*

## Set Up


### **Note**

*If socket has protective cap installed, remove it.*


*A) Insert Chair Arm into socket until it locks into place.*

# Operation

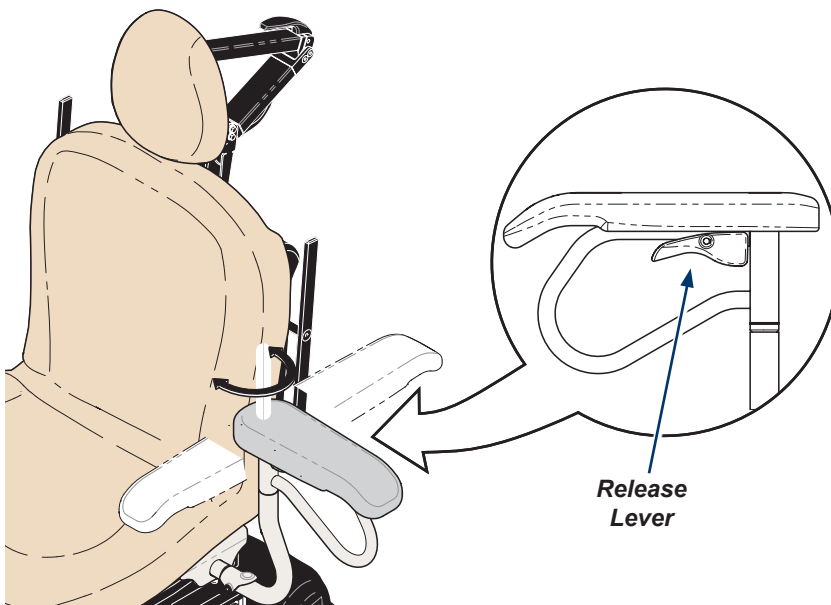- A) To position Chair Arm, squeeze release lever.
- B) Rotate Chair Arm to desired position.

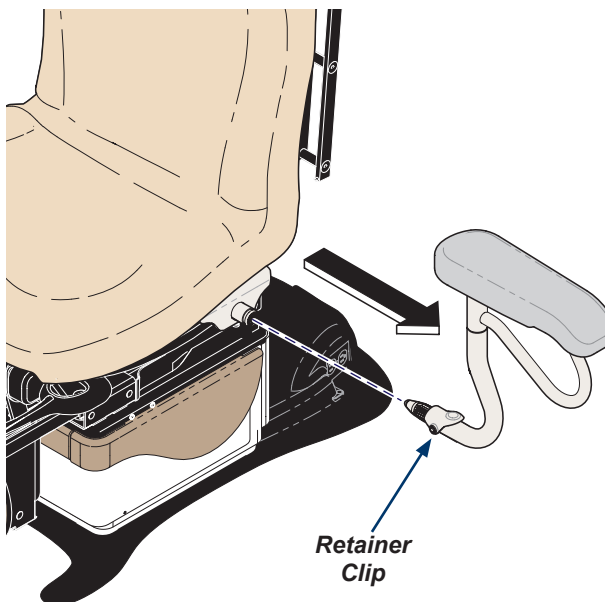To remove Chair Arm, squeeze retainer clip and pull outward.

# Maintenance

## Calling for Service

### Note

*Model / serial number information is required when calling for service.*

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM Friday 8:00 AM until 5:00 PM (ET)

[midmark.com](http://midmark.com)

## Cleaning

### Equipment Alert

*The Chair Arm is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Chair Arm.*

### Chair Arm Pad

Wash your Chair Arm weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner buildup.

Disinfect your Chair Arm with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the Chair Arm surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [midmark.com](http://midmark.com) in the Technical library under the User Information tab for your product.

### Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

## Periodic Maintenance

Periodically inspect the following areas:

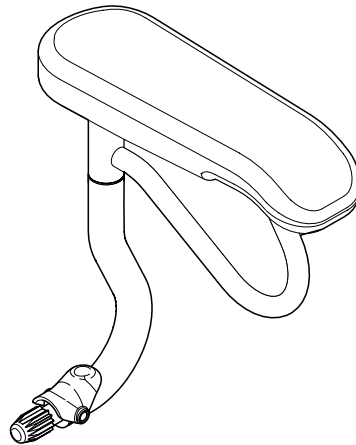
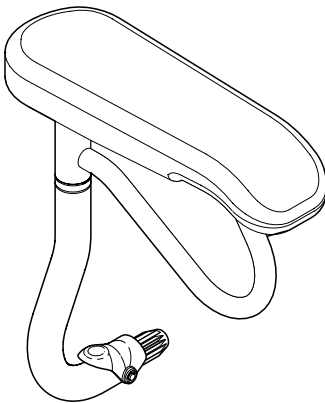
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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© Midmark Corporation 2020
## **Midmark Corporation**