Patient Support Rails 9A561

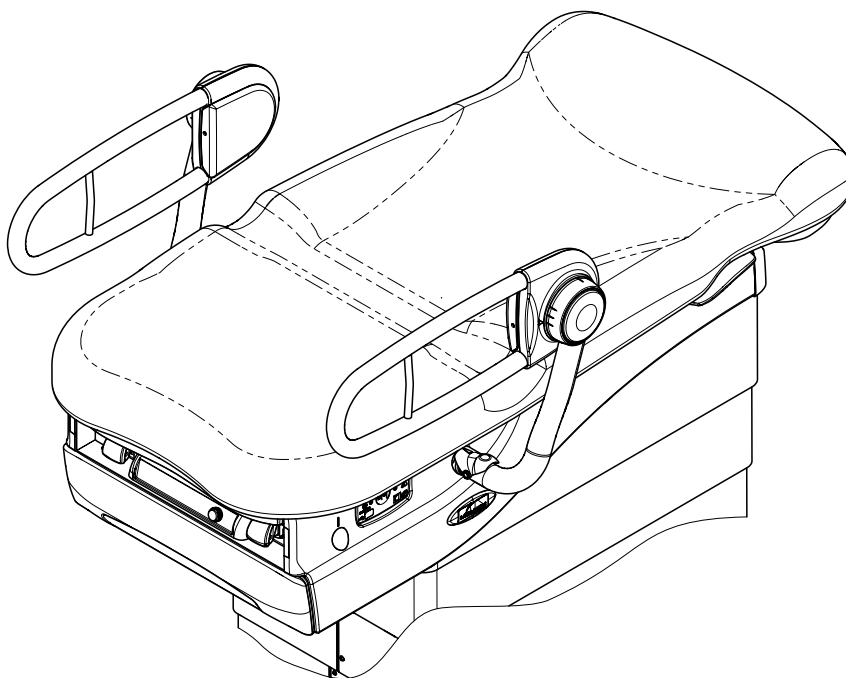
[English](#)
[Español](#)
[Français](#)

Applies to Models:

222 / 223 / 622 / 623 / 625

Special Tools:

none

Set Up / User Guide

Style P

003-10406-99 Rev. AA3 (10/5/23)

TP202 20-42-FO-00014 Rev A1 C2169

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

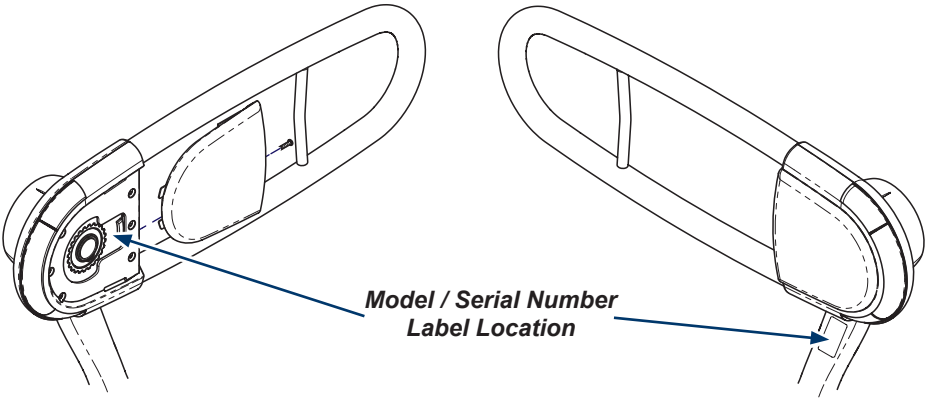
Table of Contents

Important Information

- [Product Information](#) 2
- [Product Registration](#) 4
- [Operating Conditions](#) 4
- [Disposal of Equipment](#) 4
- [Authorized Representatives](#) 4
- [Important Information](#) 6
- [Intended Use](#) 7
- [Safety Instructions](#) 7

Set Up

- [Set Up Instructions](#) 8

Operation

- [Lowest Position](#) 11
- [15° Patient Position](#) 11
- [30° Patient Position](#) 12
- [Patient Access Position](#) 12

Maintenance

- [Calling for Service](#) 13
- [Cleaning](#) 13
- [Periodic Maintenance](#) 13

Warranty Information

- [Limited Warranty](#) 14

Product Registration

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the chair, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

EC	REP
----	-----

WARNING
In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in the EU should direct all questions, incidents, and complaints to Midmark’s Authorized Representative listed below.

CEpartner4U
Esdoornlaan 13
3951 DB Maarn, The Netherlands
Phone: +31 343 442 524
Fax: +31 343 442 162

Customers in Australia should direct all questions, incidents, and complaints to Midmark’s Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Authorized Representatives - continued

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel: +91 22 4915 3000
Fax: +91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Associated Medical Supplies Company LTD. 沒有中文版使用說明
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice or condition.

Symbol Glossary

Pressure Limit

Temperature Limit

Proper shipping orientation

Keep dry

Fragile

Catalogue Number

Manufacturer

Serial Number

Medical Device

Intended Use

Assist patient entering, exiting or repositioning on the chair.

Safety Instructions

WARNING

No modification of this equipment is allowed.

WARNING

Do not sit on patient support rail.

WARNING

Do not press any button during a procedure.

WARNING

Avoid entrapment between rail and back.

Set Up

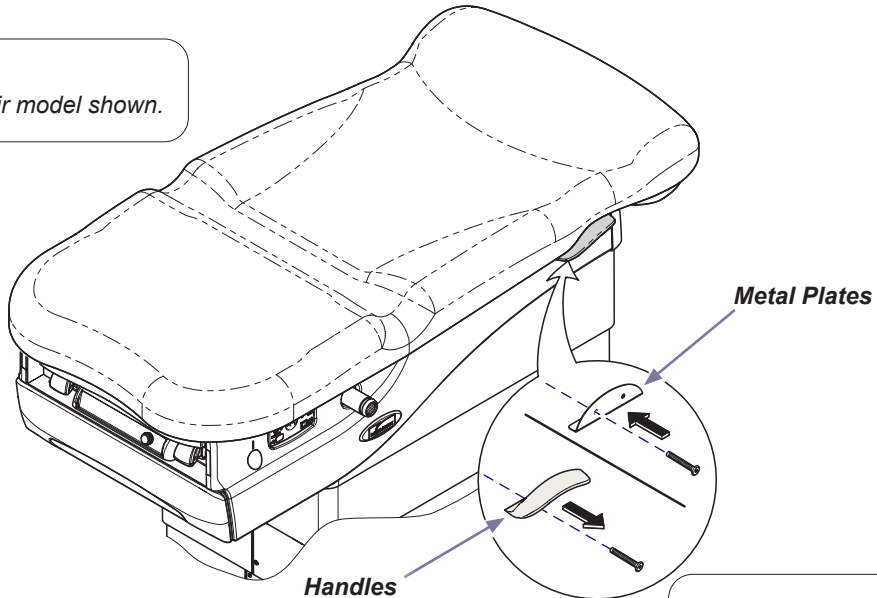
Equipment Alert

Ensure rails do not appear uneven. Rails should be horizontally straight. Unevenly positioned rails may not be locked.

Conflict when patient support rails and 253-002 exam light mounted on model 625 or 222/223 chairs. Use 253-003 exam light with extended spline mount instead.

Note

625 chair model shown.


Step 1: 223 Chairs ONLY

Remove plastic handles (2) and replace with metal cover plates (2) on upholstery back.


Note

223 Chairs **ONLY**.

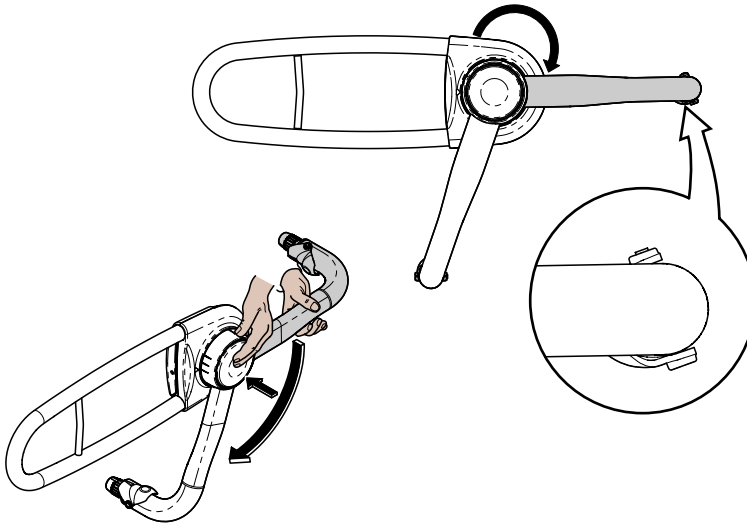Set Up - continued

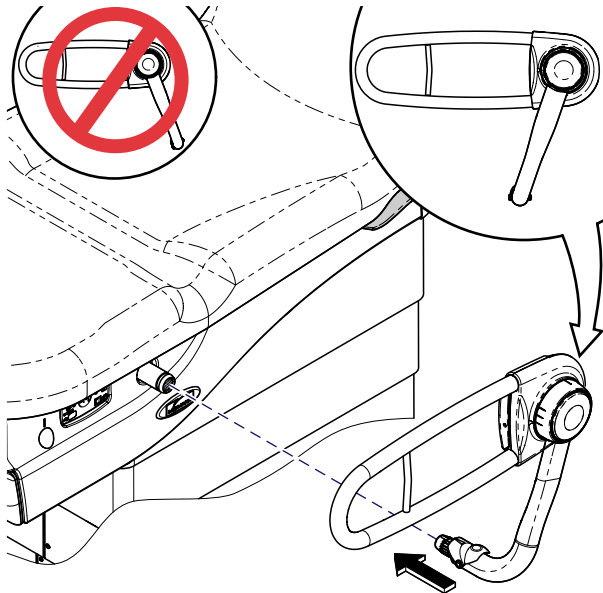
Step 2: Patient Left Hand
Push knob in and rotate as shown.


Step 3:
Install side rail into socket until release level locks into place.

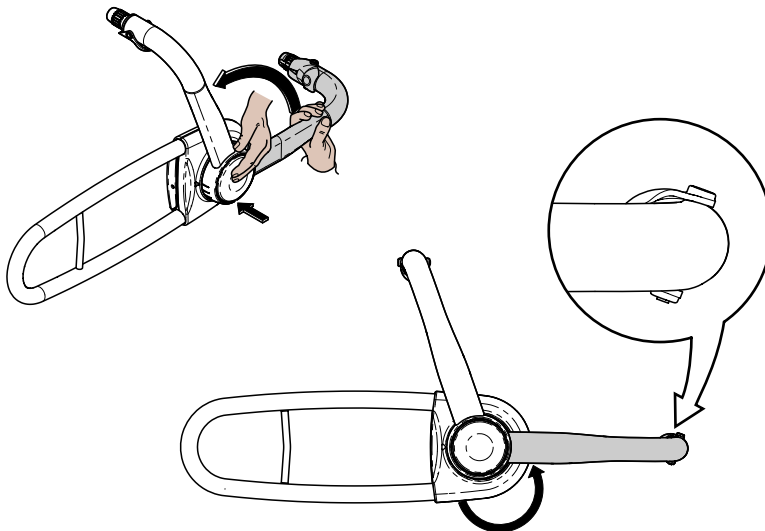Set Up - continued

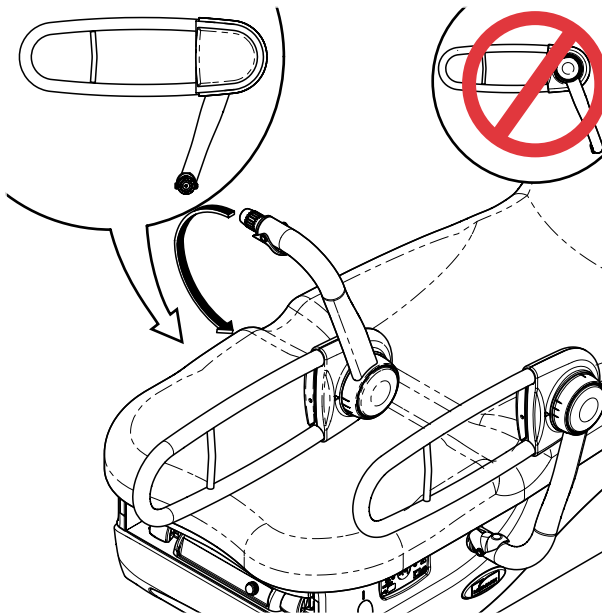
Step 4: Patient Right Hand *Push in knob in and rotate as shown.*


Step 5: *Install side rail into socket until release level locks into place.*

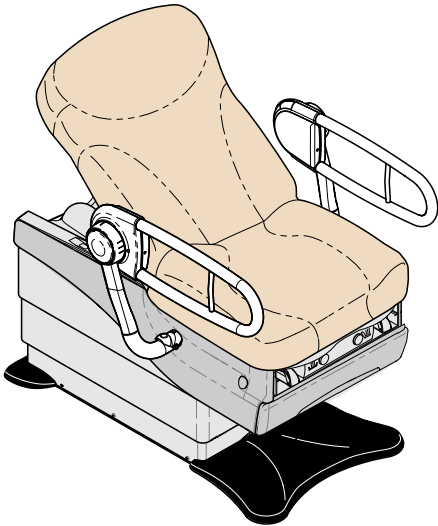Operation


Lowest position

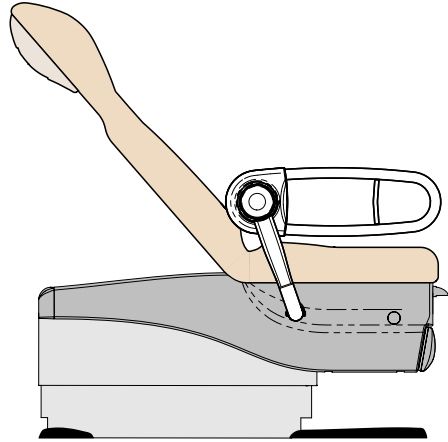Starting position of Support Rail.


Patient Right - View

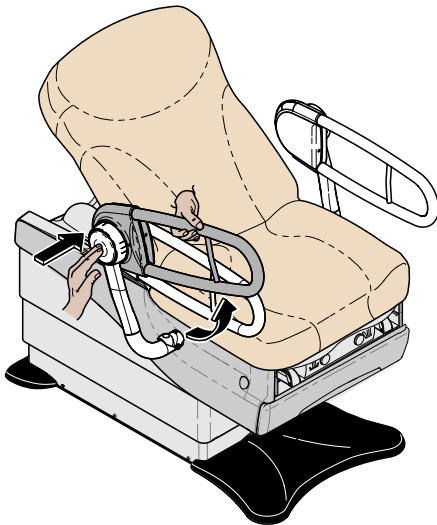This position is used for arm positioning with short patients.

Articulation of rail

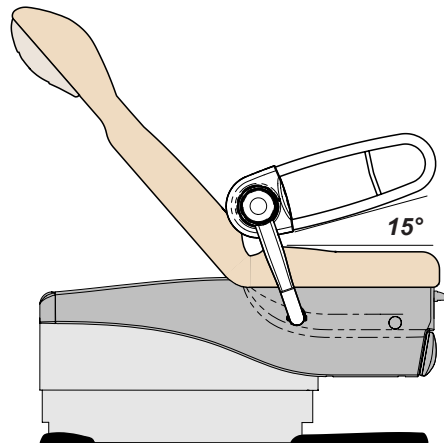
Hold rail with hand while pushing button with other hand to raise support rail into position.

Patient Right - View


The 15° position is used for arm positioning with medium height patients.

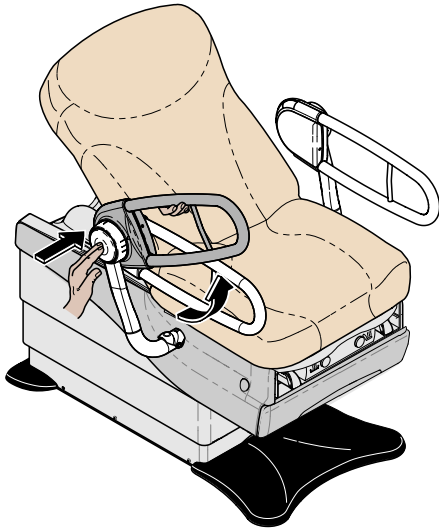Operation - continued


Articulation of rail

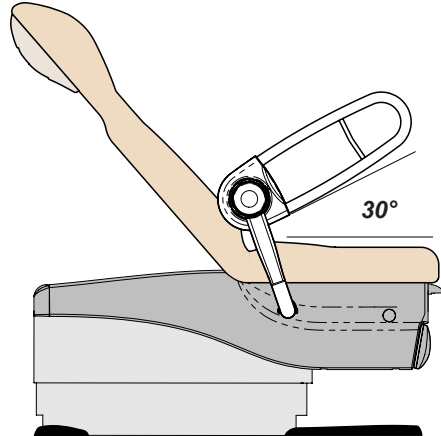Hold rail with hand while pushing button with other hand to raise support rail into position.


Patient Right - View

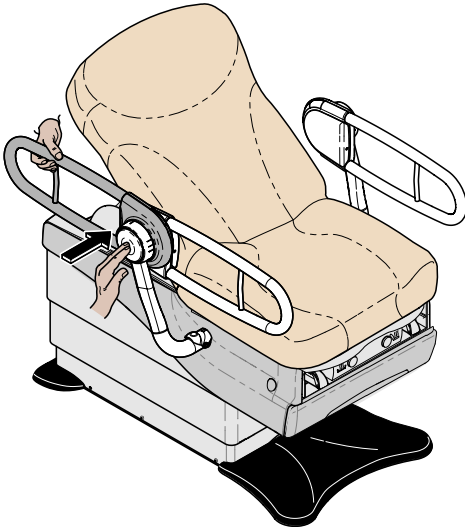The 30° position is used for arm positioning with tall patients.

Patient Access position

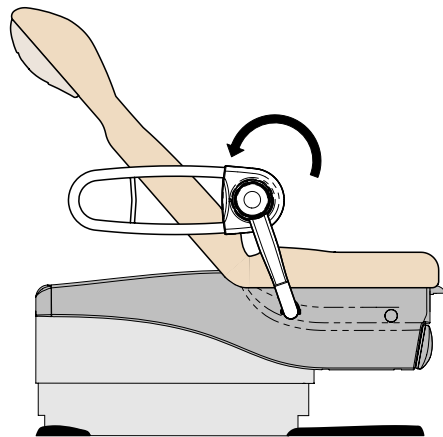Hold rail with hand while pushing button with other hand to rotate support rail out of way.

Patient Right - View

This rotation allows for patient access to chair seat.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The Support Rails are resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Support Rails.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

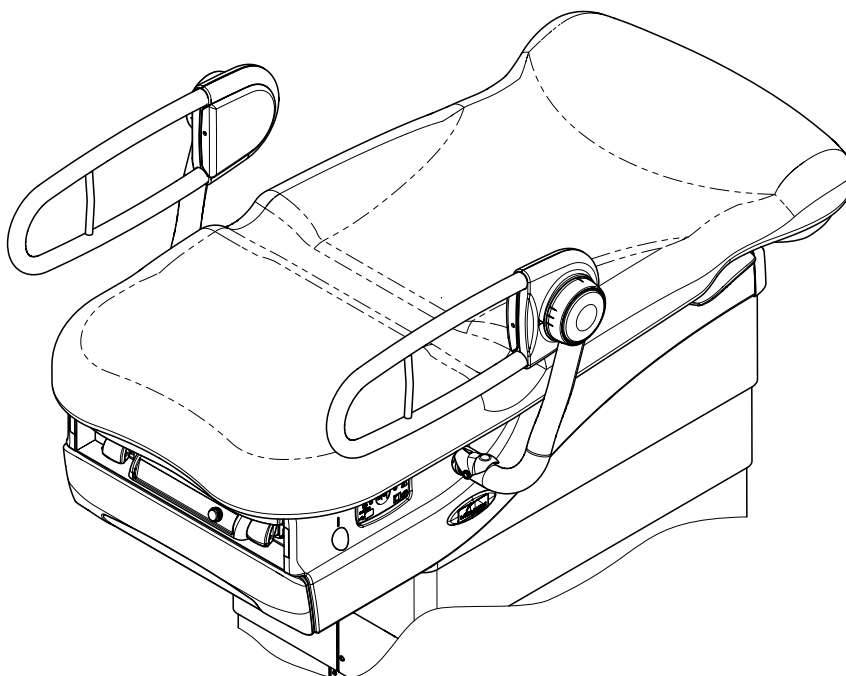
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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Midmark Corporation