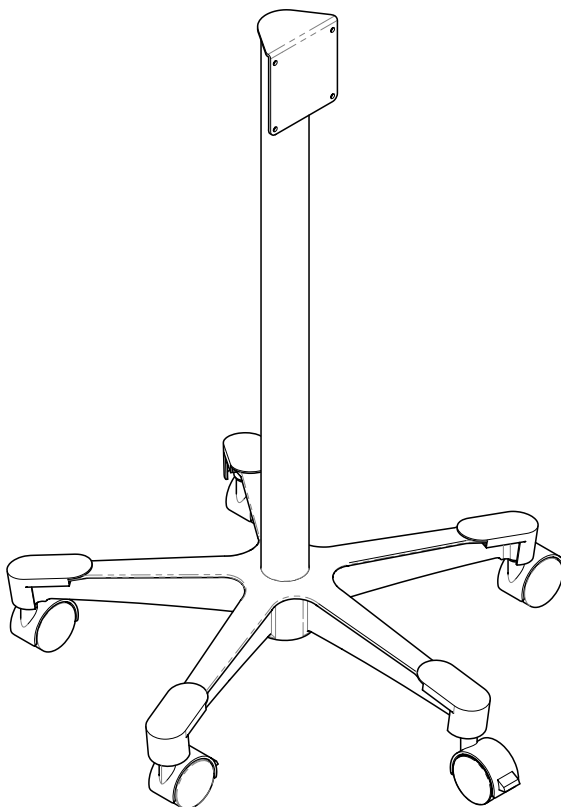
LED Light Mobile Caster Base 9A628

[English](#)
[Español](#)
[Français](#)

Applies to Models:
253

Special Tools:
none

Set Up / User Guide

Style P

003-10407-99 Rev. AA3 (10/6/23)

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

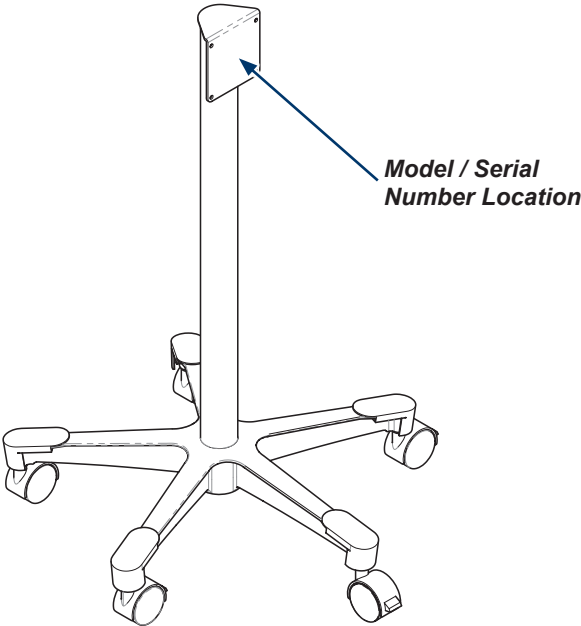
Table of Contents

Important Information

Product Information 2

Product Registration 4

Operating Conditions 4

Disposal of Equipment 4

Authorized Representatives 4

Important Information 6

Intended Use 7

Safety Instructions 7

Set Up

Set Up Instructions 8

Operation

Operating Instructions 9

Maintenance

Calling for Service 10

Cleaning 10

Periodic Maintenance 10

Warranty Information

Limited Warranty 11

Product Registration

Operating Conditions

Ambient Temperature Range:.....+10°C to +40°C (+50°F to +104°F)
Relative Humidity.....30% to 75% (non-condensing)
Altitude.....3000m or less

Disposal of Equipment

At the end of product life, the caster base, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Authorized Representatives

WARNING

*In the event of any serious incident in relation to the device,
please contact Midmark and the appropriate competent authority.*

Customers in Australia should direct all questions, incidents, and complaints to Midmark's Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel:+91 22 4915 3000
Fax:+91 22 4915 3100

Authorized Representatives - continued

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

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Associated Medical Supplies Company LTD.
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage.

Note

Amplifies a procedure, practice, or condition.

Symbol Glossary

Manufacturer

Serial Number

Catalogue Number

Intended Use

To provide mobility for Ritter® 253 exam light.

Safety Instructions

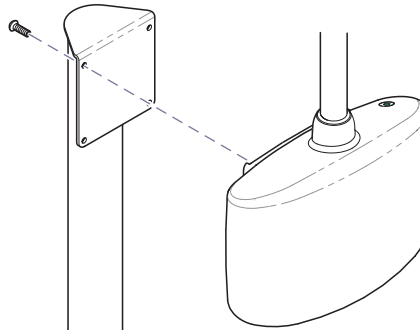
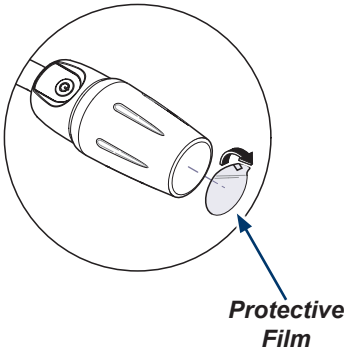
WARNING

No modification of this equipment is allowed.

Set Up

Before using light:

Remove protective film from lens.

Step 2: Install light.

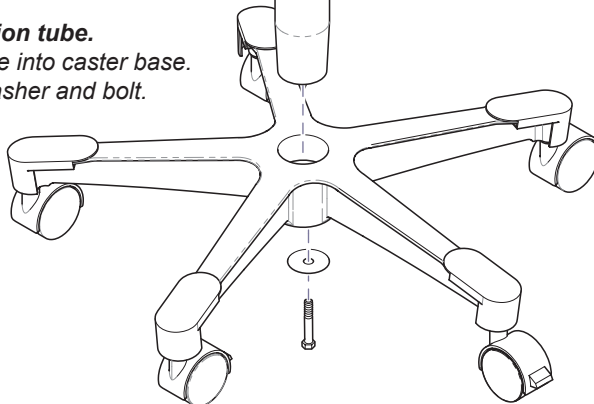
A) Align light with holes in mounting plate.

B) Secure light with four screws.

Step 1: Install extension tube.

A) Press extension tube into caster base.

B) Secure tube with washer and bolt.

Operation

To move or reposition caster base...

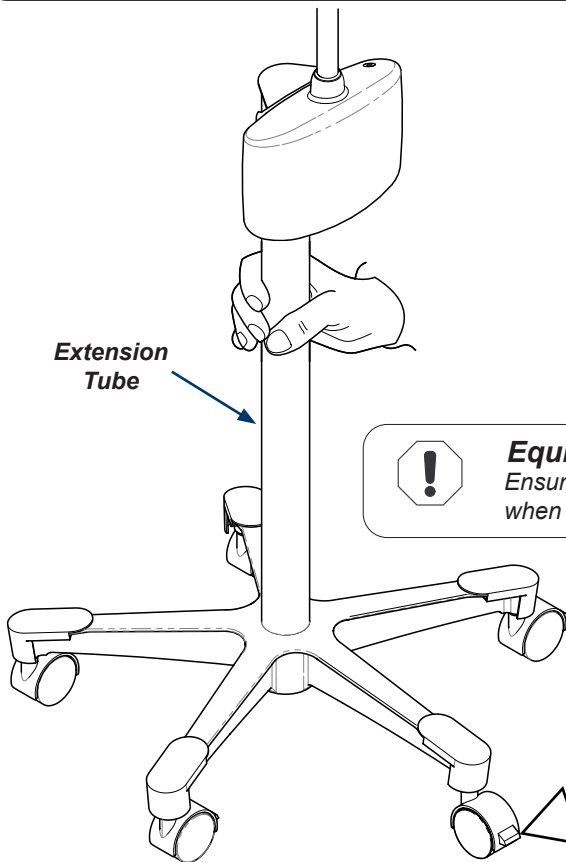
A) Unlock both locking casters.

B) Grasp extension tube and move caster base to desired location.


Equipment Alert

Grasp extension tube when moving caster base or repositioning the 253 light.

Equipment Alert

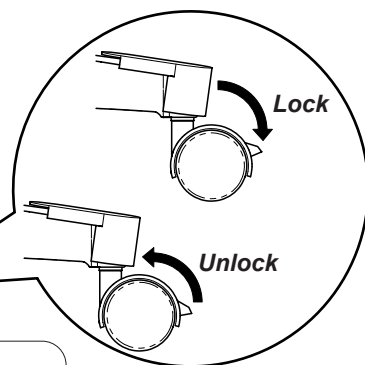Ensure both locking casters are “Unlocked” when moving the caster base.

Note

There are two locking casters on the caster base.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact Midmark directly:

US 844.856.1230 / Canada 937.526.8585

Monday-Thursday 8:00 AM until 6:00 PM

Friday 8:00 AM until 5:00 PM (ET)

midmark.com

Cleaning

Equipment Alert

The Caster Base is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the Caster Base.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth and mild cleaner.

Periodic Maintenance

Periodically inspect the following areas:

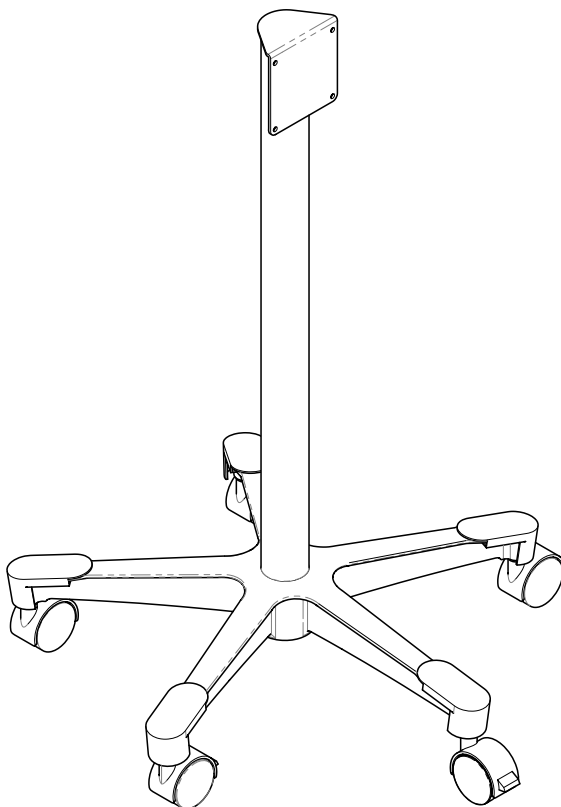
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

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Midmark Corporation