

# Debris Tray Magnet Template

003-10413-00 Rev AA2 12/14/20

1. **Cut both cutouts along the dotted lines.**
2. **Place cutout (1) in the underside of the debris tray.**
3. **Peel adhesive backs off of the thinner magnets. Place and hold in circle cutouts for 10 seconds.**
4. **Place cutout (2) on front side of the debris shelf.**
5. **Peel adhesive backs off of the thicker magnets. Place and hold in circle cutouts for 10 seconds.**
6. **Allow both sets of magnets to cure for 24 hours before use.**

