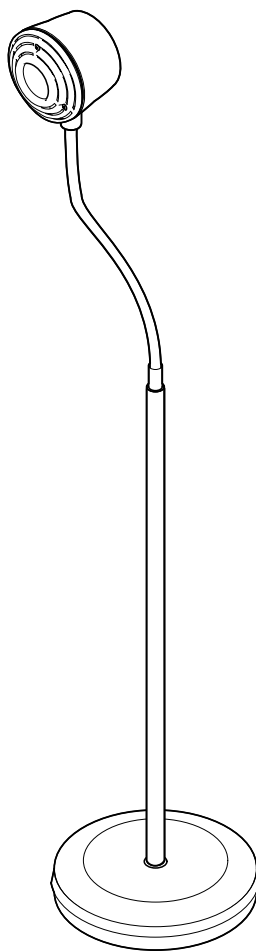
Ritter®

250 LED Exam Light

[English](#)
[Español](#)
[Français](#)

Applies to Models:
250

Special Tools:
none

Set Up / User Guide

Style P

003-10450-99 Rev. AA3 (9/21/22)

Product Information

Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

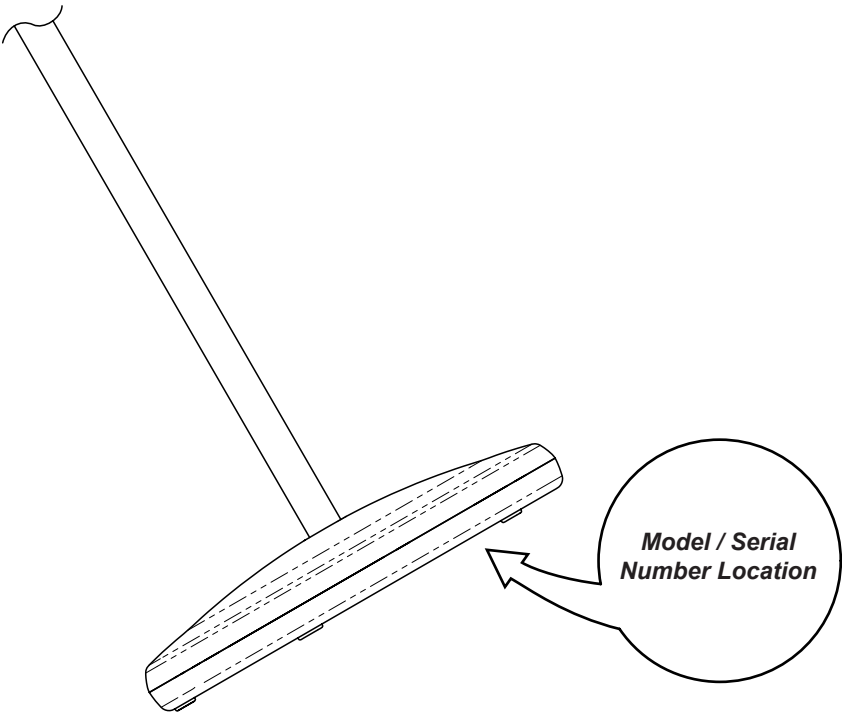
Table of Contents

Important Information

Product Information 2

Product Registration 4

Transportation / Storage Conditions 4

Disposal of Equipment..... 4

Authorized Representatives 4

Important Information - Safety Symbols 6

Intended Use 7

Electromagnetic Interference..... 7

Safety Instructions..... 7

Set Up

Set Up Instructions 8

Operation

Operating Instructions 9

Specifications

Specifications Chart..... 11

Maintenance

Calling for Service 12

Cleaning 12

Preventive Maintenance 12

Warranty Information

Limited Warranty 13

Product Registration

To register your product, go to www.midmark.com

Transportation / Storage Conditions

Ambient temperature range.....41°F to 100°F (5°C to 38°C)
Relative humidity 10% to 90% (non-condensing)
Atmospheric pressure..... 500hPa to 1060hPa (0.5 bar to 1.06 bars)

This product contains sensitive parts. It should be transported and stored with care to limit vibration and shocks.

Disposal of Equipment

At the end of product life, the light, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Australia should direct all questions, incidents, and complaints to Midmark's Sponsor listed below.

ICONA Pty Ltd.
1A/2A Westall Road
Clayton, Victoria 3168
Phone: 1300 442 662

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel: +91 22 4915 3000
Fax: +91 22 4915 3100

Authorized Representatives - continued

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Customers in Hong Kong should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

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Associated Medical Supplies Company LTD.
Room 1201-1202, 12/F, Fo Tan Industrial Center
26-28 Au Pui Wan Street, Fo Tan, Shatin
New Territories, Hong Kong
Phone: +852 2604 9389
fax: +852 2694 0866

Customers in Qatar should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Tadmur Trading
Midmac Roundabout
Opposite Al Mana Petrol Station
Doha, Qatar
Tel: +974.44337000
Fax: +974.44337100

Customers in Israel should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Medtechnica Ltd.
HaTnufa St 7
Petah Tikva, Israel
Tel: +972 3.925.4040
Fax: +972 3.924.9977

Important Information - Safety Symbols

These symbols may appear on your equipment and/or in the manuals.

WARNING
Indicates a potentially hazardous situation which could result in serious injury.

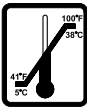
Caution
Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert
Indicates a potentially hazardous situation which could result in equipment damage.

Note
Amplifies a procedure, practice, or condition.

Symbol Glossary

	Proper Shipping Orientation		Storage Temperature Limits
	Fragile		Keep Dry
	Atmospheric Pressure Limits		Humidity Limits
	Protective Earth Ground		Class 1, Type B Applied Part Ordinary Equipment (IPX0)
	Fuse Rating Specification		Refer to instruction manual/booklet
	Serial Number		Manufacturer
	Catalogue Number		

Intended Use

To provide illumination for general examinations and procedures performed by trained professionals in a healthcare environment.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room
- Plug light into isolated circuit
- Increase separation between light and interfering device
- Contact Midmark if interference persists

Safety Instructions

WARNING

No modification of this equipment is allowed.

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic mixture with oxygen, air, or nitrous oxide.

Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Set Up

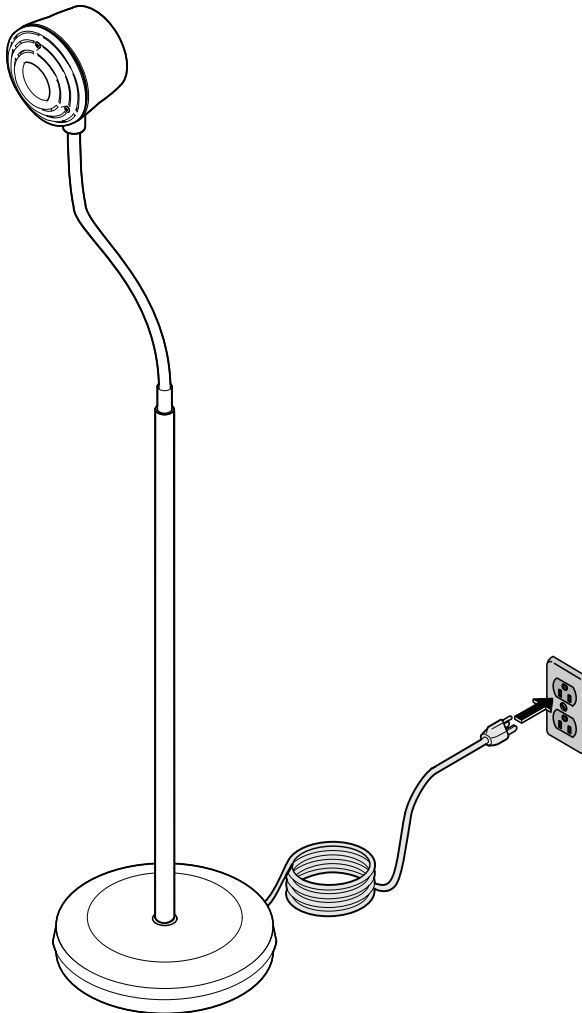
Equipment Alert

Remove light assembly out of bottom of box and remove cardboard protectors.

To Install the light...


- A) Place light base on floor.
- B) Plug power cord into 120 VAC outlet.

Operation

To operate the light...

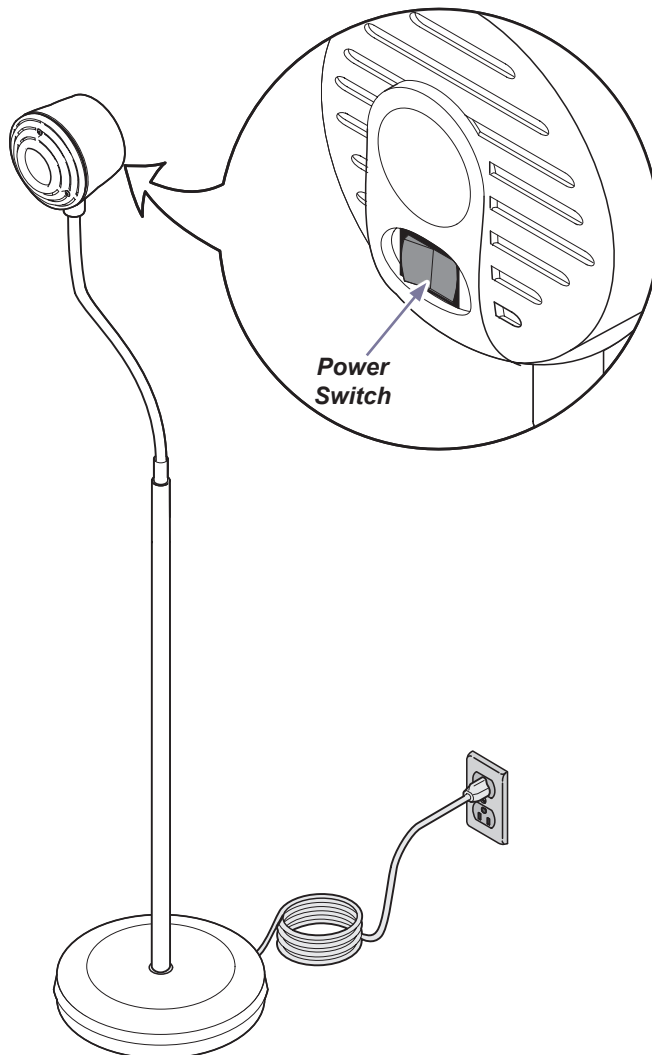Move power switch to the ON position.

Operation - continued

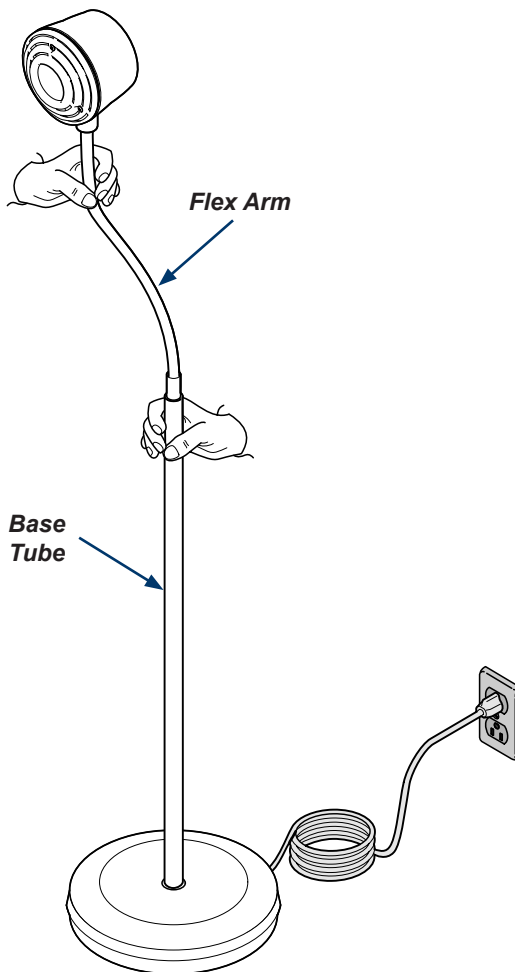
To reposition the light...

Grasp base tube directly below flex arm, then grasp flex arm and adjust for desired position.

Equipment Alert

Grasp base tube when moving or repositioning the 250 light.

Specifications

Specifications Chart	
Lighthead	
Size:	4.5" (11.4 cm) dia. x 4" (10.2 cm) long
Bulb:	12V LED / 25,000-hr life / Type MR16 (with glass cover)
Illumination @ 18" (45.7 cm):	1800 fc
Color Temperature:	4000 K
Flex Arm	
Length:	22" (55.8 cm)
Base Tube	
Length:	36" (91.4 cm) long
Standard Base Housing	
Size:	13" (33 cm) diameter x 2.25" (5.7 cm) thick
Weight:	12 lbs. (5.4 kg)
Electrical Information	
Power Cord:	8 ft. (2.5 m), Hospital Grade
Requirements:	120 VAC, 50/60 Hz, 0.12A
Fuses (qty: 2 / built into power cord):	T500L, 250V, 500mA, Time Delay
Compliance Information	
UL60601-1 (2nd Edition)	
CAN / CSA C22.2 No. 601.1 M90	

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact your authorized Midmark dealer.

To contact Midmark Versailles Tech Service directly:

1.937.526.3662

8:00 am until 7:00 pm Monday - Thursday (ET) [excluding standard US holidays]

8:00 am until 5:00 pm Friday (ET) [excluding standard US holidays]

midmark.com

Cleaning External Surfaces

Equipment Alert

Before cleaning the light, disconnect power and allow the light to cool. Do not touch the bulb or inner components with bare hands. Clean EXTERNAL SURFACES ONLY! Prevent fluids from leaking under covers to interiors or onto electrical components. DO NOT ATTEMPT to clean or disinfect interior.

For general cleaning, use a mild detergent and water solution. Wring excess solution from sponge or cloth before wiping surfaces.

In accordance with your facility's cleaning procedures:

1. Wipe external surfaces with a mild detergent and water solution.
2. Rinse all external surfaces with a soft cloth dipped in clean water.
Wring excess water from cloth before wiping.
3. Wipe all external surfaces dry.

Disinfecting External Surfaces

Equipment Alert

Use only quaternary disinfectants to disinfect light. Staining, pitting, discoloration, or softening may occur if phenolic, iodophor or glutaraldehyde-based disinfectant is used on plastic surfaces.

Midmark® + Ritter® Medical Equipment (Tables, Chairs, Physician Seating, Lights, Workstations and Sterilizers) Limited Warranty

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components. QuickClean® Ultrasonic Cleaners are warranted for a period of three (3) years.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at midmark.com; by phone at 1.800.MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

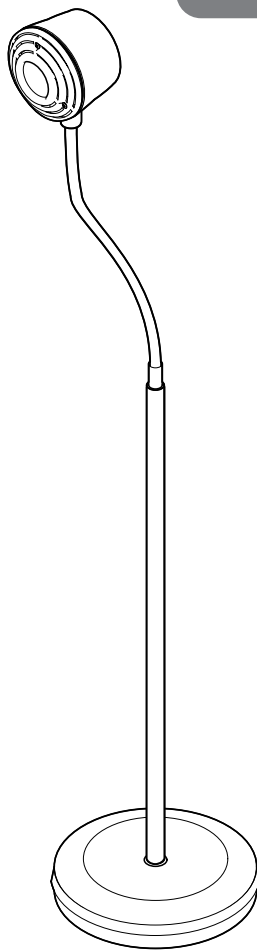
NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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© Midmark Corporation 2021